



Exa PACS/RIS 1.4.27

Feature Summary: Technologist Screen

© 2020 Konica Minolta Healthcare Americas, Inc.

Internal Use Only

The technologist-specific exam screen (technologist screen) enables technologists to more easily and efficiently complete their workflows (such as changing statuses, adding study notes, and scanning documents) in Exa PACS/RIS.

- To make the technologist screen available, you must link the user to a technologist in the table in **Setup > Resources**.
 - The user does **NOT** have to belong to the default Technologist user group.
- You must add the Technologist right to the user's user role under **Setup > User Management > User Roles > Home**.
- The technologist screen appears when a technologist user opens a worklist study for editing.
 - They can still access the conventional editing screen by right-clicking the study and selecting **Change/Edit Patient**.

The screenshot shows the 'PROVIDERS/RESOURCES' setup screen. The 'RESOURCE' tab is selected. The 'Type' dropdown is set to 'Technologist'. The 'Code' is 'HCP325TG' and the 'Inactive' checkbox is unchecked. The 'Name' is 'Danielle' with 'MI' as the middle initial and 'Niewind' as the last name. The 'Modality' list includes 'All Modalities', 'BD', 'BR', 'CT', 'MG', 'MR', 'NM', and 'RT', all of which are checked. The 'Facilities' list includes 'All Facilities', 'Garner', 'Insurance Facility', 'Jacksonville', and 'Mammogram QC', with the first three checked. The 'User Name' field is 'dniewind'. The 'LOGIN DETAILS' tab is selected, and the 'User Name' field is highlighted with a red box. Below the 'User Name' field are 'UNLINK USER' and 'RESET PASSWORD' buttons.

USER MANAGEMENT -

USERS USER ROLES USER GROUPS AD GROUPS AD USERS USERS ONLINE USER ASSIGNED STUDIES USER ASSIGNED PATIENTS THIRD PART

Group Name * Facility Admin DNU Hide AD Groups

Name * Danielle MI Niewind Suffix

User Name * dniewind Inactive

Session Interval 300 Allow Emergency Access

Access Expires After days

One-Time Access

User Must Change Password Next Login

User can change accounting dates

Dragon 360

All Facilities

Facilities *
Garner
Insurance Facility
Jacksonville

Linked Provider User Type Technologist

Technologist Niewind, Danielle

USER MANAGEMENT -

USERS **USER ROLES** USER GROUPS AD GROUPS AD USERS USERS ONLINE USER

- Adjustment Codes
- Billing Provider
- Billing Messages
- Billing Validation
- Printer Templates
- EDI Request Template
- Clearing House
- Billing
 - Claims
 - Edit Claim
 - Claim Inquiry
 - All Insurances
 - Coordination of Benefits
 - Explanation of Benefits
 - ERA Inbox
 - EOB Claims Process
 - File Insurance
 - Patient Claim
 - Patient Report
 - Payments
 - Apply Payments
 - Refund
 - Back to Ready To Validate
- Home
 - Allow reset Approved studies
 - Allow reset of Canceled and No Show studies
 - Appointment Confirmation
 - Can Edit Public Filter
 - Can Edit Public Template
 - Cancel Reason
 - CD Burn
 - Dicom View
 - Dicom Viewer
 - Image Preview
 - Import Images
 - Link Dicom Study
 - Marketing Rep Dash Board
 - Merge Study
 - Technologist
 - Opal Viewer
 - Order_additional
 - Order_ICD
 - Order_Referring_Provider
 - Order_Studies
 - QC Delete
 - QC Edit

Niewind, Danielle - 03/03/1992 - NC1577, F

4177 4178 Multiple studies in one Order are displayed as multiple tabs

SAVE

TECH START **IN PROGRESS** TECH END

Status changes and the time it takes a technologist to complete an entire exam can be tracked in the "Technologist Productivity" report in Reports > Operations

Study Notes | Schedule Notes | Patient Notes | Reason for Study

Documents

Studies

Study Forms

Appointment Type: XR Chest

Body Part: Select

Orientation: Select

Scheduled Date: 04/15/2020 3:25 PM EDT

Duration: 15 / 30 minutes

Location: Garner

Modality Room: Xray Room 1, Garner

Ordering Physician: Smith, Joey

Reading Physician: Select Reading Physician

Attorney: Select Attorney

Technologist: radmin Technologist, A

ABORT

CANCEL