



Exa PACS/RIS 1.4.28

Feature Summary: Advanced Peer Review

© 2020 Konica Minolta Healthcare Americas, Inc.

Confidential

With advanced peer review, customers can configure their peer review process based on their needs and facility workflow, and easily generate reports to evaluate productivity and progress of peer review cases.

Note: Advanced peer review is available at an additional cost, so please include Sales as needed.

- You must enable settings under **Configuration > Web Configuration > Peer Review**.
- The user can set the percentage of peer review cases based on the amount of annual approvals under **Setup > Office > Company**.

The screenshot shows the 'OFFICE - COMPANY' configuration page. The 'Peer Review' section is highlighted with a red box. The 'Peer Review Percentage of yearly approvals' is set to 75. Other sections include 'Billing Information' and 'Email Information'.

- There are two new user rights for peer review.
 - **Peer Review allowance** – enables the user to conduct peer reviews.
 - **Peer Review manual appointment** – enables the user to appoint peer reviewers.


- Users can set additional peer review appointment criteria for physicians under **Setup > Providers & Resources > Resources**. On a provider's **PEER REVIEW** sub-tab, you can filter their availability by facility, modality, CPT code, appointment type, or study description.

- To access the MY PEER REVIEW worklist filter, you must:
 - Add the Peer Review allowance right to the user's user role
 - Link the user to a Radiology Provider user type

MODALITY PR	STUDY DESCRIPTION PR	AGE-TIME OF STUDY PR	GENDER PR	REASON FOR STUDY PR
MG,SC,SR	MG Bilat Digital Screening Mammo 3D	undefined	U	

- The MY PEER REVIEW tab lists studies that are automatically or manually assigned to the user. The user can open the study to launch the Peer Review screen, where they can view the approved report, select an agreement level, and add additional comments.

Note: The Peer Review agreement levels are *not* configurable. They are aligned with the ACR’s peer review rating scale.

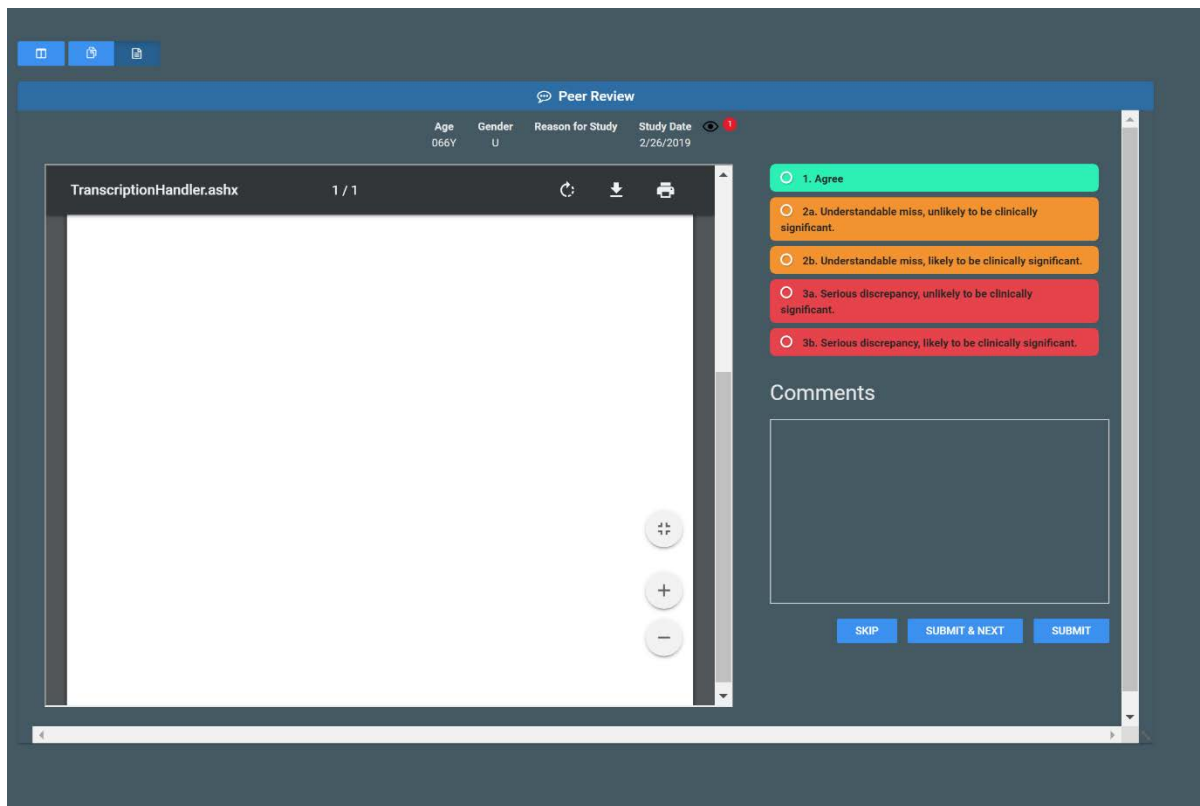

- Users with the Peer Review manual appointment right can click a peer review disagreements button in the top right of worklist to display the PEER REVIEW DISAGREEMENTS worklist filter.



Worklist									STAT	0 0 0 0 0 0 0 0 0						TAT	0 0 0 6			
ALL STUDIES									PEER REVIEW DISAGREEMENTS ⁴											
AGREEMENT LEVEL	REVIEW NOTES	PEER REVIEWING PHYSICIAN	APPROVING PHYSICIAN PR	MODALITY PR	STUDY DESCRIPTION PR	FACILITY PR	AGE-TIME OF STUDY PR	GENDER PR	REASON FOR STUDY PR											
2a	sdfasdf	Anderson, Margaret	Test, Radiologist	PT	PET CT skull base midhigh ini	My Company	084Y	F												
2a	text	Radovich, Rachel	Test, Radiologist	XA		My Company	067Y	F												
2a	I disagree	Radovich, Rachel	Radulov, Roddy	CT	CT LUMBAR SPINE W/O CO	My Company	046Y	M												
2b	Comment made over here	Radulov, Roddy	Test, Radiologist	CT,SC	CT Chest without Contrast	My Company	049Y	F												

- Users can monitor this tab for peer review cases that have assigned disagreement levels, and for any comments the peer reviewer made on the study.
- Users can right-click a reviewed study to dismiss it or reappoint it to another reviewer.

AGREEMENT LEVEL	REVIEW NOTES	PEER REVIEWING PHYSICIAN	APPROVING PHYSICIAN PR	MODALITY PR	STUDY DESCRIPTION PR	FACILITY PR	AGE-TIME OF STUDY PR	GENDER PR	REASON FOR STUDY PR
2a	sdfasdf	Anderson, Margaret	Test, Radiologist	PT	PET CT skull base midhigh ini	My Company	084Y	F	
2a	text	Radovich, Rachel	Test, Radiologist	PT	PET CT skull base midhigh ini	My Company	067Y	F	
2a	I disagree	Radovich, Rachel	Test, Radiologist	PT	CT LUMBAR SPINE W/O CO	My Company	046Y	M	

- Users can also manually appoint a peer reviewer to a study from the ALL STUDIES tab.
 - Right-click > Exam > Appoint Peer Reviewer

The screenshot shows the 'Worklist' interface with a table of studies. A right-click context menu is open over a study, and the 'Appoint Peer Reviewer' option is highlighted with a red circle. The table columns include STATUS, PATIENT, ACCESSION NO., MODALITY, STUDY DESCRIPTION, REF. PHYSICIAN, FACILITY, STUDY DATE, STUDY FLAG, TECHNOLOGIST, IMGS., and ACCOUNT NO.

The 'Appoint Peer Reviewer' dialog box is shown. It features a search field labeled 'Select provider to perform peer review' and an 'APPOINT' button. Below is a table with the following data:

Study Date	Study description	Facility
10/18/2011	Head*1PETCT_Brain	My Company

- Once a peer reviewer has been appointed to a study, the user cannot assign it to another user.

Appoint Peer Reviewer

APPOINT

Not suitable for appointment

Error Code	Study Date
Reviewing provider already assigned for this study	04/15/2020

- A new **Peer Review Operational** report is also available for customers to track productivity and progress of cases.
 - To enable the report for a user, you must add the Peer Review Operational user right.

REPORTS - Peer Review Operational
⏻

Date Range *

Date Type * Study Date Reviewed Date

Provider +

Selected Peer Review Radiologist

Facilities * (1) MY COMPANY ▾

Modalities (20)

All Modalities

Show Inactive

BD
BR
CR
CT
DG
DS
DX
ES
...

VIEW
PDF
EXCEL
CSV
XML

exa Reports » My Company » Peer Review Operational

peerReview operational report

Found 1 results

#	Peer Reviewed By	Original Approving Radiologist	Facility	Study Date	Modality	Agree Level	Notes	Review Date	Accession Number	Entered By	Second Reviewer	Second Agreement level	Secondary Notes
1	Niewind, Danielle	J, Rj	My Company	02/26/2019	MG	1		04/20/2020	13855384				

- Users with access to dashboards can view three new dashboards gadgets.

