

exa-PACS • exa-RIS


Feature Summary

CD-Burn Images and Series


© 2023 Konica Minolta Healthcare Americas, Inc.

CD-burn images and series


1. In user settings, clear the **Show Priors** checkbox.

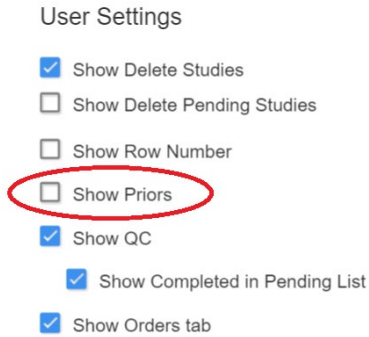


2. On the worklist, find the study containing images or series you want to burn and then select the plus button.

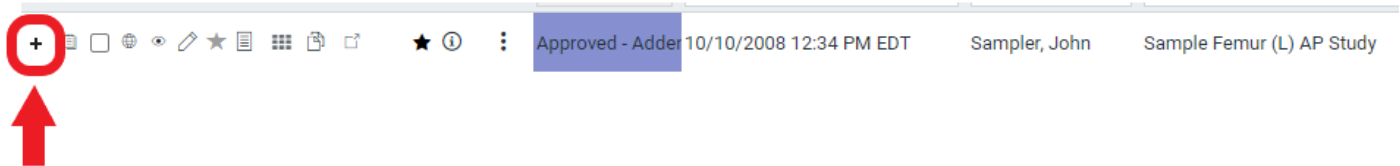

3. Select the plus button again, and then select the picture buttons to determine which images and series you want to burn.

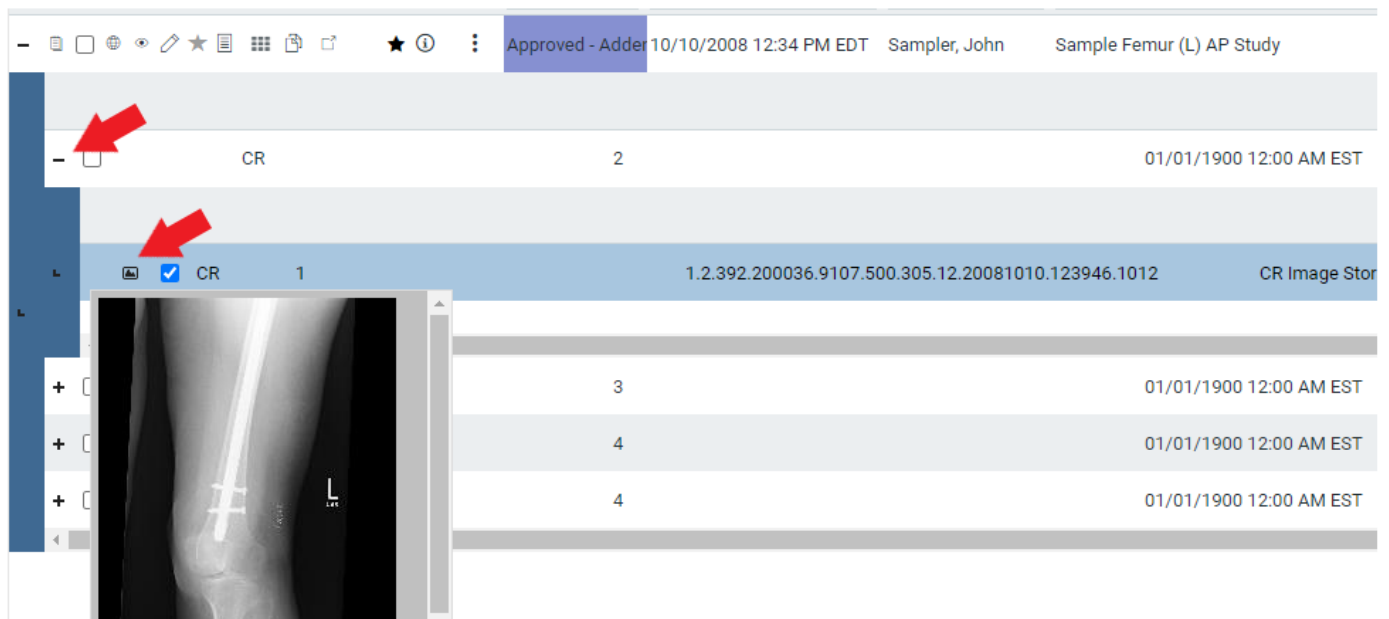

4. Select the checkboxes of the images and series that you want.
5. On the **PACS ACTIONS** menu, select **CD Burn**.

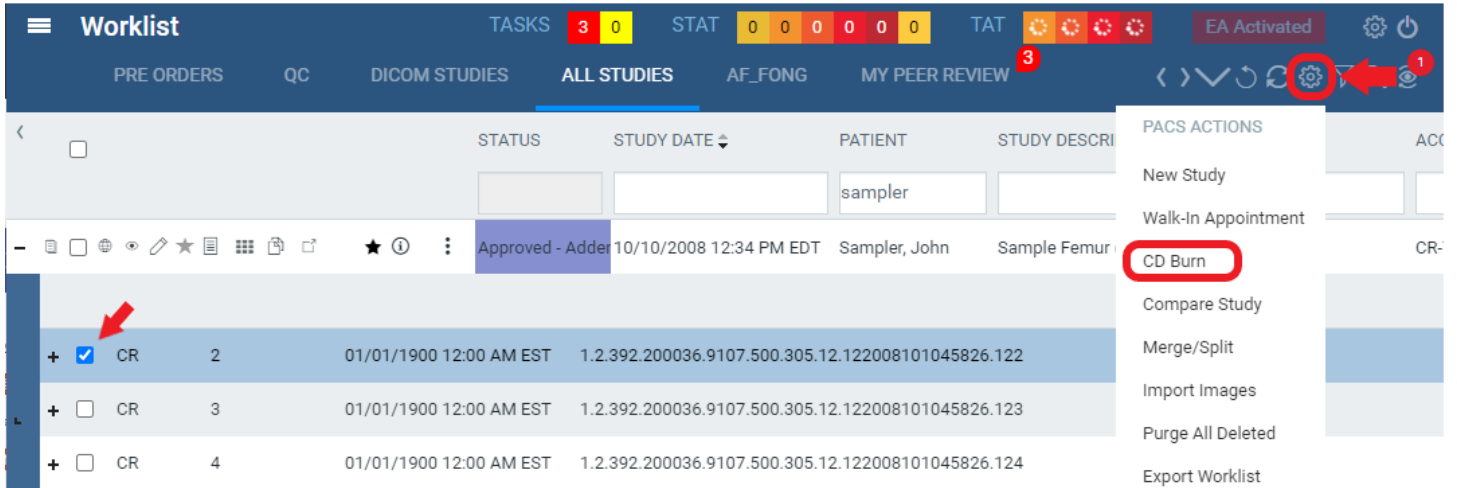

6. In **Opal CD Burning**, modify settings if necessary and then select **START**.

