



# **Exa<sup>®</sup> PACS/RIS**


## **Feature Summary: Change Your Password**

© 2020 Konica Minolta Healthcare Americas, Inc.


1. Click the hamburger menu  button.




2. Click the Konica Minolta logo.

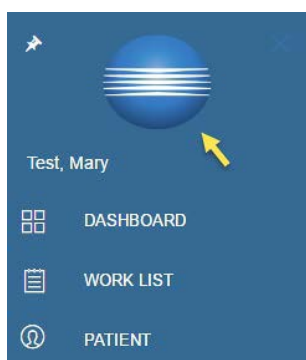

3. In the **My Profile** dialog box, click **CHANGE PASSWORD**.

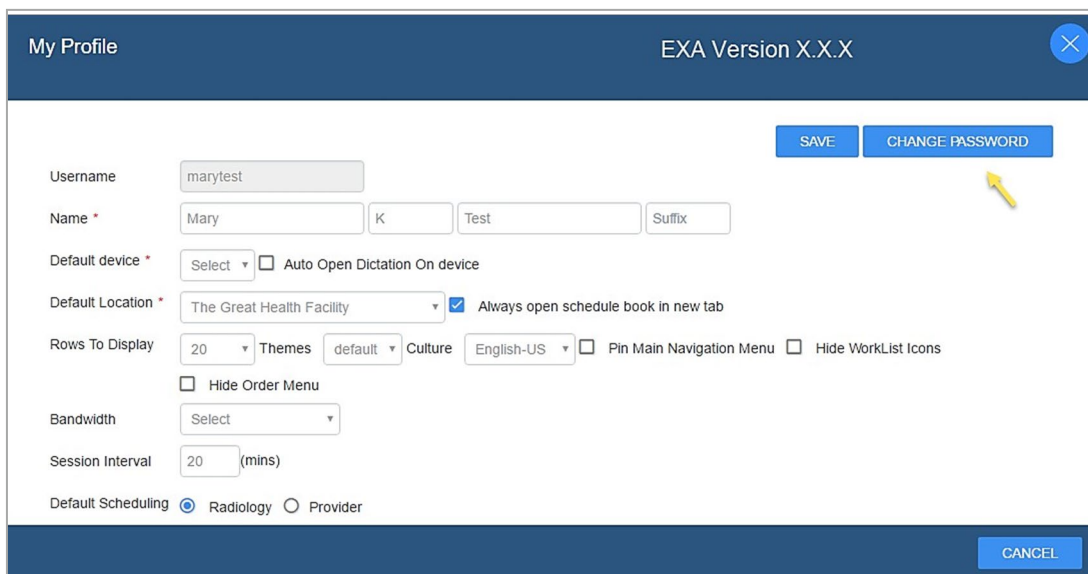

4. At the bottom of the dialog box, in the **Current Password** box, type your current password or the temporary password provided by your administrator.

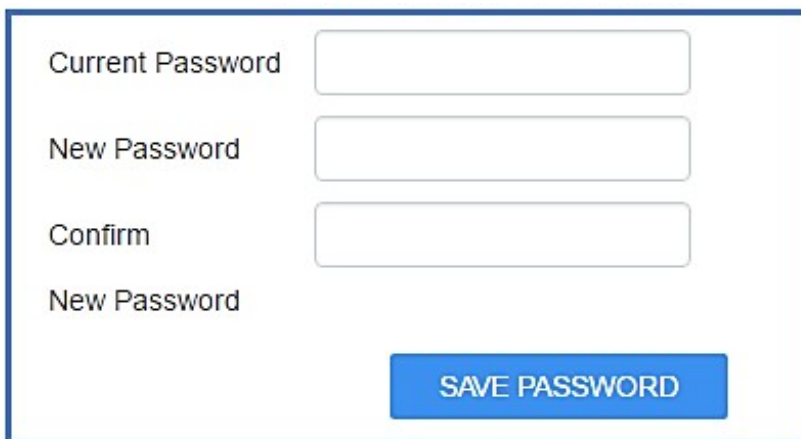

The screenshot shows a dialog box with a blue border. It contains four text input fields stacked vertically. The first field is labeled 'Current Password', the second 'New Password', the third 'Confirm', and the fourth 'New Password'. Below the input fields is a blue button with the text 'SAVE PASSWORD' in white capital letters.

5. Type your new password twice, and then click **SAVE PASSWORD**.
8. Click **SAVE**.