




Exa[®] PACS/RIS

Feature Summary: Add Tools to the Viewer Toolbar

© 2020 Konica Minolta Healthcare Americas, Inc.

You can add or remove modality-specific tools to or from the Exa PACS/RIS toolbar by following the procedure below. These settings apply to the user creating them, and they can be shared with other users.

See also “Configure the viewer” in the User’s Manual.

1. Click the settings  button, and then click **Viewer Settings**.



2. In the **Viewer Options** dialog box, click the **Tools** tab.
3. Under **Toolbar**, make necessary changes.
4. Under **Tools**, select or clear check boxes to add or remove tools.

