




Exa[®] PACS/RIS


Feature Summary: Customize the Viewer Context Menu

© 2020 Konica Minolta Healthcare Americas, Inc.

You can add or remove commands from modality-specific viewer shortcut menus by following the procedure below. The changes you make apply only to you, the current user.

See also, “Add or remove tools from the image shortcut menu” in the user’s manual.

1. On the settings  menu, select **Viewer Settings**.

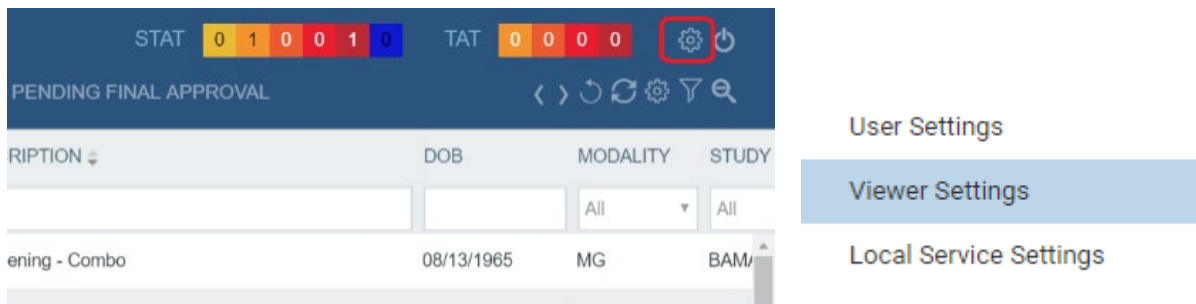

2. In the **Viewer Settings** dialog box, click the **TOOLS** tab.
3. Under **Toolbar**, click select a modality, and then click **>>CONTEXT MENU**.
4. Under **Tools**, select or clear check boxes to add or remove commands.
5. Click **SAVE**.

