

exa-PACS • exa-RIS


Feature Summary

DICOM-Send Images and Series


© 2023 Konica Minolta Healthcare Americas, Inc.

DICOM-send individual images and series


1. In user settings, clear the **Show Priors** checkbox.

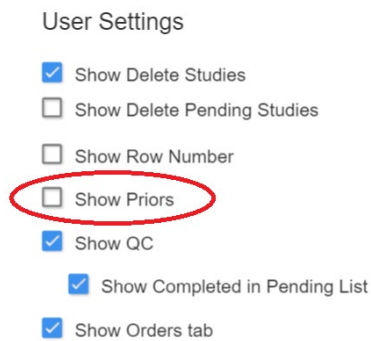

2. On the worklist, find the study containing images you want to send and then select the plus button.

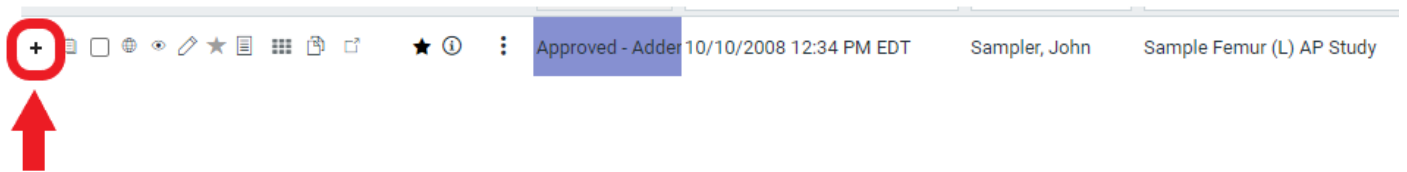


3. Select the plus button again, and then select the picture buttons to determine which images and series you want to send.

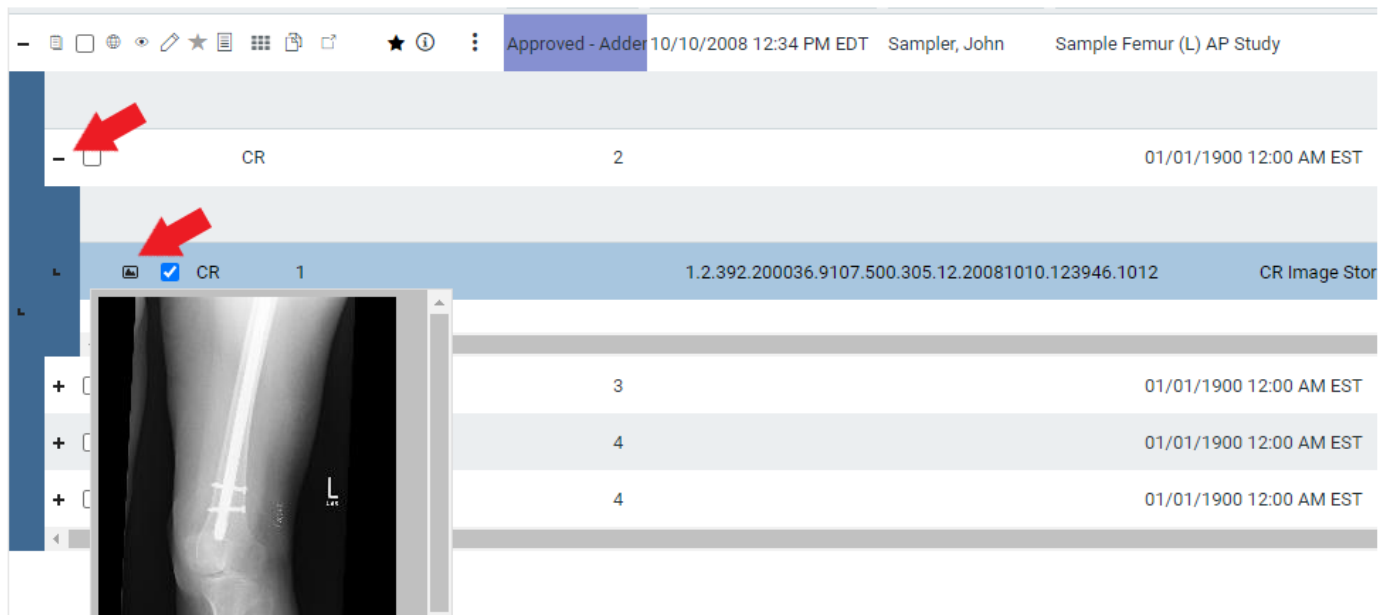

4. Select the checkboxes of the images and series that you want.

5. Right-click and select **DICOM Send**.

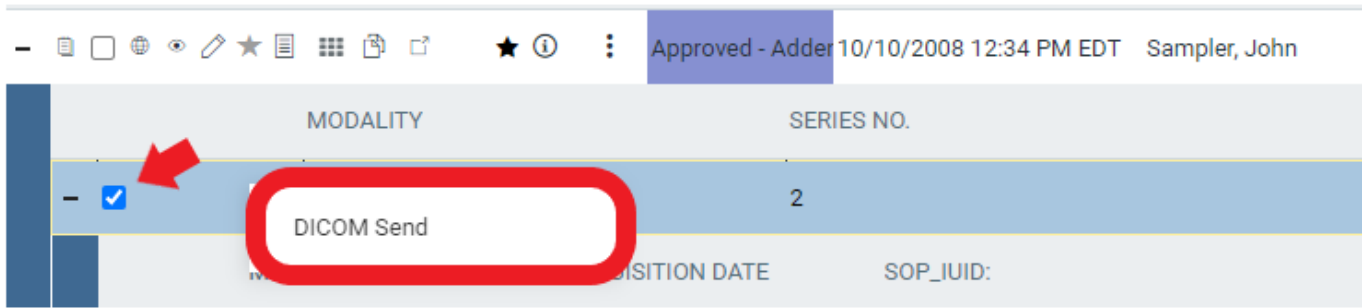

6. Select the send destination and then select **SEND TO SELECTED**.

