


exa-PACS exa-RIS

Feature Summary
SDE General Configurations
and Settings

1. Enabling SDE Report Templates for Clinical Workflow:

- a. Click on "User Settings"
- b. Under "Voice Recognition" select,
- c. "Dictation (WebTrans)"
- d. Click Save

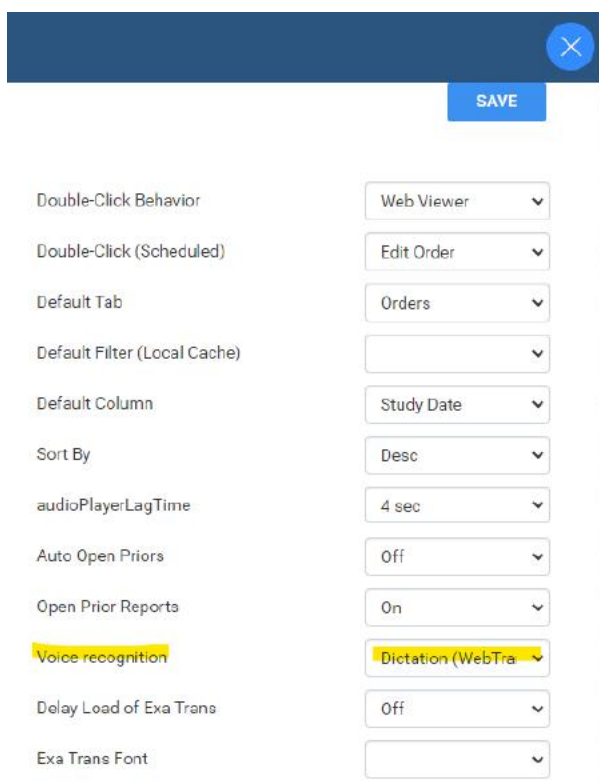

Setting	Value
Double-Click Behavior	Web Viewer
Double-Click (Scheduled)	Edit Order
Default Tab	Orders
Default Filter (Local Cache)	
Default Column	Study Date
Sort By	Desc
audioPlayerLagTime	4 sec
Auto Open Priors	Off
Open Prior Reports	On
Voice recognition	Dictation (WebTra...
Delay Load of Exa Trans	Off
Exa Trans Font	



2. Setting EXA to Launch Both Images and the SDE Report Template for Clinical Workflow:





- a. Click on "Viewer Settings".
- b. Install the Chrome_Multi-Monitor Extension.
- c. Go back to Viewer Settings.
- d. Under General>Display Settings, select Display1, select current and prior radio buttons.
- e. Under General>Display Settings, select Full Screen for Display2, then select "Transcription" under the Orders drop-down. (If not enabled, enable it under the Interface settings).

Viewer Settings

GENERAL

Display Settings

Monitors on System:  

Monitor	Current	Prior	Full Screen	Orders	T
<input checked="" type="checkbox"/> DISPLAY1			<input type="checkbox"/>	<input type="text" value=""/>	<input type="checkbox"/>
<input type="checkbox"/> DISPLAY2			<input checked="" type="checkbox"/>	Transcription <input type="text" value=""/>	<input type="checkbox"/>

Viewer Settings

GENERAL

Use Modified Toolbar

Caliper Zoom

INTERFACE

Auto-Play Cine Loops

Auto Play Cine Wait Time (ms)

Default Frame Rate for MP4

Default Outline Thickness

MAMMOGRAPHY

Auto Open Orders

- Referring Providers
- Studies
- Summary
- Transcription
- Vital Signs

OVERLAYS

TOOLS

EXTERNAL

3. Enabling Cardio Viewer and Cine Loops:

- a. Click on Viewer Settings, select Interface
- b. Enable Auto-Play Cine Loops.
- c. Enable Cardio Viewer.

Enable Cardio Viewer

Auto-Play Cine Loops

4. Splitting Static Images from being Stacked Images in Thumbnails:

- a. Click on “Viewer Settings”.
- b. Select “Modality”.
- c. Click “US”.
- d. Scroll to bottom and select checkbox for “Always” under Auto-Split Rules.
- e. Click Save.

