

**symmetry**-Mini PACS

# **Feature Summary**


## **CD Burn**

© 2023 Konica Minolta Healthcare Americas, Inc.

1. On the worklist, select the studies to burn.



2. On the **PACS Actions** menu, click **CD Burn**.

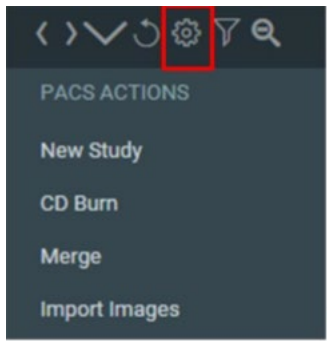

3. In the **Opal CD Burning** dialog box:
  - a. Optional. To include a viewer application, select the **Include Viewer** check box.
  - b. Optional. To write the selected studies to your local hard disk, select the **Write to Disk Only** check box, and then in the **Path** box, select a destination folder.
  - c. Optional. To anonymize studies, select the **Anonymize DICOMS** check box.
  - d. In the **Compression** list, select a type of compression.
  - e. Optional. To store a report with images, in the **Report** list, select a method.
4. Click **START**.

