

**symmetry**-Mini PACS


# **Feature Summary**

## **CD-Burn Images and Series**


© 2023 Konica Minolta Healthcare Americas, Inc.

### CD-burn images and series


1. In user settings, clear the **Show Priors** checkbox.

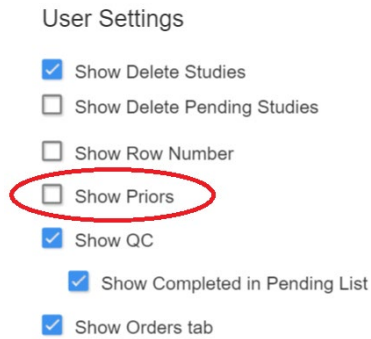

2. On the worklist, find the study containing images or series you want to burn and then select the plus button.

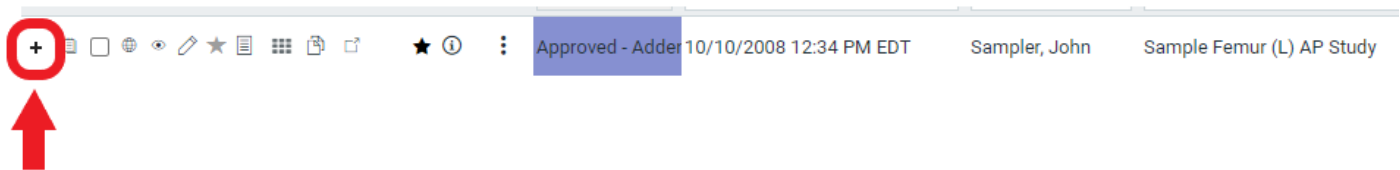

3. Select the plus button again, and then select the picture buttons to determine which images and series you want to burn.

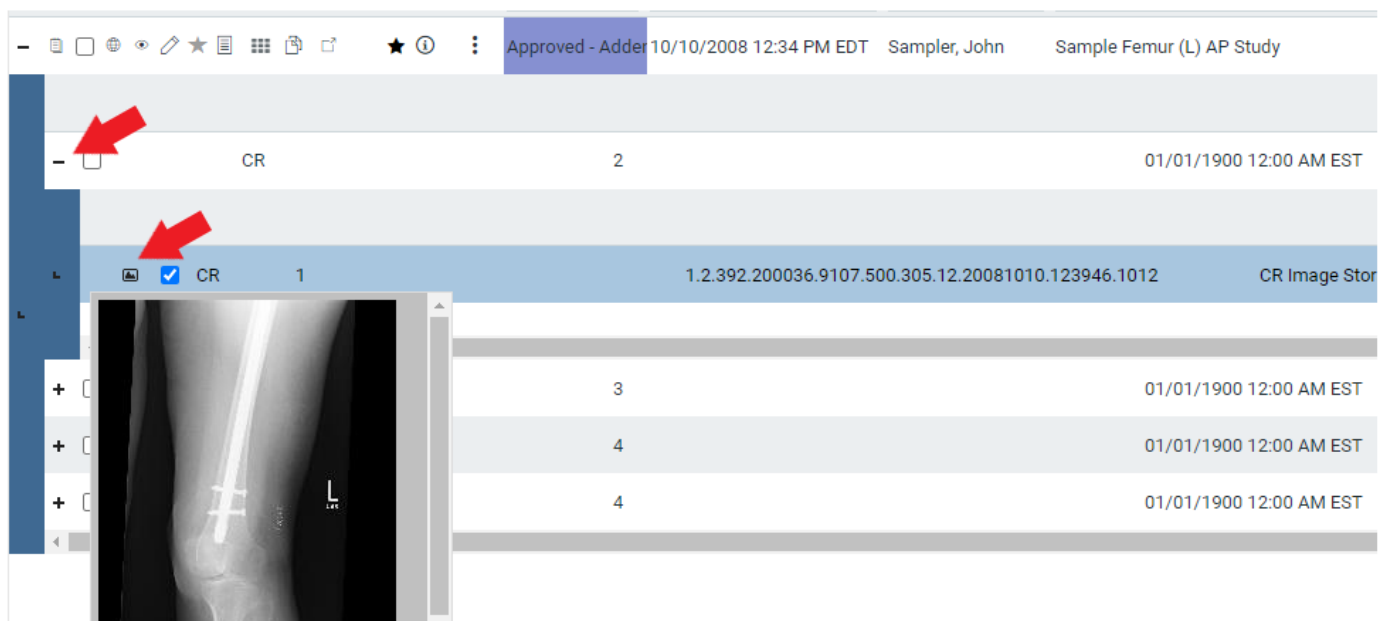

4. Select the checkboxes of the images and series that you want.
5. On the **PACS ACTIONS** menu, select **CD Burn**.

Worklist

TASKS 3 0 STAT 0 0 0 0 0 0 TAT EA Activated

PRE ORDERS QC DICOM STUDIES ALL STUDIES AF\_FONG MY PEER REVIEW

STATUS STUDY DATE PATIENT STUDY DESCRIPTION

+	CR	2	01/01/1900 12:00 AM EST	1.2.392.200036.9107.500.305.12.122008101045826.122
+	CR	3	01/01/1900 12:00 AM EST	1.2.392.200036.9107.500.305.12.122008101045826.123
+	CR	4	01/01/1900 12:00 AM EST	1.2.392.200036.9107.500.305.12.122008101045826.124

PACS ACTIONS

- New Study
- Walk-In Appointment
- CD Burn**
- Compare Study
- Merge/Split
- Import Images
- Purge All Deleted
- Export Worklist

6. In **Opal CD Burning**, modify settings if necessary and then select **START**.

Opal CD Burning

PID	NAME	DESCRIPTION	DATE
CR-Test2	Sampler^John^D^^	Sample Femur (L) AP Study	2008-

Options

- Include Viewer
- Write to Disk Only
- Cache Images
- Create Annotations Overlay
- Anonymize DICOMS
- Remove Annotations
- Fix Split BTO
- Force SOP Class ID for all files

Path: C:\Users\jhodor\Desktop\Single i ...

Compression: JPEG 2000 Lossless

Report: DoNotStore

XML Loaded. Click Start to Initiate Burning Process

Current Step Progress

Total Progress

Close START Cancel