


Symmetry Mini PACS

Feature Summary


Merge Patient Accounts

1. On the worklist, select the edit button  of a study belonging to a patient to merge.

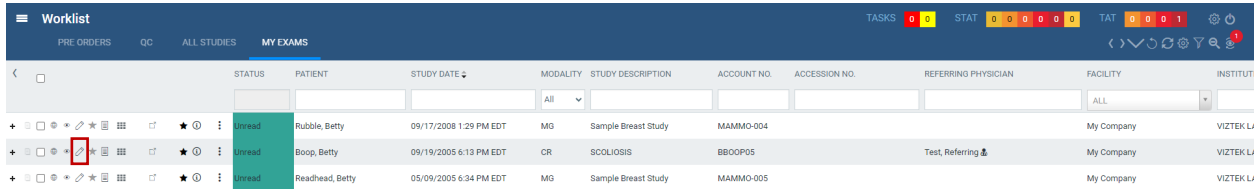


2. In the **Edit Study** screen, select **PATIENT INFORMATION**.

3. In the **DICOM PATIENT ID** box, type the account number of the destination patient (the patient to merge the current patient into). Click outside of the box.

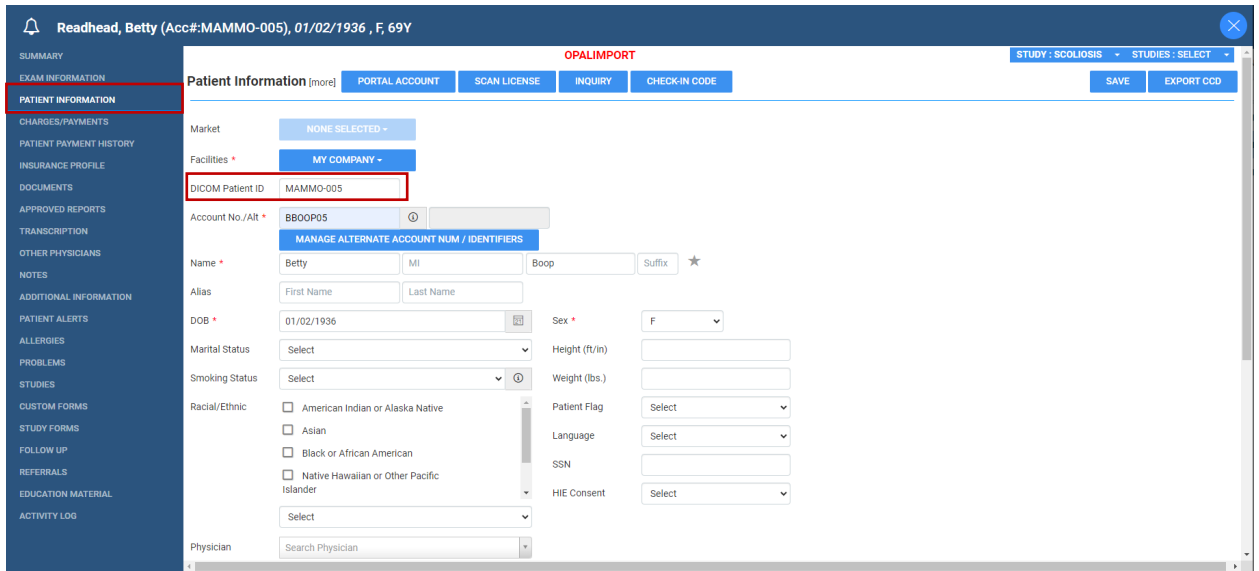

4. To merge the patient account on all studies, select **SAVE**.

To merge the patient account on selected studies, select the studies from the list and then select **SAVE SELECTED**.

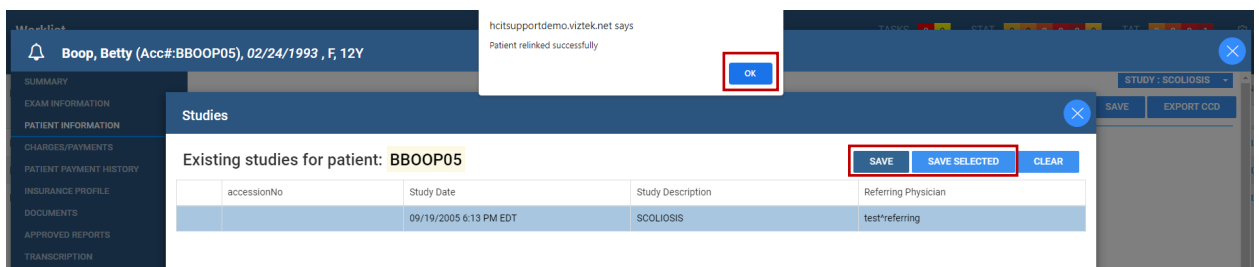

5. Select **OK** to confirm.