
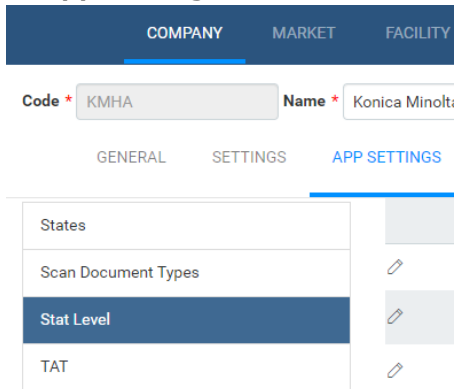


# **Symmetry Mini PACS**


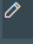

## **Feature Summary**

### **Configure Stat Levels**


1. Sign in to Symmetry Mini PACS as an administrator.
2. On the burger menu,  select **Setup > Office > Company**.
3. On the **App Settings** sub-tab, select **Stat Level**.



3. Select the edit (pencil) button to edit the description or color.

	ACTIVE	LEVEL	DESCRIPTION	COLOR
	<input checked="" type="checkbox"/>	0	Stat Off	
	<input checked="" type="checkbox"/>	1	Stat 1	<span style="background-color: #e9bb34; padding: 2px;">#e9bb34</span>
	<input type="checkbox"/>	2	Stat 2	<span style="background-color: #f39330; padding: 2px;">#f39330</span>

4. Optional. At the bottom, type a description.

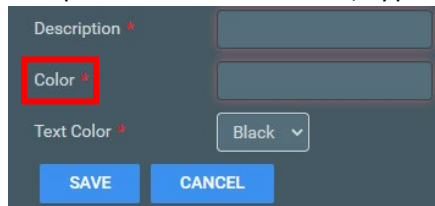

5. Optional. To add a color, click inside the color box and then use the color pickers to select a color.



6. Select **SAVE**.