

Exa® PACS/RIS

Workflow Guide


Worklist and Viewer

©2023 Konica Minolta Healthcare Americas, Inc.




User settings

To open user settings, select the **Settings** button in the upper toolbar.

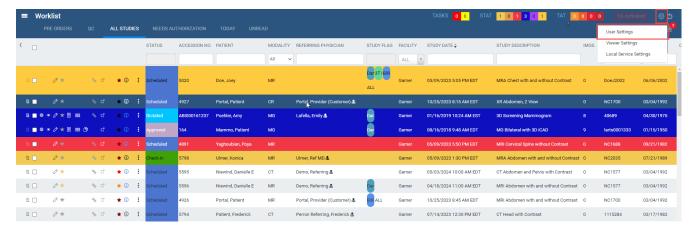


User settings control how the worklist appears and functions. These settings are user-specific; they only apply when the user who configured them signs in.

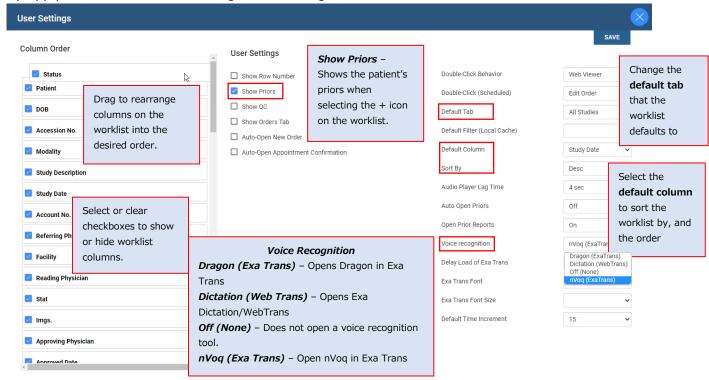




Worklist

Customize the worklist's appearance and functions.

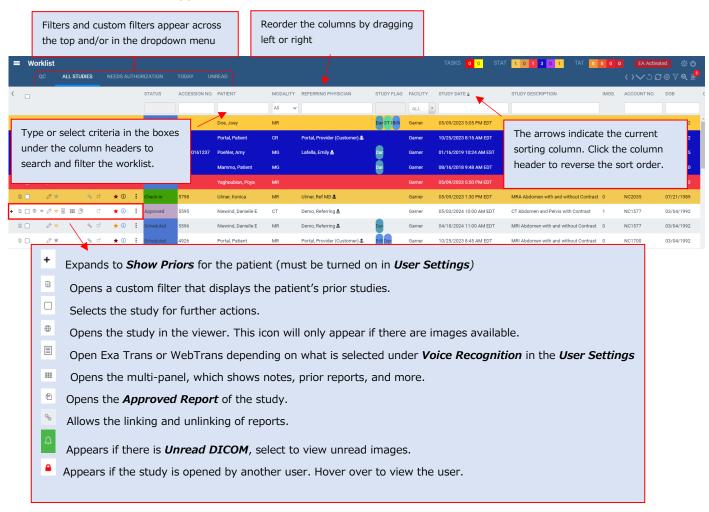

Upper and lower toolbar

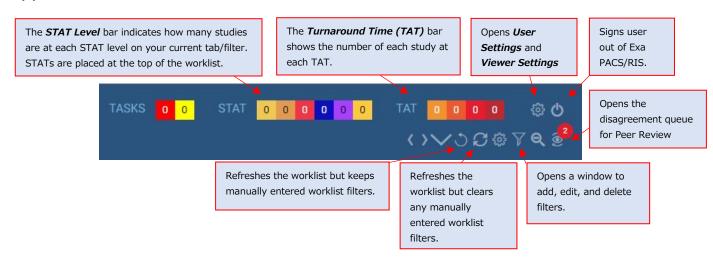



Create a filter


1. To create a filter, select the **Filter** button on the lower toolbar.



2. In the **Study Filter** screen, select **Add**.




3. Enter the Filter Name and Filter Order first. Then enter other criteria for the filter.

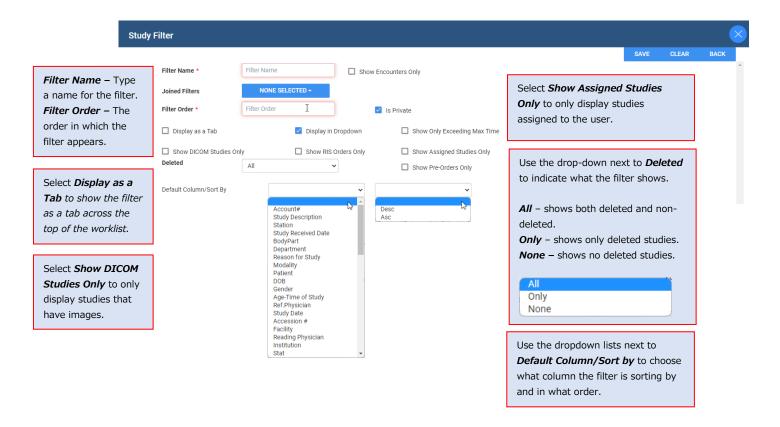




Date/time

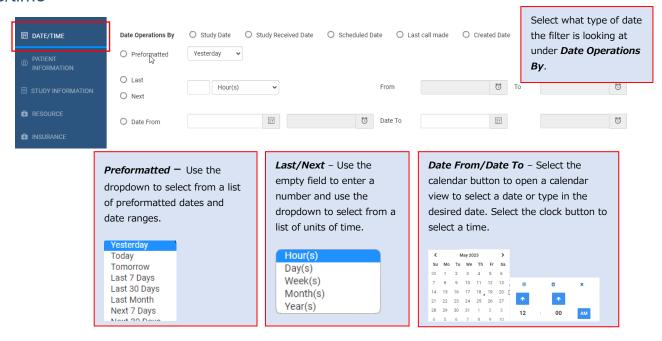


Patient information

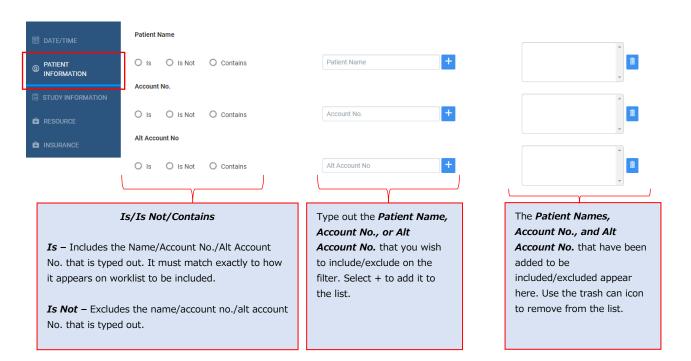




Study information

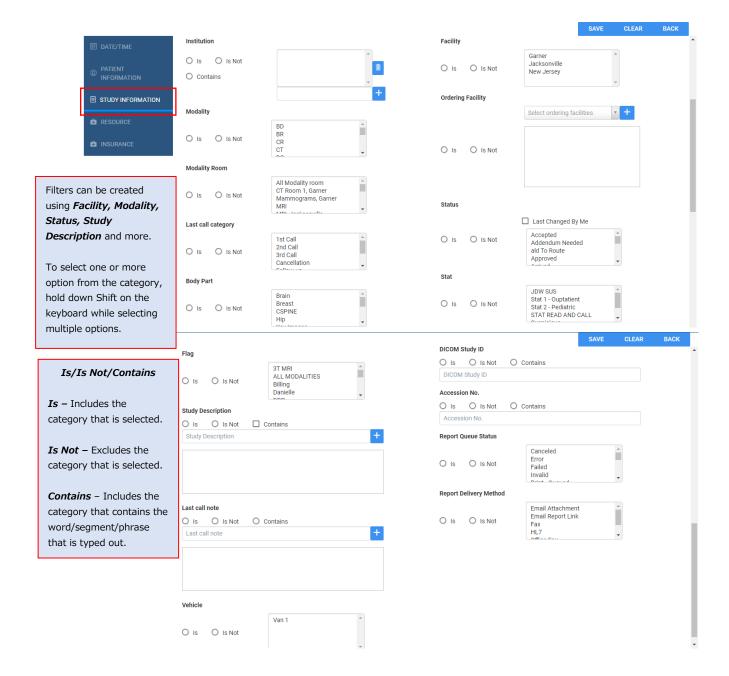




Resource

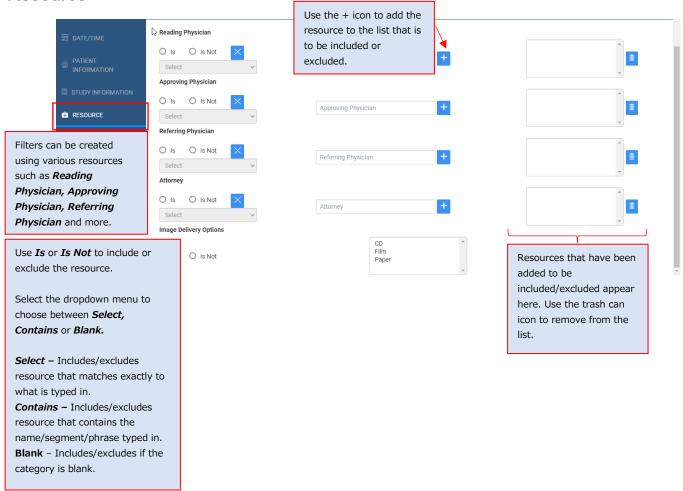


Insurance

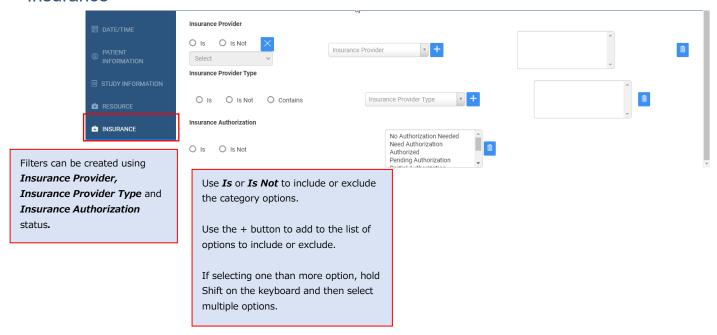



Basic viewer navigation

To open the Exa PACS/RIS viewer, select the **DICOM Viewer** button or double-click on the study on the worklist.

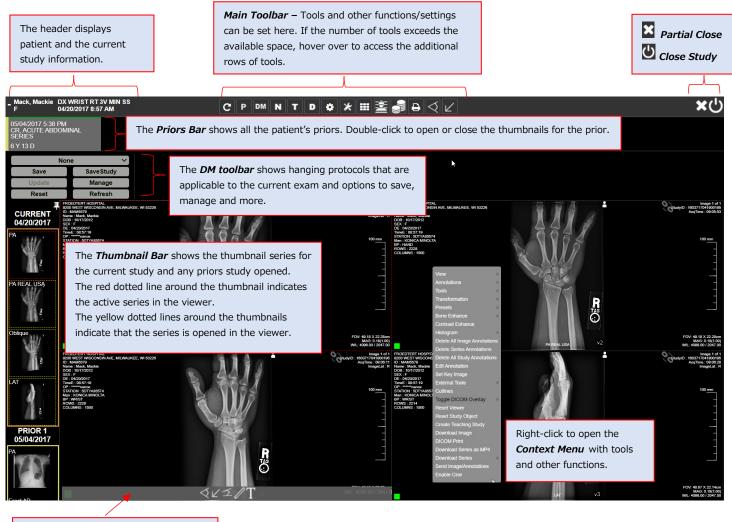

Hover over the top, left, and bottom of each frame opens floating toolbars.