

Table of Contents

Introduction
Symbols 3
Indications for use
Training4
Regulatory and compliance
System requirements
Calibration and measurement accuracy
New in version 1.4.33_P07
Getting started
Configure your working environment
Configure Exa PACS/RIS
Work with the hamburger menu 208
Work with the dashboard
Work with the worklist
Work with dictation and transcription
Work with the Exa PACS/RIS viewer
Work with Display Management
Work with Peer Review
Work with Patient Kiosk
Work with the patient chart
Work with mobile radiology
Work with query and retrieve
Work with the fax manager: Updox
Work with scheduling
Work with billing
Work with reports
Work with Exa Chat
Work with tools
Work with Patient Portal
Work with Physician Portal
Appendix
Reference

Introduction

Symbols

The following symbols may appear in the product documentation or on the product.

Symbol	Symbol Name	Symbol Description	Standard Number and Name	Symbol Reference Number
** *	Manufacturer	Indicates the name and address of the manufacturer	ISO 15223-1:2021	5.1.1
EC REP	Authorized Representative in the European Economic Area (EEA)	Indicates the Authorized Representative, responsible for the device in the European Economic Area (EEA).	ISO 15223-1:2021	5.1.2
$\sim \sim$	Date of Manufacture	Indicates the date when the device was manufactured.	ISO 15223-1:2021	5.1.3
\triangle	Caution	Indicates information that is important for preventing loss of data or misuse of the software.	ISO 15223-1:2021	5.4.4
LOT	Batch Code	Indicates the full Software Release / Version number	ISO 15233-1:2021	5.1.5
SN	Serial number	Indicates the manufacturer's serial number so that a specific medical device can be identified	ISO 15233-1:2021	5.1.7
REF	Catalogue Number	Indicates the manufacturer's catalogue number so that the device can be identified	ISO 15233-1:2021	5.1.6
i	Consult instructions for use	Indicates the need for the user to consult the instructions for use	ISO 15233-1:2021	5.4.3
R _{LOnly}	Prescription Device	Caution: Federal law restricts this device to sale by or on the order of a licensed healthcare practitioner	21 CFR 801.109(b)(1) Prescription Devices	N/A

BS EN ISO 15223-1:2021 Medical devices - Symbols to be used with information to be supplied by the manufacturer - Part 1: General requirements

Indications for use

EXA[™] is a software device that receives digital images and data from various sources (i.e. CT scanners, MR scanners, ultrasound systems, R/F Units, computed & direct radiographic devices, secondary capture devices, scanners, imaging gateways or other imaging sources). Images and data can be stored, communicated, processed, and displayed within the system and or across computer networks at distributed locations. Lossy compressed mammographic images are not intended for diagnostic review. Mammographic images should only be viewed with a monitor cleared by FDA for viewing mammographic images. For primary diagnosis, post process DICOM "for presentation" images must be used. Typical users of this system are trained professionals, nurses, and technicians.

Training

Users of this software must have received adequate training on its safe and effective use before attempting to operate the product described in this Instructions for Use. Users must make sure they receive adequate training in accordance with local laws or regulations.

Regulatory and compliance



Konica Minolta Healthcare Americas, Inc. 2217 U.S. Highway 70 East Garner, NC 27529 USA

Tel: 1-800-366-5343

System requirements

The following are the minimum system requirements for clients of Exa PACS/RIS. The workstation requirements are based on typical usage scenarios. For the recommended requirements of your specific implementation, contact your Konica Minolta representative.



RECOMMENDED: To avoid loss of functionality and data during a power loss, connect an uninterruptible power supply. When running the system by the UPS alone, save data immediately after modifying.

REQUIRED: Install anti-virus software on the server and each client.

Server

Component	Specification
Processor	Intel® Xeon® Gold 5120 CPU @ 2.20GHz
Cores	14
Memory	32 GB
Archive Storage	1 TB partition DAS
	7,200 RPM hard disk drive
OS	Windows Server 2022
	C:/ 250 GB SAS mirrored OS drive
Cache	500 GB SSD
Database	500 GB SSD
NIC	Single 1000 MB/s

Workstation - User

Component	Specification
CPU	Intel® Core™ i5 or later
RAM	8 GB or more
NIC	Single 1000 MB/s
Storage	HDD, 500 GB or more
OS	Windows 10 Pro, 64-bit
Monitor	20 inch, 1600 × 1200

Workstation - Reading Radiologist

Component	Specification
CPU	Intel Xeon® E-2176G (3.70 GHz, 12 MB) or later

Video Cards	2 NVIDIA® Quadro® P1000 cards
RAM	32 GB or more
NIC	Single 1000 MB/s
Storage	SDD, 500 GB or more
OS	Windows 10 Pro, 64-bit
Monitor	Diagnostic displays

Calibration and measurement accuracy

Measurement accuracy partially depends on image quality, which is subject to various factors including the skill of the technologist, the precision of the modalities, and image resolution. However, clinical users of Exa PACS/RIS can help ensure diagnostic image quality and accurate measurements by using appropriate viewer settings (such as window/level and zoom) and by calibrating monitors (see "Calibrate monitors").

The clinical user is responsible to judge the accuracy of the measurements based on the image quality and based on the accuracy of placed measurement points. For a table of measurement accuracies, see "Measurement Accuracy Limits" in the appendix.

Help us improve!

If you find any errors in this manual, would like us to explain something better, or would like us to cover a new topic, please let us know by visiting our online portal.

https://support.kmhapub.com/PremierCustomer/Home.aspx

Monitored: 8:00 AM-8:00 PM EST

For urgent issues, please call the Hotline.

New in version 1.4.33_P0

For an introduction to what's new in this release, see the document, New Features and Improvements. For line item changes, see the release notes.

Getting started

This chapter contains the following sections.

<u>Configure your working environment</u> <u>Configure Exa PACS/RIS</u>

Configure your working environment

Before using Exa PACS/RIS, you must configure settings and information used in various parts of the program. This section helps with the most common areas of basic configuration. For help with other detailed settings, contact your Konica Minolta representative.

Set up credentials, browser, and monitors

To set up credentials, browser, monitors, and other settings, see the following topics in this section.

Configure the language in Chrome Turn off Chrome autofill Sign in to the Exa platform About two-factor authentication Sign in using 2FA with email Sign in using 2FA with Google Authenticator Reset your password Change your password Add an avatar Edit your user profile View version information Create a shortcut for Exa PACS/RIS Calibrate monitors

Configure the language in Chrome

You must configure the browser language of your client computer as follows.

Procedure

- 1. In Chrome, select the customize [‡] button, and then select **Settings**.
- 2. On the left pane, select Languages.
- 3. If the local language and culture does not appear in the list, select **Add languages**, and then add the language.

Note: The culture must be correct. For example, in Canada, ensure that English (Canada) is installed.

- 4. Select the more actions [‡] button for the local language, and then select the **Display Google Chrome in this language** checkbox.
- 5. Select the more actions ⁱ button again, and then select **Move to Top**.
- 6. Select Relaunch.

Turn off Chrome autofill

Chrome may automatically fill in passwords, patient information, and other text in various parts of the program. This can be a security, privacy, and functional risk, therefore we highly recommend turning off these features.

Passwords

- 1. In the Chrome menu, select **Settings**.
- 2. Select Autofill and passwords > Google Password Manager > Settings.
- 3. Turn Offer to save passwords off.

Payment methods

- 1. In the Chrome menu, select **Settings**.
- 2. Select Autofill and passwords > Payment methods.
- 3. Turn off Save and fill payment methods.

Addresses and more

- 1. In the Chrome menu, select **Settings**.
- 2. Select Autofill and passwords > Addresses and more.
- 3. Turn off Save and fill addresses.

Sign in to the Exa platform

You can sign in to the main application, the patient portal, the physicians' portal, the ordering facility portal, or the attorney portal.



Procedure

- 1. In Chrome, go to the URL given to you by your Konica Minolta representative.
- 2. Enter your sign-in credentials.
- 3. Select the I agree to HIPAA Privacy Notice checkbox.
- 4. Do one of the following:
 - To sign in to Exa PACS/RIS, select **LOGIN**.
 - To sign in to a portal, select a portal button.

See also:

About two-factor authentication Sign in using 2FA with email Sign in using 2FA with Google Authenticator

About two-factor authentication

If your facility purchased two-factor authentication (2FA), administrators can use it to increase security by requiring an additional sign-in procedure. During sign-in, Exa PACS/RIS sends an *authentication token* (*or "code"*) to your email address, or one is generated by the Google Authenticator app. You then enter that authentication token on the Exa PACS/RIS sign-in screen to finish signing in.

Administrators: To make two-factor authentication available for a user, configure the user's email address and user name. For further details, see the configuration manual.

See also:

<u>Create a user</u> <u>Sign in using 2FA with email</u> <u>Sign in using 2FA with Google Authenticator</u>

Sign in using 2FA with email

If your administrator made two-factor authentication available, you will receive an email from the server with a verification token or code. Using that code, follow these steps.

Procedure

1. Sign in to Exa PACS/RIS in the usual manner.

2. In the EMAIL VERIFICATION PENDING box, paste the verification code and select VERIFY.

Result: Exa PACS/RIS signs you out.

- 3. Sign back in to Exa PACS/RIS, select the email option, and then select **REQUEST TOKEN**.
- 4. Type the token sent to the email address, and then select **LOGIN**.



Note: Every time you sign in the future, you must repeat this step, using a different token each time.

See also:

Sign in to the Exa platform About two-factor authentication Sign in using 2FA with Google Authenticator

Sign in using 2FA with Google Authenticator

After you have signed in using 2FA the first time, if you prefer, you can use Google Authenticator to sign in the future rather than an email token. To use Google Authenticator to sign in, follow these steps.



Caution: Follow these steps correctly or the setup key (the text portion of the QR code) may become useless for subsequent users.

Procedure

- 1. Download the **Google Authenticator** app onto your mobile device.
- 2. In Exa PACS/RIS, on the burger 🔜 menu, select the blue Konica-Minolta 🛄 globe.
- 3. In the **My Profile** dialog, in the **Two Factor Authentication** box, type your account password, and then select **SHOW QR CODE**.
- 4. In Google Authenticator, tap Scan a QR Code, and then scan the code on screen.
- 5. In Exa PACS/RIS, in the **One-Time Password** box, enter the code from your new Google Authenticator profile, and then select **VERIFY**.

See also:

About two-factor authentication Sign in using 2FA with email

Reset your password

If you forgot your password, contact your Exa PACS/RIS administrator for instructions on how to reset your password.

Change your password

You can change your sign-in password.

Procedure

- 1. Select the burger \blacksquare button.
- 2. At the top of the burger menu, select the blue Konica Minolta globe:



(user avatar).

- 3. In the My Profile dialog, select CHANGE PASSWORD.
- 4. Enter current and new passwords, and then select **SAVE PASSWORD**.

Add an avatar

An avatar is an image such as a photo that represents you in various parts of the program.



Prerequisite: Prepare a PNG image to use as the avatar.

Procedure

- 1. At the top of the burger 🔲 menu, select the blue Konica Minolta Solution.
- 2. In the My Profile dialog, select Choose File.
- 3. Browse for and select your avatar image, and then select **Open**.
- 4. Select SAVE.

Edit your user profile

Each user of Exa PACS/RIS has a profile with sign-in credentials and other information. After signing in, you can view and edit your information.

Procedure

- 1. Select the burger \blacksquare button.
- 2. At the top of the burger menu, select the blue Konica Minolta globe or user's avatar:



🔼 (user avatar).

Result: The My Profile screen appears.

My Profile				EXA Version UDI (01)008	
User Name	stak.				
Name *	Pace	M	Viztek	Buffer	- r
Mobile Phone / Ema	Mobile Phone	Trool			- 1
Default Device *	• 🗆 AU	to Open Dictation	on Device		
Default Location *	My Company	<u>(</u>	Always Open !	Schedule Book in a New Tab	
Rows to Display	50 ¥ Theme	n default v	Culture English-U	s 🗸 🖾 Pin Busger Menu	
	🗆 Hide Worklist Ic	ona 🗖 Hide Ori	let Menu		
Bandwidth	Select	•			
Session Interval	300 min (10-6	600)			
Default Scheduling	Radiology				
					SAVE

3. Enter the following settings.

Setting	Description
User Name	Type a user name for sign-in purposes.
Name	Type your true name.
Mobile Phone/Email	Type your mobile phone number and/or email address. An email address is required for two-factor authentication.
Default Device	[Unused]
Auto Open Dictation on Device	[Unused]
Default Location	Select a facility as your default location. When you sign in, this facility appears in various parts of the program by default. You must configure the facilities that you want to make available in this dropdown list.
Always Open Schedule Book in a New Tab	Select to automatically open a new browser tab for the schedule book when you open it.
Rows to Display	Select how many rows to display on the worklist.
Themes	Select a default theme of Bright (default) or Dark.
Culture	Select the language/culture of the user interface.
Hide Worklist Icons	Hides the lower toolbar and study row buttons on the worklist. Hidden functions remain available in shortcut menus and other controls.
Hide Order Menu	Hides the navigation menu within the Edit Study screen.

Bandwidth	Select the expected speed of the network on which your client installation runs. This helps to optimize performance in your networking environment.
Session Interval	Type the number of minutes to elapse before the program times out and returns to the sign-in screen.
Default Scheduling/Radiology	[Unused]

View version information

You can view version information about the application, host system, services, and external tools.

Procedure

- 1. Select the burger 🔲 button.
- 2. At the bottom of the burger menu, select the white Konica Minolta globe.



Result: The About screen appears.

About	
App/DB/Misc	Services
Exe: 1.4.32-p3-135 Node: v16.16.0 PostgreSQL: 14,4 NGINX: 1.23.1 Redis: Not Found TxTransportation: Not Found	C-Move Service: 1,4.32,50 DICOM Service: 1,4.32,50 Fax Service: 1,4.32,50 ffmpeg: Not Found imageserver_http:= 1,4.32,50 Image Service. Not Found
System OS Type: Windows_NT OS Platferm: win32 OS Architecture: x64 Total System Memory: 12.00 GB	MWL Service: 1.4.32.50 Opal Listener Service: 1.4.32.50 Opal Sender Service: 1.4.32.50 Dicom Print Service: 1.4.32.50 Storage Commitment Service: Not Found Tools
	Opal Tools Setup: Not Found Opal Viewer Setup: Not Found EXA Trans: Not Found EXA Dec Scan: 1.0.11.0

Create a shortcut for Exa PACS/RIS

You can create a shortcut for the local Windows user to Exa PACS/RIS and place it on the desktop.

Procedure

- 1. In Chrome, go to the Exa PACS/RIS sign-in page.
- 2. In the address bar, drag the lock 🔎 icon onto the desktop.

The Exa Login shortcut appears.

- 3. Optional. To change the shortcut icon:
 - Right-click the shortcut and then select **Properties**.
 - Select **Change Icon**, and then browse for and select an icon.
 - Select OK.

Calibrate monitors

Especially after a new installation or upgrade, you can use Exa PACS/RIS to calibrate your monitors to ensure accuracy of length and other measurements. If all images you work with contain pixel spacing information, you can skip this procedure.

Prerequisite: Enter Calibration Width and Calibration Height settings for your monitors. See <u>Set up</u> <u>connected displays</u>.

Procedure

- 1. Open an image in the viewer.
- 2. Optional: For higher accuracy, on the image shortcut menu, select **View** > **Actual Image Size**.
- 3. Hover over the upper-right corner of a frame, right-click **=**, and then select **Recalibrate**.
- 4. Select two points on the image.
- 5. In the dialog, type the length between the points and then select **OK**.



Note: If prompted to apply calibration to the series, select OK.

Results

- The calipers are modified.
- Previously annotated measurements are redrawn.
- Future measurements will appear according to the new calibration values.

Configure Exa PACS/RIS

This chapter includes the following sections with topics on core items to review or configure prior to using Exa PACS/RIS.

Set up the office: Companies Set up the office: Other assets Configure providers and resources Configure scheduling and codes Configure DICOM settings Configure billing Manage users Configure general settings Configure the viewer

For other settings, see the following topics.

See also:

Worklist settings User settings

Set up the office: Companies

Company here means the parent hospital, clinic, or other institution where Exa PACS/RIS is installed. Configure your company and then configure its child facilities (see <u>Add a facility</u>) and other locations as needed. In configuring your company you enter basic settings such as its name and address, but also depending on product and configuration—you can customize Exa PACS/RIS for your company's needs by adding preset options for such things as: sex, body parts, and ethnicity; patient alerts, critical findings, cancellation reasons; MRN and accession number formatting; passwords; and billing modifiers.

This section contains the following topics (not all topics may be available depending on version and region).

Configure your company General settings Settings App settings Reason codes MRN information Accession information AE filter Payment gateway LDAP configuration RCopia configuration HL7 Password management Modifiers Link encryption

Configure your company

To configure your company:

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Office** > **COMPANY**.

=	w Negation	Hereiter	(sectore)s		(marticle without	servicania	Mare		
Cale I ul	Nere '	MyCompany.	Timetane	- USERNIN	•0			tervine 🖸 Endle Paramet	
D Trigger Hnateg et			110001000					in an 12 forthe statute	

2. Enter the following settings.

Setting	Description
Code	Type your internal code for your institution.
Name	Type the name of your institution.
Timezone	Select the time zone used by your institution.
Enable LDAP	If your institution uses an LDAP or Active Directory server, contact your Konica Minolta representative to configure these settings.
Enable RCopia Service	Select to turn on the RCopia service, if available, and display the RCOPIA CONFIG sub-tab. Enter settings in the sub-tab, and then select SAVE.
Enable Payment Gateway	Select to turn on the Payment Gateway service, if available, and display the PAYMENT GATEWAY sub-tab. Enter settings in the sub-tab, and then select SAVE.
Trigger Routing on Study Flag Changed	Select to reprocess routing (DICOM transactions) automatically when a user changes a study flag.

3. Select SAVE.

4. Enter additional information as needed by following the steps in later subsections.



Note: When finished, select **SAVE** again at the top of the screen on the right side of the **Trigger Routing on Study Flag Changed** checkbox.

General settings

Enter the main contact information for your company and other basic settings relating to scheduling, billing, documents, security, image viewing, and peer review. For forms, email and notification templates, db tools, report queue, and other topics, see <u>Configure general settings</u>.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Office** > **COMPANY** > **GENERAL**.

			0001 1001121Da	THE DOLLAR HOUSE	
00638	the internal	ADTINGS REASON COLOR WATERATING ACCESS	Strend and the second	TITUE +47 Description	ពាមកំណ
Lonin-	Dial of the second s				
					_
Centers Esfa marbe			Docomenta		MAT ALL THE
(withink)	1013 (seg 108		114 5254	Scotlast M	
aldress lates 1			Second		
12/10/07/201	minută	un - Inner	Assessed frankright		
hote Ne			menhod		
t-st					
			Video		
Advision					

2. Enter the following settings.

Caution: A password is configured or used on this page. For recommended security, <u>turn off</u> <u>Chrome autofill</u>.

Setting	Description		
Contact Information	Enter the company address and conta	ct information.	
Scheduling	Highlight new Patient exams on ScheduleBook	Select to highlight new patient appointments on the schedule book for easier identification.	
	Highlight Color	Select the color to use to highlight new exams on the schedule book.	
	Default Distance from ZIP Code	Select the default distance for finding facilities near patients.	
	Enable schedule rule reason	Select to require users to select a schedule reason when creating a schedule block.	
	Enable reschedule reason	Select to prompt for a reason when rescheduling an appointment.	
	Enable pause reason	Select to prompt for a reason when a technologis pauses an exam.	
Billing Information	Corporate Office Type	Select Person for an individual practice. Otherwise, select Non-Person Entity.	

Setting	Description		
	NPI No.	US only. Type your national provider identifier.	
	Taxonomy Code	Type your NPI taxonomy code.	
	Tax ID	Type your national tax ID.	
	EDI Submitter ID	If using an EDI service, type your submitter ID.	
	EDI Receiver ID	If using an EDI service, type your receiver ID.	
	Modifiers in Order	Forces the user to enter modifiers before creating an order.	
	ICD9 to ICD10	Select to automatically convert diagnostic codes from the ICD9 to ICD10 standard.	
	Sales Tax	Type the tax rate at the company's address, as a percentage (e.g. type 07.50 for "seven and one-half percent").	
Documents	File Store	Select a file store for scanned documents at your company. This is separate from the file store used by AEs to store images.	
Security	Account Lockout Threshold	Set the maximum number of sign-in attempts.	
Viewer	Viewer Titlebar Text	Select to display the patient name or the accession number on the title bar when opening a study in the viewer.	
Peer Review	Schedule (how often studies are assigned to be peer-reviewed)	Select the frequency at which studies are automatically drawn and assigned for peer review.	
		To customize, select Custom settings, and then enter the cron settings below.	
	Cron custom settings	Available when you select Custom settings in the Schedule dropdown list. Type a custom frequency for assigning peer reviews by entering values in the boxes: seconds, minutes, hours, day of the month, month, and day of the week.	
	Peer Review Percentage of yearly approvals	Type the percentage of approved studies to add to the initial draw pool each draw.	
	Studies per draw amount	Type how many studies to draw from the draw pool. These studies are distributed randomly to peer reviewers.	
		Any studies left over (that are not drawn) remain in the draw pool for the next draw.	
	Cutoff period	The number of days in the past from which Peer Review draws studies (the maximum age of the study).	
	Interval	Type how often to check whether the conditions are met to start the draw process, in milliseconds.	
Email Information	Server Address	Type the address of your outgoing SMTP/mail server.	
	Port	Type the mail server port number, provided by your ISP.	

Setting	Description	
	User Name	Type the account administrator user name or email address.
	Password	Type the email account password.
	Sender Address	Type the default sender address (the administrator's email address).
	Email Subject	Type a default email subject to use if no subject is configured on the notification template.

4. Select **SAVE ALL INFO**.

Settings

Settings here means presets that appear as options in other parts of the program, mostly related to people (such as staff and patients).

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Office** > **COMPANY** > **SETTINGS**.

			kinn = 1	(Maganaria)
Exto 7 Mil	Barne	No.	Concern	0.000
anairu Uan Charles	ativnes.		ine Inela	494711225
Ter Titer			(ARCH)	3844
Madel Status	1 1 1	8	46	
tan.	10	8	ma	
Budy Forts	11	6	811	
training the law	1.01	影	305	
Cesteviule				
facial there				
Tri-siply-				
Balatanship	Onire	ptoo :	0	
Increase		* 1125		
Source				

- 2. On the left pane, select a category.
- 3. At the bottom of the screen, in the **Description** box, type a setting. The following table describes the setting for each category.

Setting	Description
User Titles	User prefixes, such as Mr., Mrs., and Dr.
Marital Status	Patient marital statuses, such as Single, Married, and Divorced.
Sex	Patient sexes.
Body Parts	Patient body parts under examination.
Employee Status	Statuses of employees at your institution.
Credentials	Physician credentials, such as Ph.D. and M.D.
Racial Identity	Patients' self-identified races.
Ethnicity	Patients' self-identified ethnicities.
Relationship	Relationships of people to patients (including self).
Priorities	Priorities that can be assigned to studies in the worklist. These are separate from stat levels.
Sources	Sources of patient arrivals, such as clinic referral, attorney office, hospital transfer, or emergency room.
Orientation	Lateralities as they pertain to the study.
Languages	Patients' preferred languages.
Specialties	Healthcare worker specialties such as oncology or sports medicine.
Communication Preferences	Patients' preferred methods of communication, such as cell phone or email.
Administration Site	Immunization modes.
Regions	Geographical regions, such as for grouping markets.
Units of Measure	Units of measure, such as for doses of medication or contrast material.
Needle Gauge	Available needle gauges.

App settings

App settings here means presets that appear as options in other parts of the program.

Regarding stat levels

Because different PACS systems use different stat levels, you need to translate the stat level of inbound studies to the stat levels you configure in Exa PACS/RIS, and conversely, you need to translate the stat levels of studies you send to other systems to their stat levels. To do this, as described in the table below, select the value of the (0040,1003) Requested Procedure Priority tag to translate "from" in inbound DICOMs, and to translate "to" in outbound DICOMs.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Office** > **COMPANY** > **APP SETTINGS**.

= officie	WHEE	: (ABLET.	H200,7V	voorm mov	:moresco
cule (in)	Marro	- 10	Gargere	Tarvaire	+ Instalies	
D transcontinue	астични	12	SAVE 1			
apalaza, arri		APP.	CTT H KO	ADON TOGS	him bi mos/s	n sinta
LPHID CONTANT						
States .				TUR		
Scellmaren'yas	1	-1	- 12			
Settant	1		146			
147	10		. R			
Television and Marchines	1		10			
Access Sectors.	2		64			
Draking Kalifu Tuno	1		100			
and the second s	1		11			
	i+m Lowe	()	V 1		. 1044	Autor)

- 2. In the list on the left side of the screen, select a setting.
- 3. For each option in the following table, enter settings.

Setting	Options	Description
States	Description	Type states, provinces, prefectures, or other regions used in various parts of the program.
Scan Document Types	Description	Type descriptions to assign to documents to be scanned, such as "Photo ID" or "Explanation of Benefits."
	Requires Review	Select to require review of the document before moving to the next status in the current workflow. See <u>Require document review</u> .
Stat Level	Description	Type descriptions for each of the stat levels (0–5).
	Color	Click in the text box, and then use the color picker to select a background color for studies on the worklist having the stat level.
	Text Color	Select a text color for studies on the worklist having the stat level.
	Inbound DICOM	Use to "translate" the DICOM Requested Procedure Priority (0040,1003) of inbound studies to the stat levels you configure in Exa PACS/RIS.
		Select the requested procedure priority of inbound studies to assign to the stat level you are configuring.
	Outbound DICOM	Use to "translate" the stat level of outbound studies to the corresponding DICOM Requested Procedure Priority (0040,1003).
		Select the requested procedure priority to assign to outbound studies that are in the stat level you are configuring.
ТАТ	Description/ Colors	Select the edit con of a TAT to modify its description, text color, and background color. You can set the max TAT for a facility under Setup > [all uppercase menu] > Office > Facility.
Patient Arrival Wait Times		ving settings to configure color-coded descriptions corresponding to lengths of atient waits upon arrival. These bands of time appear in the wait time monitor


Setting	Options	Description
	at the top of	the Patient Arrival Worklist.
	Description	Type a description for the band such as "Short Wait" or "Long Wait."
Color Background color for arriv		Background color for arrivals at the current wait time.
	Text Color	Text color for arrivals at the current wait time.
	Minutes	Type the maximum number of minutes that a patient waits to remain in the band.
Access Reasons	Reasons for a	accessing confidential patient records when using the Break the Glass function.
Code Type an inte		Type an internal code for the reason.
	Description	Type a name or description of the reason such as, "Direct patient care."
Ordering Facility Types	Description	Type a description such as "Assisted Living" or "Prison."

Reason codes

Reason codes are text strings that identify reasons for taking various actions such as cancellations, followups, and rescheduling. You can preconfigure them for use in various parts of the program.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Office** > **COMPANY** > **REASON CODES**.

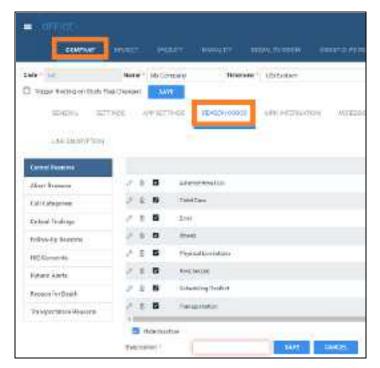

2. On the left pane, select a category.

- 3. On the right pane at the bottom, type a reason.
- 4. Optional: To make the reason temporarily unavailable, select Hide Inactive.

Setting	Options	Description
Cancel Reasons	Description	Type reasons for such canceling such things as appointments or exams in progress.
Abort Reasons	Description	Type reasons for aborting exams in progress.
Call Categories	Description	Type reasons phone calls, such as to patients for scheduling.
Critical Findings	Description	Type critical findings you can assign to studies, such as "Acute Positive."
Follow-Up Reasons	Description	Type reasons to follow up on a study, such as "Benign but suspicious."
	Code	Type your internal code for the finding.
	Follow Up	Type and select the length of time until the follow-up.
HIE Consents	Description	Type indicators as to whether a patient has consented to release of demographics and medical data through a Health Information Exchange for treatment, payment, and health care operations purposes.
Patient Alerts	Description	Type alerts to assign to patients, such as "Patient is claustrophobic."
Pause Reasons	Description	Type reasons that your company deems acceptable for pausing an exam, such as "Patient Movement."
Reason for Death	Description	Type reasons for death.
Reschedule Reasons	Description	Type reasons for rescheduling an exam.
Schedule Rule Reasons	Description	Type reasons to use when writing schedule rules to make a timeslot unavailable, such as "Machine maintenance."
	Color	Assign unique colors to each schedule rule reason for easy identification on the schedule book.
Transportation Reasons	Description	Type modes of transportation such as "Medical Transport" or "Personal Vehicle."

MRN information

MRN information here means your internal format for MRNs at your company. Exa PACS/RIS can assign MRNs automatically according to the following settings.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Office** > **COMPANY** > **MRN INFORMATION**.

Code + Line	Markin *	My Centerally:	Timaner	US.Seitury
🗍 Tagar Berling on	Dudy Five Diverged	-		
00005	3070440	W 62772604	MARGINE	www.uroesen
UNIT DECKY	TION			
DDR 1a formalitent				
_		,		
_	Detaily (e			
VARIA TAJAN *	Dendy e	Carl		
VMI Trie * Pufe	Dendy		00082469	
UDB Internation UDB Internation Partia Data Partia Data Data Data Data	Detail) (*	C #b		

2. Enter the following settings.

Setting	Description
MRN Type	Select Default to use the default formatting included with Exa PACS/RIS, or Custom to define your own formatting.
Prefix	Type a prefix to prepend to the MRN. To be able to modify the prefix, select Can Edit .
Suffix	Type a suffix to append to the MRN.
Allow Duplicates	Select to allow duplicate MRNs.
Prefix/Suffix Max.	Select Fixed Length (above), and then type or select the maximum length of the MRN prefix and suffix.
Fixed Length	Select to enter a maximum length of the MRN.

3. Select **SAVE**.

Accession information

Accession information here means global modifications you want Exa PACS/RIS to make to accession numbers.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Office** > **COMPANY** > **ACCESSION INFORMATION**.

				M19403.01141	HIRSALISH	
Cods 4 (u)	Sprus	· Mp Celump	Time	ero 1 (Classe)	-1	D Brith COX
🗊 Ta Micht Capta Davis				• Tody Filep Drospet	EME .	
IDN(Ro)	-	an dan sea	1010-1110	a anessister	nov /Keessesi ke	OF-MILES.
Activities	1027 1040	001.(1	12121	HUNDERD		
Alarskaniskorudas						
auto -						
3./fx						
	ske					

2. Enter the following settings.

Setting	Description
Prefix	Type a prefix to prepend to the accession number.
Suffix	Type a suffix to append to the accession number.

3. Select **SAVE**.

AE filter

This sub-tab is currently unused.

Payment gateway

If you selected **Enable Payment Gateway** previously (see <u>Configure your company</u>):

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Office** > **COMPANY** > **PAYMENT GATEWAY**.

100		ā. 5			6	
oden inc			Wy Company			US/Eadern
Trable REaging 5	entes 🖬 E	table Payrte	nt Sateuray 🚦	Trigget R	eating on Situ	rty flag Drunged
Scheek	lette	ii ii	+ art taks	READO	II COCHT	AREN PROCESSION
-						
PROMINENT	ISATE MAT	#COPIE	122494	H2 - 3	SASWOID N	MIT. MITHEREN
		<u> </u>				
lateway information						
Baleway Information			*			
Balency 1						
and a second second			л т			
Balency 1			R F			
Baterwy * Wede *			* *			
Baterney * Wede *	E		*			
Baleney * Vede * Cham10 *	E	201	R			

- 2. Enter settings to make a payment gateway available, such as PayPal.
- 3. Select SAVE.

LDAP configuration

Administrators, contact Konica Minolta for assistance with configuring LDAP.

RCopia configuration

If you selected **Enable RCopia Service** previously (see <u>Configure your company</u>):

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Office** > **COMPANY** > **RCOPIA CONFIG.**

	миекет машту мое	NUTY FOCULTY DOCM COCOL C	HAF BOOM NOTHER ON TH
Code * 1.1.:	Hane * My Company	Timescore * III/Eastern	⇒ 🗍 Enable LDAP
		EON CODED MEN INFORMATION A	CCERSION INFORMATION AFFRITER
RCopia Service Details			
Vendor Name *	Worsden Nierre	RCopie Practice Liter	BDopie Preefics Upor Nes
Vendor Passnerd *	Vetstor Painward	Rdant las	
ANS URL *	https://anasert.drk/st.pory	Rortal System Name	Ppral System Norro
ANS IF S *	https://enscor@.drfirst.co.	Trans interval	5 (prent)
AppNane	PAGS	Notity Interval	B (new)
AppVesion	10	Roceita User 1D	0
Pinctice Name	Prairies North	Receptor Ectoretal User 10	Remon Esternal Uner D
Station	Station.	Wendur Secret Key	Vendor Securi Kay
	SAVE		 Save Request File Bave Response Files

- 2. Enter RCopia service details.
- 3. Select **SAVE**.

HL7

This sub-tab is currently unused.

Password management

You can customize requirements for passwords that users create to sign in to Exa PACS/RIS and other related client apps.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Office** > **COMPANY** > **PASSWORD MGT**.

	· Managate and a management scores we rescard the management
Cade * ML Rave * Uy Screpany 1	Tierenere • UD/Eastern • 🗆 Reactio (DSP
Enable RCopie Service Exable Payment Catoway Trigger Rev GENERAL DETTAICS APPEETTINCS READON PAYMENT GRIENWY DESCRIPTION 16.7	
Paseword Dequirements	Other Settings
Personal Must Bebetween II and 36 Characters	📴 User Mast Charge Password on Nort Legri 1e Chenled by Dafeat
Pasewoode Main Contain Upper and Latient	🗍 Porce Usera to Revet Their Passauch as a fiel fished ale
Passwords Mari Curtan Lowercase Letters	🗇 Apply Passanet Requirements to Pation Pertai
D Paeswoode Mast Contain Namiliers	
Passwords Mast Contain Syndrols or Synsel Characters	

2. Enter the following settings.

Setting	Description	
Password Must Be between…	Type a minimum and maximum num contain.	ber of characters that passwords can
Passwords Must Contain …	Uppercase Letters	Exa PACS/RIS validates passwords
	Lowercase Letters	based on these selections.
	Numbers	
	Symbols or Special Characters	
'User Must Change Password Next Login' Is Checked by Default	Select or clear to assign the default s	tate of the setting.
Force Users to Reset Their Password on a Set Schedule	Select to require password resets of a	a specified interval.
Apply Password Requirements to Patient Portal	Select to use the same password requ	uirements for Patient Portal accounts.

3. Select **SAVE**.

Modifiers

Modifiers are alphanumeric codes that you can append to a CPT or HCPCS code to provide information to payers that helps them process claims. Professional and facility claims can include up to four modifiers per CPT/HCPCS code. Placement of the modifiers is critical for correct reimbursement.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Office** > **COMPANY** > **MODIFIERS**.

	OFTICE - TOOMPA	APRIL)	MERICE	MILINE	MORALITY POD	at steared
Code *	Ac.	Rame *	My Campony	Tiple	anne 1 UC/Easte	
1 tr	at le RC tyro Serie	ce 🖪 tassie Perro	etGanway 🕈	Trope Entry	on block Flag Cha	oper LLVE
	02-624AL	attract N	e istronose	KENDER COD	MROWIT	N RUTARE
AL AFR		ZTUNAY . HELINYA		wa seeta		COPPERS 10
2	ŧπ	10	π			
1	ir	109				
ø	51	41.7	TERAL.			
18	35	342	NOT PRINCIPUL	IS/SERVED		
e 1		122	uceo setunces			
d'		190	CONTRACTOR	minear		
0	TU	140	ears cours	NERT"		
1.000						
Doole 7	1					
		D Implicit				

2. Move to the bottom of the screen, enter the following settings.

Setting	Description	
Code	Type the CPT modifier, or other modifier code.	
Implicit	Select to visually exclude the modifier from submitted claims.	
Description	Type the description for the code.	
Fee Level	Select the fee level in the dropdown list.	
Fee Override	Type the fee override code, which takes precedence over any modifier created previously.	
	To configure an override amount: In the first dropdown list select an operator $(+ \text{ or } -)$, in the box type an amount, and then in the last dropdown list select Value or %.	
M1-M4	Select the modifier number to which to assign the code.	
Display on Schedule Book and Print Order	Select or clear to control visibility of modifiers.	

3. Select **SAVE**.

Link encryption

You can define how Exa PACS/RIS responds when a user clicks a link to a study using an encrypted URL. Settings here must match those agreed upon by the entity creating the encrypted links. You cannot edit link encryption settings, only add and delete them.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **OFFICE** > **COMPANY** > **LINK ENCRYPTION**.

k Enuryption	400	RELOAD
HEY CODE LINK ENCRYTTION, INTEGRATION NAME LINK ENTRY KEY TYPE	NEW EXPLICY	NECEMBER

2. Select ADD.

Link Encryption Type *	Steeple Viewer 👻
Integration Name *	Test
Uok Espiry	🛃 Hour(s) 👻 T
Кеу Туре 👻	AES-102 👻
Key Expiry	C1/31/2023
	CENERATE BACK

3. Enter the following settings.

Setting	Description
Link Encryption Type	Select the screen in which to open the study.
Integration Name	Type the friendly name of the integration (matching the one included in the URL, which identifies the decrypting preshared key and action to take on the decrypted URL).
Link Expiry	Select the checkbox to apply an expiration to the URL, and then use the dropdown list and text box to configure a length of time before expiration.
Кеу Туре	Select a type of encryption for the key.
Key Expiry	Select the checkbox to apply an expiration to the key, and then use the date picker to select an expiration date.

4. Select **GENERATE**.

5. In the dialog, copy the passphrase and initialization text to a safe location, and then close the dialog.



Caution: Do not close the dialog until you copy the information; Exa PACS/RIS will never display it again. If you lose the key, you must create a new one.

6. Send the passphrase and initialization text to the EMR company or other entity creating encrypted links.

Set up the office: Other assets

In addition to company settings, there are many other infrastructure and data assets that you can configure.

This section contains the following topics (not all topics may be available depending on version and region).

Add a market Add a facility Add modalities Add modality rooms Add a group chat room Add a notification Add tasks What are authorization, verification, and estimation? Add authorization rules Create a study flag Set monthly goals Configure macro notes Configure portal links Manage life cycles

Add a market

A *market* is an entity to which you can assign facilities. In other parts of the program, selecting a market makes it easy to select multiple facilities at once.

Procedure

1. Go to $\underline{SETUP} > \underline{OFFICE} > Office > MARKET$.

OFFICE-						
COMPANY	MARKET	FACILITY	MODALITY	MEDALITY ROOM	RECEIP CHAT RO	045
🖪 Hide Inactive					ADD	RELOAD
000E ¢		NAME		REGION		
				, IIA		
<創い		Market1				

2. Select ADD.

Code.*	120 🖾 Inseline
Name *	Test Market.
Region	Report +

3. Enter the following settings.

Setting	Description	
Code	Type your internal code for the market.	
Name	Type the name of the market.	
Region	Select the geographical region in which the market is located.	

4. Select **SAVE**.

Add a facility

Facilities are typically where healthcare services are rendered. You must add at least one facility to your company by following the procedure below.

Common tasks after adding a facility

After adding your facilities by using the procedure below, see the following topics on other tasks you may need to complete.

- <u>Configure study statuses</u>
- <u>Configure application entities</u>
- <u>Configure a file store</u>
- <u>Configure routing rules</u>
- <u>Configure AE scripts</u>
- <u>Understanding receiver rules</u>
- Configure a receiver rule
- <u>Configure study flow</u>
- Add modalities
- <u>Add modality rooms</u>
- Add a report template
- <u>Configure a schedule template and schedule rules</u>
- <u>Configure appointment types</u>
- <u>Create a user</u>
- <u>Configure an ordering facility</u>

Procedure



Caution: For recommended security, please <u>turn off Chrome autofill</u>. A password and/or patient information is configured or used on this page.

1. Go to $\underline{SETUP} > \underline{OFFICE} > Office > FACILITY$.

							0
9	OMPANY	MARKET	FACILITY	MODALITY	MODALITY ROOM	GRDI/P O IA	
Hide i	nactive					(ADD.)	RELOAD
CODE :			NAME			MARKET	
						All	
②唐 AD			sinks	Pecility 1			
CE ang1			Angie F	aciity			
2 🕅 F120			Meeral	Facility			

2. Select **ADD**, and then enter the following settings for the facility.

DAda =	C Inclin		dataves Line 1			
lana i	1		Emiliane Line 2			
Marter	Date Value [4]		ChyciPenti23P		Salare & 20 Cests	
Consult No.			FileDove		~	
he he			0	Appelline ell'appeintri	ent tages in this facility.	
Dans Reports to Fax	0		0	C Stow Peters Alena		
ua o e aporta ta - a e	<u>u</u>		0	D toos Pecers Schedurer		
			C trasis sestimater tegenvine			
Enel			0	Moti is Rod Lispatching	1.519000	
Enai Kepan Low	0		D to Net Star Durity programmers in Israelant			
Terrort Passo and			 Import Documents into Study as DIODXIs Onder Autor Prior Attractive Recept 			
EvelAtastriat						
Titrease r	Select	•	C Gellow Repida			
tarene berrya @			0	Exclude from #3/Portals	(AS Partiels & Black the Black)	
Man 1.01	de wini		Secured Selds at	Teaso) Sinesse totet	2	
eren a	Gratile Alt Aug. No.		Scheduling.	Beforing Pojsician Bady Fart		
1	T flessine 33M		Required Fields for	Reser.	(4)	
1	3 #wate firms that		Creating ePriorder	Schenurg Motes Referring Providiates Bridg Park		
Ройсбай Лехропін			Doucy Satura offers	Select V		
Appar Account G			Patient Antona *			
lest for miliator			Study Status often	Deboi 👻		
iprost25000	Chaste File 1d No choses		Petietti in Rokimi r			
			Report Boley to Palant	(in days)		
Surger Lage	No taqui uprosoed		Partal			
Remove Lógé	REMOVE LOOD		Formal Name			
			Facility Notes			

Setting	Description			
Code	Type your internal code for the facility (up to eight characters).			
Name	Type the name of your facility.			
Market	Select the market that the facility serves.			
Contact No.	Type the phone number of the primary contact person at the facility.			
Fax No.	Type the fax number for receiving approved reports.			
Send Reports to Fax No.	Select to automatically send reports to the fax number you entered previously.			
Email	Type the email address of the primary contact at the facility.			
Email Report Link	Select to automatically send an email when a report is created to the email address entered earlier that contains a link to view the reports on the ReportLink service.			
Report Password	Type the password needed to view reports on ReportLink.			
Email Attachment	Select to receive reports by attachment.			
Timezone	Select the time zone used by your facility.			
Mammo License ID	Type the facility's mammography license ID.			
Max TAT	Type the maximum allowed turnaround time, in minutes, before a breach of contract occurs.			
Enable Alt. Acc. No.	Select to use alternate account numbers (more than one account number) for a patient.			
Required SSN	Select to require a social security number to provide treatment.			
Require Primary Phys.	Select to require a primary physician's information to provide treatment.			
PokitDok Response	[Unused]			
Updox Account ID	If using Updox, enter the account ID.			
Send fax via Updox	Select to use Updox to send faxes.			
Upload Logo	Select Choose File, browse for and select a logo file for the facility, and then select Open. Logos can be in JPEG or PNG format.			
Remove Logo	To remove the current logo, select REMOVE LOGO.			
[Address]	Type the address of the facility.			
File Store	Select the default file store to use at the facility. See also $\frac{\text{Configure a file}}{\text{store}}$.			
Associate all appointment types to this facility	Select to associate all appointment types to this facility. Saves time because you don't have to add the facility to each appointment type configuration.			
Show Patient Alerts	Select to show a patient alert window when scheduling in the schedule book.			
Show Recent Schedules	Select to display the recent schedules dialog when: 1) scheduling a preorder; and 2) double-clicking a patient in the Patient tab of the New Appointment screen.			
	Days: Type the number of days in the past and future to control which exams appear in the recent schedule screen.			
Enable Veterinary Registration	Select to be able to register veterinarians as physicians.			

Setting	Description
Mobile Rad Dispatching Address	Type the address of the office that dispatches mobile radiology.
Do Not Allow Overlapping Procedures to Be Scheduled	Select to prevent users from scheduling more than one procedure in the same timeslot in the schedule book.
Import Documents into Study as DICOMs	Select to automatically convert documents to DICOM images before importing. If not selected, the user can select whether to convert at the time of import.
Global Auto-Print	Select to automatically send radiology reports to a printer that is configured on the Exa platform server.
Abbreviated Receipt	Select to use an abbreviated formatting for payment receipts.
Custom Receipt	Select to use a custom receipt for payments, such as co-pays.
Exclude from all Portals (All Portals & Break the Glass)	Select to exclude all studies associated with the facility from all portals, including when "breaking the glass."
Required Fields at Scheduling	Select options to require when scheduling an exam.
Required Fields for Creating a Preorder	Select options to require when creating a preorder.
Study Status When Patient Arrives	Select the status to assign to the study when the patient arrives at check-in.
Study Status When Patient in Room	Select the status to assign to the study when the patient arrives in the exam room.
Report Delay to Patient Portal	Type the number of days to wait before making reports available on the patient portal.
Formal Name	Type the formal name of the facility, if different from the commonly used name.
Facility Notes	Type additional notes for the facility if needed.
MRN Information: Inherit	Select to copy the MRN from your company settings or EMR of origin. Clear to define a dedicated format for the facility.
Other MRN settings	See MRN information.
The following settings become a	vailable after you save.
NPI No.	Type the national provider identifier of the facility.
Enable Insurance Eligibility	Select if using PokitDok.
Federal Tax ID	Type the tax ID of the facility.
Enable Insurance Claims	[Unused]
ImagineSoft Provider ID	Type the "facility ID" used by ImagineSoftware for Exa Clear.
Default Billing Provider	If using Exa Billing, select the default billing provider (the provider issuing invoices for treatment).
Service Facility	The facility providing treatment.
POS Type	Select the place of service code for the facility.
	To use this value as the default for all facilities, select the Always set POS to This Value checkbox.
Fee Schedule	Select the fee schedule for the facility.
Rendering Provider	The primary radiologist at the facility.

Setting	Description
Sales Tax	Type the percentage of sales tax charged by the facility, or select Inherit to use the percentage charged by the facility's company.

3. Select **SAVE & CLOSE**.

Add modalities

You must set up each modality available to your facility or institution in Exa PACS/RIS. In particular, you must add modalities before entering DICOM settings.



Caution: Do not add non-DICOM modalities, as this could result in problems sending studies to external PACS.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Office** > **MODALITY**.

CFRC							Ф
e	DMEMNY	MARKET	FACILITY	MODALITY	MODALITY ROOM	DROUP CHINT	ROOMS 5
Hido In:	octive :					A00	RELOAD
NO.	CODE			Nob	Ŧ		
∂*≣ ⊚1	80			Box	Densitometry		ň
28 G G2	8R.			BRM	(odslity		
∕*≣⊗⊚a	ст			Com	puted Tomography		

2. Select ADD.

Code *	5.0	[] Inche
Name *	RRModality	🔲 Hidden on Physician Portal

3. Enter the following settings.

Setting	Description
Code	Type the standard DICOM modality code. For example, for ultrasound, type US.
Name	Type a unique name for the modality.
Hidden on Physician Portal	Select to hide the modality name on the physician portal to prevent unwanted scheduling. Useful for hiding non-scheduled modalities such as

Setting	Description
	SR.

- 4. Select **SAVE**.
- 5. In the list of modalities, select the \odot \odot buttons to order the new modality.



Note: During modality-related tasks, Exa PACS/RIS matches modalities by the order in the list. Therefore, for example, you should list MG before SR.

Add modality rooms

Modality rooms are named locations where modalities are used for exams, and must be configured for use in scheduling exams. Add a modality room for each modality at your facility.



Prerequisite: Add modalities, Add a facility, and DICOM modality worklist (see <u>Configure</u> application entities).

Avoid common mistakes

- Give your modality rooms **intuitive names** that are easy to find when creating schedule templates
- Set the **From** date to a date in the past to view past scheduled appointments
- Set the **To** date to a date far in the future (such as 1/1/2099) unless you plan to permanently close the room on a certain date.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Office** > **MODALITY ROOM**.

COFFICE						0
COMPANY	MARKET	FACILITY	MODALITY	MODALITY ROOM	ORIGUE CHAT	iooms 🔉
🛃 Hide inactive					ADD	RELOAD
RADILITY	coor ;		sia	ALITY NICES	COLOR CODE	
之意 stinkaFacRty I	A01		US x	raning		
🖉 🗐 annika Pacility I	A02		CT x	aong		1.1
	ang		Angi	1		1

2. Select ADD.

ods *	AD	D hutter	Modulities *		DAWL
acting -	works metting t		All Modeltas		
Name *	00 aparteg		C =0	1	
from Exter To Date 1	01/01/2022 🔟 01/01/2224 🔟	2	🗆 an		
Östar Goda	#116/2		Π σ	- 1	
Display Order =	-		I we		
			D VE		
			🗇 अभ्य		

3. Enter the following settings.

Setting	Description
Code	Type your unique internal code or name for the modality room.
Facility	Select the facility where the modality room is located.
Name	Type a name for the modality room.
From Date/To Date	Select the long-term date range in which you plan to keep using the room.
Color Code	Click inside the box, and then select a color from the picker to assign to the modality room. The color appears in a bar below the name of the modality room in the schedule book.
Display Order	Type a number to determine in which column the modality room appears in the schedule book. Lower numbers appear on the left side of higher numbers.
Modalities	Select all modalities that are in use in the modality room. All appointment types with the modalities you select become available to schedule in the room (unless excluded by schedule rules).
DMWL	Select the AEs in the room to which to serve the worklist. Only the selected AEs will receive orders for that room.

- 4. On the **MODALITYTIMES** sub-tab, select the here link, and then add a schedule template to the room.
- 5. Optional: On the **LINKED RESOURCES** sub-tab, select which treatment resource is associated with the modality room (technologist or Mobile RAD vehicle), and then type or select the name of the resource.
- 6. Select **SAVE**.

Add a group chat room

For use with <u>Exa Chat</u>, you can create group chat rooms, manage members of the rooms, and make the room available to selected users to join.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Office** > **GROUP CHAT ROOMS**.

	0				Dat yrny tee	n has been added to an every th	1	U
		COMMANY	MARTI	Lightery	- MARALIN	NODINLITY FOLCH	CROUP CHAT ROOMS	
							906	RELOAD
		CHAT BOOM HAT	E.			GHAT BOOM MEMBERS	USERS WITH CHAI	ROOM VIERLIN
						CELECT MEMORY	KU select weater	e (* 1
d^{0}	8	TestifietRoom				2		

2. Select ADD.

COMINNY	MANT	i Naturi V	MODALITY	MODALITY FOOM	CIOUP CHAT ROOMS	NOTIFICATION (
					SAVE CL	EAR BACK
Room Name *	TeetRoom					
Usera	falact transfer		1 . T	Available to	faller Mertini	· +

3. Enter the following settings.

Setting	Description
Room Name	Type a name for the room.
Room Users	Select the members of the group chat room.
Available to	Select users who can find the group chat room when searching for a room to join.

4. Select SAVE.

Add a notification

You can preconfigure notifications (for example, a welcome message) to appear on the dashboard and/or portal sign-in pages under "Organizational News." These notifications are fixed messages, and not related to email or fax notifications.



Prerequisite: Add at least one facility.

Procedure

1. Go to $\underline{SETUP} > \underline{OFFICE} > Office > NOTIFICATION$.

	COMENNE	MODE)	(ABBLED)	MODALITY	MODALITYRCOM	GROWF CHAT ROOMS	NOTIFICATIO
Search		9	ADD.				
Nex Hire		<u></u>					

2. Select ADD.

	WANT MARKET	-BASUTY	MOBALITY	MODALITY ROOM.	SED # CIMTRODIES	NOTIFIC)
1200	mana ang ang ang ang ang ang ang ang ang	STREET C.	MODALO	SWOLING HOUSE.	SAVE	CLEAR	ATION) BACK
Tide 1	New Hires	D 1997740	🔄 Showaa P	hovacker Portal Logia 🔲	Shise on Attorney Portal Lo	394	
Description *	Welcome our new	provideta)					
	All Locations						
Office Location *	📑 Ange Facility	h.		1			
	🗋 ÖpbikaTeatfa	clite		1			
	Meesfacity Meesfacity			2			

3. Enter the following settings.

Setting	Description
Title	Type a title for the notification.
Show on Provider Portal Login	Select to display the notification on the sign-in page of the physician portal.
Show on Attorney Portal Login	Select to display the notification on the sign-in page of the attorney portal.

Setting	Description
Description	Type the body of the notification.
Office Location	Select one or more facilities to which to send the notification.

Add tasks

Administrators can add tasks for marketing representatives, such as "Tell referring physician offices that we will start offering 3D mammography in January."

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Office** > **TASKS**.

J	INTERIORI						G
	COURTER TION	TASKS	TRAM ALTIVORYATION	THAT DO	VONTINY GOALS		PORT
						ADO	RELOAD
	THE .	QESI	SIPTION		COMPLE	TE DOE DATE	
					ai s		
ō	🖞 Net Phys Recorders		et y mannogophy is April urtaise forms			63/31/2028	

2. Select ADD.

OFFICE- (annextion)	TASKS EXAM AUTHORIZATION STUDY FLAG MONTHLY	v abnus - Minaria
Ditle *	Ref Phys Remindera	SAVE
Description *	Offering mammography in April New intake forms).
Due Date	03/31/2023	

3. Enter the following settings.

Setting	Description
Title	Type a title for the task.
Description	Type a description of the task.

Setting	Description		
Due Date	Select a due date for the task.		
Completed	Select when the task is completed.		

What are authorization, verification, and estimation?

Insurance companies and other providers may require authorization and verification of eligibility.

- Authorization involves contacting the insurance company about a procedure, providing proof of medical necessity, and obtaining permission for the procedure in the form of an authorization number. Not common in Canada.
- *Verification* (of eligibility) means to check whether a patient's insurance policies cover medical products or services.
- *Estimation* refers to estimating coverage in the form of amounts payable by insurance companies, patients, and other parties.


Eligibility verification and insurance coverage estimation are performed electronically with a clearinghouse through PokitDok, Exa Clear, or other service. If the verification service is not able to determine eligibility, it can be determined manually by clinical staff.

See also:

Add authorization rules Authorize an exam View authorization history

Add authorization rules

Authorization rules help prevent you from scheduling an exam on a date sooner than the amount of time required to obtain insurance authorization. Based on the rules you configure, Exa PACS/RIS automatically adds a color-coded authorization star to studies on the worklist to indicate authorization status, and displays prompts during scheduling.

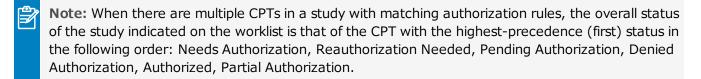



Prerequisite: Configure at least one procedure code and insurance provider.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Office** > **EXAM AUTHORIZATION**.

=	0	FIOE-								<u> 8</u>
	¢	TADES TADES	CXAN	AUTHORIZA	TION	STERVELAG	- 30	ADMITHLY OF ALS	MACRO NOTCH	PORT.
									ADD	RELOAD
		FACILITIES		AND ALL THES	DA.	n ALLOW SCHEE	NUM	PROCEDURE CODES	NICLEGANCE	ROWCERS
		AU.		A1	4	AL	19			
3	8	Angle Facility		BOCTERMO	MRNMUT	*		3.5dected	3 Selected	
2	-	Airmitia Tent Factility		XA.	2	2		2 lateral	3 Selected	
ġ.	8	arinika Facility 1		AKEUTO	2			3 Delected	1 Selected	

2. Select ADD.

Facility *	C Select all current and future facilities 3 Choose facilities					
	WY COMPANY 9					
Modelity *	ALL COLLOTED (19) -					
Days *						
	Allow Scheduling					
	Apply to Secondary/Tertiery incorrece					
	No Authorization Needed					
Procedure Codes	2 CTOES SELECTED					
Invariance Providera	1 PROVIDERS SELECTED					

- 3. Under Facility, to make different authorization rules for different facilities, Choose facilities.
- 4. Configure the following settings.

Setting	Description
Facility	Select all, or one or more facilities in the dropdown list that will use the authorization rule.
Modality	Select one or more modalities to which the authorization rule will apply.
Days	Type the minimum number of days required for authorization (the exam cannot be scheduled during the number of days you enter).
Allow Scheduling	Select to allow staff to schedule exams that are currently unauthorized, but that can be authorized in the future. Gives the scheduler the option to schedule the exam, create a preorder instead, or cancel.
Apply to Secondary/Tertiary Insurance	Clear : Apply the rule to the primary insurance policy. Select : Apply the rule to all (primary, secondary, and tertiary) insurance policies that are available for the exam.
No Authorization Needed	Select to not require authorization for any study to which the rule applies.

Setting	Description
Procedure Codes	Select one or more procedure codes. The authorization rule only applies to the procedures you select.
Insurance Providers	Select one or more insurance providers. The authorization rule only applies to the providers you select.

See also:

Authorize an exam What are authorization, verification, and estimation?

Create a study flag

You can create a study flag that users can apply to studies and orders on the worklist. Sorting the worklist by flags is one way for users to easily find groups of studies or orders to work on. You can also use them to trigger routing rules. When you create a study flag, you can restrict its availability by institution, facility, modality, and other parameters.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Office** > **STUDY FLAG**.

OFFREE							8
CONFIGATION	TASKE	CRAM ALTHORIZATION	STUDYFLAG	MONTHLY GOALE	м	GRO NOTE	.09000
						A00	RELIGION
DESCRIPTION :	94 507 67	NG MS	501/059	MODALITES		COLOF CODE	
			ALL	1 mil -	٠		
J' E Flagt							
∂ II risgi							
o? 🗄 febiga							

2. Select ADD.

							e
Carterine (TARE	EXAM AUTHERIZATION	STUDY FLAG	ONTHEY DRALS	MAURO NETTE	PORTALS	
Desenation *	Fiest	🗍 hette	🗋 Analistile in Portei Fanilitie	📋 Ves Ond in Suledulo B		SAVE	BACH
Altretuno Labet Hattal	and the second se	+		allies artitet		2 1 2	

- 3. In the **Description** box, type a name for the flag.
- 4. Enter the following settings.

Setting	Description
Available in Portal	Select to make the flag available in the physicians' portal. If selected, all other settings become unavailable.
View Study Flag on Schedule Book	Select to show any study flags added to a study on the appointment card in the schedule book.
Color Code	Select a color in the picker. This color appears in the Study Flag column in the worklist.
Institutions	Select the institutions for which the study flag is available.
Facilities	Select the facilities for which the study flag is available.
Modalities	Select the modalities for which the study flag is available.

Set monthly goals

You can set monthly goals for the number of studies to perform, for example.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Office** > **STUDY FLAG**.

	orne	E:				,				3
	4 000			HAM ASH			MONTHEY GOALS	MACKEMITTE	еритака	
									.Aco	-101090
		HODAR	96 - 30	NUM. PORTO (2010)	IDA/ MONTAN	F/808E)7	91100115 EATEPH	CMONTH UND	N HORE	
		ALC: N								
Z.		MD		February 2022	100		21			

2. Select ADD.

Modality *	MG	~
Goal Period (Month/Year) *	m	March 202
Monthly Goal *	108	
Working Days per Month *	21	

3. Enter the following settings.

Setting	Description
Modality	Select the modality for which to set a goal.
Goal Period	Select the month for the goal.
Monthly Goal	Type the target number of studies to perform with the selected modality.
Working Days per Month	Type the number of days in the selected month.

4. Select SAVE.

Configure macro notes

Macro notes are preconfigured blocks of text and metadata that you can add as notes to items in various parts of the program such as studies, schedules, and claims. Configuring macro notes ahead of time saves you from having to re-enter the same notes on multiple occasions.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Office** > **MACRO NOTES**.

		14190	EXMANDIO	- 50.0771.45	MORPHY COLUMN	MACKE NOTES	PORDALS	3
Ì	DESCRIPTION -	19990	time		MORPHY MORES	- Construction of the local division of the	ADD MAXXES	ANIMA
21	04 stady rates		ζωύγ		My Gampary, Nevella	Test Pacify sinhalfy Al		

2. Select ADD.

				evie evie	NECEL BACK
Dourgean?	Six of upy notice				
Малт Тураз т	- STUDY -	Facilities.*	SHEETID -	Modakty *	ALL SELECTED (12) -
Mecro Text *				Procedues	
Teeting - notes	ocenarios			ICITARSONTM	CONT PRESS
				Alternation	Argues
				1	

3. Enter the following settings.

Setting	Description
Description	Type a short description for the macro note to appear as the selectable option when adding notes in other parts of the program.
Macro Types	Select the context in which the macro note is available.
Facilities	Select the facilities where the macro note is available.
Modalities	Select the modality with which the macro note is available.
Edit Appointment Types	Select to add or remove the appointment types for which the macro note is available.
Macro Text	Type the text of the macro note.

4. Select SAVE.

Configure portal links

You can add up to two custom links (URLs) to the patient, provider, or attorney portal. For example, you could add a link to the patient portal that sends your patients to your company's payment website.

Procedure

1. Go to $\underline{SETUP} > \underline{OFFICE} > Office > PORTALS$.

=	OFFICE					
	Constantion TABLE EX	AM AUTHORIZATION	STUDY FLAD	(WONTHEY DOALD))	NACRO NOTES	PORTALS
	PATIENT PORTAL PROVIDER P		PORTAL			
Labe	Our Pasient Portal	8			SAVE	
1.mk	http://www.teetpportal.or	6				

2. Select a portal sub-tab (PATIENT, PROVIDER, or ATTORNEY), and then enter the following settings.

Setting	Description
Label	Type the link text that appears to the user. For example, Pay Now.
Link	Type the URL. For example:
	https:/www.ourcompname.com/customer_payment_site

3. Select SAVE.

Manage life cycles

To automatically purge aging images from the database (such as for legal or efficiency purposes), you can create life cycle management (LCM) rules. Exa PACS/RIS automatically purges images according to your rules, in FIFO (first-in-first-out) order.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Office** > **LIFE CYCLE MANAGEMENT**.

							1	
!</th <th>4 restri</th> <th>CARRY AND CONTRACTOR</th> <th>- STORE HERE</th> <th>- ARDVINE TUBLE</th> <th>MACHENINES</th> <th></th> <th>IFF CHILF MA</th> <th>NAGEMENT</th>	4 restri	CARRY AND CONTRACTOR	- STORE HERE	- ARDVINE TUBLE	MACHENINES		IFF CHILF MA	NAGEMENT
							499	RELEVAD
	81416 =	aurarie :		AR VALUE	AND DEPENDEN	ACARD I	821016	
		48	¥.	140	14		. Yes	

2. Select ADD.

= 10F006- <+			Suppose	landreywords.	adden (de da	iomie.	uncern	11 X X 10	EVENT
LON Rule Name *	Pergelline	D Inseries					54	¥C	SHOR
Select Factory and	d Modality		Select Data to Pur	94	Sele	et Alter to Purg	é.,		
Teaday -	CON	AT EXTENDED TY -	E Diages			Ago of Study *	Novite w	+	1
		Manapata	E Ferrira			🖲 Divis Date	O Shudy Revel	ved Date	e.
Medably *	NO	ALERCER THE	🗆 teoraris		0	AgeofPatent			
			🗆 табулысыны	c .	0	Verily image list			

- 3. In the **LCM Rule Name** box, type a name for the rule.
- 4. Enter the following settings.

Setting	Description
Facility/Modality	Select the facilities and modalities whose data is subject to purging by the rule.
Data to Purge	Select one or more types of data to purge.
When to Purge	Age of Study : Select the checkbox, and then enter the number days, months, or years.
	Select one of the following:
	Study Date : Determine the Age of Study by its study date. Purge by study date, oldest studies first.
	Study Received Date : Determine the Age of Study by its received date. Purge by study received date, oldest studies first.
	Age of Patient : Purge by age of patient. Select the checkbox, and then enter the age of the patient.
	Verify image instance before removing : Select the checkbox, and then select the databases from which to purge the images.

Configure providers and resources

You can set up individual healthcare providers, their groups and facilities, and provider pay schedules.

This section contains the following topics (not all topics may be available depending on version and region).

Configure an ordering facility Import an ordering facility Add a location or contact to an ordering facility About provider organizations, and provider groups and locations Add a provider organization Add a provider group or location Add a resource Configure detailed resource settings Merge resources Convert or merge a system provider resource Configure a technologist Configure a provider pay schedule

Configure an ordering facility

An ordering facility (*OF*)—such as a skilled nursing facility—is a facility that hires a mobile radiology company (see <u>About mobile billing</u>). You can configure an ordering facility at your institution to be able to submit electronic orders for OF-related exams. The orders submitted appear on the PRE ORDERS tab of the worklist for scheduling.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Providers & Resources** > **ORDERING FACILITY**.

	RDERING FACILITY	IDDATES OF THE PROPERTY OF THE	- PROVIDENCE		Elevela.	escope		
					(A00)	RECOMU	MPORT	HEIM
1480		ADURESS						
0086	RAME	ADURESS	PROMENE	DAVEND	EROAT		WARDT	
		11.			- 8		SAL.	÷
	My Oxferlig Failby	3217 US Highways TO first	c					
°≣ MYOF	nà orient circà							
CE MYOF	0510	Actor						

2. Select **ADD**, and then enter the following information.

EXO-PLATFORM

					300	SWELCOR	e aise
Code "	Cost+	I isotire		Market	NONE	SELLOSTED -	
Name *	faile.			Fedilites	NONE	BLEGTED -	i i
Address Line 1.4	Autom	estine 1		NPI No.	1		
Addens Life 2	notes	1.001.0		Provider Agreement Code	540tt •	2	
CIT/SOURCE.			Select +	Ently identifier code	Teles		
tovladt mos Map Medicarie Provider Ni CLUM Ne. Notes		ь търг	х 	EIN State Lisense No E'IN Federal Tax ID Medicael Provide the Toxonerry Code Modicore URIN SSIN			
				Contract Obert Date Fee Onteclule Locartonal Contacts	sevenation Select	ti 🕴 🕴	1
				Selected Provider Contacts			

Setting	Description
Code	Type the OF's internal facility code.
Name	Type the name of the OF.
[Address]	Type or select the address and country or region of the OF.
Default POS Map	If you configured POS maps, you can select one here.
Medicare Provider No.	Type the Medicare provider number of the OF.
CLIA No.	Type the clinical laboratory improvement amendments number of the OF.
Notes	For mobile billing, type notes to the OF, and apply formatting by using the word processing tools provided.
Market	Select markets served by the OF.
Facilities	Select facilities associated with the OF.
NPI No.	Type the national provider identifier of the OF.
Provider Agreement Code	Select the Medicare provider agreement code of the OF.
Entity Identifier Code	Select the entity identifier code of the OF (refer to the USHIK).
EIN	Type the employer identification number of the OF.
State License No.	Type the state medical license number of the OF.

Setting	Description			
ETIN	Type the ETIN to be able to use electronic transfers.			
Federal Tax ID	Type the federal tax ID of the OF.			
Medicaid Provider No.	Type the Medicaid provider number.			
Taxonomy Code	The Medicare/Medicaid taxonomy code (refer to CMS.gov).			
Medicare UPIN	If there is only one healthcare provider at the OF, and the provider has no NPI, type the unique physician identification number of the provider.			
SSN	If there is only one healthcare provider at the OF, type the social security number of the provider, if available.			
Contract Start Date	The start date of any mobile billing contract made with the OF.			
Fee Schedule	Select a fee schedule from the dropdown list. To add fee schedules, see <u>Configure a fee schedule</u> .			
Locations/Contacts	Select a provider associated with the OF, and then select the plus the button.			

4. Enter the following settings on the sub-tabs that appear.

Locations/Contacts

See Add a location or contact to an ordering facility.

Providers

To view the providers at the ordering facility, select the **PROVIDERS** sub-tab.

Setting	Description
[None]	Lists providers added to the ordering facility.

Summary

To generate summaries of patients and studies at the ordering facility, select the **SUMMARY** sub-tab.

Setting	Description
Patients/Study	Select whether display patients or studies in the ordering facility summary report.
From Date/To Date	Enter a date range for the summary report.

• Select **GO** to view the summary report.

Marketing Rep

To configure a marketing rep. for the ordering facility, select the **MARKETING REP**. sub-tab.

Setting	Description
Marketing Rep.	Select the marketing representative for the facility.
Inherited From	If inherited, enter the contact information of the origin.

- 1. Select SAVE.
- 2. In the **Notes**, **Follow-Ups/Tasks**, and **Contracts** areas, select **ADD** to add the corresponding items to the marketing rep.
- 3. In the Patient Documents area, select UPLOAD to upload documents.

Login Details

You can link users who can sign in to the ordering facility portal, and assign sign-in credentials.

- 1. Select a user and then select **LINK USER**.
- 2. Optional. To add a new user, enter credentials and select **CREATE USER**.

Credentialed Technologists

You can assign technologists to ordering facilities for which they are credentialed (such as for working at prisons). This ensures that mobile sites do not assign non-credentialed technologists.

On the **CREDENTIALED TECHNOLOGISTS** sub-tab, select one or more technologists in the dropdown list.

Credentialed Radiologists

You can assign radiologists to ordering facilities for which they are credentialed. Only assigned radiologists can read studies of the ordering facility. If no radiologists are assigned, all radiologists can read studies of the ordering facility.

• On the **CREDENTIALED RADIOLOGISTS** sub-tab, select one or more radiologists in the dropdown list.

Linked Resources

To link technologist or vehicle resources to the ordering facility, select the **LINKED RESOURCES** subtab.

Setting	Description
Technologist	Select a technologist to link.
Vehicle	Select a vehicle to link.

Billing Rules

See <u>Add billing rules</u> and <u>About billing rules</u>.

Import an ordering facility

If you obtain exported facility information you can import those facilities.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Providers & Resources** > **ORDERING FACILITY**.

	NDERING FACILITY	10000001 090 A NOOD	- PROMOTION	GURALOGATION /	Terevita.	escope		
)	(400)	RECORD	WPOIT	HEIP
14 (0)	danting .							
0086	ASASAL.	ACURISS	PRONEME	OAX NO	1009L		WAREF	
							AL.	
<*≣ MYGP	My Orderlag Failing	2217 05 Highways 10 fee	e					
J E 010	0F10	Addr						
2 E 010								

2. On the **ORDERING FACILITY** tab, select **IMPORT**.



- 3. Select Choose File, browse for and select the file to import, and then select Open.
- 4. Optional. To replace existing facilities, select the **Update Existing** checkbox.
- 5. Select IMPORT.

Add a location or contact to an ordering facility

Ordering facilities can have multiple locations. For example, skilled nursing facilities (SNFs) can have different halls, each with different nurse stations or fax lines. To address this, you can add locations or contacts to an ordering facility.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Providers & Resources** > **ORDERING FACILITY**.

- 11									24
		ERING FACILITY	10000.000 090AN(20009).	- CPREMOTION	IONIN IOGADICH /	- Internation			
						((460))	RECORD	week	HELP
	Hide the	inter .							
00	38	RAVE	ADDRESS	PROME NO.	DAX RD	8869L		MARGER	
								AL.	
d'≣ NY		My Ordenlig Faility	2217 05 Highways 70 See						
Ø ≝ 01	0	0710	Addr						
12 M		1143	Atkined						

2. On the **ORDERING FACILITY** tab, open an ordering facility to edit, and then select the **LOCATIONS/CONTACTS** sub-tab.

	LOCATIONS/CONTACTS PROVIDERS SUMMARY
	CREDENTIALED RADIOLOGISTS LINKED RESOURCES
	LOCATION NAME PHONE NUMBER FAX NO.
e.	My Ordering Facility
NEW	CONTACT

3. Select **NEW CONTACT**, and then enter the following settings.

Caution: A password is configured or used on this page. For recommended security, <u>turn off</u> <u>Chrome autofill</u>.


Setting	Description
Location Name	Type the name of the location.
Phone/Fax/Email	Type the contact information for the facility.
Send Fax/Email	Select the checkboxes to indicate how to receive approved reports.
Report Password	Type the password that the ordering facility must use to view reports generated from exams that they ordered.
Ordering Facility Type	Select the type of ordering facility for the location.
POS Type	Select the place of service code for the location.
Primary Contact	Select if this location is the primary contact for the ordering facility.
Billing Type	Available in selected regions.

Setting	Description
	Select the billing type of the location. What is a billing type?

- 5. Select **SAVE CONTACT**.
- 6. Select **SAVE & CLOSE**.

About provider organizations, and provider groups and locations

In Exa PACS/RIS you can organize your healthcare institutions hierarchically by parent Companies and child Facilities. In a similar manner, you can organize your referring providers by parent <u>provider</u> <u>organizations</u> and child Groups/Locations (which are typically "brick-and-mortar" offices and clinics). You can then associate individual referring providers to those groups and locations (see <u>Add a provider group</u> <u>or location</u>).

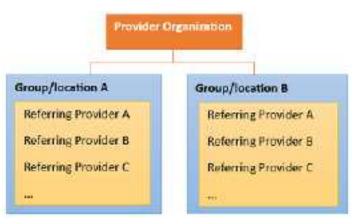

Notes:

- Configuring provider groups and locations in advance is recommended because it makes it easier to enter settings for resources, providers, and other assets.
- Configuring provider groups and locations is required for setting up the physician portal.

Add a provider organization

A *provider organization* is an optional parent entity for *provider groups and locations*, which are typically "brick-and-mortar" offices and clinics for referring providers. To add a provider organization:

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Providers & Resources** > **PROVIDER ORGANIZATION**.

= PR	INTERIS IN PERS	MRGEN:			0
	CHINESES YOCK ITY	PRIVIDER ORGANIZATION		1000000	
					ADD IIE040
0000		19	IOVIEER ORIGANIZATION #		PROVIDER GROUPS
J7 ≣ 0)		Pe	teider Organization Text		My Provider Group1 AgioTool

2. Select **Add**, and then enter the following settings.

Setting	Description
Code	Type your internal code for the organization.
Description	Type a description.

3. Select **SAVE & CLOSE**.

See also:

About provider organizations, and provider groups and locations Add a provider group or location

Add a provider group or location

Provider groups and locations are typically "brick-and-mortar" offices and clinics to which you can associate individual referring providers. If your institution works with a provider group or location, add it to the system as follows.



Caution: A password is configured or used on this page. For recommended security, <u>turn off</u> <u>Chrome autofill</u>.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Providers & Resources** > **PROVIDER GROUP/LOCATION**.

3	DREINGTACLITY	FROMER ORGANIZATION	PROVIDES CROUP/LOCATION	ILLOWICE	PROVIDED INVECTIONLE
				-	ADD RELOAD
cone	NAME 7	acontist	PHONE NO.	PAKHE	ENAL.
Pi≊ ang	AgieTextPra6rop	1/54 west 12			
I MITO	My Preside Doug?	2217 125 Highwar 70	Zaul		

2. Select Add.

CINOCHINO DA	CUTEA ¹¹¹ I BROMDER OROM	(CATINGH	FROMOUS OROUP/LOCATION	
Code 1	\$evps			
liame ?	My Provider Group?			
Pundor Organization	Poseider Organization Text = *			
Address Line 1 *	2217 00 Highway 70 East			
Address Line 2	Address Line 2			
Oty/State/2P		Geleot 🛩		
2.8	201 Mark			
Phone No.	Passa Ar.			
FarNo	Tat We			
	Sortel Paie			
Emai	tinal			
Réport Password				
PRO #0175	MORETING REP.			
COCK NAME		NPING.	MARKINGHER	

3. Enter the following settings.

Setting	Description				
Code	Type your internal code for the group.				
Name	Type the name of the group.				
Provider Organization	Select the provider organization to which the provider group or location belongs.				
[Address]	Type and select the address and country or region of the group.				
Phone/Fax No.	Type the primary contact information for the group.				
Email	Type the email address of the primary contact at the group.				
Report Password	Type the password that the provider group must use to view reports generated from exams that they ordered.				

- 4. Select **SAVE**.
- 5. Use the **PROVIDERS** sub-tab to view or edit the list of configured providers.
- 6. Use the **MARKETING REP** sub-tab to add a marketing rep to the provider group.

See also:

About provider organizations, and provider groups and locations

Add a resource

A *resource* is an attorney, nurse, provider, laboratory, radiologist, technologist, or referring provider. You can add individual resources and add them to one or more provider groups.



Caution: <u>Turn off Chrome autofill</u> feature before entering sensitive or security related information such as dates of birth and passwords.

Why add a "resource" at all?

A user (such as an individual technologist) cannot be directly assigned to a study. Instead, you assign the resource (such as a resource of type Technologist) to a study, and then link a user to the resource. See "LOGIN DETAILS" in <u>Configure detailed resource settings</u>. Resources also serve as a grouping mechanism to make users easier to find in various parts of the program.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Providers & Resources** > **RESOURCE**.

	CRILA RESO		n silsanization	(FRIONE	icin ceccu	IVLOCATIC	n	FILS	ouna	•	PITOY	IDCFT PMV	801	6000
			SSARCHINGEP	ROW DEF ON W	421	A00	-	nii i	- 13	инон	μi.	MEORT		- 112
çone e	kida i	NELS.	MUNCETING RG	ADDRENT	PHONE	e skee	P	entit	ieo	uici;	MH I	-	15	Átti si
							A	(-	AL.	-	A2		Ŷ.	
• 21201	Tool, Attorney			2217.06 Hips	e.	(333)32	22		anon	199				5
+ //\$m	Alterney, Griftera Te	e.		Autornal Autors	10				anne					J.
·	HIGH. Alg.			12.7280 FC0					malar	rig H				v

2. Select **Add**, and then enter the settings in the figure below.

Note: Actual settings differ depending on your selection in the Type dropdown list.

Type '	Referring Provider 😽		Macket.	NONE SELECTED -
Cade	HCP12RF	Inactive	fecilities *	a pet rerro -
Name -	Referring	0,07		NONE SELECTED -
	Provider	i Butto	Specialty	
Title				
PLNo:	Ĩ.	1	Ucense No.	
eronomy Code			-Diverse two	
			Medicare Provider No.	100
ic. Office Name			Medicaid Provider No.	
SN			Prov Agreement Code	~ ?
ederal Tax II)			10000	Property in the
			POSType	Talett 👻
Pi Nu.			Fee Schedule	Select 🐱
Titl No.				
ET (1988)			Max TAT	1 (in min)
fedicare UP/N				

Setting	Description			
Туре	Select the provider type.			
Code	Type your internal code for the provider.			
Name	Type the name of the provider.			
Modality	Select the technologist's allowed modalities.			
Title	Type the academic credentials of the provider (e.g. MD)			
NPI No.	Type the national provider identifier of the facility.			
Taxonomy Code	The Medicare/Medicaid taxonomy code (refer to CMS.gov).			
Dr. Office Name	Type the business name of the provider's office.			
SSN	Type the social security number of the provider.			
Federal Tax ID	Type the federal tax ID of the provider.			
EIN No.	Type the employer identification number of the provider.			
ETIN No.	Type the ETIN to be able to use electronic transfers.			
Medicare UPIN	If the provider has no NPI, type the unique physician identification number of the provider.			
Market	Select the market that the resource serves. All facilities in the Facilities dropdown list that are associated with the selected market become selected.			
Facilities	Select all facilities with which the provider is associated.			
	CAUTION: Confusion can arise if you select Facility "A" here, and then later link a user to this resource who does not have rights to view studies from Facility "A." Keep track of which resources and users have access to which facilities and other assets.			

Setting	Description			
Specialty	Select the specialties of the provider.			
License No.	Type the state medical license number of the provider.			
Medicare Provider No.	Type the Medicare provider number of the provider.			
Medicaid Provider No.	Type the Medicaid provider number of the provider.			
Prov. Agreement Code	Type the code for the legal provider agreement code.			
POS Type	Displayed when Referring Provider is selected as the provider type. Select the place of service type for the referring provider's location.			
Fee Schedule	Displayed when Referring Provider is selected as the provider type.			
Max TAT	Assign a maximum turnaround time for the provider, in minutes. Exa PACS/RIS divides this time into 4 segments on the TAT monitor on the worklist.			

3. Under **Contact Information**, enter the following settings.

Contact Information	1			
Code *		hactive	Office Pilone	
Provider Organization	Goled provider procession		Office Fax	
Group	Relatit pro other group	+	Mpbile No.	
Email			Pager No.	
Contact Name			Phone No.	
Country	United States 🗢		Fax No.	
Addition Line 1.*			Pomary Contact	
Address Life 7				
City/State/ZIP		Belest 🐱		
	21P-Pfu			
Provider Alerto				

Setting	Description
Code	Type your internal code for the provider.
Provider Organization	Select the provider organization of the provider.
Group	Select the provider group to which the provider belongs.
	If the provider uses Physician Portal, you must select the portal user's group.
Email	Type the email address of the provider. This is required to receive email reports, notifications, or attachments.
Contact Name	Type a contact name, such as the name of the administrative assistant of the provider.
[Country and Address]	Type or select the country or region and address of the provider.

Setting	Description
Provider Alerts	Type any alerts for the provider, such as "only takes referrals."
Office Phone/Fax	Type the contact information for the provider's office.
Mobile/Pager No.	Type the mobile and/or pager number of the provider.
Phone No.	Type the personal phone number of the provider.
Fax No.	Type the personal fax number of the provider.
Primary Contact	Select to send reports to the referring provider only.

4. Under **Reports to Me**, enter the following settings.

Reports to Me					
Contact Information			Reports to Droup		
	💭 Ensel Report Link	El Shall Attechment		🖾 Email Report Link	🗋 Brial Attochment
	D Pertal Kdull	🖾 928		D Postal Mali	E Fax
	🗆 office Pax			🔲 Office Pax	
Report Paccoord			Reports to Patient Po	stal	
			Delay	(in days)	
Image Delivery	Options				
	🖸 c1				
	🗇 film				
	D Paper				
Notification Se	ttings				
	🛱 Email				
	□ F=x				
	Added as CC Provider				

Group	Setting	Description
Reports to Me	Contact Information	
	Email Report Link Email Attachment Postal Mail Fax Office Fax HL7	Select to send reports to the individual provider location/contact, and select which methods to use. Note : If you select Email Report Link and/or Email Attachment, you must enter the recipient email address in the Contact Information area.
	Reports to Group Email Report Link Email Attachment Postal Mail Fax	Select to send reports to all members of the provider location/contact's provider group, and select which methods to use.

Group	Setting	Description
	Office Fax HL7	
	Report Password	If you selected Email Report Link previously, type the password that the provider must use to view reports generated from exams that they ordered.
		CAUTION : If you don't add a password, the default password (123456) is used. Because this is easy to guess, please create a unique password as soon as possible.
	Reports to Patient Portal Delay	Type the number of days to wait before posting reports on the patient portal.
Image Delivery Options	CD Film Paper	Select which media to use to deliver images.
Notification	Email/Fax	Select to receive notifications by email and/or fax.
Settings	Receive When Added as CC Provider	Select to have the provider receive notifications by email if they are added as a CC (carbon copy) provider.

Result: The resource is saved, and sub-tabs appear for detailed settings.

6. Continue to <u>Configure detailed resource settings</u>.

See also:

<u>Configure a technologist</u> <u>Deliver reports</u> <u>Configure a notification template</u>

Configure detailed resource settings

If you open the settings of a resource, sub-tabs appear for detailed settings. Select a sub-tab, enter settings as described below, and then select **SAVE & CLOSE**.



Prerequisite: Add a resource.

LOCATIONS/CONTACTS

If the resource is available at multiple locations (such as physical addresses) or points of contact (such as different offices with different phone and fax numbers), you can create a new location/contact by selecting **NEW CONTACT**.

	LOCATIO	INS/CONTACTS	LOGIN DETAILS	ALT: NAME	MOBILE SCH	HEDULE TEMPLATES
	SIGNATE	IRE SUMMA	RY PEER REVIEW	v		
	CODE	CONTACT NAM G	ROUP ORDERIN	G FAC ADDRESS 1	FHONE #	MARKETING RE PRIMA
0	RF24					Yes
4	l.					
	NEW CONTACT					

LOGIN DETAILS

In this tab, you can link an individual user to the resource.

1. Select the user and then select **LINK USER**.

LOCATION	S-CONTACTS	LOON DETAILS	ALT. HANK	MORTE SCHEDULE TEMPLA
Select Over	Ritest			• •
		INCOLOR		

2. Optional. If the correct user has not yet been configured, you can add them "on the fly" by entering the following settings.

New User *	New Unie	
Peaaword *	Paraword	
Confirm Password *	ConfirmPassword	
Access Expires After	dsyn	×
	One-Time Access	
Group Name *	Portal(PTL0)	🗴 🔹 🛃 Hide AD Groupa
		CREATE USER

ALT. NAME

If the resource goes by different names, add them here.

ALLOWED APPOINTMENTS

For technologists. You can select EDIT, and then search for appointments that the technologist is allowed to perform. Leave blank to allow all appointments.

MOBILE SCHEDULE TEMPLATES

For technologists. Use this sub-tab to define the technologist's working hours for Mobile Rad.

SCHEDULE RULES

For technologists. You can view which schedule rules are associated with the technologist.

SIGNATURE

Attach a JPG image to use as the resource's electronic signature in other parts of the program.

SUMMARY

Generate basic reports about which patients, studies, and encounters the resource interacted with over a specified range of time.

PEER REVIEW

Use to specify which studies a radiologist resource can peer review. See <u>Configure peer reviewing</u> <u>provider filters</u>.

Merge resources

It is possible to accidentally configure the same resource more than once, for example by configuring separate instances of a resource for multiple locations rather than configuring the resource once and adding multiple locations. If you find such duplicate resources it is best to merge them.

- 1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Providers & Resources** > **RESOURCE**.
- 2. In the **RESOURCE TYPE** column, select the type containing the duplicate resources.

TON RESOURCE	FIROVIDER PM	SCHEDULE			
	MERGE	SEARCH FOR PR	OVIDER ON WEB	ABD	RELOAD
ADDRESS	PHONE #	ŦAX #	SPECIALITY	RESOUR	ICETYPE MAR
			AL	At	~ AI
	Becomes a when you s resource ty	elect the	ē.		tury sr Rodiology ng Provider

- 3. In the list of resources, select one of the duplicated resources and select MERGE.
- 4. In the dialog, select **Next**, and then select a destination provider.

	C.1	
	Selected for merge	
ABARCA,	MD (25917)	

5. Select **Confirm** to complete the merge.

	Summary
Source	
ABARCA,	C MD (1998)
Destination	
Abarca.	(1710287019)

Convert or merge a system provider resource

When Exa PACS/RIS receives a study with providers that it does not recognize, it automatically creates a *system provider* resource and associates the study with that system provider. System providers are not available in other parts of the program, but you can convert them to a usable provider or merge them with an existing one.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Providers & Resources** > **RESOURCE**.

GRD	CITING EVENINE	(novici	IN GROANIZATION	(EB9)0	CRITIC GROUP	VLOCATIO		RES	OURC	ε	PROVI	DCTT PMV	вана	TOULE
			SSARCHINGEP	REVOLUER DAV W	A21	ADD .	-	iii	ŧ	xPor	i i	MEOR		180
cone e	kabel	NER	MARKETING RS	ADDREED	PHONE	tile a	19R	iiiii	acie	i iid	MARGET		- 54	in sei
							A1	+	AL.		A2		÷.	
• 2°201	Tool, Attorney			22173.8 Heps	e,	(333)33	2		Atten	nqq-			ŝ	
+ //\$is	Alterney Granets	1		(utribelition)	-				any	-			3	5
· diang	High Alg			tation 400					malar					*

2. On the SYSTEM PROVIDER column, select Yes.

		CONVERT SE	LECTED	SEARC	H FOR PROVI	DER ON WEB		ABO	PIELO	iADi	EXPORT
	C00E#	NAME	NEIP	MARKETING	ADDRESD	GPECIALI	V.RS	SOURCE	HO3/VE	SYSTEM	APROVIDER
		10				ΔI	- 4	a v	Yes 👻	Yes	
+	HCP21RF.	NR KRISTY SU			ISYSTEM PRO	MDC	Re	feming Pr			*

- 1. Select a system provider in the list, and then select **CONVERT SELECTED**.
- 2. To create a new provider, select **CONVERT DIRECTLY**.
- 3. To merge with an existing provider, select **USE EXISTING**, and then select a provider in the list.



4. Select SAVE.

Configure a technologist

Technologists have access to special features in Exa PACS/RIS, and therefore require a slightly more detailed configuration. For example, you can select which appointment types the technologist is allowed to perform, and assign schedule rules to the technologist, which prevents scheduling them for an appointment that they are not allowed to perform. To configure a technologist:

Add a technologist resource

- Follow the steps in <u>Add a resource</u> to add a resource of type **Technologist**, and select **SAVE** (do not select **SAVE & CLOSE**).
- 2. On the ALLOWED APPOINTMENTS sub-tab, select EDIT.

Tape T Cade	Terbologii •		Wald	ANNERSON -		
Mail+-	Textp	WI Tech	Matany 🚺	40.0030300/0 <u>9</u> -		
			NOTAT DE METALE TEMPLATER	3240411010	EMINTW	PERIODALA
ine.	6.001	HERE, YY	- 29	dury -		101

- 3. In the **Edit Appointment Types** dialog, select the checkboxes for all appointments that the technologist is allowed to perform, and then select **SAVE**.
- 4. Optional. On the **SCHEDULE RULES** sub-tab, select an edit button to modify a schedule rule.
- 5. Select **SAVE & CLOSE**.

Create a technologist user role

- 1. Go to <u>SETUP</u> > <u>OFFICE</u> > User Management > USER ROLES.
- 2. Select **ADD**, type a name and description of the role (such as "TECH"), and the select **SAVE**.
- 3. In the **User Role Permission** area, select the **Technologist** right, plus any other rights you want to grant all technologists.
- 4. Select **SAVE & CLOSE**.

Create a technologist user group

1. On the **USER GROUPS** tab, select **ADD**, and type a code, name, and description for the group.

USER M/	NAGEMENT		1	
USERS	USER ROLES	USER GROUPS	AD GROUPS AD U	SERS USERS CHUNE AS
Group Code * Group Name * Group Description *	TECH Technologist Technologist	Inactive	Document Types	All Documents
Roles			Navigation	Incurance Card +
Billing Facility Admin	1		🖸 Dashboard	
🗋 Frontidesk			D Patient Arriv	al Worklist
Portal RAD			Patient	6
REF			Schedule	
TRANS			Paymenta	

- 2. Type a In the **Roles** dropdown list, select the role you created earlier (such as **TECH**).
- 3. Enter other settings for the group (see <u>Create a user group</u>), and then select **SAVE**.

Create a technologist user

Configure an individual technologist as a technologist user.

- 1. On the **USERS** tab, select **ADD**.
- 2. In the Linked Provider User Type dropdown list, select Technologist.

10100	THE HOUSE	HINE SHO IN	ATTRACTOR.	*******	ALCONG BORNING	Abox - OKTU	en anteresta	n Hite
					SAVE	EWE & CLOSE	HESET FASSWORD	UNLOG
tiroup heree *	Terretinger		1 🖬 m	www.All Groups	Assess Desires	Attai	dayh 😽	
Nora*	Tesh	.94	Teur	G.(Ta-		C Deel	ITS Access	
Minile Phone	Malifie PScott					D (her)	dan Denge Farmed N	ention
E-Mail	freat					🖾 User d	an sherye assocrating ite	ites .
Uou Mene+	Test Det	D. 196	2010			C Drops	A2M0	
Second standard	19	C Ale	u Smorgeney Adu	104	Marce!		ARE REFERENCED IN	
					factoris *	- ALL	30.00703 (0 +	_
					United Provider	Use Tipe Technol	tapit +	
					Technologist	Tauls 7m	4	1

- 3. In the **Technologist** dropdown list, select the technologist resource you created earlier.
- 4. Enter all other settings for the user (see <u>Create a user</u>).
- 5. Select **SAVE**.

Configure a provider pay schedule

To help with billing, you can configure a pay schedule for each organization that provides radiology reading services. The schedule defines the charges by procedure, modality, and radiologist. Another benefit of configuring provider pay schedules is that it enables you to generate **Fees by Radiologist and Modality** reports.



Prerequisite: Obtain the Provider Pay Schedule right.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Providers & Resources** > **PROVIDER PAY SCHEDULE**.

	WE WANTED BOT OF THE OWNER OWNER OF THE OWNER OWNE	statestic densities all in	HANDING	PROMORTING SCHOLER	
			-	100	mne
SALL C			TRAILERIE	250 0472	

2. Select ADD.

Charles Contraction (Inactive		
Start Date/End Date * MM.	00,7777 III u	M/DD/YYYY 3	1	
Provider * Test	Radinlogist	× - +		
Selected Providers				
Test, Facilologist 🔀				
Modalities [7] A		20	Modality Fee 00.00	+
1 A	l Modelities BR		accounty i at	
Committee of the local division of the local	or			
8	MG			
	MR			
Summury				
Appointment Types 30.9	NOR I&R CT MR USA	DIN REQ POSTR	X - Appointment Type	FF80 100 +

3. Enter the following settings.

Setting	Description			
Pay Schedule Name	Type a name for the pay schedule. Typically, this is the name or organization of the radiologist.			
Start/End Date	Select a date range during which the pay schedule is valid.			
Provider	Select a radiologist to assign to the providing organization, and then select the plus 🛨 button. You can add multiple radiologists.			
Modalities/Fee	Select one or more modalities to which to assign a fee. Type the fee to charge when using the selected modality during an exam, and then select the plus button. Add all modality/fee combinations that the provider offers.			
Appointment Types/Fee	Select one or more appointment (exam) types to which to assign a fee. Type the fee for the selected exam, and then select the plus to button. Add all appointment type/fee combinations that the provider offers. Note: Appointment type fees overwrite any overlapping modality fees.			

4. Select SAVE.

Configure scheduling and codes

You can set up information related to scheduling exams, including fees, body parts, appointment types, diagnostic and procedure codes, and various kinds of templates.

This section contains the following topics (not all topics may be available depending on version and region).

About fee schedules Configure a fee schedule Import or export a fee schedule Configure a facility fee schedule Configure body parts Configure diagnostic codes Import diagnostic codes About procedure codes, appointment types, and appointment type procedures Configure procedure codes Import specimen catalogs Import procedure codes **Configure NDCs** About schedule blocks About schedule templates and schedule rules Configure a schedule template and schedule rules Configure a schedule filter Configure study statuses Configure study flow Add a transcription template Add a report template Configure functional and cognitive statuses What are lead time and expected arrival time? Configure appointment types Configure exam prep instructions Configure grouped appointments View locked slots

About fee schedules

Fee schedules define what your healthcare institution charges for procedures.

Types of fee schedule

• **Billing fee schedule**: The primary type of fee schedule. Lists the global fees before any other considerations.

- Allowed fee schedule: The contract rate agreed upon with insurance companies. Used for reference purposes.
- Self-pay schedule: Reflects discounts for cash payments made by individual patients.

Creating fee schedules

To create fee schedules, see <u>Configure a fee schedule</u>.

Using fee schedules


To use a fee schedule, assign it to an insurance provider, facility, and/or ordering facility as described in these topics:

Add an insurance provider Configure an ordering facility Add a facility


Ordering facilities: which fees are used from which schedules?

Exa PACS/RIS uses sophisticated logic to determine which fees to apply based on any fee schedules that you create. In addition to the criteria in the decision trees below, Exa PACS/RIS attempts to select a facility fee schedule that includes the ordering facility of the study. If it cannot find one, it looks for one that does not include the ordering facility.

For ordering purposes

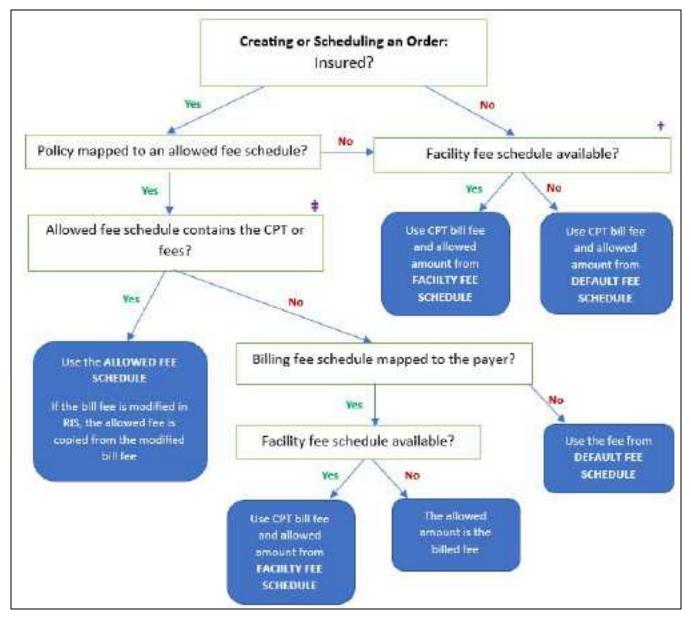

For billing purposes

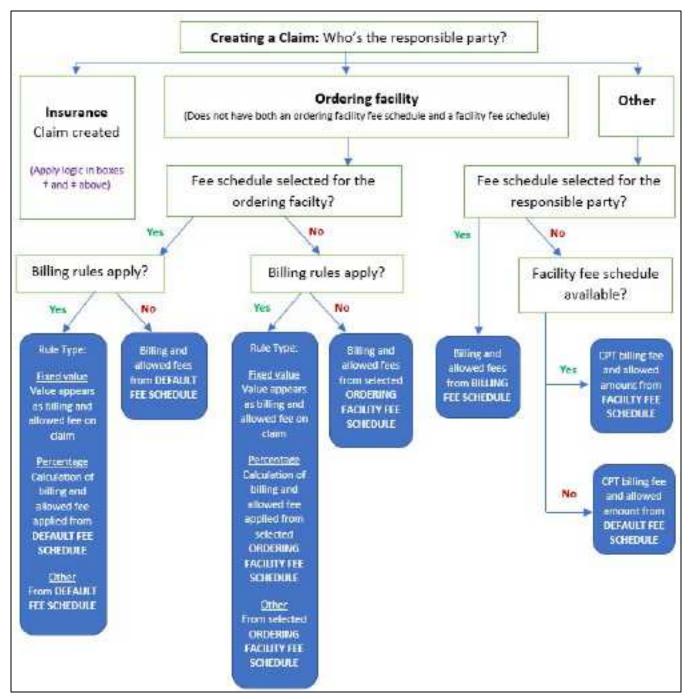

Configure a fee schedule

You can configure fee schedules by following these steps:



Prerequisite: Configure procedure codes

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Scheduling & Codes** > **FEE SCHEDULE**.

					(
HE STREET	 anov 9446	liament contes	PROTEINED	later the second	-
				ALL	
MADE *	3999		ARTINITATIO	TODATO	
	(AP)	28			
d' Delaut	Dring Fee Sche	dile -	10/01/01/0	01/20/2393	
d New Select	Defect the Sch		00/28/0029	00.010.025	
/ werning and	Emples for	nie .	60/28/3029	15,900393	

2. Select **ADD**, and then enter the following settings.

Setting	Description
Fee Schedule Name	Type a name for the fee schedule.
From Date/To Date	Select a date range during which the fee schedule is valid. The From date must be the current day or earlier.
Туре	Select a type from the dropdown.

- 3. Select SAVE.
- 4. On the **FEE SCHEDULE** tab, find the schedule you just added, and then double-click to open it.

🖨 samangta	NA A STATE				0
H# 934	andra and the state	anormers encanderer	a elocatercitate	Seconds ()	iliana ina 🔉
	ADD PRODUCTURE	NEON TESO IDULE DOOT	TERENICOULE HELOAD	DAVE BACK	ane .
Fee Schedus Same * Horn Date to Date * Type	Teart and theo period teart and the cosystem and a set of the Relating Free Schemulan (19)	90 -			
COEN	089C497DOA	365MILE TYPE	ARODAL/TY	80011	0000
		[M]	1412.00	$\pm (240)$	(w)
r 🖹 00017	HIRT FAILLIFE ABRESSED			write facility (Angle See A(4) 10
/ 🗐 0001A	IVM 4EMIN:SKR000V2.2EMI0.5	0.3ML S.L. RECON-		attila fecilia 1	Angle Paulity C 22
/ = 000%	I IVM ADMIC SARSON 2 STACO	T. BALL DE RECENCI		anta-Ricey a	Ange Record an

5. Select ADD PROCEDURE, select one or more procedures in the list, and then select SAVE.

To find procedures to add, enter one or more of the following search criteria, and then select **SEARCH**.

Setting	Description
Add To	Shows the name of the current fee schedule.

Setting	Description			
Facilities	Select the facilities that provide the procedure.			
Code	Type the code for the procedure.			
Short Description	Type one or more characters in the description of the procedure.			
Modalities	Select the modalities used in the procedure.			
Ref. Code	Type the reference code for the procedure. In most cases this can be the CPT or LOINC code.			
NDC Code	Type the national drug code associated with the procedure.			
Service Type	Select a service type in the dropdown list.			

6. On the FEE SCHEDULE tab, select SAVE.

Result: The Fee Schedule tab reappears with the procedures you added in the list.

7. Open one of the procedures, and in the Fee Details dialog, enter the following fees:

900	8001F
Short Description	
	HERT FAILLEFE ASSESSED
Description	
	HHIT PRILURE ASSESSED
Global Fee	
	0.00
Professional Fee	
	0.00
Technical Fees	
	0.00

8. Add fees to all of the procedures in the fee schedule, and then select **SAVE**.

See also:

About fee schedules

Import or export a fee schedule

You can import a fee schedule from another Exa PACS/RIS installation, or export a fee schedule from your current installation. When importing a fee schedule that already exists in Exa PACS/RIS but contains new CPT or billing codes, the new codes are added to the program. However as of version 1.4.32_P6, new appointment types are not added.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Scheduling & Codes** > **FEE SCHEDULE**.

I REHERVLING ING	Q PIES					<pre></pre>
HE STREET	MACRO THEF SCHOOLS	400004444	the second second	PROXIMATION	Anti-	Sealest an inc.
1.1.1					AD	
80.00E +		3999		ARESOLUTION	TO DATE.	
		M.	28			
Default		Dring Fee Sche	ster (10/01/09/09	01/00/2390	
New Select		Defend the Sch		02/28/0625	01471/2.023	
* test all far attack		Emples for	vi.ie	60/28/3022	11,500100	

- 2. Double-click an existing fee schedule to open it.
- 3. To import a fee schedule:
 - a. Select IMPORT FEE SCHEDULE.
 - b. Select **Choose File**, browse for and select a file to import, and then select **Open**.
 - c. Select IMPORT.

111.14	illiure,	FACILITY FOR SCIEDULE	NUCLAY PARTY IN A CO	KINDER GEREEN IN INCLUDE	i coutr	NEXT.	2014
		ADD PROCEDURE	IMPORT FEESCHEDULE	EXPORT FEESOHEDULE	RELOND	BAV	E i
Fee Bohedule Name *	feat bill fee	solied					
	[] Institue						
From Date/To Date =	02/28/2023	12/30/2023 II	0				
type	Inting tee	Schedule 👻					
Chinese .							
cone	000	countrol -	SCRUGE THR		VICAUTO		nyci
1. AND A. A.	0ES	OUMDON .	SCRADE TYPE		AT	Ŷ	
100000	i.	DUPDON HALLINE ASSESSED				*	AT I

5. To export the current fee schedule to your Windows Downloads folder, select **EXPORT FEE SCHEDULE**.

Configure a facility fee schedule

It can be useful to configure individual facility fee schedules if your fees differ by facility, or for teleradiology, or if you generate the Fees by Facility or Fees by Modality report.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Scheduling & Codes** > **FACILITY FEE SCHEDULE**.

	COLUET					
HE IS STOLE	RELITY FEE BO-EDINE	BOON PARES	DAMONTHCODE	Fill Control and a control	HEIG DOLLER	somerne)
					434	and the second se
NAME				sorr name -	010/04/3	
J 🕆 hunda Test Facility			0.0	2/34/1011	01/36/2030	

2. Select ADD.

Fee Name *	Fit Fee DI	I Inactive		
Start Cate Twilliam 1	3322-05-03 🗵 2024-0	8-00 III		
Taillies -	D all exciting		Onlering Pacificies	
	Salari hedilari Myttempany III Ramta	araaany ar 80000		Ng Deburg Fability
Woddites	C AlWoldten	Modelity Foo 10001	+	
Gammery BD 5101.00 BR 5101.00 GR 5101.00				

3. Enter the following settings.

Setting	Description
Fee Name	Type a name for the facility fee schedule.
Start Date/End Date	Select a date range during which the schedule is valid.
Facilities	Select one or more facilities to which the schedule applies.
Modalities	Select one or more modalities to which the fee applies.
Modality Fee	Type the fee for the modality, and then select the plus to button. You can add multiple modality/fee combinations.
Ordering Facilities	Select which ordering facilities use the fee schedule. If you make a selection, the Summary of Fees by Facility/Modality report pertains to

Setting	Description
	studies associated with those ordering facilities.
Summary	Shows the total fees by modality. If at least one ordering facility is selected, the Summary applies only to OF fees.

4. Select SAVE.

See also:

About fee schedules

Configure body parts

You can define body parts for use in other parts of the program.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Scheduling & Codes** > **BODY PARTS**.

	CHEDULING 4.C	open -					0
		real resources	DODY PARTS	Linkinkance opera	recently cours	-	stenessast ens. >
						10	a ana ana ana ana ana ana ana ana ana a
	WWVE 5			N.T. WWWE			
7 E	Adames			Nalawari			
0.0	Asthe			Ar A Iz			
12				340			

- 2. Select ADD.
- 3. Type the name and alternative name for the body part, and then select **SAVE.**

Configure diagnostic codes

You can enter SNOMED, ICD, or LOINC codes for use in other parts of the program.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Scheduling & Codes** > **DIAGNOSTIC CODES**.

Transmur	NATION OF TANKING THE BARMANDECO	ord Piperniae creat		southarr
	120	A00 -	10.040	uncar inte
1000+	DESCRIPTION /		099	PC00m
			# 1.	· · · · · ·
10A B 1	Oradonacidade to: Vitoria of adonate CT, biotype utravier av		10292	1
at win	Chemistra ID YORO (Temar CLower etc.		100	
/ \$ 804	Orden augentier		8000	

2. Select ADD.

Code *	A00.0	Inactive
Description *	Cholera due to Vibrio chol	
Code Type	10010 ~	

3. Enter the following settings.

Code – Type the code to enter.

Description – Type the code's diagnosis description.

Code Type – Select the code standard to which the code belongs.

4. Select SAVE.

Import diagnostic codes

If you obtain exported diagnostic codes, such as from the CMS or AMA, you can import them.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > Scheduling & Codes > DIAGNOSTIC CODES.

annan ann an Anna an An	NARAWATACHTERE REPORTS BACHGEROCOD	nickening street		southeart
		A00		urear Ha
mire.	Last surflow 2		1099	ACTUM
			21.	* (m. *
1 1 ASD 0	Onderso due to Vicilo d'admas CT, bioyar chalerae		10012	
CTE MILL	Dicession to York of General Lower etc.		100	*
7 \$ MR.	Orden auperder		82230	

- 2. Select **IMPORT**.
- 3. Select Choose File, browse for and select the file to import, and select Open.
- 4. Optional. To replace existing facilities, select the **Update Existing** checkbox.
- 5. Select **IMPORT**.

About procedure codes, appointment types, and appointment type procedures

Procedure code

A *procedure code* (or *CPT code*) is an internal code that identifies a medical procedure. You can preconfigure procedure codes primarily *for purposes of billing*.

Appointment types and appointment type procedures

An *appointment type* defines a set of related procedures to perform during an exam *for purposes of scheduling and medical examinations* (by the front desk, radiologists, and technologists).

An appointment type includes one or more *appointment type procedures* (which are the same as "procedures") to perform during a single appointment. A common example would be an appointment type that includes an imaging study (one procedure) with contrast (another procedure).

Where do I use these?

Exa PACS/RIS prompts you for either a procedure or appointment type (or both) depending on which part of the program you are using. For example, when *scheduling* the exam, the scheduler enters appointment types. When *reading* the exam, the radiologist also enters appointment types. When *billing* the exam, the biller enters procedure codes.

Grouped appointments

For even more flexibility when scheduling, you can create *grouped appointments* that contain one or more appointment types. As an example, a grouped appointment could have the following hierarchy.

Grouped appointment	Appointment type A	Appointment type procedure A
	Appointment type B	Appointment type procedure A
		Appointment type procedure B
		Appointment type procedure C

What's the difference?

- Appointment types are intended to contain multiple similar or related procedures.
- **Appointment types** can be scheduled from the schedule book or by using the Find Slots feature.
- **Grouped appointments** are intended to contain different exam types, such as an ultrasound and a mammogram—studies that take place in different modality rooms.
- **Grouped appointments** can only be scheduled by using the Find Slots feature.

Procedure code charge types

You can select a *charge type* for individual procedure codes that determines whether the procedures are billed to an ordering facility or insurance carrier, regardless of the claim's responsible payer. When you create a claim, Exa PACS/RIS automatically creates separate invoices for all procedure codes of each charge type—one invoice for the insurance carrier, and one for the ordering facility. Specifying charge types is especially useful when working with ordering facility billing rules that automatically add special charges to ordering facility invoices. The charge types are:

Ordering facility invoice

• Charges are placed on the ordering facility invoice.

No Split - Insurance

- Charges are placed on the insurance company invoice if the OF billing type is Global, or when the claim is created from the census queue and Global is selected.
- If the insurance provider is set up for split (technical and professional) claims, charges are added to the technical claim only, without a modifier, and with the fee from the default fee schedule (not the fee schedule mapped to the OF).

See also:

<u>Configure procedure codes</u> <u>Configure appointment types</u> <u>Configure grouped appointments</u>

Configure procedure codes

A *procedure code* (CPT code) is an internal code that identifies a medical procedure. Additionally, procedure code entries serve as detailed sets of information about procedures that Exa PACS/RIS uses for billing, ordering, and other functions.

When you configure a procedure code, you can add NDCs that technologists can select when performing the exam. This makes the process of searching for and adding the appropriate contrast material and other medications more efficient and accurate.

Prerequisite: For adding NDCs to a procedure code, complete the steps in <u>Configure NDCs</u>.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Scheduling & Codes** > **PROCEDURE CODES**.

		uilla a conce Signe - Mourve	ne bolikovat – / k	ioov muir	nii a	ve cenic som		NHOREDARE COD	a koc	006E	lenovieno
	i i i i i i i i i i i	-the				Abo as	ian.	MHORITOPI	1945	MEN CRIME	N HAP
	0000:+		EXAMPTER IN STRUCT	CLOCKL 1	1 01,5979.04	NADERTY.		NODHLITY	HODITIERS	307/97	TTTE BYR.
						AL.	-	at w	48	19 AL	Ψ.
100	NUM-	IMPLADMIN SAESCOVO		9	100	smoothcity1A	rgene	and the second			1
100	HEF	NRT FLECKE ASSESSE		ŧ.	16	unin Facily 1.0	nge Far				
	100234	ILIM ADMIN CARDOVS			15	rivin Facility 1/4		100 C			

2. Select ADD.

Facilities *	Al Facilities	Service Type *	Select
	Briecz tocilities •	Ref. Code	0001A
		NDC Code	
		NDC Measure	
		Color Code	
Code *	9001A Diractive	SDE Study	0
Short Description *	IMM ADMIN SARSCOV2 2		
Description *	NM ADAM SORSCOV2 8		
DENERAL	NDC CODE EXAM PREP INSTRUCTIONS DIATH	IDSTIC CODES DEFAILS	TIROPORT TEMPLATE ES
-	NEC CODE EXAM PREP INSTRUCTIONS DIADA	DUTIE OFFICE DEFAILS	TREPORT TEMPLATE ES
-			
-	Select v	Duration *	15
-	Sellect • Non-Trenscribable Require Copey	Duration * Default Units	15
Body Part	Sellect • Non-Trenscribable Require Copey	Duration * Default Units RVU	15

3. Enter the following settings.

Setting	Description
Single/Empty Code	[unused]
Facilities	Select one or more facilities that perform the procedure.
Code	Type your code for the procedure.

Setting	Description
Short Description	Type a short description for the procedure.
Description	Type a full description of the procedure.
Service Type	Select the service type to which the procedure belongs.
Ref. Code	Type the reference code of the procedure. In most cases this can be the CPT or LOINC code.
NDC Code	Type the national drug code associated with the procedure, if any. For a more advanced NDC function, see later steps in this topic.
NDC Measure	Type the unit of measure for contrast and/or liquid medications. The value for this is usually UN.
Color Code	Click inside the box, and then select a color in the picker to assign to the procedure.
SDE Study	Select the checkbox if the procedure is associated with echo ultrasound.

4. Select **SAVE**.

5. Optional. On the **GENERAL** sub-tab, enter the following settings, and then select **SAVE**.

Setting	Description				
Body Part	Select the body part associated with the procedure.				
Non-Transcribable	Select if the procedure cannot be transcribed.				
Require Copay	Select if the procedure requires a copayment.				
Require Physician	Select if the procedure must be performed by a physician.				
Require Waiting Time	Select if the procedure requires a waiting time before beginning (such as drinking Barium some number of hours before a procedure).				
Notes	Type notes for the procedure as needed.				
Charge Type	Select on which invoice to add the charge. For details, see <u>About</u> <u>procedure codes</u> , <u>appointment types</u> , <u>and appointment type</u> <u>procedures</u> .				
	Ordering Facility Invoice				
	No Split - Insurance				
Duration	Type the expected duration of the procedure.				
Default Units	Type the number of procedure units.				
RVU	Type or select the relative value units of the procedure for Medicare reimbursement.				
Level	The provider level code that determines the pay rate to the radiologist.				
Modalities	Select one or more modalities used in the procedure.				

- 6. Optional. To add NDCs to the procedure code, do the following.
 - a. On the **NDC** sub-tab, select **Add**.
 - b. Search for a code by entering criteria in the search boxes at the top and selecting **Enter**.

- c. Select the checkboxes of the NDCs to add, and then select SAVE.
- d. Optional: To make NDCs available to add with charges in the Technologist screen and elsewhere, add the procedure you created to an appointment type (see <u>Configure</u> <u>appointment types</u>).
- 7. Optional. For **EXAM PREP. INSTRUCTIONS**, see <u>Configure exam prep instructions</u>.
- 8. Optional. To add diagnostic codes (other than standard ones already on your system), on the **DIAGNOSTIC CODES** sub-tab, select codes from the dropdown list.
- 9. Optional. To associate a diagnostic code with the procedure code, on the **DIAGNOSTIC CODES** sub-tab, select a code.
- 10. Optional. To select a report template to be used for the procedure, on the **DEFAULT REPORT TEMPLATE** sub-tab, do one of the following.
 - In the **Template** List, select an existing template. To make templates available in this list, see <u>Add a report template</u>.
 - Select NEW TEMPLATE, type a template name, create a template in the word processing area, and then select SAVE TEMPLATE.
- 11. Select **SAVE & CLOSE**.

See also:

About procedure codes, appointment types, and appointment type procedures

Import specimen catalogs

If you obtain specimen catalogs in CSV format, you can import them.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Scheduling & Codes** > **PROCEDURE CODES**.

	100105	IDAE MOUTVH	E BOHERRAET	iolov mikin	ii ii	All CETIC COL	n 1	VIOREDURE COR	es Ho	1000E	licitore	İ
						400	astan	MERITOR	t Life	SMARK CRITAL	00 I	- 64
12	Hide Inact	h#										
	10000	descention .	EXAMPLE IN LTRUC	CLOCHL 1	0.049700	E PADEITY.		NODALITY	HODITORS	307/6	atiff 3	рv,
						AL.	13	AL Y	Evan .	(* AL	*	
12	HEA.	MR 40MR SARECOV		ф	15	ni.			(48	All.	*	-
	HEA.	MM ANNE SARACINA ART FALCHE SSSESS		a A	15		LAnge Tec	and a second second	48	trei] Alt.	*	

- 2. Select **SPECIMEN CATALOG**.
- 3. Select Choose File, browse for and select the file to import, and select Open.

- 4. Optional. To replace existing facilities, select the **Update Existing** checkbox.
- 5. Select **IMPORT**.

Import procedure codes

If you obtain exported procedure (CPT) codes, such as from the AMA or third-party billing companies, you can import them.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Scheduling & Codes** > **PROCEDURE CODES**.

	10116	ERAE RAGETVH	TE BON ERRALE - // (KOOV PINIT	ii i	white entropy	60 ji	PROBEDURE COD	68 Hold	00068	licitio az
						400	4000	MARKET CAR	t Lines	MAN CRIME	n H
12	Heleinard	hue.									
	20000	descention.	EXAMPLE: NUTROS	CLODAL 1	0.0470	N PADEITY.		NODALITY	NODITIERS	307/92	TITE IS
						(AL.	11	AL W	(48	W AL	4
12	HELA .	MR 40MR SARDOW		ф.	15	AL.			(all	Ψ All	٣
	HEA.	INTRACING SARESCOVE ART FRADE ASSESSE		я я	15 15		1Arge the	1010	40	. Ψ][A I.	¥

- 2. Select **IMPORT CPT**.
- 3. Select Choose File, browse for and select the file to import, and select Open.
- 4. Optional. To replace existing facilities, select the **Update Existing** checkbox.
- 5. Select IMPORT.

Configure NDCs

An NDC (national drug code) is an 11-digit unique identifier assigned to medications listed under Section 510 of the United States Federal Food, Drug, and Cosmetic Act. The first 5 digits identify the manufacturer of the drug, and are assigned by the FDA. The remaining 6 digits identify the product package and size, and are assigned by the manufacturer. You can follow the procedures below to preconfigure NDCs either manually or by importing them.

When adding charges such as during technologist exams or in the Edit Study screen, you can add the NDCs you create here to the charges. To make this available you must first:

- Associate the NDCs with procedure codes. For example, create a procedure code for contrast (service type Diagnostic X-ray) and add the NDCs for contrast. See <u>Configure procedure</u> <u>codes</u>.
- Add the NDC-associated procedure codes to appointment types. See <u>Configure appointment</u> types.

Add an NDC manually

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Scheduling & Codes** > **NDC CODES**.

									1
	FERMINAL	PROVINGENERAL	Showing	Involution code	i inte	iloreccada	HOCCOL	65	ioionisno)
					AUDI		minor	Links	e eae
	PROPERTIES NAME :	ICE REPRESENTATION	3010 HE 0106	SECURE DESCRIPTION OF	LESC FOLDS	- (88 00	CONIC		
								AP .	
1	B Excelan ProTrole 4/100	n Acesannoones, Augurn () (7203	8	10 F0 (2>++++ 80x >++ 84	Recontrar	Comune Tables	Convec	7465	

2. Select ADD.

115 Section 10		00071-0072	alimente anna	(REACHARD COLORS)	NOC COORS 10-10
					-14446
Proprietary Name -	Excedit: PrinTriple Action Ga	pletat	Labeier fileree *	Gescardt Aline Core	une Heathcare Hald
Non Propiletely Name 1	Aptantropher, Aprin (nue	freduid In A du	Product NDC *	Table, Cost of	
NOC Package Code *	572060		llocte	Nesd	~
Package Description *	10 POUCH in 1 20% > 1 PATE	Hin 1 POUCH /			
10,000	3	cm g 👻			

3. Enter the following settings.

Setting	Description
Proprietary Name	Type the trade name, or the name of the drug chosen by the labeler.
Non Proprietary Name	Type the generic name or the active ingredient in the drug
NDC Package Code	Type the code from the manufacturer.
	Exa Billing users : Remove dashes and add zeros to make the NDC package code 11 digits, conforming to these format examples:
	$XXXX-XXXX-XX \rightarrow 0XXXXXXXXX$
	$XXXXX-XXX-XX \rightarrow XXXXX0XXXX$
	$XXXXX-XXXX-X \rightarrow XXXXXXXX0X$
Package Description	Type the description of the package from the manufacturer.
Strength	Type the strength of the active ingredient, then select the units of strength in the UOM dropdown list.
Labeler Name	Type the name of company corresponding to the labeler code segment in the Product NDC.
Product NDC	Type the labeler code and product code segments of the NDC number, separated by a hyphen.

Setting	Description
Route	Select the administration method for the drug.

4. Select SAVE.

Import NDCs

1. Create a CSV file of NDCs that include the same options in the previous table.

Hint: Select the **Help** button to view a list of the column names and their order.

- 2. Go to **OFFICE** > **Scheduling & Codes**.
- 3. On the NDC CODES tab, select IMPORT.
- 4. Select Choose File, browse for and select the CSV file, and then select Open.

Optional: To append (and not replace) the CSV file to existing NDCs, select the **Update Existing** checkbox.

5. Select IMPORT.

See also:

Add charges and NDCs in the exam screen for technologists

About schedule blocks

You can reserve blocks of time on the schedule book for modality rooms at selected facilities, and you can select whether to be able to schedule exams during the block. Each block is given a *reason* (such as a specific patient's name), so that if scheduling is allowed, the scheduler knows what exams they can order during the block (such as ones needed by that specific patient). There are three types of schedule block:

Hard block: Cannot overlap with other blocks

Soft block: Can overlap with other blocks

Quick block: A schedule blocks that you can create "on the fly" directly from the schedule book

To create a schedule block, you create a schedule rule within a schedule template and select the "Display as a block on schedule book" option. See <u>Configure a schedule template and schedule rules</u>.

See also:

About schedule templates and schedule rules Create a quick block

About schedule templates and schedule rules

A *schedule rule* determines when appointment types and modality rooms can be scheduled. Think of schedule rules as the "office hours" for exams. When working in the schedule book, Exa PACS/RIS prevents you from scheduling an exam that breaks any rule you configure, such as by hiding modality rooms on specified days, or by warning you with a message.

Inclusion and exclusion rules

- An *inclusion rule* defines when something is *available*
- An *exclusion rule* defines when something is *unavailable*

A *schedule template* is a container for a set of schedule rules. It also links to one or more appointment types and one or more modality rooms.

Schedule rules and the Find Slots feature

Find Slots uses schedule rules to determine the availability of timeslots in the following manner:

- 1. If **no rule** applies to a slot, the slot is always available.
- 2. If an **exclusion rule** applies to a slot, the slot is not available.
- 3. If an **inclusion rule** applies to a slot, regardless of time:
 - All other slots are unavailable.
 - A priority is set for the inclusion rules in the following order: Specific date, yearly, monthly, weekly, daily.

For example:

- If a specific date and a daily inclusion rule apply, the specific date rule is applied and the daily rule is ignored.
- If two equal priority rules exist (such as two daily rules), they both apply (they "stack").
- Inclusion and exclusion rules can apply together, such as a weekly inclusion rule and a daily exclusion rule.

See also:

Configure a schedule template and schedule rules Use the Find Slots feature

Configure a schedule template and schedule rules

You can configure a schedule template, and then add schedule rules to the template that define scheduling availability of your modality rooms and other resources.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Scheduling & Codes** > **SCHEDULE TEMPLATE**.

	in a contest Invester i func	110-14-111		-cooninvesta	ciús	bdress	LE TEMPLA	mai (Ali	101	nver me	1.1	LDOILLI	100011
Tomptico Nama (1992)	t ettödun tempiote		3	D stownsow	2			L	Act	TUMPLATE		SMIC 1	uru
Schedule Rule	85										REU	siko	400
RACEMANE	(MOLITY)	- 29	0094.2	MODALITY/ROOM		DOMINANT.	010 DATC	REDUKIREE		APPOINTNED	THE	A/AEA	FADTHE
	1.441	10	ii y	-	÷			+(÷	(4)		#1.9P	111 -
ASTOCAR .	Werpulia Text Eaclif	y c	t : :	ct									~

- 2. Select **ADD TEMPLATE**.
- 3. Type a name and description for the template and then select **SAVE**.
- 4. Under Schedule Rules, select ADD.
- 5. In the schedule rule screen, enter the following settings (see example screen below).

Setting	Description
Template Name	Select the template to which to add the schedule rule.
Rule Name	Select Predefined or Free Text , and then type or select a name for the schedule rule.
	The Predefined list contains "schedule rule reasons," which you can define here: Reason codes
Modality Rooms	Select a combination of these items along with dates and times to define
Appointment Types	the rule. For example, your rule could state that:
Resource	"Resource A can perform appointment type B in modality room C."
Туре	Select Repeating if you want the schedule rule to recur, or Date/Time if not.
Not Available	Select Not Available to create an exclusion rule (to define when <i>not</i> to schedule appointments). Clear the checkbox to create an inclusion rule.
Display as a block on schedule book	Select if this rule defines a schedule block on the schedule book. Select the hexadecimal color code box, and then use the color picker to assign a color to the block.
Ignore rule in scheduling	Available when you select Display as a block on schedule book .
engine (use only for displaying text/color label)	Select this Ignore rule checkbox to display the block on the schedule book, without the program taking it into account when finding slots.
Autoblocking	If you select a technologist resource for the rule, and the technologist is available to work in two modality rooms at the same time, select Autoblocking to prevent double-booking that technologist.

- 6. Do one of the following
 - To save the current rule and add another rule, select **SAVE**.
 - Select SAVE & CLOSE.

inactive			
Template Name *	Jienn's Termplete		
Rule Name *	O Predefined 🛛 📵 Free Te	aut -	
	Lunch 12-1pm		
Modality Rooma	EDIT MODALITY BOOMS	Appointment Types	EDIT APPOINTMENT TYPES
	80-H (60-H) 0TH (6TH)		
	MGH (MGH) MRH (MRH	2	
	UBH (UBH)		
	XRAY-H (XRAY-H)		
Type *	Repeating 🐱	Start Date	19/03/2022
	C Not Aveilable		
	Display as a block on schedule (book	
	I grote rule in scheduling engine	(100	
	only for displaying text/color label)		
Repeato	Daily 🗸	Every	1 v days
	From 12:00 PM 20	Te 1:00 PM	
Summary			
	day from 12:00 PM to 1:00 PM begin	ning 10/03/2022	

Example of a schedule rule: Technologist lunch break

See also:

About schedule templates and schedule rules About schedule blocks Create a quick block

Configure a schedule filter

You can create a filter that controls what appears in the schedule book. For example, you could create a filter that only shows two modalities during a four-day period.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Scheduling & Codes** > **SCHEDULE FILTER**.

	SCHEDGLIN	CINCODES		a			ं
	(Internation	wennes	NOHOLEHURN	THOUGHAINS	Telescolettica (marcate	 H HILTH	e- nr- 3
			1 A 2			400	111.040
	PETER MANN +				inve.	 -	
)# 1	ll Tex				14.)		

2. Select ADD.

Fital Norse * No. of Dens. *	Toat:			
Locations Modelity Rooms	My Contrainty	1	Splortpri Roome	+
C managery and the	Computer Torns View	3		
	Note: Held Cat or Shift to	velect multiple raudality soom		

3. Enter the following settings.

Setting	Description
Filter Name	Type a name for the filter.
No. of Days	Type the number of days to display on a single page of the schedule book. If the content of the page exceeds the available display width, a scroll bar appears.
Locations	Select a facility in the dropdown list. The modality rooms at the selected facility appear in the Modality Rooms list.
Modality Rooms	Select one or more modalities in the list, and then select the right arrow button to add them to the Selected Rooms list. These are the only modality rooms (columns) that appear in the schedule book when you apply the filter.

4. Select SAVE.

Configure study statuses

Workflows in Exa PACS/RIS are status-driven. To create or tailor workflows, you can create a set of study statuses and then configure how studies move from status to status (see <u>Configure study flow</u>). Statuses are unique to each facility, so if you want a status to be available in multiple facilities, repeat the steps for each facility. To add statuses:

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Scheduling & Codes** > **STUDY STATUS** > **STUDY STATUS**.

=	SCHEDULING	F& 000E3	1		2
		NOC IDDIES	no-rou.c+icm	STUDY STATUS	(180
	SUTATION STATUS	BTUDY FLO	W		
Focility	* My C	empany	* worth	2	A00
	ORD/Ordered	0%e)	10		
	PR/Precheckin	(466)			
18	SCH Scheduled	Citer			
	CON/Confirmed	Ser			
	CHI/Check-In	Orter			

2. In the **Facility** dropdown list, select a facility, and then select **ADD**.

Genalal Information			Things to Validate	Always abox valuation weekee
Utstas Code *			Patient Address (Address, Gity State, 2)P)	🛄 сет соде
	Add to Study Bho	tout Menu on Waitilist		Diogricolo Code
Status Gesenation *		C Excherence Invested	D Patient 20M	Pyley, Alem
Colur Citide		C Other Related	E Falient Contact Number (Home or	🗌 Reason for Shaty
Max Well Time	000.040		Mobile)	Document Review
		And a subscription of the subscription of the	🔲 Paferring Physician	
		SAME CANCE	Primary insurance	
Notification Bettings				
tenui Template				

3. Enter the following settings.

Group	Setting	Description
General	Status Code	Type your internal code for the status.
Information	Add to Study Shortcut Menu on Worklist	Select to add the status to the study row shortcut menu on the worklist.
	Status Description	Type the name of the status.
	MobileRad Related	*Required for the Mobile RAD app.
		Select if the status is for Mobile RAD.
	Color Code	Click inside the box, and then select a color from the picker to color code the status.

Group	Setting	Description
	Order Related	Select if the status applies to entire orders. Clear if the status applies to studies.
	Max Wait Time	Type the number of minutes to complete check-in. If the time is exceeded, the "Exceeds maximum time limit" count at the bottom of the worklist increments.
Things to Validate (not available for	Always Show Validation Window	Displays the validation results window whether or not you select any Things to Validate.
all statuses)	[Other checkboxes]	Select the items that Exa PACS/RIS validates (checks for inclusion) before moving the study to the next status. If items are missing the validation window opens for you to add them.
	Document Review	Select to require document review to advance to the next status in the study flow. Select the documents for which to require review in the button list.
Notification Settings	Email/Fax Template	Select templates to enable notification to be sent when a study arrives at the status. To select who receives the notifications, turn on Notification settings of individual resources (see <u>Add a resource</u>).

4. Select SAVE.

See also:

Require document review

Configure study flow

Exa PACS/RIS provides a graphical representation of the flow of studies from status to status at each facility. You can define these flows, and also apply routing rules to control whether and how data can move to the next status in the flow. The easiest way to start a new flow is to copy one to a new facility by selecting the source and destination facilities in the lists (see figure) and selecting **COPY**.

Note: You do not need to apply routing rules for manual DICOM send operations.

Caution:

- Changing the study flow incorrectly can "orphan" studies, leaving them unable to move to the next status in the workflow.
- Selecting RESET clears all non-saved changes and restores the default flow.

Procedure

=

- 1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Scheduling & Codes** > **STUDY STATUS** > **STUDY FLOW**.
- 2. In the **Facility** dropdown list, select a facility.

STUDY STATUS STUDY FLOW		
Facility - My Company - Source Basar	Angle Facility Gookafeic/Facility Meena Fecility Meena Fecility Meena Fecility	
Didnet Straties	the Content Stand	
	Eréchecias	
Caeping -	ateins	
	Trais Dant Conditional	
A p	Creato	D
Section Section	Tech Statt	

- 3. To modify the flow:
 - Drag a status from the right pane into the main flow diagram.
 - Drag the orange bar in a status node to another status, and drag arrows.
- 4. Optional: To add a routing rule between statuses:
 - Selecting the triangle on an arrow.
 - Under Actions, select a routing rule, and then select OK.

Result: When studies change between the statuses, they do so per criteria defined in the selected routing rule. For example, the routing rule might send the study to a specified AE along with user-defined priors. See <u>Configure routing rules</u>.

- 5. Optional: To force flow between statuses:
 - Selecting the triangle on an arrow.
 - Under Force Flow After, type or select a number of minutes.
 - If available, select modalities to which the force flow applies.
 - Select OK.

Result: Studies automatically flow between the statuses after the specified number of minutes.

6. Select **SAVE.**

See also:

Configure study statuses

Add a transcription template

You can create a template that contains pre-formatted text and merge fields to streamline transcription. A *transcription template* is for formatting a transcription, whereas a *report template* is for final formatting of approved reports. To use a transcription template, see <u>Dictation and transcription with Exa Voice and Exa Trans</u>. To create a transcription template:

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Scheduling & Codes** > **TRANSCRIPTION TEMPLATE**.

-[D.A.CODELL					e
	Currictero.	sociaces	SOUTH LEVEL	atusi atensi	TRANSCRIPTION TEMPLATE	Internet interaction	FUNCTIONAL DTK.)
-	TEMPLATE NAME	0				· ·	ARM IRLEAD
	E ELAPCOTE MAVE	20					
2.8	Tropinte free						
	Techoo						
1.8	TRACEAND						

2. Select ADD.


						NAME ADDRESS	a
Tensine Rane 1	1	100	HALLING IN	ARE WILL REPORTED WHY	HICOPING REPRESENCE		
SIRAY TROD		2	1	Ne VIEW A	NEXE: SHE	E . Pauplase Na	100
La Inactive D Global Maura Represent		Dyt	-1	tee i i i vers, stil Z · A ·	第二日日) Per	(0·£·2·≡·	N Pr
Vapi		(k) (*.		····· • ··· • · · · • · •			
guarrettan.	8 +	1010					
				IXAM: RADIOGRAPH OF THE			
11 12 - 12 - 12 - 12 - 12 - 12 - 12 - 12		3	1	echnique: PA, oblique and lateral w	ens of the hand are	submitted.	
Appolitizers Types		- C	0	Inical Data: Pilin			
being appearance these	6 +	- C					
P		ê l	P	nor Studies			
		100000-00000000000000000000000000000000	19.00 C	indings: There is regional esteopen intigace randowing sciences and is at levels, and first carpal-imstacarps builtedicatofied density along the ta- hanges associated with provious that cute fracture or other acute observes.	surring present. Early or 6 joint. Correlate with e asso of the first metacal ama. Soft to succare un	roove changes PIP at roove osteoarthatic rat may reflect soft too	nd DIP Sup
C Regula All Metty		3		noression			
Facilities				it is a subility of the sub-			
Botest / Auditors	8 *	(1) (1) (1) (1) (1) (1) (1) (1) (1) (1)	F	lecommendation:			
and the second s		196.32.15	nerrol a	el Cantal	1 7 3		¢0-
Modully *							_

3. On the left pane, enter the following settings.

Setting	Description
Template Name	Type a name for the template.
Global	Select to make the template available to all users.
Macro Keyword	Type a macro keyword that can be voice-recognized to open a template.
	Tip: Add a starter word in front of the keyword to make it easier to call up macros/templates.
	Example: macro chest
User	You can restrict availability of the template by assigning users to it. Select a user in the list and then select the plus template to multiple users.
Appointment Types	Select for which appointment types (CPT codes) the template is available.
Require All Match	The template is only available to users and studies who match all criteria you enter here.
Facilities	Select for which facilities the template is available.

Setting	Description
Modality	Select for which modalities the template is available.
Body Part	Select for which body parts the template is available.
Study Description	The template is only available for studies with the study description you enter here.
Institution	Select for which institution the template is available.

- 4. Select SAVE.
- 5. In the template editor, type text and apply formatting for the template, and then select **SAVE**.
- 6. Optional. To paste items onto the template that you copied from outside Exa PACS/RIS, on the **HOME** tab, select **Paste**, and then in the button shortcut menu, select **Paste**.
- To add merge fields, select REPORTING (1) > Insert Merge Field (2), and then select a merge field (3).

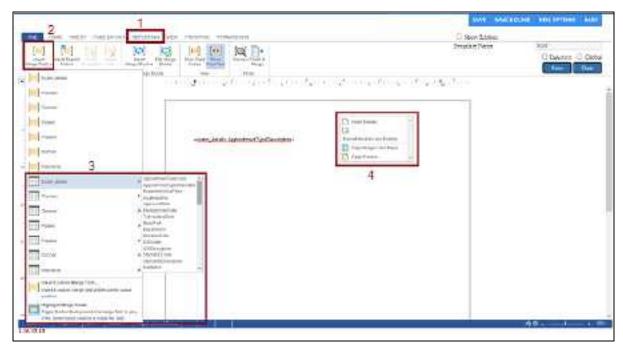

- 8. For formatting options (4), right-click in the body of the template.
- 9. When finished with your template, select SAVE & CLOSE.

Add a report template

A *report template* is for final formatting of approved reports, whereas a *transcription template* is for preparing transcriptions. You can create report templates and enhance them with logos, tables, and merge fields. When you approve a study, Exa PACS/RIS generates the approved report using a template you create here.

Which template is used?

The template settings must match the facility and modality of the study. If several templates match, the highest one in the list is used. If none match, none are used, and the approved report contains only the body of the transcription.

Add a report template

Add a report template with the following steps, and then continue to the next procedure to edit the header or footer.

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Scheduling & Codes** > **REPORT TEMPLATE**.

SCHEDULING & CO	ones.					5
 (income) (income) 	ciana sourcarround	rusinario	Transformed Televise	IEPORT TEMPLATE	a conti	NOVAL USE
				-	480	111040
TERRE ATE NAME +	Cavinson -		WGUTY.		MARKET .	
			1 Xiii	12	141	
A BHMM Server	Sargle (might	u ál muðaltisa	Argin Facil	(),My Swigany,Nirin⊟a Tes	£)	
dalime.			Argetest	ty data we have have by Alwers	Internet	

- 2. Do one of the following.
 - To base the new template on an existing one, select the copy ¹/₂ button of the existing template.
 - To edit an existing template, select its edit 🖉 button.
 - To start a new template from scratch, select **ADD**.

Territe lans Trac Reporter ales 🔹 report	Modatity #	Dady Decorption
Manat King Malagoo		
Alcoses * BUDHAUTY *		
Asttana		
	Sames	
Addendare Laustice D. The 🛞 Boltom		

3. Enter the following settings.

Setting	Description
Template Name	Type a name for the template.
Market	Optional. Select the market to associate with the template. All facilities in the selected market become selected in the Facilities dropdown list.
Facilities	Select all facilities that use the template.
Institution	Select institutions that use the template.
Modality	Select all modalities to which the template can apply.
Summary	Type explanatory information about the template.
Study Description	Type study descriptions for the templates to be applied. For example, US Abdomen could have a different template than MRI Abdomen.
Addendum Location	Top : Select to place addenda before the original report (top of the first page).
	Bottom : Select to place addenda after the original report (bottom of the last page).

4. Select SAVE.

5. Perform the steps in the following subsections as needed.

Add a header and footer

You can configure the header and footer of a report template, or edit existing ones.

1. At the bottom of the page, select the **PAGE HEADER/FOOTER** sub-tab.

Result: The word processor appears (it may take a few moments).

2. On the **INSERT** tab, select **Header** or **Footer**, and then in the button shortcut menu, select **Edit Header** or **Edit Footer**.

2	NGE HEADE	R/FOOTER	REPORT	SIGNATURE				
FILE	HOME	INSERT	PAGE LAYOUT	REPORTS VI	ew ya	BLE LAYOUT		
Dimik Pope	1 Table	invige S	Rospes Chart	Hapedrik Bookmank	Header	Forter Page	Teut Prame •	Ω Symbol
Pages	Tables	n.	skalium	Links	Head	icas & Fealtria	Text	Syntal

- 3. Type information for the header or footer.
- 4. Select SAVE.

Add a table

Although not required, it can be helpful to organize information on your header with a table.

- 1. On the **INSERT** tab, select **Table**, and then in the button menu:
 - Select a cell layout, or

- Select Insert Table dialog, enter table settings and then select OK.
- 2. Type text in cells, and select **Save**.
- 3. See below to add logos and merge fields to the table.

Add a logo

1. Prepare a bitmap image and save it in the following:

[drive]:/EXA/WebImages

2. Place the cursor in the header or footer where you want to place the logo.

Note: It may be helpful to add carriage returns to make room for the logo.

- 3. On the **INSERT** tab, select **Image**, and then in the button shortcut menu, select **Image**.
- 4. In the **Open File** dialog, type the name of the bitmap image and then select **Open**.
- 5. In the image shortcut menu, select **Format**.
- 6. In the Image Attributes dialog, under Wrapping Style, select In Line, and then select OK.
- 7. Optional. On the **HOME** tab, select the text alignment tools, or drag the image to position it.
- 8. Select Save.

Add merge fields

- 1. Place the cursor in the header or footer where you want to add the merge field.
- 2. On the **REPORTING** tab, select **Insert Merge Field**.

	FORTING VEW	PROCIFING PER
issert special card card	set Est Wege Block • Blocks	(iii) Show Field Coden View
Common higher- level marge fields Lower-level, detailed merge fields → Practors Incom Incom	AppartmentTypeC AppartmentTypeC AppartmentTypeC ExpensetAmmaTim AcpuriveCatte ProApproveCatte TranscribedDate BodyPart Department DictationCate ICDCope ICDCopertplan StudyCDDepartplan StudyCDDepartplan	cockydio
Provider	thattution	
C General		

- 3. On the **Merge Field** button shortcut menu, select a merge field, or choose a field from one of the sub-menus.
- 4. Select Save.

Add a report signature line

- 1. Select the **REPORT SIGNATURE** sub-tab.
- 2. Type text and add merge fields as described earlier.

	REFERENCESSION OF Statements Sectory of Sover Sover Sectory of Sover Secto	Dear
Lange Control Gamma	Image Entropy Manue + Social manue finais Image Final manue finais Image Final manue finais Image Final manue finais 2 1	
4	Electronically Signed By: *Recipite Signatures *Dioble Signatures *SignatureSoft	

3. Optional: To include a key image in the signature area, add the **<<keyimages>>** merge field.

4. Select the **SAVE** buttons on the top and bottom of the screen.

Configure functional and cognitive statuses

Functional and cognitive statuses are preset descriptions of a patient's level of physical and mental functioning that can be used in other parts of the program.

Procedure

- 1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Scheduling & Codes**.
- 2. On the **FUNCTIONAL STATUS** tab, in the **Description** box, type a functional status and then select **SAVE**.

i i	REN REN	OIT TEMPLATE	FUNCTIONAL STATUS	COUNTIVE STATUS	BENEDULE TEMPLATEB
		DESCRIPTION			
ō	п	Ambulatory			
5	8	Care/waller			
ription	· 1				

3. On the **COGNITIVE STATUS** tab, in the **Description** box, type a cognitive status and then select **SAVE.**

What are lead time and expected arrival time?

The *lead time* is the number of minutes prior to the appointment time that the scheduler wants the patient to arrive. The program calculates an expected patient arrival date and time (the *expected arrival*

time) by subtracting the lead time from the appointment time. The scheduler may find it convenient to inform the patient of this expected arrival time rather than the formal start time of the appointment.

If an appointment contains multiple exams, the program calculates the expected arrival time based on the lead time of the first exam. If the first exam has no lead time, it uses that of the second exam, and so on. If no exams have lead times, the expected arrival time is simply the time of the first exam.

Configure appointment types

An appointment type defines a set of procedures to perform during an exam. You can configure appointment types for use in scheduling and other parts of the program.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Scheduling & Codes** > **APPOINTMENT TYPES**.

	-	14.	neccine	ATALIS	a ta Milana	1000	a main and	APUNT	MENT TYPES		009 MAT
tioner.	s	LEISTED -	4				Report	EXPORT	HELP	400	ani.
(core	ere.come	NAME.				11001			Gen	LLET MADE	mo i Al
									11/44	4) M	- 4 H
ESATT	CAPP1	ComboAgo	true!						cr		
II water	INTESOL IN	angest.							1908	KNN (

2. Select ADD.

lioda *	CAPPH	🗆 tustie		Facilities *	ALL SILLECTED (N -		
net code	C81991			moters-	m+		
Name *	Combo4p	offypet		Modality Room	NEHIC BELEVITED -		
Description	Country Ser			Reprints	tayvoid 4		
Duration *	15						
Add on a type	6 E 🗌						
LexitTime							
Dias Court	1						
	0.0008	Appointment					
A	reastwear TV	EPROCEMPES	EXAMPLE/ROOMLEVEL CONTRE				
						Alt:	16000
	CODE	cescrum	non				
1.5	240.75	UT 480 1	PEUXIS NO CONTRACT				
1.2	70128	CT 68/1	PICAMIN.				

3. Enter the following settings.

Setting	Description
Code	Type your internal code for the appointment type.
Ref. Code	Type the reference code.
Name	Type a name for the appointment type.
Description	Type a description for the appointment type.
Duration	Type the expected duration of the appointment type.
Additional Time	Type the amount of additional time needed beyond the default duration.
Lead Time	Type the number of minutes to arrive before the exam begins.
Exam Count	Type the number of exams to be included.
Grouped Appointment	Select for grouped appointments (see <u>Configure grouped appointments</u>).
Facilities	Select one or more facilities where the appointment type is available.
Modality	Select one or more modalities to use for the appointment type.
Modality Room	Select one or more modality rooms where the appointment type can be scheduled.
Keywords	To make the appointment type easier to find during scheduling, type a keyword and select the plus button. Add as many keywords as needed.

- 4. On the **APPOINTMENT TYPE PROCEDURES** sub-tab, select **ADD**.
- 5. In the **Appointment Procedure** dialog, select a **CPT Description** (procedure), up to four applicable modifiers, and up to four diagnosis codes, and then select **SAVE.**

Note: When you schedule the appointment type or add it to an order, the modifiers and diagnosis codes you enter here are automatically added to the order.

- 6. On the **EXAM PREP/ROOM LEVEL CONFIG** sub-tab, add or edit exam prep instructions as needed.
- 7. Select SAVE.

See also:

<u>Configure exam prep instructions</u> <u>About procedure codes, appointment types, and appointment type procedures</u>

Configure exam prep instructions

For each appointment type you can assign exam and room preparation instructions. The instructions are linked to modality rooms so that if an exam is rescheduled to a different room, the instructions change accordingly. To configure exam prep instructions, start by following the procedure in <u>Configure</u> <u>appointment types</u> and then:

Procedure

1. On the EXAM PREP/ROOM LEVEL CONFIG sub-tab, select ADD.

	* 🔲 Select All Modality Room	12.5
	Select modulity rooms	+
Duration		
Additional Time		
Additional Time		
Additional Time		

2. Enter the following settings.

Setting	Description
Modality Room	Select the modality room to which to apply the prep instructions.
Duration	Type the estimated duration of the exam.
Additional Time	Type the amount of additional time needed beyond the default duration.
Lead Time	Type the number of minutes to arrive before the exam begins.
Prep Instructions	Type the exam and room preparation instructions.

3. Select SAVE.

Configure grouped appointments

A *grouped appointment* is an appointment type that contains more than one billable procedure. Grouped appointments make it easy to schedule more than one procedure or exam during a single patient appointment. To configure a grouped appointment:

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Scheduling & Codes** > **APPOINTMENT TYPES**.

			CONTRACTOR OF STATISTICS	STREET, THEF, STREET,	MPONI	ART TYPES	10	un secus
	s			NATION T	EXPORT.	Imr	400	10.04
Cacilles		CECTED C						
(008	****	NARM.	1012.00P	1111.5			LTT MARKED	TO FROM
							114	4.14
E DATE	CATT!	SphiloAap/Tale!				cr		*

- 2. Select ADD.
- 3. Select the **Grouped Appointments** checkbox.

on Charles	Facilities *	ALL SELECTED	(6) -
BEH 0040W0M0K	Keyurarda	Raywurd	+
OrcupExample			
Description			
0 minutes (pulled from one of the appointment type	es below)	2	
2			
Couped Appointment			
	BEN-006QW0MUK DroupExample Description Consistent (pulled from one of the appointment type 2	SEN-006QW0MUK Keywords DroupExample	BENJOOSOWOMJK Keywords DroupExample

4. Enter the following settings.

Setting	Description
Code	Type a unique code for the grouped appointment. The code does not need to match an existing procedure code.
Ref. Code	Type the reference code, if any.
Name	Type a name for the grouped appointment type.
Description	Type a description for the appointment type (it can be the same as the name).
Lead Time	[Determined automatically]
Exam Count	Type the number of procedures to be included.
Grouped Appointment	[Already selected]
Facilities	Select one or more facilities where the appointment type is available.
Keywords	To make the appointment type easier to find during scheduling, type a keyword and then select the plus button. Add as many keywords as needed.

5. Select **SAVE**, and then in the message box, select **YES**.

6. In the **Appointment Types** dialog, select an appointment type to add to the grouped appointment.

Appointment Types						
Appointment Type *	Select an appointment type	Ŧ				
Code •	Code					
Minutes to wait	0					

- 7. Optional. In the **Minutes to wait after exam** box, type the expected minutes the patient must wait between appointment types.
- 8. Select **SAVE**, and then in the message box, select **YES**.
- 9. Repeat steps above until all appointment types are added to the grouped appointment.

IMPORTANT: Add the appointment types in the order that they are performed, or in the correct scheduling priority.

10. On the APPOINTMENT TYPES tab, select SAVE.

View locked slots

Exa PACS/RIS "locks" timeslots as soon as a scheduler selects them. The prevents other schedulers from simultaneously double-booking in the same slot. You can view which slots are currently locked with the following steps.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Scheduling & Codes** > **LOCKED SLOTS**.

3	Here wath all		commissions.	STREET IN COMPANY	ANTIRIMANTINGS	EDOGED SERVIN				
Select	a facility to see the li	st of thet facility's mod	ality rooms, Then click	e modality soom to view	reserve; or release loci	ked time slots.				
Facilitie	es									
a second s										
Locked Data	Fectorial	Facility Harm	5	addity Coale						
	Pacific M	Enclity Name								
Lucked Slots			a a a a a a a a a a a a a a a a a a a							

Configure DICOM settings

Administrators can configure DICOM settings such as for AEs, file stores, and receiver and routing rules.



Note: The default AE title for Exa PACS/RIS is EXA_SCP, or EXA_MWL when acting as a worklist source. These titles are configured elsewhere in the program by a Konica Minolta installation engineer.

This section contains the following topics (not all topics may be available depending on version and region).

Configure application entities Configure a DICOM printer Configure a file store Configure routing rules Configure AE scripts Understanding receiver rules Configure a receiver rule Configure matching rules Configure matching rules Configure hanging protocols Manage jobs in the transfer and SR queues Configure an issuer of a PID Automatically retrieve priors from an external PACS

Configure application entities

An *application entity* (AE) *title* is a named configuration for a DICOM application that includes information such as its type, host computer, and port number. You must set up AE titles for Modality Worklist, send, receive/store, print, query/retrieve, and other functions to make them available on Exa PACS/RIS. To configure an AE, use the following procedure.



Caution: Failure to correctly configure application entities can cause general failure of DICOM communication between Exa PACS/RIS and those entities.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **DICOM** > **AE TITLE**.

	-	-stating stats (ACTORNIS	, activitations,	MATCHINE RANKS	Watane water	- • e
🖸 Hida Inagtiva						420 88	0/0
TITLES	HOCT	PERT	DEDUILITION.		1.37 OFE	ALTYN	
						42	
a provinciona	10.227.25.76	3444.85	EU-102-02	Ers	Server Etsages	sandysectories of	TERVISET IN
CLD, AND THE			OLD ARDITIVE	007	NUL7 01002	PECCIVE.	
21 OPALIZAR	10.227.24 300	88834	OP44_15_98	Ever.	Sarver Images	SENDARCEVED	IERV/RETR

2. Select ADD.

4E Type :	sense PRAS	e)	AE Flag	Select 🗸
	I IECENE	1	bonutos.	
	OUCH/RETRIEVE		file 28% *	Select v
At Tite (Return) 1	The second se	Vector	Pom *	C Percine 224
	🗍 ianare Scan Dasa (Senii)		Tiansfel Suitar (Sendi	Selari V
My AE Title			Transfer Syntax	Select v
Host Maine *			(RECEIVE)	
Description *		Disable QC2LIVE	Veticle	Any mobile unit
Reties	6	Cand Single Incluize per Study		
Max Censultant	36		SDEMOOHIN	0
Receive Facility -	Select factory + O		WOreday Approvald Babor	
Send Pacifices	NO RESIDENTION	0	The set of	
Send Annotations as	Belevit .	*		
Send Unique Invage UID				
Finst Viewer-				
Ignore inbound Stat	ج 🗆			
DMWL Masislees *	0 #0	÷.		
	🗆 នា 🗋 ពា			
DMW, Fectber *	argaunitae Argebrachty degeFacity	Ť.		
	D. Brenant Facility			
losse: Tirge 1	S NOW O SINE C	Dynamic .		

3. Enter the following settings, and then select **SAVE.**

ÿ

Note: The actual settings that appear on screen differ by the AE types you select.

Setting	Description
АЕ Туре	Select one or more transactions that Exa PACS/RIS makes with the AE.
AE Title (Remote)	Type a title for the AE of up to 16 characters. If an AE title is preconfigured on the device hosting the AE, use the same title.
Ignore Scan Docs (Send)	Select to prevent sending of scanned documents to the AE (only DICOM studies are sent).
My AE Title	Type a title of up to 16 characters to set the identity of Exa PACS/RIS individually for each DICOM node. In most cases, leave blank to use the default (EXA_SCP for PACS/RIS functions, and EXA_MWL for modality worklist functions).
Host Name	Type the host name or IP address of the AE.
Description	Type a description for easy identification of the AE.
Disable QC2LIVE	Select to disable the automatic transfer of studies from QC to live.
Retries	Type the number of times the system attempts to reconnect with the AE after a communication failure.
Send Single Instance per Study	Select to send one image per study.
Max Concurrent	Type the maximum number of concurrent threads that the system can use for the transaction.
Receive Facility	Select a facility to receive studies sent to the AE.
Send Facilities	If you select facilities here, only users assigned to those facilities can select them for DICOM-sending (leave blank for all facilities and users). This is effectively a user filter for the send destinations list in the Send Study dialog.
Ignore Inbound Stat Value	Select to keep the Stat level of the study from Exa PACS/RIS.
	Clear to use the Stat level of the study as received.
Send Annotations as	Select how to process annotations for sending.
	Burned in: "Flattens" the image and overlay into a single image.
	DICOM Overlay : Converts the annotations to a DICOM overlay and sends along with the study.
	GSPS/PR Object : Converts the annotations to a presentation state and sends along with the study.
Send Unique Image UID from Viewer	Select to generate a new UID for each image (rather than just for the series or study).
DMWL Modalities	Select to which modalities to serve a modality worklist.
DMWL Facilities	Select to which facility to serve a modality worklist.
Issuer Type	Select the type of issuer of PID:
	None : DICOM receive and query/retrieve takes place as if no issuer was configured.
	Static: Select an entity from the list (see "Issuer" below).
	Dynamic : Automatically define the issuer based on retrieved results, and allow multiple issuer/alternate ID combinations.

Setting	Description			
Issuer	When you select an issuer type of Static, select the entity that issues the patient's PID, MRN, or account number. This makes available tracking and correct assignment of account numbers for sending and receiving studies when more than one are available.			
AE Flag	Select one of the following services to use to send studies. DICOM : C-MOVE DICOM SSL : C-MOVE, secured HL7 : Standard HL7 protocols. OPAL : The Opal Transfer service.			
Institution	Type the name of the institution for easier identification.			
File Store	Select a file store for storing received data.			
Port	Type the port number through which the AE can connect to Exa PACS/RIS (the Exa PACS/RIS listening port).			
Require SSL	Select this checkbox if: 1) you selected the DICOM SSL AE flag; and 2) the port number requires an SSL certificate.			
Transfer Syntax (Send)	Select the transfer syntax for sending.			
Transfer Syntax (Receive)	Select the transfer syntax for storing and receiving.			
Vehicle	Select vehicles to which to assign received studies.			
SDE Modality	Select if the AE is a dedicated echo ultrasound.			
Include Approved Reports	Select to include approved reports when sending as DICOM. Note: When DICOM-sending, the report status (such as Approved or Transcribed) is not included. To include the status, send via Opal.			

See also:

Configure a DICOM printer

Configure a DICOM printer

Configure and save AE with AE type of "Print," and then perform the steps below. For each printer you can save multiple configurations for different print media and other settings. At print time, you can select the configuration you need.

Procedure

- 1. Go to $\underline{SETUP} > \underline{OFFICE} > DICOM > AE TITLE$.
- 2. Configure a print AE, and then under **Extra Printer Options**, enter the following settings.

Extra	a Print	ter Opti	ons						
ARDM	Timeout					Max PDU Lan	gth		
Read To	Incore					Sorder Margins 2			
Write 7	Passar					True Size			
Printe	er Config	jurations						1	
								REFFESH	ADD CONTIGUTIATION
	New	Meilun	The life	Dordei Denstry	detication	otherbolize	Magnification	Empty Density	Memory Allacation
98	Peri	PAREN	SMKDIN	BLACK	MAGAZERE	PORTRACT	REFLICATE	BLACK	Aide

Setting	Description
ARTIM Timeout	Type the number of minutes to wait for an acceptance or rejection response to an association request before timing out.
Read Timeout	Type the number of minutes the printer waits to receive data before timing out.
Write Timeout	Type the number of minutes the printer waits to send data before timing out.
Max PDU Length	Type the maximum number of bytes of a protocol data unit that the printer can receive.
Border Margins	[Unused]
True Size	[Unused]

3. Select ADD CONFIGURATION.

Configuration		
Name	Film Destination	MAG4ZINE ¥
	Film Drientation	PORTRAT ~
Medium	Megnification	REPLICATE V
рарея 🗸	Empty Density	BLACK -
Film Size		1.0000000000000000000000000000000000000
6NX10N ¥		
Border Density BLACK 🐱		
Memory Allocation (HE)		
Auto (necession) deal)		SAVE GANCEL

4. Enter the following settings.

Setting	Description
Name	Type a name for the printer.
Medium	Select the printer medium.

Setting	Description
Film Size	Select the film size for the printer.
Border Density	Select a border density of Black or White.
Memory Allocation (KB)	For most printers, accept the default of Auto. Otherwise, type the amount of memory in kilobytes to allocate for a print session.
Film Destination	Select the destination for exposed film.
Film Orientation	Select the orientation of the film.
Magnification	Select a magnification factor for printing images.
Empty Density	Select the density of the image box area on the film that contains no image.

5. Select SAVE.

Configure a file store

File stores are directories on servers that Exa PACS/RIS uses to store files received in a DICOM transaction, such as images from a modality.

Caution: Changing an existing file store can prevent the study from opening.

Procedure

1. Go to $\underline{SETUP} > \underline{OFFICE} > DICOM > FILE STORE$.

	MCMM-		ې ۱
	APPENDE PLE STORE	antennes artempte entretens automotions	pasting memory a
			ADD REDAD
3	Wrate: +	works	POLT INVESTORY
e	BEWLETETORE	Unprocessed DrCOM studies	PAEKWUNINOWS DATIONS
1.4	Realtanne ettaget	Processed DCOM stotles	1 UKAN magee
1.3	Bearing & Doca	Summed Desymptotic In Patient Gase	PVD4Advanced Images

- 2. Select ADD.
- 3. Type the server name, root directory, and notes, and then select **SAVE.**

Configure routing rules

With routing rules, you can define how certain types of data flow between Exa PACS/RIS and other AEs. For example, you could create a routing rule that tells Exa PACS/RIS to automatically send all data that it receives from ultrasound modalities to another PACS. You can also add criteria to your rule for more precise control. For example, your rule could limit sending of ultrasound data from only a specific facility.

Use the following procedure to configure a routing rule, and then apply the rule by following the steps in "Configure study flow." It is not necessary to apply routing rules for manual DICOM send operations.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **DICOM** > **ROUTING RULES**.

	ticost							e
Щ. —	-titele	TACATHER	BOUTING RIALES	AC ADMITTO	ette offenulte	AND DESCRIPTION OF	NUMBER P	
							400	PELOND
	W.8.9 14100 2		er mie				atheiti,	
						*		
2.0	Bj41		£84,912,922			2		

2. Select ADD.

Rule Nama *	Example			🗆 lastine	
Type *	BEND	-	Filoray *	1	
AE THE *	EXAV33	~	Attempts +	2	
Number of Priors to					
	\$				

3. Enter the following settings. The available settings change depending on the AE type that you select in the Type dropdown list.

Setting	Туре	Description
Rule Name	All	Type a name for the rule.
Туре	All	Select the transaction to perform with the AE (for example, SEND).
Interface/Trigger Name	HL7 Send	Select an interface and/or trigger for HL7 send actions.

Setting	Туре	Description
AE Title	SEND	Select the AE with which to perform the transaction (for example, select
	PRINT	Remote_Hospital to send to that AE).
	QUERY/RETRIEVE	
	Send Report (Opal)	
	LOCAL_CACHE	
Priority	SEND	Type a number to indicate which rules to evaluate first. Lower numbers
	PRINT	take higher priority. Refer to your current list of rules on the ROUTING RULES tab to determine the priority to use for the rule you are adding.
	QUERY/RETRIEVE	
	Send Report (Opal)	
	HL7 Send	
	LOCAL_CACHE	
Attempts	SEND	Type the number of attempts the program should make to apply the rule.
	PRINT	Tule.
	QUERY/RETRIEVE	
	Send Report (Opal)	
	HL7 Send	
	LOCAL_CACHE	
Priors	QUERY/RETRIEVE	Type the number of priors to retrieve. Type 0 (zero) to retrieve instances for the current study only.
Modality	QUERY/RETRIEVE	Select an option to filter which priors to retrieve.
Prior Options		
Number of	SEND	Type the number of priors to send along with the study.
Priors to Send	LOCAL_CACHE	
Auto Send Priors		Select to send any priors found in the system along with the current study.
	LOCAL_CACHE	 The system sends as many priors as it can find, up to the Number of Priors to Send.
		• The system sends the most recent priors first.
Relevant Priors	SEND	Select to send relevant priors when auto-sending, along with any non-
	LOCAL_CACHE	relevant priors that the system found. Enter criteria to define what is relevant (see below).
		 If you do not define relevant priors (see "Define relevant priors" later in this topic), the system considers relevant priors to be studies of the same modality and/or body part.
		 The system sends as many relevant priors as it can find, up to the Numbers of Priors to Send.
		• If the system finds fewer relevant priors than the Number of Priors to Send , it adds non-relevant priors up to the number.
Force Relevant	SEND	Select to send only relevant priors.
Only	LOCAL_CACHE	

Setting	Туре	Description
		• If the system finds fewer relevant priors than the Number of Priors to Send , it does not send any additional priors.
[Define relevant priors]	SEND LOCAL_CACHE	Available when you select Relevant Priors . See "Define relevant priors" later in this topic.

- 4. Optional. Configure a criterion for the rule:
 - a. In the Field dropdown list, select an argument and a logical operator.

For example, *Modality* and *Is*

b. Select a matching value for the argument.

For example, in the *Modality* list, select *MG*.

Field			O is Not
Modeliny			0. 11100
Modality		BD BR CT MG MR	
ADD TO F	RILLES		

c. Select **ADD TO RULES.**

Result: In this example, when Exa PACS/RIS receives an MG study, it automatically sends it to the AE named Remote_Hospital.

- 6. Optional. Add more criteria.
- 7. Select SAVE.

Auto Query/Retrieve: use study's accession number

To facilitate Auto QR of studies from external sources such as the AHS, configure a query-retrieve routing rule with a placeholder to match the study's accession number rather than the study instance UID (SUID) in Exa PACS/RIS. This rule only applies if the Priors count is 0. Create the routing rule as follows:

- 1. In the **TYPE** dropdown list, select **QUERY/RETRIEVE**.
- 2. In the **Priors** box, type 0 (zero).
- 3. In the **Field** dropdown list, select **Accession** #, and select the **Is** option.
- 4. In the criterion box, type {{accession_no}}.

Prior settings on the left...

5. Select SAVE.

Define relevant priors

Defining relevant priors allows you to send only priors that are relevant to the current study based on study description, modality, or body part. To write definitions for which priors are relevant you use a syntax called *relevant prior logic*:

STRING or (STRING STRING ...) {.*(STRING STRING ...)}

"A single string, or multiple strings delimited with a bar, optionally followed by ".*" plus multiple strings delimited with a bar."

Enter your relevant prior logic in a table, such as in this example:

Prior Modality	Prior Budy Part	Prior Description	Main Study Modelity	Main Study Body Part	Main Study Description		
C1			MR			1	
	(INEEPATELLA)			KNUEE		1	
		ORCELICASED FILEFTEDDD			OVERLIGALCO POLEFTILTERO	1	

...main study settings on the right

<u>Row 1</u>: Does not require relevant prior logic. When the main study is an **MR**, relevant priors are **CT**s.

Row 2: When the main study is of the knee, relevant priors are either of the knee or patella.

<u>Row 3</u>: When the main study description contains "**HEEL**" or "**CALC**" followed by "**LEFT**," or "**LT**," or "**BI**," relevant priors are those whose descriptions also contain "**HEEL**" or "**CALC**" followed by "**LEFT**," or "**LT**," or "**BI**."

Note: You can also define relevant priors that you want to auto-open in the viewer (see <u>Configure modality-specific viewing options</u>).

Configure AE scripts

You can write scripts that modify DICOM tags when carrying out a DICOM-related transaction such as sending, printing, or receiving.

Procedure

1. Go to $\underline{SETUP} > \underline{OFFICE} > DICOM > AE SCRIPTS$.

			 - 40	011141 St. 8 10	Анхоних	ANCOUNT MARINE	MATCHIC SALES.	no interesti	ہ د سست
				(Party)	Lermatanicad			100	101000
	white	RIE ARTHON		05.11036		Sofat	da an		
		AB		14		. VIII			
而有	Ť	8£HD		EN4-932-932		Stripts			

2. Select ADD.

Script Description	Script Description	АЕ Туре *	Select	~ 🗆	Inactive
Asynchronous	Asynchronous	AE Title *	Select 🛩		
		Script Description			
	Script - L	Contrate 2		onous	

3. Enter the following settings.

Setting	Description
АЕ Туре	Select a transaction and node. The script runs when Exa makes the
AE Title	selected transaction with the selected node.
All	Select all AEs of the selected AE Type.
Script Description	Type a description of the script.
Asynchronous	Select to run the script asynchronously.

4. In the **Script** area, type the script to run when the previous conditions are true.

Note: For syntax, select the help ? button.

- 5. Select **COMPILE**, and correct any errors that occur.
- 6. When the script is free of errors, select **SAVE**.

Understanding receiver rules

A *receiver rule* is a detailed set of criteria and instructions that tells Exa PACS/RIS where to place studies that it receives, and what status to assign them. For example, you can configure a receiver rule such that studies with unrecognized account numbers are placed on the QC tab to await reconciliation by the user before going live.

Note: Receiver rules also have an important role in accelerating the processing of incoming data. Exa PACS/RIS processes each receiver rule in order on a particular computer. Therefore, to improve speed, you can apply different rules on different computers.

To understand how receiver rules are evaluated and applied, it is helpful to imagine a 4-step process that Exa PACS/RIS performs for each incoming study.

- Find a receiver rule that matches the study.
- Validate the study against existing PACS studies and/or match it to existing RIS orders.
- Process the successes (studies that passed validation and/or matched orders).
- Process the failures (studies that failed validation or did not match an order).

A detailed explanation of each of these steps follows.

Step 1 - Find a matching receiver rule

When Exa PACS/RIS receives a DICOM study, it evaluates the study against each receiver rule on the RECEIVER RULES tab in order of "Priority" (starting from the top) until it finds one that matches.

			PRIORITY :	RULE NAME	1	0	t	4	12	Kforwarder7
2	(1)	+	4	PMT2	- 2	jįtį	+	÷	13	Kforwarder8
2	1		2	PMT	2		+		14	MMD Liv e
2			3	File Import	2				15	DEFAULT BULE

From top...

...to bottom

If no user-defined rule matches, Exa PACS/RIS uses a "default" rule that has a minimum number of criteria to ensure that all legitimate studies match at least one rule. (If a study did not match any rule, Exa PACS/RIS would not receive it.)

To evaluate whether a study matches a rule, Exa PACS/RIS compares the settings you enter under **Rule Info and Filter** (in the green box in the following figure) to the information in the study. If all relevant criteria match, the rule becomes active, and the process continues to step 2.

Rule info and Filter

Rule Name *			🗆 Inac
AE Title	All	x	
Facility	All		•
Modality	All 🔹		
Institution			
Status	Urvead		
Interval (Seo) *	120		

Step 2 - Validate and match against PACS and/or RIS

A "rule" is actually not a single rule, but rather a combination of the previous "Rule Info and Filter," plus one of four possible preset "Applied Rules," such as in the following figure.

Ap	plied Rules
	Reconciliation Mode: Pacs only site
	Emit Live Update
٠	Precache Study
	Precache Priors
	No. of priors: 3
	Pick study description from DICOM
٠	Patient
~	Account #/Dicom Patient ID
	Last name
	Date of Birth
	· Move Anyway (Even in exception)
	· Move to live when there is no-match
٠	Study
	Study UID
_	· Move to live when there is no-match

Figure: Applied rules in the PACS ONLY preset.

Exa PACS/RIS compares the items in the green boxes from the incoming study against PACS, RIS, or both, depending on the reconciliation mode (the first bullet item in the figure). The items under Patient (such as "Last name" in the figure) and Study ("Study UID") must match a patient in PACS and/or a RIS order. Exa PACS/RIS stores the result of the match (success or failure) in this step, and then evaluation continues to step 3 or 4.

Step 3 – Process successes

If matching succeeded, Exa PACS/RIS waits the number of seconds specified in the **Interval** setting from step 1, moves the study from the QC tab to the ALL STUDIES tab (the study "goes live"), and assigns to the study the status selected in the **Status** dropdown list from step 1. At the same time, it performs the actions and applies the options in the blue box in the following figure.

Applied Rules

- Reconciliation Mode: Pacs only site
 Emit Live Update
 Precache Study
 Precache Priors
 No. of. priors: 3
 Pick study description from DICOM
 Patient

 Account #/Dicom Patient ID
 Last name
 Date of Birth
 Move Anyway (Even in exception)
 Move to live when there is no-match
 - · Study UID
 - · Move to live when there is no-match

Step 4 – Process failures

If matching failed, the study initially remains on the QC tab with a status of *Conflicts* to await reconciliation by the user. However, there is one more set of evaluations left to make, based on the items in the orange boxes in the following figure.

Applied Rules

- · Reconciliation Mode: Pacs only site
- Emit Live Update
- · Precache Study
- Precache Priors
- · No. of, priors: 3
- · Pick study description from DICOM
- · Patient
 - Account #/Dicom Patient ID
 - Last name
 - Date of Birth
- Move Anyway (Even in exception)
- Move to live when there is no-match
- Study
 - Study UID
 - Move to live when there is no-match

If any of these items are present, the study status is set to Completed in the QC tab. Additionally, each item results in the following.

Move Anyway (Even in exception)

If duplicate matching demographics are found, or if at least one matching criterion fails, the study also appears on the ALL STUDIES tab with a status of "FIX_."

Patient: Move to live when there is no-match

The study moves to the ALL STUDIES tab even if patient demographic validation fails. Generally, this is used only when no modality worklist is available.

Study: Move to live when there is no-match

The study moves to the ALL STUDIES tab, even if no matching order is found.



Note: If "Move to live when there is no-match" is present both under Patient and Study, Exa PACS/RIS processes the study as a success.

See also:

Configure a receiver rule

Configure a receiver rule

You can configure receiver rules to tell Exa PACS/RIS where to place studies that it receives, and what status to assign to them. For more information, see <u>Understanding receiver rules</u>.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **DICOM** > **RECEIVER RULES**.

	346	ame II	neuriote 👘	năumia	i ku in		RICEMER	AULUS	Kistanika enellä	HANDRIGH	-inclusive
										A00	RELEAD
	PRICE	RILENAME	ACTIVITE		EAGL/TV	3	VILLAGE		arminen -	CTATES	
			Eat.	14	ais"	T411	AR.			(i 44	
78	1	INFORT	3441470	et .	48		4			Unnell	
1	2	attene star	× 14		.25	3	a -			Linnati	

2. Select ADD.

Rule Informatio	n and Filter	t;)		Applied Rules
Rule Name * AE Trile Facility Modulity Institution	Rulet 28 All • 28 •	•	azb-s	 Reconciliation Mode: PACS Only Site Emit Live Update Precache Study Precache Priors No. of Priors: 3 Select Study description from DICOM
Stotum Interval (Saz.) =	Unread 120	4		 Patient Account No./DICOM Patient ID Last Name Date of Birth
Trigger Routing Rules Isouer Of Altomoto Account Number Reserver Pula	0			 Move Anyway (Even in Exception) Move to Live on No-Match Study Study UID
PACE ONLY	RAGE/TE:	PAGE/DMML	MIGRATION	 Move to Live on No-Match

Example showing the PACS ONLY rules

3. Under **Rule Information and Filter**, enter the following settings as needed.

Setting	Description
Rule Name	Type a name for the rule.
AE Title	Use these settings as filtering criteria. For example, to evaluate the rule
Facility	only against mammography studies, on the Modality dropdown list, select MG. To evaluate the rule only if the study comes from a specific
Modality	AE, select one under AE Title.
Institution	
Status	Select the status to assign the study if it matches the rule.
Interval	Select a number of seconds to wait before moving the study from the QC tab to the ALL STUDIES tab (go live).

Setting	Description
Trigger Routing Rules	Select to force evaluation of matching studies against routing rules.
Issuer of Alternate Account Number	Select to make alternate account numbers available. See <u>Add alternate</u> account numbers to a patient chart.

4. Select one of the following preset buttons to assign a reconciliation mode and to add "applied rules" (matching criteria and actions to perform) to your receiver rule.

Preset Button	Reconciliation Mode	Description
PACS ONLY	PACS	Validate patient and study items against existing PACS records.
PACS/RIS	PACS + RIS	Validate patient and study items against existing PACS records and/or RIS orders.
PACS/DMWL	PACS + RIS	same
MIGRATION	PACS	Use when migrating data from another PACS. Allows everything to be received.

5. Select SAVE.

See also:

Understanding receiver rules

Configure matching rules

See <u>Work with Display Management</u>.

Configure hanging protocols

See Work with Display Management.

Manage jobs in the transfer and SR queues

DICOM studies that you send appear as jobs in the transfer queue. You can view and change job status and priority, cancel jobs, and requeue failed jobs.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **DICOM** > **TRANSFER QUEUE** or **SR QUEUE**.

🖌 🗤 selilets	S Million		autows-auto-		Harrickes.	CRAS	FER QUEUE	nia meritan	C LA BURN	-
				24	Provide States	- 60	INTERPROCESS F4	i ku	A1110201	40.00
CONTRACT	nonici inici	exer pive	LAST UPSAT		staters	TAT	Addest	dute:	PACIENT NAME	AND .

2. Perform any of the following tasks.

View job status

In the list of jobs, the status is listed in the CURRENT STATUS column. The available statuses are as follows.

Queued	Processing not yet started.
In-progress	Sending of data has started but not finished.
Completed	Sending of all data is finished.
Canceled	Sending of data canceled by the user. You can only cancel a job with a status of Queued.
Error	Sending was aborted due to failed association or the request was rejected by the destination.

View demographics

To view the demographics of a study in the transfer queue, select the expand + button.

Change the job priority

To change the priority of a job, select the 🥙 edit button, select a new priority, and then select **SAVE**.

View job details

To view demographics and other queue details, select the view \square button.

Cancel a job

To cancel a job, select the cancel [@] button.

Requeue a job

To *requeue* a job means to repeat or retry sending. If the status of a job is Queued, Error, Completed, or Cancelled, you can requeue the job by selecting the requeue \checkmark button. Requeued jobs remain in the transfer queue for 15 minutes before reprocessing begins.

Reprocess non-failed jobs

Rather than requeuing jobs one at a time, you can select multiple jobs for reprocessing, and then select REPROCESS SELECTED.

Reprocess failed jobs

Failed jobs are ones with a status of "Error." You can select a range of failed jobs (such as by filtering the worklist) and reprocess them all at once.

- 1. On the transfer queue, select **REPROCESS FAILED**.
- 2. In the date boxes, enter a date range of jobs to reprocess.



Note: To select all dates, leave the boxes blank.

- 3. In the AE Title dropdown list, select the AE whose jobs you want to reprocess.
- 4. Select **REPROCESS**.

Reprint DICOM Print jobs

- 1. On the transfer queue, find a job with type **PRINT** and status **Completed**.
- 2. Right-click the job, and select **REPRINT**.

Cancel a DICOM Print job

- 1. On the transfer queue, find a job with type **PRINT** and status **Queued**.
- 2. Right-click the job, and select Cancel.

Configure an issuer of a PID

See <u>Configure alternate account number functionality</u>.

Automatically retrieve priors from an external PACS

When a patient is referred from another facility, it can be efficient to automatically retrieve priors from the referrer's PACS that are not already present in Exa PACS/RIS, and to control the timing of that retrieval. That is, you can configure Exa PACS/RIS to retrieve priors when it becomes likely that the exam will take place (such as when confirmed by the patient), but not so close to exam time (such as upon check-in) that waiting to fetch priors could delay reading. To set up automatic retrieval, follow this example in which we configure Exa PACS/RIS to retrieve priors when the study changes from Scheduled to Confirmed status.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **DICOM** > **AE TITLE**.

			ACTORNIS	HARDING MULTIN	MARCHINE R. LES	strations, and t	ور م
🖬 Hida Inagtive.						410 8	HLOND V
TITLES	HOCT	PERT	DESURPTION		370/E	ALTING	
						42	
n niversitä (10.227 25.78	3444.85	EU-102-02	£13	Severatingen	SEMIO/RECEIVED	OURNARTS
CLD.AND IVE			OLO ARDI-SVE		NULT 07002	PECCIVE.	
2 1 OPALISTAN	10.227.24.300	88836	0944_315_58		Загин тгодог	SEND/RECEIVE)	

2. Select **ADD**, and configure a query/retrieve AE.

3. Go to <u>SETUP</u> > <u>OFFICE</u> > **DICOM** > **ROUTING RULES**.

4. Configure a routing rule with the following settings.

Setting	Description
Rule Name	Q/R Old PACS
Туре	QUERY/RETRIEVE
AE Title	Select the query/retrieve AE you configured earlier.
Priority	1
Attempts	10
Priors	10
Modality	Modality of Current Study
Summary - Field	Patient Identifier
Account Number	Select to query the external PACS by the patient's account number, alternate account number, or alternate account number via issuer.
Issuer	Select the issuer
Name	Select Yes to include the patient's name in the query.
DOB	Select Yes to include the patient's date of birth in the query (in YYYY-MM-DD formatting).

5. Configure a study flow from Scheduled to Confirmed status using the previous routing rule.

See also:

Configure application entities Configure routing rules Configure study flow

Configure billing

Exa Billing uses patient and insurance information that you entered in other Exa PACS/RIS modules, but you must also enter other information directly into Exa Billing such as codes, statuses, templates, providers, and validation rules.

This section contains the following topics (not all topics may be available depending on version and region).

Configure columns on the Studies and Claims screens Add codes and other basic data Configure the billing collections process Configure claim validation Configure a billing provider Create or edit an EDI or ERA template Configure an EDI clearinghouse Add billing messages Add an insurance provider About active and inactive insurance Import an insurance provider Add an insurance provider payer type Add a provider level code Insurance mapping Create a printer template Select default printer templates

Configure columns on the Studies and Claims screens

You can show, hide, and reorder columns on the Studies and Claims screens. You can also configure how the studies and claims lists are sorted by selecting default columns and their default sort orders.

Procedure

1. Go to <u>BILLING</u> > settings

Jolumn Order	Settings	\$199
C Storty Date	Default Column	¥
I Patient Nerro	Defeul: Surt	
C Account No.	Order	
Date Of Sirth	Printer Temp	lates
13 SSN	Paper Claim	Saleot w
D Place Of Service	(98W)	
Referring Providence	Paper Claim	Select -
C Rendering Providers	(880)	
Billing Fee	Direct involce	Select v
III Payer Type	Patient trouce	Select 🛩
C Clearing House	Special Form	Baleet 🔗
🗆 Balance		
- million classe	1	

- 2. Under **Column Order**, do one or more of the following.
 - To show or hide a column, select or clear its checkbox.
 - To reorder a column, drag it to a new position in the list.
- 3. Under **Settings**, select a default column and its default sort order.
- 4. Select **SAVE**.

Add codes and other basic data

Use this procedure to add or edit: adjustment codes, billing codes, billing classes, claim statuses, billing providers, provider ID code qualifiers, billing messages, payment reasons, CAS group codes, CAS reason codes, and status color codes.

Procedure

- 1. Go to <u>BILLING</u> > <u>Setup</u> > **Adjustment Codes**.
- 2. In the left pane, select the type of data that you want to add.
- 3. In the upper-right side of the page, select **ADD**.

SETUP		
ADJUSTMENT CODES	Code +	Inective
BILLING CLASSES	Entry * select Type	
CLAIM STATUS		
COLLECTIONS PROCESS		
HER AV BEARING		

4. Enter the information in the dialog, and then select **SAVE**.

See also:

Configure claim validation Create or edit an EDI or ERA template Configure an EDI clearinghouse Insurance mapping Create a printer template

Configure the billing collections process

You can configure Exa Billing to automatically assign collection review status to claims meeting criteria that you enter. You can also automatically write off the balances of claims in collections.

Procedure

1. Go to <u>BILLING</u> > <u>Setup</u> > **Collections Process**.

ADJUSTMENT CODES BILLING CODES BILLING CLASSES CLAIM STATUS	Process Collections Automatically Minimum Account Balance Collections Review Criteria Multi select at least one option
PROCESS	Change claim status to Collections Review if no payment is applied to the
DELAY REASONS	patients account days after statements are sent
BILLING PROVIDERS	Change claim status to Collections Review if no payment is applied to the
PROVIDER ID CODE QUALIFIERS	patient account within days of the last patient payment
BILLING MESSAGES	When not selected claim belances will remain
PAYMENT REASONS	Write off the balance when there is a claim status in Claim in Collections
CAS GROUP CODES	
CAS REASON CODES	

2. Enter the following settings.

Setting	Description
Process Collections Automatically	Select to turn on automatic collections processing.
Minimum Account Balance	Type a dollar amount. If the claim balance is lower than the amount you type, it will not be processed for collections.
Collections Review Criteria	Select one or more criteria, and then type the values in the boxes.
Write off the balance when there is a claim status in Claim in Collections	Select to automatically write off the balance of claims that are in the Claim in Collections status.
Debit/Credit adjustment Code	When writing off balances, select a debit or credit adjustment code to use for the write-off.

3. Select SAVE.


Configure claim validation

Exa Billing validates claims by confirming that the information satisfies the requirements of various public and private health agencies (if applicable), and by checking whether information was entered in required options. You can select which options are required.

Procedure

1. Go to <u>BILLING</u> > <u>Setup</u> > **Billing Validations**.

2. On the top of the billing validations page, select a billing method of **Electronic**, **Invoice**, or **Patient**.

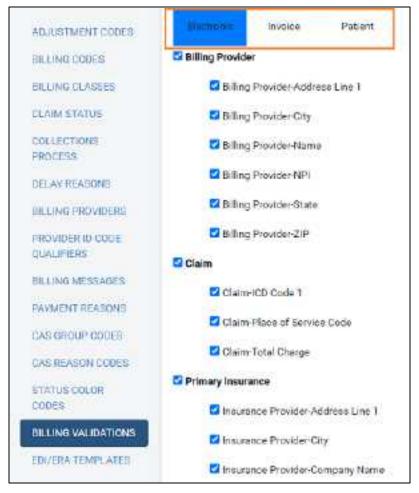

- 3. In the list of options, select the ones that you want to be required.
- 4. Select **SAVE**.

Configure a billing provider

A *billing provider* is the provider who issues invoices for an exam. In order to create claims, you must configure at least one billing provider.

Procedure

1. Go to <u>BILLING</u> > <u>Setup</u> > **Billing Providers**, and then select **ADD**.

ADJUSTMENT CODES BILLING CODES BILLING CLASSES CLAIM STATUS	Nome * Code * Short * Description Federal * Tax ID			[] inacto	 NPI No. * Taxonomy * Code	1	
COLLECTIONS PROCESS DELAY REASONS	Address Info	Č.			Pay To Addr	ess	
BILLING FROMDERS	Context * Marte				Address1 Address2	1	
QUALIFIERS BILLING MESSAGED	Address7 *				City/State/ZP	IT STATE	\$
PAYMENT READOND	City/State/ZIF*		s	• 20₽	Phone	29	
CAS REPSON CODES	Phone *	ZIP			Fax		
STATUS COLOR CODES	Fax 7				Emad		
EDUERS VALIDATIONS	Email Web.URL						
EDI							


2. Enter information as needed, and then select SAVE.

Create or edit an EDI or ERA template

You can create or edit an EDI template to format electronic claims submissions. You can also edit the ERA template.

Procedure

- 1. Go to <u>BILLING</u> > <u>Setup</u> > **EDI/ERA Templates**.
- 2. On the top of the page, select **EDI Template**, and then select **ADD**.
- 3. In the **Template Creation** dialog, type a name for the template and then select **SAVE TEMPLATE**.

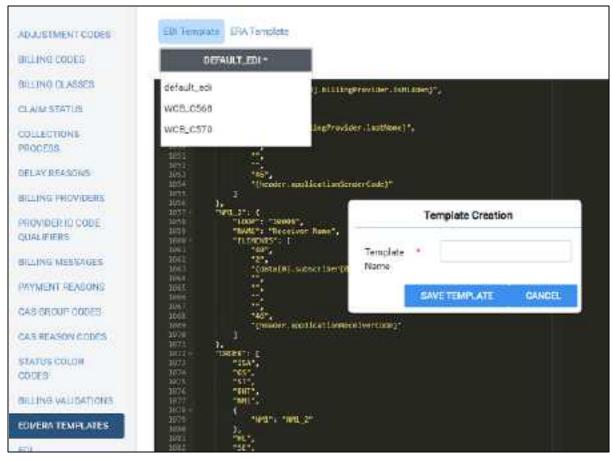

- 4. Below the **EDI Template** button, in the template button menu, select your template.
- 5. In the template editing area, edit the template.
- 6. To edit the ERA template, select the **ERA Template** button and then edit the template as needed.
- 7. Select SAVE.

Configure an EDI clearinghouse

To process transactions through a clearinghouse, you must configure one as follows. If you transact with the Alberta, Canada healthcare system, add the Alberta WCB as a clearinghouse.

Prerequisite: Obtain credentials and other connection information from the clearinghouse.

Procedure

1. Go to <u>BILLING</u> > <u>Setup</u> > **EDI CLEARINGHOUSES** > **ADD**.

ADJRIGTIMENT CODES BILLING CODES BILLING CLASSED	Description * Dode *	🛄 kastivę	Receiver * Name Receiver III *
CLAIM STATUS	Cleaning House Control Header		
EDULECTIONS PROCESS DELW FEASOND	Auto Info * Decarity, Info *		Repetition Separator(SC10) Element/Sub-
BILLING PROVIDERS	Gender ID *		Hemeni Gelmiter
PROVIDER ID CODE QUALIFIERS	inter Crif Stand E		Segment Torminator DReposited?
BILLING KESSAGES	intex Cull Version Norther		Usage * O Production O Testing
PAYMENT REASOND CASI DROLIN CODER			EDI File 1 Extension
CAS REASON CODES			CRATIle * Estension
ETATUS COLOR CODES	Functional Control Header		
BILLING VALIDATIONS	Application		Application *
EDUFRATEMPLATES	Bernher Code		Pecetary Gode
EDI GLEARINGHOUSES	Regionable * Agency Cade		Vet Rel.tsd.ID Code
INSULVINCE MUST ING	Transaction Set Header		Cleaning House info

2. Enter all relevant information for the clearinghouse and select SAVE.

Add billing messages

You can associate customized messages with accounts receivable codes. Billing messages print on the patient statement based on the age of the oldest claim included in the activity.

Procedure

- 1. Go to <u>BILLING</u> > <u>Setup</u> > **Billing Messages**.
- 2. To find a code, enter a criterion in the **CODE** box and then select **Enter**.

ADJUSTMENT CODED	CODE
DELING CODES	
THE LINE CLASSES	∂ infectors
CLAIM STATUS	Ø ==120
COLLECTIONS	p* 105-120
UELKY REASONS	Ø 61-90
BLLING PROVIDERS	J 35-60
HIGVIDERID CODE	Ø 030
HIDVIDERED CODE GUALIFERD	P 040
PAYMENT REASONS	

3. In the list of codes, double-click a row.

Cittle 1	01-228
Decemption *	A long time to wait for payment

4. Enter a description, and then select SAVE.

Add an insurance provider

Before you can bill an insurance provider, you must set the provider up in Exa PACS/RIS. Note that *insurance provider* in this context means any institution that can act as a payer, including insurance companies, financial firms, and attorneys.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Billing** > **INSURANCE PROVIDER**.

	INSURANCE PROVIDER	INSURANCE PREVAILING PR	Constitution and a second		(11)5364P			
				100	GELOAD	INDERF	HB	eP.
00064	CODE: NAVE	PAYER ID PROVIDENTYPE	CONTACT INFORMATION	TABLITY		WARKET		ж
				ALC: Y		48		÷
(\$ioon)	NETRO HEALTH PLAN		PE-803 25518 juile 1822/ID-6	CND VITRA FAC	tily Lange Field	Indexed Apple		*
12.1A	RANF HERSTH REVENUE		PERIO 2498 NAME TRATIA		Ity LANGE Facil	tyte totarkar 1		
15434	APPINITY MEDICARE ADD		PO BOX 4018 SCRANTON	winks fac	Ity Large Feel	tyh Market 1		*

2. Select ADD.

Code	29	🖾 inclus	Market	INVESTIGATION -	
Hourierce Norse *	AABP HEALTH ADVANTA	Riquine AUC	Pacifices *	ALL SELECTED CO +	
Payer (D *	87726		Pulsikier Type	Select	
Address Line 7 +	PO BEX 746819	1	SOP Paver	Select	*
Address Line 2			Fee Schedule	Select +	
City/State/2P	ATLANTA	94 ¥	Altoned See Schuckle	Selant w	
	38574 million		fundamentary for	Concernation of the second	
Phone No.	1803(227)3789		Toding Parties ID	and unstated and state	
Parchio			Abamaletmagne		F .
Additional Phone Nam	nbere		PsesiderID		
Reference Note	Dhose No.		Website		
HUNDER MORE	Phote No.		Prourance Drowcker Alerta		

3. Enter the following information for the provider. Depending on customer options, some fields are not available.

Setting	Description		
Code	Type the insurance provider's plan ID.		
Inactive	Select this checkbox to:		
	1. Make the insurance provider unavailable in other parts of the program.		
	Make all associated patient insurance profiles unavailable in other parts of the program.		
	3. Set the insurance profile Valid To date to yesterday's date.		
Insurance Name	Type the name of the provider.		
Require AUC	Select if the provider requires the physician to satisfy AUC before submitting orders for Medicare. Selecting this checkbox will automatically open the CDSM application when creating the order in the physician's portal.		
Payer ID	Type the insurance provider's ID (or ID based on information from your clearinghouse).		
	Paper claims: 99999		
	Attorneys: 88888		
[Address]	Type the address of the provider for sending claims.		
Phone/Fax No.	Type the primary phone and fax numbers of the provider.		
Additional Phone Numbers	Type additional phone numbers, if available.		
Market	Select the market that the insurance provider serves. All facilities associated with the selected market become selected in the Facilities dropdown list.		

Setting	Description		
Facilities	Select the facilities that the insurance provider serves.		
Provider Type	Select a provider (insurance) type.		
SOP Payer	Select the standardized source of payment code for the provider.		
Fee Schedule	Select the default billing fee schedule used for the provider.		
Allowed Fee Schedule	Select the allowed fee schedule for the provider.		
Alternate Imagine Provider ID	Type an alternate provider ID for use by ImagineSoftware for Exa Clear.		
	Note : Only use if an alternate NPI is required by an insurance company. Leave blank to use the provider ID configured for the facility.		
Trading Partner ID	Type the ID assigned to the clearinghouse, or the PokitDok or ImagineSoftware trading partner ID. For PokitDok, you can find the trading partner ID at: <u>https://dashboard.pokitdok.com/#/trading- partners?code=270&name=Eligibility</u>		
Website	Type the URL to the provider's website, if available.		
Insurance Provider Alerts	Type alerts as needed.		

4. Select SAVE.

See also:

Configure a fee schedule About CDSM and AUC Add an insurance provider payer type Verify eligibility of preorders and studies with PokitDok Verify eligibility and estimate coverage with Exa Clear

About active and inactive insurance

Insurance providers can be inactivated, but individual patient insurance policies from those providers can also be inactivated from the patient chart. When adding insurance to an order, the following describes the relationship between the two.

Insurance	Order Status	Policy Valid To Date	Policy Status
Provider			
Active	Unscheduled	< Current date	Inactive and unavailable.
		≥ Current date	Active and available.
		None	Active and available
	Scheduled	< Scheduled date	Inactive and unavailable.
		≥ Scheduled date	Active and available.
		None	Active and available
Inactive	Scheduled or Unscheduled	Any	Inactive and unavailable.

* *Unavailable* means that it cannot be added to the order, or selected anywhere in Exa PACS/RIS. If the policy was previously added to a patient chart, it is still be viewable but cannot be edited or used for billing or other purposes.

See also:

Add an insurance provider

Import an insurance provider

If you obtain exported insurance provider files, you can import them.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Billing** > **INSURANCE PROVIDER**.

1 101.4								
	INSURANCE PROVIDER	Insurance encoders es			missee			
				100	GECOMD	MINORF	HB	ыр
CODE: HWIVE	NAVE	PAVER ID PROVIDENTYPE	CONTACT INFORMATION	TABLITY		MARKET.		ж,
				302.5		48.		4
(\$100m)	SETING HEALTH FLAN	Settin 100	PE-00X35516,6454-102200-0	ACIND VITURE Fect	lily Lange Fect	tyh Marios I		*
Eza.	SARE HEALTH ACTION OF	08.87228	-10.803 740819,5084101,410	ANTA SHIRR FACT	Ity Lange Facili	tyn narkar t		
18434	APPINITY MEDICARE ADV	AU 13383	PO BEX 4018.3CRANTON	winke Faci	Ity LAnge Fect	trh Market T		*

2. Select IMPORT.



- 3. Select Choose File, browse for and select the file to import, and select Open.
- 4. Optional. To replace existing facilities, select the **Update Existing** checkbox.

Add an insurance provider payer type

You can preconfigure insurance provider payer types for selection in other parts of the program.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Billing** > **INSURANCE PROVIDER PAYER TYPES**.

		¢					
		наличног римпири	RISLINANCE PROVIDER PAYOR TYPES	MONDAL PLANT COLL	Inserver.		
		Ţ				(abb \	121040
		COREC	DESCHI	F00			
	-		No. 1070				
Ľ.,	8	Δ₽		unice Policy			
2	8	Ċ1	Gamme	sia .			
1	8	CP	Madaze	Conditionally Primary			

- 2. Select ADD.
- 3. Enter your internal code and description, and then select SAVE.

Add a provider level code

You can preconfigure provider level codes to associate with Procedure Codes.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Billing** > **PROVIDER LEVEL CODE**.

neidettice (normitie)	INSUBANCE PROVIDER DAYESTYPES	PRONDER LEVEL CODE	Pauser		
				(A00)	RELAA
CODES	CESCRIPTION				

- 2. Select ADD.
- 3. Enter your internal code, description, and level, and then select SAVE.

Insurance mapping

Insurance mapping means to assign a billing method (such as direct, paper, or electronic), clearinghouse (for electronic billing), and other settings to an insurance provider or other payer.



Procedure

1. Go to <u>BILLING</u> > <u>Setup</u> > **Insurance Mapping**.

2. Double-click an insurance provider in the list.

CAS GROUP CODES	insurance / Name	AETNA SENIOR SUPPLEMENTAL
CAS REASON CODES	Insurance	ATSENIOR
STATUS COLOR CODES	Code Billing Method *	Direct Billing 🗸
BILLING VALIDATIONS	EDI Code	v
EDIJERA TEMPLATES	Claim File	
EDI CLEARINGHOUSES	Indicator Code	France
INSURANCE MAPPINE		file for Paper Claim Form
QUALIFIERS	Participating Provide	r Addreve


- 3. In the **Billing Method** dropdown list, select an item, and then enter information as needed in the other options.
- 4. Select **SAVE.**

Create a printer template

Exa Billing comes with templates for standard billing forms, but you can also create custom templates.

Procedure

- 1. Go to <u>BILLING</u> > <u>Setup</u> > **Printer Templates**.
- 2. At the upper-right, select **ADD**.

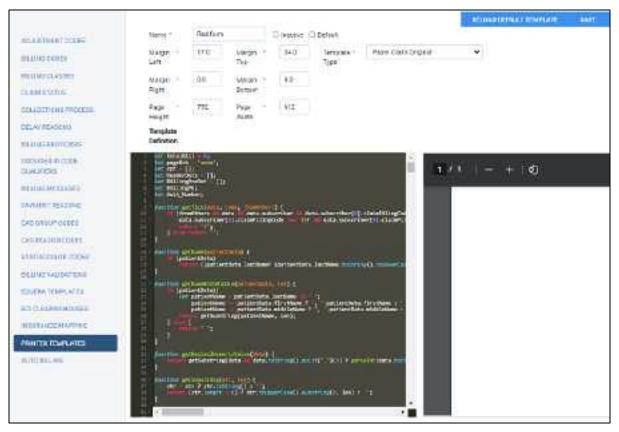

- 3. Optional: To base your template on the default, select **RELOAD DEFAULT TEMPLATE**.
- 4. Enter the name, template type, and margins.
- 5. In the **Template Definition** area, type the JSON code for the template.

A preview of your template appears in the right pane.

6. Select **SAVE.**

Select default printer templates

You can select which templates to use by default for printing out standardized forms and customized invoices. Exa Billing comes with several templates, and you can also create your own.

Procedure

1. Go to <u>BILLING</u> > settings

There is a survey of		7	
Column Order	Settings	4	SAME:
C Study Date	Default Column	×	
23 Pollert Marro	Defeul: Surt		
Account No.	Order		
🗆 Date Of Sirth	Printer Temp	lates	
13 SSN	Papar Claim	Silico w	
D Place Of Service	(56W)		
Referring Providers	Paper Claim	Select 👻	
D Rendering Providers	(980)		
Billing Fee	Direct involoe	Salect 💌	
II Payer Type	Patient troops	Select 👻	
C Clearing House	Special Form	Beleet 🔗	
🗆 Balanzo			
- matrix return	2		

2. Under **Printer Templates**, make the following selections.

Paper Claim (B&W) – The standard "black and white" claims form.

Paper Claim (RED) – The standard "red" claim form.

Direct Invoice – Select a template for billing insurance companies.

Patient Invoice – Select a template for billing patients.

Special Form - Select to use special-case forms, such as for claims in New Jersey.

3. Select SAVE.

See also:

Create a printer template

Manage users

Administrators can configure individual users, user groups, Active Directory, and assign roles (rights) to users. In Exa PACS/RIS, you assign user roles to user groups, and then you assign user groups to individual users.



Caution: Incorrect user configuration can lead to security risks such as leaked patient information.

This section contains the following topics (not all topics may be available depending on version and region).

<u>Create a user role</u> <u>Create a user group</u> <u>Create a user</u> <u>Reset a user's password as an administrator</u> <u>Configure an administered worklist filter</u> <u>Manage who is online</u> <u>View assigned studies and patients</u> <u>Copy settings from one user to another</u>

Create a user role

A *user role* is a named collection of rights and permissions that you can assign to user groups. To create a user role, complete the following steps.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > User Management > USER ROLES.

								9
	15005	LINER HERES	Installations	-4016423195	ALLERSON	STATISTICS. INC.	and the second second	
	-							ADD RELIND
	BRIDESE:				PELE DES	2127.000	1.00	
_								
.× 0	élleu.				\$tilling.			
10	Foring Lavon				Facility data			
10	from geals				fortikk			

2. Select **ADD**, type a name and description of the role, and the select **SAVE**.

For example, type BILLING, and then select **SAVE**.

3. In the **User Role Permission** area, select all of the rights that you want to assign to the user role.

Hint: To find rights, type Ctrl + F.

Role Name *	Billing	Inactive	
Role Description *	Billing		
User Role Per	mission		
Billing/Setup			Order ICD
E1 E	Silling Codes		Order Referring Provider
E E	Billing Class		Order Studies
E c	Claim Status		QC Delete
E (CAS Group Code		OC Edil
E F	Provider ID Code Qualifier		QC Move to Studies

4. Select **SAVE & CLOSE**.

See also:

<u>User rights</u> <u>Create a user group</u> <u>Create a user</u>

Create a user group

A *user group* is a named collection of individual users to which you can assign roles and access to certain document types and areas of the program.

\bigotimes

Prerequisite: Create a user role.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > User Management > USER GROUPS.

	CONTRACTOR AND A CONTRACTOR OF A CONTRACTOR AND A CONTRAC									
	Meters.	(INPRIMALES	INTRACTO		ABBISTR	MSHRS UNLINF	ASSAULTED AT LOUIS	ASYMMET >		
							(ADD ;;	HILONI		
	- From Tractive									
	660.04 1213	33	0078344		940.F B	ACREOUX:				
2.0	51.	8	ц.Рю		BLUND.					
10	648	Fa	nity Kome		Factivy 5.8	inimetroty (
J 8	PROT	ň	nteletin		Fort desk	personnel				

2. Select ADD.

Greup Code *	BIL.	Inactive	Al Documents				
Group Name *	BILLING		Document Types *				
Group Description *	BILLING		🖾 Assurance Card				
			📑 Reparts				
Roles			Navigation				
🛃 Billing			🖸 Dashboard				
E Facility Admin			Worklist:				
🔲 Front-desk			Parient Antival Workfur				
D Portal			E Patient				
C RAD			Ex Manager				
SEF			Schedule				
🗆 тесн			S Sfing				
TRANS			E Payments				

3. Enter the following settings.

Setting	Description		
Group Code	Type your internal code for the group. For example, RAD.		
	Do not edit the names of built-in group codes as they are tied to other functionality.		
Group Name	Type your internal name for the group. For example, Radiologist.		
Group Description	Type a description for the group.		
Document Types	Select the document types that group members can access.		
	All Documents: Selects all current and future documents.		
Roles	Select the user role that you want the group members to have. Choose only one unless assigning elevated permissions to specific users.		
	Example : The Technologist User group has the Technologist User role. Only the lead technologist is permitted to purge studies and merge studies. To accommodate this, create a user role with only the merge and purge rights, then create a separate user group and select the technologist user role and the new user role. Finally, apply the separate user group to the Lead Tech User.		
Navigation	Select the areas of the program that group members can access (items available on the burger menu).		
	Note:		

Setting	Description
	• This only allows opening the item, and certain user rights may be required to work in a given area.
	• Even if selected here, the Fax Manager requires activation to open.

4. Select **SAVE**.

See also:

Create a user

Create a user

A *user* is a member of a user group for whom you can configure custom rights and limitations in addition to those defined in user roles.



Prerequisite: Create a user group.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > User Management > USERS.

= USER MAN								3
UNERE	MACH HOLKS	STREET GROUPS	AD GROOM	ADVERS	anacuta	(A20124)	O TE LA	America 1
3 Show Warding					ADD	RELOAD	MPORT	HELP
I TERM NAME 🛎		MADIA.		HELPARS		train 17	on 160	19 16 T
						13	- Al	4
-Siringin		Repts Ange		Facility Admin		Others		
/ Ellerytest		Tect, Davry		Facility Admin		Othere	M	
CERNO.		Test, HII		Faility Johns		Othere	M3	55

2. Select ADD.

			80/6	SIVE & CLOSE	UNLOCK ACCOUNT	2448	HACK
Group Neme *	Salara George	÷		enz Expires After	sta		
	Thie AD Groups			the supress of the	C One-Time Access		
Neue -	Plaine an	(MI)			E Unit Mat Ching		Tabel:
	Land Horns	Eaths.					
Motile Phena	Multile Phone.				Assign Cliene to I Usei	follow-Og Quana	for Another
E-Max1	Emiel;				D the cart thange i	country dates	6
User Name *	View Name	I HAROTAN			Diagon 160		
Painment *	Hannerd		Var	kot	NOME SELECTS	10 - I	
Confirm Password *	Conduct theorem		Fac	lites *	NONE SELECTION	Ð -	
Sebulos Interval				ed Provide: Liker Type	Salert 4		
	🗇 Allaw Emergency Acco	***			Bits Order Moni		

3. Enter the following settings.

Not all settings may be available depending on your product and configuration.

Caution: A password is configured or used on this page. For recommended security, <u>turn off</u> <u>Chrome autofill</u>.

Setting	Description
Group Name	Select the user group to which the user belongs.
	Select Portal to limit the user to a portal; this removes any Exa or Symmetry rights.
	Note : This does not grant portal access to the user. To grant access, set the Linked Provider User Type (see below) to one with a portal, such as Provider .
Name	Type the user's name.
Mobile Phone/ E-Mail	Type the user's mobile phone number and/or email address. The email address must be unique, and is required if using two-factor authentication.
User Name	Type a sign-in user name for the user.
Inactive	Select to disable the user account. If extra security is needed, also change the user's password.
Login with Google	When selected, the user can sign in through their Google account.
Netcare access	Select to grant the right to access Alberta Netcare (Alberta only).
Password	Type a sign in password for the user. This option is only available the first time you configure the user.
Session Interval	Type or select the number of minutes before the user's session times out.
Allow Emergency Access	Select to assign near "super user" (full) rights to the user regardless of roles or groups.
Linked Ordering Facility	Available when you select Ordering Facility in the Linked Provider User Type dropdown list.

Setting	Description
	Select ordering facilities to appear on orders the user creates in the Ordering Facility portal.
Access Expires After	Type and select a duration after which the user's account expires. Leave blank to keep the account open indefinitely.
One-Time Access	When selected, the user can sign in only one time.
User Must Change Password Next Login	Select to require the user to create their own password after signing in the first time.
Assign Claims to Follow-Up Queue for Another User	Select to give the user the ability to add claims to another user's follow-up queue.
User Can Change Accounting Dates	When selected, the user can edit accounting dates under Billing > Payments.
Dragon 360	[To be discontinued]
Market	Select the market that the user serves. This setting narrows search results in other parts of the program. For example, when the user searches for a patient chart, Exa PACS/RIS only returns charts of patients in the same market as the user.
Facilities	Select the facilities that the user can view and modify. Select All means "Select all current and future facilities."
Linked Provider User Type	If the user is also a resource, select the user type to link, and then select the user in the dropdown list that appears.
	This also has the effect of giving access to a portal (Ordering Facility, Provider, Attorney) or the exam screen for technologists.
Hide Order Menu	[To be discontinued]
Ordering Facility	Available when you select Ordering Facility.
	Select the ordering facility whose portal the user can sign into.
Patient Search Type	Available when you select Provider or Ordering Facility . Select the range of patient and other data that the user may search:
	Associated Patients Only : The provider only sees patients associated with their resource on worklists.
	Full Database: The provider is not restricted to seeing their patients only.
Scheduled By	Available when Ordering Facility is selected.
	Select an option to control the dates available to the user when requesting an appointment in the ordering facility portal.

4. Select **SAVE**.

- 5. Optional.
 - To limit what the user can view on the worklist, see <u>Configure an administered worklist filter</u>.
 - To view a user's devices, select the **ATTACHED DEVICES** sub-tab.
 - To restrict the user's access to specific ordering facilities, select the **USER PERMISSIONS** sub-tab.

See also:

Create a user role

Reset a user's password as an administrator

If a user forgets their password, an administrator can reset it or prompt the user to reset it the next time they log in.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > User Management > USERS.

	ADEMENT -			
MRHS.	anna anns	nun secure	()AR940044	ADDRESS CO
Bhow Insotive				
UNTRAMMET		nuut		SHOUT NAME
/ Badast		FE-esh Argin		Ficiet deal

- 2. Double-click the user in the list and do one of the following.
 - Select **RESET PASSWORD**, and then choose a new password for the user.
 - Select the User Must Change Password Next Login checkbox.

RESE	T PASSWORD	UNLOCK ACCOUNT	BACK						
After		days Choose one							
	Cne-Timedecesa								
	User Must Change Password Next Login								
	Asaign Ci User	aims to Follow-Up Queue fo	r Another						
	User can	User can change accounting dates							
	Dragon 30	50							

- 3. Select SAVE.
- 4. Inform the user to sign out and then sign in again with the new password, or follow the on-screen instructions to reset their password.

Configure an administered worklist filter

In addition to user roles, administrators can use the following steps to configure worklist filters that control which studies, patients, and other information are available to individual users and user groups. Users can configure their own worklist filters but cannot edit filters that administrators configured for them.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **User Management** > **USERS**.

🛋 URER MANA	CEMENT'S			
uters 1		(100950099)	100500095	ADUSTRIC
🗇 Skon Inopäre				
usernoor :		habe		NO PARE
/ Estuti		RDeak, Angle		Front deak
/ Einner		Get, Anna		Ascelly Letters
stimute.		Regin, Andre		Feelity whole

2. Double-click a user row to open a user, and select the **WORKLIST FILTER** sub-tab.

WORKLIST	ATTACHED	NEVICES USE	R PERVISSIONS				
SCINERAL	Show Fee	ounters Only [] Disable Right-Cl	ck an W	erli 🕞 517/1 Sturber O	verside Current Sab-Filter	
PADEHI	Show DIC	ON Studies Only [Show RIS Orders	Orly.	Shoe Assigned	Studies Cely	
INFORMATION.	Deloted				AD	~	
III IIIIOV INFORMATION	Date Operations By	🤔 Gludy Data	O Bludy Receiv	oć Deto	O Oxheduled Date) Q Loot call made	
te Hessunce	Proformatted	Call index					
e numari	O Lest O Nett	Haro	e		han	11 To	M
	O Late From		Ð	a	Пане То	6	ß

3. Perform one or more of the following procedures.

Configure general user worklist filter settings

You can finely control the information on the worklist to which the user has access.

1. In the left pane, select **GENERAL**.

2. Enter the following settings, and then select **SAVE**.

Setting	Description
Show Encounters Only	Select to show only studies involving interaction between a patient and provider.
Show DICOM Studies Only	Select to show only DICOM (imaging) studies.
Disable Right-click on Worklist	Select to prevent the user from opening the worklist shortcut menu.
Show RIS Orders Only	Select to show only orders from RIS.
STAT Studies Override Current Sub-Filter	Select to always show studies with assigned stat levels, regardless of any other filtering criteria.
Show Assigned Studies Only	Select to show only studies to which a user is assigned.
Deleted	Select whether to show all, only, or no (None) deleted studies.
Configure the following settings to	gether to compose a single filtering condition.
Date Operations By	Select a study property against which to apply criteria.
	Study Date
	Study Received Date
	Scheduled Date
	Last Call Made
	Created Date
Preformatted	Select from a list of preformatted date ranges, rather than manually entering one by using the settings described later.
	Example:
	Date Operations = Study Date
	Preformatted = Last 7 Days
	"If the study date falls within the last 7 days, display the study on the user's worklist"
Last/Next	Select a time range for the selected property.
	Example:
	Date Operations = Study Received Date
	Select Last, type 8, select Week(s)
	"If the study was received some time during the last 8 weeks, display the study on the user's worklist."
Date From / Date To	Select and enter a date range for the selected property.
From / To	Select and enter a time range for the selected property.

Limit access to patient information

You can limit the worklist to only display studies for patients who satisfy certain criteria.

1. In the left pane, select **PATIENT INFORMATION**.

- Under Patient Name, select a logic option, type the exact portion of the patient name that corresponds to the selected logic option, and then select the plus + button to add the criterion.
- 3. Under **Account No.**, perform the previous step as appropriate for the account number.
- 4. Select **SAVE**.

Limit access to study information

You can limit the worklist to only display studies that satisfy certain criteria.

- 1. In the left pane, select **STUDY INFORMATION**.
- 2. For each criterion, select a logic option, and then type or select a value.
- 3. For example, to display only abdominal studies, under Body Part:
 - Select the **Is** option
 - In the list of body parts, select Abdomen
- 4. Select SAVE.

Limit access to information related to specific resources

You can limit the worklist to only display studies whose resources satisfy certain criteria.

- 1. In the left pane, select **RESOURCE**.
- 2. For each criterion, select a logic option, and then type or select a value.

For example, to display only studies whose referring physician is Jane Doe, under **Ref. Phys.**:

- Select the **Is** option
- In the box, type JANE DOE, and then select the plus 📩 button.
- 3. Select SAVE.

Limit access to information from specific insurance providers

You can limit the worklist to only display studies for patients who satisfy certain criteria.

- 1. In the left pane, select **INSURANCE**.
- 2. For each criterion, select a logic option, and then type or select a value. For example, to hide studies under litigation, under **Insurance Provider Type**:
 - Select the **Is Not** option
 - In the box, select **Litigation**, and then select the plus **t** button.
- 3. Select SAVE.

See also:

Create a user worklist filter

Manage who is online

Users with user management rights can view a list of users who are online, view a user's activity log, and terminate their session.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **User Management** > **USERS ONLINE**.

	DIERMANA					0
		MHORONE: LIEBCHIOPE:	AN 1999 - A		ARRENT DITIONI	Minning >
						=81050
	USERMANE	1008409/0075	(3,640.95	LANTAG	cesses 2	
		03/12/0028 - 08/14/20	22.			
ð	Victols Page	00/14/2023 7/07 AM 607	10217,834	0 Simeps		
3	Serves densits	-5974/10018/s1 AMART		87 m ago	0	Zeimisate
3	Dervice, Asherr	CO/14/2003 7:41 AM 807		27 m age	63 - C	Territate

- 2. Optional: To view a session log for a user, select the log 🖄 button.
- 3. Optional: To terminate a session, select the **Terminate** link.

View assigned studies and patients

You can view lists of studies and patients that are assigned to you (the current user).

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **User Management**.

	AGEMENT:						
Kacal ar	ASSIGNED STUDIE	ATHONCO PACIFIETE	LINEP FARTY TOOLS.	- COPY	PROFILGACTION	DOLLAR WELL	
-	unis variation i					BELOAD	
LAGERS AND	rilati seroobia	PERMIT					
Green Annumero 5		4552946 2000 #2		SULEY STAT	tin Astronomete me	Assessed on	
] Rudiologict, Richard	1166 3	ар кыргана ат менеристи х кра	EPESTPOK	Appraveil	Text, Fast	03/85/2020	
Hattoopo, Petraid	- 4	IORELEDIO 194 SPUCIES		hereas	en presia	00.00650000	
Religiogial Richard	20	ANOTH ACTIVIL UNI SUCTIVITIES		Age and	har timella	01/16/2020	

- 2. Do one or both of the following.
 - To view studies assigned to you, select the **ASSIGNED STUDIES** tab, and then select a subtab (such as Users or Reading Provider) to filter by user type.
 - To view patients assigned to you, select the **ASSIGNED PATIENTS** tab. Patients can be assigned from the patient chart by using the **Users Assigned** dropdown list.

Copy settings from one user to another

To create new users more quickly, you can add a user and then copy settings from an existing user.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > User Management > COPY PROFILE SETTINGS.

10.04	(99) (8378990)	anasia.	ASSIGNED PATIENCIS	11 100 PM/IN 100 5	COTY PROFILE BETTINGS	OKTOREN MUTH
						Howe
outos Usar *	Seart Sauna Land					
estimation thire.*	3440000000	- 11	===			

- 2. In the **Source User** dropdown list, select the user whose settings you want to copy.
- 3. In the **Destination User** dropdown list, select a user to configure with the source user's settings, and then select the plus **button**.



Caution: Never overwrite the **Rad test** or **Ref test** users. Instead, copy to an account of a different name or create a new account.

- 4. Optional. Repeat to configure additional users with the source user settings.
- 5. Select SAVE.

Configure general settings

You can set up study forms, custom forms, templates, and queues, and perform limited database queries.

This section contains the following topics (not all topics may be available depending on version and region).

Create a study form Export and import a study form Create a custom form Perform a quick query of database totals About email templates and notification templates Configure an email template Set up emailing of results and reports Configure a notification template Set up sending of notifications Create fax cover sheets Work with the report queue

Create a study form

Study forms are interactive electronic forms that people inside and outside your organization can use to gather or provide needed information quickly—and often automatically. Examples include consent forms for HIPPA or treatments, screening forms, pain sheets, affidavits, customer payment plans, and procedure checklists. For information on how to use forms that you create, see <u>Add a study form to a study</u>.

Set up a study form

Use the following procedure to initially set up the form, and then see later in this section to add and edit form contents.

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **General** > **FORM BUILDER**.

· O ENERAL-					0
FORV BULICE D.STONTCOV	INAL TENTATE	1011001001001		DOVER BULL	m m r.)
		(A00)	nilloko,	DIPORT	INFORT
DOCUMENT NAME				MATHETE	
				All	
P.B. Contrast Conservation Stational Days					- ii
Ø ≧ 07 Spreeing Fam					
🖉 🗟 Tehocardogram Walksheel					

- 2. Select ADD.
- 3. Type a Form Name, and then select the **ASSIGN** sub-tab.

Form Name* To	eat	Silmage 🛛 Checkle
PROF	EFFTIES ASSIGN MER	GE FIEL04
Document Type	Select	×
🗆 Viewable on F	Patient Portal	
Auto Assign t	o Study	
Insurance		
Provider *	Search Carrier	
Provider *		
	Empty	
Allow All with	Empty	
Allow All with	Empty	•
Allow All with	Empty	
Allow All with	Empty	

4. Enter the following settings to define the scope of the form and filter its availability.

Setting	Description
Document Type	Select a type in order to group forms and make them easier to find.
Viewable on Patient Portal	Select to make the form available to add to the patient portal and Patient Kiosk.
	Note : Selecting this does not immediately add the form to the portal and kiosk. Instead, it adds the ability to add the form to the portal and kiosk when you add the form to a study later on.
Auto Assign to Study	Select and then enter study criteria to automatically assign the form to those studies when you create them (such as in the New Appointment screen).
	Assigned forms appear in the Study Forms tab of the Edit Study screen, making them available to users without performing dedicated searches.
Insurance Provider	Select to restrict the form to specific insurance providers.

Allow All with Empty	Select to make available to all providers if none are specified.
Allow All without Empty	Select to make the setting required.
Physician	Select to restrict the form to specific physicians.
Allow All with Empty	Select to make available to all physicians if none are specified.
Allow All without Empty	Select to make the setting required.
CPT Codes	Select to restrict the form to studies with specific CPT codes.
Allow All with Empty	Select to make available for all CPT codes if none are specified.
Allow All without Empty	Select to make the setting required.
ICD Codes	Select to restrict the form to studies with specific ICD codes.
Allow All with Empty	Select to make available for all ICD codes if none are specified.
Allow All without Empty	Select to make the setting required.
Markets	The form is available to patients in, or having exams scheduled in the markets you select. All facilities in the selected markets become selected in the Facilities dropdown list.
Facilities	The form is available to patients in, or having exams scheduled in the facilities you select.
Modalities	Select to restrict availability to the form to studies from specific modalities.
Male Patient Only	Select to restrict availability to the form for male patients only, such as for prostate exams.
Female Patient Only	Select to restrict availability to the form for female patients only, such as for HSG exams.
Mammo Patient Only	Select to restrict availability to the form for mammography patients only.

5. Select **SAVE**.

Add a header to a study form

- 1. At the top of the **FORM BUILDER** tab, select the **Header** checkbox.
- 2. In the form area, select the word "Header."
- 3. In the left pane, on the **PROPERTIES** tab, select **CHANGE LOGO**.
- 4. Browse for and select a logo, and then select **Open**.
- In your form header, select the word "Header" and then replace it by typing a new field name.
 For example, type Study Date:
- 6. In the left pane, on the **MERGE FIELDS** tab, in the search box, type search term to look for a relevant merge field.

For example, type study.

atName		Study Date	Header" with new header "\$\$\$1udyDate\$\$ Patient:	text, for example: \$\$Lastname\$\$, \$\$FirstName\$3
ge Fields tient ut Name	PROPERTIES	ASSIGN	MERGE FIELDS	Brag
at Name	Caller -		9 ~	
name -	dieot ntName	/		

7. In the list of results, double-click a merge field to add it to the form header.

For example, double-click **StudyDate**.

8. To the right of the **\$\$StudyDate\$\$** merge field, type a new field name. For example:

Patient Name:

9. Repeat earlier steps to find and add a merge field for the patient name.

Hint: Search for "name" rather than "patient."

- 10. In the left pane, on the **PROPERTIES** tab, use the **Spacing** and **Style** controls to modify the layout.
- 11. Select Save.

Another example header:



Patient Name: \$\$Fuliname\$\$ Study Description: \$\$StudyDescription\$\$

Add controls to a study form

Use controls to **gather information from** or **provide information to** the person filling out the form. To add a control, drag its button into the body of the form. To configure the control, select it and enter options in the left pane.

📾 image 🖉 Checkbox	® Redis Dutton Text Box Text Area Text E Drokep. ₽
MERCE FIELDS	
i N	Overwrite "Header" with new besder text, for example: Study Date: SSStudyDateSS Patient: SSLestnameSS, SSFirstNam
	This is a Text Box
o Ali	These are Checkbaxes
	Walk-in Emergency
	This is a Text Area
•	The user can enter text here.
	These are Radio Buttons
	Option 1 Option 2
*	This is a Free Text box
*	Enter some static text
italic 🔲 Underline	

Details on form controls

The following sections in this topic provide additional information and instructions about form controls.

Text controls

Text Boxes to gather one-line user input.

Text Areas to provide boilerplates.

Free Text boxes to gather or provide information longer than one line.

To add a text control:

- 1. Drag the **Text Box**, **Text Area**, or **Free Text** button onto the form.
- 2. Select the word "Untitled," and then in the left pane, in the **Question** box, type text to introduce or label the text input box. For example:

For an information gathering box, type Please list all allergies.

For an information providing box, type Liability waiver.

3. If the text control is for providing information, enter the information in the box.

Adjust vertical spacing:

- To adjust the space between the question and the box, use the Line Height slider.
- To adjust the space before and after the text control, use the **Padding** slider.
- To apply adjustments to all controls on the form at once, select **Apply Spacing to All** checkbox, and then use the sliders.
- If using a Text Box, select the following options for **Inline Controls**.

Setting	Description
Fit on One Line	Select to place the label one the same line as, and to the right of, the text input box.
Inline Multiple Controls	Select this option for two adjacent text boxes to place both on the same line.
Question First	Select to move the label to the left of the text input box.
Append Text Input	Select to add a new input box between the label and the original input box. This new input box is only available to the person filling out the form.
Dynamic Input	Select to fit the height and width of the text box to the text it contains.

- If using Free Text to provide information, apply formatting to your text as needed by using the **Style** controls.
- To adjust the width of the text control, select an option in the Size dropdown list.

4. Select **SAVE**.

Add an image to a study form

- 1. To add an image, drag the **Image** button to the form.
- 2. To give the image a title, select **Untitled**, and then type a title.
- 3. Double-click the box below the title, and then browse for and select an image file.
- 4. On the left pane, use the controls to modify the alignment and size.



Note: If you select a size of Original, the image expands to fit the width of the page.

Add checkboxes to a study form

- 1. To add a group of checkboxes, drag the **Checkbox** button onto the form.
- 2. Select the word "Untitled," and then in the left pane, in the **Question** box, type text to label the checkbox group.

For example, type: Allergies.

- 3. Adjust the vertical **Spacing**, **Inline Controls**, and checkbox label **Style** (see "Adjust vertical spacing" earlier in this topic).
- 4. To add a new checkbox to the group, select **ADD ANSWER**.
- 5. To label individual checkboxes, type labels in the boxes under **Answers**.

For example, type <code>Peanuts</code> for the first checkbox, and <code>Mold</code> for the second.

- 6. To arrange the checkboxes in columns, select a number in the **No. of columns for Answers** dropdown list.
- 7. To add an "Other" checkbox with accompanying text input box, select the **Add Other** checkbox.
- 8. Select **SAVE**.

Add options to a study form

Add checkboxes for "multiple choice," but add options for "either/or" settings. You add an option group by using the same procedure as for checkbox groups, except that you drag the **Radio Button** button onto the form instead of the Checkbox button.

Add a group box to a study form

A *group box* is a named group that can contain one or more checkbox groups and/or one or more option groups. To add a group box:

- 1. Drag the **Group** button onto the form.
- 2. Select the word "Group Title," and then in the left pane, in the **Question** box, type text to label the group.
- 3. To add a background image to the group, in the left pane, select **CHANGE IMAGE**, browse for and select an image, and then select **Open**.
- 4. To divide the group into columns, in the **Grid Columns** dropdown list, select a number of columns.
- To add a checkbox group or radio button group into a particular column, in the New Control Placement dropdown list, select the number of the column, and then drag the Checkbox or Radio Button button onto the group.

- 6. Configure the checkboxes and/or radio buttons as described in "Add checkboxes to your form."
- 7. Select **SAVE**.

Add a footer to a study form

You can add and modify footers in the same way as headers. However, we often recommend using the footer for a signature line with a signature merge field. After the person electronically signs the form, their signature appears in the location of the merge field. To create a signature line in the footer:

- 1. At the top of the FORM BUILDER tab, select the Footer checkbox.
- 2. In the form area, select the word "Footer," and replace it by typing Signature.
- 3. In the left pane, on the **MERGE FIELDS** tab, search for "signature" and then double-click SudyFormSignature in the list of results.
- 4. Select SAVE.

See also:

Add a study form to a patient record Add a custom form to a patient record

Export and import a study form

To save time creating a form for "Facility A," it may be helpful to import an existing form from "Facility B" and then modify it.

Export an existing study form

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **General** > **FORM BUILDER** > **EXPORT**.

FORM BISLOPR	GILLOW ROOMS	DESTINATE	CONTRACTOR				0.111.13
				SARE:	(088.000)	-Existin	MPORT
DODUMENT NAME						NARETS	
						East 1	19

Result: The form appears in the Windows Downloads folder as a JSON file.

2. Copy the JSON file to the computer at Facility A.

Import a study form

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **General** > **FORM BUILDER** > **IMPORT**.

- 2. In the **Import Study Forms** dialog, select **CHOOSE FILE**, browse for and select the copied JSON file, and then select **Open**.
- 3. On the FORM BUILDER tab, find the new form and edit it for use at Facility A.

Create a custom form

Custom forms are non-interactive electronic forms intended for delivering information. For interactive electronic forms that can both deliver and gather information, see <u>Create a study form</u>.

Set up a custom form

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **General** > **CUSTOM FORMS** > **ADD**.

= GENER	RAL -			
FO	EM BUILDER	CUSTOM FORMS	DETOTALS	EMAIL TEMPLATE N
Jocument Name *	Custom Form	Test	Labels	
Sescription	An example	1		Patient Review
				Patient Mammo Ferm
				 Education Material Reference Document

2. Enter the following settings.

Setting	Description
Document Name	Type a name for the form.
Description	Type a general description.
Labels	[Unused]
Patient Review	[Unused]
Patient Mammo Form	
Education Material	Select to make the form available in the EDUCATIONAL MATERIAL area of the patient chart.
Reference Document	Select to make the form available in the DOCUMENTS and CUSTOM FORMS areas of the patient chart.

3. Move down the page to Assign Document.

Assign Docu	iment				
Modalities	Select modality			+	m CT imes m MR imes
Facility	My Company		× •	+	
ICD Codes *			× •		
	Allow All with Empty				ADD.9.Cholera, un:
Medication *	Medication Allow All with Empty Allow All without Empty	+			15uprofen

4. Enter the following settings.

Setting	Description			
Modalities	Select modalities to which to assign the custom form.			
Facility	Select a facility to which to assign the custom form, and then select the add + button.			
ICD Codes	Select to restrict availability for specific ICD codes.			
Allow All with Empty	Select to make available for all ICD codes if none are specified.			
Allow All without Empty	Select to make the setting required.			
Medication	Select to restrict availability for specific medications.			
Allow All with Empty	Select to be available for all medications if none are specified.			
Allow All without Empty	Select to make the setting required.			
Lab Codes	Select to restrict availability for specific lab codes.			
Allow All with Empty	Select to make available for all lab codes if none are specified.			
Allow All without Empty	Select to make the setting required.			
Male/Female/Mammo Patient Only	Select to restrict availability of the form to male, female, and/or mammography patients.			

5. Select **SAVE**.

About adding contents to a custom form

The custom form word processing area provides commonly recognizable tools to type, align, and apply formatting to text, and insert images and merge fields. You can copy and paste contents by using the **Ctrl+C** and **Ctrl+V** keyboard shortcuts. To change font sizes, select text that you want to resize, and then select a size in the **Font Size** dropdown list.



Add merge fields to a custom form

In this example we show you how to add a signature merge field to a custom form. You must add a signature merge field to be able to electronically sign the form.

1. In the right pane, in the **Merge Fields** area, in the search box, type search term to look for a relevant merge field.

For example, type sign.

2. In the list of results, double-click a merge field to add it to the form.

For example, double-click **CustomFormSign**.

3. To the left of the \$\$CustomFormSignature\$\$ merge field, type a new field name.

For example, type Signature:

4. Select SAVE.

Editing a custom form

- **Note:** If you edit an existing form, your changes will *not* appear on forms that are already attached to a patient chart. To edit a form:
- 1. On the burger 🔲 menu, select **SETUP**.
- 2. On the **OFFICE** menu, select **General**.
- 3. On the **CUSTOM FORMS** tab, select the form 🖹 button of the custom form that you want to edit.

4. Edit the form as needed, and then select **SAVE**.

Perform a quick query of database totals

As an alternative to reports, you can quickly search the PACS and RIS databases with customizable queries. A "DB total" search tells you how many patients, studies, and images in your databases match the search criteria.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **General** > **DB TOTALS**.

, rem	edinter of	ttele roman	DO TOTALS	TWN TOWNAR
licto#2ngo	02/14/2029	115	08/14/2028	121
	Standy Date	Q. Study P	lecenos Date O	Approved Date
Richty	48	1		
Nodalitate	1.00000000	m- 🗖	14	
Glady Description		10 AI		
antuteee			- A	
			e	
	LEGATETICAL	s esve	apeter to	

2. Enter the following search criteria.

Note: If you performed a search previously during the current session, refresh your browser before entering new criteria.

Setting	Description			
Date Range	Select a date range within which to search.			
Study Date	Select a category for the date range:			
Study Received Date	Search all studies with a study date within the date range.			
	Search studies whose DICOM receive date is within the date range.			
Approved Date	Search studies that were approved during the date range.			
Facility	Filter the search by facility.			
Modalities	Filter the search by modality.			
Study Description	Filter the search by study description.			
Institutions	Filter the search by institution.			

Ī

To add a new institution to the list, clear the All checkbox, type the name
of the institution in the box, and then select ADD.

- 3. To run the search, select **UPDATE TOTALS**.
- 4. The search results appear below the button under **DB Details**.
- 5. Optional. To export a more detailed report of your search and results, select **CSV REPORTS**.

About email templates and notification templates

Email templates and *notification templates* contain pre-formatted text that you can automatically send through and along with various media. Note that the original term *email template* now also applies to non-email media such as faxes. Commonly, you use **email templates** to auto-send reports to providers and patients, and **notification templates** to auto-send study status changes to referring providers. The following table summarizes the differences in auto-sending media based on the type of template.

To create templates, see <u>Configure an email template</u> and <u>Configure a notification template</u>. To configure auto-sending of media using the templates, see <u>Set up emailing of exam results</u> and <u>Set up sending of notifications</u>.

Feature	Auto-Sending of Media Based On:	
	Email Templates	Notification Templates
Where to configure template	Setup > [all caps button menu] > General > Email Template	Setup > [all caps button menu] > General > Notification Templates
Who can receive media	Referring provider locations/contacts, referring provider group, referring provider's patient (in Patient Portal)	Referring provider locations/contacts
What media are sent	In addition to text defined in the template:	Email or fax of status changes, including text defined in the template.
	Reports	
	PDF attached to email	
	Link attached to email	
	Print and mail	
	Print on selected printers	
	Fax to referring providers	
	Fax to referring provider's office	
	Images	
	CD, Film, Paper	
How to turn	Select checkboxes in "Reports to Me"	• Select checkboxes is "Notification Settings," and
ON/OFF auto- sending		 Select notification template in each status at which you want to auto-send.

Configure an email template

Follow the steps in this topic to configure an email template. To use the template to auto-send emails and other media, see <u>Set up emailing of exam results</u>.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **General** > **EMAIL TEMPLATE**.

=	OBNERAL:		1027				
	11 E	2,970M HOUSE		MPUTE: NOTIFICATION TAMPLETE		COYENSHEETS-	(EPO)
						A00	RELOAD
	TEMPLATE WANK	REPORT TYPE .	RBACT	6404DP		BANNET	
		A0 -		N-1-		Al	12
121	Notification Phy	Study Careabilion	Test	Angia Society/Ay Company/Hemil	a Testi F	2	
17.8	construction	e continuation become	periet apportune confere	atter schall ing congrangite state society	inia	1	


2. Select ADD.

Templete Name *	Default Confirmation Scre		Inactive
Category	Confirmation Screen	*	
Subject *	Appointment Confirmation Email	1	

3. Enter the following settings.

Setting	Description
Template Name	Type a name for the template.
Category	Select the type or purpose of the template.
Subject	Type text to appear on the Subject line.
Markets	Select the market for which the template is available. All facilities associated with the selected market become selected in the Facilities list.
Facilities	Select the facilities to whose studies the template applies.

4. Use the word processing tools to compose the body of the email.

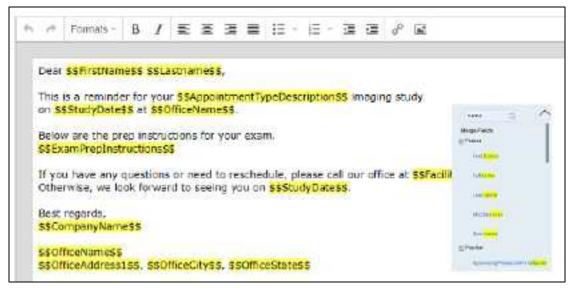

- 5. To automatically insert exam details into the email:
 - a. In the right pane, in the **Search here** box, type a category of information (e.g. Patient).

Note: Category searches are case-sensitive.


- b. In the resultant Merge Fields list, select a merge field group (e.g. Patient).
- c. Double-click a merge field to add it (e.g. FirstName).
- 6. Select SAVE.

See also:

About email templates and notification templates

Set up emailing of results and reports

By completing the following tasks, Exa PACS/RIS automatically sends exam results, approved reports, and appointment confirmations to the referring provider locations/contacts you modify.

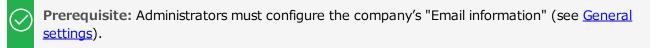

Tasks:

- <u>Configure an email template</u>.
- Add a facility, and configure all email- and fax-related settings.
- If you want to email a provider group or location, <u>Add a provider group or location</u>.
- Configure a referring provider location/contact, including the **email address**, **fax numbers**, and options under **Reports to Me** (see <u>Add a resource</u>).

See also:

About email templates and notification templates

Configure a notification template

Follow the steps in this topic to configure a notification template. To use the template to auto-send status change emails or faxes, see <u>Set up sending of notifications</u>.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **General** > **NOTIFICATION TEMPLATES**.

	GEHENN.					23	20
	CORDER NOR BOTT	SUGTEM FORME	DESCRIPTION OF	DWATERAT	NOTIFICATION TEMPLATES	CONTRACTOR	нсеоп
						ADD	RELOAD
	TEMPLATENANC *	OUBJEET		TADULTY .		NATION	
				1.84		20	*
58	a detrophistication	Teol		Ange Fødit	Wy Coopers Noville Test Pacificati		

2. Select ADD.

GENERAL			
(romanitates) (cons	ON FORMS ON TOTALS	HANG HIRFEAM	NOTIFICATION TEMPLATES COVER UPDET
Template Name * RetPhylaoSiscotion	Subject Teat		EAVE BACK Market None season eo - Pacimos * A SALACTED -
⇒ # Formati - B I ≡ S\$Fullname\$\$ \$\$AccessionNo		3 3 0 R	name C
			Middiename User same

3. Enter the following settings.

Setting	Description
Template Name	Type a name for the template.
Subject	Type text to appear on the email's Subject line.
Market	Select the market for which the template is available. All facilities associated with the selected market become selected in the Facilities list.
Facilities	Select the facilities to whose studies the template applies.

- 4. Use the word processing tools to compose the template.
- 5. To automatically insert exam details into the notification:
 - a. In the right pane, in the **Search here** box, type a category of information (e.g. Patient).



Note: Category searches are case-sensitive.

- b. In the resultant Merge Fields list, select a merge field group (e.g. Patient).
- c. Double-click a merge field to add it (e.g. Fullname).
- 6. Select SAVE.

See also:

About email templates and notification templates Merge fields

Set up sending of notifications

By completing the following tasks, Exa PACS/RIS automatically sends status change email notifications to the referring provider locations/contacts you modify.



Prerequisite: Administrators must configure the company's "Email information" (see <u>General</u> <u>settings</u>).

Tasks:

- <u>Configure a notification template</u>.
- For each status of which you want to auto-send notification, select the notification template under "Notification Settings" (see <u>Configure study statuses</u>).
- Configure a referring provider location/contact, and select the location/contact's **Email/Fax** and/or **Receive When Added as a CC Provider** checkboxes (see <u>Add a resource</u>).

See also:

About email templates and notification templates

Create fax cover sheets

You can create a cover sheet to automatically attach to faxes. You can create one cover sheet for each facility. To create a cover sheet:

Procedure

- 1. Go to <u>SETUP</u> > <u>OFFICE</u> > **General** > **COVER SHEETS** tab, select **ADD**.
- 2. Type a name for your cover sheet, select the facility where the cover sheet will be available, and then select **SAVE**.
- 3. Use the word processing tools create your cover sheet, and then select **SAVE**.
- 4. Select **SAVE & CLOSE**.

Work with the report queue

You can open a report queue that shows reports that you previously generated or that are currently in process. You can resend (reprocess) individual reports, or reprocess them in batches.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **General** > **REPORT QUEUE**.

-														
		DBITOTALS	CMAU	TIMELKI	t	NOTO	GATION I	EŴ	PLATES	COVER 3	08015	RP	ORT OU	900
						T	REPROC	524	SELECTED	1819	ROCENS F/	ALC:U	RE	.080
000.8	D DATE 2 LAET OPDATE	STUDY BOTE		CENER TO		usikit ja	нсамани	194	PATIENT	100.00	I DELIVERY	inpes	C.RRE	
Lucen			A	a v	AT	-	erenter.	1	*****	43 w			61 *****	ă.
eos.co	2923145	05/05/2023 1.4	otheo line	fela L	1025	17	-	9	Test Permer		(745)454	5454	0.etel	e.
inter a	2025 14505/98/2025 10	405/05/2023 110	01160 Bi	dana -	ierd icel				Test, Farrier	Émel	(initia)	ythorn	s Svert	
(503/CB)	20281325	10/10/2013 12:	11165 BA	dening Proc	каррго	VAC RAT	est Hefers	19	Test, Prarte	Fit	(705)054-	1000	Quebed	

- 2. To resend a report, right-click it in the list and then select **Resend**.
- 3. To resend a batch of reports, at the top of the list, select **REPROCESS SELECTED** or **REPROCESS FAILED**.
- 4. To resend a report automatically at a later time, right-click it in the list and then select **Requeue**.

Configure the viewer

In the Viewer Settings dialog, you can configure a wide variety of viewer options including display, interface, modality, toolbars, and shortcut menus. There are two ways to open the Viewer Settings dialog.

• In the Exa PACS/RIS viewer, in the upper toolbar, select the setting 🖸 button.

СРИМ КТ Р 🐼 🗶 🎟 🙋 🖹 🦹 🔣 📚 🧟 †+ 🖷 🕹 💕

• In the worklist, on the upper toolbar, select the settings button, and then in the button shortcut menu, select **Viewer Settings**.

This section contains the following topics (not all topics may be available depending on version and region).

Set up connected displays Configure autosave options Configure the toolbar Configure calipers Configure other interface settings Configure annotation tool functionality Configure annotation colors Configure annotation size and scaling Configure recording Show or hide tool buttons About ROI and SUV values and units Configure fusion, ROI, and SUV options Configure modality-specific viewing options Configure mammography Configure overlays Add or remove toolbar tools Add or remove tools from the image shortcut menu Assign toolbar keyboard shortcuts Assign keyboard shortcuts for Exa Trans

Set up connected displays

Configure all connected monitors that you intend to use with Exa PACS/RIS. These settings act as default behavior for what study items open in which monitor. If DMs conflict with the settings you enter here, the DM takes precedence.



Prerequisite: Add the Chrome extension.

Procedure

Ŵ

- 1. Go to <u>Viewer Settings</u> > <u>GENERAL</u>.
- 2. Under **Display Settings**, in the **Monitors on System** box, select the update \bigcirc button to update the monitor count (or enter manually), and then select the identify O button.

Noniturs on Syst	en	1		CO			
Monitor	Current	Prior	Full Screen	Orders	Туре	Calibration Width (mm)	Calibration Height (mm)
	0	0		-		_	-

3. In the table of monitors, enter the settings in the following table.

Note: At any time you can select the update 💭 button to preview your changes.

Setting	Description
Monitor	Select to make monitors available for image viewing. To display orders and studies on the monitor, clear this option.
Current	Select to make monitors available to display current studies.
Prior	Select to make monitors available to display prior studies, if available.
Full Screen	Select to default to full screen view.
Orders	Select a screen of the program to open in the monitor when manually opening it with a viewer tool, or when configured to open automatically.
	Documents : Opens the document list when you select the button in the viewer, or when Auto Open Documents is enabled.
	Notes: Opens the Notes screen.
	Transcription : Opens the Transcription screen (Exa Trans, Web Trans, or other).
	Other : Select other screens in the list. To make options available, select them in the Auto Open Orders list (see <u>Configure other interface settings</u>).
Туре	The color or grayscale display type.
Calibration Width/mm	Type a manual calibration width (appears on horizontal calipers)
Calibration Height/mm	Type a manual calibration height (appears on vertical calipers)

4. Select SAVE.

Configure autosave options

You can configure how the viewer performs automatic saving and other functions.

Procedure

1. Go to <u>Viewer Settings</u> > <u>GENERAL</u>.



2. Under **Autosave Options**, enter the settings in the following table.

Setting	Description
Autosave Image Properties on Close	Automatically saves image properties such as the W/L when the image is closed. Saved properties are restored the next time the image is opened.
Autosave Image Annotations on Close	Automatically saves image annotations when the image is closed. Saved annotations are restored the next time the image is opened.
Auto Mark Read Study on Close	Automatically marks the study as "Read" when the study is closed. If Exa Dictation is in use and the workflow includes dictation and transcription, this option is disabled.
Auto Upload Dictation on Close	Automatically uploads recorded dictations to add to the study when the study is closed.
Autosave Viewer Settings on Close	Automatically saves any changed viewer settings such as toolbar configuration when closing the viewer.
Auto Open Next Study	When the current study is closed, automatically opens the next study in the worklist that is not partially deleted.
Auto Send Annotated Images on Close	Automatically sends images if they include annotations when the current study is closed.

3. Select SAVE.

Configure the toolbar

You can configure the appearance and other properties of the viewer toolbar. To configure specific toolbar buttons, see <u>Add or remove toolbar tools</u>.

Procedure

- 1. Go to <u>Viewer Settings</u> > <u>INTERFACE</u>.
- 2. Under **Toolbar**, enter settings described in the table below.

Toolbar	
Toolbar Button Scale	
Tiumbnail Bar Scale	
Magnifying Glass Size	200 Pr. Factor
W/L Acceleration	
Span Sensitivity Auto Dfn Low	High
Thumbeall Column/Rows	•
Thumbnail Bar Orientation	Loft 🗸
Hoader Menu	Top 🗸
DM Toolhar	Top v

Note: You may need to refresh the browser or viewer for changes to take effect.

Setting	Description
Toolbar Button Scale	Adjusts the size of toolbar buttons as a percentage of the screen resolution.
Thumbnail Bar Scale	Adjusts the size of the thumbnail bar.
Magnifying Glass Size	Adjusts the size of the area affected by the magnifying glass, and the pixel magnification factor (the single digit in the text box).

Setting	Description
W/L Acceleration	Sets the rate or sensitivity of W/L adjustments. Typical range is 100–600.
Span Sensitivity	Sets the rate or sensitivity of span adjustments.
	Auto - Adjusts the speed automatically based on the number of images in the series.
	Dflt - The default sensitivity after upgrading to version 1.4.32_P1. This is the same sensitivity as the "0/Auto" sensitivity used in version 1.4.29.
	Low/High - Spans slowly or quickly.
	Span as Scroll - Spans in correlation with the scroll bar. For example, if the series has 400 images, you can drag from the middle to the bottom of the frame on image 1 to span from 1 to 200.
Thumbnail Column/Rows	Sets the number of columns and rows for the thumbnail bar.
Thumbnail Bar Orientation	Sets the position and resultant orientation of the thumbnail bar. If you select Mirror, the thumbnail bar in the left monitor appears on the left side of the screen, and the thumbnail bar in the right monitor appears on the right side of the screen (available when the modified toolbar is not available).
Header Menu	Sets the position of the toolbar and other header items.
DM Toolbar	Sets the position of the DM toolbar.

See also:

Configure other interface settings

Configure calipers

You can configure the number and placement of calipers.

Procedure

- 1. Go to <u>Viewer Settings</u> > <u>INTERFACE</u>.
- 2. Under **Caliper Position**, select checkboxes for each caliper that you want to display, and then select **SAVE**.



Configure other interface settings

You can configure miscellaneous viewer toolbar and other settings.

Procedure

- 1. Go to <u>Viewer Settings</u> > <u>INTERFACE</u>.
- 2. Under **Other Toolbar**, enter the settings in table below.

	-	Hub Partiel Clase Button
Other Toolbar		Hide All Weinter Cayout Charge Button
Ask on Close	Auto Open Orders	D Shaw Label in DM Presime
C Hec en Crese	🗇 Artistybes 👘	Shew Middle Silos Trumbrusi
💭 Wegle Instance Made	🗇 Additional Information	The property of the property of the property of
C Sto Fames in Crie to Mantan Plaghasi Sparel	🖾 Allegias	🔲 Loppie Cards Vinner
D Show Cutimus for First Last Images	Appred Reports	🗆 Disable-Querring 🔺 🗸
🗇 Øper Keylmagen militale lengen	Auto Rhow Prives Dar	🖾 Stretch Picor Bar
💭 Uan Madified Tooline	🔟 Fecusin on Zoom Point	CI-Sheev Prior Friday
🗇 Calije Zoom	2 Fill Empty DM Calls	Aute Open STAT
🖾 AsterHay Ona Except	D High-Quality Interpolation	□ Spin W/K Repton
Auto Play Cine West Tene (me) 100	C Exultor R35/VER Swap	Color Ovy as Last Monitor
💷 Delauk France Rote for MP4 🛛 15	Enable Developer Mode	🗇 Clear-Side Windum/Level for US
Datail: Cutino Thelesee 1	Show Galibrated Ultrassauld Region	🗇 Show Saan Oversig

ſ

Тор...

...middle...

...bottom

Setting	Description
Ask on Close	Prompts you to specify which items to save before closing the viewer, related to the <u>Autosave</u> options.
Single Instance Mode	Limits the viewer to one display frame per browser.
Skip Frames in Cine to Maintain Playback Speed	Skips large images in a cine loop so that the cine plays a fixed speed. Helpful with slower computers or video cards.
Show Cutlines for First/Last Images	Shows any available cutlines only on the first and last images of a series.
Open Key Images as Single Images	Displays any available key images separately in the thumbnail area. When cleared, shows all series images in the key image thumbnail area.
Use Modified Toolbar	Temporarily hides the thumbnail bar, and adds a Prior bar at the top of the screen that displays patient priors in order. You can hover over a prior to display series/image thumbnails, and select the thumbnails to open the images. Works in conjunction with the Auto Show Priors Bar option.
Caliper Zoom	If selected, you can drag the calipers to zoom the current image.
Auto-Play Cine Loops	Automatically plays cine loops when opened.
Auto Play Cine Wait Time (ms)	Sets a pause time before auto-playing cine loops.
Default Frame Rate for MP4	Sets a default frame rate for MP4 files.
Default Cutline Thickness	Sets a default thickness for cutlines.

Setting	Description	
Auto Open Orders	Selects the options that appear in the Orders dropdown list in Viewer Settings > General > Display Settings.	
Auto Show Priors Bar	Shows all prior studies in a separate bar at the top of the viewer.	
Focus-In on Zoom Point	When selected, the user can drag on an image to zoom in and out at the initial pointer position.	
Fill Empty DM Cells	Fills undefined DM frames with the next available images in the current study.	
High-Quality Interpolation	Select to use high-quality image interpolation. Frequently used to enhance X-ray quality. You can use this option to remove unwanted "gridlines" if switching between earlier and high-resolution monitors.	
Enable RGB/YBR Swap	Makes the color settings supported by the user video settings available. Corrects Red/Blue color reversal errors in venous US and other color-based modalities.	
Enable Developer Mode	Shows development mode logs, and displays a vertical green line in the right side of each frame when the server performs a prefetch.	
Show Calibrated Ultrasound Region	Server side setting.	
Hide Partial Close Button	Hides the partial close button, an "X" that appears next to the logoff button. Partial closing closes the study, but leaves the viewer open for faster loading of future studies.	
Hide All Monitor Layout Change Button	Hides the toolbar's monitor layout change button that appears when two or more monitors are made available.	
Show Label in DM Preview	Shows the previews of hanging protocols in the DM bar.	
Show Middle Slice Thumbnail	Shows only the thumbnail of the middle slice in a series on the thumbnail list.	
Enable Cardio Viewer	[Unused]	
Disable Queuing	Forces synchronous communication; client message and server response occur one at a time. CAUTION: We do not recommend changing this setting. Consult with your Konica- Minolta representative.	
Stretch Prior Bar	Extends the prior bar across multiple monitors. If the prior bar exceeds the total available screen width, a scroll bar appears.	
Show Prior Index	Displays the Prior index number in the thumbnail and Prior bars.	
Auto open STAT	When you open a study in the Exa PACS/RIS viewer, this option automatically opens any stat images, if included.	
Split W/L Region	[Unused]	
Color Only as Last Monitor	Chooses the color monitor as the last monitor on which to display images.	
Client-Side Window/Level for US	Processes the Window/Level for ultrasounds on the client side. May reduce lag by bypassing the server.	
Show Span Overlay	Shows a span sensitivity adjustment slider on images.	


See also:

Configure the toolbar

Configure annotation tool functionality

You can configure annotation tool functionality.

- 1. Go to <u>Viewer Settings</u> > <u>INTERFACE</u>.
- 2. Under **Annotation**, enter the settings in the table below.

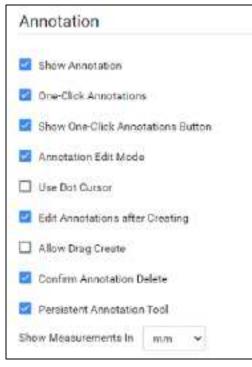

Setting	Description
Show Annotation	Shows annotations by default. The user can still hide them manually in the viewer.
One-Click Annotations	Changes from the selected annotation tool to the default cursor after each use. If cleared, the user can reuse the tool repeatedly without re-selecting the tool.
Show One-Click Annotations Button	Shows a button for turning one-click annotation on and off.
Annotation Edit Mode	Selects the Edit command in the image shortcut menu by default, to move, resize, or delete annotations.
Use Dot Cursor	Changes the standard arrow pointer to a dot pointer.

Setting	Description
Edit Annotations after Creating	Causes the viewer to enter Edit mode after initial placement of an annotation.
Allow Drag Create	Creates annotations by dragging instead of selecting.
Confirm Annotation Delete	Prompts the user for confirmation when deleting annotations.
Persistent Annotation Tool	Makes a selected annotation tool available until the user selects a different tool. If cleared, the pointer changes to the default operation after using the tool.
Show Measurements In	Sets the units for any length measurements taken.

Configure annotation colors

You can configure the color of annotations to indicate whether they are selected, in edit mode, or applied.

- 1. Go to <u>Viewer Settings</u> > <u>INTERFACE</u>.
- 2. Under **Colors**, to add an annotation color, select inside a color or grayscale box and then use the color picker to select a color.

Colors	
	Color Orayscale
Anno Color 1	#FFFF00
	#FFFFFF
Anno Color 2	1008F900
	#FFFFFF
Anno Color 3	
	#FFFFFF
Anno Color 4	100000
	#EFEFFF
Anno Color 5	#FPFFFF
	AFFFFF
Anno Color Edit	anning -
	AFFFFF
Anno Color Hover	MARGONAL
	MEFFEOD

Anno Color 1-5: Annotations appear in these colors by default.

Anno Color Edit: Annotations change to this color when you edit them.

Anno Color Hover: Annotations change to this color when you hover over them.

3. Select SAVE.

Configure annotation size and scaling

You can configure the pixel or font size of annotations, and scale annotation points.

- 1. Go to <u>Viewer Settings</u> > <u>INTERFACE</u>.
- 2. Under **Miscellaneous**, enter the settings in the table below.

Miscellaneou	IS
Scale in Pixel Size	
Scale Annotation Font By	2
Scale Annotation Point By	
Invert SR Reports	
De <mark>fault SR Report</mark> Font Size	•

Setting	Description
Scale in Pixel Size	Determines the annotation font size by the pixel size.
Scale Annotation Font By	Sets the font size for annotations.
	Scale Pixel Size ON : Default is approximately 10
	Scale Pixel Size OFF : Default is approximately 1 to 1.5
Scale Annotation Point By	Sets the size of annotation handles (from 0.1 to 5).
Invert SR Reports	If an SR appears within a cell, displays it in reverse video (such as white text on black background).
Default SR Report Font Size	Sets the default font size for text in SR reports that appear within a cell.

Configure recording

You can configure audio recording options for Exa Dictation.

- 1. Go to <u>Viewer Settings</u> > <u>INTERFACE</u>.
- 2. Under **Record Control**, enter the settings in the table below.

Recor	ding Control
Hold	for Record
🗆 Веер	on Record (heard on recording)
🗆 Blink	on Record Pause
🗆 Moni	key Chatter on Rewind
FFWD/R/	WD Factor (200-500)
Lag (0-5	000)

Setting	Description
Hold for Record	Requires you to hold down the button for the duration of recording.
Beep on Record	Beeps when recording starts. The beep may be audible on resultant recordings.
Blink on Record Pause	Causes the microphone light to blink when recording is paused.
Monkey Chatter on Rewind	Plays "monkey chatter" sound when rewinding.
FFWD/RWD Factor (200-500)	Changes the rate of fast forward and rewind, in milliseconds.
Lag (0–5000)	Delays the response after selecting play or record by the specified amount of time, in milliseconds.

Show or hide tool buttons

You can hide certain tool buttons.

- 1. Go to <u>Viewer Settings</u> > <u>INTERFACE</u>.
- 2. Under **Tool Buttons**, select the checkboxes of the items that you want to hide.

Tool Buttons		
Hide Document		
Hide Notes		
Hide Print		
Hide Reports		
Hide Transcription		

About ROI and SUV values and units

Exa PACS/RIS can calculate selected types of SUV values and display them in selected units. When using these functions, consider the following.

- DICOM data from Philips To calculate SUV values based on DICOM data from a Philips modality, if the data uses CNTS units, Exa PACS/RIS converts pixel values to BQML by using Philips private tags Scaling Factor [7053,xx00] and Activity Concentration Factor [7053,xx09]. However, if those tags are missing, the CNTS units are retained regardless of the SUV type you configure.
- If data required for calculations is not found in the DICOM source file, Exa PACS/RIS displays N/A in place of measurements.
- If ROI Selection Weighting is set to **Exclusive**, largest-valued or smallest-valued pixels falling partially outside an ROI are excluded from calculations.
- If DICOM-sending images with ROI annotations, they are automatically set to **Inclusive** regardless of the user selection.

See also:

Configure fusion, ROI, and SUV options

Configure fusion, ROI, and SUV options

You can configure options for fusion, ROI, and SUV.

- 1. Go to <u>Viewer Settings</u> > <u>INTERFACE</u>.
- 3. Under **Fusion**, and under **ROI & SUV**, enter the settings in the table below.

Fusion
Auto Adjust for CT/PT Frame Count Mismatch
ROI & SUV
ROI Show Size
Show Number of Pixels In ROI Selection
Show Number of Slices in ROI Selection
ROI Selection WEIGHTED V
Enable SUV Calculations
Show SUV Type and Measurements in the ROI Header
Show SUV Units after Measurements
Use Standard SUV Type Units (g/ml, om²/ml)
Default SUV Type SUVbw 🗸

Group	Setting	Description
Fusion	Auto Adjust for CT/PT Frame Count Mismatch	Select to reconstruct PET anatomy on fusion overlays between absent slices. Disable to display PET overlays only for slices originally included in the PT series.
ROI & SUV	ROI Show Size	Adds length and area measurements to ROI.
	Show Number of Pixels in ROI Selection	Select to show the number of pixels selected when annotating an ROI or selecting with the freehand tool.
	Show Number of Slices in ROI Selection	Select to show the number of slices selected when annotating an ROI or selecting with the freehand tool.
	ROI Selection Weighting	Select which pixels are included in an ROI. When DICOM-sending images with ROI annotations, they are sent as Inclusive regardless of this setting. Therefore the Min, Mean, StdDev, and Pixel count may differ slightly from what appears in Exa PACS/RIS with the Weighted or Exclusive options.
		Weighted : Include pixels touching or within the ROI border, and assign a value based on the percentage of the pixel lying within the ROI.

Group	Setting	Description
		Inclusive : Include pixels touching or within the ROI border, and use the full pixel value.
		Exclusive : Include only pixels completely within the ROI border, and use the full pixel value.
	Enable SUV Calculations	Select to add SUV calculations to ROI annotations of PET SUV series and CT-PET fusion studies. Prerequisite: Your system administrator must also enable SUV in a separate procedure.
	Show SUV Type and Measurements in the ROI Header	Select to include the SUV type and measurements in the ROI header.
	Show SUV Units after Measurements	Select to include the SUV units following the measured values in the ROI header.
	Use Standard SUV Type Units (g/ml, cm2/ml)	Select to display SUV values using standard units for the SUV type. Clear to use the values stored in the DICOM file.
	Default SUV Type	Select the type of SUV calculation to perform.

See also:

About ROI and SUV values and units

Configure modality-specific viewing options

You can configure how studies appear in the viewer for all modalities, or for each specific modality.

Procedure

1. Go to <u>Viewer Settings</u> > **MODALITY**.

FITINGS					LANS.
					2075
UENERAL	- 44	Layout		Mouse Butte	ona
	.90	10			MP-
INTERFACE:	-DB	Screen Layour	111	Eath	None 🐱
	57	10000000	(1)	Sagin	Sterie 👻
MOCALITY	MG	Serves Layout	141.392	1.5	Signa 👻
a descentition of the second sec	MR			Nidele	None 🛩
	NM	Compression			
MANIMOGRAPHIC	101			Loft+Right	Nore 🛩
	1.5	Nor+ Dispensio	*		
OVERLAVO	100				
	100	Presets			
	D04	Presets			
TOOLS	ΨT		and an entropy	Window Witten	Window Center
	11		Key Decorption	WEDAK PRET	Window Center
DITERNAL TOOLS	80	0			80
	XX				
	3F				
	τQ				
	.46	Options			
	58	The state initian	9		

- 2. In the list of modalities, select a modality to configure.
- 3. Enter the settings in the following table.

Section	Setting	Description
Layout	Screen Layout	Changes the number and layout of series frames.
	Series Layout	Changes the number and layout of image frames within series frames.
Mouse Buttons	Left/Right/Middle/Left+Right	Assigns functions to mouse buttons. This is frequently used by CAD users to assign functions to extra mouse buttons.
Compression		Sets the image quality, which inversely affects viewer performance.
Presets	Кеу	Assigns shortcut keys to W/L presets.
	Description	Type a name for the W/L preset.
	Window Width	Sets the window width of the W/L preset.
	Window Center	Sets the window center of the W/L preset.
	Set	Select when finished entering Preset settings.
Options	Auto Linking	Automatically numerically links the current series projection (Sagittal, Coronal, and Axial) with all other like series projections in that study.
	Assume Color Study	Forces studies of the selected modality to appear in the default color monitor set in the

Section	Setting	Description
		display settings regardless of color.
	Assure All Viewed	Displays a check mark in thumbnails of images that were opened, and warns the user when closing before all images are opened.
	Auto CLAHE	Automatically applies CLAHE, enhancing image contrast.
	Auto Hang DM	Enables the use of DMs. Makes the first instance on the DM preview available to hang on the viewer.
	Auto DM Wrap	Enables moving from the final step of a DM directly to the first step, and back again.
	Auto Hang Priors	Automatically loads the first prior when the viewer opens. Works in conjunction with Auto Open Priors.
	Auto Replace Priors	Automatically replaces prior images in the current layout.
	Auto-Invert SC Images	Inverts white and black of scanned documents such as prior reports. Reduces eye fatigue.
	Auto Next Series	When moving beyond the last image of the current series, automatically opens the next series.
	Auto Next Series Cine	Automatically starts cine play of the next series when the user opens it.
	Auto Next Series Wrap	Automatically opens the first series after viewing the last image of the last series in a study.
	Auto Next Study	Automatically opens the next study in the worklist after closing the current study.
	Auto Swap Red/Blue	Inverts colors that may not be displayed from the modality properly (such as with non-DICOM ultrasounds).
	Auto Show DICOM Overlays	Automatically displays DICOM overlays, if present.
	Auto Show SR Overlays	Automatically displays SR overlays, if present.
	Auto Show Cutlines	Automatically displays cutlines, if present. This setting also applies to synthesized mammogram views, in which the cutlines indicate the position of the currently viewed image on the corresponding BTO view.
	Auto Show Cine	Shows the media control bar when a series is opened.
	Auto Show W/L Bar	Shows the window/level bar when the viewer opens.
	Detect Pixel Padding	Turns detection of pixel padding ON/OFF.
	DM Allow Missing Cell	Shows a blank cell in the DM if the assigned image is not available.

Section	Setting	Description
	DM Save W/L	Saves the window/level with the DM.
	Save Study DM on Close	Creates a new DM for each study when it closes, or when the next study auto-opens. The DM is named with the accession number, and contains the last displayed stack and monitor position.
	Show PDF in Last Stack	Moves PDF files to the last place in the stack. In the viewer, they are loaded last if an empty cell is available.
	Instance Window/Level	Select to show each image with its own W/L. Clear to show each image with the W/L of the first instance.
	Do Not Skip Images	Prevents moving forward if subsequent images are not yet loaded.
	Ignore Frame of Reference within Study	Disables referencing of the study's DICOM frame of reference UID when performing linking and cutline functionality.
	Ignore LUT	If LUT values are corrupted, Exa PACS/RIS ignores the LUT values, and attempts to render the best image possible.
	Ignore Presentation LUT	Same as Ignore LUT, but select for certain vendors that use a "Presentation LUT."
	Keep Rotate	Applies the current rotation to all images in the series.
	Keep W/L	Applies the current W/L to all images in the series.
	Keep Zoom	Applies the current zoom level to all images in the series.
	Extend Image Display (if Stretch)	When Stretch Across Monitors is selected, extends a single image across multiple monitors.
	Pixel Padding as Background	Interpolates missing pixel data to fill in "dead" spots in an image.
		Note : When using this feature, reading physicians should be aware that some pixels may be synthesized.
	Pre-generate Bitmaps	Pre-generates bitmaps automatically if imported into PACS.
	Reset W/L for Individual Images	Select to use the original W/L of each individual image.
		Clear to use the first W/L sent by the modality for all images.
		Frequently used to optimize MRI images.
	Select Last Contrast Entry	When DICOM tags include multiple W/L values, uses the last value for all images in the series. Clear to use the first value.

Section	Setting	Description	
	Show 3D Spine Labels	Shows the 3D spine label tool on the toolbar.	
	Show 'Bone Enhance' on Toolbar	Shows the Bone Enhance (sharp mask) tool on the toolbar.	
	Show DM Toolbar	Shows/hides the DM toolbar on initial load.	
	Show Spine Labels	Shows the spine labels tool on the toolbar.	
	Sort Thumbnails by Date/Time	Sorts thumbnails by date/time of acquisition.	
	Stop Thumbnail from Updating	Prevents W/L changes to the current image from affecting its thumbnail.	
	Stretch Across Monitors	Turns on extending of images across all active monitors.	
	Cine Direction	Sets the cine playback to forward or backward.	
	Default FPS	Causes cine play to occur at the default frames per second, depending on modality.	
	Enable 4DM	Opens the 4DM viewer for post-processing. (Requires configuration to integrate with the 4DM viewer.)	
	Disable CINE Scrolling	Disables cine play of series.	
	Active Frame Based Cineplay	Playing a cine loop on one series automatically plays all linked series.	
	Disable Caliper on Cine	Hides calipers during cine playback.	
Auto-Split Rules	Always	Splits US series into individual images.	
	Differing Echo Time	Splits by echo time.	
	Differing Series Number	Splits by series number.	
	Differing Series Time	Splits by series time.	
	Differing Acquisition Number	Splits by acquisition number to create stacks for separate acquisitions/scans.	
	Differing Diffusion Value	Splits images within similar series UID by b- value (0018,9087 MR Diffusion Value) into separate series.	
	Differing Trigger Time	Splits by trigger time (0018,1060).	
	Isolate MPEG	Splits by MPEG file.	
	Isolate Multiframe	Splits cine loops into multiple images.	
Auto Bone Enhance		Highlights the bone portions of images by the percentage you type.	
Auto WL Type		Selects which W/L to use (such as when values are not available from the modality).	
		Normal : Use the W/L tags from the modality, if available.	
		Raw Img : Ignore W/L tags and allow the viewer to auto-contrast the image.	
		Full: Allows full range of W/L by bit depth:	

Section	Setting	Description
		8 bit = 256, 10 bit = 1024, 12 bit = 4096
Auto Reorder Images		Automatically changes the order of images in a series to the selected order.
		Image Number : Displays images by DICOM instance number, lowest to highest.
		Image Number Reverse : Displays images by DICOM instance number, highest to lowest.
		Slice Location : Displays images by DICOM slice location, lowest to highest.
		Slice Location Reverse : Displays images by DICOM slice location, highest to lowest.
		Image Time : Displays images by DICOM acquisition time, lowest to highest.
Prior Options	Auto Open Prior	Automatically opens prior studies.
	Relevant Priors	When automatically opening priors, also opens relevant priors from other modalities. See "Define relevant priors" later in this topic.
	Auto Open Prior Count	When automatically opening priors, opens the selected number of studies.
	Force Relevant Only	Opens only the defined relevant priors (see "Define relevant priors" later in this topic).
	Prioritize Current Modality First	When opening relevant priors, display same- modality priors first.

5. Optional. If you selected Relevant Priors, continue to "Define relevant priors."

Define relevant priors

Defining relevant priors allows you to auto-open only priors that are relevant to the current study based on study description, modality, or body part. To write definitions for which priors are relevant you use a syntax called *relevant prior logic*:

STRING or (STRING|STRING|...){.*(STRING|STRING|...)}

"A single string, or multiple strings delimited with a bar, optionally followed by ".*" plus multiple strings delimited with a bar."

Example 1

When the main study description contains "CHEST" or "THORAX," the relevant priors to auto-open are CT studies whose descriptions also contain "CHEST" or "THORAX."

	o Open Prilor					
	icoant Priora en Pika Goun		Port Day	n Rolevart.	C Prioritize Curr	rant Modality
Modality	Bolly Pert	Description	Main Study Body (Part	Main Diudy Dess		

Prior settings on the left...

...main study settings on the right

Example 2

- When the main study description contains "CLAVICLE," SCAPULA," or "SHOULDER" followed by "LEFT," "LT," or "BI," the relevant priors to auto-open are those whose descriptions also match those criteria.
- The same is true for the **RIGHT** shoulder.
- Priors of the same **modality** as the main study are opened first.

🖬 440	Opun Prim						
ALES CO	levant Prians en Pelor Court		<mark>ca</mark> For Only	e Relevant	📒 Prinsitiae Current M Firet	odelity	
santality	fiedy Part	Description	Moin Study Budy Part	Main Study Des	ergalen	ľ	
58x5dity	Stody Part	Description (TEAV/REDGAPIEASHOREDER) * (LEFTETRI)	Baudy		eription PLLASHOULDER().*	R	

Note: You can also define relevant priors to auto-send (see <u>Configure routing rules</u>).

Configure mammography

You can configure mammography viewing options.

Procedure

F.

1. Go to <u>Viewer Settings</u> > **MAMMOGRAPHY**.

ETTING5		
GENERAL	Mammography	SR CAD
INTERFACE	Use 2+ Vear-Old Priore Stack Some View	Breast Density Breast Becimienty
MODALITY	Auto Orient, Memmo Views	Catofication Clusters Individual Calorication
MAMMOGRAPHY	Seriap LaftyRight Breast Positions (with Auto Onerti Only) Remove BTO Projections	Neple Male
OVERLAYS	Allow Wammo Switching via Overlay Shuw BTO Orientation Overlay	Apply Changes to All Images Enable CAD Findings on ALL, BTO Images
TOOLS	 Show Quick Maron Unit US Broast Prices shar MG Prices 	
EKTERNAL TOOLS	Enable CM Satel Enable State Scrutting	
	Chaicle BTO Sciolling	
	Geoup DM Priorie by Prior Show Priora in Chronological Order Actor State Transis None	

2. Enter the settings in the following table.

Section	Setting	Description
Mammography	Use 2+ Year-Old Priors	Makes available the display of priors that are 2 years old or more.
	Stack Same View	When Auto Next Series is turned on, stacking of 100% matches for a cell becomes available in the Display Manager.
	Auto Orient Mammo Views	Automatically orients mammo views.
	Swap Left/Right Breast Positions (with AutoOrient Only)	When Auto Orient Mammo Views is selected, switches the left and right breast images.
	Remove BTO Projections	Hides the BTO orientation overlay.
	Allow Mammo Switching via Overlay	When selected, the user can click the series name in the top of the cell to switch between 2D and 3D.
	Show BTO Orientation Overlay	Shows a bar in lower left indicating the cranial or caudal location in the breast when moving through a 3D image.
	Show Quick History	Show the Quick History of studies.
	List US Breast Priors after MG Priors	Orders priors in the prior bar.
	Enable DM Scroll	Turns on moving through DMs in the DM toolbar by using the mouse wheel.

Section	Setting	Description
	Enable Stack Scrolling	In special cases (such as BTO) multiple images are present in the same series, which is received as a single DICOM. In these cases, turn on the stack scroll.
	Disable BTO Scrolling	Turns off moving through BTO views.
	Group DM Preview by Prior	Saves space on the prior bar by stacking DMs with the same prior logic. You can hover over a prior to display matching priors in a vertical pop-out.
	Show Priors in Chronological Order	Orders priors in the prior bar.
	Auto Slab Tomo	Automatically applies a slab thickness to tomosynthesis studies.
	Stack By View And Prior Sort By	Makes it possible to move through prior mammo images within a hanging protocol without needing a specific DM. Select Asc or Desc to move by age of prior.
	Enable SkinLine	Maximizes breast anatomy based on the skin line without degrading image quality.
SR CAD Options	[Select]	Sets which CAD overlays (data) are displayed in the viewer, if present.
	Apply Changes to All Images	Shows all available SR CADS in the images.
		Malc - Select to view Malc findings when viewing Hologic studies.
	Enable CAD Findings on ALL BTO Images	Displays all findings within the series on all images. CAD findings on non-referenced images are 67% transparent.

Configure overlays

You can configure the appearance and contents of overlays.

Procedure

1. Go to <u>Viewer Settings</u> > **OVERLAYS**.

SETTINGS								
								SAVE
REMERAL	PoinTandy	Ártaí w	Size	10	٠			
INTERFACE	Madelle	Al +						
MODALITY	Left Overk	ay				Right Overl	ay	
MANINGGRAPTN	neki					heid		
OVERLAYS	Prefix	Mask		- 40		Partix	Mack	1.400
And and a	× institutio	n(0008,0080)			1ñ	$ imes$ image_inst	tanceindes	1
70068	× institutio	a,addr(0000,0001)				× StudyID_di	som_study_ici(0000.00	10
	× m.dicom	.patient_1d(0010.00	20)			× Spaning_s	nacing_hetween_slices	0018.00880
EXTERNAL TODED	× Name,pa	cient_nome(0010,00	10)			× AN, arrost	inn_no(0008,0050)	
10021	× DOB_bird	h_date(0010,0030)			1	× Rip, Rip, a	ngle(0018,1314)	
	× sex_puti	est_ses(0010,0040)			17	× Loc, size)	location(00/20,1041)	
	× III study	+inono altora	. A.			X This siles	thislese among a dosion	

2. Enter the settings in the following table.

Section	Setting	Description				
	Font Family	Select the font used for overlays.				
	Size	Select the font size used for overlays.				
	Modality	Select to which modality the overlay settings apply. You can enter modality-specific settings or select all modalities.				
Left/Right	Field	Click inside the box and then:				
Overlay		Select a property from the list, or				
		• Type a DICOM tag (e.g. 0020,0010)				
	Prefix	If you typed a DICOM tag in the Field box, type a tag prefix. You can use an intuitive prefix such as Series #, DOB, or Time.				
	Mask	Optional. Type attributes that describe mask operations for a multi- frame image (see the DICOM standard).				
Bottom Overlay	FOV/MAG/W/L	Displays the field of view, magnification, and W/L for each series in the bottom corner.				

3. Select ADD

Result: The tag appears at the bottom of the list.

- 4. Optional. Drag the tag to a new position in the list.
- 5. Select **SAVE**.

Add or remove toolbar tools


You can add or remove tools from various viewer toolbars in the program, and customize your choices by modality.



Note: Not all toolbar tools can be removed.

Procedure

- 1. Go to <u>Viewer Settings</u> > <u>TOOLS</u>.
- 2. Under **Toolbar**, select the modality whose toolbar you want to customize.

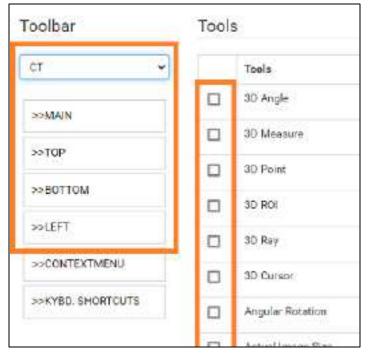

IMPORTANT: The Toolbar **All** setting only defines the default tool settings. If any modalityspecific tools were configured, they override the default. Therefore we recommend configuring each modality you plan to use (such as "CT" in the figure above).

3. In the list, select one of the following toolbars to customize.

MAIN – The toolbar at the top of the viewer.

TOP/BOTTOM/LEFT – The toolbar accessible by pointing to the top, bottom, or left edge of an image.

- 4. Under **Tools**, select or clear the checkboxes of the tools that you want to add or remove.
- 5. Select **SAVE.**

See also:

Add or remove tools from the image shortcut menu

Assign toolbar keyboard shortcuts Assign keyboard shortcuts for Exa Trans Viewer tools and keyboard shortcuts

Add or remove tools from the image shortcut menu

You can add or remove tools from the shortcut menu (context menu) that appears when you right-click an image, and customize your choices by modality.

Procedure

- 1. Go to <u>Viewer Settings</u> > <u>TOOLS</u>.
- 2. Under Toolbar, select the modality whose shortcut menu you want to customize.

oolbar	Tool	3
ct 👻		Toola
>>MAIN		3D Angle
		30 Measure
>>TOP		3D Point
SOBOTTOM		30 R01
>>LEFT	0	3D Rey
>>CONTEXTMENU	0	30 Carsor
>>Kybd. Shortcuts	0	Angular Rotation
		Artiss Imana Size

- 3. In the list, select **CONTEXT MENU**.
- 4. Under **Tools**, select or clear the checkboxes for tools that you want to add or remove.
- 5. Select SAVE.

See also:

Add or remove toolbar tools Assign toolbar keyboard shortcuts Assign keyboard shortcuts for Exa Trans Viewer tools and keyboard shortcuts

Assign toolbar keyboard shortcuts

You can assign keyboard shortcuts ("hot keys") to tools, and customize shortcuts by modality.

Procedure

- 1. Go to <u>Viewer Settings</u> > <u>TOOLS</u>.
- 2. Under **Toolbar**, select the modality for which you want to assign shortcuts.

Toolbar	Tools	
ल	-	Tools
396746	0	10 Andis
23200312 23200312	- D.	20 Massaure
NOTEC	0	SC Plaint
39700TT0M		80-808
29/EA1	0	IC Ray
>>CONTEXTIVENU	0	10 Q.r. Kar
PARYED SHOWTCUTS	0	Jeigutze Relation
20-EXA TRANS SHORTCUTS	D	Actual image Bize
		Deale Teaching Budy
		Made Marker

- 3. In the list, select **KEYBOARD SHORTCUTS**.
- 4. Under **Tools**, select the checkboxes of the tools to which you want to assign shortcuts.
- 5. In the **Shortcut** box, type the keyboard shortcut. The shortcut can be one of the following, either by itself or preceded by the **Ctrl** key:
 - Alphanumeric character (A–Z, a–z, 0–9)
 - Arrow key (**Up**, **Down**, **Left**, or **Right**)
 - Basic math operator (+, -, *, /)
- 6. Select SAVE.

See also:

Add or remove toolbar tools Add or remove tools from the image shortcut menu Assign keyboard shortcuts for Exa Trans Viewer tools and keyboard shortcuts

Assign keyboard shortcuts for Exa Trans

You can assign keyboard shortcuts ("hot keys") to common functions in Exa Trans. To use a keyboard shortcut that you assign here, Exa Trans must be the active application. For built-in keyboard shortcuts, see <u>Keyboard shortcuts and commands for Exa Trans</u>.

Procedure

- 1. Go to <u>Viewer Settings</u> > <u>TOOLS</u>.
- 2. Under **Toolbar**, In the list, select **EXA TRANS SHORTCUTS**.

er 🔸		Toels	Shortest
>INSUN		approve	
>v70P		Approve and heat	
>>BOTTOM		Pre-Approve	
NISET		Seve.	0
		15 kp	1
WKYBD SHORTOUTS	-		

- 3. Under **Tools**, select the checkboxes of the tools to which you want to assign shortcuts.
- 4. In the **Shortcut** box, type the keyboard shortcut. The shortcut can be one of the following, either by itself or preceded by the **Ctrl** key:
 - Alphanumeric character (A–Z, a–z, 0–9)
 - Arrow key (**Up**, **Down**, **Left**, or **Right**)
 - Basic math operator (+, -, *, /)
- 5. Select **SAVE.**

View logs

To view the various logs that are available in Exa PACS/RIS, do the following.


- 1. Go to $\underline{\text{SETUP}} > \underline{\text{OFFICE}} > \textbf{Log}$.
- 2. Select tabs (AUDIT LOG, USER LOG, etc.) to view different logs.

-	1001							((e)
	MUNTLOG	OBILIDE	ATTEC ACCELLINGTON	DEDIN FLATRE 100	MONE REPORTED OF	- I OWGE HEREBOULDEDE	HOPEOC	18
Freem	10/0/00 1200 M	MES E	Ta OLITARZUELIKS	NINFARD E		Sector Provide	The second se	
	LOUID DOLL	ACCESSION N	up.	WIEW.	- Coastanciary	ol		
	40/14/1828 /8-45 and 60		Springer (Park)		tizz (Kambu	nineed tweeto-tim ru	Ans (emiliary))	st/fe
6 m	BUTHERID IN AD ANY DO	a	darlers (KAN)		Devision	estated		
0	10-14-1820 (6-41-444 D)	er :	Cyclem (Div)		Dier protoider	n versplanet, immerica (160), inte	dia amina (2).	na

3. Double-click entries to view details.

Work with the hamburger menu

The hamburger (burger) menu is the main menu of the program. To open it, select the burger menu button circled in red in the following figure. Some of the options in the figure may not be available depending on your product and configuration.

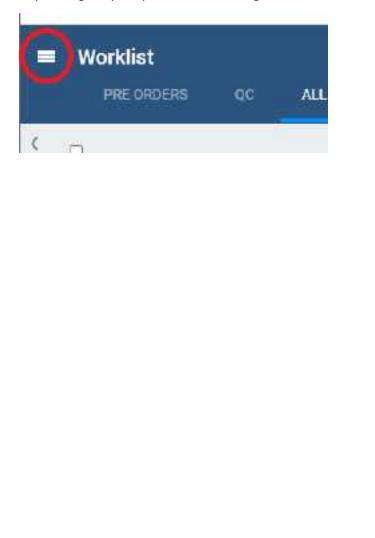

B WORKLIST 盲 PATIENT ANNVAL WURKLIST ۲ PATHENT DISPATCHING DAUHUGARD Q QUERT/RETRIEVE BICHEDULE 目 **HILLING** PATHAINTE > Ö REPORTS 4 > TOILE 11 DICTUP INTERACE Shiph

...and menu (your options may vary)

The burger menu button...

Work with the dashboard

What is the Dashboard?

The Dashboard is a page that displays gadgets, which contain information in graph, table, and other formats to help you understand the status of work at your facility. The Dashboard is separate from the Dispatching Dashboard, which is dedicated to <u>Mobile Radiology</u>.

This section contains the following topics (not all topics may be available depending on version and region).

<u>Open the dashboard</u> <u>Add or remove gadgets from the dashboard</u> <u>Change the graph type of a gadget</u> <u>Change the facility of a graph</u> <u>Change the date range of a graph</u>

Open the dashboard

• On the burger 🔲 menu, select **DASHBOARD**.

Add or remove gadgets from the dashboard

The dashboard displays an array of user-customizable *gadgets*, and a gadget with a button for adding more gadgets.

Procedure

- 1. On the burger menu, select **DASHBOARD**.
- 2. To remove a gadget, select its remove X button.
- **3.** To add a gadget, find **Add Gadget** and select its add ^O button.
- **4.** In the list of gadget types, select a type.

Change the graph type of a gadget

You can select a graph type of bar, column, line, or pie.

- 1. On the burger 🔲 menu, select DASHBOARD.
- **2.** On a gadget, select the graph **button**.
- **3.** In the button shortcut menu, select a type.

Change the facility of a graph

You can change the source facility of the graph in a gadget. The available facilities are ones that you set up in advance in Exa PACS/RIS.

Procedure

- **1.** On the burger menu, select **DASHBOARD**.
- **2.** On a gadget, select the facility **B** button.
- 3. In the button shortcut menu, select a facility or select All Facilities.

Change the date range of a graph

You can change the date range of the data source of the graph in a gadget.

- 1. On the burger menu, select DASHBOARD.
- 2. On a gadget, select the date range 🛅 button.
- **3.** In the button shortcut menu, select a date range and then select **Apply**.

Work with the worklist

The worklist is the starting point for most operations in Exa PACS/RIS. The worklist is a searchable and highly customizable list of patients, studies, and series, providing full access to demographics and images.

Procedure

1. Go to <u>burger menu</u> > <u>WORKLIST</u>.

	•	lorklis (mile		5			iii		MI	ទាបថ	B		STAT 1. 2	0000		00 79.9
4												BTATUS	STUDY BATE :	INTERT	MobA	STUDY DEEK
	-											_			lla	
+	1 () 1	e + 2	•	Ξ	ш	3		1		0	Ŀ	Approximit	01/10/2003 0:30 PM PST	Test Helen	34	ANGREE A
÷	a (1)	2	*	ġ.	=	13	8	2		۲	ŧ	Approved.	01/18/2023 7:15 PM PST	Test Tom	CR	CHOLANS
+	0 🗆	0	+1		m	3	q.	zî.		•	÷	Approvent	01/18/2021 7:00 PM PST	Test Tom	CR	BRONCHO

See also in this chapter:

Worklist settings User settings Viewer settings Local service settings Use the worklist PACS Actions menu Worklist shortcut menus Edit Study screen Exam screen for technologists

<u>QC operations</u>

Worklist settings

This section contains the following topics (not all topics may be available depending on version and region).

About toolbars and elements in the worklist Display a filter of the worklist Create a user worklist filter Edit or delete a worklist filter Show or hide a worklist filter Select a quick filter of the pre orders tab

About toolbars and elements in the worklist

The worklist provides the following toolbars and other items. Notice that there is a settings menu with button in both the upper and lower toolbar, but they open different menus.

Upper toolbar

TASKS	<mark>o stat</mark>	0 0 0 0 0 0 0 0 TAT 0 0 0 34	60
1	2	3	4 5

The upper toolbar contains the following elements, from left to right.

1	Tasks	Shows the number of tasks by urgency. Select to jump to the Tasks list.
2	STAT meter	Shows the number of studies at each stat (urgency) level.
3	TAT meter	Shows the number of studies at each TAT (turnaround time).
4	Settings	Opens the settings menu.
5	Log off	Signs the current user out.

Lower toolbar



The lower toolbar contains the following elements.

1	Move tabs left/right	Left/right justifies the worklist filter tabs.
2	Filter menu	Select from any available filter tab.
3	Refresh	Updates the worklist, but keeps manually entered filters.

- 4 **Refresh all** Updates the worklist, clearing all filters.
 - PACS Actions Opens the PACS Actions menu.
- 6 Filters Opens the Study Filters dialog.
- 7 Search tools Shows/hides the search bar.
- 8 Disagreements Opens the disagreements queue (requires a separate license for Peer Review).

Filters

5



Filter tabs (figure above) appear on the left side of the worklist by their labels. The Exceeds Maximum Time Limit filter (figure below) appears at the bottom of the worklist.



The following table describes the four types of filter tab (filters) that are available in the worklist.

Туре	Name	Description				
		Lists unscheduled orders. Pre orders often have no				
(Can be hidden)		referring physician, and can come from Exa RIS or portals (such as exams requested by the patient).				
	QC	Lists received studies needing reconciliation.				
		See <u>QC operations</u> .				
Permanent ALL STUDIES		Lists all DICOM studies and scheduled RIS orders that are				
(Cannot be hidden)		not currently on the QC tab.				
User Filters	[user-defined]	Provide a highly customized view of the worklist.				
(Can be hidden)		See <u>Create a user worklist filter</u> .				
Administered worklist filters	[administrator- defined]	Same as a user filter, but created by an administrator for a user, and cannot be changed by the user.				
		See Configure an administered worklist filter.				
Temporary	Exceeds Maximum Time Limit	When you select the Exceeds Maximum Time Limit link at the bottom of the worklist, this tab opens and lists relevant studies. Select the pushpin button on the tab to display a dialog with options, including "Remove."				

Display a filter of the worklist

You can display optional filters (such as PRE ORDERS or QC) in the following two ways. To display user filters, you can use these steps if the filter was configured to appear as a tab or in the tabs list.

Procedure

1. On the worklist, in the filter bar, select a tab to display the filter.



2. In the lower toolbar, select the **Show all Tabs by list** \ge button, and then select a filter.



Create a user worklist filter

Exa PACS/RIS comes with the PRE ORDERS, QC, and other filters, but you can also create your own filters to customize the worklist at a high level of precision.

- 1. On the worklist <u>lower toolbar</u>, select the filter worklist.
- 2. In the **Study Filter** dialog, select **ADD**.
- 3. Enter the following criteria. See an **example** below the table.

Setting	Description
Filter Name	Type a name for the filter
Show Encounters Only	Shows only studies for outpatient encounters.
Joined Filters	Selects another filter to combine with the current filter. Studies must meet the criteria in both filters to appear in the worklist.
Filter Order	Determines the order in which filters appear as tabs or list items, from low to high.
Is Private	Restricts availability of the filter to the user who created it.
Display as a Tab	Shows the tab for the filter on the worklist.
Display in Dropdown	Shows the filter in the Show Tabs by List list on the worklist.
Show Only Exceeding Max Time	Shows only studies that are exceeding their max time as specified in "Configure study statuses."
Show DICOM Studies Only	Shows only DICOM studies (studies containing images).
Show RIS Orders Only	Shows only RIS orders.
Show Assigned Studies Only	Shows only studies that were assigned to the current user.

Setting	Description
Show Pre-Orders Only	Shows only preorders.
Deleted	Shows all (deleted and non-deleted) studies, no deleted studies, or only deleted studies.
Assign	Available when you select Is Private. Administrators can use these settings to assign the filter to specific users or user groups.
Default Column/Sort By	Sorts the worklist by the column and order that you select.
DATE/TIME	Filters studies by a range of dates and times.
PATIENT INFORMATION	Filters studies by patient or account number.
STUDY INFORMATION	Filters studies by institution, facility, modality, body part, and other categories.
RESOURCE	Filters studies by physician or attorney.
INSURANCE	Filters studies by insurance provider.

Note: When available, you can use the **Is** operator and **Blank** criteria to search for studies that are missing information. For example, in the **INSURANCE** category, under **Insurance Provider**, select **Is**, and then select **Blank** to search for studies with no insurance.

Example

The settings in the figure below define a filter with the following properties:

- Appears as the **first** available user filter
- Is **Private** (only available to the user who created it)
- Appears as a **tab** on the worklist
- Appears as a **menu** option in the list of filters
- Shows only **DICOM studies**
- Is only available to users of type Radiologist
- Only displays studies in Unread status

Study Filter									
(1997) 13	-	1.22					SAVE	CLENH.	BALR
Piller Name 1	Should,		Show Derma	ders Gely					
Juised Filter	s NON	IESELECTED+							
Filter Order *	1		🖸 ta Pa	rale					
🖬 Display a	s a Tal:	🖬 Oligiley in Dr	oodowe 🗆	Shin Drive	aceeding Max Time	8			
	2012 Studies Only	🛄 1940 70.0	der Dely 🖸	Ohon Assign	est Durites Only				
Deleted	-1		* 0	fine Per 0	data Only				
Assign	Select Univ.	<u>e</u> + a	fadiologint	• • +					
			adobaist -						
Default Colum	an/Sort By		1		*				
elocita regeli									
	Institution				Feelity				
E GATI/10.1	O is			-		Anale	Facility		
- Remark	O INNO.				O in O to Not	Gabiks	Tentracity Facility	- 1	
" INFORMATION	O Containe			1	Contract		mpany	+	
					Ordering Faci	ing .			
NYORMATION	Medality					Orter	arderes fai	Rice + +	
0 15506.005		BD		1		-			
	O Is O Is Not	BN DR		ñ	O in				
a Hasiliyada		<u>cr</u>			O ishot				
	Medašty Reom								
	O IN	Arigia Concuter Tomi		÷.					
	O Is Not	80		Ū	Status				
		<u><u> </u></u>				Elizati	Charged By	il.	
	Last call rate g	and the second second			(B) (c)	Tachi			
		Followup			Q falint	Tacht			
	O In O is Not	Other Ro-schodute				1000	al.	×	
						1 Middler			

Edit or delete a worklist filter

You can edit or delete a filter that you created.

1. On the worklist <u>lower toolbar</u>, select the filter worklist.

52002 CT 2010 CT 2010					AUD	RELOAL
Default Tab	All Studies	v				
HOTERMAME			PATER TYPE		FILTER DA	
			All	×		
All Studies			Global		0	
2 III Delete or Edit Me			Private		2	

2. Select the edit \checkmark or delete \blacksquare button of the filter to delete.

Show or hide a worklist filter

You can show filters as tabs on the worklist, as items in the filter menu, as both, or as neither (you can hide them for later use without deleting them).

Procedure

- 1. On the worklist, on the lower toolbar, select the filter worklist.
- 2. In the **Study Filter** dialog, double-click the filter to show or hide.
- 3. In the second **Study Filter** dialog, select or clear the **Display as a Tab** checkbox and the **Display in Dropdown** checkbox.

Select a quick filter of the pre orders tab

At the bottom of the PRE ORDERS tab, you can select from the following checkboxes to quickly filter the list of preorders.

□ / * ★ 03/17/2029.8:20 AM PDT 0.0~	Nimila Test Facility	Tast, Martin	CR	BONE AG
	i Nimila Test Facility	Test, Martin	TA	30 RNDR
□ × ★ # 03/14/2023 11 06 AM POT 2004	Numilla Test Facility	Test, Justin	XA	ANGRPH.

Quick Filter Checkbox	Description
Waiting for Authorization	Shows only unauthorized preorders.
Show Only Expired	Shows only preorders that are past the authorization date.
Show Only Physician Orders	Show only preorders created from Physician Portal.
Show Only Ordering Facility Orders	Show only preorders created from the ordering facility portal.

User settings

User settings (from the worklist) control how the worklist appears and functions. The settings only apply for the user who configured them, so all users can have their own unique settings. User settings override global settings, except for security settings.

This section contains the following topics (not all topics may be available depending on version and region).

<u>Configure worklist columns</u> <u>Configure other user settings</u>

Configure worklist columns

You can show, hide, and order columns in the worklist.

Procedure

1. Go to <u>Worklist</u> > **Settings** > <u>User Settings</u>.

User Settings						
Column Order	User Set					
🛃 Status	Show 1					
Study Date	Show 1					
E Facility	Show I					
Patient	She					
🔄 Modality	Auto-0					
Study Description	Auto-0					
C Account No.						
122	1					

- 2. Do one or more of the following.
 - To show or hide a column, select or clear its checkbox.
 - To reorder a column, drag the column to a new position in the list.



Note: While viewing the worklist, you can also drag columns directly to new positions.


3. Select SAVE.

Configure other user settings

You can customize behaviors and settings for worklist tabs, columns and rows, and configure other user settings.

Procedure

1. Go to <u>Worklist</u> > **Settings** > <u>User Settings</u>.

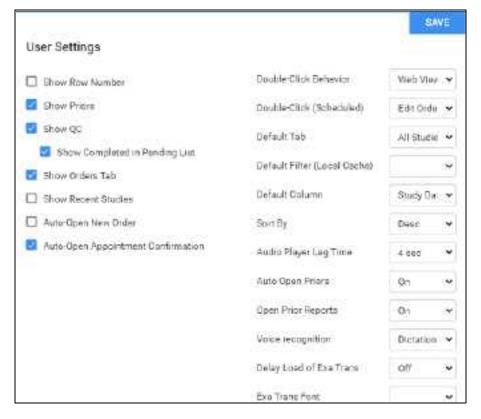

2. Enter the following settings.

Setting	Description
Show Row Number	Select to display the row number column in the worklist. The column is labeled as "#" or "No."
Show Priors	Select to show prior studies on the worklist, or clear to show series and images instead. Whether you choose to show priors or series, you can view them by selecting the expand for collapse button in a study row.
Show QC	Select to show the QC tab, and select the Show Completed in Pending List checkbox to show include completed orders in the QC tab.
Show Orders Tab	Select to show the PRE ORDERS tab in the worklist.

Setting	Description
Show Recent Studies	Select to show the RECENT STUDIES tab in the worklist. This filter contains the 25 most recent studies that were opened in the viewer.
Auto-Open New Order	Select to automatically open the Edit Study screen after selecting the CREATE ORDER button (Exa PACS/RIS only).
Auto-Open Appointment Confirmation	Select to automatically open the appointment confirmation screen after selecting the CREATE ORDER button (Exa PACS/RIS only). See also <u>Confirm an appointment</u> .
Double-Click Behavior	The view or app in which a study appears when double-clicking a row, unless the study is scheduled.
Double-Click (Scheduled)	The view in which a scheduled study appears when you double-click it.
Default Tab	The default tab for the worklist (the filter that initially appears when you open the worklist).
Default Filter (Local Cache)	The default filter for local cache. If local cache is installed and configured, studies in this filter are locally cached per settings.
Default Column	The default column for the worklist that determines the sort.
Sort By	Whether the list is sorted in ascending or descending order.
Audio Player Lag Time	The amount of delay before starting audio playback.
Auto Open Priors	Turns automatic opening of priors in the viewer on and off.
Open Prior Reports	Automatically opens prior reports when opening a study.
Voice recognition	When opening a DICOM study in the Exa PACS/RIS viewer:
	Dragon (Exa Trans) : Opens Dragon Naturally Speaking for radiologist transcriptions rather than Exa Dictation. You must install Exa Trans on the local workstation to use this option.
	Dictation (Web Trans): Opens Exa Dictation.
	Other (Exa Trans/SDE) : Opens Exa Trans with no Exa platform-based voice recognition app (which can be faster).
	Off (None): Does not open a voice recognition tool.
	nVoq (Exa Trans) : Opens Exa Trans with nVoq voice recognition.
Delay Load of Exa Trans	Delays loading of the Exa Trans transcription screen, to focus attention on images before the transcription.
Exa Trans Font	Sets the font used by Exa Trans.
Exa Trans Font Size	Sets the font size used by Exa Trans.
Default Time Increment	Sets the default time increment for time blocks in the schedule book.

3. Select SAVE.

Viewer settings

See Configure the viewer.

Local service settings

See Install and configure local cache.

Use the worklist

This section contains the following topics (not all topics may be available depending on version and region).


Find preorders and studies View series and images View priors Open a pop-up window for physicians and ordering facilities Use the study toolbar buttons Use the order toolbar buttons About Linked Reporting Link reports Unlink a report Authorize an exam Unauthorize an exam View authorization history Update the worklist or viewer About color highlighting on the worklist About drawing attention to orders and studies Use local cache

Find preorders and studies

You can find a study and open it to view or edit details.

Procedure

1. On the <u>Worklist</u>, select **PRE ORDERS** or **ALL STUDIES**.

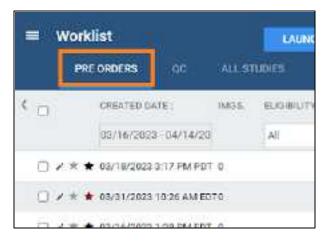

2. In the search bar, do one or both of the following:

- In a column header (such as **FACILITY**), type or select one or more criteria and press **Enter**.
- Select a column header to sort the list.

Worklist						
' a				FACILITY MY COMPANY	STATUS STUDV	
+ 9 0	2*	6.0	٠	My Company	Schedulat 03/17/2	
+ 40	1.0	40	*0	i My Company	Scheduled 03/17/2	
0	2*	% (f.	* 0	My Compeny	Scheduled 03/17/2	

3. Double-click a study in the list to open it for viewing or editing.

See also:

<u>Search syntax</u> <u>View approved reports</u>

View series and images

You can view series and images of any study in the worklist that includes them.

Procedure

1. In <u>User Settings</u>, clear the **Show Priors** checkbox and select **SAVE**.



2. In the worklist, find a study containing images and select its expand + button.

- 0() 0 +	2*8800 *	© E America AJA 10/10/2005 12:34 Pf	N EDT - Bamplet John - Sample Pennar (L) A	PStudy
.#		¥.	31/01/1	00 1200 AM EST
		1.3.312.200634	9187 500 006 12 2308 101 0 12 3046 1012	CR image Stor
• 0		a)	\$1/01/0	X00 12:00 AM 681
• 0		+	01/01/1	12:00 AM EET
• 9	1 - L	4	\$1/01/t	T05 WA 00 11 00
1				

- 3. Expand series rows by selecting the + button.
- 4. To view a thumbnail of an image, select the 📠 button.



Note: To view an image thumbnail, the status must be Incomplete or Unread.

5. To open series and images in the viewer, double-click the main study.

View priors

You can view the prior studies of any study in the worklist that includes them.

Procedure

1. In <u>User Settings</u>, select the **Show Priors** checkbox and select **SAVE**.



2. In the worklist, find a study and select its expand + button.

きんし かい書書語	a to the state of	Field Landak	CIVATO LIDEMPOT	
0 248=3	ct \star 0) [Newlig	Test Facility approved-a01/	02/20247-15 PW PST	Excitef.Ende(A
Nim Ba Test Facility	= 100441 01/02/2024 T 15 PM	PST - Enduef, Erndrøf	Test, Valerie	64
Neralla Teo Pacility	(7460-0) 0301712923 1.00 PM	FOT	Test, Valerie	st
NITER THE RICELY	AN 02104121 0210412083 12:00 4N	4 5 (2) (2)	Test, valerie	62
Nimita Tini Pacity	NEW DOLLARS TO PM	Priors	open Test, yasre	64
terrella facilitaciity	111ee1 0113112023 4:15 PM		Test, Valide	CH.
Investigation of the sectory.		PAT .	Test, taken	174
Remitia Tael Bacelly	27-641 01/31/2011.350 PM	PST -	Test, Values	.01

Result: The priors appear in rows below the study row.

See also:


View approved reports

View recent studies


You can view a worklist filter tab containing the 25 studies that were most recently opened in the viewer. This makes it easy to find studies you viewed recently and want to reopen. Most standard worklist functions are available to you on the Recent Studies tab, but see "Details" below for exceptions.

Procedure

1. In <u>User Settings</u>, select the **Show Recent Studies** checkbox and select **SAVE**.

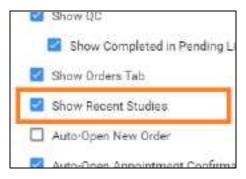

2. In the worklist, select the **RECENT STUDIES** worklist filter.

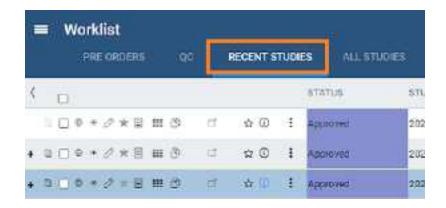

Details

The Recent Studies tab works like the All Studies tab except:

- You can't sort it (it's automatically sorted in first-in-first-out (FIFO) order by date/time of viewing).
- Priors opened during a viewer session do not appear on this tab, but priors originally opened from the worklist do.
- Stat level has no effect on the sorting of the list.
- The program creates a unique recent studies tab for each user.
- If you open a study from the worklist more than once, only one entry appears on the recent studies tab.

Open a pop-up window for physicians and ordering facilities

From the worklist, you can open a pop-up window to display the contact information of a study's physician or ordering facility.

Procedure

1. On the <u>Worklist</u>, in the **REFERRING PHYSICIAN** or **ORDERING FACILITY** column, select the doctor **a** symbol.

+ #O	10.4	1.85 (7)	• ©	1 WyCampen	Indextant and 012925 Part Partiel	No. ba
	10.00	10-12		E AyCompany	Determined 000/00/2025/9/27 PM (D) Entited Territed	10.10
• = 2	27.91	(and)	•0	1 By Campany	Generative 00/10/00/2000 Porch1 Swith Swith Street & Contra	of the select last
+ + 0	27.90	5.0	.0	i Ny Campany	Constant of 10 gage 545 P. Following Praviders Unit	and the second
6.80	2.4	(6)01	. ♦	i Uy Curiyany	References 00/17/2000 S-ft = and st and st 2017 US Highwes 70 East	-
€.≋0	2.90	5.5	* @	E My Company	Selections 03/17/2323 5/25 # Fax No.	Ares.
• = = = -	30.6	10.111		E WyCampen	Serverised 03/13/2025 4.00 Flexue	*C.181+

Use the study toolbar buttons

Each study on the ALL STUDIES tab or corresponding user filters of the worklist has its own toolbar on the left side of the study row. Depending on the properties of the study, one or more of the following tools are available.

• 8 ■ * * / ★ 8 = 0 0 * • ★ 0 :

Tool	Name	Description
+/-	Expand/Collapse	Expands or collapses the study row to show or hide series or priors.
	Prior	Select to display a list of prior studies in a custom filter.
	Checkbox	Select to select the study for further processing by subsequent commands.
0	DICOM Viewer	Open the study in the Exa PACS viewer. This button is only available for studies that contain images.
۲	Opal Viewer	Open the study in the Opal viewer (see <u>Install Opal Viewer</u>). Available for studies that contain images.
0	Edit	Opens the study for editing in the Edit Study screen.
*	VIP	If solid green, the study is for a VIP. Users with rights can change the VIP status in the patient chart.
	View Transcription	Opens the dictation and transcription editor.
	Multipanel	Opens the study in the multipanel, an operating panel for working with dictation, transcription, documents, notes, and other information.
0	Approved Report	Opens approved reports of the study.
00	Linked Studies	Opens the Link Studies dialog where you can link approved reports. See <u>About Linked Reporting</u> .
ď	External app.	Opens the current study in an external application if configured. See <u>Configure opening of a third-party program</u> .
Д.	Unread DICOMs	Select to view unread images. In the Unread DICOMs dialog, select MARK AS READ, or OPEN IN VIEWER to read the images in the Exa PACS/RIS viewer.
*	Authorization	US only. Opens the manual authorization screen. See <u>Authorize an exam</u> .
0	Notes	Opens a screen for entering and viewing notes related to patient, study, schedule, and reason for study, and for updating the call log. The button turns blue if study notes are available.
1	Study menu	Displays the study shortcut menu (you can also right-click anywhere on the study row to display the shortcut menu).

See also:

Work with the Exa PACS/RIS viewer

Edit study screen

Dictation and transcription with WebTrans

Worklist shortcut menus

Use the order toolbar buttons


Each order on the PRE ORDER tab or corresponding user filters of the worklist has its own toolbar on the left side of the order row. Depending on the properties of the order, one or more of the following tools are available.

Tool	Name	Description			
	Select	Select to select the order for deletion.			
1	Edit	Select to open the order in the Edit Order screen.			
*	VIP	If solid green, the study is conducted on a VIP.			
*	Authorization	US only. Select to authorize the order. See <u>Authorize an exam</u> .			

About Linked Reporting

With *Linked Reporting*, radiologists can save time by dictating multiple related studies into a single report. When the report is approved, all of the linked studies move to the Approved status automatically. When you open an approved report or addendum from any of the linked studies (studies 1, 2, or 3 in the figure below), you see the same combined report so that all information is available to you regardless of the study or report you open.

Linking reports also links the studies containing those reports, so that all studies follow the main study in terms of stat levels, study statuses, and report approval statuses. For example, if the main report moves to the Dictated status, so do its linked reports. If you later decide to unlink a report, those statuses revert to their previous states (with exceptions, see <u>Details on Linked Reporting</u>).

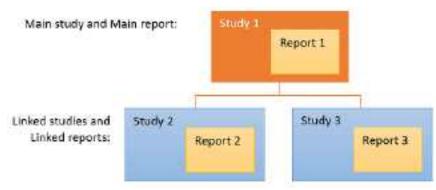

Once linked, Reports 1, 2, and 3 become one combined report

Note

- You can link reports any time before approval.
- You can unlink reports at any time.

- You cannot link a report that is already approved.
- If an addendum is added to a linked report, it applies to all linked studies, and is available for viewing from any linked study.


What do linked reports look like?

Based on the merge fields you include in your report templates, Exa PACS/RIS automatically fills the **header** with information about the main report and its linked reports. (Due to space constraints, some fields only display information from the main report, such as the referring physician.)

The **body** of the report contains the radiologist's findings for the main study and all linked studies.

Approved reports always appear based on the transcription template of the main study, even if the linked studies originally used different transcription templates.

An example of a linked report:

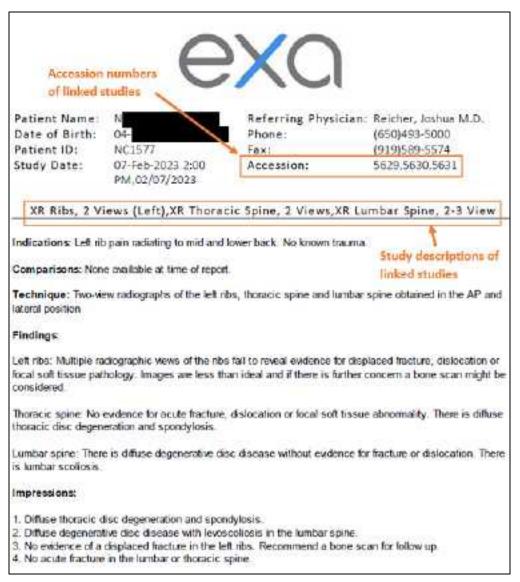

Link reports

To link reports:



Prerequisite: Obtain the Link Report right from your administrator.

Procedure

1. On the worklist, find a study that is available for linking, and select the gray link button.



2. In the Link Studies dialog, select the studies whose reports you want to link to the main report.

Link	Studies: Tes	t, Eric (Ac	c#TesE1077), 20060101, M, 023Y						
30	RNDRIGR	CT MRI	US/OTH X REQ POSTPCX					Baller-	LANC
	Stady Case 141	2/23/95	Accession His 1162 Padlays	simila Test facility				-	
	Histolity us		Referring Physical Test, Referring						
Sele	of asiddles to	link or un	link for Reporting:						
	diat All								
	where the second	SIDDLOFY	STLEN DEALERWEIN	STHEY DATE	200	THE MONTH	TENDERIC PHYSICAR	MATHONY	
Ū.	=23.4dated	LR.	took viscolen and a month cash and	protra Alda Islan	1.40		Last Astrong & Court	egilevatori lati a	ninj.
ő	≥teitiet	in'	DOR EDHERT. REEL ARE THAT DERIVATED	a lectrones	- 110	Colorega	novecia ust		÷.
ĊĂ.	invested	1(81)	TO REAL AND THE REAL PROPERTY OF	200200307	- 316	Test, Role	interg (grown: 70 East		ŵ
-							Shorkdaningte borr		
						Spills.	(746)484 6464		
						NO NETRU	INCOME BOARDING		

- 3. Optional: Click the physician is symbol to display details, and click the physician's name in the pop-up window to edit properties.
- 4. Select **SAVE**.

Result: The reports and studies are linked.

See also:

About Linked Reporting Unlink a report View approved reports **Details on Linked Reporting**

Unlink a report

To unlink a report:



Prerequisite: Obtain the Unlink Report right from your administrator.

Procedure

1. In the worklist, find a main or linked study (studies with the blue Linked Studies Studies button), and select the button.

Result: Whether you chose a main or linked study, the **Link Studies** dialog opens with the main study shown at the top with its linked studies listed below it.

- 2. In the Link Studies dialog, clear the checkboxes of the studies whose reports you want to unlink from the main report.
- 3. Select **SAVE**.

See also:


About Linked Reporting Link reports **Details on Linked Reporting**


Authorize an exam

When you configure authorization rules (see Add authorization rules) Exa PACS/RIS automatically labels exams on the worklist with these "authorization stars:"




🕱 (Black) No authorization needed.

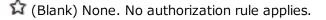

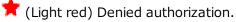



🗯 (Orange) Reauthorization needed.

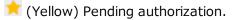

🖈 (Green) Authorized.

When you receive an authorization number from the insurance provider you can authorize the study, either from the worklist or from the Edit Study screen. When doing so, you specify under which insurance policy the exam was authorized by selecting the insurance **level** (primary, secondary, or tertiary insurance policy).

Authorize from the worklist

1. On the worklist study row, select the dark red \star orange 📩 or yellow \star star.

C 🖬 🖉 🕫 🖬 🖬 🖬 Scheduled (1234321)

Result: The Manual Authorization screen opens to the CURRENT AUTHORIZATION tab.

1	Aan	ual Authorizati	on					6
0	HAT	DRICAL ANTI-IORIZAT	ION CURRENT		SATION			
in	ana na na	e Previder (Primary):		W.				
		Shotue	Authorization No.	Casto	Boccriptice	Ettoctae Date	Experition Date	Notes
		need automanos		10.91	OF ANCIONRAMMY PELSIS W/COMPRESS/INCRICO/FRAST			
1	ŀ	Need Autopsyston		72(90	OT PELVIS C-WATRL			
in a	uanc	e Previcier (Secondar	y) ABSOLUTE TOTA	LCARE				
		Statury	Autorization No.	Gody	Description	Effective Inte	Expiration Date	Notes
1	*	Nasd Autorization		721.91	TANEDOGRAPHY FELVIE W/CONTINUE/ NEWCONTINET			
1	*	Read Jurhorsbor		72190	CT RELVINC. MATRI,			

2. Select the edit button of a study to authorize, and then enter the following settings.

EXG-PLATFORM

Ebglelite / Extend	tion				EANS:	Ť.	AVE & BLOSE	0.40
esusanno Name			CPTCode	De	ocription			
Notice Test			12101		INNOIOGRAPHIN P CONTRAST INCM			
ACCOMPANY NO.			Phone No.	72	e Ma			
PO BOX 25510 RICHWOND, VA 28			180010222 5558					
Salas	Need statforgation	*		Coloration Date	COSCHERE)	5		
Activity ten No.				Tifactive Date	уууулган	52		
Eferal Marolee				itosan Dale	York-Make	耐		
date a				Accused Date	2009-04848	-		
				Assident Type	C chalanter	ai i	🗌 Aito Accel	e t
					Center Acc	GIT.		
100001				Annajert Ototo	Delvis . w			

Authorization	
Status	Select the new status.
Authorization No.	When authorized, type the authorization number.
Referral Number	Type the referral number of the study needing authorization.
Notes / Reason	Type notes and reasons for the study needing authorization.
Expiration / Effective / Issue Date	Select authorization dates.
Accident Date	Select the date of the accident, if applicable.
Accident Type	Select the type of accident.
Accident State	Select the state or province in which the accident occurred.

- 3. Optional. Select **Eligibility/Estimation**, and then verify eligibility and estimate insurance coverage. See <u>Verify eligibility of preorders and studies with PokitDok</u> and <u>Verify eligibility and estimate coverage with Exa Clear</u>. (Available with Eligibility rights.)
- 4. Select **SAVE**.

Authorize from the Edit Study screen

- 1. Go to <u>Worklist</u> > <u>Edit Study screen</u> > **INSURANCE PROFILE.**
- 2. Open the insurance level to authorize, edit the **Authorization**, and then enter the settings as needed.
- 3. Select **SAVE**.

What if an authorized exam changes before the appointment?

If you need to make certain changes to an exam, Exa PACS/RIS helps you manage its authorizations. See the following topics.

About changing appointment types Change an authorized appointment Reschedule an appointment

Authorize from the Edit Order screen

- 1. On the worklist, open the study or order to authorize in the Edit Study screen.
- 2. On the **EXAM INFORMATION** tab, select the **ORDER** sub-tab.
- 3. At the bottom of the screen, select the order's authorization star.
- 4. Authorize the order as described earlier in "Authorize from the worklist."

See also:

What are authorization, verification, and estimation? Add authorization rules View authorization history

Unauthorize an exam

If a previously authorized exam is no longer authorized, you can unauthorize the exam by deleting the original authorization.

Procedure

- 1. In the worklist, find an authorized study (green star \star), and then select the star.
- 2. In the **Manual Authorization** dialog, select the edit button of the exam to unauthorize, and then select **DELETE**.

See also:

What are authorization, verification, and estimation? Authorize an exam

View authorization history

You can view a list of all authorization changes made to an exam.

- 1. In the worklist, select the authorization star (such as the green star \star) of a study.
- 2. In the Manual Authorization dialog, select the HISTORICAL AUTHORIZATION tab.

Manual Authoria	zation					
HISTORICAL AUTHOR	US NOTASIR	REENV. AUTHOR:	ZATION			
BUTATUR	#UT1-682A	7.0N.NC 000E	ococumica	DIDUDANCE PROVIDER	10/05	MOTEO
🖉 🍺 Resuttonzetion N	leeded ullin	76020	ODP VELOCIMIETEN FTU V	INDLICAL AABBOLUTE TOTAL ONTE Dev	ted Filmory	

3. Optional: To view details of an authorization change in the list, select its edit 🖉 button.

See also:

<u>Authorize an exam</u> <u>Unauthorize an exam</u>

Update the worklist or viewer

After performing tasks that modify studies it may be necessary to update the worklist or viewer to show changes.

• To update the worklist or viewer, select **F5**.

About color highlighting on the worklist

When the globe ^{(IIII}) button for studies on the worklist is highlighted, this indicates the caching status as follows.



Caching completed



Caching in progress



Some images could not be cached

About drawing attention to orders and studies

There are several methods to draw attention to orders and studies that are of special consideration. The following gives a general description of the available options.

You	e medical urgency. Setting a stat level places the study at the top of the worklist. I can edit stat levels and descriptions, and add new ones. Stat levels drive rkflows.
-----	--

ТАТ	The turnaround time. Studies that are near or beyond the TAT are flagged in red and appear at the top of a radiologist's unread studies worklist.
Priority	By default you can set the priority to High, Medium, and Low, but you can edit these and add more options. Priority is primarily used to sort the worklist (on the Priority column).
Critical findings	Medical findings that you can customize. In Exa Trans, you can select critical findings in a list to send notification to the referring physician. You can filter the worklist by critical findings.
Flag	With flags you can mark a study with a customized description that can serve as a filter. You can only assign one flag per study.

See also:

Change the stat level of an order or study Assign critical findings to an order or study Flag an order or study Change the priority of an order

Use local cache

A *local cache* is a dedicated area on a workstation or server to temporarily store Opal and DICOM studies for faster subsequent viewing. When using the local cache function, studies are sent from the modality (or forwarder) to the local cache on the workstation and to the image server in advance. When the radiologist opens the study from the worklist, Exa PACS/RIS opens the local copy, which is faster than downloading them from the image server.




Prerequisite: Install and configure local cache.

Perform a server prefetch

You can manually initiate a server prefetch from the worklist, whereby the server decompresses a study and copies it to a "local" cache on the server. If large studies will be opened in the near future, you can perform a server prefetch of those studies for faster loading.

1. On a worklist study shortcut menu, select **Images** > **Server Prefetch**.

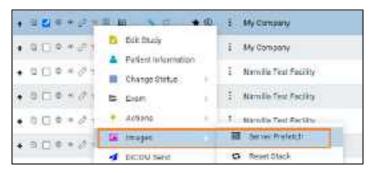

Prefetch studies to your local cache

You can manually prefetch studies "on demand."

1. On a worklist study shortcut menu, select **Images** > **Add to My Local Cache**.

Result: The server queues the images for sending.

Prefetch studies to other local caches

- 1. On the worklist, select the **ALL STUDIES** tab.
- 2. On a worklist study shortcut menu, select **Images** > **Send to Local Cache**.
- 3. In the **Local Cache** screen, in the list of available caches, select to the left of one or more destination caches.
- 4. Do one of the following.
 - To send to the selected caches, select LOCAL CACHE SELECTED.
 - To send to all caches, select LOCAL CACHE ALL.

Auto-route studies to local cache

You can create a routing rule to send studies that satisfy criteria to your local cache. Complete the following procedures:

- <u>Configure an application entity</u> for your local cache.
- <u>Configure a routing rule</u> for type **LOCAL_CACHE**, and select the AE title you created.

Use an on call worklist filter to locally cache studies

You can configure a worklist filter that sends matching studies to local cache.

- 1. Create a user worklist filter.
- 2. In <u>User Settings</u>, in the **Default Filter (Local Cache)** dropdown list, select the worklist filter.
- 3. Select **SAVE**.
- 4. On your computer, in a text editor, open the **localCache.cfg** file, located at the following path by default:

C:\Viztek\exa\cfg\localCache.cfg


5. Find the line for "oncall_enabled" and set it as follows:

"oncall_enabled": "true",

6. Save and close the file.

PACS Actions menu

The PACS Actions menu on the worklist provides top-level access to common functions.

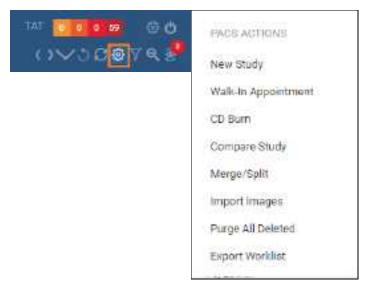

This section contains the following topics (not all topics may be available depending on version and region).

Create a new study Schedule walk-ins Burn studies, series, or images to media Compare studies Merge or split studies Import DICOM images Purge all deleted Export the worklist

Create a new study

A common way to create a new study or create a new order (or to "enter a new study" or "enter a new order") is to start from the schedule book by following the steps in <u>A typical scheduling workflow</u>. You can also jump into this workflow from the worklist:

- 1. On the <u>PACS Actions</u> menu, select **NEW STUDY**.
- 2. Follow the steps in <u>Set up a patient</u> and subsequent topics.

See also:

Schedule an unscheduled order Schedule a walk-in appointment

Schedule walk-ins

See <u>Schedule a walk-in appointment</u>.

Burn studies, series, or images to media

You can burn studies to a removable disc (such as a CD or DVD) or hard drive for sharing with other facilities, physicians, or patients. To burn images and series, see later in this topic.



Note: Key image references are visible when opening the media at other sites, and when viewing in Opal Viewer or Opal Light.

Prerequisite: Install Opal tools and Install Exa Client Viewer.

Burn studies

- 1. Place a disc in the drive.
- 2. In the worklist, select studies whose combined size does not exceed the capacity of the disc.
- 3. On the <u>PACS Actions</u> menu, select **CD Burn**.

PID	NAME		DATE	
AXIA-2263081	Maynard^Malinda^P		2023-4pr-27 15:38:33	9 9
ε		_		,
ptons Include Vewer Cadhe Images Anonymize DICOMS Fix Split BTO Create Annotations Remove Annotations Force SOP Class ID For		Pathi	a Hard Drive 🔘 Burn To Ds frem Finishen India	
Current Step	Progress: 0%	,	Total Progress: 0%	

4. In the **Exa Burner** dialog, enter the following settings as needed.

Setting	Description
Include Viewer	Select to burn a viewer program onto the disc along with the items. Required for viewing off site.
Cache Images	Select to keep items in the Viewer cache folder (and not remove them after burning).
Anonymize DICOMs	Select to remove identifying patient demographics.
Fix Split BTO	Select to combine multi-frame mammograms into a single DICOM file.
Create Annotations Overlay	Select to convert annotations to an overlay and include them with the images.
Remove Annotations	Select to omit annotations from the burned items.
Force SOP Class ID for All Files	Select to set the Modality tag (0008, 0060) to the value in the DICOM file's SOP Class ID.
Compression	Select a compression algorithm.
Report	Select a report storing method.
Write to Hard Drive	Select whether to write to the hard disc or removable disc.

Setting	Description
Burn to Disc	
Path	Select the ellipsis button and then browse for and select a destination drive and/or path.
Eject When Finished	Select to eject the disc after burning.
Close Media	Select to finalize the media after burning so that it is no longer available for writing.

5. Select **START**.




Note: If the connection is interrupted, burning will automatically resume when the connection is re-established.

6. Optional. After burning, test the results by opening an image from the disc using the included viewer.


Burn images and series

Use this procedure to burn individual images and/or series.

1. In <u>User Settings</u>, clear the **Show Priors** checkbox.

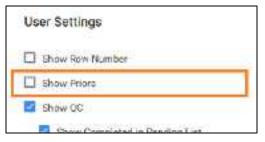

2. On the worklist, find images or series to burn by expanding the study row nodes and image preview buttons.

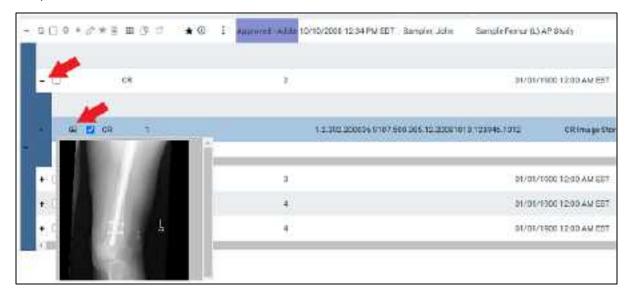

- 3. Select the checkboxes of the images and series.
- 4. On the <u>PACS Actions</u> menu, select **CD Burn**.

Compare studies

You can open two studies of the same patient in separate viewer screens for purposes of comparison.

Prerequisite: Configure two or more monitors (see <u>Set up connected displays</u>).

Procedure

1. In <u>Viewer Settings</u>, clear the **Single Instance Mode** checkbox.

Viewer Settin	98.
OCHERAL	Other Toolbar
	Ask on Close
INTERFACE	🔲 Single Instance Mode
MODALITY	Sip Frames in One to Meintain Playbook Speed

2. In the worklist, select two DICOM studies of the same patient to compare.

*	^a	U	a.	3	10	A	31	-	3		P	.*	9	1	Nimilia Test Fecility
+	0	0	ð.		Ð.	.sé	R	m	ð	*	197	*	Ø	1	Namila Test Facility
•	1	0	a	•	ø	×	2	-11	э	*	et :	*	¢	1	Normalia Test Pacility
t	9	0	-	×	Ż	×	э	ш	2		(F)	*	φ	.i.	Nimile Test Fasilty
÷	3	ò	a	ŵ	đ	*	ġį,	in		8	160		œ	1	Namilia Test Facility

3. On the <u>PACS Actions</u> menu, select **Compare Study**.


Result: The two studies open in the Exa PACS/RIS viewer.

Merge or split studies

In Exa PACS/RIS, *merge* and *split* refer to a category of operations in which you move series and images from one study to another. Additionally, *split* refers to moving images from a study with an incorrect accession number to the correct order. There are many ways to perform merge/split operations, but the following procedures are useful examples. Moving all series and images out of a study deletes the study. As an alternative to merging studies, see <u>Add alternate account numbers to a patient chart</u>.

Move or copy a series from one study to another study

1. In the worklist, select two different studies that contain series.

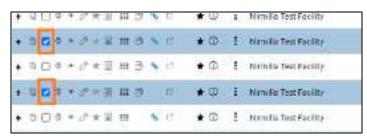

- 2. On the PACS Actions menu, select Merge/Split.
- 3. Select the button to choose Move Selected or Copy Selected mode.



- 4. in the **Merge/Split From** column, select the plus sign of the *first* study (the one on top) to reveal its series.
- 5. Drag the series onto the *second* study (the one on the lower-right) in the **Merge/Split To** column.



Note: As you drag over the second study, it turns green to indicate when you can release the mouse button.

6. Select **SAVE CHANGES**.

Move or copy all images in one series to a series in another study

- 1. In the Merge/Split To column, select the plus sign of a study to reveal its series.
- 2. Drag a series in the left column onto a series in the right column.
- 3. Select SAVE CHANGES.

Other moves or copies

In the same manner, you can expand nodes and move or copy any study, series, or image to another study or series.

Split images into their correct order

1. In the Merge/Split Study screen, select SELECT ORDER.

Merge/Split Study		
Move Sciented		SELECT ORDER
Merge/Split From	Merge	e/Split To
Name: Test. Tom(Acc#: TesT102 Accession No. 11		Name Test, Tom(Acc#: TesT102

2. Find the correct RIS order, and then select its SELECT button.

Merge	Merge/Split Study							
RIS O	Charles and the							
	STUDY DATE	PATIENT NAME	DOB	STUDY DESCRIPT: ACCESSION NO	ACCOUNT N			
SELE	01 03/17/2023 4:0	0 P Test, Valerie	01/01/2000	CT ABD & PELVIS V1371	TesV1001			
SELE	CT 8/17/2023 51	G P Test, Andrew	01/01/2000	DOP VELOCIMETH 1370	TOCASE			
SELE	CT 03/17/2023 5.0	0 P Test, Andrew	01/01/2000	BONE AGE STUDIE 1369	Tes A83			

3. Drag the images to the correct order, and then select SAVE CHANGES.

Add a new series to a study

1. In the Merge/Split To column, select the new series 🖻 button.



Note: The new series is added to the bottom of the list.

- 2. On the new series row, select the edit \square button.
- 3. In the Edit Study Info. dialog, type a description and then select the EDIT button.



Note: You can move series or images into the new series.

4. Select **SAVE CHANGES**.

Clone a study

1. In the **Merge/Split From** column, select the Clone Study 🗖 button.



Note: You can move series or images into or out of the new study.

2. Select SAVE CHANGES.

See also:

Add studies to the merge queue About the PID, MRN, and account numbers

Import DICOM images

You can import DICOM images to a study.



Prerequisites:

- Install Opal tools.
- If the study originates from a facility associated with the OPALIMPORT or EXAIMP* AE Titles, obtain access.

Procedure

- 1. In the worklist, select a destination study for the imported images.
- 2. On the <u>PACS Actions</u> menu, select **Import Images**.
- 3. In the pop-up window, select **Open Opal Import**.
- 4. In OpalImport, select Select Files/Folder or DICOMDIR to Import.

A Lingerand	14 N	a Open	
2.15.08 641.0	12011-11-10151	Livels IT 🚺 Blacket	- ⊡ b @ • ⊡·
EductPilog/Folder at 00004000 to Import	Service Galacia Cale Sale Society	Dada scores Dado scores Dados Destos Piconspra	
OpolDE Driv OpolDE Driv Sore Before Encode Drotes Male ModRy Demographics Mex File Str. Ker Production Flock Str. Ker Production Flock Str. Ker Drote Flock	Coper Tie Liet Save Tie Liet	Likawa Tha PE Nerwak	
# Ties Fablers Se	ected for Incorn	ten nave.	Setted

5. In the **Open** dialog, browse for and select files to import, and then select **Open**.

6. In **OpalImport**, configure the settings in the following table.

Note: This is a partial list. Other settings are self-explanatory or generally unused.

Setting	Description			
Do Not Compress	Select to store in DICOM Little Endian format. Clear to compress in DICOM JPEG200 Lossless format.			
Overwrite	Select to replace any existing instances of the studies.			
Opal DB Only	Select to import DICOM data only (no images).			
Save before Encode	For troubleshooting purposes: Select to place the images in a folder on the server, and not update the database.			
Isolate Fails	Select to place files that could not be imported into a separate folder.			
Modify Demographics	Select to reconcile demographics before import.			
Max Files	Type a maximum number of files to import.			
Max Size	Type a maximum data size to import.			

7. Select Import.

- 8. Optional. If you selected the Modify Demographics checkbox, the Modify/Confirm Demographics dialog appears. Under **New Info**, do one of the following.
 - Edit the demographics and then select **MODIFY**.
 - Confirm that the demographics are correct, and then select USE CURRENT.
- 9. Select Exit.

Purge all deleted

See <u>Undelete or purge a deleted study</u>.

Export the worklist

You can export the worklist to a CSV file.

Procedure

1. On the <u>PACS Actions</u> menu, select **Export Worklist**.



- 2. In the **Export Worklist** dialog:
 - Optional: In the **Filename** box, edit the default file name.
 - Modify the name of the exported file by selecting or clearing the **Include Timestamp** and **Include Filter Name** checkboxes.
- 3. Select the **Export** button.

Worklist shortcut menus

Exa PACS/RIS and Symmetry PACS have a study shortcut menu for studies in the Unread and later statuses. Additionally, Exa PACS/RIS has an order shortcut menu for PRE ORDERS and studies in the Ordered status. You open shortcut menus by right-clicking anywhere on a study row. The tasks you can perform by using each shortcut menu are described in the topics in this section. Note that the commands available on the shortcut menus vary depending on the order or study status and other factors.



Caution: Any errors in patient and study information resulting from incorrectly performing the procedures in this section can result in problems with data integrity. Take care to ensure that entered and imported data is correct.

This section contains the following topics (not all topics may be available depending on version and region).

Schedule an unscheduled order Edit an existing order Manually verify or clear eligibility of orders Change the priority of an order or study Assign critical findings to an order or study Flag an order or study Change the stat level of an order or study Delete an order Require document review Modify or reset the order authorization days Assign a study to users Add studies to the merge queue Create a teaching study Reset a study Delete a study Download approved reports Undelete or purge a deleted study Copy the API URL Reset a stack Lock and unlock a study Edit or view a patient, study, or order Change the study status DICOM-send studies, series, or images Send an approved report via Opal Send an approved report in a fax or email View and export audit log entries About linking patients Link patients

A Open the call log ttach (upload, scan) non-DICOM documents Attach (upload, scan) DICOM documents Open the call log


Schedule an unscheduled order

If you previously entered an order (or a study with a status of "Ordered") without scheduling it, you can schedule it from the worklist.

Procedure

Ż

1. <u>Worklist</u> > right-click an order > Schedule Appointment.

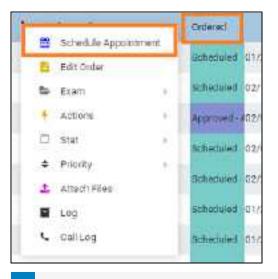

Note: If messages appear, refer to related topics in this manual.

Result: The modified Find Slots screen appears. (For a full discussion of the Find Slots feature, see <u>Use the Find Slots feature</u>).

istatice from Patient	Pridera 217 Code	Palest 2P Pi	e Gerter	Appointments		
10 y relati	27529	TPPLE	US'Excert	Salari mayor (*	2023-03-22 7.05 AM	-8:10 AN US Eastern
Appointment Type Facility				week ty hoor	XA	Argie
SPIRICE MITCH MITCH. •			RIED-	NONE SELECTED -	75700 ANORPH ARVEN SH	
uto.	fast live	54	e Theor	4347.998	& 10	Q15
CONTENTS	140 M	5	(27.7M) (7.	00	Pending	KA
ayo of the Meek		Mitoral Time			75877 ID PNDP 359 CT MPU	SIOTH REC POSTPCX
0000	000	0		80	Q 20	
a ticta Found					Aletta	

2. Filter available timeslots by entering the following criteria.

Setting	Description
Distance from Patient	Type the patient's ZIP code (see next row), and then select a distance. Timeslots are shown only at facilities within the distance from the patient that you select.
Patient ZIP Code (Postal Code and ZIP Plus)	Type the patient's ZIP code, postal code, and/or ZIP Plus to make the Distance from Patient filter available.
Carrier	Select the patient's insurance carrier.
Appointment Type	Shows the first appointment type in the order (read-only).
Facility	Select one or more facilities where the exam could be performed.
Modality Room	Select one or more modality rooms where the exam could be performed.
Date	Select to only show timeslots falling within a date range.
Start/End Time	Select to only show timeslots starting within a time range.
AM/PM	Select one or both to limit your search to daytime or nighttime.
Days of the Week	Select one or more days that the patient is available.
Additional Time	Select if the exam requires more time than the standard duration.

3. Select a timeslot in the list, and then select **UPDATE ORDER.**

See also:


<u>A typical scheduling workflow</u> <u>Schedule appointments back to back</u>

Edit an existing order


You can edit a previously entered order from the worklist.

Procedure

1. <u>Worklist</u> > right-click an order > Edit Order.

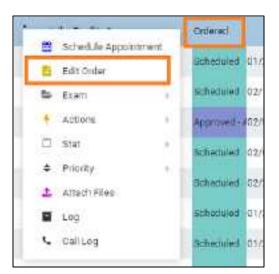

2. Edit the order and then select **SAVE.**

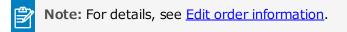

Manually verify or clear eligibility of orders


If you determine that an order, preorder, or study on the worklist satisfies payment provider eligibility requirements, you can mark it as Eligible (green check mark) on the worklist. If a study was previously manually verified, you can clear the Eligible status. Verifying or clearing does not affect the eligibility status in other parts of the program. To view the eligibility status, display the Eligibility column.



Prerequisite: Obtain Eligibility rights.

Procedure

- 1. Optional. In User Settings, display the Eligibility column.
- 2. <u>Worklist</u> > right-click an order > Manually Verify Eligibility.

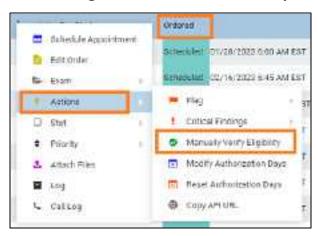

To clear the eligibility status, select Remove Manual Eligibility Verification.

3. Optional. In the **Eligibility** column, confirm that the green checkbox appears or disappears.



See also:


What are authorization, verification, and estimation? Verify eligibility of preorders and studies with PokitDok Verify eligibility and estimate coverage with Exa Clear Configure worklist columns

Change the priority of an order or study

You can set the priority of an order.

Procedure

1. <u>Worklist</u> > right-click an order or study > Priority.

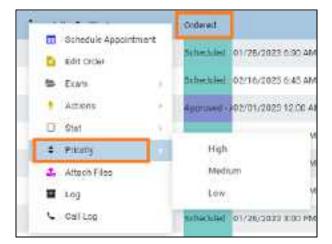

2. Select a priority.

See also:

About drawing attention to orders and studies

Assign critical findings to an order or study


You can assign critical findings to an order or study (for example, "Acute Positive").



Prerequisite: Configure critical findings. See <u>Reason codes</u>.

Procedure

1. <u>Worklist</u> > right-click an order or study > Actions > Critical Findings.

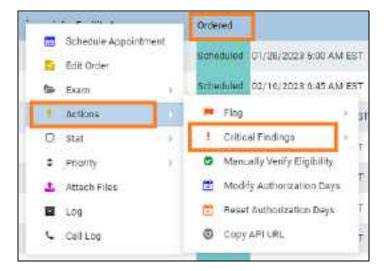


2. In the sub-menu, select a finding.

See also:

About drawing attention to orders and studies


Flag an order or study

You can flag an order or study in the worklist as being of special consideration. You can only apply one flag to an order. After flagging orders or studies, you can filter the worklist by those flags.

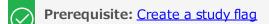

Procedure

1. <u>Worklist</u> > right-click an order or study > Actions > Flag.

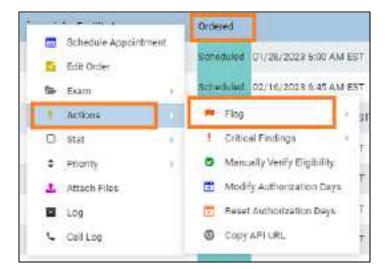

2. On the sub-menu, select one or more flags.


See also:

About drawing attention to orders and studies

Change the stat level of an order or study

You can assign or change the stat level or urgency (Stat 1–5, or Off) of an order in the worklist.

Procedure

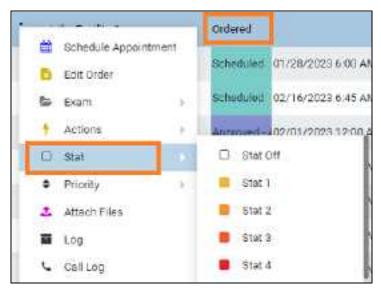

1. <u>Worklist</u> > right-click an order or study > Stat.

2. In the sub-menu, select a stat level.

See also:

About drawing attention to orders and studies

Delete an order

You can delete orders from the worklist.

• On the worklist, select the **PRE ORDERS** tab.

Delete one order

• On the order shortcut menu, hover over **Exam**, and then select **Delete Study**.

Delete multiple orders

• Select the checkboxes of multiple orders, and then on the shortcut menu, select **Delete Study**.

Require document review

When creating a scan document (or "document") type, you can require that users review the document before moving forward in the study workflow. For example, you can require technologists to review physician orders prior to performing an exam to ensure order accuracy. To review a document, open it from the Edit Study screen or Technologist screen and select the **Reviewed** checkbox. To require document review:

Procedure

- 1. In <u>App settings</u>, add a **Scan Document Type** and select the **Requires Review** checkbox.
- 2. <u>Add a study status</u>, select the **Document Review** checkbox, and then select the document types to review.

See also:

Attach (upload, scan) non-DICOM documents Configure study statuses

Modify or reset the order authorization days

This command lets you quickly change the number of days specified in the authorization rule associated with an exam. You can reset the days to the number of days set in the rule, or modify the days to an arbitrary number.

Procedure

1. <u>Worklist</u> > right-click an order > Actions > Reset Authorization Days, or Modify Authorization Days.

*	Ordered
Schedule Appointment	Echeduled 01/26/2028 6:00 AM EST
🕼 Exam	Scheduled 02/10/2023 & 45 AM EST
1 Actions	🗮 Flog 💚 g1
O stat	I Critical Findings
C Priority	Mancally Venify Eightlity
1 Attach Files	🖆 Modify Authorization Days
Log	🖹 Reset Authorization Days T
Call Log	Copy API URL

2. In the **Days Count** dropdown list, select a new number of days.

Modify Authori	zation	
Days Court	6	÷
Dayslott	0	
	EAVE	

3. Select SAVE.

See also:


Add authorization rules

Assign a study to users

You can assign one study to one or more users, radiologists, or peer reviewers. When the user signs in, they can create a filter of the worklist that displays all studies assigned to them. This feature also works in conjunction with the "Show assigned studies only" option under Setup > User Management > [user profile] > WORKLIST FILTER.

Procedure

1. <u>Worklist</u> > right-click an approved study > Exam > Assign Study.

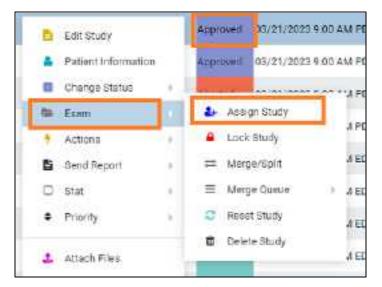

2. In the **Study Assignment** dialog, in the **Assign to User** dropdown list, select a user and then select the add the button.

tudy Assignment			
dige to Unor Austrame to averge	Assign to Refiningian	Antigate Peer Preisoner Delest mitcheyet to secure Colord Antiocological Peer Levi	349
	Dady Suta Davis description		Eastitiy
	82/21/2822 ANGERIARIAN DOUNT POLI		Rendla Tact Facility

- 3. Optional. In the **Assign to Radiologist** dropdown list, select a radiologist as the referring provider.
- 4. Optional. In the **Assign to Peer Reviewer** dropdown list, select a radiologist as the peer reviewer.
- 5. Select SAVE.

See also:


Assign a peer reviewer for a study Create a user worklist filter

Add studies to the merge queue

If you plan on doing "merge work" on a multiple studies you can add studies to a merge queue from the worklist first, and then merge or split them by following the procedures in <u>Merge or split studies</u>.

Procedure

- 1. <u>Worklist</u> > right-click a study > Exam.
- 2. Optional. To start merging or splitting with one study at a time, on a study shortcut menu, select **Merge/Split.**

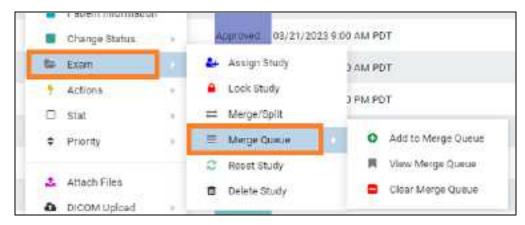


- 3. On the Merge Queue sub-menu, select Add to Merge Queue.
- 4. Repeat the previous step for all studies that you want to work with.
- 5. Right-click one of the studies that you added to the merge queue, hover over **Exam**, hover over **Merge Queue**, and then select **View Merge Queue**.

Create a teaching study

You can create a teaching study based on an existing DICOM study. The teaching study is anonymized.

Procedure

1. <u>Worklist</u> > right-click a DICOM study > Images > Create Teaching Study.

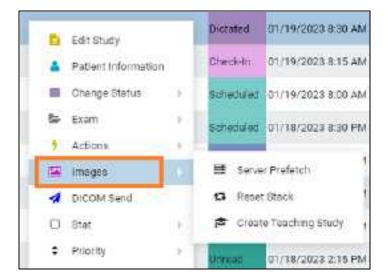

2. In the **Teaching Study** dialog, in the **AE Title** dropdown list, select an **AE title**.

ETela			
EXA, V32-V32 (10.227.26.76.)	13336.)		~
		CREATE TEACHING STUDY	CANCEL
Field	Current Value	New Volue	
Accession No.	105	rOv8MacH5hCad100	
Palient ID		844ΖΚΙ Ο406/JTΚΤριά	
Patient Name	Text Busic	Teaching_XA_105	
Study Description	ANORPH BRACH RTROR R561	ANGREEN BRACH RETIGE RESI	
Patient Contine	*	Other.	
Referring Phys. Name		[No Volue]	
Institution Name		[No Value]	


- 3. In the **New Value** column, type or select anonymous values for the new study.
- 4. Select **CREATE TEACHING STUDY**.

Reset a study

If the status of a study is beyond "Scheduled" in the study flow, you can reset the status to "Scheduled."

Procedure

1. <u>Worklist</u> > right-click a study > Reset Study.

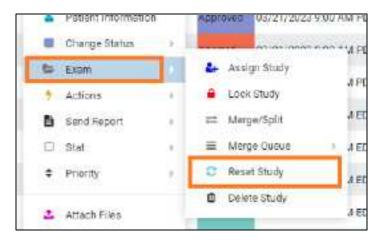


2. In the confirmation dialog, select YES.

Delete a study

When you delete a study it remains on the worklist but Exa PACS/RIS applies strikethrough text formatting to it and disables access. You can undelete or purge a deleted study.

Procedure

1. <u>Worklist</u> > right-click a study > Exam > Delete Study.

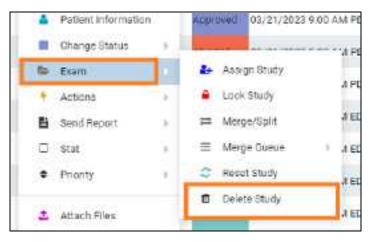

2. In the confirmation dialog, select **YES**.

Delete Reason		
Cancel/Delete Reason *	Illness	¥
	SAVE	


- 3. In the Delete Reason dialog, select a reason
- 4. Select **SAVE.**

Download approved reports

If an approved report is attached to a study, you can download the approved reports directly from the worklist.

Procedure

1. <u>Worklist</u> > right-click an approved study > Send Report > Download.

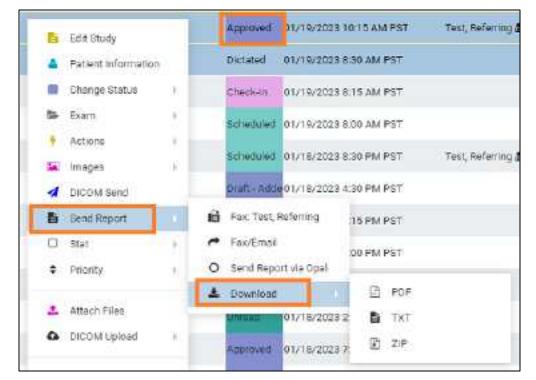

2. Select the format of the report.

Result: Exa PACS/RIS downloads the report to your Windows Downloads folder.

Undelete or purge a deleted study

Undeleting a study removes the strikethrough text formatting and restores availability. *Purging* a study permanently removes it from the worklist and the database.

Procedure

1. <u>Worklist</u> > right-click a **deleted** study.

4	₽	:	Ninnilla Tant Souther	
\$ 17	• 0	1	nobkaTest	Restore Study Purge (Permanently)
36.37	* 0	:	GobshaTest. www.	


- To undelete, select **Restore Study**.
- To purge a deleted study, on the study shortcut menu, select **Purge (Permanently)**, and then select **OK**.
- 2. To purge all deleted studies at once, on the <u>PACS Actions</u> menu, select **Purge All Deleted**.

Copy the API URL

If you need the API URL, you can copy it from the worklist.

Procedure

1. <u>Worklist</u> > right-click a study > **Actions** > **Copy API URL**.

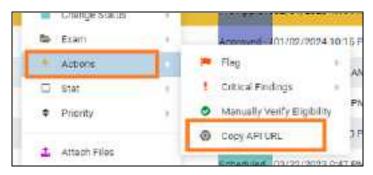

Result: A pop-up window appears with the API URL.


2. Press Ctrl + C to copy, and then select OK.

Reset a stack

To *reset* a stack means to restore its images to their last-saved ordering. It can also be helpful to reset the stack if any problems occur when displaying it in the viewer.

Procedure

- 1. On the worklist, select the **ALL STUDIES** tab.
- 2. On the shortcut menu of a study that contains images, hover over **Images** and then select **Reset Stack**.

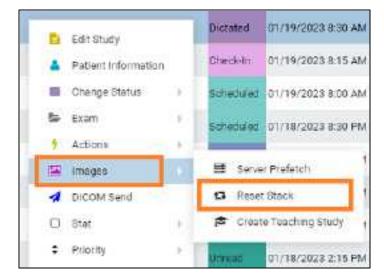


3. In the confirmation dialog, select **YES**.

Lock and unlock a study

You can lock a study to warn other users attempting to open it, and to prevent other users from using transcription. You can unlock a study that someone previously locked.

Procedure

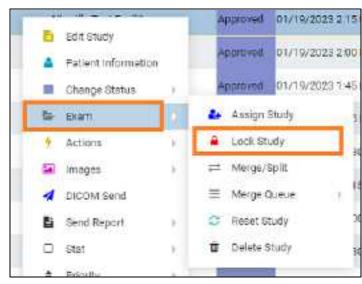

1. <u>Worklist</u> > right-click a study > Exam > Lock Study.

2. To unlock a locked study, on the study shortcut menu, hover over **Exam** > **Unlock Study**.

Edit or view a patient, study, or order

To edit a patient, study, or order you can access the Edit Study screen from the worklist by selecting the Edit Study, Edit Order, or Patient Information commands on the shortcut menu.

e	Schedule Appointme	π.		Scheckle appoint	their
ß,	Edit Study			Edit Order	
4	Patient information			Esem	
	Change Status			Actiona	
8	Exam		D	Stat	
,	Actions	(6)	\$	Priority	
	Stat	1963	2	Attach Files	
-	Driastu	1.67		1.00	
				Call Log	

Study shortcut menu

Order shortcut menu

See also:

Edit study screen

Change the study status


You can assign or change the status (such as Approved or Transcribed) of a study in the worklist.



Prerequisite: Configure study statuses. See <u>Configure study flow</u>.

Procedure

1. <u>Worklist</u> > right-click a study > Change Status, and then select a status.

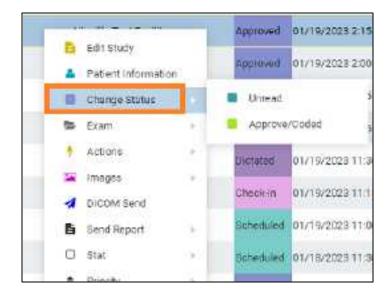

DICOM-send studies, series, or images

You can DICOM-send studies, series, or images to PACS or another destination. You can include any approved reports in the study, but note that the report status (such as Approved or Transcribed) is not sent. To send an approved report with its status, <u>Send an approved report via Opal</u>.

Prerequisite: Configure send destinations (see Configure application entities).

Send studies

1. <u>Worklist</u> > right-click a **DICOM** study > **DICOM Send**.



2. In the **Send Study** dialog, to filter the list of available AEs, type at least one character in the filter box and then select **Enter**.

ACCESSION NO.	STUDY DATE MOD	ALF BRUDY BESCRIPTION	ADDOUNT	THO, FACIENT	DOD	REFERENCE PHYCECLAN
ozketekzakos	01/08/1025 T00 ACT	PISKUL TO THEN WIT	82589	St ke J	88°84/1998	Bassett, benns Randolpt
/Apr Dearch	ptian Q	Priority	3	🗌 Include Appro	ned Report	SEND TO SELECTED


- 3. In the **Priority** dropdown list, select a priority.
- 4. Select destinations in the list, and then select **SEND TO SELECTED**.

Optional: To send any attached approved reports, select the **Include Approved Report** checkbox.

5. Optional. To manage in-process send jobs, see Manage jobs in the transfer and SR queues.

Send series or images

- 1. In **User Settings**, clear the **Show Priors** checkbox.
- 2. On the worklist, find images or series to send by expanding the study row nodes and image preview buttons.

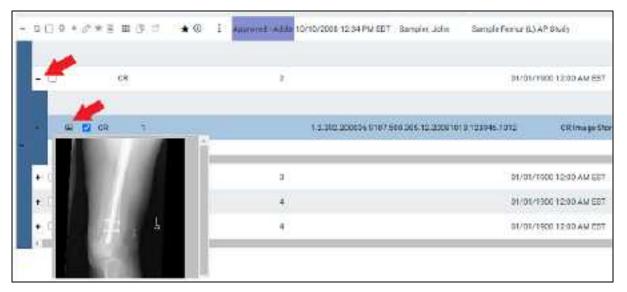

3. Select the checkboxes of the images or series to send.

- 4. Right-click and select **DICOM Send**.
- 5. In the **Send Study** dialog, in the **Priority** dropdown list, select a priority.
- 6. Select the send destinations in the list, and then select **SEND TO SELECTED**.
- 7. Optional: To send any attached approved reports, select the **Include Approved Report** checkbox.
- 8. Optional. To manage in-process send jobs, see Manage jobs in the transfer and SR queues.

Send an approved report via Opal


You can send approved reports of DICOM studies to another Opal or Exa platform system.



- Configure the referring physician on the sending server (your Exa PACS/RIS server) and receiving server. You may need to contact the receiving facility to confirm.
- Install OPAL tools.

Procedure

1. <u>Worklist</u> > right-click an **approved** study > **Send Report** > **Send Report** via **Opal**.

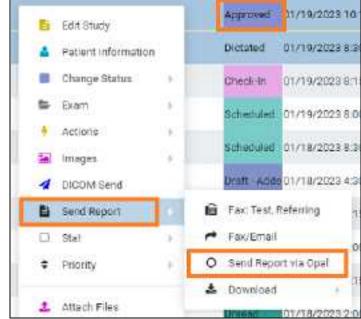

- 2.
- 3. In the **Send Report via Opal** dialog, to filter the list of available AEs, type at least one character in the filter box and then select **Enter**.

CCESSION NO.	STLEY DATE	MORALE STUDY BEED	SEPTION ACCOUNT #	D. PATERT	008	REFERENCE PRIVEIDAN
20	01/19/2023 2:00	FUB 00P ECHO F	TL SPECTRAL I TwaT1064	Text Ted	01/01/2008	Test Refering
Filter Down	rtar: C	Pnorty	0			SENO TO SELECTED
	often S	1000	0			SENO TO SELECTED
V EXAVS		vae)	Ø			SENO TO SELECTED


- 4. In the **Priority** dropdown list, select a priority.
- 5. Select send destinations in the list.
- 6. Select SEND TO SELECTED, or SEND TO ALL.

Send an approved report in a fax or email

You can send approved reports of DICOM studies in a fax or email.

Procedure

1. <u>Worklist</u> > right-click an approved study > **Send Report**.

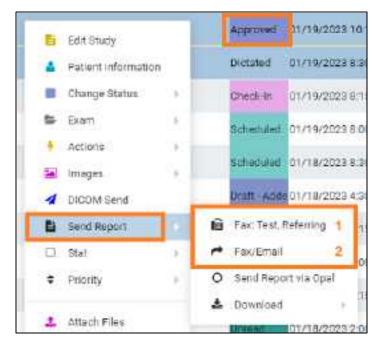

To fax to the default recipient (1), select it.

Result: The report is sent to the fax queue.

- 2. To send to selected recipients (2), select Fax/Email.
- In the Send Report dialog, select recipients and then select the SEND FAX and/or SEND EMAIL button.

Send Fax			Report Queue
Send To =	Ordering Physician 🐱	Ordering Physician	Test, Releting ((745)454-5454)
Send Email			Sector (MA
Sand To 🔹	Ordering Physician 👻	Ordering Physician	Teot, Referring (nimilia regilikonicaminoita


4. Optional. Repeat the previous step to send to additional recipients.

View and export audit log entries

You can view audit log entries, and export them to an XLS file.

Procedure

1. <u>Worklist</u> > right-click an order or study > **Log**.

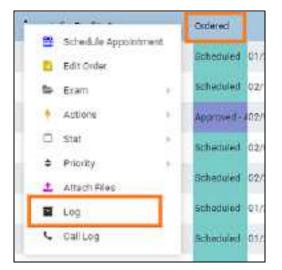

2. In the **Activity Log** screen, select a date range for the log entries to view, and then select **RELOAD**.

A REAL PROPERTY AND			91400 CBC	and the test of the second
	here: BUBLORD TROUGH AND		Te REPORTED AND MARCE	And And Address of the Owner of
		and the		
	() 🛢 monorman in early state		and the	and monotode laws (Teve strained
en la	0 E 100000 10100 107	Tanas Main	Same Char	control patient plan.
	IT S HARDER HAR ME UT		ADDRESS.	taatter attest and
	0. BE SUPACKOP 11.15 (49-11)	(01111046)	marine ((invite that into
A DESCRIPTION OF THE OWNER OWNER OF THE OWNER OWNER OF THE OWNER OWNER OF THE OWNER	o b mann i sideirt	Relating from an	emplei.	wig ten beiereg fremin fast beiereg with)

- 3. If any entries match the date range, they appear in the list.
- 4. Optional. To export an entry from the list, select it and then select **EXPORT TO EXCEL**.
- 5. To view information in a log, double-click the log entry in the list.

See also:

View activity logs

About linking patients

Two ways to handle duplicate patient records are linking and merging.

The first way is to *link patients*, which means to merge patients to the master patient record by using alternate account numbers. You can link patients from the worklist "on the fly" by following the procedure in <u>Link patients</u>, or from the patient chart by following the procedures in <u>Add alternate account numbers</u> to a patient chart.

The second way is to <u>merge patient charts</u>, which is generally used to move studies, series, and images between patients.



Note: Linking patients is different from Linked Reporting.

See also:

<u>Use alternate account numbers</u> <u>Merge or split studies</u>

Link patients

If a patient has multiple records, each with different account numbers (or MRNs or PIDs), you can link them from the worklist.




Caution: Linking an external patient record to an internal one results in deleting the external record.



Prerequisite: Configure alternate account number functionality.

Procedure

1. <u>Worklist</u> > right-click an order or study > Link Patient.

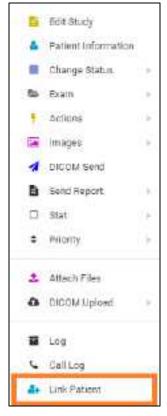

2. In the **Link Patient** dialog, in the **Alt Account No** search box, enter the exact account number to link to the current account number, and then select **Enter**.

Vain Account No	Tex51034			
Nt Account No	753	9		
ecuer.	Salact 🐱			
	ADO			
Main Account N	10	Alt Account No	Issuer	Merge Status

3. In the Issuer dropdown list, select the issuer of the PID, and then select ADD.

Note: If the account number was found (it already exists in Exa PACS/RIS), a message appears. Type a different number or select a different issuer.

4. Select SAVE.

See also:

About linking patients

Attach (upload, scan) non-DICOM documents

In various parts of the program you can *attach*, *upload*, and *scan* non-DICOM documents. All of these result in adding files to the DOCUMENTS section of the patient chart and the Edit Study screen, but differ slightly in procedure and scope.

Scan – Scan paper documents and add to a patients or studies.

Upload – Select existing document and add to patients or studies.

Attach – Open a screen from which you can upload or scan documents.



Procedure


1. Choose one of the following:

From the worklist

- <u>Worklist</u> > right-click an order or study > **Attach Files**.
- <u>Worklist</u> > <u>Edit Study screen</u> > **DOCUMENTS**.

From the schedule book

• <u>SCHEDULE</u> > right-click an appointment block > Attach Files.

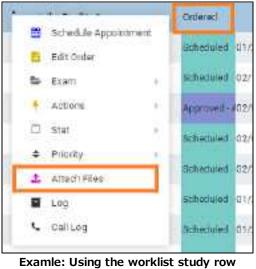

Examle: Using the worklist study row shortcut menu

Result: The Patient Documents screen appears:

							and the second s	ALCOHOL: DOLLARS	-
effe	eni	Dot	unerts				\$129.3041	UIUND -	12.040
	ġ.	-	All bonut with an all	CONTRACTOR OF THE OWNER			0	and the second second	
-			CONTRACTOR OF THE OWNER.	A-15-1010					
				600.400 hone	institut.	Administration and a second se	areast an owner.	terranerian	÷.
			100	-				H	
8	2		ole	Nerrighted (https://	1005,7440	100240412112111998221	1026 M 1020 1020 1020	couly	
61			1010	"with a first the second	108.741	1000000 0000000	TOWNED CORPORED	PRINT	

2. In the upper-right, select one of the following options.

QUICK SCAN - UPL			
QUICK SCAN TO PATIENT			
FULLISCAN	UFLOAD		RELOAD
FULL SCAN TO PATIENT	UPLOAD	TO PATE	ENT

Setting	Process	Attach Document To
QUICK SCAN	Scan document with basic options	Study
QUICK SCAN TO PATIENT	Scan document with basic options	Patient
FULL SCAN	Scan document with full options	Study
FULL SCAN TO PATIENT	Scan document with full options	Patient
UPLOAD	Upload document	Study
UPLOAD TO PATIENT	Upload document	Patient

- 3. Based on your choice above:
 - For **Quick** (basic) **scanning**, select a document type, type notes, and then select **Scan Now**.



• For **Full scanning**, use the **Document Scan** screen to edit the image as needed, and then select **Scan Now**.

Document Scin)
Pabent		Edit Image					
Patient Name: Tool, Todd	19 Ackleune: 192.168.242.13				1. 3	2 3	- Beset
Gynteen Manne : DIEW-L3-CH	Decement Level: PatientLovel	Brightness	Contrast	Zoos		1 10	C - Ante-
1							10000
							2.7
							Print
							Curri
							0.44
							1111
							-
							- 2
							Scon
							Sec.
							and the second
							Acces
							Decine
							and the second second
tal Pages 0							
canner Settings	Scanner		Decur	ent interm	neite		
Resolution ScienType	Belect Scamer	- 10			0.00000		
(0) Gray Seale (Delaute	and the second sec		Dot: T	ne:			v
ican/soe (1.1.2711.51 volve) Castone	C Rate Freeder O Hat lied		Notive	811			
CONTRACTOR AND A CONTRACTOR	Scanned By: Wetek, Poco						

• For **Uploading**, select options, use **SELECT FILES** to browse for and select files to upload, and then select **SAVE**.

Document Type *	Other 👻			
Created Date/Time *	2023-06-22	1	1.06.PM (DT	63
Requested By	Patient. 👻			
Notes	Test document.			
Δ.	laxitaum file size: 20 MB			

See also:

Attach (upload, scan) DICOM documents Require document review

Attach (upload, scan) DICOM documents


You can upload or scan files to DICOM format and add them to an order or study. Documents added in this way are available in the Exa PACS/RIS viewer.



Prerequisite (scanning only): Install Exa Scans

Procedure

- 1. Choose one of the following:
 - <u>Worklist</u> > right-click a DICOM study > **DICOM Upload** > **Upload File**.
 - <u>Worklist</u> > right-click a DICOM study > **DICOM Upload** > **Scan to Study**.
 - <u>Worklist</u> > right-click a DICOM study > **DICOM Upload** > **Scan to Order**.

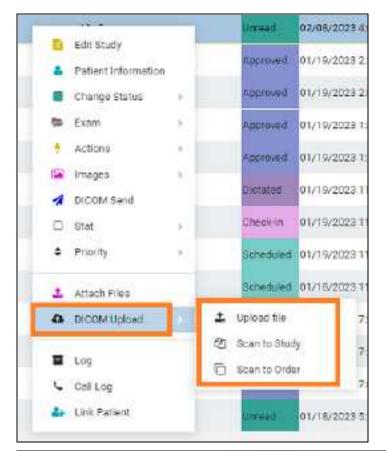

Setting	Process	Attach Document To
Upload file	Upload document, convert to DICOM	Patient
Scan to Study	Scan document, convert to DICOM	Study
Scan to Order	Scan document, convert to DICOM	Order

- 2. Based on your choice above:
 - For **Uploading**, select a series description, use **SELECT FILES** to browse for and select files to upload, and then select **UPLOAD**.

Upload for Cornelius, Si	aroara (N	ma#12109141129)	
Series Description		Com ×	1
#EN		e	
Maximum Resort 204/3		Oppman + Name Image: Company Image: Non-Arrivation Image: Company	
		Napara Alfile Ci -	

• Use the **Document Scan** screen to edit the image as needed, and then select **Scan Now**.

🗐 Document Scin		1993 C			- = ×
Patient Name: Teol, Todd System Name: DEV4.3-08	19 Addense: 192,168,262,13 Document Level: PatientLevel	Edit Image Distriction	Contrast Za		
					∳ ∳ Peine A4 Bann Now Bann Now Cecline EXII
Total Pages 0	(Minute				
Scanner Settings Resolution ScanType 10) Gray Scale ScanStor Art 22775 Structure Castorne	Scanner Belect Bearner (§ Auto Freder) Hat Bed Soarred By, Videk, Poco		Documore Doc. True Notes	information	v

See also:


Attach (upload, scan) non-DICOM documents Require document review

Open the call log

You can use the call log to save a record of communication to and from a patient regarding an appointment or order.

Procedure

1. <u>Worklist</u> > right-click a study or order > **Call Log**.

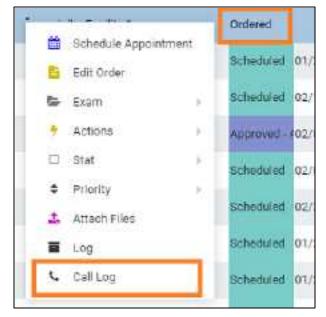


2. The Notes tab of the patient chart opens. See <u>Update the patient call log</u>.

Edit Study screen

In the Edit Study screen (or *study chart*) you can view and edit most information about an order or study. For a list of topics in this chapter, see <u>Topics on the Edit Study screen</u>.

Open the Edit Study screen

To open the Edit Study screen, follow the procedure below.

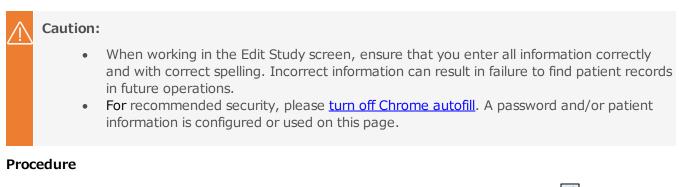

• On the Worklist ALL STUDIES tab, double-click study row or select Edit Study 🖉.



...OR

On the Worklist PRE ORDERS tab, double-click an order row or select Edit Order



Result: The Edit Study screen opens to the EXAM INFORMATION screen, with the most recent study sub-tab selected.

FENRINT	i i			676071
EXAM EXPORTANCE	Critich	273 278		
MODIF FEIDINATION	1.00000			
SUNDER WARDIN				
NUTLIC AVAILABLE ARTIST	Direct Bo	WE WIEGENEE	Repeatr	
HINANSEMPHINA	Accession No.	774		
1003040111				
APPROXID REFERENCE	Beady UID	CONTRACTOR NUMBER	Shady Netro	Geboor Manza Hartes
PERMORRATION	Den	32/38/2023 145/94/201 +		
of the latter to the latter of	Multy Status	Dis Approval		
ALT IN THE REAL PROPERTY AND INTERPOPERTY AND INT	- story softer	(revenue)		
MANDAL MADINA	Budy Pw1	Selmi +	Disgunis Codes	Soluti Biografia (200
AVISOR ALBERTS	Distances	delet w		
u u u si				
PROFESSION.	Roportment Type	aparameter NN abort (ROLE)	8	
ET LINES	- Diele	91377	Fefering Physician	Refeberen Presier
Carlos fore	. 470,05	CMMC.	Sulping Physican	111
property and	0.000		and the statement	lisädeteilipinen
moneter	AUCAME		Resident Havailan	Solici ryd, rmacar
weeking .	Mainley Diving	(ma)	Atlantes	Solari Mining (*

Topics on the Edit Study screen

This chapter (<u>Edit Study screen</u>) contains the following topics (not all topics may be available depending on version and region).

View patient alerts View a summary of an order Edit order information Edit study information About changing appointment types Change an authorized appointment Edit basic patient demographics Add or edit a patient's portal account View and print a patient statement from the claim inquiry Scan a patient's driver's license Generate a patient check-in code Export a patient CCD Enter a new charge Enter payments and print receipts View payment history and print receipts Add or edit an insurance profile Verify eligibility of pre-orders and studies with PokitDok Verify eligibility and estimate coverage with Exa Clear View or add patient documents from Edit Study

View approved reports Open the transcription screen Manage other physicians Enter notes and reasons for study Update the patient call log Enter employment and transfer-related information Assign or remove patient alerts Add allergy information View problems View prior studies and add a new study Add a custom form to a study Add a study form to a study Draw on a study form image Specify a follow-up exam Refer a patient View activity logs

View patient alerts

You can quickly check whether the patient of a study has any alerts that might affect billing, scheduling, exam preparation, or other factors.

Procedure

1. Go to <u>Worklist</u> > <u>Edit Study screen</u>, and then select the alert \square button.



Result: The Patient Alerts dialog box appears.

Pelion Acits
Alerts
· Balance Due
• CT Contract Attemp
• Pat Alait
Notce
TOTAL CONTRACTOR
a second s

2. Optional: Enter notes and click **SAVE**.

View a summary of an order

You can view a summary of an order and its studies.

Procedure

1. Go to <u>Worklist</u> > <u>Edit Study screen</u>, and then select **SUMMARY**.

SWNAR	÷			6710
CARACIPATION NO.	Order Information			
NEXT HERE A	Expected Animal 03/03	(3322 1:45 PK PST	Order Statuc:	Chesk
PATRON PROMINE INSTANC			ICO Coder:	
PERSONAL PROPERTY.		TER NA DE F ESES	Issuance Profile	
2022/00/10 2011/00/10	Constant by Ba			
CRAWCHPTING.	STORIES - Vale Same Dro	p tomaties		
menter and the	1.7			
4210	Austoine No.	750	BPT Cades	765
NOTIONAL PROPERTOR	Study Description	30 RHDR IAR CT MRIVANITH RDD ROCTOCK	Body Part:	

Result: The order summary appears in the right pane.

Edit order information

An order defines information for one or more studies. You can edit properties of an order.

Procedure

 Go to <u>Worklist</u> > <u>Edit Study screen</u> > **EXAM INFORMATION**, and then select the **ORDER** subtab.

19449	c# RegT1014), 200	WWW	BEUDVICE	ANNOUNCES FRANK WOOMIN
LOAM INFORMATION	Coper	inia	1	
INCIDENT INFORMATION				
D-STREETWARMENTS.				
INTERT INVALUE RECEIVE	Facility	Peglatida -	Schollag Notes	Dalact Marent Nota
MURANSCOTORI,C	Modelly	D se al		Tree Bilingule Notes Note
DOCUMENTE -		L] er		
APPROVED REPORTS		0 cf		
TRANSCRIPTION	Requesting Date	flue ti	Junfoston if	
DT-CT-TANKING	worksows pare	AAAA 2000 To Total	Moltike Spreice	
W200	Execute-	Anglicterials		
ALTER TOWAL PERSONNER	OrderingFacility	withe Oldering Facility (Astheoider, a 1	Tavinnois grat	Securi connection *
INTROFF ALERTS		(A)	Discharge	1
MAGNE	Pattere Location	Seloct -	instructions	
TISOLEME	124/122.502	Landa a	1 10 10 10 10 10	
BTURNER.	Patient Room No.		Physician Order	delice +
DUCTION FORMS	Paters Curvition	Select 👻	Status	1 March 19
NUMBER POPULATION	Doarce	field v		
FOLLOWIN				
NUTERINA DI	Place of Service	Select w		
STRATEGY MATTRAL	Reparations	Insert Dagarstory Chystenes 🕈		
Activitica	Paracies			The second se

Result: The Edit Order screen appears.

2. Near the top of the screen, select the **ORDER** sub-tab and make changes to the following settings.

Setting	Description		
Facility	The facilities performing the studies (to change, reschedule).		
Modality	The modalities used for the studies (to change, reschedule).		
Requesting Date	The date requested for the order. Users with rights can edit this date.		
Resource	The resources used for the studies (to change, reschedule).		
Ordering Facility	Available with Mobile RAD only. Select the ordering facility or location of the studies. If the patient resides at the ordering facility location, select S as Home Address .		
Patient Location	Select the current location of the patient.		
Patient Room No.	Type the room number of the patient location.		
Patient Condition	Select a patient condition to consider for studies (such as "wheelchair").		
Source	Select the source of the patient (such as referral or previous medical facility).		
Place of Service	Select the place of service for billing purposes (Box24B).		
Supervising Physician	Select the supervising physician (Medicare requires a supervising physician to be present in the department or location during the exam).		

Setting	Description		
Functional Status	Select the patient's functional status.		
Cognitive Status	Select the patient's cognitive status.		
Transition of Care in	[Unused]		
Patient Visit No.	The enterprise-specific serial number for the study.		
SDE Study	Select the checkbox if the order contains an echo ultrasound-related study.		
Schedule Notes	Type or select schedule notes (such as messages from the front desk).		
Justification of Mobile Service	Type the justification of mobile service for reference by the physician.		
Technologist	Select the technologist performing the studies.		
	Unavailable if both Mobile Billing and Mobile Rad are in use.		
Discharge Instructions	Type any discharge instructions.		
Physician Order Status	Status from the Referring Provider portal.		
PRINT ORDER	To print a summary of the order information, select one or more of the following options, and then select PRINT ORDER .		
Include Disclaimer/ Signature Area	Includes a preconfigured disclaimer on the printed order to be signed by the patient.		
Include Notes	Includes schedule and study notes on the printed order.		
Include Barcode Page	Prints a barcode on the order.		

- 3. If DICOM settings are not available to edit, at the bottom of the screen, select **DICOM STUDY**, and then make changes to DICOM settings as needed.
- 4. To edit or add CPT codes, at the bottom of the screen, select the edit 🖉 or add 🛨 button, enter changes, and then select **SAVE.**
- 5. Select **SAVE.**

Edit study information

You can edit properties of an ordered study, including DICOM settings and CPT codes.

Procedure

1. Go to <u>Worklist</u> > <u>Edit Study screen</u> > **EXAM INFORMATION**, and then select a study sub-tab.

FURNALIST	â.			676.071
EXAMINE DENILSTICH	Critten	tra tra		
MODIT RECOMMENDE	1.54556			
CHARGED WHICH TH		2.000		
WITE OF ANY ADDRESS OF A CANY	Dreet By	ALC YERCOMER	Repetry	
NOULANYC PHOTOL 2003/WORKS	Auszakosta,	774		
Annous assess	Trucky U-D	CO() 8711/0030409	Shady Nation	Geboor Marcia Harba
TRANSCRIPTION	Dara	32/98/2023 545 PAVENT *		
al analysi kindanan	Unity Station	(in Approval		
MANDAUL INCOME.	Budy Pw1	Selmi +	Disgunis Codes	Soluti Hisperite (au
NUMBER ADDRESS	Diargasso	Select v		
PROFESSION.	Roportment Type	DOWNSHITTNE WATCHDIEL	3	
PURC	Dude	7077	Fiefering Physician	Referrin Trener
	arcsex :		Sularing Physician	Itoficial Training
morener	AUC Abde		Personal Physician	Select well Provident
errerers.	Marcha Brian	1411	Atlantes	Solut Mining 4

An order containing studies with accession numbers 773 and 774

2. Make changes to the following settings.

Setting	Description
Order No.	The order number to which the study belongs (to change, reschedule).
Accession No.	The accession number of the study (users with rights can edit here).
Study UID	The study UID as determined by the selected appointment type.
Date	The date of the study (to change, reschedule).
Study Status	The current status of the study in the Exa PACS/RIS study flow.
Body Part	Select the relevant body part for the study (for example, hand).
Orientation	Select which side of the body (for example, right).
Appointment Type	Select the study's appointment type (the type of exam for scheduling purposes).
Code	The internal procedure code of the study (associated with the appointment type).
G-Code	If CDSM was used to create the order, the G-Code appears, which identifies the CDSM application (such as LogicNets).
AUC Audit	If CDSM was used to create the order, a link is available to the AUC audit, which shows the AUC decision tree used by the physician.
Modality Room	The modality room where the study is performed (to change, reschedule).
Reason	Type the patient-specific reason for performing the study.
Study Notes	Type or select study notes (such as basic study history or notes from technologist to radiologist).
Diagnosis Codes	Select diagnosis (e.g. ICD10) codes for the study.
Referring Physician	Select the physician who referred the patient for the study.

Setting	Description	
	If available, you can select the provider ${ar {ar {ar {a}}}}$ button to view provider details.	
Ordering Physician	Available when using Mobile RAD. Select the physician who ordered the study. If left blank, Exa PACS/RIS automatically enters the referring physician.	
	If available, you can select the provider ${ extsf{B}}$ button to view provider details.	
Reading Physician	Select the radiologist or other physician who will read the study.	
	If available, you can select the provider ${ar {ar {a}}}$ button to view provider details.	
Attorney	Select the attorney for the patient. This attorney is given access to the study throu Attorney Portal.	
	If available, you can select the provider ${ar {ar {a}}}$ button to view provider details.	
Department	Type the department of the facility performing the study.	
Institution	Type the institution of the facility performing the study.	
Station	Type an identifier for the station of the location performing the study.	

- 3. If DICOM settings are not available to edit, at the bottom of the screen, select **DICOM STUDY**, and then make changes to DICOM settings as needed.
- 4. To edit or add CPT codes, at the bottom of the screen, select the edit 🖉 or add 🛨 button, enter changes, and then select **SAVE.**
- 5. Select SAVE.

About changing appointment types

Authorized appointments

Appointment types contain one or more CPTs. If you change a study's appointment type from one that has only one CPT to another than also has only one CPT, Exa PACS/RIS automatically copies any authorization information from the original CPT to the new one. However, if the old and/or new appointment types have multiple CPTs, you can copy insurance authorization information from selected CPTs in the original appointment type to any matching CPTs in the new one.

Clinically relevant Medicare appointments (CDSM)

If you change the appointment type of a study that was approved by CDSM and there exists a CPT that matches the old and new appointment types, the approval remains (Exa PACS/RIS copies the AUC modifier and G-code, and retains the original AUC Audit link). However, if a matching CPT does not exist, new modifiers and codes are not copied, and the AUC Audit link is removed.

See also:

Change an authorized appointment

Change an authorized appointment


The following feature is not available on all installations. If you attempt to change an appointment type in a study that is already authorized, Exa PACS/RIS presents you with a dialog for copying the authorization information to the new appointment type.

Procedure

1. Go to <u>Worklist</u> > <u>Edit Study screen</u> > **EXAM INFORMATION >** study sub-tab.

mature	â				
FRAMING DRAWING	anpen	QRD(2) 2212 724			
FACTOR AND		-			
COLUMN TWO IS NOT		A			
NUMBER OF STREET	0 sppr Ke	645-10000500	-108		
Independent of the	Volkeppion Nuc.	774			
E0300ENTE	101000000000				
ATTRACTOR	St-dyUD	1333342-111232009	3,00		
TRANSCONTING	Dett	07/06/2022 S-459/MIEST			
OTHORNADAWA	Shudy Storus	Da Assesso			
HUTH	Shary She as	(TTALEVIS)			
ADDITIONAL REPORTATION	Dody Twit	Select e	044		
FRANCIN ALCONT.	Userston.	Salart 4			
4110000	-		-		
FEDRICIES	Agesteriors Terr	an monital et socialisti	teice. E		
There are a second s	-	Control 10	0.0		
Internet contents	Code	1000	1045		

- 2. In the **Appointment Type** dropdown list, select a different appointment type.
- 3. In the **Change Appointment Type** dialog, to copy authorization information, select **YES**.
- 4. In the **Authorization** screen, select the **COPY** button of any CPTs whose authorization information you want to copy.

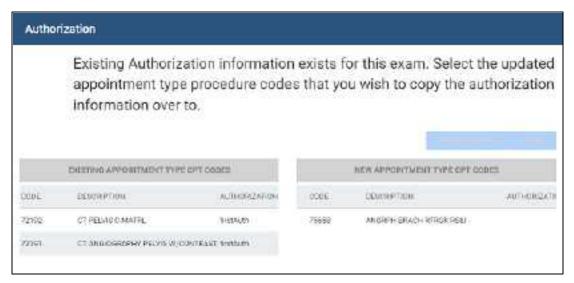

5. Select SAVE.

See also:

About changing appointment types Authorize an exam

Edit basic patient demographics

The Edit Study screen gives you convenient access to some patient demographics in the patient chart.

Procedure

1. Go to <u>Worklist</u> > <u>Edit Study screen</u> > **PATIENT INFORMATION**.

	Petient Inform	uefion seed	nitivi Alson	e solvi Los	susc.	NUL	-	13 1940 GF
NAMES OF TAXABLE PARTY	Nate	NORSEE	SULEA V				Licenia No 2014	
NUMBER AND ADDRESS	Pailos *	HICCH	www.				this face	NAMES IN
connuma Manage National	CROW/Parkets (B	-					Stp. Seka	2000
	Advant No. (J.), 1	Peningo	Ð				Stylesettere	
THE OWNER AND A		MANUEL AL	INCOME INCOME.	CHINA, COURSES			ing taken	
ATTER	Rote 1	Tarr	14				05/069/25	
THE CHARMEN BRANCKS		Palatel	a da	-10				24.1
Namakianto e	Alasia	The Harm	1.414	Harna			Turne Fax	
	008.1	01/80/1811	17	3m ⁺	N.	37		
	Nata Itou	Select.	- 2	Augest Relat				C From
NAMES IN TRACES	Studies Calor	blick	- Q	We git the [August	
1.Cov (e conseni	Petersterner	C Inanian Informe Alasha	Î.	Perettig	34irt			
0.1100 UP 010000.0		tiane tiane		Lang/sor	Select	<u></u>		

2. Make changes, and then select SAVE.

See also:

Create and modify a patient chart

Add or edit a patient's portal account

You can edit a patient's portal credentials and other account information.

Procedure

1. Go to <u>Worklist</u> > <u>Edit Study screen</u> > <u>PATIENT INFORMATION</u> > **PORTAL ACCOUNT**.

Patient Account	Representa	tive Account -	10
Rani	Select 1	*)	
trui	Fourstane		
burster that	Lost News		
CAUN YAR	Enal		
	Decisionary Croat		
	Restaume	80eil	

Create a patient account

1. In the **Create Portal Account** dialog, under **Patient Account**, type the patient's email address, and then click **CREATE**.

Result: An email is sent to the patient to verify and finalize their patient portal account.

2. Click SAVE.

Create a representative account

• Under **Representative Account**, fill in the options and click the save 🔟 button.

Result: A email is sent to the representative to verify and finalize their patient portal account.

Assign a representative account

If you created more than one representative, you can select which one to assign to the current patient.

Under Representative Account, select the account in the dropdown list and then click the save
 button.

Edit a representative account

- 1. Under **Representative Account**, select the account in the dropdown list.
- 2. Change information as needed, and then click the plus + button.

Reset passwords

After an account is verified, you can reset the password.



Caution: A password is configured or used on this page. For recommended security, <u>turn off</u> <u>Chrome autofill</u>.

- 1. Under Patient Account or Representative Account, click RESET PASSWORD.
- 2. Type a new password, and click the corresponding Save button.

View and print a patient statement from the claim inquiry

While in the Edit Study screen, you can view and print a patient statement from the claim inquiry.

Procedure

1. Go to <u>Worklist</u> > <u>Edit Study screen</u> > <u>PATIENT INFORMATION</u> > **INQUIRY**.

and a second	NAMES.		10001		NACT TROUBLE	TTTL INTERTOMATIC	e LLADOR 1	1010	i come	- 14		
	1970) gala tila na 197		12-11	jiros:		na	Course of Course	190	ng basanan	-	PRESING	contraste
		-		UNDER	1		 (10.040		
-011	и		0.12	CNET	92 1.4	-	(1210	10.00		81.03	
Persona Descandado			8111 1018	CNET	943	1		1010	11.00		BLUE. BLUE SE	

2. In the **Patient Claim Inquiry** dialog, select **PRINT**.

A printable claim summary appears in a separate browser window.

Scan a patient's driver's license

You can search for a patient by the name and DOB on their driver's license. If Exa PACS/RIS finds a matching patient in the system, it opens a dialog in which you can reconcile that record against the information on the license. If it does not find a matching patient, it uses the information on the license as a basis for you to create a new patient.

Procedure

1. Go to <u>Worklist</u> > <u>Edit Study screen</u> > <u>PATIENT INFORMATION</u> > **SCAN LICENSE**.



2. When the **Scan License** dialog appears, scan the driver's license.

Result: Relevant information appears in fields.

Generate a patient check-in code

You can generate a code that the patient can use to check-in for an exam when using Patient Kiosk.

Procedure

1. Go to <u>Worklist</u> > <u>Edit Study screen</u> > <u>PATIENT INFORMATION</u> > CHECK-IN CODE.

Check-in Code	
Test, Richard	
	B1QPQK
	0011 To 0.445040

2. Optional. Select **COPY TO CLIPBOARD**, and then paste the code in an email to send to the patient.

See also:

Use Patient Kiosk as a patient

Export a patient CCD

A patient CCD (continuity of care document) contains medical information that you can send to other providers without loss of meaning. You can export the information, download it to your Windows Downloads folder, and send it to an email recipient.

Procedure

1. Go to <u>Worklist</u> > <u>Edit Study screen</u> > <u>PATIENT INFORMATION</u> > **EXPORT CCD**.



2. Select type and encryption functions.

- 3. Select PROCESS PATIENT INFO, and then select DOWNLOAD.
- 4. To send the zipped file, select **SEND** and follow the on-screen instructions.

Enter a new charge

You can modify charges for health services from the Edit Study screen.

Procedure

1. Go to <u>Worklist</u> > <u>Edit Study screen</u> > **CHARGES/PAYMENTS**.

	÷					river, of Alicenter	ni Jawas	HEORIE .	ACTY.
	NDO	Accession No.	Dete	GPT Opda	SPT Bescription			N0	542
APRESTATIVENTS	$+ \times$,	2923	2008-04641 (0.00.4)4	merer 👌	. ADDRESS AND T	we have a group of			
KANECITERILE	+ ×	100	2004/06/11/2014/04	1000	(Increasing	Whotester, Name 14	E		
E ALLANCE .									
nesali marana Kalent Mar Kalent Mar	HEN CHATCH	and .							
ncar more North Le Electronic	HEN CHANGE Deta	Aneral	Pyrel Mode 0	SectorCard No.	Dynt Gaaroo	Received By 1	lyvet. 10	Net	•
		Ansont	Pyrek Mode 🕴 Ö	SerlitCard No.	Fyret Dames	Received By 1	lywt. ID	Net	••.3

- 2. Edit settings for an existing charge, or select **NEW CHARGE**, enter charge information, and then select **SAVE**.
- 3. Optional: In the **NDC** column, select the plus button to add an NDC to the charge.
- 4. Optional. Select **Eligibility/Estimation** follow the steps in the topics below to verify eligibility and estimate coverage. (Available with Eligibility rights.)
- 5. Select **SAVE.**

See also:

Enter a claim for a study Verify eligibility and estimate coverage with Exa Clear Verify eligibility of preorders and studies with PokitDok Manually verify or clear eligibility of orders

Enter payments and print receipts

You can enter payments, print receipts, and create an invoice for the balance due from the Edit Study screen.

Procedure

1. Go to <u>Worklist</u> > <u>Edit Study screen</u> > **CHARGE/PAYMENTS**.

REALIZED PROFILIER	- + :×	1254	09/8	472023 4.16 PM	13	1000	листония	AGA INTERNA	ai lei T	
PETERST ANYMANT RETURN INSURALIZED FOR THE EXCLUSION FROM	+ ×	1354		4/2023 4 15 PW	G	ninettyr toor 🦻	Selectary		10	
APPROVE APPEND	NUM CH	ome sove	Arreart	Furst Made		Check/Card No.	Pyref. Reason		Powt 10	Netes
NUTS AND REPORTED	+ ×	(01-14.000) III	9	Select		()	Solent v	vtztęk		
NUTSH ALINT ALINTEE YNDALDAD	NEW PAO	TatalEaprart	30/947	TREAM						

- 2. Select NEW PAYMENT, enter payment information, and then select SAVE.
- 3. Optional. To print a receipt, select **PRINT RECEIPT**.
- 4. Select **SAVE.**

See also:

Record a payment

View payment history and print receipts

You can view a history of payments made on behalf of the current patient, and print receipts.

Procedure

1. Go to <u>Worklist</u> > <u>Edit Study screen</u> > **PATIENT PAYMENT HISTORY**.

internet inter	MANUAT HISTORY MANUAL	ALCONG AN ADDRESS OF
naradar narashadana Indanashadan	THREAD IN AND ADDRESS OF ADDRESS ADDRE	restant among senare in
	No set artic Auror.	

2. Select a payment for which to print a receipt, and then select **PAYMENT RECEIPT**.

Result: The receipt opens in a browser tab.

3. Use the functions of the browser to download or print the receipt.

Add or edit an insurance profile

An *insurance profile* contains all relevant information about an insurance policy applied to an order. By creating primary, secondary, and/or tertiary insurance profiles for your patients who have insurance policies, you can take advantage of billing and other features throughout the patient encounter workflow.

You can create an insurance profile from the patient chart or the Edit Study screen, but in the Edit Study screen you can modify authorization information specific to the associated order or study.

Editing an insurance profile with authorizations

When you edit an insurance profile (change the carrier or policy number) of an order that is partially or fully authorized, and the new profile contains one or more of the authorized studies, Exa PACS/RIS prompts you with the choice to copy the authorization information to the edited insurance profile. This saves you from having to reauthorize studies unnecessarily.

Procedure

1. Go to <u>Worklist</u> > <u>Edit Study screen</u> > **INSURANCE PROFILE**.

And an address of				6.	etv" - 10 (2404 - 26) of 144	100
CERTIFICATION CON	1000 m	9977	1101010-012	Projecto more some orange)	*	1000
	23 MINO -	AND ALC: AND A	4	(Cath)		100 200

2. Select ADD.

nsurance Deta	ils	skvr: c	EAR GANGEL
Laval *	Primary Insertive	Address	P0 B0X 25519
Insurance Carner *	AETNA HEALTH PLAN .		RICHMOND, VA 23260
Insurance Provider		Phone No.	(900)222-5553
Туре		Website	
Policy Number *	TeosAdtro2	Group Name	
Group No.		Valid From Date	MM/DG/YYY DE
Pre-Cart. Phone		Valid To Date	MM/00/200
Pre-Cort. Fax		Medicare Ins. Type	Select 🛩

3. Enter the following insurance details.

Setting	Description
Insurance Details	
Set As Default ²	Select to set the current policy as the default policy. Useful when the patient has multiple policies at the same level.
Level	Select whether the insurance is the patient's primary, secondary, or tertiary policy.
Inactive	Select to inactivate the policy while retaining its information.
	The Valid To Date is automatically set to the current date.
Existing Insurance ¹	To use or edit a previously entered policy, select it in the dropdown list.
Insurance Carrier	Select the insurance carrier.
Insurance Provider Type	Entered automatically.
Policy Number	Type the patient's insurance policy number.
Group No.	Type the insurance group number of the policy.
Pre-Cert. Phone / Fax	Type contact information for getting pre-certification.
Accept Assignment	Select if the insurance policy can accept assignments.
Address / Phone No.	The contact information of the insurance carrier.
Website	The website of the insurance carrier.
Group Name	Type the name of the insurance group.
Valid From/To Date	Select the period during which the current policy is effective.
Medicare Ins. Type	Select the Medicare insurance type of the policy.

Available:

1. When adding from Edit Study; 2 = On the Patient chart

4. Enter contact information for the patient, and determine eligibility and/or obtain an insurance estimate (your options may vary).

Eligibility & Esti	mation					
Elipi68ity / Es	timation					
Contact Inform	ation					
Relationship *	Solt 🕶			DOB *	01/01/2000	55
Subscriber Name *	Tèza	M		Employee Status	Saloct	•
	Regin	\$.ffiz		Work Phone		
Dex	Ε	1		Home Phone		
Dountry	United States 👻					
Address Line 1						
Address Line 2						
City/State/ZIP		172235				
Chipp States and		Select	¢ •			
	ZIP Code	sib hitm				
Eligibility & Est	timation					
Service Type		Select to sea	rch for and select a	a service type.		
Benefit On Date		Select the da	ate on which the po	licy becomes eff	fective.	
Eligibility & Estima	ation		ify eligibility and/or coverage with Exa		age. See <u>Verif</u>	<u>fy eligibility</u>
Contact Inform	nation					
Relationship		Select an em	ergency contact pe	erson for the pat	ient.	
Subscriber Name		Type the nar	me of the insurance	e policy subscribe	er.	
Sex		Select the se	x of the subscriber.			
Country		Select the co	ountry of the subscr	riber.		
Address		Type the add	lress of the subscrib	ber.		
DOB		Select the da	ate of birth of the su	ubscriber.		
Employee Status		Select the en	nployment status of	f the subscriber.		
Work / Home Pho	ne	Type the pho	one numbers of the	subscriber.		

- 5. Optional: To authorize the study, see <u>Authorize an exam</u>.
- 6. Select SAVE.

See also:

About active and inactive insurance

Verify eligibility of pre-orders and studies with PokitDok

If the insurance provider is a PokitDok trading partner, you can verify eligibility by using the following procedure.

Procedure

1. Go to <u>Worklist</u> > <u>Edit Study screen</u> > **INSURANCE PROFILE.**

n manager La man an an Andria an	1				mil/+13/364:62/** #	X10
STREET AND DREET ON	1200	GINT	THEFT	more so more same one set	*	A PLACEMENT
	23 MM	and the first of the loss of t	4.	- Elistia	6	ाम् <u>स</u>

- 2. Double-click a profile to open it.
- 3. Select a service type, and then select the green Eligibility / Estimation button.

Service Type	SURGICIAL (2)÷
Benefit Cin Date	03/14/2023	臣

Result: If the verification succeeds, an eligibility report appears.

See also:

Verify eligibility and estimate coverage with Exa Clear Manually verify or clear eligibility of orders

Verify eligibility and estimate coverage with Exa Clear

With purchase of a license you can use the Exa Clear feature to verify insurance eligibility and estimate primary insurance coverage. You can also generate and print the Good Faith Estimate that satisfies the No Surprises Act.

The following procedure describes accessing the verification and estimation screen from the Edit Study screen, but the verification and estimation steps are the same when you access the screen from other parts of the program including when creating an order, or from the patient chart.

Prerequisites:

- <u>Add an insurance provider</u> and include the trading partner ID for ImagineSoftware.
- Obtain Eligibility rights from your administrator.

Procedure

 \checkmark

1. Go to <u>Worklist</u> > <u>Edit Study screen</u> > **INSURANCE PROFILE.**

🕰 EARFSPatientLS, EARFSPatientF5 (Accel:EARE3436), 1989-01-01, F, 35Y							
planetary turns in the state	1				ETHON (ADD RELOAD	
I FREEKT HISTORIECH	E STARGE	OVIDER	(994)0817//6_>	DUDIE NO DOLIZYWSINE ODOLE NG	REATONSHIP	DUDDOTINICE	
	2 E Prinary	AND MEDICARE EN		(102)945-54976240	Sw1	Europaisentii, Earlipe	

- 2. Double-click a profile to open it.
- 3. Under Eligibility & Estimation, select a service type, and then select the Eligibility / Estimation button.

Eligibility & Estimation						
Service Type	NONE SELECTED +	Benefit On Date	2028-07-29	E	fligbliky / Extension	

4. Continue to the topics below.

Verify eligibility

1. Select the **ELIGIBILITY** sub-tab.



Result: If not eligible, the screen above appears. If eligible, the eligibility breakdown appears.

tigildity Status	C Benefits Date C4/06/2023	
LIGHLITY	ESTIMATION	
Plan Details		
ACTIVE Point	of Service (POS) - Health Benefit Plan Coverage - Plan Name: AHF Choice POS & Diagnostic X-Ray, MRI/CAT Scan	
Co-imprance		
(herselies id)	gproeting 3. Hay, MRIVCAT Brown - Visit realing are considered on-Pain-temeori. All other in Network Providers. Contoes making Graw Theo Call Sciencizios5 APPLES TO OUT OF produk Imaging Professional Companies, 32(HE 449UE2 TO 2011 OF POCK2T	201
	spreatie X-Ray, MRI/CAT Sean - Visit milled are considered in Pain Network. All Otherts Retwerk Providers, distpation? Xivy and Lab (CONIC APPLIES TO OUT OF PODIET)	201
Nervefite id	sphoetis X-Ray, MRI/CAT Scen - Visit molest an considered Out Of Plan Metwers. Complex imaging Other Than CAT Scen.00WS AVRUSE TO OUT OF POCKET. Complex Imaging al Components Onto APPLICA TO OUT OF POCKET. Colpusitors Kny and Lab.COMO APPLIES TO OUT OF POCKET	501
Co-Pay		
Bevefits id	gnossie X-Ray, MRI/CAT Scare - Visit etdfied an considered is ("se Network, All Other In Network Providers, Compline Inseping Other TransCAT Scan, Compline Imaging Professional 1. Lizzarbert Xino and Lat.	\$15
	agrootile X Bay, MRI/CAT Seas - Viall en Bail an completed D.4 O Planillaryees. Complex unaging Offer Than CAT Scien Complex broging Professional Component. Oxforbert &	\$30
Deductible		
	aith Benefit Plan Coverage - Colemiae Year - Visit milliad are considered is Flan Network. Flan Eagle: 01.91 2022. All Other is Harwork Providers. OED MOUUDID IN COP	\$450
Individual - He	alth Benefit Plan Coverage - Remaining - Visit	50633.0

- 2. To view patient, study, and insurance details, select the down arrow 😇 button.
- 3. Optional. If conditions changed since the benefits on date, select **RECHECK ELIGIBILITY**.
- 4. Optional. To print a copy of the verification, select **PRINT**.

Estimate coverage

1. Select the **ESTIMATION** sub-tab.



Result: If not eligible, the screen above appears. If eligible, the coverage details are shown including the estimated patient balance.

lightity Status 🤡 Be	nofile Date 2023-07-03				1
		REASTIMATE	GOOD PARTICIST MATE	LITTER	PHIN
e Contasurance	610.00	Estim	ated Patient Baland	te: \$155.	23
O Co-Pay	3100.00	The balance above a the estimated amount to be collected from the patient including the deductifies copy, and connectances amounts to			
O Deductibile / Max	\$100.00 / \$200.00	time galacted (
Max Balance Due	\$0.00	T Abdar	non and Poivis without Contrast	CPT 74176	
Max Out-of-Packet	\$1500.00	CT Abdomen and Pelvis with Contrast - CPT 74177, 09967			
	ear studies to see the cost of the visit	1100 C 2125	eatimete for a specific pro al procedures, onange kelacitad i IETRE.		fillioicnt nei click

- 2. To view patient, study, and insurance details, select the down arrow 😇 button.
- 3. Optional. If conditions changed since the benefits on date, select RE-ESTIMATE.
- 4. Optional. To generate and/or print the Good Faith Estimate, select GOOD FAITH ESTIMATE.
- 5. Optional. To print a copy of the estimate, select **PRINT**.
- 6. Optional. To generate a formal estimation letter, select **LETTER**.
 - a) To print the letter, select **PRINT**.
 - b) To send the letter in an email, type the recipient's email address and select **EMAIL**.

See also:

Verify eligibility of preorders and studies with PokitDok Manually verify or clear eligibility of orders

View or add patient documents from Edit Study

You can view all documents attached to a patient record and their metadata, and add new documents.

Procedure

1. Go to <u>Worklist</u> > <u>Edit Study screen</u> > **DOCUMENTS**.

🕂 Test, Grad (Acc4:1	(coB1112), (1/01/2000 M	234			
namer Den selasten Senstaanse	1.	Documents Ins 42 linearcete				ov, so more a streat provin SSC HOM - Prove
The second secon		RECORDER FOR	1000000 44-00	-ina siaw	CHINADIST	antironanes, i rea
	14	the		1000,700	epreddintsawe	ett (0.1000) (1544e107 Sur
This sector (

- 2. To view a document in the list, double-click it.
- 3. To review a document, open a document in the list and select the **Reviewed** check box (see <u>Require document review</u>).
- 4. To add a document by scanning, place the document in the scanner, and do one of the following.
 - To scan one page without scanning options, select **QUICK SCAN**.
 - To scan with scanning options, select the **QUICK SCAN** arrow button, and in the shortcut menu, select a scan type.
- 5. To upload the document to RIS, select **UPLOAD**. To add document to the patient chart, select the arrow button for and select **Patient Document**.

			STUDY : 30 RINGH MAR CT MIRLU	S/OTH INQ POSTPCX	+ STUDIES :	SELEUT -
Patient Docume	ints				SAVE	BACK
	1	20	SCRATCH 1x1	0.65 HB	REMOVE	
Document Туре *	Other	~			41	1.1.1
Croated Date/Time	•					
	03/14/2023	圓				
	72:28 PM LD 0					
Tequested By	Select 🛩					
Notes	Gample					
	Maxterian file size: 20 M	13				
	SELECT HURS					
Beviewed						

- Enter metadata for the document.
- Select **SELECT FILES**, browse for and select a file, and then select **Open**.
- 6. Select **SAVE.**

See also:

Attach (upload, scan) non-DICOM documents Attach (upload, scan) DICOM documents

View approved reports

Procedure

Go to <u>Worklist</u> > <u>Edit Study screen</u> > **APPROVED REPORTS** to open a window and display the report.

Result: If an approved report is available, it opens (may take time).

CTest, Bred (Acc4:T	Ex81113) 01/01/2000	M, 20Y			
EARNARY MANUAL PROPERTY.	Transplanting	la , iba	81 (* -	(196) + U	0
APPROVED ALFCAILS		Patient Saves Referring Physicality	Best Musi	Patient III Date of liketin	Teolettis In Jan 2000
artistic permittane		Pecesara Koomisch Nurder:	SCHINDRIGH CTANI LEPONERICI PUBLIPUK 790	Date of they budy KiD cost	00/01/2025 2 kit PM
AND A DECKARTER		Exam: RADIOGI	MPH OF THE HAN	D	
ALCONOM TO A CONTRACT		Teshnique PA, a	blue and latent views	of de hand are si	brited
TRADING		Clinical Datas Ps			
Carrow Cone		Prior Studies			
					egenerative changes with two changes PIP and DIP

Open the transcription screen

Often you open transcription from the worklist, but you can also open the transcription multipanel from the Edit Study screen.

Procedure

1. Go to <u>Worklist</u> > <u>Edit Study screen</u> > **TRANSCRIPTION** to open a window and display the transcription editor within the multipanel.

C Test, Leura (Acc#:TesL	Test Lours (MRN Test 4 Accession No: 167, Modelity: CT Study Date Stack Department 30 DNDP ISB CT MDI U
	C An-directive Tore Study Description; 3D RNDR ISR CT MBLU
Period approximation	
La sull'anno 11 anno 11	Transcriptico editor 🕴
SHEETEARTS	The first and business sectors and business and
Particlet Participal recount	Construction of the second second for the second se
Subayers retrict	ATTLE AND
CONTRACTOR OF THE OWNER	Approx
and the second s	
TRANSPORTER	Autor (Net)
THE PROPERTY OF	HODING A MUNICIPALITY AND A MUNICIPALITY A MUN
-61174	Read of Second Second
NORTHWALL STOPPLICE UP	Datest Terrer First Laws Patent 2
Autorit manys	- State or the W
A Owner 1	Processory 2019/04/2011 144/02/2014/4001 144/07/2016/
	31600032
PROBATING _	And a second second second second second
STATES	Essay: RADIOGRAPH OF THE HAND

See also:

Dictation and transcription with WebTrans Dictation and transcription with Exa Trans and Dragon Dictation and transcription with Exa Voice and Exa Trans

Manage other physicians

You can add or delete other physicians (ones other than the primary, referring, or reading physicians) who are associated with the current study.

Procedure

1. Go to <u>Worklist</u> > <u>Edit Study screen</u> > **OTHER PHYSICIANS**.

R.MAAAAY	CODE	OTHER PRIVISIONAL NUME	AUGRESS	C. C
EXAMUNICEMATION INTERTINFORMATION ORANGESI'NAMENTU INTERTINAVMENT HISTORY INGURANCE PROFILE DOCUMENTU	₩ H7.00,01	HL7_OB_CC_CdlC1CR	807 First Fluor	(987)
ACTION CONTRACTORS	Silint Ohr Phr	iniae 💽 🔍		

- 2. Optional. To add a physician, in the Select Other Physician dropdown list, select a physician.
- 3. Optional. To remove a physician, select the trash 🔳 button.

Enter notes and reasons for study

You can enter patient notes, study notes, schedule notes, and reasons for the patient's studies. Generally, you can use: *patient notes* for patient specific information; *study notes* for basic study history or notes from technologist to radiologist; *schedule notes* for messages from the front desk; and *reason notes* for more specific study history.



Procedure

1. Go to <u>Worklist</u> > <u>Edit Study screen</u> > **NOTES**.

A Test, Laura (Acces)	6esL43), 01/01/2000 , F, 2	27	
EURAAPN Staat Entratummen Antoine Sectorements Disates Administration	of coverage	NYTENT NOTES CONSIDELL NOTES	
	-	average.	sharikatta
ALTERNIE STORES ALTERNIE STORES SAARGORFTER CLARK PHYSICAL	🖑 vzsk Pala	Parama electropic and	s me tak nanaly street
ANTES			

- 2. Select the PATIENT NOTES, STUDY NOTES, or SCHEDULE NOTES sub-tab.
- 3. Select ADD, type notes, and then select SAVE.

	STUDY NOTES	PATIENT NOTES	SCHEDULE NOTES	REASON FOR STUDY	CALL LOG
User		Viziek, Paus			
Date/Time		03/16/2021 8-44 AM	A ED/C		
Notes *		Patient requested	a second opinion.		


4. Select the **REASON FOR STUDY** tab.

STUDY NOTES	PATIENT NOTES	SCHEDULE NOTES	REASON FOR STUDY	CALLIOS
Reason for Study	Patient complaine	ed of chronic headache	5	

5. Type reasons in the box, and then select **SAVE**.

Update the patient call log

You can use the call log to save a record of communication to and from a patient regarding an appointment or order. The worklist has columns for displaying the following information that you enter in the call log: last call made, last call category, and last call notes. To update the call log:

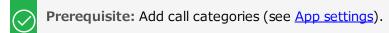

Procedure

1. Go to <u>Worklist</u> > <u>Edit Study screen</u> > **NOTES**.



2. On the CALL LOG sub-tab, select ADD.

STUDY NOTES	PATIENT NOTES	SCHEDULE NOTES	REASON FOR STUDY	CALL LOG
User	Viller, Pacs			-
Date/Time	03/16/2025 0:40 A/	N ED T		
Call Category *	Schedule v			
Call note *	Don't schedule or	Wedneodoya.		

- 3. In the **Call Category** dropdown list, select a call category.
- 4. In the **Call Note** box, type notes, and then select **SAVE**.

Enter employment and transfer-related information

You can enter various information on whether the illness or injury was employment-related, the dates of illness, work absence, and hospitalization. You can also enter information related to the source of a transferred patient.

Procedure

1. Go to <u>Worklist</u> > <u>Edit Study screen</u> > **ADDITIONAL INFORMATION**.

- 40
-
1

2. Enter the information as needed and then select **SAVE.**


Assign or remove patient alerts

Patient alerts are important notices to staff regarding a patient. You can assign or remove alerts for a patient.

Caution: Failure to add a needed patient alert could result in incorrect treatment or diagnosis.

Procedure

1. Go to <u>Worklist</u> > <u>Edit Study screen</u> > **PATIENT ALERTS**.

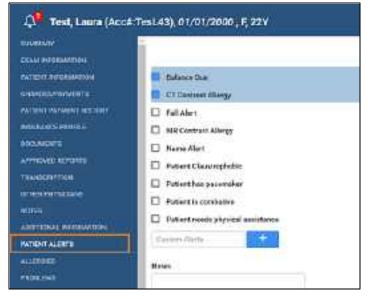

- 2. Select or clear the checkboxes corresponding to the alerts that you want to assign or remove.
- 3. Optional. To create a new alert, type it in the **Custom Alerts** box and then select the plus toutton.
- 4. Select SAVE.

Add allergy information

If your facility uses Rcopia, you can add information about a patient's allergies.

Procedure

1. Go to <u>Worklist</u> > <u>Edit Study screen</u> > **ALLERGIES**.



- 2. Select **ADD**, and then add the information as needed.
- 3. Select **SAVE.**

View problems

You can view problems associated with the study.

Procedure

1. Go to <u>Worklist</u> > <u>Edit Study screen</u> > **PROBLEMS**.

SIMMAT	
DOW INFORMATION	
INTERVISE COMATION	NO. COOL DESCRIPTION
CHARLES PAYNEW TE	No records found
PATIENT ENVIRENT HEILTERY	
NS RANCE PROPERTY	
ROCI RIMATE I	
APPROVED REPORTS	
TRANSCRIPTION	
COLD PHYSICIALS	
NOTES	
ADDITIONAL INFORMATION	
PATION ALDITS	
ALLERGES	
PROLONS	
aruoss	

View prior studies and add a new study

You can view a list of prior studies for a patient, and view reports attached to the priors. You can also start the workflow to add a new study from this part of the Edit Study screen.

Procedure

1. Go to <u>Worklist</u> > <u>Edit Study screen</u> > **STUDIES**.

Taxa and the second sec	Î.						100		A CONTRACTOR OF A	NTERNE NTERNE
	1000	interests.	100	90	el la			artyrines arms		m
SOCIETING			41	- 14	-					
And International Contract			sensitive family	44			***	Tay arrighter		- 3 (d+)
Statement of the local division of the local	1000		high the second	=			1100	Preservation		
	1.1		investigation franks	14			144	INTERNATION OF COMMUNICATION OF COMMUNICATIONO OF COMMUNICATIONO OF COMMUNICATIONO OF COMMUNICATIONO OF COMUNICATIONO OFO OFO OFO OFO OFO OFO OFO OFO OFO	0.270	644
tame to the	1.5		meth for Inda	-			-	In the supervision	wore:	
NTCO:	1.1	1010103-001	everythe fact factory	19		See.	+*	(in dissection)	Loid wet (critcher)	-
COLUMN DE DATE	10.15		ANT DE TRATE	m	- 04		10.1		NUTSAN STUDIET	10.04.50
NUMBER ADDRESS		entration nut in	transmisted being	-	atu	mini i	-	advantation to the	U/CT N	-
sianata	0		AND REPORTED	14.	1.10			DRUG AND AND	1000	11.44
101 March		STREET, SHIP	Aventa Sea tarma	- 11	10	terni i	-	10PED-0712-1511	1000	- 60
aurus Inter	100	CHINESE IN PR	Averagia for Lectral	14.	- 19		1446	I DRUG AND HIS	0.000	1.90
	1.1	in many loss in	trouble lat Large	14		in the second	10.	IONGISCEN WHI	1 1000	60

- 2. Optional. If a prior study includes reports, select the reports \square button on the study row to open the reports.
- 3. Optional. To add a new study, select **NEW STUDY** and then see <u>Set up a patient</u>.

Add a custom form to a study

Custom forms are non-interactive electronic forms intended for delivering information, whereas *study forms* can both deliver and gather information (see later topic in this section).

You can open a custom form template, fill it out, save the completed form to the current patient record, sign it electronically, fax, and print it.




Prerequisite:

- <u>Create a custom form</u>.
- To make electronic signatures available, add a signature merge field when setting up the form.

Procedure

- 1. In the worklist, select the edit \checkmark button of a study.
- 2. In the Edit Study screen, on the left pane, select CUSTOM FORMS.

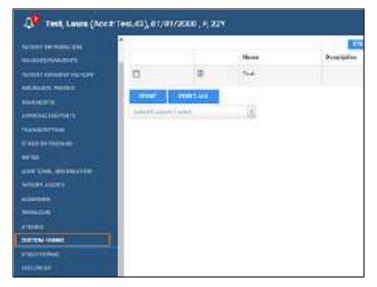

- 3. In the Select Custom Forms dropdown list, select a form to add it to the forms table.
- 4. To view, modify, or sign a form, select the form \square button in the table.
- 5. Optional. To add an electronic signature, select **SIGNATURE**, enter signature information, sign by drawing with the mouse, and then select **SAVE**.
- 6. Optional: To print the form, select **PRINT**.

7. Optional: To fax the form, select **FAX**, select the recipient physician, enter optional information under **Other**, and then select **SEND FAX**.

Add a study form to a study


Custom forms are non-interactive electronic forms intended for delivering information (see earlier topic in this section), whereas *study forms* can both deliver and gather information. You can open a study form template, fill it out, save the completed form to the current patient record, sign it electronically, and print it. You can also make forms available on the patient portal so that patients can fill them out before an appointment.



Prerequisite: Create a study form.

Procedure

1. Go to <u>Worklist</u> > <u>Edit Study screen</u> > **STUDY FORMS**.

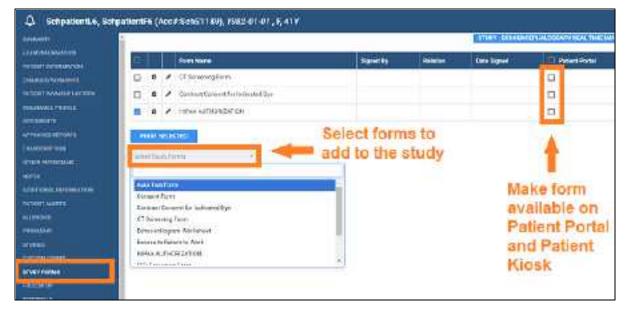

- 2. In the **Select Study Forms** dropdown list, select a form to add it to the forms table.
- 3. To view, fill in, or sign the form, double-click it in the table.

CT Screening Form				
	ADD SIGNATURE	PRIMI	SAVE	SAVE TO PATIENT DOCUMENTS
			Comput	ted Tomography (CT) Screening
exc)		D	PATIENT Eric Test OB:01/01/2000 MRN:TesE1077
			(19) 	ACCESSION:1192
			REFERR	ING PHYSICIAN: Referring Test
CHIEF COMPLAINT/REASON FOR ST	TUDY:			
🔲 No 🔲 Yes Is your reason for	this explain related to an	injury? Data of	injury:	
How were you injured? 🔲 MVA 🛛	🗆 Work 🔲 Other			

- 4. Fill in as necessary, and then select SAVE.
- 5. Optional. To add an electronic signature, select **ADD SIGNATURE**, enter signature information, sign by drawing with the mouse, select **SAVE** or **SAVE TO PATIENT DOCUMENTS**, and then close the signature dialog.
- 6. Close the form.
- 7. Optional: To print forms in the list, select their checkboxes and then select **PRINT SELECTED**.
- 8. Optional: In the forms table, next to the **Patient Portal** column header, select the checkbox to make the forms available in the patient portal.

See also:

Draw on a study form image

Draw on a study form image

While filling out a form, you can draw basic shapes on any images included in the form.



Procedure

1. Go to <u>Worklist</u> > <u>Edit Study screen</u> > **STUDY FORMS**.

- 2. Double-click the form name to open it.
- 2. Hover over an image on the form to display the drawing toolbar.



1 2 3 4 5 6 7

Do any of the following:

- Select **Color Picker** (7) and then select a color to apply to items that you draw.
- Select **Rectangle** (2), **Ellipse** (3), and then drag on the image to place the shape.
- Select **X-Stamp** (4), and then click to place the stamp.
- To edit a shape, select **Select** (5), select the shape, and then drag to move the shape, or drag the shape's handles to resize.
- To delete selected shapes or stamps, select them and then select **Delete** (6).
- To delete all shapes and stamps, select **Clear** (1).

See also:


Add a study form to a patient record

Specify a follow-up exam

If a patient needs a follow-up exam, you can specify the need for the follow up and the general time frame. To schedule follow-up exams, see <u>Schedule a follow-up</u>.

Procedure

1. Go to <u>Worklist</u> > <u>Edit Study screen</u> > **FOLLOW UP**.

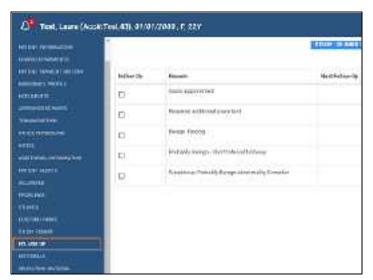

- 2. In the Follow Up column, select a reason for the follow-up.
- 3. In the **Next Follow-Up** column, select a time horizon for the follow-up.

4. Select SAVE.

Refer a patient

Referrals is a legacy Meaningful Use feature that is no longer available.

View activity logs

You can view a log of program activity and export the log to Excel.

Procedure

1. Go to <u>Worklist</u> > <u>Edit Study screen</u> > **ACTIVITY LOG**.

- veniminwe			91490-C401	our present of the particular of the state of the
2000-011-010-000000 10-000-010-000-000-00	heres BARLOND TRADER AND	11) P	Te REPORTED AND MARKED	
	Alexandra d			
	() 🖬 mericinan kranten bar		and the	lakal men di den hann Tinng an and
erte	0 E 100000 1010 10 107	Tates i Maria	Ren Vin-	control patient along
are cons promotion.	a Storade trace ut		ADDIAL.	taatte athet ant
1.000.0000	0. B (1/4/302113) (411)	(011111044)	matter ((investments)
and the second sec	D B REPORTED Instanting	Ninerghanian	emiliei I	why new largering Provide Card Selecting which
rtalet urmunitale nuomuni ulumuni 19990000000000000000000000000000000000				

- 2. Change the range of dates to filter the activity that you want to view.
- 3. To view details of an entry, double-click the entry.
- 4. Optional. To export the longs, select **EXPORT TO EXCEL**.

Exam screen for technologists

Users signing in with a technologist user account see a technologist-specific version of the Edit Study screen called the *Exam screen for technologists* (or the *Technologist screen*) that helps them work more efficiently. To open the screen, select the edit

CensusI4, Cen	susf4 - 01/01/1999 - CenC103	Δ ····· ···
1235		Thirty balls and the second se
Appointment Type	OF DOP HOND FTS PUBLICITY	
Sorly Part	Select 🖌	
Orientiation	Select w	
		Tablet Market More (1
Scheduled Date	ONCOURSE DATION ID/	
Durwton	12 / 12 minutes	Mildy Holes
I must be		

Exam screen for technologists: upper-left (left) and upper-right (right)

See also:

<u>Configure a technologist</u> <u>Use the exam screen for technologists</u> <u>Add charges and NDCs in the exam screen for technologists</u>

Use the exam screen for technologists

To use the exam screen for technologists (the *Technologist screen*):



Prerequisite: Configure a technologist.

Procedure

- 1. Sign in as a technologist (or a user linked to a technologist).
- 2. Select the edit dutton in a worklist study row.

Result: The Technologist screen opens.

3. Near the upper-left part of the screen, select a study tab (an accession number).

1207	192	Q
Againmin Typ Baily Pin: Diversitie	tit menanaskern tinbr enternet (g lans = lans =	Annual Annual Annual Annual Annual
formers Dav Danton Looren Horschy Pour	Interface of the second s	304.596.0.505
Roburny Prairies Langengelingenation Descring Properties	Landon by Second Processor [2] Landon (control) Processor [4] Landon Decemp Processor [4]	And here to broken here the second se
Anama <u>n</u> Technologia	Metal Research (2)	Studies Study Forms
		🗄 Add Charges

4. Proceed with technologist workflow, using the settings and tools described in the following table.

Setting	Description
φ.	View patient alerts.
Appointment Type	If the original appointment type was incorrect or otherwise changed after scheduling, you can select a new one. Note: It may be necessary to authorize all or parts of any new appointment type you select.
[Other settings]	Select as needed.
TECH START/END	Select upon starting and ending the exam. Exam duration statistics are recorded based on the times these buttons are selected, and are used in reporting.
TECH PAUSE	Select when an exam is interrupted, such as by a difficult patient.
UNREAD	Select to reset the study's status to Unread.
Notes (Study, Schedule, Patient, Reason)	Select a category of note, and then type notes. If macro notes were configured (see <u>Configure macro notes</u>), you can select them from the dropdown list.
ABORT	Select to abort the exam, such as in an emergency.

- 5. At any time, select **SAVE** to save changes.
- 6. In the lower-right part of the screen, you can use the following:

Setting	Description
Documents	Select to view or add documents to the study or order.
	See <u>View or add patient documents</u> .

Setting	Description
Studies	Select to view a list of prior studies for the current study. Opens the STUDIES tab of the Edit Study screen.
Study Forms	Select to add, edit, or delete study forms from the study.
	Opens the STUDY FORMS tab of the Edit Study screen.
Add Charges	Add new charges to an exam (see <u>Add charges and NDCs in the exam</u> <u>screen for technologists</u>).

7. When the exam is complete, select **TECH END**, and then **SAVE**.

Add charges and NDCs in the exam screen for technologists

Technologists can add charges to an exam if needed for such things as contrast material. The charges correspond to CPT or health service codes, and any associated NDC codes.

Prerequisites:

- <u>Configure NDCs</u>
- To make NDCs available here, add a CPT code with NDCs to the appointment type being performed. See <u>Configure appointment types</u>.


Procedure

 \checkmark

- 1. Open a study in the **Technologist** screen, and then select **Add Charges**.
- 2. In the Add Additional Charges dialog, select ADD CHARGE.

31		10
1.043		
(4)	1	
ĺ		

3. In the **CPT Code** column, select the CPT code for the additional charge.

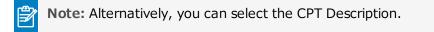

4. Optional: In the **NDC** column, select the add **•** button, and then search for a code to add in the **Search NDC Codes** list.

units of

Search NDC Code *	Search ArDit Cryde		
NOG Package Code		Package Description	
Strongth			
		Lobeler Name	
that of Message			
Let Number ?		Expiration Date *	YYYY MM DO ET
Amount Used *		Unita *	
Routo	Salact w	Administration Sta	Select or
Needla Gauge	Select w	Administered By	Belect 👻
Reaction	🖾 Yea		

5. Modify the following settings as necessary.

	Note: Use the same	Note: Use the same value for the Amount used and Units .						
	Setting	Description						
NDC Package Code		Type the code from the manufacturer.						
	Strength	Type the strength of the active ingredient, then select the strength in the list.						
	Unit of Moscuro	Type the unit of moscure						

	strength in the list.					
Unit of Measure	Type the unit of measure.					
Lot Number	Type the lot number of the drug.					
Amount Used	Type the amount of drug administered.					
Route	Select the administration method for the drug.					
Needle Gauge	if a needle was used, select the gauge.					
Reaction	Select if any reaction occurred.					
Package Description	Type the description of the package from the manufacturer.					
Labeler Name	Type the name of company corresponding to the labeler code segment in the Product NDC.					
Expiration Date	Type the expiration date of the drug.					
Units	Type the number units (e.g. vials) used.					
Admin Site	Select the site where the drug was administered.					
Administered By	Select the person who administered the drug.					

6. Select **SAVE**.

See also:

Use the exam screen for technologists Add charges and NDCs for wasted contrast

Add charges and NDCs for wasted contrast

When entering charges in the Technologist screen, you may be required to enter charges and corresponding NDCs for the amount of contrast material used, and for the amount wasted (if any).

Procedure

- 1. Open a study in the **Technologist** screen, and then select **Studies** > **CHARGES/PAYMENTS**.
- 2. In the Additional Charges dialog, enter a charge for the used contrast, and select SAVE.
- 3. In the **NDC** column, select the plus **•** button.
- 4. Enter information for the NDC in the usual manner, entering the amount of contrast **used** in both the **Amount Used** and **Units** fields. Select **SAVE**.
- 5. Back in the **Technologist** screen, repeat the above steps to enter a charge and NDC for the wasted amount of contrast, entering the amount of **wasted** contrast in both the **Amount Used** and **Units** fields. Select **SAVE**.
- 6. In the **CHARGES/PAYMENTS** screen, in the **M1** box of the charge for wasted contrast, type the **JW** modifier.

										8.7	* ===	1010
		***	Attended	im .	union	principal and a second		44	Act.		-	
	4.1		-	\$100,000 (H 4) (H	100.10	81 810-0		10				
1			-011	monomer et al dat	400 S			1		1		
10	+ 1	ameter	411	anamika			COMPANY OF THE OWNER WITH A DESIGNATION OF THE OWNER OF THE	1.00	14	111		
		imaa v	ec.	4	(a	Service and the service of the servi		C)	1		116
1		100	ki .							4		
1		(1000	10-000	0-10-0				- 14	
			Past Tarrent	1.00.								

The two charges should appear something like the following.

QC operations

All DICOM studies from other AEs initially appear on the QC tab, and are automatically evaluated for conflicts with PACS studies, RIS orders, and other data based on user-defined receiver rules. If no conflicts are found, the studies are automatically moved to the worklist (this is knows as the "QC-to-Live" process). If conflicts are found, the studies remain on the QC tab for you to reconcile.

The following topics in this section describe how to use the QC tab and reconcile these conflicts.

<u>Prepare the QC tab</u> <u>Reconcile conflicts by matching to a RIS order</u> <u>Reconcile conflicts by manually editing demographics</u> <u>Reconcile conflicts by creating an order</u> <u>Move OC studies to the All Studies tab</u>

See also:

Understanding receiver rules

Prepare the QC tab


When received studies fail to match studies in the database, they appear on the QC tab of the worklist. To find conflicts needing reconciliation more easily, you can sort studies in the QC tab of the worklist by their QC status.



Prerequisite: Obtain QC user rights.

Procedure

- 1. Go to <u>Worklist</u> > **Settings** > <u>User Settings</u>.
- 2. Under User Settings, select Show QC.

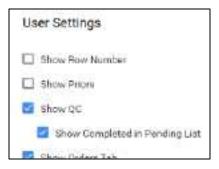

3. To show exams in Completed status, select Show Completed in Pending List.

To show exams in *Conflict* status only, clear the checkbox.

4. Select SAVE.

5. On the worklist, select the **QC** tab.

-	Worklie	H NDHRE QC	AL S	ulaes				
3		Rota	stan/a	amoust center;	ildes.	HSTINGT	MOCOLITY	STUCY DESCRIPTION
							All ~	
• 21	0.4	MP NOVA	L'e male bei	2022-06-17 9 35 AM EDT	.2	Rend, Todti	ПК	Through an
• 🐔	**	DEFAULT RULE	Completed	2022-06-09 3:29 PM EDT	444	Masquet Garlos Ariel	WR	OOLANGICREBO
• 0 •	0 = 1	MY NOVA	Coupline .	1122-06-06 & 29 MM EDL		EXA' MP1	MR	COLAMINGREES
anner.	412	Taxaa ka	and the state		14	ACCESSION OF	110	- All and the second

- 6. On the search bar, click inside the **STATUS** box.
- 7. Select a status (such as "Conflicts") and select **APPLY FILTER**.

See also:

Reconcile conflicts by matching to a RIS order Reconcile conflicts by manually editing demographics Reconcile conflicts by creating an order Move QC studies to the All Studies tab

Reconcile conflicts by matching to a RIS order

When received studies fail to match studies in the database, they appear on the QC tab of the worklist. To reconcile conflicts, start by trying to match the conflicting study to a RIS order.



Prerequisite: Prepare the QC tab.

Procedure

1. On the worklist **QC** tab, find a study whose status is **Conflicts**, and select the "reconciliation dot."



2. In the **Reconcile** dialog, under **Suggestions**, if an order in the list matches the **Current Study**, select its **SELECT** button, and then select **OK**. If no suggestion matches, skip to the next step.

Reconciliation (Study)		
SUMMARY CXAM REPORTATION PATIENT REPORTATION CHARGE LAWRING THE PATIENT PROVIDED REPORT	conflictReason : No patients matched, Move to Live option(STUDY) disabled Current Study	ļ
DESTRUCT PERMIT	Patient Nortiss DXA/INITE (EXAMP) DCB Sarviss Description Backt 6044603872401 COLLAND CREDCHANCIA* Bady Pure Study Dates 7022-00-03 5 29 PM EDT Betering Physician	
ADDITIONAL PROCEEDING	Suggestions	IIILOND
PROFERENCE STREAM	ETHEM BATE FATERATE HANKE BIDE STICH BEESEN ADDEDDEN N ADDEDDITION GODEDD. No wordd foand	PA HEITENEMSPHIPEI

- 3. Under **RIS Orders**, search for more potential matches by typing criteria in the search boxes. If a match is found, select its **SELECT** button and continue as above.
- 4. Hint: Find patients in your EHR or other sources for possible matches.

RIS Orders					RELOAD
STUDY DATE	PATIENT NAME	DOB	STUDY DEBCR ACCESSION N	AGCOUNT NO	ORDER DA' REFERING PHYS
\$EL 2022.08-01 8	30ne*Txo2	1990-10-10	BONE LENGTH TT568959930	5625142	202.2-08-01
SEL 2022-68-01 8	9 Colina_01, Rog 5 F	2000-05-04	BONE LENGTH College1_01	CellUT	2022-05-01Colina, Referring

- 5. On the QC tab, right-click the study and select Reprocess.
- 6. Confirm that the study no longer has a status of Conflicts.

See also:

Reconcile conflicts by manually editing demographics Reconcile conflicts by creating an order Move QC studies to the All Studies tab

Reconcile conflicts by manually editing demographics

When received studies fail to match studies in the database, they appear on the QC tab of the worklist. If the conflict reason on screen is **Multiple matches/Account# conflicts for patients**, you can edit the demographics of the conflicting study to match a patient in the Exa PACS/RIS database, or vice versa (edit a patient in the database to match the conflicting study).



Prerequisite: Prepare the QC tab.

Procedure

1. On the worklist QC tab, find a study whose status is Conflicts, and select the "reconciliation dot."



- 2. In **Chrome**, open another instance of Exa PACS/RIS, and on the burger end menu, select **PATIENT**.
- 3. Search for a patient chart that is likely to be the match for the conflicting study.

Hint: Find patients in your EHR or other sources for possible matches.

-	E Patient 😔								
-	Starts Wit	n y Al	• 🗆	Show inactive 🔲 Break	k the Glass				
Γ	Last Name	First	Name	123456	BBN	Phone	DOB		
	9	A CREATE NEW	SCAN LIDENSE	CLEAR SEARCH	IMPORT PATIE	NT ?			
	boost	boost'breast - FIX_123456, 123456							
	Brown	01/01/1900 iling, Patrick - 12345677 02/08/1977	128756						
	22552693	Dental*Patient - 12345677, 12345677 DOB: 10/10/2000							
		ULTRA - 123456, 1234 04/05/2015	56						


In the example, the MRN matches multiple studies -- select the correct one

- 4. Open the likely matching chart, edit demographics as necessary, and then select SAVE.
- 5. On the QC tab, right-click the study and select Reprocess.
- 6. Confirm that the study no longer has a status of Conflicts.

Reconcile conflicts by matching to a RIS order Reconcile conflicts by creating an order Move QC studies to the All Studies tab

Reconcile conflicts by creating an order

When received studies fail to match studies in the database, they appear on the QC tab of the worklist. If you cannot find a RIS order that matches or should match a conflicting study, you can reconcile by creating a new order that matches.

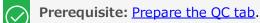

Procedure

1. On the worklist **QC** tab, find a study whose status is **Conflicts**.

Write down the key information from the study that you would need to create an order (patient demographics, MRN, study information, etc.).

- 2. Create an order for the conflicting study by following the steps in <u>A typical scheduling workflow</u>.
- 3. On the QC tab, in the conflicting study's shortcut menu, select Reconciliation.
- 4. In the **Reconciliation** dialog, under **RIS Orders**, find the order you just created and select its **SELECT** button.
- 5. On the **QC** tab, right-click the study and select **Reprocess**.
- 6. Confirm that the study no longer has a status of Conflicts.

Reconcile conflicts by matching to a RIS order Reconcile conflicts by manually editing demographics Move QC studies to the All Studies tab

Move QC studies to the All Studies tab

If you want to use a study in the QC tab even though it is not reconciled, you can manually move it to the All Studies tab.



Prerequisite: Prepare the QC tab.

Procedure

- 1. In the worklist, select the **QC** tab, and then find a study that you want to move.
- 2. In the study's shortcut menu, select **Move to All Studies**.



Note: If Move to All Studies does not appear in the shortcut menu, try selecting **Reprocess** first.

-	Worklis PRE 0	st RDERS QC	//ALL ST	IUDIES				
<		RULE	STATUS	STUDY DATE :		IMGS.	PATIENT	5
+ 0	0.*	MY NOVA	Completed	2022-06-17 9:36 AM ED	T	2	Rand, To	dd
+ 0	0 *	DEFAULT RULE	Completed	2022-06-09 9:29 PM ED	ŕ.	446	Macquel	, carlo
+ 0	• • • •	MY NOVA	Conflicts	2022-06-09 9:29 PM	Delete			
+ 0	0 +	MY NOVA	completed	2022-06-09 8:42 PM	Reproce	135		nport
+ 0	@ *	IMPORT	Completed	2022-06-09 8:42 PM	Reconci			12
+ 0	0.4	DEFAULT RULE	Completed	2022-06-09 8:42 PM	Move to	Allistudi	86	Carlos
a 104	1.142121		Contrastico.		2			

Reconcile conflicts by matching to a RIS order Reconcile conflicts by manually editing demographics Reconcile conflicts by creating an order

Work with dictation and transcription

Exa PACS/RIS offers three main solutions for dictation and transcription.

- Dictation and transcription with Exa Voice and Exa Trans
- Dictation and transcription with Exa Trans and Dragon
- Exa Dictation and transcription with Web Trans (comes standard)

Dictation and transcription with Exa Voice and Exa Trans

If you purchased Exa Voice with Exa Trans, you can dictate and transcribe simultaneously. Exa Voice is a "zero footprint" solution for higher performance.

If you did not purchase Exa Voice, you can use <u>Dictation and transcription with Web Trans</u> to dictate findings and transcribe them manually, and use the features of the multipanel for transcription related tasks.

This section contains the following topics.

Turn on Exa Voice and Exa Trans Dictate and transcribe a study Edit and approve transcriptions Add a dictation or transcription addendum Manage transcription templates Configure substitutions Configure vocabulary Customize Exa Trans Use voice commands Change text to uppercase

See also:

Assign keyboard shortcuts for Exa Trans Dictation and transcription with Exa Trans and Dragon

Turn on Exa Voice and Exa Trans

You must turn on Exa Voice and Exa Trans before using them.

Prerequisites:

- An administrator must create an Exa Voice account for each user.
- Install Exa Trans.

Procedure

 \checkmark

- 1. Go to <u>Worklist</u> > Settings > <u>User Settings</u>.
- 2. On the right side, under Voice recognition, select nVoq (Exa Trans).



3. Select SAVE.

4. Sign out of Exa PACS/RIS, and then sign in again.

Next: Dictate and transcribe a study

Parent: Dictation and transcription with Exa Voice and Exa Trans

Dictate and transcribe a study


With Exa Voice and Exa Trans you can dictate and transcribe findings simultaneously. If you pause dictation, during the next 30 seconds you can use voice commands or resume dictation without losing the connection to Exa Voice. After the 30 seconds, dictation "stops," and the connection closes. To dictate multiple studies at once, see <u>About Linked Reporting</u>.



Prerequisite: <u>Add a transcription template</u>.

Procedure

- 1. In the worklist, find an unread study and select its transcription 📓 button.
- 2. In the Exa Trans window, in the Templates dropdown list, select a template.
 - You can "call in" (open) a template by speaking a macro keyword if it appears in the template list.
 - Select **All** to show all templates, or clear to show only templates applicable to the current study.

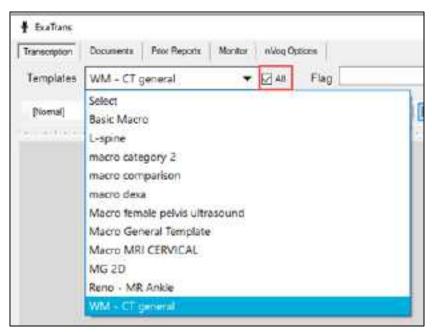

3. Press the button on the microphone, and then begin dictating. Press the button again to pause/stop.

Notes:

- Depending on configuration, you may need to hold the microphone button down continuously to record.
- You can also select the dictation 🔌 button in the bottom-right corner of the screen to start recording, and 🎐 to pause.
- You can use voice commands and buttons to move between fields (until approval).
- 4. Optional. To view the hypothesis text during dictation, select the **Show Hypothesis** link.
- 5. Select Save.

Next: Edit and approve transcriptions Previous: Turn on Exa Voice and Exa Trans Parent: Dictation and transcription with Exa Voice and Exa Trans

Edit and approve transcriptions

You can edit and approve saved transcriptions.

Note: If approving a main study, any activity related to TAT recording affects any of its linked studies (see <u>About Linked Reporting</u>).

Procedure

- 1. In the worklist, select the transcription 🔳 button of a dictated study.
- 2. In the editing screen, review the transcription, make any needed changes, and then select **Save**.
- 3. Select e-Sign & Approve, or Approve and Next.

Next: Add a dictation or transcription addendum Previous: Dictate and transcribe a study Parent: Dictation and transcription with Exa Voice and Exa Trans

Add a dictation or transcription addendum

After approval, transcriptions cannot be edited. However, physicians and transcriptionists can add addendums in order to correct mistakes or insert new information.

Procedure

- 1. In the worklist, select the transcription 📕 button of an approved study.
- 2. To add a *dictation* addendum, repeat the steps in <u>Dictate and transcribe a study</u>.

The status of the dictation reverts from Approved to Dictated.

3. To add a *transcription* addendum, select Add Addendum.

- 4. Type new text, and then select **Addendum Save**.
- 5. Select Addendum Submit for Review.

The status of the dictation reverts from Approved to Transcribed.

Previous: Edit and approve transcriptions **Parent**: Dictation and transcription with Exa Voice and Exa Trans

Manage transcription templates

In Exa Trans, you start dictation by selecting a transcription template that you prepare in advance as described in <u>Add a transcription template</u>. This topic introduces the Template Manager, which you can use to manage your transcription templates.



Prerequisites: Obtain the Transcription Template user right.

Procedure

- 1. In the worklist, find an unread study and select its transcription 🔲 button.
- 2. In **Exa Trans**, in the upper-right corner, select the transcription template manager \overline{V} button.
- 3. In the **Template Manager** dialog, on the **New** tab, you can create a new private template based on the current template.



Setting	Description
[pencil 📝 button]	Select to open the transcription template configuration screen.
Name	Type a name for the new template.
Macro Keyword	Type a macro keyword that can be voice-recognized to call in (open) the template.
Modality	Select modalities to which to apply the template.

4. Select SAVE.

5. Optional: On the **Update** and **Delete** tabs, select active non-global templates and then edit or delete them.

See also:

Dictation and transcription with Exa Voice and Exa Trans

Configure substitutions


You can add substitutions (voice macros) to Exa Voice so that you can speak something short to type something long. With the example substitution below, when you say "mammo" Exa Voice types "mammography."

Procedure

- 1. In the worklist, find an unread study and select its transcription 🔳 button.
- 2. In Exa Trans, on the nVoq Options tab, select Substitutions.

4 failes.					. Г		1
Tenerston	Documents	Ptor Flaporta	Muster	N/oc Optione		Substitutions	

- 3. Optional. To view previously saved substitutions, select **Substitution List**.
- 4. In the **New Substitution** dialog, enter the following settings.

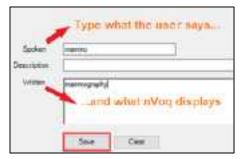

Setting	Description
Level	Select Account to make the substitution available only to you. Select Organization to make the substitution available to all.
Spoken	Type the voice macro command (the word or phrase spoken to activate the substitution).
Description	Type a description of the substitution.
Written	Type the text that nVoq substitutes (inserts) when the voice macro command is spoken.

5. Select Save.

See also:

Dictation and transcription with Exa Voice and Exa Trans

Configure vocabulary

You can add words and phrases to nVoq's vocabulary so that it can recognize proper names of people, places, product names, and uncommon words.



Prerequisite: <u>Turn on Exa Trans and nVoq</u>.

Procedure

- 1. In the worklist, find an unread study and select its transcription 📕 button.
- 2. In Exa Trans, on the nVoq Options tab, select Vocabulary.



- 3. In the Vocabulary screen, select one of the following:
 - Account The current vocabulary is available only to the current user.
 - **Organization** The current vocabulary is available to all users in the facility.



- 4. In the **Written** box, type a word or phrase.
- 5. Optional: In the **Sounds Like** box, type pronunciation.



Note: Usually leave this blank. Try adding pronunciation if Exa Voice frequently fails to recognize the word or phrase.

6. Select Save.

See also:

Dictation and transcription with Exa Voice and Exa Trans

Customize Exa Trans

If supported by your version you can modify Exa Trans settings in the Options tab, or by editing the configuration file.

Edit settings in the Options tab

In **Exa Trans**, select the **Options** tab, and then modify the following settings.

Application	Mcrophone		
There Uge ~	Hold Button 1	e Record	
Show Hypothesis	Buton Mappings	s	
👩 Pin Hypothesis	Button	Action	Parameter
Show "Enable Approvals" Checkbox	FastForward Reverse Seek/Stat Seek/End	Nast Field Frenktum Field None None	
Editor	-366(210)	TAURE	
C Keep Delimiters			
Replace Entire Word			
nVieq			
Superhubons			
Vocabulary			

Command	Description
Theme	Select Light or Dark.
Show Hypothesis	Show hypothesis text as you dictate.
Pin Hypothesis	Pin the hypothesis text to the left.
Show "Enable Approvals" Checkbox	Show the checkbox.
Keep Delimiters	When replacing variables with text, leaves the brackets (or other delimiting characters).
Replace Entire Word	Even if only a part of a word is selected, dictated text overwrites the entire word.
Hold Button to Record	Select: Hold down the button during recording.
	Clear: Press the button once to record, again to stop.
Button Mappings	Configure buttons for SpeechMikes and PowerMics.
nVoq	Available if using nVoq. See <u>Configure substitutions</u> and <u>Configure</u> <u>vocabulary</u> .

Edit the configuration file

If supported in your version, you can directly open and edit Exa Trans settings for nVoq.

- 1. In a text editor, open [drive]:/Viztek/Exa/trans/bin/KMHA.exaTrans.WinApp.exe.Config.
- 2. Edit the following lines as needed.

Setting	Line	Value
Show/hide hypothesis	<add <br="" key="ViewHypothesisText">value="1" /></add>	0 = Hide 1 = Show
text		
	<add <="" key="PinHypothesisText" td=""><td>0 = Unpin (text moves with the cursor)</td></add>	0 = Unpin (text moves with the cursor)
text	value="1" />	1 = Pin (pins text to the left side of the Exa Trans window)
Mic button triggers	<add <br="" key="ApproveOnMic">value="Off" /></add>	Off = Mic buttons do not trigger Approve and Approve & Next
Approve		On = Mic buttons trigger Approve and Approve & Next
Light or dark	<add key="Theme" value="Dark"></add>	Dark = Darker user interface
theme		Light = Lighter user interface
Recording	<add <="" key="PauseRecordingTimeout" td=""><td>Setting range: 0-30</td></add>	Setting range: 0-30
timeout	alue="30" />	0 = Recording stops immediately without entering Paused state
		30 = Recording stops 30 seconds after entering the Paused state

Dictation and transcription with Exa Voice and Exa Trans

Use voice commands

When using Exa Voice, in addition to substitutions, you can use the voice commands described in the following table.

Command	Description
[voice macro keyword]	Runs the preconfigured voice macro to open a transcription template.
Select [word or phrase]	Selects words and phrases.
	Example: Say, "Select cardiac arrest" to select the word "cardiac" followed by the word "arrest."
Select [word] through [word]	Selects a range of words
	Example: Say, "Select food through department" to select the phrase "food and beverage department."
Next variable	Selects the next set of brackets within the report and their contents.
Next field	
Previous variable	Selects the previous set of brackets within the report and their contents.
Previous field	
Next line	Adds a line after the current line
New line	
Letter	Capitalizes the current letter
Capital	
Caps	Capitalizes the first letter of the current word

Command	Description	
Caps on	Turns capitalization off the first letter of all words ON and OFF	
Caps off		
All caps	Capitalizes all letters in the current word	
All caps on	Turns capitalization of all letters of all words ON and OFF	
All caps off		
Lowercase	Sets the first dictated word in lowercase	
Overwrite		
Numeral one period	Creates a numbered list and capitalizes the first letter of each list item	
One period		
Number next	Creates a numbered list, capitalizes the first letter of each list item, and adds	
Next number	a new line before each item	
Letter next	Creates a lettered list, capitalizes the first letter of each list item, and adds a	
Next letter	new line before each list item.	
Scratch that	If text is currently selected, deletes the selection. If text is not currently	
Delete that	selected, deletes the previously transcribed phrase. Repeat the command to delete the 10 most recently entered instances of stable text.	
Undo	Undoes the 10 most recent user actions, including dictation of text or use of other voice commands (same as the Windows Undo command).	
Punctuation		
Period		
Comma	,	
Colon	:	
Semicolon	;	
Hyphen	-	
Dash		
Minus sign		
Question mark	?	
Exclamation mark	!	
Exclamation point		
Open paren	(
Open parenthesis		
Close paren)	
Close parenthesis		
Open brace	{	
Open curly brace		
Close brace	}	
Close curly brace		

Command	Description
Open bracket	[
Open square bracket	
Close bracket]
Close square bracket	
Less than sign	<
Open angle bracket	
Greater than sign	>
Close angle bracket	
Hash sign	#
Number sign	
Pound sign	
At sign	@
Asterisk	*
Asterisk sign	
Ampersand	&
And sign	
Tilde sign	~
Forward slash	1
Slash	
Back slash	λ.
Double slash	//
Quote	"
Open quote	
Open double quote	
Start quote	
Unquote	"
Close quote	
Close double quote	
End quote	
Open single quote	í
Close single quote	,
Apostrophe	,
Apostrophe S	's
Percent sign	%
Ellipsis	

Dollar sign§Equal sign=Plus sign-Plus sign·Multiplication sign·Division sign·Plus or minus sign·Underscore sign-One quarter sign/One quarter sign//Multiplication sign·Three quarter sign//March seventeen two thousand March 17, 2023Three seventeen two thousand wenty-threeMarch 17, 2023Three seventeen two thousand wenty twenty-threeThree seventeen table3/17/2023Three seventeen table3/17/2023Three seventeen tableSolon minusThree seventeen tableSolon hoursThree	Command	Description
Plus sign+Multiplication sign·Division sign·Division sign·Plus or minus sign·Vertical barIOne quarter sign½One quarter sign½Three quarters sign¾Dates and TimesMarch seventeen two thousand twenty-threeMarch 17, 2023Three seventeen twenty twenty- threeThree seventeen dash twenty twenty- threeThree dash seve	Dollar sign	\$
Multiplication sign • Division sign ÷ Plus or minus sign ± Underscore sign	Equal sign	=
Handputation sign+Division sign+Plus or minus sign-Vertical bar One quarter signVertical bar One quarter sign½Three quarter sign½March seventeen two thousand twenty-threeMarch 17, 2023March seventeen two thousand twenty-three3/17/2023Three sash seventeen twenty twenty- three3/17/2023Three sash seventeen twenty twenty- three3/17/2023Three sash seventeen dash twenty-three3-17-2023Three dash seventeen dash twenty twenty- three3-17-2023Three dash seventeen dash twenty twenty-three3/17/2023Three sash seventeen dash twenty twenty-three3-17-2023Three dash seventeen dash twenty twenty-three3/17/2023Three dash seventeen dash twenty twenty-three3-17-2023Twenteen twenty twenty-three0.000 hoursThree dash seventeen dash twenty twenty-three3-17-2023Three dash seventeen dash twenty twenty-three3-17-2023Three dash seventeen dash twenty twenty-three3-17-2023Twentinty an2:30 a.m.Setwent two p and five p mBetween 2 p.m. and 5 p.m.Zero eight hundred hours0020 hoursZero zero two two hours0022 hoursZero zero two two hours0.0022 hoursDe hundred hours100One hundred twenty120Three hundred fifty two0.5Zero point five0.5Seventy five dollars an	Plus sign	+
Plus or minus sign ± Underscore sign	Multiplication sign	*
Underscore sign	Division sign	÷
Vertical barImage: Constraint of the second of	Plus or minus sign	±
One quarter sign 1/4 One half sign 1/4 One half sign 1/4 Three quarters sign 1/4 Dates and Times 1/4 Dates and Times March seventeen two thousand twenty-three March seventeen two thousand twenty-three March 17, 2023 Three seventeen twenty twenty-three 3/17/2023 Three seventeen slash seventeen slash twenty twenty-three 3/17/2023 Three dash seventeen slash twenty twenty-three 3/17/2023 Two p m 2 p.m. Two p m 0 p.m. Zero eight hundred hours 0800 hours Ohe eight hundred hours 0800 hours Ohe dight hundred hours 0022 hours Zero zero twenty two hours 0022 hours Zero zero two hours 120 Numbers, dollars, and cents 120 One hundred fifty two 352 Point five 0.5 Zero point five 0.5	Underscore sign	_
One half sign½Three quarters sign¾Dates and TimesMarch seventeen two thousand twenty-threeMarch 17, 2023March seventeen two thousand twenty-threeMarch 17, 2023Three seventeen twenty twenty- three3/17/2023Three saks seventeen twenty twenty- three3/17/2023Three dash seventeen dash twenty twenty-three3-17-2023Two p m2 p.m.Two p m2 p.m.Zero eight hundred hours0800 hoursOn eight hundred hours0022 hoursZero zero two two hours0022 hoursDe hundred100one hundred fifty two352Point five0.5Zero point five0.5Zero point five0.5Zero point five0.5Zero point five0.5	Vertical bar	
Three quarters sign 3/4 Dates and Times March seventeen two thousand twenty-three March seventeen two thousand twenty-three March 17, 2023 Three seventeen twenty twenty-three 3/17/2023 Three seventeen twenty twenty-three 3/17/2023 Three dash seventeen slash twenty-three 3/17/2023 Three dash seventeen dash twenty-three 2 p.m. Two p m 2 p.m. Two p m 2:30 a.m. Between two p m and five p m Between 2 p.m. and 5 p.m. Zero eight hundred hours 0022 hours Zero zero twenty two hours 0022 hours Zero zero two two hours 100 One hundred wenty 120 Three hundred fifty two 352 Point five 0.5 Zero point five 5.5	One quarter sign	1⁄4
Dates and Times March seventeen two thousand twenty-three March 17, 2023 March seventeen twenty twenty-three 3/17/2023 Three seventeen twenty twenty-three 3/17/2023 Three sacks seventeen slash twenty-three 3/17/2023 Three dash seventeen dash twenty-three 3-17-2023 Two p m 2 p.m. Two p m 2:30 a.m. Between two p m and five p m Between 2 p.m. and 5 p.m. Zero eight hundred hours 0800 hours Oh eight hundred hours 0022 hours Zero zero twenty two hours 0022 hours Zero zero two two hours 120 Numbers, dollars, and cents 120 One hundred fifty two 352 Point five 0.5 Zero point five 0.5	One half sign	1/2
March seventeen two thousand twenty-threeMarch 17, 2023March seventeen twenty twenty- three3/17/2023Three seventeen twenty twenty- three3/17/2023Three slash seventeen slash twenty twenty-three3-17-2023Three dash seventeen dash twenty twenty-three3-17-2023Two p m2 p.m.Two p m2 p.m.Two thirty a m8etween 2 p.m. and 5 p.m.Between two p m and five p mBetween 2 p.m. and 5 p.m.Zero eight hundred hours0800 hoursOh eight hundred hours0022 hoursZero zero twenty two hours0022 hoursZero zero two two hours100One hundred100One hundred fifty two352Point five0.5Zero point five0.5Zero point five0.5Zero point five0.5Zero point five0.5	Three quarters sign	3⁄4
twenty-threeMarch seventeen twenty twenty- threeThree seventeen twenty twenty- threeThree seventeen twenty twenty- threeThree slash seventeen slash twenty twenty-threeThree dash seventeen dash twenty twenty-threeThree dash seventeen dash twenty twenty-threeThree dash seventeen dash twenty twenty-threeTwo p mQ p.m.Two p mQ p.m.Two thirty a mBetween two p m and five p mBetween 2 p.m. and 5 p.m.Zero eight hundred hoursOh eight hundred hoursZero zero twenty two hoursZero zero two two hoursDone hundredOne hundred twentyThree hundred fifty twoPoint fiveZero point fiveSeventy five dollars and fiftySeventy five dollars and fiftySeventy five dollars and fifty	Dates and Times	
three Image: Seventeen twenty twenty-three Three seventeen twenty twenty-three 3/17/2023 Three slash seventeen slash twenty twenty-three 3/17/2023 Three dash seventeen dash twenty twenty-three 3-17-2023 Two p m 3-17-2023 Two p m 2 p.m. Two thirty a m 2 p.m. Setween two p m and five p m Between 2 p.m. and 5 p.m. Deter two p m and five p m 0800 hours One hundred hours 0022 hours Zero zero twenty two hours 0022 hours Zero zero two hours 0022 hours Den hundred nours 100 One hundred twenty 120 Three hundred fifty two 352 Point five 0.5 Zero point five 5		March 17, 2023
threeThree slash seventeen slash twenty twenty-threeThree dash seventeen dash twenty twenty-threeThree dash seventeen dash twenty twenty-threeTwo p m2 p.m.Two p mQ p.m.Two thirty a mBetween two p m and five p mBetween 2 p.m. and 5 p.m.Zero eight hundred hoursOh eight hundred hoursCero zero twenty two hoursZero zero two two hoursDne hundredOne hundredOne hundred twentyThree hundred fifty twoSeventy five dollars and fiftySeventy five dollars and fifty		
twenty twenty-threeImage: constraint of the system of the sys		3/17/2023
twenty twenty-three2 p.m.Two p m2 p.m.Two thirty a m2:30 a.m.Between two p m and five p mBetween 2 p.m. and 5 p.m.Zero eight hundred hours0800 hoursOh eight hundred hours2Zero zero twenty two hours0022 hoursZero zero two two hours0022 hoursDone hundred100one hundred fifty two352Point five0.5Zero point five\$75.50		
Two thirty a m2:30 a.m.Between two p m and five p mBetween 2 p.m. and 5 p.m.Zero eight hundred hours0800 hoursOh eight hundred hours0800 hoursZero zero twenty two hours0022 hoursZero zero two two hours0022 hoursNumbers, dollars, and cents100One hundred100one hundred twenty352Point five0.5Zero point five\$75.50		3-17-2023
Between two p m and five p mBetween 2 p.m. and 5 p.m.Zero eight hundred hours0800 hoursOh eight hundred hours0022 hoursZero zero twenty two hours0022 hoursZero zero two two hours0022 hoursNumbers, dollars, and cents100One hundred100one hundred twenty120Three hundred fifty two352Point five0.5Zero point five\$75.50	Two p m	2 p.m.
Zero eight hundred hours0800 hoursOh eight hundred hours0022 hoursZero zero twenty two hours0022 hoursZero zero two two hours0022 hoursNumbers, dollars, and cents000One hundred100one hundred twenty120Three hundred fifty two352Point five0.5Zero point five\$75.50	Two thirty a m	2:30 a.m.
Oh eight hundred hoursImage: Constraint of the system of the	Between two p m and five p m	Between 2 p.m. and 5 p.m.
Zero zero twenty two hours0022 hoursZero zero two two hours0022 hoursNumbers, dollars, and cents0002 hoursOne hundred100one hundred twenty120Three hundred fifty two352Point five0.5Zero point five50Seventy five dollars and fifty\$75.50	Zero eight hundred hours	0800 hours
Zero zero two hoursNumbers, dollars, and centsOne hundred100one hundred twenty120Three hundred fifty two352Point five0.5Zero point five\$75.50	Oh eight hundred hours	
Numbers, dollars, and centsOne hundred100one hundred twenty120Three hundred fifty two352Point five0.5Zero point fiveSeventy five dollars and fifty\$75.50	Zero zero twenty two hours	0022 hours
One hundred100one hundred twenty120Three hundred fifty two352Point five0.5Zero point fiveSeventy five dollars and fifty\$75.50		
one hundred twenty120Three hundred fifty two352Point five0.5Zero point fiveSeventy five dollars and fifty\$75.50		
Three hundred fifty two352Point five0.5Zero point fiveSeventy five dollars and fifty\$75.50		
Point five 0.5 Zero point five Seventy five dollars and fifty \$75.50		
Zero point five Seventy five dollars and fifty \$75.50		
Seventy five dollars and fifty \$75.50		0.5
cents	Seventy five dollars and fifty	\$75.50

Command	Description
Roman fifteen	XV
(1-19 available only)	

Dictation and transcription with Exa Voice and Exa Trans

Change text to uppercase

In Exa Trans, you can change selected text from all lowercase, or mixed uppercase-lowercase, to all uppercase.

Procedure

- 1. In the Exa Trans template, select text to change.
- 2. In the upper-right corner of the screen, select the Case \mathbf{A} button.

Result: The text changes to all uppercase.

```
Example: tHis IS sample TEXT \rightarrow THIS IS SAMPLE TEXT
```

See also:

Dictation and transcription with Exa Voice and Exa Trans

Dictation and transcription with Exa Trans and Dragon

If you purchased Exa Trans with Dragon, you can dictate and transcribe simultaneously. Otherwise, you can use <u>Dictation and transcription with Web Trans</u> to dictate findings and transcribe them manually, and use the features of the Exa Panel for transcription related tasks.



Note: Often, *Transcriptionists* use Exa Dictation with Web Trans, whereas *Radiologists* use Exa Trans with Exa Voice or Dragon.

This section contains the following topics.

Turn on Exa Trans and Dragon Dictate and transcribe a study Edit and approve transcriptions Add a dictation or transcription addendum About using Exa Trans and Web Trans

Turn on Exa Trans and Dragon

You must turn on Exa Trans before using it by turning on the Dragon speech-to-text option in the user settings.



Prerequisite: Install Exa Trans.

Procedure

- 1. Go to <u>Worklist</u> > **Settings** > <u>User Settings</u>.
- 2. On the right side, under Voice recognition, select Dragon.



3. Select **SAVE.**

Next: Dictate and transcribe a study Parent: Dictation and transcription with Exa Trans and Dragon

Dictate and transcribe a study

With the integrated Dragon speech-to-text application, you can use Exa Trans to dictate and transcribe findings simultaneously.

Procedure

- 1. In the worklist, find an unread study and select its transcription 🔳 button.
- 2. In the **Exa Trans** window, in the **Templates** dropdown list, select a template.
- 3. Optional. To use a voice command, with the **Templates** dropdown list open, say, "Macro [*template name*].
- 4. Place the cursor where you want to add text.
- On the Dragon toolbar, select the record button, and then begin dictating.
 Select stop when finished.
- 6. Select **Save**.

Next: Edit and approve transcriptions Previous: Turn on Exa Trans and Dragon Parent: Dictation and transcription with Exa Trans and Dragon

Edit and approve transcriptions

In Exa Trans with Dragon, you can edit and approve saved transcriptions. If approving a main study, any activity related to TAT recording affects any of its linked studies (see <u>About Linked Reporting</u>).

 \oslash

Prerequisite: <u>Turn on Exa Trans and Dragon</u>.

Procedure

- 1. In the worklist, select the transcription 📕 button of a dictated study.
- 2. In the editing screen, review the transcription, make any needed changes, and then select **Save**.
- 3. Select e-Sign & Approve, or Approve and Next.

Next: Add a dictation or transcription addendum Previous: Dictate and transcribe a study Parent: Dictation and transcription with Exa Trans and Dragon

Add a dictation or transcription addendum

In Exa Trans with Dragon, approved transcriptions cannot be edited. However, physicians and transcriptionists can add addendums in order to correct mistakes or insert new information.



Prerequisite: Turn on Exa Trans and Dragon.

Procedure

1. In the worklist, select the transcription 🔳 button of an approved study.

2. To add a dictation addendum, repeat the steps in "Dictate and transcribe a study."

Exa PACS/RIS reverts the status of the dictation from Approved to Dictated.

- 3. To add a *transcription* addendum, select **Add Addendum**.
- 4. Type new text, and then select **Addendum Save**.
- 5. If not using Dragon, select Addendum Submit for Review.

Exa PACS/RIS reverts the status of the dictation from Approved to Transcribed.

Previous: Edit and approve transcriptions Parent: Dictation and transcription with Exa Trans and Dragon

About using Exa Trans and Web Trans

Exa Trans and Web Trans (with the multipanel) are designed to work in sync. Exact behavior depends on various rights, actions, and settings, as in the following examples.

Action/Condition	Effect
Close Exa Trans	Also closes Web Trans
Web Trans rights assigned	Web Trans available
Exa Trans rights assigned	Exa Trans available only if the Dragon user setting is also turned ON.
Auto-Open Orders + "Documents" selected	If an order auto-opens, the Documents screen auto-opens.
Auto Open Orders enabled AND Viewer Settings > General > Display Settings > Monitor 1 = Documents	Documents auto-open in the Documents screen of the multipanel in Monitor 1.
Auto Open Orders disabled AND Viewer Settings > General > Display Settings > Monitor 1 = Documents	Documents do not auto-open, but if manually opened, they open in the multipanel in monitor 1.
Exa Trans and Web Trans enabled within Auto Open Orders	Exa Trans and Web Trans both open when an order auto-opens. You can configure which monitor they open in in the Display Settings, or leave blank to open in the main monitor.
Exa Trans set to auto-open AND Web Trans not enabled in Auto Open Orders	Only Exa Trans opens when a study opens (but you can still manually open Web Trans).
Set Exa Trans or Web Trans to open on a specified monitor	Manually opening Exa Trans or Web Trans opens it in the specified monitor.
Web Trans set to auto open in Auto Open Orders, but not Exa Trans	Web Trans auto opens when the viewer is opened, in the specified monitor, if so configured.
User selects the Exa Trans or Web Trans button on the worklist	Exa Trans or Web Trans opens on the same monitor as the worklist.
Exa Trans	Study is approved, viewer closes, Exa Trans minimized.
Esign & Approve	
Exa Trans	Study is approved, viewer and Exa Trans open with next available study.

Action/Condition	Effect
Approve & Next	
Exa Trans	Study status changes to Not Approved, viewer closes, and Exa Trans
Not Approve	minimizes.
Exa Trans	Study stays in Unread status, viewer and Exa Trans open with the next
Skip	available study
Exa Trans	Study status changes to Draft, viewer and Exa Trans remain open with
Save	current study
Exa Trans	Study status changes to Pre-Approved, viewer closes, and Exa Trans
Pre Approve	minimizes
Exa Trans and Web Trans	Exa Trans closes, Web Trans minimizes or closes
Esign & Approve	
Exa Trans and Web Trans	Study status changes to Approve, viewer, Exa Trans, and Web Trans open
Approve & Next	with the next available study
Exa Trans and Web Trans	Study status changes to Not Approved, viewer, Exa Trans, and Web Trans
Not Approve	minimize or close.
Exa Trans and Web Trans	Study status remains Unread, the viewer, Exa Trans, and Web Trans open the
Skip	next available patient or study.
Exa Trans and Web Trans	Viewer, Exa Trans, and Web Trans remain open. If the user closes the
Save	viewer, Exa Trans and Web Trans also close.
Exa Trans and Web Trans	Study status changes to Pre-Approved, viewer and Web Trans close.
Pre Approve	

Next: Edit and approve transcriptions

Previous: Turn on Exa Trans and Dragon

Parent: Dictation and transcription with Exa Trans and Dragon

Exa Dictation and transcription with Web Trans

As the entry-level dictation and transcription solution in Exa PACS/RIS, radiologists can dictate findings in Exa Dictation, and the transcriptionist can transcribe them in Web Trans. Web Trans features "the multipanel," a centralized screen for managing transcriptions and other assets. For more advanced solutions, see <u>Dictation and transcription with Exa Trans and Dragon</u> and <u>Dictation and transcription with Exa Voice and Exa Trans</u>.

This section contains the following topics.

Turn on Exa Dictation and Web Trans Dictate a study Edit and approve transcriptions in Web Trans Enter notes in Web Trans Manage documents in Web Trans Manage prior reports in Web Trans

Turn on Exa Dictation and Web Trans

To get started, configure Voice Recognition as follows.



Prerequisite: The dictating radiologist and transcriptionist must both <u>Install Exa Dictation</u> on their workstations.

Procedure

- 1. Go to <u>Worklist</u> > **Settings** > <u>User Settings</u>.
- 2. On the right side of the screen, under **Voice recognition**, select the following:

Vision recognition	Diagonie 😁
	Dragen (Coa Trans)
Delay Load of Eas Trate	Detation (WebTrans)
	Other (this Train/SD)
Exe Trans Font	Off (Nune)

- <u>Transcriptionist</u>: **Dictation (WebTrans)**
- <u>Radiologist</u>: If you will be signing off on reports, select **Dictation (WebTrans)**. Otherwise, select **Off (None)**.
- 3. Select SAVE.

Next: <u>Dictate a study</u> Parent: <u>Exa Dictation and transcription with Web Trans</u>

Dictate a study

In Exa Dictation, radiologists can record dictation without simultaneous transcription by using the dictation toolbar



Prerequisite: Add a transcription template

Procedure

- 1. In the worklist, double-click an unread study to open it in the viewer.
- 2. When prompted, select to open Exa Dictation or Web Trans.
- 3. Select a template from the dropdown list, or use a voice macro to open it.
- 4. On the dictation toolbar, select the record \mathbf{W} button, and then dictate your findings.
- 5. Use **Pause** during dictation if needed, and when finished, select the **Stop** button.
- 6. To review your dictation, rewind the recording and then select the play button.

Note: You can record over parts of your dictation that you want to change.

7. When you are satisfied with the results, select the upload up button to add your dictation to the study.



Note: Skip this step if you selected Auto Upload Dictation on Close (see <u>Configure autosave</u> <u>options</u>).

Next: Edit and approve transcriptions in Web Trans Previous: Turn on Exa Dictation and Web Trans Parent: Exa Dictation and transcription with Web Trans

Edit and approve transcriptions in Web Trans

The Web Trans transcription editor does not currently support dictation playback, but you can edit and approve submitted transcriptions.



Note: If approving a main study, any activity related to TAT recording affects any of its linked studies (see <u>About Linked Reporting</u>).

Procedure

- 1. On the worklist, in the relevant study row, select the multipanel 📰 button.
- 2. In the Web Trans multipanel, on the toolbar, select one or more of the following buttons:



- 1 Open or close all panels
- 2 Open or close the **Transcription** panel.
- **3** Open or close the **Notes** panel
- 4 Open or close the **Documents** panel
- 5 Open or close the **Prior Reports** panel

Note: You can drag panels within the Web Trans screen to reposition them.

- 3. In the **Transcription** panel, review the transcription, make any needed changes, and then select **Save**.
- 4. Select Approve or Approve & Next.
- 5. In the **Providers** dialog, select the approving provider and select **Approve**.

Next: Enter notes in Web Trans Previous: Dictate a study Parent: Exa Dictation and transcription with Web Trans

Enter notes in Web Trans

You can add study notes and reasons for study in the Notes panel of Web Trans.

Procedure

- 1. In the multipanel, open the **Notes** panel, and select the **STUDY NOTES** or **REASON FOR STUDY** tab.
- 2. If entering a study note, select a study date.
- 3. Type notes in the space provided, and then select **SAVE**.


Next: <u>Manage documents in Web Trans</u> Previous: <u>Edit and approve transcriptions in Web Trans</u> Parent: <u>Exa Dictation and transcription with Web Trans</u>

Manage documents in Web Trans

You can view all documents attached to a study in the Documents panel of Web Trans. The Documents panel displays up to two documents at once, and you can select which documents to display.

Procedure

- 1. On the worklist, in the relevant study row, select the multipanel \blacksquare button.
- 2. In the **Documents** panel, select the document buttons to show or hide currently open documents in viewing frames.

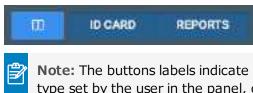

Note: The buttons labels indicate the document selected for viewing, the default document type set by the user in the panel, or the last-viewed document types.

3. If more than two documents are available, you can open a new one by dragging a document metadata block from the top of the panel into a document viewing frame.

		Documents	
Show All Documenta	SET AS DEFAULT	2	367
Insurance Card D	Order/Prescription (11/01/2021	Order/Prescription	Order/Prescription 11/01/2021
Order/Prescription (2)	1 Order/Bracocintion	Deter/Resectation	Order/Prescription 11/03/2001
0ther C	onto buttor		Patien Forms 12/10/2021
Patient Forms	03/31/2021	03/31/2021	07/20/2021
Patient Portal Ipload 🗍 08/18/2021	Patient Portal Upload (06/18/2021	Patient Portal Upload B8/18/2021	Pasient Portal Upla 01/21/2021
Prior Report	Tech Worksheet (07/23/2021	3 Unknown 0 10/11/2021	
DROP DOL	MENT TO PREVIEW	DROP DOCUMENT TO PIEVIE	

Next: Manage prior reports in Web Trans Previous: Enter notes in Web Trans

Parent: Exa Dictation and transcription with Web Trans

Manage prior reports in Web Trans

You can view, download, and print prior reports from the Prior reports panel of Web Trans.

Procedure

1. On the worklist, in the relevant study row, select the multipanel 🖽 button.

If the report is linked, the link \Im symbol appears in the block.

2. In the **Prior reports** panel, in the left pane, select the block of metadata corresponding to the report that you want to view.

8	Prior reports	6
PE/22-0021 11-12 PM MRL MITTOPLUE DOS 104PLE : CLERENT	🚍 trasscriptor-landlet anter 👔 († 2. – 1945.) + 🖽 🕹	
Sector Call	See Addendum Below	а
NULISISSES POD AM MILISIACHISTO MALARCOMUPHY	Potiert Name Testy, Jewafer R DOB: 25-Mar-1968	

3. To download or print the report, in the right pane, hover over the report, and then select the download or print button in the toolbar that appears.

Previous: <u>Manage documents in Web Trans</u> Parent: <u>Exa Dictation and transcription with Web Trans</u>

Work with the Exa PACS/RIS viewer

This chapter contains the following sections (not all sections may be available depending on version and region).

Standard viewing tasks Configure integration with third-party apps Lines, curves, shapes, and text Markers **Measurements** Spine labels <u>3D</u> **Shutters** Other annotation tasks Change elements in the viewer Scaling images Linking images Moving and jumping to images Use MPR tools Work with PET fusion studies Gather information from images

See also:

Work with Display Management

Standard viewing tasks

This section contains the following topics (not all topics may be available depending on version and region).

Open a study in a viewer View priors and prior reports About viewing CAD findings View studies with CAD findings Send an image from the viewer DICOM-print a study Delete images and series

Open a study in a viewer

In this manual, *viewer* refers to the Exa PACS/RIS viewer unless otherwise specified. To open a study, on a worklist study row, do one of the following:

• Select the 🔎 button to open the Exa PACS/RIS viewer.



- Select the 💌 button to open the Opal viewer.
- Configure Exa PACS/RIS to open studies in the viewer when you double-click them. Follow the steps in "Miscellaneous user settings" to configure the double-click behavior setting.



Caution: Check for the presence of the lossless indicator in the lower-left corner of the image frame before proceeding with a diagnosis. If you adjust the image, wait until the lossless indicator turns green before diagnosis.



The red lossless indicator means the image has not yet fully loaded.

View priors and prior reports

From the Exa PACS/RIS viewer, you can open priors from a list or the prior bar, and open prior reports.

Open a prior from a list

1. On the viewer toolbar, select the **Show Prior List button** to open the list.

				Pitur Patient Exams					
0	R	M	Description	Referring	Exam Date	Status			
	R	U5	DOP VELOCIMETRY FTLUMBILICALART	Test Reterring	01/19/2023 11:15 AM	Approved			
	R*	115	DOP ECHO FTL SPECTRAL DISPLAY COMPL	Test Reterring	01/19/2823 11:00 AM	Approved			

If an **R** appears in the R column, the prior includes a report. If the priors are linked:

- **R'** The report is the main report.
- **R*** The report is a linked report (linked to the main report).
- 2. In the list, double-click a prior to open it in the viewer.

Open priors with the Prior bar

If priors are included in a study that you open, you can view them by using the Prior bar. To show the Prior bar in the viewer, configure the **Use Modified Toolbar** setting (see <u>Configure other interface</u> <u>settings</u>).

LCC RCC LMLQ RMLQ CVIEW RC			18335 PM dard Screening - 6 D	NG, Stan Tomohio 1 Y S M 1	is to:17 Am Imography, Ing digital: T	OST1/201 MG, MAN SCREEN CURREN
b d b d	BTO RCC	OVIEW RC	RMLQ	LMLO	RCC	LCC

The Prior bar contains the current and prior studies on the top row, and their image thumbnails on the bottom row (see figure). The study boxes show the age of the study, and depending on configuration, let you do one or more of the following:

- Hover over a study to display its details.
- Select a study to open the thumbnail bar.
- Double-click a thumbnail to open its image or series.
- If a report is included, select its symbol to view.

Open a prior report

- 1. On the viewer toolbar, select the Prior Reports 尾 button.
- 2. In the **Approved Reports** screen, on the left pane, select a prior report.

Result: The report appears in the right pane (may take time to load).



Note: If reports are linked, the link symbol appears (in the red circle in the figure).



About viewing CAD findings


The Exa PACS/RIS viewer automatically displays image markers contained in SR data from supported manufacturers such as iCAD, DiA, Hologic (Malc findings, see below), and CureMetrix (such as the CureMetrix mammography calcification and density markings). To control which markings to display, select options in the SR CAD list (<u>Configure mammography</u>).

Exa PACS/RIS uses tracking IDs in the SR data to ensure that when you click a 2D/synthesized image annotation it jumps correctly to the corresponding 3D/BTO image. Additionally:

- If the BTO series is not already open, it opens next to the 2D/synthesized view in a 1 x 2 layout. You can go back to the original layout by selecting the backslash key (if the DM was not interrupted).
- When 3D CAD SR markings are present and you hover over a CAD SR marking on the synthesized or 2D view, the color changes to the hover color configured in viewer interface settings.

Hologic Malc findings

Hologic ImageChecker CAD has a finding type called a Malc, which is a combination of one mammography breast density and one calcification cluster. When Malc findings are present in a CAD SR, the viewer displays them as a "compass" shape. The center point marking of the compass is determined by the center point of the child breast density. The total count of Malc findings appears in the top overlay next to the count for densities and calcifications.

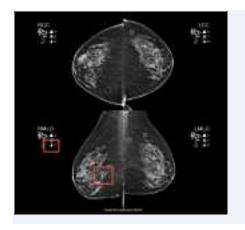



The Malc "compass" marking and count

Hologic Intelligent View

Hologic 3D CAD mammography software features a synthesized "Intelligent View," which is acquired at a higher resolution than their C-View. To identify Hologic Intelligent View images, "i2D" is added to the image header, such as i2D LXCC. For example, the header of the image below in Exa PACS/RIS would appear as **i2D RMLO**.



See also:

<u>Configure annotation colors</u> <u>View studies with CAD findings</u>

View studies with CAD findings

The Exa PACS/RIS viewer supports 2D CAD findings for mammography and 3D tomographic series. Findings are displayed as overlays. To view CAD findings:

Prerequisite (Exa PACS/RIS only): Configure mammography (especially SR CAD options).

Procedure

1. <u>Open a study</u> containing CAD findings in the viewer.

Result: The CAD button becomes available.

2. Select the CAD we button, and then select findings to show.

Example: Calcification Clusters(3) > Show

3. Optional: Select a finding to select a value for the 2D or synthesized view.

Result: The screen changes to a 1×2 temporary view with the 2D image in one cell and the BTO series moved to the slice with the finding in the other. For example, if you select the RCC finding, the BTO series opens at the RCC slice.

CAD-related keyboard shortcuts

To easily move through series that include 3D CAD findings, you can use keyboard shortcuts (see <u>Assign toolbar keyboard shortcuts</u>). For example, you can use a keyboard shortcut to jump to the next finding in a BTO study, or turn Malc markings on and off.

See also:

About viewing CAD findings

Send an image from the viewer

You can send the current image in the viewer to an application entity. If the image includes annotations, those annotations are included with the image according to the method you specify in the **Send Annotations as** setting (see <u>Configure application entities</u>).

Procedure

- Optional: To send an annotated image back to its original AE, select Viewer Settings > General > Auto Send Annotated Images on Close.
- 2. <u>Open a study in a viewer</u>.
- 3. Optional: Add annotations.



Note: ROI annotations are automatically sent as inclusive.

4. Right-click the image and select **Send Image/Annotation**.



5. In the **Send Study** dialog, in the bottom pane, select one or more AEs.

ACCESSION NO. 1	TWO YOUTE	MODALE BEUDY E	COCRIPTION	ADDOUNT	THO, FATIENT	DOD	REFERRING PHYCROLAN
ozikeleszakos k	12.408/3029 T DB 4	RT PISKOU	TO THIR WOT	82589	SIL Ion J	08/09/19/1988	Bassett, bennis Raedolpt
/ikurijearspti	20 Q	Priority	ų	3	🗌 Freihele Appro	une: Report	SEND TO SELECTED
	209 Q		u	2	🗌 Freihale Appro	uwel Respons	BEND TO SELECTED

6. Select SEND TO SELECTED or SEND TO ALL.

See also:

DICOM-send studies, series, or images

DICOM-print a study

You can print the currently viewed image to a preconfigured DICOM printer. The current image prints, including any displayed annotations and overlays. See also <u>Print images to a standard printer</u>. You can also add selected images and series to the DICOM print queue, and print them later from the queue. This can be useful for large studies (see later in this topic).




Note: You cannot print multiframe MR/CT or MG tomography images, or US echocardio series.



DICOM-print the current image or series

- 1. <u>Open a study</u> in the Exa PACS/RIS viewer, right-click any image, and then select **DICOM Print**.
- 2. In the **DICOM Print** dialog, in the **Printer** dropdown list, select a printer.

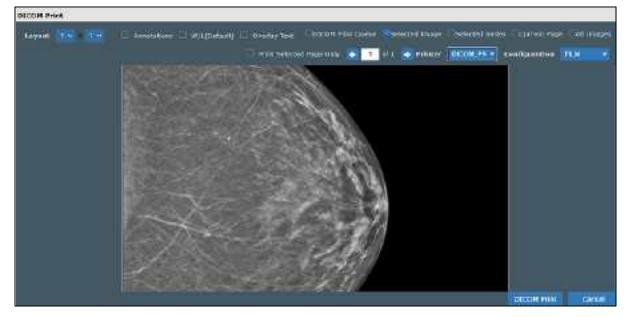

3. Enter the following remaining settings.

Setting	Description
DICOM Print Queue	Select to print the current contents of the DICOM print queue.
Selected Image	Select to print selected image, selected series, current page of images, key
Selected Series	images, or all images.
Current Page	
Key Images	
All Images	
Layout	Select Selected Series above, and then adjust the frame layout.
	The layout you select applies to the current and subsequent pages, and you can change layouts on any page.
Annotations	Select to include annotations, or clear to exclude.
W/L (Default)	Select to print the image with its original window/level settings. Clear to select the current window/level setting.
Overlay Text	Select to include the DICOM overlay (as per-image detail), or clear to exclude.
Print Selected Page Only	If viewing a series, select to print only the current page.
Prev / Next	Select to move through pages of the print preview.


Setting	Description
Printer	Select a printer (selected earlier in this procedure).
Configuration	Select a printer configuration for the target print medium. To add or edit configurations, see <u>Configure a DICOM printer</u> .

4. Select **DICOM Print**.


Use the DICOM print queue

You can add images and series to the DICOM print queue, and then print everything in the queue at once.

 On an image shortcut menu, select Add to DICOM Print Queue - Image or Add to DICOM Print Queue - Series.

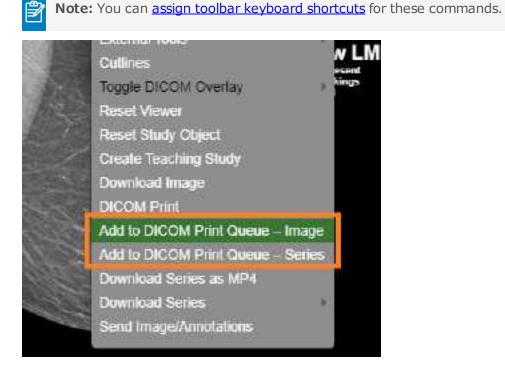

2. On an image shortcut menu, select **DICOM Print**, and then in the DICOM Print preview, select the **DICOM Print Queue** option.

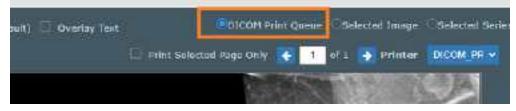

3. Select other options as needed, and then select **DICOM Print**.

Delete images and series

You can delete images and series from unread studies. When you delete a series, only images in the current stack and cell are deleted (even if a series is split across multiple cells). You cannot delete MPR-generated images.



Prerequisite:

- Obtain the **Study Delete** right from your administrator.
- Add the **Delete Series** and/or **Delete Images** commands to the shortcut menu (see <u>Add</u> <u>or remove tools from the image shortcut menu</u>).

Procedure

1. In the Exa PACS/RIS viewer, right-click an image or series.



2. In the shortcut menu, select **Delete Image** or **Delete Series**.


Configure integration with third-party apps

The following topics in this section describe integration with third-party apps. With the Exa platform, refer to topics in which you can integrate third-party apps such as PenRad, OrthoView, and various RIS solutions.

Configure opening of a third-party program Configure opening of PenRad Configure opening of MModal Configure opening of OrthoView Open a study in OrthoView Configure opening of Visage RIS Configure integration with a third-party RIS

Configure opening of a third-party program

You can configure Exa PACS/RIS to open third-party programs such as Ikonopedia, PowerScribe 360, and MModal.

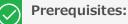

- Install Exa Launcher
- Add a trusted IP or URL

Procedure

- 1. Go to <u>Viewer Settings</u> > <u>GENERAL</u>.
- 2. Under **Display Settings**, in the table of monitors, on the row corresponding to the monitor on which to open the third-party application, select the row's **DISPLAY** checkbox.

Monitar	Current	Prior	Full Screen	Orders		Туре	Calibration Width (mm)	Celibration Height (mm)
DISPLAY1	0	0		P\$360	6	~	(

- 3. In the row's **Orders** dropdown list, select the application (such as PS360 or MModal).
- 4. In the left pane, select **INTERFACE**.
- Under Other Toolbar, in the Auto Open Orders group, select the application (such as PS360 or MModal).



- 6. In the left pane, select **TOOLS**.
- 7. Under **Toolbar**, select **MAIN**, and then in the right pane, select the **External Tools** checkbox.

Foolbar		Dot
CT	•	Edit Annotation
>>MAIN		flipse
>>TOP		External Tools
VARIATION		FH Measure

8. In the left pane, select **EXTERNAL TOOLS**.

GENERAL	Title *				Title
INTERFACE	Description *		2	æ	pervad
	Commend				
MODALITY	Argumenta				
MAMMOGRAPHY	initial Director				
	Keyboard Sho				
OVERLAYS	Toolbar Butto	n			
	Do not Besed	64 encode U\$	E angu	ments	
TOOLS	Use facility ID	for subdired	tory		
EXTERNAL TOOLS	ADD CLEAR				

9. Enter the settings in the following table, and then select **ADD**.

Setting	Description	
Title	Type a title for the program.	

Sotting	Description
Setting	Description
Description	Type a description of the program.
Command	Type the command to run when you select the toolbar button (such as the path to the executable, or a URL).
Arguments	Type arguments to pass to the command at runtime.
	Exa PACS/RIS supports the following arguments:
	{{account_no}} (patient ID)
	{{id}} (study ID) {{accession_no}} {{facility_id}}
	{{username}}
	{{user_id}}
	{{patientName}} {{patientDOB}} {{patientSex}} {{studyDescription}}
	{{series_uid}}
	{{image_uid}}
	To base-64 encode the URL, surround the arguments with the \$\$ marker. For example:
	exal://launch/q= \$\$ patient_id={{account_no}}&accession_no={{accession_no}} &cmd=powerscribe&study_id={{id}} \$\$
Initial Directory	Type the path to the root folder.
Keyboard Shortcut	Optional: Type a keyboard key sequence to use as a shortcut for opening the program.
Toolbar Button	Optional: Select to add a dedicated button for the application on the viewer toolbar.
Do not Base-64 Encode URL Arguments	
Use Facility ID for Subdirectory	Do not select (reserved for PenRad).

See also:

<u>Configure opening of PenRad</u> <u>Configure opening of MModal</u> <u>Configure opening of OrthoView</u>

Configure opening of PenRad

You can configure Exa PACS/RIS to open mammography and ultrasound breast studies in PenRad by selecting a button on the viewer toolbar. Each instance of PenRad automatically opens in the configuration corresponding to the facility ID of the study. You must perform this configuration procedure for each user.



Prerequisite: Install Exa Launcher.

Procedure

- 1. Follow the steps in <u>Configure opening of a third-party program</u>, with the following modifications.
 - In **Display Settings**, in the **Orders** dropdown list, select **PenRad**.
- 2. In the EXTERNAL TOOLS screen, Enter the following settings, and then select ADD.

Setting	Description
Title	PenRad
Description	PenRad
Command	exal://launch/q=
Arguments1 (variable filename)	<pre>patient_id={{account_no}}&accession_no={{accession_no}} &cmd=penrad&study_id={{id}}&facility_id={{facility_id}}</pre>
Arguments2 (filename)	<pre>patient_id={{account_no}}&accession_no={{accession_no}} &cmd=penrad&study_id=file&facility_id={{facility_id}}</pre>
Initial Directory	[Leave blank]
Keyboard Shortcut	Optional: Type a keyboard shortcut.
Toolbar Button	Select (adds the PenRad button to the viewer toolbar).
Do Not Base-64 Encode URL Arguments	
Use Facility ID for Subdirectory	Optional: Select to open different instances of the application that are configured for different facilities on the same Exa PACS/RIS installation.

1. Creates a study_id.xml file (e.g. if the study ID is 1234, the file created is 1234.xml).

2. Always creates a file.xml file regardless of the study ID.

Configure opening of MModal

You can configure Exa PACS/RIS to open mammography and ultrasound breast studies in MModal by selecting a button on the viewer toolbar. Each instance of MModal automatically opens in the configuration corresponding to the facility ID of the study. You must perform this configuration procedure for each user.



Prerequisite: Install Exa Launcher.

Procedure

- 1. Follow the steps in <u>Configure opening of a third-party program</u>, with the following modifications.
 - In **Display Settings**, in the **Orders** dropdown list, select **MModal**.
- 2. In the EXTERNAL TOOLS screen, Enter the following settings, and then select ADD.

Setting	Description
Title	MModal
Description	MModal
Command	exal://launch/q=
Arguments1 (variable filename)	<pre>patient_id={{account_no}}&accession_no={{accession_no}} &cmd=mmodal&study_id={{id}}</pre>
Initial Directory	[Leave blank]
Keyboard Shortcut	Optional: Type a keyboard shortcut.
Toolbar Button	Select (adds the MModal button to the viewer toolbar).
Do Not Base-64 Encode URL Arguments	
Use Facility ID for Subdirectory	Optional: Select to open different instances of the application that are configured for different facilities on the same Exa PACS/RIS installation.

- 1. Creates a study_id.xml file (e.g. if the study ID is 1234, the file created is 1234.xml).
- 2. Always creates a file.xml file regardless of the study ID.
- 3. The XML file contains the MRN.

Configure opening of OrthoView

You can configure Exa PACS/RIS viewer to open a study in OrthoView by selecting the external app button on a worklist study row.



Prerequisite: Install Exa Tool.

Procedure

- On the server, use a text editor such as Notepad to open the exatool.toml file (default location is C:\viztek\exa\cfg).
- 2. Add or edit the file to include the following within the **[tools]** section:

```
# Materialize OrthoView
[tools.orthoview]
launchURL = `http://server_address:90\OrthoView\start.htm'
```

- Where server_address is the IP or fqdn of the server hosting OrthoView.
- If needed, substitute http with https.
- 3. Save and close the file.
- 4. On the burger Honner menu, select **Setup**.
- 5. On the OFFICE menu, select User Management.
- 6. On the **EXTERNAL APPS** tab, select **Add**, and type the following settings.

Integration Name OrthoView

URL exatool:///?tool=orthoview&study_id={{study_id}}&username={{username}}

Note: You can use the following variables in place of, or in addition to those shown in colored text on the previous line.

{{accession_no}}	The accession number of the study
{{account_no}}	The account number of the patient

- {{facility_id}} The database ID for the facility of the study
- {{study_id}} The database ID for the study
- {{user_id}} The database ID for the currently signed-in user
- 7. Select **SAVE**.

See also:

Configure opening of a third-party program

Open a study in OrthoView

If OrthoView is installed on your server, you can use the Exa PACS/RIS worklist or viewer to open studies in OrthoView.



Prerequisite: Configure opening of OrthoView.

Procedure

- 1. On the worklist, select the external app 💷 button.
- 2. In the button shortcut menu, select **OrthoView**.

Result: Your browser downloads a .jnlp file.

3. Open the .jnlp file.

Configure opening of Visage RIS

You can configure Exa PACS/RIS to integrate with Visage RIS.

Prerequisite: <u>Install Exa Launcher</u> and <u>Install Exa Tool</u>.

Enable integration with Visage RIS

- 1. Go to <u>Viewer Settings</u> > <u>INTERFACE</u>.
- 2. In the Auto Open Orders list, select Visage RIS.



Select a monitor

- 1. In the left pane, select **GENERAL**.
- 2. Under **Display Settings**, on the **Orders** list of the monitor to use, select **Visage RIS**.

GENERAL	Display Set	tings					
INTERFACE	Monitors on Syst	en	+	¢©			
MODALITY	Monitor	Current	Prior	Full Screen	Orders	Туре	
MAMMOGRAPHY	DISPLAY1	0	0		ExaTrane2 M 👻	L.	
OVERLAYS	Autosave Options			Falide Ups Bionopiedia Inimuniciationia Insurance Profile Lab Gräens MMedal			
TOOLS	Autopave Image Properties on Close Medic				Medical History Medicaliants		
	Autosave Image Annotations on Close				P\$360 Patient Alecta	- 1	
EXTERNAL	Auto Mark Read Study on Close			Patient to fo Penited			
TOOLS	Auto Uploed Dictation on Close				Prescriptions Prior Studies Problems Referrals Referrals		
	Autosave Viewer Settings on Close						
	Auto Open Next Study						
	Auto Send Annotated Images on Close				Visage R/S		

Create a profile for Visage RIS

1. In the left pane, select **EXTERNAL TOOLS**.

GENERAL	Title *	Visage
INTERFACE	Description *	Visage R/S
	Command	exat://launch/q=
MODALITY	Argumenta	accession_no=((accession_nc))\$
MAMMOGRAPHY	Initial Directory	
MAMMOGILAPHY	Keyboard Shortcut	
OVERLAYS		🛃 Toolbor Button
TOOLS		Do not Base-64 encode URL argaments
routs		Use facility ID for subdirectory
EXTERNAL TOOLS		ADD DLEAR

2. Enter the following settings, and then select **ADD**.

Setting	Description
Title	Any, such as "Visage"
Description	Any, such as "Visage RIS"
Command	exal://launch/q=
Arguments	accession_no={{accession_no}}&cmd=visageris
Initial Directory	[Leave blank]
Keyboard Shortcut	Optional: Type a keyboard shortcut.
Toolbar Button	Select (adds a Visage RIS button to the viewer toolbar).
Do Not Base-64 Encode URL Arguments	
Use Facility ID for Subdirectory	[Leave cleared]

3. Select **SAVE**.

Configure integration with a third-party RIS

This is an example in which Exa Tool was configured to generate an XML file in a pre-determined directory that a third-party RIS application could read to open a study from Exa PACS/RIS. Contact your Konica Minolta representative for assistance with your particular application.

1. Create an Exa Tool profile to generate the following XML file.

```
<?xml version="1.0"?>
<ChunkData>
<Event>
<Event>Login</Event1>
<User>
<LoginID>{{username}}</LoginID>
<Key>{{dataHash}}</Key>
<UtcTimestamp>{{UTC_Timestamp}}</UtcTimestamp>
</User>
</Event>
</ChunkData>
```

Setting	Description
{{username}}	The signed-in user. Must be all lowercase.
	Example: jwilliams
{{UTC_Timestemp}}	Current UTC DTS in the format: YYYY-MM-DDTHH:MM:SS
	Example: 2022-12-16T17:00:00
{{dataHash}}	The HMAC data must be hashed in the following format, including commas and spaces, but omitting ().
	SHA-256/HMAC(sharedSecret, lowercaseUsername, UtcTimestamp) w/Base-64 output.
	Example where RIS XML Integration Key value (shared secret)= AAAAAA.
	HMAC(AAAAAA, jwilliams, 2022-12-16T17:00:00)
	Results in:
	mj+darm8HKqNaiYaf9xiBckhXQgGjzGGqmQgxAN14QM=

2. In the Exa Tool profile, configure the path and file name of the XML file as follows:

```
C:\Temp\Study.xml
```

3. In this example, the resultant file contents are as follows:

```
<?xml version="1.0"?>
<ChunkData>
<Event>
<Event1>Login</Event1>
<User>
```

```
<LoginID>jwilliams</LoginID>
<Key>mj+darm8HKqNaiYaf9xiBckhXQgGjzGGqmQgxAN14QM=</Key>
<UtcTimestamp>2022-12-16T17:00:00</UtcTimestamp>
</User>
</Event>
```

</ChunkData>

Lines, curves, shapes, and text

You can draw linear shapes and add text to images.

Draw lines and shapes Draw freehand lines Draw a curved shape Add text

Draw lines and shapes

You can draw rectangles and ellipses on images.



Prerequisite: Add the relevant toolbar tools.

Procedure

1. On the viewer toolbar, select one of the following tools.

П

12

- Rectangle
- Ellipse
- Straight line
- Arrow
- 2. Drag on the image to create the shape.

Optional:

- To move a shape, drag any part of its outline.
- To resize a shape, drag any of its handles.

Draw freehand lines

You can draw freehand lines on images.



Prerequisite: Add the relevant toolbar tools.

- 1. On the viewer toolbar, select the pencil \blacksquare tool.
- 2. Drag on the image to draw a freehand line.
- 3. Optional. You can drag any points along the line to modify it.

Draw a curved shape

You can draw curved shapes on images.



Prerequisite: Add the relevant toolbar tools.

Procedure

- 2. Click the image at least three times to specify a start point, a vertex, and a second vertex.
- 3. Double-click to close the shape.

Add text

You can add a text annotation to an image.



Prerequisite: Add the relevant toolbar tools.

Procedure

- 1. On the viewer toolbar, select the text ${\rm I\!I}$ tool.
- 2. In the box, type text and then select Enter.



Note: There is no character limit, but carriage returns are not supported, and long text may extend beyond the visible image boundary.

Markers

You can add markers to images, such as for left and right.

Add a dot Add a vertical or horizontal line Add a left or right marker

Add a dot

You can add a dot marker to an image.



Prerequisite: Add the relevant toolbar tools.

Procedure

- 1. On the viewer toolbar, select the dot $\hfill \ensuremath{\square}$ tool.
- 2. Click the image to place the dot.

Add a vertical or horizontal line

You can add a vertical or horizontal line to an image.



Prerequisite: Add the relevant toolbar tools.

Procedure

- 1. On the viewer toolbar, select the vertical **I** or horizontal **I** line tool.
- 2. To add the line, click the image.
- 3. Drag the line to position it.

Add a left or right marker

You can add a left or right marker to an image.



Prerequisite: Add the relevant toolbar tools.

Note: To reveal the endpoints, you can drag a vertical line up or down, or a horizontal line left or right.

- 1. On the viewer toolbar, select the left 0 or right 0 marker tool.
- 2. To add the marker, click the image.
- 3. Drag the marker to reposition it.

Measurements

You can take various linear and angular measurements of images. Completed measurements remain as annotations. To adjust measurements easily, we recommend turning on Edit mode before taking measurements (see <u>Edit annotations</u>).

Measure an angleMeasure a Cobb angleMeasure an HO angleMeasure a radial angle or lengthMeasure an elliptical or freehand ROIMeasure an elliptical or freehand ROIMeasure and compare two lengthsTake a freehand measurementFind a center pointDraw a circle and measure its radius

Measure an angle

You can measure an angle on an image.



Prerequisite: Add the relevant toolbar tools.

Procedure

- 1. On the viewer toolbar, select the angle dol.
- 2. Click the image three times to place a vertex and two endpoints.
- 3. Move the pointer to position the angle measurement, and then click to finish.

Measure a Cobb angle

You can measure a Cobb angle on an image.



Prerequisite: Add the relevant toolbar tools.

- 1. On the viewer toolbar, select the Cobb angle 🜌 tool.
- 2. Drag on the image to place the first angle line.
- 3. Drag again to place the second line.
- 4. Move the pointer to position the angle measurement, and then click to finish the angle.

Measure an HO angle

You can measure an HO angle on an image. An HO angle measures the degrees of the angle drawn relative to true vertical and horizontal.



Prerequisite: Add the relevant toolbar tools.

Procedure

- 1. On the viewer toolbar, select the HO angle \mathbb{Z} tool.
- 2. Click the image to place a starting point.
- 3. Click to place the end point.

Measure a radial angle or length

You can measure a radial angle or length on an image.



Prerequisite: Add the relevant toolbar tools.

Procedure

- 1. On the viewer toolbar, select the radial angle 🖿 or radial length 🛅 tool.
- 2. Click the image to place the axes.
- 3. Drag the axes or their endpoints to take measurements.

Measure an elliptical or freehand ROI

You can measure properties of an ROI by drawing an ellipse or a freehand shape. The viewer displays the ROI's length (mm), area (mm²), and the minimum, maximum, mean, and standard deviation in Hounsfield units.



Prerequisite: Add the relevant toolbar tools.

- 1. On the viewer toolbar, select the ROI 🔤 or free ROI 🔄 tool.
- 2. Drag on the image to place the ellipse or the freehand shape.
- 3. Drag the outline or handles of the shape to adjust the ROI.

Measure a length

You can measure a length (mm) on an image.



Prerequisite: Add the relevant toolbar tools.

Procedure

- 1. On the viewer toolbar, select the measure 🖉 tool.
- 2. Drag on the image to place the measurement line.
- 3. Drag the outline or endpoints of the line to adjust the measurement.

Measure and compare two lengths

You can measure the lengths (mm) from two starting points to an endpoint for comparison.



Prerequisite: Add the relevant toolbar tools.

Procedure

- 1. On the viewer toolbar, select the M compare 🔳 tool.
- 2. Click the image three times to place a starting point, endpoint, and second starting point.
- 3. Drag any of the three points to adjust the measurement.

Take a freehand measurement

A freehand measurement is similar to freehand ROI, but it measures different values: angle; radius (mm); area (mm²); and length (circumference in mm).



Prerequisite: Add the relevant toolbar tools.

- 1. On the image shortcut menu, select **Annotations** > **Measures** > **FH Measure**.
- 2. Click the image three times to place a vertex and two endpoints.
- 3. Move the pointer to position the circle measurements, and then click to finish.

Find a center point

You can find the center between two reference points.



Prerequisite: Add the relevant tools to the image shortcut menu.

Procedure

- 1. On the image shortcut menu, select **Annotations** > **Measures** > **Center Point**.
- 2. Click the image twice to place the first and second reference points.



Note: The center point is indicated by an x.

3. Drag either of the reference points to adjust the measurement.

Draw a circle and measure its radius

You can draw a circle and measure its radius (mm).



Prerequisite: Add the relevant toolbar tools.

- 1. On the viewer toolbar, select the circle \square tool.
- 2. Click the image twice to place two loci.
- 3. Move the pointer to adjust the circle, and then click to add a third locus.
- 4. Drag any locus to resize or reposition the circle.

Spine labels

You can add preset labels for spinal vertebrae and discs to MR images. Labels are grouped by region (cervical, thoracic, lumbar, and sacral), plus another group for all discs. Basic labels do not include height and width measurements, whereas non-basic labels do. 3D labels appear in all frames and planes, whereas non-3D labels appear only in the individual frames to which you add them.

Add basic spine labels Add basic 3D spine labels Add 3D spine labels Use a spine label shortcut

Add basic spine labels


You can add basic spine labels to vertebrae in ascending or descending order.



Prerequisite: Add the relevant toolbar tools.

Procedure

- 1. On the viewer toolbar, select the basic spine label \square tool.
- 2. In the button shortcut menu, select a region and starting vertebra.

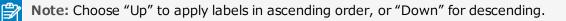

3. Click the first vertebra to add the first label, and then click the second vertebra, and so on.

Add basic 3D spine labels

You can add basic 3D spine labels to vertebrae in ascending or descending order.



Prerequisite: Add the relevant toolbar tools.

Procedure

- 1. On the viewer toolbar, select the quick 3D spine label $\stackrel{[mathbb{m}]}{>}$ tool.
- 2. In the button shortcut menu, select a region and starting vertebra.



Note: Choose "Up" to add labels in ascending order, or "Down" for descending.

3. Click the first vertebra to add the first label, and then click on the second vertebra, and so on.

Add 3D spine labels

You can add 3D spine labels in ascending or descending order.



Prerequisite: Add the relevant toolbar tools.

Procedure

- 1. On the viewer toolbar, select the 3D spine label shortcut \blacksquare tool.
- 2. In the button shortcut menu, select a region and starting vertebra.

Note: Choose "Up" to add labels in ascending order, or "Down" for descending.

- 3. Click a vertebra twice to place a starting point and endpoint.
- 4. Repeat the previous step to label additional vertebrae.

Use a spine label shortcut

You can use a spine label shortcut to quickly start labeling from a common starting vertebra.



Prerequisite: Add the relevant toolbar tools.

- 1. On the viewer toolbar, select the spine label shortcut Z, 3D spine label shortcut , or 3D quick spine label shortcut Z tool.
- 2. In the button shortcut menu, click a starting vertebra.
- 3. Add labels.

3D

3D labels appear in the image where you add them, and the same slice in other series.

Use the 3D cursor Add a 3D point Add a 3D ray Add a 3D ROI Add a 3D length measurement Add a 3D angle measurement

Use the 3D cursor

When you place the 3D cursor on one series, the cursor also appears in the corresponding location in any cell of the same study that is open in the viewer.




Prerequisite: Add the relevant toolbar tools.

Procedure

- 1. Open two or more series of the same study (for example, an axial, sagittal, and coronal view).
- 2. On the viewer toolbar, select the 3D cursor 🖻 tool.
- 3. Hover over an image in one cell.
- 4. The cursor appears in the same location in the other cells.

Add a 3D point

You can mark a 3D point on an image. The point you place in the current image appears in the corresponding position in all frames and planes.

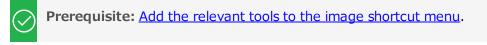

- 1. On the image shortcut menu, select **Annotations** > **3D Point**.
- 2. Click the image to place the point.

Add a 3D ray

You can mark a 3D ray on an image. The vertex you place in one frame extends as a ray through the stack and is visible in other planes.



Prerequisite: Add the relevant tools to the image shortcut menu.

Procedure

- 1. On the image shortcut menu, select **Annotations** > **3D Ray**.
- 2. Click the image to place the vertex of the ray.

Add a 3D ROI

You can mark a circular ROI in one image, and the program adds the ROI to the same image in other series in the viewer. The viewer displays the ROI's length (circumference in mm), area (mm²), and the maximum, minimum, mean, and standard deviation in Hounsfield units.



Procedure

- 1. On the image shortcut menu, select **Annotations** > **3D ROI**.
- 2. Click the image twice to place the center point and a tangent point.

Add a 3D length measurement

You can take a linear measurement (mm) in one image, and the program adds the measurement to the same image in other series in the viewer.



Prerequisite: Add the relevant tools to the image shortcut menu.

- 1. On the image shortcut menu, select **Annotations** > **3D Measure**.
- 2. Click the image twice to place the first and second endpoints of the measurement line.

Add a 3D angle measurement

You can take a 3D angle measurement in one image, and the program adds the measurement to the same image in other series in the viewer.



Prerequisite: Add the relevant tools to the image shortcut menu.

- 1. On the image shortcut menu, select **Annotations** > **3D Angle**.
- 2. Click the image three times to place the vertex and the first and second endpoints of the angle.

Shutters

You can add rectangular, elliptical, or freehand shutters.

Add a rectangular or elliptical shutter Add a freehand shutter

Add a rectangular or elliptical shutter

You can add a rectangular or elliptical shutter to "crop" unwanted areas of an image.



Procedure

- 1. On the image shortcut menu, select **Annotations** > **Shutters** > **Shutter Box** or **Shutter Ellipse**.
- 2. Click the image twice to place a starting point and endpoint for the shutter.

Add a freehand shutter

You can add a freehand-drawn shutter to "crop" unwanted areas of an image.

Prerequisite: <u>Add the relevant tools to the image shortcut menu</u>.

- 1. On the image shortcut menu, select **Annotations** > **Shutters** > **Freehand Shutter**.
- 2. Drag on the image to draw the shutter.

Other annotation tasks

The following topics introduce additional annotation features.

Add an AR annotation

Edit annotations Saving and deleting annotations Annotate ultrasounds with DiA Annotate series Annotate tagged images Reset an image in the viewer

Add an AR annotation

You can plot points to represent angular rotation.



Prerequisite: Add the relevant toolbar tools.

Procedure

- 1. On the viewer toolbar, select the AR M tool.
- 2. Click the image six times to place guide points.

Edit annotations

After you finish an annotation, it is fixed in place to prevent accidental alteration. To move, edit, or delete an annotation, either select the tool that created it, or select the edit is tool to enter Edit mode. You can select the edit tool again to exit Edit mode.

Saving and deleting annotations

You can save or delete annotations as follows.



- To save the current annotations, on the toolbar, select the save annotations ${}^{\hbox{\mbox\m\m\m\m\m\m\m\m\m\m\mbox{\mbox{\mbox{\mbox{\m$
- To delete all annotations, on the toolbar, select the delete all annotations 🔤 button.
- To delete all annotations for the current series, on the image shortcut menu, select **Delete Series Anno**.

 To delete all annotations for the current study, on the image shortcut menu, select **Delete Study** Anno.

Annotate ultrasounds with DiA

With the DiA ultrasound analysis application you can automatically annotate 2-, 3-, and 4-chamber cardio ultrasounds, and populate SDE reports with SR measurements.

Caution: Incorrectly tagging cardio images can result in misdiagnosis. It is the user's responsibility to correctly tag series as 2-, 3-, or 4-chamber studies for purposes of DiA analysis. Neither Konica Minolta nor DiA Imaging Analysis are responsible for any outcomes resulting from mis-tagged images.



Prerequisite: Purchase a license for DiA.

Procedure

- 1. Open one or more ultrasound images or series in the Exa PACS/RIS viewer.
- 2. On the toolbar, select the **DiA** white button.
- 3. In the center of each open frame, select the **2CH**, **3CH**, or **4CH** button to tag the image or series as a 2-chamber, 3-chamber, or 4-chamber item.



Note: Select only one type.

- 4. In the upper right corner of each frame, select the **SG**, **EF**, or **Both** button to perform annotation and measurement.
- 5. To save the results, select the save annotations \square button.
- 6. To view the SR data, open the study in the cardio report editor.

Annotate series

You can add the same annotation in the same location to all images in a series. This feature is not available with 3D annotations. Any subsequent edits you make to annotations added in this way only apply to the image you edit, not the entire series.



Prerequisite: Add the relevant toolbar tools.

Procedure

1. In the Exa PACS/RIS viewer, open a series.

- 2. On the toolbar, select the **Annotate Series** ^{ASS} tool.
- 3. Select another annotation tool (for example, the Ellipse tool).
- 4. Add the annotation to the current image (for example, draw an ellipse).

Result: The annotation (such as an ellipse) appears on all images in the series.

Annotate tagged images

To annotate all images in a series, see <u>Annotate series</u>. If you want to annotate multiple—but not all images in a series, use the procedure below to *tag* (select) them and then annotate only the tagged images. Any subsequent edits you make to annotations added in this way only apply to the image you edit, not the entire series. This feature is not available with 3D annotations.




Prerequisite: Add the relevant toolbar tools.

Procedure

- 1. Open series or multiple images in the Exa PACS/RIS viewer.
- 2. On the viewer toolbar, select the **Tag Images III** tool.
- 3. Click on an image to tag it.

Result: The image is tagged for annotation.

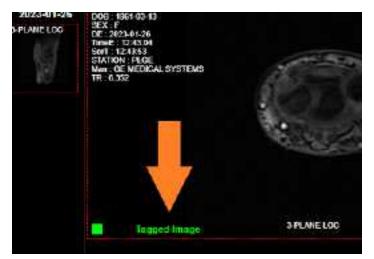

- 4. Repeat for all images you want to annotate.
- 5. On the toolbar, select the **Annotate Tagged Images M** tool.
- 6. Select another annotation tool (for example, the Ellipse tool).
- 7. Add the annotation to one of the tagged images (for example, draw an ellipse).

Result: The annotation (such as an ellipse) appears on all tagged images, and the images are no longer tagged.

ÿ

Note: These multi-image annotations are not "linked," so you can edit annotations in one image without modifying the corresponding annotations in other images.

Reset an image in the viewer

When making non-permanent modifications to images in the viewer (zoom, rotation, Window/Level etc.), you can reset the image to its original state.



Prerequisite: Add the relevant toolbar tools.

• On the viewer toolbar, select the reset viewer G button.

Change elements in the viewer

The following topics are contained in this section.

Show or hide overlays Show or hide annotations Show series and stacks Show or hide cutlines Split series by slab thickness Recalibrate the scale Change the series layout Reorder a series

Show or hide overlays

You can turn overlays on or off. This includes PR (GSPS) DICOM objects from Intelerad, which are supported as overlays.



Prerequisite: Add the relevant toolbar tools.

 On the viewer toolbar, select the Overlays or Toggle DICOM Overlay button to turn overlays on or off.



See also:

Configure overlays

Show or hide annotations

You can turn annotations on or off.



Prerequisite: Add the relevant toolbar tools.

Procedure

• On the viewer toolbar, select the toggle annotations button to turn annotations on or off.



Show series and stacks

If the viewer is showing only a single frame of a series or stack, you can change it to display all series and stacks.



Prerequisite: Add the relevant toolbar tools.

● On the viewer toolbar, select the reset series and stacks Select the reset series and stacks and stacks select the reset series and series series and select the reset series and series series series series series and series series series series and series s

Show or hide cutlines

Cutlines show the intersection between two open series. The cutlines update dynamically as the user moves through a series. A dashed cutline indicates where the first image starts and the last image ends. A solid cutline indicates the intersection of the currently displayed locations in the series. If a series includes intersecting images, you can show or hide cutlines.



Prerequisite: Add the relevant toolbar tools.

• On the viewer toolbar, select the cutlines 🖻 button.

Split series by slab thickness

You can use slab tools to split series by image thickness.

Procedure

- 1. In the upper-right corner of a frame, right-click the menu icon \blacksquare .
- 2. In the shortcut menu, select **Slab Tools**, and then select a slab thickness.



Recalibrate the scale

You can manually recalibrate the scale measurement, which is shown on the caliper of the current series.

- 1. In the upper-right corner of a frame, right-click the menu icon \blacksquare .
- 2. In the shortcut menu, select **Recalibrate**.



3. In the dialog, type a new value (mm) for the scale, and then select **OK**.

Change the series layout

You can change the number of series panels displayed in the viewer.

Procedure

- 1. In the upper-right corner of a frame, right-click the menu icon \blacksquare .
- 2. In the shortcut menu, select **Series Layout**, and then select a layout.



Reorder a series

You can change the order of images in a series.

- 1. In the upper-right corner of a frame, right-click the menu icon \blacksquare .
- 2. In the shortcut menu, select **Reorder**, and then select one of the following options.



Image Number – Arranges the images by number in ascending order.

Image Number Inverse – Arranges the images by number in descending order.

Slice Location – Arranges the images by relative location (in mm) in the plane in ascending order.

Slice Location Inverse – Arranges the images by relative location in the plane in descending order.

Image Time – Arranges the images by the time they were taken.

Change the appearance of images

The following topics are contained in this section.

How the viewer chooses the initial window/level Adjust the window/level Apply a window/level preset Turn on auto window/level Apply CLAHE enhancement Apply bone enhancement Invert colors Add color to images Apply sigmoid processing Reset and Reset study object Add to new UNQ study

How the viewer chooses the initial window/level

There are many sources of window/level settings. When the viewer displays an image, it chooses the window/level settings from the first available of the following sources, in order.

- Last display state, if "Auto saved image properties on close" is turned on
- LUT (from prefetch), unless "Ignore LUT" or "Ignore Presentation LUT" is turned on
- DICOM tags the window center and window width tags
- VOI LUT DICOM tags value of interest LUT tags
- Manual W/L settings, if adjusted by the user

Adjust the window/level

You can manually adjust the window/level.



Prerequisite: Add the relevant toolbar tools.

- On the viewer toolbar, select the window/level 🔯 button.
- Drag horizontally over the image to adjust the window.
- Drag vertically over the image to adjust the level.

Apply a window/level preset

You can apply a window/level preset to the current study.



Prerequisite: Add the relevant tools to the image shortcut menu.

• On the image shortcut menu, select **Presets**, and then select a preset.

Turn on auto window/level

The viewer can automatically adjust the window/level so that images are, to the extent possible, neither too dim nor too bright.



Prerequisite: Add the relevant toolbar tools.

• On the viewer toolbar, select the auto window/level 🔳 button.

Apply CLAHE enhancement

You can apply CLAHE to enhance local contrast.



Prerequisite: Add the relevant toolbar tools.

• On the viewer toolbar, select the CLAHE 💶 button to turn CLAHE on or off.

Apply bone enhancement

You can use different levels of bone enhancement to optimize the contrast between bone and other tissues. You can also apply color inversion to further highlight fractures and other features of bone.



Prerequisite: Add the relevant toolbar tools.

- 1. On the viewer toolbar, select the bone enhance w button.
- 2. In the button shortcut menu, select a percentage.

Invert colors

You can invert black and white values to make certain images easier to read.



Prerequisite: Add the relevant toolbar tools.


• On the viewer toolbar, select the invert colors 🔟 button.

Add color to images

You can add color to images.

Procedure

1. Right-click the overlay in the lower-right of the image, and then in the shortcut menu, select **Palettes**.

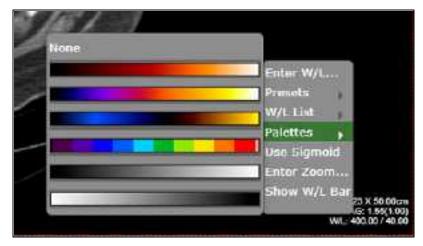

2. Select a color in a palette.

Apply sigmoid processing

You can apply sigmoid processing to enhance low-contrast images.

• Right-click the overlay in the lower-right of the image, and then in the shortcut menu, select **Sigmoid**.

Reset and Reset study object

You can access the Reset and Reset study object commands in the image shortcut menu. *Reset* undoes all unsaved changes (such as W/L and annotations) made to the current series. *Reset study object* updates the viewer after new images are added to a study that is currently open.

Add to new UNQ study

A *UNQ* (unique) study is a teaching study. When you select this command in the image shortcut menu, a button appears on the toolbar to open the teaching study screen. See, "Create a teaching study."

Scaling images

The following topics are contained in this section.

Zoom images Display one to one Display images in their actual size Fit images to window Use the magnifying glass

Zoom images

You can zoom in on (enlarge) or out of (reduce) images for optimum viewing. There are several ways to zoom:



Prerequisite: Add the relevant toolbar tools.

- On the viewer toolbar, select the zoom 🖸 tool and then drag on the image.
- Select the zoom tool in the image shortcut menu: Tools > Zoom.
- Drag the pointer across a caliper (if Caliper Zoom is selected, see <u>Configure other interface settings</u>).
- Right-click the lower-right overlay, select Enter Zoom, and then type a value from .05 to 7.5.
- "Quad zoom": Use the left or right angle bracket keys (< >).

Note: Available for image data only. If Skin Line is turned ON, when using quad zoom for mammography images, the zoom factor is based on the skin line to ensure that anatomy gets maximal placement within the cells.

Display one to one

If you zoomed an image you can restore it to its original DICOM size. This is essentially "life size."



Ŵ

Prerequisite: Add the relevant toolbar tools.

• On the viewer toolbar, select the one to one 🛄 button.

Display images in their actual size

You can display images in their original size.



Prerequisite: Add the relevant toolbar tools.

• On the viewer toolbar, select the actual image size 🛄 button.

Fit images to window

You can automatically zoom an image so that its longest dimension fits in the visible frame.



Prerequisite: Add the relevant toolbar tools.

• On the viewer toolbar, select the fit to window $\begin{tabular}{ll} \hline \end{tabular}$ button.

Use the magnifying glass

You can use the magnifying glass tool to zoom an area of the current image.



Prerequisite: Add the relevant toolbar tools.

• On the viewer toolbar, select the magnifying glass tool, and then select and hold over the area to magnify.



Linking images

When two series are open side-by-side in the viewer, *linking* them causes moving through images in a second series when you move through images in the first series. Refer to the following topics.

Auto link images Manually link images Use the context tool Reset linked series

Auto link images

Auto linking means that linking turns on automatically when two or more series of the same plane (such as axial) are open. With auto-linking, the system uses the image numbers to automatically match up the frames in each series so that the frame in the second series is as near in stack order to the image in the first series as possible.



Prerequisite: Add the relevant toolbar tools.

• To auto link images, on the viewer toolbar, select the auto link 🔤 button.

Manually link images

Manual linking means that you turn linking on and off manually when two or more series are open. This is frequently used to move through a recent image and a corresponding prior, side-by-side.



Ì

Prerequisite: Add the relevant toolbar tools.

- To manually link images, on the viewer toolbar, select the manual link 🔤 button.
 - **Note:** For mammogram images, you can only link LCC and RCC with each other; similarly you can only link LMLO and RMLO with each other.

Use the context tool

You can automatically jump to an image in a linked series that most closely matches the angle (X, Y, and Z position) of the image in the current series.



Prerequisite: Add the relevant toolbar tools.

On the viewer toolbar, select the context tool I.

Reset linked series

You can reset (undo) image linking by selecting the reset linked series \square button. This may take time depending on the size of the series.

Moving and jumping to images

The following topics are contained in this section.

Pan images Move or span through images Flip or rotate images Play cine loops Use key images Open a series image in a 1×1 frame Jump to a BTO image by SR finding Triangulate images

Pan images

You can pan an image to view any portions that extend beyond the visible frame.



Prerequisite: Add the relevant toolbar tools.

Procedure

- 1. On the viewer toolbar, select the pan 🖤 tool.
- 2. Drag the image in any direction.

Move or span through images

You can move or span through images in a series. *Spanning* means to move rapidly with minimal hand or wheel motion.

To adjust the span sensitivity, see Configure the toolbar.

- To move: Hover over the image and rotate the mouse wheel.
- To span: On the image shortcut menu, select **Tools** > **Span Images**, and then drag on the image.

Flip or rotate images

You can flip images 180 degrees, or rotate them 90 degrees.



Prerequisite: <u>Add the relevant tools to the image shortcut menu</u>.

From the image shortcut menu:

- Select Transformation > Flip Horizontal or Flip Vertical.
- Select **Transformation** > **Rotate Right** or **Rotate Left**.

Play cine loops


When viewing a series, you can play cine loops and shuttle between frames.



Prerequisite: Add the relevant toolbar tools.

Procedure

- 1. On the viewer toolbar, select the cine 1 button.
- 2. Cine play starts, and the media control bar appears.

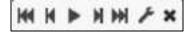

3. Use the media control bar to pause, move to the first or last frame, and move to the previous or next frame.

Use key images

You can specify key images in a series that you can jump to at any time. You can add key image thumbnails on reports.



Specify a key image

- 1. Display the image that you want to be a key image.
- 2. On the image shortcut menu, select **Set Key Image**.

Jump to a key image

1. At the top of the thumbnail pane on the left side, select **Keyimage**.



The thumbnail pane shows thumbnails of your key images.

2. Double-click a key image thumbnail to open the key image.

Open a series image in a 1×1 frame

If a series is open in a layout other than 1×1 , double-click any image to open it in a 1×1 layout.

Jump to a BTO image by SR finding

To quickly open a BTO image that has a specific SR finding:

Procedure

- 1. Open the tomosynthesis series that contains the image.
- 2. On the 2D or synthesized view, select the SR finding of interest.

Result: The corresponding slice opens. Note the following:

- The slice that opens is of the same view type (e.g. select a finding on an RCC to open a BTO RCC slice).
- If the BTO series is not displayed within the current layout, the series opens next to the 2D or synthesized view in a 1x2 layout. To restore the original layout, select the backslash (\) key.

Triangulate images

You can move to images and points in multiple series at the same time.



Prerequisite: Add the relevant toolbar tools.

Procedure

- 1. Open two or more series.
- 2. On the viewer toolbar, select the Triangulation \square button.
- 3. Do one or both of the following.
 - Double-click a series to open the corresponding slice in the other series.
 - Move the pointer in one series to move the pointer to the corresponding position in the other series.

Use MPR tools

Multiplanar reformation (MPR) creates images in orthogonal planes (axial, coronal, sagittal or MIP) relative to a base image. The MPR tool is hidden until you open a series. Refer to the following topics.

Perform simple MPR Perform standard MPR Handle MPR images Save an MPR series

Perform simple MPR

You can perform simple MPR to generate a single alternate planar view from the base view.



Prerequisites:

- If performing MPR on a fusion study, wait until both series are fully pre-fetched.
- Add the MPR tool.

Procedure

- 1. Open a series, and then double-click an image to view it full screen.
- 2. Select the arrow on the right side of the MPR MFR button.



- 3. In the button shortcut menu, select a plane.
- 4. The viewer performs MPR and displays the new plane.

Perform standard MPR

You can perform standard MPR to generate three additional planar views of the base image.

Prerequisites:

- If performing MPR on a fusion study, wait until both series are fully pre-fetched.
- Add the MPR tool.
- Open a series, and then select the MPR MER T button.

 \checkmark

Note: The plane of the base image determines the default MPR plane.

Handle MPR images

You can handle MPR images in the following ways by using controls in the base image.

- To move through MPR images, **drag the blue region bars**. You can also **drag the endpoints** to adjust the viewing angle.
- To move both region bars at the same time, on the toolbar, select the context tool **I**, and then drag on the base image.
- To change the slice thickness, **double-click the thickness value**, and then type a new value (or drag the thickness value).
- To change the slab width, select **Region**, double-click the slab width value, and then enter a new value (or drag the slab width value).
- To display an MPR image full screen with a large thumbnail of the base image, **double-click an MPR image**.

Save an MPR series

After you perform MPR, you can save the resultant images and series for later viewing.

Procedure

- 1. <u>Perform simple MPR</u> or <u>Perform standard MPR</u>.
- 2. Right-click a generated cell, point to Save MPR, and then select Instance or Series.



3. In the **Create MPR Series/Instance** dialog, type a description, and then click **save**.



Result: A thumbnail for the saved instance or series appears on the thumbnail bar.

Work with PET fusion studies

The following topics are contained in this section.


Turn PET fusion display on and off Color and blend PET fusion source studies

Turn PET fusion display on and off

If a PET study was fused with another study such as a CT scan, you can turn display of the fused study on or off.

Procedure

- 1. To turn on fusion display, hover over the top of an image to display the image's top shortcut menu, and then select **PT**.
- 2. In the PT button shortcut menu, under **PET selection**, select a source study.

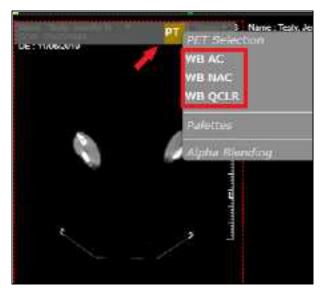

"Fused" appears in the lower-left part of the image to indicate the display state.

3. To turn off fusion display, in the PT button shortcut menu, select Remove current PET.

See also:

Color and blend PET fusion source studies

Color and blend PET fusion source studies


You can add color to source studies in a PET fusion study, and control alpha blending of the two studies. Specifying a color in one image applies the color to all images in the series.



Prerequisite: Turn PET fusion display on.

Procedure

1. In the PT button shortcut menu, under **PET selection**, select the source study to color.

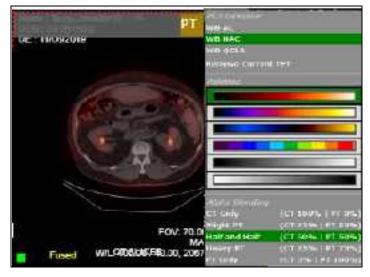

- 2. In the PT button shortcut menu, under **Palettes**, select a color.
- 3. In the PT button shortcut menu, under **Alpha Blending**, select a blending option.

Gather information from images

The following topics are contained in this section.

View pixel values in Hounsfield units Display DICOM values Export DICOM values Print images to a standard printer Open a third-party application or function Download images

View pixel values in Hounsfield units

You can view the value of a selected pixel in Hounsfield units.



Prerequisite: Add the relevant toolbar tools.

• On the viewer toolbar, select the Hounsfield units Hu tool, and then select and hold the mouse button on the pixel of interest.



Display DICOM values

You can display the DICOM tag values associated with the current image by doing one of the following.



Prerequisite: Add the relevant toolbar tools.

- On the viewer toolbar, select the DICOM values \blacksquare button.
- On an image shortcut menu, select **View** > **DICOM Values**.

DICOM Values		_		
		0		
T_0008_0000_GenericGroupLength	UL	1	4	44G
T_0008_0005_SpecificCharacterSet	CS	1	10	ISO_IR 100
T_0008_0006_tmageType	cs	3	22	ORIGINAL\PRIMARY\AXIAL
T_0008_0012_InstanceCreationDate	DA	1	8	20230302
T_0008_0013_InstanceCreationTime	TM	1	6	081901
T_0008_0016_SOPCIessUID	ut	1	25	1.2.840 10008.5 1.4 1.1.2
T_0008_0018_SOPInstanceUID	UI.	1.	36	1.3.6.1.4.1.11157.2023.3.8.0.4.32.8
T 0008 0020 ShidyDate	E1A	1	8	0020020

Export DICOM values

When you view DICOM values in the viewer you can export them to a CSV. This CSV file can **only** be displayed correctly in **Notepad**, **Notepad++**, or **Wordpad**.



Prerequisite: Add the relevant tools to the image shortcut menu.

Procedure

- 1. On an image shortcut menu, select View > DICOM Values.
- 2. In the title bar of the screen that appears, select the download \blacksquare button.

Result: The file is downloaded to your Windows Downloads folder.

Print images to a standard printer

You can print the current image, series, or study to a TWAIN (non-DICOM) printer. It is a convenient, less-expensive alternative to DICOM printing, but is not suitable for diagnostic purposes. See also <u>DICOM-print a study</u>.



Prerequisite: Add the relevant toolbar tools.

Procedure

1. On the viewer toolbar, select the print 🕒 button.



2. In the Print Image dialog, enter the following settings.

Layout – Select the number and arrangement of frames per page.

Page Size – Select the paper size and print orientation.

Compress Large Images – Select to compress large images (for lower quality but faster printing).

Selected Image – Select to print the current image.

Selected Series – Select to print all images in the current series.

Current Page – Select to print the current image in all displayed frames.

Key Images – Select to print key images from the current series.

All Images – Select to print all images in the current study.

- 3. Select Print.
- 4. In the Windows print dialog, configure options and select **Print**.

Open a third-party application or function

If you configured a third-party application or function, you can open it from the toolbar.


Prerequisites: <u>Configure opening of a third-party program</u>

• On the viewer toolbar, select the external tool button corresponding to the item to open.

Download images

You can download images and series from studies to your desktop in various formats.

• To download the selected image as a PNG (regardless of the current series layout), on the image shortcut menu, select **Download Image**.

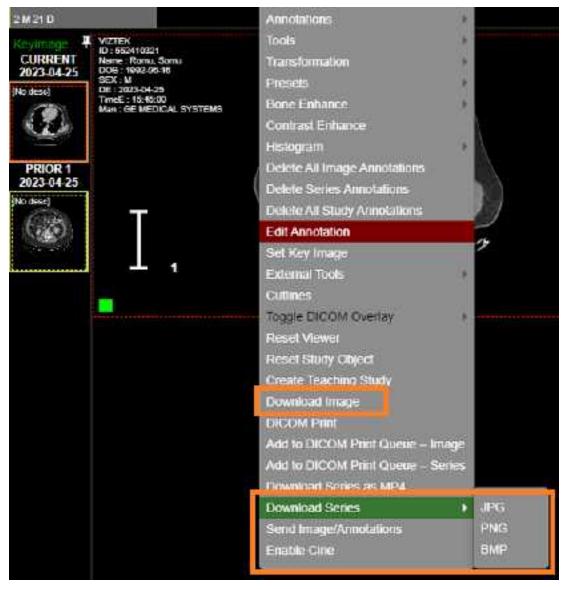

• To download each image in the current series as a separate file, on the image shortcut menu, select **Download Series**, and then select the file format.

Work with Display Management

With the viewer's display management functions you can create and manage highly customized "DMs." The term *DM* refers to *display management* in general, or to an implementation of a hanging protocol. By using DMs you can precisely customize how images "hang" in the viewer.

This chapter contains the following sections. <u>Setup and basics</u> <u>Create a DM</u> <u>Use DM groups</u> <u>More example DMs</u>

Setup and basics

This section contains the following topics.

Configure viewer settings for DMs Use the DM toolbar Display images with a DM Use DM Manager

Configure viewer settings for DMs

Before working with DMs, configure the settings described in the following sections in this manual.

<u>Configure modality-specific viewing options</u> <u>Configure mammography</u> <u>Set up connected displays</u> Next, configure viewer settings as follows.

Procedure

- 1. Go to <u>Viewer Settings</u> > **MODALITY**.
- 2. In the list of modalities, select a modality that you work with (such as CT or MG).
- 3. To turn on DMs, in the settings area under **Options**, select the following checkboxes.
 - Auto Hang DM
 - DM Allow Missing Cell
 - Auto Hang Priors
- 4. To view priors, under **Prior Options**, configure the following.
 - a. Select the Auto Open Prior checkbox.
 - b. In the **Auto Open Prior Count** dropdown list, select the maximum number of priors to open at once.

5. Select **SAVE.**

6. Repeat for each modality that you work with.

Use the DM toolbar

The DM toolbar gives you quick access to DM related functions.

Show the DM toolbar

You can manually display the DM toolbar in the viewer, or configure the viewer to display the DM toolbar automatically for selected modalities.

- Manually: In the viewer, select the display manager [™] button, and then in the button shortcut menu, select **Show DM Toolbar**.
- Automatically: On the viewer toolbar, select the settings button. Select **MODALITY**, select modalities, select the **Show DM Toolbar** checkbox, and then select **SAVE**.

Use the DM toolbar

In the example in the figure below, the toolbar contains tools on the left, and DMs (DM and mg_DM) on the right. "DIAGNOSTIC MAMMO" is the name of a user-created DM group. Use the toolbar as follows.



- **[Top button]**: If available, select a DM group to display the group's DMs on the right.
- **Save**: Saves a new DM based on the current layout.
- Save Study: Saves the current configuration for the current study only.
- **Update**: Overwrites the current DM with properties taken from currently displayed images.

Caution: Selecting Update deletes all settings of the current DM.

- Manage: Opens the DM manager.
- **Reset**: Select to apply any newly created or edited DMs.
- **Refresh**: You can also *refresh* DMs to update the list of applicable DMs.
- [DM]: Select any DM on the DM toolbar to apply it in the viewer.

Display images with a DM

You can select from a list of previously configured DMs to hang the current study. If you show the DM toolbar in the viewer, you can select a DM on the toolbar (see <u>Use the DM toolbar</u>). Otherwise, you can use the DM button on the viewer toolbar as follows:

Procedure

- 1. Open a study in the viewer.
- 2. On the toolbar, select the DM 🔤 button.
- 3. On the button shortcut menu, select one of the DMs in the list.



Result: Images hang according to the settings in the DM. When multiple images hang in a single cell, they appear in the same order as in the DM.

- 4. Optional. You can change the view as follows.
 - Rotate the wheel to page through any additional images in the study that the DM can hang.
 - Drag a thumbnail onto a cell to display it there.
 - Use the left or right bracket keys ([or]) to move to the next or previous DM step.

Use DM Manager

In DM Manager, you can create, edit, and manage all your DMs from a centralized screen.

- 1. On the viewer toolbar, select the display manager we button.
- 2. In the button shortcut menu, select DM Manager.

H Mane	per -								
une (2 March 1	Gelart		tisses	Mintell	1	rdiere	Servill	٣
EDPO	misuten	ED EXCERNIT AL	a binger	Show hact	Nostan Ap	7			
Moda	¥χ.	MC(Mammagraphy)			LEOF	- +	Nome		
			1004	AGH REFREER	ADD HEW DH	00.010			
New	e !	erites Senates	Analability	Chemer	12	iiiin			
0 ma)	DM 1	MQ	Olobal	icul				0 8 3	i iti i
а ви	1	MG	Cloba	iest 🔺	T		1	0.0.1	n w re
Herr		Drag t	o reord	ier 🕴		+		0 1 1	t∋ đ

Each DM appears in its own row (light blue, bottom half)

Note: When you open a study in the viewer, it hangs in the highest DM in the list that matches the study.

3. To search for a DM, enter one or more of the following search criteria.

You can also click column headers to sort by, for example, the DM name or its number of monitors.

Setting	Description
User Groups	Select the group to which the DM user belongs.
Users	Select users associated with the DM.
Filters	Select whether the DM is private (user-specific) or public.
Monitors	Select how many monitors are configured in the DM.
Modality	Select the modality associated with the DM.
DM Group	Select DM groups associated with the DM.
Name	Type all or part of the name of the DM.

4. Select **SEARCH**.

Result: DMs matching your search criteria appear in the list at the bottom of the dialog.

5. To open the DM for editing, double-click it, or select its edit 🖉 button.

Use DM tools

Each DM has four buttons on the right side of the list:

• 2 i u i

Add – Adds the DM to a DM group that you can select.

Edit – Opens the DM for editing (for configuration, see <u>Create a general DM</u>).

Delete – Deletes the DM from the list.

Duplicate – Duplicates the DM and prompts you to name the duplicate.

Copy – Sends a copy of the DM to a user group or user.

Create a DM

DMs are very powerful tools for customizing the way studies are displayed in the Exa PACS/RIS viewer.



Prerequisite: Configure viewer settings for DMs

This section contains the following topics (not all topics may be available depending on region and version).

<u>Create a general DM</u> <u>Use the Display Management sub-tab</u> <u>Create a mammography DM</u> <u>Add a mammo rule for breast implant displacement</u>

Create a general DM

The following is an example of how to create a basic general (non-mammo) DM.

Procedure

- 1. Open a study in the viewer that you want to hang in the DM you are creating. Set the series layout and drag images into the cells.
- 2. On the viewer toolbar, select the display manager we button, and then select **DM Manager**.
- 3. In the DM Manager dialog, select ADD NEW DM.

The local sectors of the local
Filter
WDM

4. In the second **DM Manager** dialog, enter the following settings.

HP Propertie	5	
Name *	-	D Inoctive D Private
HP Identifier	Modality	
Modalities *	67+	
Body Part		

Setting	Description
Name	Type a name for the DM.
Inactive	Optional: Select to disable the DM until ready to use.
Private	Optional: Select to make the DM available only to you.
Modalities	Select all modalities that potentially could hang in the DM.
	Note : This must be the modality of the currently opened study, or the validation percentage may not reach 100.
Body Part	Optional: Prevents all but the selected body part to display.
	To display any body part, leave blank.

- 5. In the **Preview** area, under **Monitors**, select the number of monitors you use *for viewing images*.
- 6. Under **Preview**, in the upper-left corner of the frame, select "**1*****1**" and then select a cell layout (for the study-level series/images within the monitor).
- 7. Optional. Select **1*****1** in one of the cells, and select a series layout (for the series-level images within a cell).

8. Optional. To allow missing cells, in the upper-right corner of a page or cell, select the checkbox (see <u>About missing cells</u>).

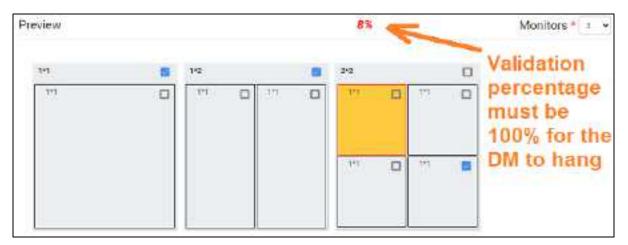


Figure. In this example, 3 pages (monitors) are configured, with 1, 2, and 4 cells added to each page, respectively. On the second page, **Allow Missing Cells** is selected at the page level, **which means that either the left, right, or both cells do not have to match**. On the third page, the study will hang if images for the lower-right cell are missing, but not if they are missing in any other cells. The upper left cell is selected for adding a matching rule group and/or matching rule. Because the validation percentage is not yet 100%, matching rules must be added.

Add general matching rules

Matching rules define which images can hang in which cells. The DM's modality, body part, and allow missing cell settings *roughly* define this, but you need matching rules to *finely* define this. The validation percentage tells you whether your rules are sufficient to hang the currently opened study--add matching rules until it reaches 100%. In addition to matching rules, you can also add prior rules (see later in this topic).

- 1. Under **Preview**, select a cell.
- 2. At the bottom of the window, on the MATCHING RULES tab, select ADD GROUP.

Note: Adding a group to contain your rules is optional, but we recommend doing so for most applications because you can combine rules with AND/OR logic (see the OR dropdown in the figure below).

Result: The new group initially appears as a row of buttons under the ADD GROUP button.

		ADD CHOUP	ADD GENERIAL POLLE	ADD MAMMO FILE	ADD FRIDE R.L.
--	--	-----------	--------------------	----------------	----------------

3. In the group, select ADD GENERAL RULE.

Ŵ

4. In the Matching Tag dialog, enter the following settings.



This rule allows CTs with chest-related series descriptions to hang in the cell.

Setting	Description
DICOM Field/Tag	Select an item to match (the argument of the rule).
Operator	Select a matching operator.
Matching Value	Type a value that must match for the rule to be satisfied, and then select the plus sign. Add as many values as could potentially match.

5. Repeat to add rules for all other cells, until the validation percentage is 100%.

• **Note:** The order of your rules matters. Images hang in the order of rules in the DM (previously, they hung in the order they appeared on thumbnails).

- 6. In **DM Manager**, select **SAVE**.
- 7. To automatically apply additional processing to cells, see <u>Use the Display Management sub-tab</u>.

Add a prior rule

Ŵ

If you want to hang priors in a cell, create a prior rule instead of a matching rule.

- 1. Select a cell, and on the Matching Rules tab, select ADD PRIOR RULE.
- 2. Select the index of the prior to hang:
 - **ANY** Open any prior that matches
 - **1** Open prior number 1
 - **2** Open prior number 2

Use the Display Management sub-tab

To automatically apply additional processing to all images that hang in a DM, you can use the Display Management tab in the DM manager.

Procedure

- 1. In **DM Manager**, open a DM, and then select a cell.
- 2. On the **DISPLAY MANAGEMENT** sub-tab, enter the following settings.

MATCHING I	ULES DISPLAY	MANAGEMENT			URDATE
Auto-Orient (Blank to	Right Edge	Bottom Edge	LUT Bone Enhance	Acto v	0.00.00.00.00
Disable) Rotation (Clockwise)	None 👻	Mirror	Window/Level Reprocess	O Default O Custom Select	
New (Zoom)		igmoid 🗇 Linear Fusion 🗐 CAD	11111111111		
Setting	Current MS t				

Setting	Description
Auto-Orient	[Unused]
Rotation	Select a degree of clockwise rotation.
Mirror	Select to switch the left and right breast view.
View	Select a zoom type.
Invert	Select one or more processing functions to apply.
Sigmoid	
Linear	
CLAHE	
Fusion	
CAD	
Current MG Only	Exempts the current cell from the Stack by View and Prior option.
CAD	Select to display CAD values according to the SR CAD settings when the user reaches this step of the hanging protocol.
LUT	Select an LUT to apply.
Bone Enhance	Select a percentage of bone enhancement.
Window/Level	Select Default or Custom. If Custom, type the values to apply.
Reprocess	Select to generate an additional view from the base view.

3. Select **UPDATE**, and then **SAVE**.

Create a mammography DM

The following is an example of how to build a DM that displays priors on top and current images (RCC, LCC, RMLO, LMLO) on the bottom. This DM shows all main views and specialty views such as spot and cleavage, synthesized views (including Hologic Intelligent Views), and implant displaced views.



Prerequisite: For the modalities in the example, configure viewer settings (see <u>Configure viewer</u> <u>settings for DMs</u>).


Procedure

- 1. In the viewer, on the toolbar, select the **DM** button, and then in the button shortcut menu, select **DM Manager**.
- 2. Select ADD New DM.
- 3. Type a name for the DM, and then enter required settings.

OPTIONAL: In the **Body Part** list, select **Breast**.

DM Managar		
HP Propertie	s	34
Name *	MG TEST	D Inactive D Private
HP Identifier	Modality	
Modalities *	MC-	
Body Part *	Great •	

4. In the **Page Count** box, type the number of **monitors**.

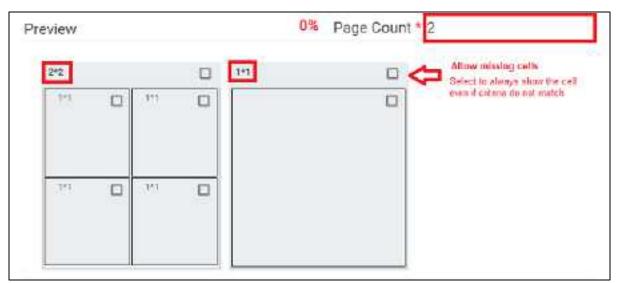

5. Select the layout numbers (such as **2*2** or **1*1**), and then select the number of cells.

- 6. Select the **Allow missing cells** check box for each page and/or cell that you want to always display, even if empty (such as for a unilateral diagnostic).
- 7. Select a cell, and then on the **MATCHING RULES** tab, select **ADD GROUP**.
- 8. In the list, select **OR**, and then select **ADD MAMMO RULE**.

-	5		1	17
	ADD GROUP	ADD GENERAL RULE	ADD MAMMO RULE	ADD PRIOR RULE
		-		

- "OR" means "this rule, OR any subsequent rules."
- **Groups** allow you to have the system look for multiple views, and use one DM for multiple scenarios.
- 9. For the upper-left cell, add the following rule criteria:

"Right breast, CC Family, Any specialty view except ID, and processed."

Enda *	O Let	
	S Regit	11
	07	
trainus =	CC Family 4	
	DONT CARE V	
Piccessing Type *	O unprocessed	
	C fonosyntees	
	C Synthesized View	
	S Processed	
	01	- 12
In a get index	Arty 🕶	

Setting	Options	Description
Side	Left	Left breast
	Right	Right breast
	?	Either
View	[Various]	Select the primary breast view.
	[Various]	Select the secondary breast view. Select DON'T CARE to allow any of the secondary views to display except ID.
Processing Type	Unprocessed	Raw images

Setting	Options	Description
	Tomosynthesis	3D images
	CView	2D images taken from a tomosynthesis study
	Processed	Standard 2D images
	?	Images without the "0008,0068" Presentation Intent Type DICOM tag.
Image Index	ANY	Images of any index can hang
	1–20	Only an image of the selected index can hang.

10. Select SAVE.

Result: Rules matching your criteria appear in the list.

- 11. Within your group, select **ADD MAMMO RULE** and add the following additional rules.
 - Right, CC Family, Don't Care, Synthesized
 - Right, CC Family, ID, Processed
 - Right, CC Family, ID, Synthesized
- 12. From outside the group, select **ADD PRIOR RULE** for this cell.

	App group	ADD GENERAL MALE	ADD MAMNO RULE	ADD THE	REAL
O GROUP ALD GROVERAL PULL		A R DELETE GROUP		0E	

Note: The button outside the group applies the rule to the entire group.

- 13. For Study Index, select one of the following options.
 - **Any**: Display all allowable MG priors (the allowed number depends on Auto Open settings for the MG modality).
 - **1**: Show the first prior
 - 2: Show the second prior
- 14. In the **Display Management** tab, customize the way the image displays with the following options (see <u>Use the Display Management sub-tab</u>).
- 15. Repeat steps above to configure each cell of your DM.
- 16. Select **SAVE** at the top.

See also:

Create a general DM Mammography views

ß

Add a mammo rule for breast implant displacement

You can create a DM for breast views that allows you to displace (effectively hide) the implant.

Procedure

- 1. Add a mammo rule to a DM cell, and in the **DON'T CARE** list, select **ID**.
- 2. When viewing implant images, rotate the wheel to switch between displaced and non-displaced views.

See also:

<u>Create a mammography DM</u> <u>Mammography views</u>

About missing cells

By default, DM pages and cells do not allow *missing cells*. This means that if no image exists in the study that can hang in one of the cells on a page, no images hang at all on the page. For example, if you configured a DM to display a current image in the left cell and a prior in the right cell, but no prior is found, neither current nor prior image hang unless you select to allow missing cells.

It can be preferable to allow missing cells. For example, if you configured your DM to display up to three priors, but only two priors are found, you still want to display the current image and the two priors.

Create a DM from an existing layout

To create a DM more quickly, you can arrange a layout in the viewer, and then save it as a DM. Available for all modalities except mammography.

Caution: The rules that are created with this method include as many parameters from the source studies as possible, and may be overly restrictive for general use, or apply unexpected image processing. Therefore to ensure proper hanging and rendering, review the resultant rules in both the MATCHING RULES and DISPLAY MANAGEMENT tabs and modify if necessary.

Procedure

- 1. Open a study in the viewer and configure the display to your liking.
- 2. On the toolbar, select the **DM** button, and then in the button shortcut menu, select **DM Save**.
- 3. In the **DM Information** dialog, type a name for the DM and select or clear the **Private** checkbox.
- 4. Select **SAVE**.

Export and import DMs

You can import and export DMs for use in other Konica Minolta systems.

Procedure

- 1. On the viewer toolbar, select the display manager we button.
- 2. In the button shortcut menu, select DM Manager.

M Manager													
User-Groups	Select			Users		vistek			*		Filte	ra	
	Show All	(v)											
EXPORT SELEC	TTED EXPO	RT ALL	IMPORT	Show	inactive	Mon	ñore	v IA					
Modality	MG(Mammogr	sphy)	*	DM-Group	SELE	CTC		*	÷	1			
	Nome				SEAS	ан 👘	REFI	IESH)	ADO	NEW	DM		
No.	edor Sammer	Availability	j Owner.			Pruvia	*						
mg_DM †	w0	Olobel	wittek						۲	0	ii	#	
	MG	Global	wintek		П	Π			æ	0		ar:	

3. Select **EXPORT SELECTED** or **EXPORT ALL**.

The DMs are added to a single JSON file and downloaded to your computer.

4. Copy the file to the destination system, display the **DM Manager** dialog, select **IMPORT**, and then open the exported JSON file.

Use DM groups

DM groups help you organize and share your DMs. This section contains the following topics (not all topics may be available depending on region and version).

Add a DM group Share DMs with a DM group Share DM groups with users About DM groups and private DMs

Add a DM group

To organize your hanging protocols, you can place them in groups.

Procedure

- 1. Go to $\underline{SETUP} > \underline{OFFICE} > \mathbf{DICOM}$.
- 3. On the **HANGING PROTOCOLS** tab, under **DM Group**, select the plus **+** button.
- 4. In the **DM Group** dialog, on the left, type descriptive information for your DM group.

DM Group			😣
Group Name	Associate group to stu	dy by:	
[APPOINTMENT TYPE	OR	STUDY DESCRIPTION
Modelity			
Body Part	_		
Description			
SAVE			

- 5. Under **Associate group to study by**, do one of the following:
 - Select **APPOINTMENT TYPE**, select one or more appointment types in the list, and then select **SAVE**.
 - Select STUDY DESCRIPTION. In the Study Description box, type a description, and then select the plus + button.
- 6. Click **SAVE**.

Share DMs with a DM group

To add (share) DMs with a DM group:

Procedure

- 1. Go to $\underline{SETUP} > \underline{OFFICE} > \mathbf{DICOM}$.
- 3. On the **HANGING PROTOCOLS** tab, find a DM to add to a group.

- 4. On the right, select the share \checkmark button.
- 5. In the **Share** dialog, select the group with which to share the DM, and then select **SHARE**.

Share DM groups with users

By default, DM groups that you create are only available to you. To share a DM groups with other users or user groups:

Procedure

- 1. Go to $\underline{\text{SETUP}} > \underline{\text{OFFICE}} > \mathbf{DICOM}$.
- 3. On the HANGING PROTOCOLS tab, under DM Group, select a DM group to share.
- 4. On the right, select the edit \checkmark button.
- 5. In the Edit DM Group dialog, select SHARE.
- 6. In the **Copy** dialog, select users or groups with which to share the DM group, and then select **COPY**.

About DM groups and private DMs

When a DM is created by a member of a DM group, the following apply.

Availability	Instance	Users	Access
Private**	Original	Creator	Read/Write
		All Others	None
	Copy Shared with users R		Read/Write
	Shared with user groups		Read-Only
	Copy of shared copy	Shared with users or user groups	Read/Write
Non-Private	Original	Creator	Read/Write
		Group members	Read-Only
	Сору	Shared with user group or DM group members	Read-Only

**Read* here means view, use to hang studies, or copy to other users.

**Imported DMs are automatically marked as Private for the importing user.

More example DMs

This section contains the following topics (not all topics may be available depending on region and version).

Example DM with a general matching rule Example DM for ultrasound

Example DM with a general matching rule

In this example we add a DM for viewing chest X-rays with an AP view on the left and a lateral view on the right. Continuing after step 3 of "DM examples:"

Add the DM

- 1. In Name box, type CHEST TEST.
- 2. the Modality list, select CR, and then select DX.
- 3. In the **Page Count** box, type or select **1**.
- 4. Under **Preview**, on the page header, select **1*1** and select a **1 x 2** cell layout.

Add a rule group and general rules for the left cell

- 1. Select the left cell, and on the MATCHING RULES tab, select ADD GROUP.
- 2. In the new group, select **ADD GENERAL RULE**.
- 3. In the Matching Tag dialog:
 - In the **DICOM Field/Tag** list, select **Modality**.
 - In the **Operator** list, select **Equal**.
 - In the Matching Value box, type CR, select the plus (+) button, and then select SAVE.



Note: In practice you may also want to add the DX modality.

- 4. Following the previous step, add another general rule:
 - DICOM Field/Tag = SeriesDescription
 - **Operator** = Contains
 - Matching Value = ap
- 5. On the right side of the matching rule group, in the list, select **AND**.

This means that the image must satisfy both rules to hang in the cell.

Add general rules for the right cell

- 1. Select the right cell, and select **ADD GENERAL RULE**.
- 2. In the **Matching Tag** dialog, add the following rule:
 - **DICOM Field/Tag** = SeriesDescription
 - **Operator** = Contains
 - Matching Value = lat
- 3. Next to **Preview**, confirm that the validation percentage is 100%.
- 4. Select **SAVE**, and then close the DM manager.
- 5. Select **F5** to update the viewer with your new DM.

Example DM for ultrasound

In this example we add a DM that hangs a 3 x 2 series layout (3 rows, 2 columns) of only ultrasound images.

- 1. For the modalities in the example, configure viewer settings (see <u>Configure viewer settings for</u> <u>DMs</u>).
- 2. Open a US study of the relevant type in the viewer.
- 3. Click the **DM** button, and on the button shortcut menu, click **DM Manager**.
- 4. Select the relevant modality and then click **ADD NEW DM**.

Add the DM

- 1. In Name box, type US TEST.
- 2. the Modality list, select US.
- 3. In the **Monitors** box, select **1**.
- 4. Under **Preview**, select the 1*1 cell (not the page header), and select a 3 x 2 series layout.

Add a general rule for all images

- 1. Select inside the page's 1*1 cell to highlight everything around the 3 x 2 series-level cells.
- 2. Add the following general rule.
 - **DICOM Field/Tag** = Modality
 - **Operator** = Equal
 - Matching Value = US

3. Confirm that the validation percentage is 100%, and then save the DM.

Work with Peer Review

With purchase of a separate license, Exa PACS/RIS provides an advanced workflow for conducting peer reviews of approved studies. This chapter contains the following sections.

Configure peer review Use peer review

Configure peer review

Before using peer review, perform the following configuration tasks.

- Configure peer review in your company's General settings.
- <u>Set up peer review user rights</u>
- Link peer review users to resources
- <u>Configure peer reviewing provider filters</u>

Set up peer review user rights

Perform the following steps to give peer review functionality to radiologists.

Procedure

- 1. Go to <u>SETUP</u> > <u>OFFICE</u> > User Management > USER ROLES.
- 2. Double-click a user role to use for peer reviewing, such as the Radiologist user role (or if useful, you can create a dedicated Peer Reviewer user role).

i E	USER MANAGEMENT-							
	UGLAD	USER ROLES	USER GROUPS	AD CROW				
	ROLE NAME \$							
0 E	EFFing							
0.1	RAD							



Note: The user role must be one that is assigned to the user group whose members you want to be peer reviewers.

3. In the **User Role Permission** area, select the applicable user rights.

PeerReview	
	Peer Review allowance
	Peer Review manual appointment
	Peer Review Operational

Setting	Description
Peer Review allowance	The right for radiologists to perform peer review.
Peer Review manual appointment	The right to manually appoint a peer reviewer to a radiologist.
Peer Review Operational	The right to view the Peer Review Operational report.

- 4. Select **SAVE & CLOSE**.
- 5. Continue to Link peer review users to resources.

Link peer review users to resources

Link all of your peer-reviewing radiologists to a Provider Radiology resource by using the following procedure.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Providers & Resources** > **RESOURCE** > **ADD**.

- Anton	winter a discrimina	energeneren	RELATIONAL RESOLUTION	INEV MOD IN
Nie .	Provider Racestogy 👻		Madeat	HOME SALED TO -
code	HCPTOPE		Facilities *	a Residence .
sare "	Wetnet	ia #statiget	Seciely	HONE BELEVITED
	0.mie			
Fella				

2. Enter the relevant settings. Most are self-explanatory, but note the following.

Setting	Description
Туре	Select Provider-Radiology.
Code	Type your internal resource code.
Facilities	Select the facilities whose radiologists perform peer review.

3. Select SAVE.

4. On the **LOGIN DETAILS** sub-tab, in the **Select User** dropdown list, select the radiologist and select **LINK USER**.

LOCATION	GOCONTACTS LOGIN CRITALS	SET DAME		SCHEDULE TIMP (ATES	BIOADCTU	e 11	MMARY, PERS REVEN
Select User	Select user	D P	Rì	New Clear 1	Here, Quart		
	Link Litten			Papaward *	(fairward)		
-				Confirm Possward *	Colori Pala	4012	
				Access Righter		-	
				H'her	D Ore Time A		
				Broup Name *	177-18-19-19-19-19-19-19-19-19-19-19-19-19-19-	2028111	
				Pault of a girth (FAD2)			Hinte AD Groups
						ENTERINE	R

Alternatively, you can create a user "on the fly" in the right side of the screen.

- 5. Select **SAVE & CLOSE**.
- 6. Optional: Continue to Configure peer reviewing provider filters.

Configure peer reviewing provider filters

Exa PACS/RIS automatically assigns random studies to each peer reviewing provider that are relevant to those providers. To specify which studies are relevant to which providers, configure filters for each peer reviewing provider as follows.

Procedure

- 1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Providers & Resources** > **RESOURCE**.
- 2. Double-click the peer-reviewing radiologist resource.
- 3. On the **PEER REVIEW** sub-tab, enter filtering criteria.

For example, if the provider can only review studies from the "Angela Facility" facility:

- a. Under Facility, select Is.
- b. In the list of facilities, select Angela Facility.

icity.	Mechility
I IS C & NX Ang/LeftS.de Angle Racity SobiaTeoffacility	
PT Code	Appointment Type
a Q a Nz	O = O = Net
lalaar a CPT caala	Debatan ATTT yya [👘 🕂
Note:	

- If no criteria are specified in this screen, the radiologist can peer review any study.
- Leave settings blank to "select all." For example, leave the Study Description setting blank if the provider can review studies with any description.
- 4. Select **SAVE**.

Use peer review

This chapter contains the following sections (not all sections may be available depending on region and version).

Assign a peer reviewer for a study Reassign peer review Conduct a peer review of a study Manage peer review disagreements Create a peer review report

Assign a peer reviewer for a study

Exa PACS/RIS automatically appoints peer reviewers and adds studies to their MY PEER REVIEW worklist filters. Additionally, users with rights can manually appoint peer reviewers for their studies with the following procedure.




Prerequisites:

- <u>Configure peer review</u>
- Obtain the Peer Review manual appointment right

Procedure

1. On the worklist, right-click a DICOM study, hover over **Exam**, and then select **Assign Study**.

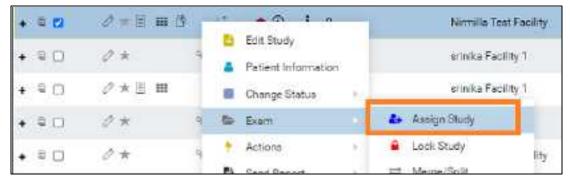

2. In the **Study Assignment** dialog, on the **Assign for peer review** dropdown list, select a peer reviewer resource.

labigit to Unio	Arrige to Sudialogist	Antiga to Par		-
Delast aver for mange	Padologus, Picharit * *	Tair, Rodok	Station and the Western	
	Stoty Daw Siath Resetution	-	Facility	Cannot Proved
	01/06/2024 AAGDINACTIL B SLITVINGS		Renola Text Facility	Radiologist. Richwid

3. Select **SAVE**.

Reassign peer review

Administrators can reassign a previously appointed peer review.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > User Management > ASSIGNED STUDIES > PEER REVIEW.

	DEER MANAGEMENT-		
		AND GROUPS ADDRESS	ASSEMED STUDIES
	MIDELS PEACING PERCENT	remember	
	ASSESSED HADDLOURD	SHOW DESCRIPTION	ACCREATE BY
0	Redelaget, Rahard	CINDSC US SU	Ray Nirnila
	Rudelogist Rohard	A MARKAN PROPERTY	Roy Harwita
	Terr, Recessingian	HISTORY REP	Any, General a
0	Radialogist, Rohard	ANORPH VISCILLETV/SUPRAGUETV BSM	Quelen Autgement

- 2. Select one or more studies, right-click, and then select **Reassign**.
- 3. In the **Appoint Peer Reviewer** dialog, appoint a new peer reviewer.

Appoint Peer Reviewer		12		(
Eesligt is uses	Annigh to Wartington	Assignito Parts R	ks level	BN/C
berr borrhermel [2 +	Haldogs, Robert 👘 🔄	(www.raikGing		
	Elvely Dato Study description	and the second second	rucitly	Caroot Provider
Radologet Sicture ((Flac)	01/16/2020 OCHE LENSTH 670D/EE		Normitie Post Facility	Radokuptet. Attravit

4. Select **SAVE**.

Conduct a peer review of a study

When peer reviewing radiologists sign in to Exa PACS/RIS, a MY PEER REVIEW worklist filter appears with all studies assigned to the user for peer review.

Procedure

1. On the worklist, select the MY PEER REVIEW tab

	Worklist	oc.	ALL STUDIES	MY PEER REVIEW		
<	MODALITY PR	STUDY DESC		ASE-TIME OF STUDY PR	GENDER PR	REASON FOR STUDY PR
•	жA	ANGREH AD	RNL BI SLETV RS&I	undefined	(#	

The symbols accompanying studies indicate the following:

- Awaiting first review
- Awaiting second review
- Awaiting third review (disagreement)
- 2. Double-click a study to peer-review.
- 3. In the **Peer Review** panel, select buttons on the toolbar to open any panels you need to conduct the review.



Buttons: Peer Review (always ON), Prior Reports, Documents

D Poer Review	
≡ 1/1 - + 13 Ø ≛ e i	T.Agite
NOR RADIOSTAPH OF THE HELL HAND	 In trade modate ross, onliney in the claimally significant.
echnique PA, célégur and mean deve af Par 1977 fund as subschool. Anies Data: Par. novemberigan	20. Understandable mans, likely to be close ally suggestioned.
vier Stadios	O Bu System Roomputery artists in the allocaty significant
Indings: There is reployed exceptoria. Custored moderate degenerative charges with bolid space servicing density and sporting present. Early excive charges PP and DP pickfaults, and first servicin-reducing all birt. Consider with recover exceptions. Destruction density across the total consideration and reducing a may reflect early charges associated with present manys. Set is seen an unernary solo. No context active vectors or of care denses absorming v.	O But the layer division particular to be a state of the
njmasion	Comments
lacormonifation:	

- 4. Optional. To add documents, in the **Documents** area, select **FULL SCAN** (display options before scanning) scan or **QUICK SCAN** (scan without options).
- 5. In the top-right corner of the **Peer Review** area, select an agreement level.
- 6. If you didn't select Agree, type your reason in the **Comments** box.
- 7. Select one of the following:

SKIP – Withhold your review temporarily and return to the study later.

SUBMIT & NEXT – Submit your review and open the next study in your list.

SUBMIT – Submit your review.

Manage peer review disagreements

The worklist provides a Peer Review Disagreements tab for administrators that lists all peer-reviewed studies for which a reviewer selected *Disagree*. On the Peer Review Disagreements tab, administrators can reassign a study to a specific user for a third opinion, or dismiss the study, such as when a third

opinion is not available. Users who are assigned for a third opinion peer review receive the study in their My Peer Review tab, and perform the new peer review in the usual manner. To manage peer review disagreements:



Prerequisite: Sign in as an administrator with the Peer Review manual appointment right.

Procedure

1. In the upper right corner of the worklist, select the disagreement button.



2. In the **PEER REVIEW DISAGREEMENTS** tab, right-click a study.

= Workli	st				-			STAT 💽
me		200	ALLERIOLS	MCPERIORDIN	100	FEVEW DISAD	NECHINIS - 🎴	
WODUITYPR		LEACH P		TROUTS PR			NATIVITIES GERALDER PER	INTERNAL FORST TOWN RELEASE
80 ()	ators	PH GDRN.	BUBLICTV HSB	Diamon Study Reasonan for Adottorial R	-	EL4Y	8	

- To get a third opinion, select **Reassign for Additional Review**, and then assign a new peer reviewer (see <u>Assign a peer reviewer for a study</u>).
- To dismiss the study, select **Dismiss Study**.

Create a peer review report


You can generate a report that shows details on studies that were peer-reviewed.



Prerequisite: Obtain the Peer Review Operational right.

Procedure

1. On the burger 🔲 menu, select **Reports**.

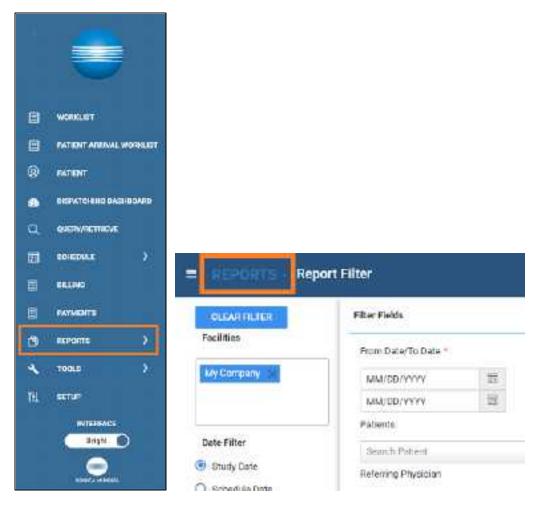

2. Next to the burger button, select the word **REPORTS**, and then select **Peer Review Operational**.

Schedule & Exports	Operationa	Patients	My R
Daily Schedules	Cancellation Reason	Worksheet	My Pe
Export Completed	Scheduler Activity		
Studies	Unfinished Studies		
Marketing Rep Activities	Referrals Variance		
Unsigned Orders	Studies Breakdown		
a configuration of a contraction	Studies by Modality		
	Studies by Modelity Room		
	Modality Breakdown		
	Technologist Productivity		
	Peer Review Operational		
	Fees by Facility and Modality		
	Fees by Recipiocist		

3. Enter filtering criteria, and then select a button corresponding to the format in which you want to generate the report.

Work with Patient Kiosk

This chapter contains the following topic and sections (not all may be available depending on region and version).

<u>What is Patient Kiosk?</u> <u>Configure Patient Kiosk</u> <u>Use Patient Kiosk as a patient</u> Use Patient Kiosk as a healthcare staff member

What is Patient Kiosk?

With a license for Patient Kiosk, Exa PACS/RIS provides functions that streamline the check-in process.

For Patients

Patient Kiosk enables patients to check in for their appointments on a touchscreen device at a healthcare provider's facility. During check-in they can view and update demographic and insurance information, and complete and sign electronic forms. Checking in through Patient Kiosk ensures that patients are correctly matched to their records.

For Staff

Staff can use functionality within Exa PACS/RIS to process appointments and information coming from patients using Patient Kiosk.

Configure Patient Kiosk

This section contains the following topics.

<u>Make study forms available on Patient Kiosk</u> <u>Configure the wait time monitor</u>

Make study forms available on Patient Kiosk

You can configure study forms to be available for viewing and signing on Patient Kiosk.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **General** > **FORM BUILDER**.

TORY GULCER D.STONTCOVE DICTORIE INVILLEMENT	TE NOT RECATION TEMPERATES	COVER 18-1411	ە • • • • • • •
	ACD BLLOAD	- DIPORT -	intern.
DOCIMENT NAME		MATHETE	
		Al	
A II contrast conset for schouter bys			1
2 El CT Spreezing Farm			
🖉 🖥 Tehocardogram Workshael			

- 2. Double-click the form you want to make available.
- 3. On the **ASSIGN** tab, select the **Viewable on Patient Portal** checkbox.

Form Name* Echocardiogram Wo	riche	Marketa	NONE SELECTED -	
PROPERTIES	MERGE FIELDS	Fecilities *	2 DELECTED -	
Document Type Select.	10 B 11	Modelities	Salar; transfallity	+
🔄 Viexable on Patient Portal			LIS	
Auto Assign to Study	12 CHV			

- 4. Move down to **Facility**, and select the checkboxes for the facilities where the form will be available.
- 5. Select SAVE.

See also:

Add a study form to a patient record

Configure the wait time monitor

The wait time monitor appears in the upper right corner of the patient arrival worklist. It is similar to the TAT and STAT monitors on the traditional worklist, but indicates how long and how many patients have been waiting since sign-in on Patient Kiosk. By default, the monitor includes three time bands (10, 20, and 30 minutes), but you can change the number of bands, their colors, times, and descriptions.

Procedure

- 1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Office** > **COMPANY** > **APP SETTINGS**.
- 2. In the left pane, select Patient Arrival Wait Time.

COLONNY INTER	in individual i	enascre anto	вајтенови на	star)satradanan marta	HALON LINE	AVABABLICED
tore - La	n v McConteny	Trains * 1	V.Destorn	V 🖸 Report of	aP 🗔 Ristow Response	nin 🗋 katakyari
1004043. 10778-0	APPORTATION	NUMBER OF T	R31307/P91421(P)	0.02210.0219944509	oregione incr	96250/80.001
mates	10/2	UPTION .			WARD	03,09
Admittacement types	2 E Ber	4			12	600660
Cattern	a long	ine :			20	errol .
PI	The second	ane accordence or a	ini.		¢.	entititi)
Patient Annual Rull Three Research annual	mentro:-					
Centures (Savity Type):	9.87 °					
	Text Colur-	Bith Y				

- 3. In the right pane:
 - To add a new band, at the bottom of the list, enter settings (such as Description and Color).
 - To edit an existing band, double-click it in the list, and change settings as needed.
- 4. Select SAVE.

Use Patient Kiosk as a patient

Main topic: Sign in as a patient.

Sign in as a patient

Patients can use Patient Kiosk to sign in at the point of care.

To start, tap a facility or type its name to search. Next:

Procedure

- 1. On the Welcome page:
 - New patients: Tap New patient, and then skip to step 3.
 - **Existing patients**: Tap Existing patient.
- 2. On the **Check-in** page:
 - If you were given a check-in code prior to arrival, tap **Check-in code**. On the **Check-in Code** page, enter your code, select the **I agree to the HIPAA privacy notice** checkbox, and then tap **Check In**.
 - If you do not have a check-in code, tap **Patient portal credentials**, enter your credentials, select the **I agree to the HIPAA privacy notice** checkbox, and then tap **Check In**.
- 3. On the **Demographics** page, enter or edit your personal information, and then tap **Next**.

	Demographics	
	-	> Next
TRIPANE -	HOSOLE HAVE	
LEST MARY +	6.4712	

4. On the **Contact information** page, enter or edit your contact information, and then tap **Next**.

	Contact information		
Provincis <	-	> Next]
e additi	4004ess silve 1 *		
ADDATE 55 LINE 2	GITE +-		1

5. On the **Responsible party** page, enter or edit information for the entity paying for the procedure, and then tap **Next**.

	oonsible party (patient gu	
Previous: K	-	> Next
Silutionies Select		
Set Set Child Spouve Others		

6. On the **Primary insurance** page, enter or edit information for your primary insurance provider, if any, and then tap **Next**.

Primary Salat a previo marsers previou	Insurance
Previous <	> Next
NUB IC RIBER PERLATION THE	
PERST NUMBER 1	Owners in strated in .
A B R THARE -	NECCIE LANSE

Note: If you do not know the policy number or other details, you can leave them blank for now.

7. On the **Secondary insurance** page, enter or edit information for the secondary insurance provider, and then do one of the following.

- If the Submit button appears, tap it to finish sign-in.
- Tap **Next**, and continue to the next step.
- 8. On the **Forms** page:
 - a. In the **Who Is Signing** list, select the signer.
 - b. In the **Name** box, type the name of the signer.
 - c. Tap a form in the list, read/edit the form, and then tap **SAVE**.
 - d. Use your finger or stylus to sign your name in the signature box, and then tap **Save Signature**.
 - e. Repeat for all other forms in the list, and then tap **Submit**.

Use Patient Kiosk as a healthcare staff member

As a healthcare staff member you can use Patient Kiosk to check in patients for appointments. To check in patients from the schedule book, see <u>Check in a patient for an appointment</u>. The following topics are available in this section.

Generate a check-in code for a patient. What is the patient arrival worklist? Check in a patient for a scheduled appointment Check in a patient if new or their chart is not found Check in a patient if demographics do not match Check in a patient for a walk-in appointment

Generate a check-in code for a patient

To facilitate patients checking in for their appointments through Patient Kiosk, you can either configure a patient portal account or provide them with a check-in code.

Procedure

1. Go to <u>Worklist</u> > <u>Edit Study screen</u> > **PATIENT INFORMATION**.

ENULIF	5 C		STUDY OT PELVIS	OF MATRIC	STUDIES - MELEDT
HOMINIPORTAL	Patient Information (ment	PORTAL ACCOUNT	SCANDONSE	NOTIF	OWNERCOOL

2. Select CHECK-IN CODE.

Result: A dialog appears with the check-in code.



3. Send the code to the patient by using email, phone, or other means.

See also:

Add or edit a patient's portal account

What is the patient arrival worklist?

The *patient arrival worklist* is a screen within Exa PACS/RIS that staff can use to manage the patient check-in process. When a patient checks in for an appointment by using Patient Kiosk, a patient row is added to the patient arrival worklist. Staff confirm or update the information in each row, and complete the check-in. They can also take receipt of electronically signed patient intake forms, and perform other registration tasks.

= Patient	Arrivals		WAIT TIME 🕥 💶 💈	0
Facility	ANGELA FACILITY -			
👗 Test, Val	erie Volley	01/31/2000 (23)	FIX_TesV1001	
10. ED. 50. E. 6. 10. E. F.	ENSITY STUDY 1+ 5/75 APPENDICULAR SKEL	Amval - 1:57 PM	Scheduled - IROD PM	
()) Test, V	/alerie	01/31/2000 (23)	New Patient	
No study di	escapition .	Arrival - 2.07 FM	No study scheduled	

The symbols to the left of the patient name indicate the registration status.

- Existing patient, no verification required.
- New patient, or matching record not found. Requires searching for a matching record and/or verification.
- Sign-in information differs from the patient chart, requires verification.

The color of the square on each row of the patient arrival worklist indicates how long the patient has been waiting to be checked in. By default, the colors indicate the following wait times. To change the colors, see <u>Configure the wait time monitor</u>.

Waiting for less than 10 minutes
Waiting for up to 10 minutes
Waiting for 11 to 20 minutes
Waiting for 21 to 30 minutes

See also:

<u>Check in a patient for a scheduled appointment</u> <u>Check in a patient if demographics do not match</u> <u>Check in a patient if their chart is not found</u> <u>Check in a patient for a walk-in appointment</u>

Check in a patient for a scheduled appointment

If the patient symbol is blue , and has a study scheduled within the next 12 hours, you can check in that patient. If a patient has multiple studies scheduled for a single visit, all of the studies are checked in at once.

Procedure

- 1. Go to <u>burger menu</u> > <u>PATIENT ARRIVAL WORKLIST</u>.
- 2. In the **Facility** button list, select your facility.

atient Arrivals			
Fecility	ANGELA FACILITY +		
👗 Test, Val	erie Volley	01/31/2000 (23)	F
DXA BONE DE	NSITY STUDY 1+ SITS APPENDICULAR SKEL	Arrival - 1:57 PM	S
	Check in All	Edit Patient Information Up	aod

3. Select the patient row to view patient information.

The information is color-coded as follows.

Green Information that the patient added or edited during check-in.

Black Original, unchanged information.

(Parens) Information prior to editing by the patient during check-in.

If any is missing or incorrect, select **Edit Patient Information** to open the patient chart and make changes.

- 4. Optional: To attach documents to the chart, select **Upload**.
- 5. Select Check In All.

The patient is checked in, and the row is removed from the patient arrival worklist.

See also:

<u>Check in a patient if demographics do not match</u> <u>Check in a patient if their chart is not found</u> <u>Check in a patient for a walk-in appointment</u>

Check in a patient if new or their chart is not found

If the patient symbol is green . the patient is either new or the information they provided could not be matched to a patient chart. For such patients you can manually search for and attach the patient chart or create a new patient chart directly from the patient arrival worklist.

Procedure

- 1. Go to <u>burger menu</u> **=** > <u>PATIENT ARRIVAL WORKLIST</u>.
- 2. In the **Facility** button list, select your facility.
- 3. Select a patient row to view details.

Facility	ANGELA FACILITY -		
A Test, Val	erie Volley INSITY STUDY 1+ SITS APPENDICULAR SKEL	01/31/2000 (23) Arrival - 1:57 PM	FIX_TesV1001 Scheduled - 3:00 PM
C Test, Val		01/31/2000 (23) Arrival - 2:07 PM	Now Patient No stiaty acheavies
Patient Infor		Bearch Create New Patter	
First Name	Valerie	Helationship	Self
East Name	Test	First Name	Valerie
Goedor	Male	Lost Nome	Test
Date of Ilicti	01/31/2000	Gender	Male
Address3	234 Street	Date of Birth	01/31/2000
Citte	Virginia Kaach	frankbo.	23.4 Street

- 4. If the patient is new, select **Create New Patient**, and then fill in the patient chart and save. If the **Check-in All** button appears, select it to complete the check-in. Otherwise, continue.
- 5. Select **Patient Search**.
- 6. In the **Patient Search** dialog, enter search criteria and then double-click a patient in the list of results.
- 7. Wait until the registration status changes, and if the **Check-in All** button appears, select it to complete the check-in. Otherwise, see <u>Check in a patient if demographics do not match</u> or <u>Check in a patient for a walk-in appointment</u>.

Check in a patient if demographics do not match

If the patient symbol is red . the information the patient entered into the kiosk differs significantly from the scheduled appointment. For such patients you can accept all the information entered into the kiosk and automatically update the patient chart and study information accordingly, or you can reject all the information and automatically replace it with the original patient and study information.

Procedure

- 1. Go to <u>burger menu</u> \blacksquare > <u>PATIENT ARRIVAL WORKLIST</u>.
- 2. In the **Facility** button list, select your facility.
- 3. On the patient arrival worklist, select the patient row to view details.

A Testy, Pe	arl NSITY STUDY 1+ SITS APPENDICULAR SKEL	01/01/2000 (23) Arrival - 9:38 AM	TesP1 Sched
		Il Changes Reject All Changes	
Patient Inform	Pearl		
Last Marrie	Testy (Test)		
Gander	Male		
Date of Birth	01/01/2000		
Address1	234 Street		
City	Oakland		
State	CA		
ZIP Code	94508		

The information is color-coded as follows.

Green	Information that the patient added or edited during check-in.
Black	Original, unchanged information.
(Parens)	Information prior to editing by the patient during check-in.

- 4. To update the patient chart and study information with the information displayed, select **Accept All Changes**.
- 5. To revert all information to the existing patient chart and study information, select **Reject All Changes**.

👗 Test, Pea DX4 SONE DE	r NGTY STUDY 14 5175 APPENDICULAR SHEL	0170172000 (23) Antval - 9:36 AM	TesP1152 Scheduled - 11 60 AM	
Patient Infor	Civetta AT	Edit Putrien folgemation	af .	
That Name	Peal			
Lair Nurre	Test			
Garminy	Male			
Date of Arth	01/01/2000			

After selecting Reject All Changes

Note: If you want to accept some but not all changes, you can select **Accept All Changes** and then open the patient chart and make modifications.

6. Select **Check-in All** to complete the check-in.

Check in a patient for a walk-in appointment

If a patient signs in to the patient kiosk who does not have an appointment, you can create a walk-in appointment for that patient. Walk-ins are considered exceptions and do not appear on the schedule book, but they do appear on the worklist so that staff can make accommodations.

Procedure

- 1. Go to <u>burger menu</u> > <u>PATIENT ARRIVAL WORKLIST</u>.
- 2. In the **Facility** button list, select your facility.
- 3. Select the patient row, and then select **Walk-in Appointment**.

E Patient Arrivals	Andele Facility -	1		WATTING
	, Valerie Volley Automator	01/91/2000 (23) 20194 - 117 PM	FIX_TeoV1001 Sci trugy schemater	
		Weise According Tall Parties of a	aton 1	
	infantination	Sweeks.		
Contribution And edited in the Local Date	eren Volles	Pedatensing Prod Roma Matthe Name	Sell Volana Volky	

4. In the **New Appointment** screen, create an appointment in the usual manner (see <u>A typical</u> <u>scheduling workflow</u>).

Work with the patient chart

Exa PACS/RIS gives you full viewing access to patient demographics, and lets you add, edit, and import a patient chart (or an electronic medical record or EMR). The Edit Study screen contains much of the same information as the patient chart.

This chapter contains the following sections. <u>Find and open a patient chart</u> <u>Create and modify a patient chart</u>

Find and open a patient chart

This section contains the following topics (not all topics may be available depending on region and version).

Find a patient chart by entering search criteria Find a patient chart by scanning a driver's license Access confidential patient records (break the glass) About VIPs

Find a patient chart by entering search criteria

You can find an existing patient chart by searching on various patient demographics.



Caution: For recommended security, please <u>turn off Chrome autofill</u>. A password and/or patient information is configured or used on this page.

Procedure

1. Go to <u>PATIENT</u> > **SEARCH**.

Note: If the tab bar is hidden, select the arrow 🙆 next to Patient to display it.

2. In the **Starts With** dropdown list, select a target for your search criteria.

This target applies to all boxes in the search bar.

- 3. Optional: To narrow the scope of your search, in the **All** dropdown list, type or select a search filter (these are typically facilities).
- 4. In the search bar, in one or more of the boxes, type one or more characters as search criteria.

Result: Patient charts matching your criteria appear in the list.

Service and Children	CHET INT OF MATION	method gravity and			CARLE COLORING	
• 4	· 15	Stow Marthee 🗆 Br	eal the Glass			
Firet	Norw	MEN	SON	- Phone	0.00	í.
L CREATENEW	SCAN LICERSE	CLUAR SEARCH	IMPORT PAT	ient ?		
REARCH RESULTS					RED	INT. MATH
x - tesE1145 9/09/1999. (09807654) 6 1. address 2 IECTADY, NY 13345 Cojae - 456	921, (129)45 6-79 99			0		
	Fret 1. CREATENEW IEARCH RESULTS : 4. IESEI145 8./08/1599, (090)765.4 1. address 2 ECTADY, NY 12345	Find Norw CREATENEW: SCAN LIDENSE IEARCH RESEARCES * -tesE1145 * -tesE1145 # 09(1)765 4321, (123)455-7599 1. address 2 ECTADY, NY 12345	First Norme MEM IL CREATENEW: SCAN LIDENSE CLEAR SEARCH IEARCH RESEARCES	Fret Norme MEN SCH IL CREATE NEW SCAN LICENSE CLEAR SEARCH IMPORT FAX IEARCH RESILETS	Fret Numme MEM SCH Phone IL CREATE NEW SCAN LICENSE CLEAR SEARCH IMPORT FATTENT ? IEARCH RESILTS .	Fret Numme MEM SCH Phone DOI IL CREATE NEW SCAN LICENSE CLEAR SEARCH IMPORT FATTENT ? IEARCH RESILTS SCAN LICENSE CLEAR SEARCH IMPORT FATTENT ? IEARCH RESILTS SCAN LICENSE CLEAR SEARCH IMPORT FATTENT ? IEARCH RESILTS SCAN LICENSE EECO IEARCH RESILTS IECO IEARCH RESILTS SCAN LICENSE IECO IECO IECO

5. Double-click a patient row, such as **Test, Table** in the figure above.

Result: The patient chart opens to the Patient Information tab.

■ Patient Test, Table	DOB:0	9/09/2000 MR	N: tes	11144			
 A (10) 	RCH PADE	NT INFORMATION	EW	IENT GUNRÀ	TON	INTIGHT ALER	ITS INSURANCE F
Patient Inform	ation	PORTAL ACCOUN	r	SCAN LICE	NSE	INQUIRY	CHECK-IN CODE
Mariost	NONE 58	LECTED +					
Facilities *	MY CO	MPANY.~					
DICOM Patient ID							
Account No./Alt *	test7144	Ø					
	MANAGE A	LIERNATE ACCOUNT	NUM /	DENTIFIERS	2		
Name *	table	MI			test		
	Surfac 🖈						
Alias	First Nadao	E LONCE	lama :				
POB *	09/09/2000			Sex *		м	*
Mantal Status	Select		~	Height (fv	in)		
Smoking Status	Salari		0	Weight (ib	1		

- 6. To view more fields, click [more].
- 7. To view more pages, next to the **Patient** page title, select **O**, and then select a tab (such as PATIENT GUARANTOR, or INSURANCE PROFILES)

See also:

<u>Create a patient chart</u> <u>Find a patient chart by scanning a driver's license</u> <u>Edit basic patient information</u> <u>Edit other patient information</u>

Find a patient chart by scanning a driver's license

You can find an existing patient chart by scanning a driver's license.

Procedure

1. Go to <u>PATIENT</u> > **SEARCH**.

Note: If the tab bar is hidden, select the arrow 🥺 next to Patient to display it.

2. Select **SCAN LICENSE**.

J



3. Use your barcode reader to scan the driver's license.

The results appear in the **PATIENT SEARCH RESULTS** list.

Access confidential patient records (break the glass)

In emergencies you can use the Break the Glass function to open patient records that are usually off limits. All Break the Glass operations are added to the audit trail report. Users of type Associated Patients Only can also "break the glass" from the New Appointment screen. To "break the glass" in the patient search screen or on the physician's portal, follow the steps below.

Prerequisites:

- Configure access reasons (see <u>App settings</u>).
- Obtain the Break the Glass user right (see <u>Create a user role</u>).

On the patient chart search screen

1. Go to <u>PATIENT</u> > **SEARCH**, and select the **Break the Glass** checkbox.

Patient				
٢.	SEARCH	PRIMI		
Starts W	th 💌 AR		Show Inactive 📴 Brea	ik the Glass
	e.	Film Manne	MEN	15N

2. Find a patient chart in the usual manner.

On Physician Portal

- 1. On the **My Exams** worklist, select the **Break the Glass** checkbox.
- 2. In the **Break the Glass Warning** dialog, use the search tools to find a patient, and then doubleclick the patient record.
- 3. In the second **Break the Glass Warning** dialog, in the **Access Reason** list, select a justification for accessing the confidential record.
- 4. In the **Verify Your Credential** boxes, type your portal sign-in credentials, and then select **ACCEPT**.

Result: Studies for the selected patient appear in the My Exams screen.

About VIPs

VIPs (very important patients) are patients who the facility wants to identify for special consideration by healthcare staff. For non-VIP patients, a gray star appears next to their names on the worklist, the patient chart, and other areas of the program. With the appropriate rights, you can select the VIP star in the patient chart to change it to a solid 📩 star and thereby designate the patient as a VIP.

Create and modify a patient chart

This section contains the following topics (not all topics may be available depending on region and version).

Create a patient chart Import a patient chart Edit basic patient information Edit other patient information About the PID, MRN, and account numbers Configure alternate account number functionality Add alternate account numbers to a patient chart Manage studies in the patient chart Assign a guarantor for a patient Send reports from the patient chart Merge patient charts


Create a patient chart

You can import a patient chart, or create one by following the steps below.

Caution: For recommended security, please <u>turn off Chrome autofill</u>. A password and/or patient information is configured or used on this page.

Procedure

1. Go to <u>PATIENT</u> > **SEARCH** > **CREATE NEW**.

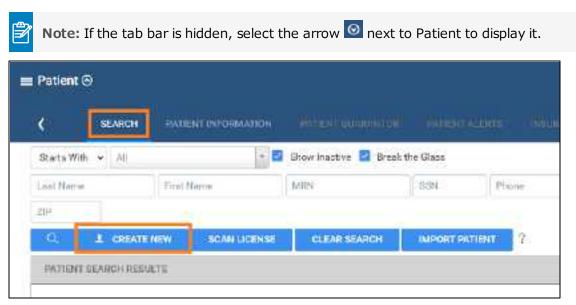

- 2. Enter information in the **PATIENT INFORMATION** tab as described <u>Edit basic patient</u> <u>information</u>.
- 3. Select **CREATE PATIENT**.
- 4. Enter information in other tabs by referring to Edit other patient information.

Import a patient chart

You can import a patient record (.CSV) into Exa PACS/RIS. If the patient already exists in Exa PACS/RIS, you can update the existing record with the imported information, or create a new record.

Note: To view which options must be present for the update to be successful, select the help button on the right side of the PATIENT IMPORT button.

Procedure

1. Go to <u>PATIENT</u> > **SEARCH** > **IMPORT PATIENT**.

Patient 🔊						
۱ ک	SEARCH	HATTER	INFORMATION :			
Starts With	▼ _AJ)	-		Show Inactive 🔽 Break	the Glass	
		First Man		MON	EEN.	Phone
part Viloria		FIRST MARK		INTER		

2. In the **Import Patient** dialog, select **Choose File**, and then browse for and select the file containing the patient record.

Import Patient	Ci Optri			×
inger Patient So	+ + 🗄 + THEPC + Docu	ижк ч 🗅	P. Send Roune	ets .
Etopse File No file closes	Organiza * New folder			
Undetse Existing	 This PC ■ 10 Objects ■ Oniting: ■ Oniting: ■ Oniting: ■ Oniting: ■ Oniting: ■ Oniting: ■ Notice ■ Years ■ Years ■ Stand Dave (Ling (P) 	Clares Discons Office Temp Discologiti Expense reports Teo FeetbeckHub Helg+ Vanual My Shapes DireVent Histobooks RiverSoft Crassed Disc meet		
	Filegang	-	ALTHOUGH	-
			Ωpen	Cancel

- 3. Optional. To update an existing patient record, select the **Update Existing** checkbox.
- 4. Select IMPORT.

Edit basic patient information

You can edit or add the basic information described below to a patient chart. You can also access these same settings from an order tab in the Edit Study screen.

Procedure

- 1. Go to <u>PATIENT</u> > **SEARCH**.
- 2. Find and open a patient chart (see Find a patient chart by entering search criteria).
- 3. On the **PATIENT INFORMATION** tab, type or select information for the following options.

Patient Inform	nation (Jess)				
Market	NONE SELECTED -				
Facilities *	му сомелли -				
DICOM Patient ID					
Account No./Alt *		1			
	MANAGE ALTERNATE ACCO	UNT NUM / IDENTIFIERS			
Name *	First Name	WE .	Last Nome	Quffu-	*
A1:68	First Nome	Lost Norte]		
008 ⁴	VYYY-MM-DD		Sex *	Select	~
Marital Status	Select	v	Height (fu'in)		
Smoking Status	Select	*	Weight (ibs.)		
Reciel/Ethnic	🔲 American Indian or Alaska I	Native	Patient Flog	Select	¥
	Asian		Language	Select	v
	Black or African American Native Hawatan or Other P		55/4		
	Lalander	acino 👻	HIE Consent	Select	~
	Select	v			
Physician	Gearah Physician	÷			
Ordering Facility	No-Ordering Facility Location S	elested • Q			

Actual settings may vary by region and version

Setting	Description
Market	Select the markets that serve the patient.
Facilities	Select facilities that serve the patient.
DICOM Patient ID	Type the PID or patient MRN.
Account No./Alt.	Type the patient's account or alternate account number. See <u>About the PID, MRN</u> , and account numbers
Name	Type the patient's first, and last name.
	Optional: Type the middle name, middle initials, and/or suffix.
★ , 📩	Grey = Non-VIP. Click to change to Yellow (VIP).
Alias	Type first and last aliases.
DOB	Select the patient's date of birth.
Marital Status	Select the patient's marital status.

Setting	Description			
Smoking Status	Select the patient's smoking status.			
Racial/Ethnic	Checkboxes: Select all that apply.			
	Dropdown list: Select the Hispanic or Latino ethnicity or non-ethnicity.			
Physician	Select the patient's primary physician.			
Ordering Facility	Available when using Mobile RAD. Select the ordering facility location or contact that serves the patient.			
Sex	Select the sex of the patient.			
Height/Weight	Type the patient's height and weight.			
Patient Flag	Select a patient flag.			
Language	Select the patient's preferred language.			
HIE Consent	Select whether the patient has consented to release of demographics and medical data through a Health Information Exchange for treatment, payment, and health care operations purposes.			
Select [more] next to words Patient Information to view the following settings				
License No./Province	Type the patient's driver's license number, and select the issuing state.			
Exp. Date	Select the expiration date of the driver's license.			
Emp. Status	Select the patient's employment status.			
Employer Name	Type the name of the patient's employer.			
Emp. Address	Type the street address of the patient's employer.			
City/Province/Postal	Type or select the city, state or province, and ZIP or postal code of the patient's employer.			
Phone/Fax	Type the phone and/or fax number of the patient's employer.			
Reason for Death	If the patient is deceased, select the checkbox and then select the reason for death in the dropdown list.			
Notes	Type any notes you wish to include in the chart.			
Users Assigned	Dynamically search for and select a user to assign to the patient, and then click the add + button. Optional: Repeat to assign more users.			
Home Address	Enter the patient's primary contact information.			
	SMS / Email Consent: Select whether the patient consents to be contacted by SMS text and/or email.			
Contact Information	Enter the patient's secondary contact information.			

4. Select **SAVE**.

See also:

Edit other patient information Create a patient chart

Edit other patient information

In addition to basic patient information, you can enter a wide range of other patient-related information in the patient chart. Most of this information is mirrored in the Edit Study screen, giving you access to it through multiple workflows. To enter or edit this other information from a patient chart:

Procedure

- 1. Go to <u>PATIENT</u> > **SEARCH**.
- 2. Find and open a patient chart (see Find a patient chart by entering search criteria).

Note: If the tab bar is hidden, select the arrow 🙆 next to Patient to display it.

3. Select a tab and then enter settings.

= Potlest G											
Test, 3105	DO	008 2000-04-30 MRRe tos31118									
108	EH		INTERT SWEWEDE	PRINCIPAL ALCOHOL:	INDROGE PROPILE	map	naporna	in avras			
	1104										

Not all tabs shown

Refer to the descriptions and topics shown for details.

Patient Chart Tab	Торіс						
PATIENT GUARANTOR	Assign a guarantor for a patient						
PATIENT ALERTS	Assign or remove patient alerts						
INSURANCE PROFILES	Add or edit an insurance profile						
STUDIES	Manage studies in the patient chart						
ORDERS	Double-click an order in the list to open it in the Edit Study screen.						
STUDY FORMS	Add a study form to a patient record						
DOCUMENTS	View or add patient documents						
PAYMENT HISTORY	View payment history and print receipts						
ALLERGIES	Add allergy information						
ACTIVITY LOG	View activity logs						
NOTES	Enter notes and reasons for study						
	Open the call log						

See also:

Edit basic patient information Create a patient chart Find a patient chart by entering search criteria

About the PID, MRN, and account numbers

The patient ID (PID) is usually the same as the MRN or account number. When they differ, it often is because the PID comes from DICOM sources and the account number comes from RIS sources. This can happen when an incoming DICOM study with an existing PID conflicts with a receiver rule, and Exa RIS compensates by creating a unique account number. Also, staff often enter account numbers manually when creating preorders for new patients, because no PID would exist yet.

Alternate account numbers

Patients receive a different MRN, PID, or account number when they undergo exams at different facilities. When Exa PACS/RIS receives such external studies, you can merge them into the master patient record by linking the external MRNs, PIDs, or account numbers as *alternate account numbers* of the Exa PACS/RIS account number. The benefits of this are:

- The viewer opens images from all records at once
- Any subsequently incoming studies having one of the account numbers are automatically linked to the other account numbers
- Helps prevent duplicate patient records
- Reduces the need to merge patient records

About issuers of PID

An *issuer* is the primary system (such as an EMR) that an institution uses to generate account numbers for their medical records. As such, it is not specific to a facility. For example, one institution could have 10 facilities, but only 3 account number systems (issuers). When importing or migrating customer medical records into Exa PACS/RIS in such cases, specifying issuers and alternate account numbers ensures that patients with duplicate primary PIDs/MRNs can be added without conflicts.

See also:

<u>Configure alternate account number functionality</u> <u>Add alternate account numbers to a patient chart</u>

Configure alternate account number functionality

To be able to use <u>alternate account numbers</u>, perform the following procedures. Alternate account numbers can be PIDs or MRNs, and in Canada they can also be ULI/PHNs or registration numbers.

Configure an issuer of a PID

The *issuer of a PID* is an alias for the facility that issued the incoming PID (or MRN, ULI/PHN, registration number, or account number). You must configure an issuer of a PID in advance for each facility whose PIDs you want to use as alternate account numbers.

Some issuers require a standardized formatting for account numbers. You can prevent users from entering invalid account numbers by typing a regular expression against which the account number must match. Exa PACS/RIS will validate all future account number entries against your regular expression.

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **DICOM** > **ISSUERS**.

= Dicoli(-)						1.1	ø
< scores	In the rest of the last of	address and the sum	Hermitele	mocus	minericality	SIGHER OF A	TRALIFICATION IN CONTRACT
						ADD.	PELDAD
Twister				mind .			
44				AL			24
<i>i</i> [↑] Ξ Rejption table				Teglezitirri	larda-		
2 2 september barber	(sever point or)			togeneries i	Interpretation		
ST E CENTRAL				ALC: N-N			

2. On the > **ISSUERS** select **ADD**.

Name *	UCVPHN.	Inactive
Туре	ULI / PHN	Ÿ
Pattern	*[0-9](9)\$	✓ Velid Pattern
Pattern Test		Einter text to text your pettern
Pettern Help *	Must be a 9-digit	number.

- 3. Type a **Name** for the issuing entity, and select a **Type**.
- 4. Optional. In the **Pattern** box, type a regular expression. Exa PACS/RIS will prevent users from adding PID/account numbers that fail to match the expression.

```
Example 1.Type the pattern: AB?
```

Means: PIDs and account numbers must start with **AB** and be followed by **one character**, such as **AB1**, **ABC**, and **ABz**.

Example 2. Type ^[0-9]{9}\$

Means: Nothing in front of the number, a digit from 0 to 9, 9 instances of such a digit, nothing after the number.

- 5. Optional: In the **Pattern Test** box, type a known alternate account number to make sure it matches your regular expression.
- 6. In the **Pattern Help** box, type the requirements for the number to help the user.
- 7. Select SAVE.

Designate AEs to use alternate account numbers

For each application entity that receives studies whose account numbers (or MRNs, PIDs) you want to use as alternate account numbers, configure the following:

- 1. Go to <u>SETUP</u> > <u>OFFICE</u> > **DICOM** > **AE TITLE.**
- 2. Double-click an AE in the list to open it.

AET	THE THEOTORE	TOUTING BALLS	ACCERTING THE	SEVER INTER MATCHING THE	a invitante referede
					A00
Hide in artists					
Hido inarmini TUTUE 5	11003	rant	ucacierman	FLE STORE	AETOPE

3. Under **Issuer Type**, select **Static** or **Dynamic**. If you select **Static**, in the **Issuer** dropdown list, select the issuer.

Issuer Type *	O None	Static	O Dynamic
Issuer *	Select 👻		

4. Select SAVE.

Configure a receiver rule for use with alternate account numbers

For each relevant receiver rule:

- 1. Go to <u>SETUP</u> > <u>OFFICE</u> > **DICOM** > **RECEIVER RULES**.
- 2. Double-click a receiver rule in the list to open it.

-	picos									
	Ą	ETHLE PAG	STORE NO	UTRIO	MIA EB	AE SCIEFE	W.	RECEIVER RU	LES	MATCHING RU
	PRIORIT	Y RULENAME	AE TITLE		FACILITY			MODALITY		INSTITUTION
			AR	*	315		×	AH.	. v	
ćΞ	4	IMPORT	OPALIMPORT	V	A8			48		
0	2	DERAULT BLLL	Ali		Al			Al		

3. Under **Rule Information and Filter**, select the **Issuer of Alternate Account Number** checkbox.



4. Select **SAVE**.

See also:

About the PID, MRN, and account numbers Add alternate account numbers to a patient chart

Add alternate account numbers to a patient chart

By adding alternate account numbers to an existing account (patient chart) you can link multiple records from the same patient together. Complete the following steps.

Prerequisite: <u>Configure alternate account number functionality</u>.

Procedure

- 1. Open the patient chart to which you want to add alternate account numbers (see <u>Find a patient</u> <u>chart by entering search criteria</u>).
- 2. On the **PATIENT INFORMATION** tab, select **MANAGE ALTERNATE ACCOUNT NUM/IDENTIFIERS**.

Test, Poo	ojaa	DOB: 05/15/2019 MRN: 456								
(s	EARCH	PATIENT INFORM	ATION	PATIENT GUARANTOR	PATIENTAL	ERTS				
Patient Infor	mation #	PORTAL A	CCOUNT	SCAN LICENSE	INQUIRY	CHE				
Market	N	ONE SELECTED +								
Facilities *	NURM	AILLIA TEST FACIL	-							
DICOM Patient ID										
	456		0							
Account No./Alt	1100		100 a 100 m 11							
Account No./Alt	1	AGE ALTERNATE A	CCOUNTIN	JM / IDENTIFIERS						

- 3. In the Alternate Account Num/Identifiers dialog, select ADD.
- 4. In the **Issuer Name and Type** dropdown list, select an option, and then type the alternate account number or identifier.

		SAVE	SAVE & CLOSE	BACK
Issuer Name and Type *				
UE//PHIN - UEL / I	PHN			
Alternate Account Num / Identifiers *	123456789	V Jesta	hatches pattern	
Muer be a	9-digit number			
Country *	Carrada 💌			
State/Province	Alberta	*		

- 5. If using multiple alternate account numbers, select the **Primary** checkbox if you want the current number to be the primary one.
- 6. Select the country and state/province where the number is valid.
- 7. Select **SAVE**.

You can repeat this procedure to add multiple alternate account numbers.

See also:

About the PID, MRN, and account numbers Configure alternate account number functionality Merge patient charts

Manage studies in the patient chart

From the STUDIES tab of the patient chart you can create, schedule, and view payment status of prior or future studies of the patient.

Procedure

- 1. Go to <u>PATIENT</u> > **SEARCH**.
- 2. Find and open a patient chart (see Find a patient chart by entering search criteria).

🖹 Note: If the tab bar is hidden, select the arrow 🥺 next to Patient to display it.

4		allyn minning			HEATSY AND	APRIL SUIP	and the second s	II In riskowy we were
atient Studies						THE GLIFTS	NCH STUDY 14	ALMIN OF CERTIFICATE
mithe	energearity 6	arear JA a	HUNDIT.		Artimutica	1919 20100100	11005.000	
0.0	RD-DORDTH MAM	sents far facily	34	ang.	1246	IS THE REPORT HAVE NO	NATED IN	Operate
er p	12/16/2011 12/00 AM	B Self Sport	1.04	· Factoria		ANGERS ADDR. DE NOT		Адріон
07 +	REPORT OF ME	termine the blocking	84.1	84	446.	at worker of the same	failleast.	-
				Main scre	en			
	(11111)	pane :		100414	10101	TTRAF.	(TERMIN)	144
Report Reservable		an a	\$4.08		tini	100 HE	AL 10	30100
IPars Reported		96.00	0508	30	000	00.00	34.34	21.09
See.		80.00	29.04		101	00.001	16.00	81.01

Payment status summary at bottom of screen

3. On the **STUDIES** tab, perform one or more of the following tasks.

Task	Торіс
Find, view, and edit a study	Search for a study in the list. Double-click a study to open it in the Edit Study screen.
View approved reports	Select the 🖻 approved report button of a study.
View payment status	An aggregate summary of study payments appears at the bottom of the screen.
Fax/Email approved reports	Right-click an approved study, point to Send Report and select Fax/Email . See also <u>Send reports from the patient chart</u> .
View the claim inquiry screen	Right-click a study that has claim history, and then select Claim Inquiry .

Task	Торіс
Schedule an ordered study	Right-click an unscheduled study, and then select Schedule Appointment to open the study in the Find Slots screen (see <u>Use the</u> <u>Find Slots feature</u>).
Create a study for the patient	Select one of the following:
	NEW STUDY : See Enter appointment information
	WALK-IN: See Schedule a walk-in appointment
Order images	1. Select a study whose images you want to order in the list, and then select ORDER IMAGE .
	2. Select the patient and/or physicians to whom to send the images.
	3. Type sending methods and notes, and then select PLACE ALL ORDERS .

Assign a guarantor for a patient

If the patient requires a guarantor for legal or insurance purposes, you can add one to the patient's chart.

Procedure

- 1. Find an open the patient's chart.
- 2. On the **PATIENT GUARANTOR** tab, select **ADD**.
- 3. Enter the information for the guarantor, and then select SAVE.

Send reports from the patient chart

You can send approved reports by email or fax directly from the patient chart.

Procedure

1. Open a patient chart, and then select the **STUDIES** tab.

Test,	Valerie	008.01/01/	2000 WRN: TeeNTU	001							-
4		-	alline mint				100300/1002	annes a	outes -	waren silika dakam	
Patient:								THE BLAT	L HOND	NEY: 1 MALNIN	CFEED \$1448E
	mithe	STREET, STREET, ST	(H) (C)		uniery.	Are	ACCOMPANY	1949200303	-	4100C8.0049	
				*	A8 - +	1					
0.0		1010/02/01110/54	several Tarriacity		54	ang.	1246				فغننون
84.	2	12/14/08/11/12/09 AM	B Destroyed	-		· Fasteral		ANGENIA ADMIN	PRINCIPAL		Appoint
62		BURNLINGTON PORT	Territor Tacilities.mg		841	847	410	it wor seen		45	Subscript of

2. In the list of studies, right-click a study containing an approved report, select **Send Report**, and then select **Fax/Email**.

Send Report					
Send Fax					Report Guese
Send To *	Paniders	¥	Providera	Select Provide	
Send Email				SEEND FAX	
Send To 🔹	Percelans	v	Purviders	Salact Provide	*
				DEND EMAIL	

- 3. In the Send Report dialog, in the Send Fax and/or Send Email sections:
 - a. In the Send To dropdown list, select a category of recipient.
 - b. In the dropdown list on the right, select an individual recipient.
- 4. Select **SEND FAX** and/or **SEND EMAIL**.

Merge patient charts

If two patient charts are accidentally created for a single patient, you can merge the charts.

- 1. Go to <u>PATIENT</u> > **SEARCH.**
- 2. In the **MRN** box, enter the account number of the records to merge.

Starte With 👻 🖉	1	• 🖾	Show Inactive 🔲 Brea	the Glass	
and Name	1 tial N	lame	test	SSN	Phone

3. In the list of results, select the row containing the *incorrect* patient chart, hover and select the settings button, and then select **Merge**.



- 4. Repeat the previous two steps for the row containing the *correct* patient chart.
- 5. In the **Patient Merge** screen, confirm the contents of the merge and then select **MERGE**.

Merge a "FIX_" study

If the DOB, gender, first name, and last name of a new patient chart are the same as an existing one, Exa PACS/RIS prepends "FIX_" to the account number in the worklist. You can fix these by merging them with the existing chart. This can occur in such cases as a name change due to marriage or divorce, an updated account number, or an error in patient entry.

1. On the search bar of the worklist, in the **ACCOUNT NO**. box, type FIX and then select **Enter**.

C	= 14	/orklist						
		PRE ORDERS	99	ALL STUDIES	989	CRM //		
×	Ó				MDS.	ACCOUNT NO.	STATUS	FACILITY
						FIX		ALL
+	20	0*	€ 2	★©	e	FIX_TecV1001	Schaduled	Angela Facili
+	= ()	2 *	19/12	*© i	0	FDK_TesV1001	Scheduled	Angelo Facili
+	20	2 *	đ	★ © i	0	FIX_TesV1001	Cancelled	GobikaTestF

- 2. In the list of results, copy the account number (without the "FIX_" prefix) of a study.
- 3. Paste the account number in the **ACCOUNT NO.** box, and then select **Enter**.
- 4. In the list of results, compare the "FIX_" record with the other to determine whether they are for the same patient.
- 5. If the same, merge the records by following the earlier procedure.

Work with mobile radiology

Exa Mobile (mobile radiology) is a set of features that provides ordering facilities and contracted mobile radiology providers tools to order, dispatch, and bill for mobile radiology services. This chapter contains the following sections.

<u>Configure mobile radiology</u> <u>Mobile radiology workflow</u> <u>Mobile billing</u>

Configure Exa Mobile

The topics in this section provide guidance on configuring Exa Mobile.

Konica Minolta staff: To enable and configure Exa Mobile for the customer, see "Configure Exa Mobile" in the configuration manual.

This section contains the following topics (not all topics may be available depending on region and version).

Configure Exa Mobile users Assign a mobile schedule template for a technologist Configure ordering facilities for Exa Mobile Add a POS map Configure vehicles Configure study statuses for mobile radiology

Configure Exa Mobile users

Configure certain users of Exa Mobile as follows.

Dispatchers and facility administrators

For dispatchers, facility administrators, or other users who may need it, grant access to the dispatching dashboard.

- 1. Go to <u>SETUP</u> > <u>OFFICE</u> > User Management > USER GROUPS.
- 2. Double-click the user group to modify (e.g. Facility Admin).
- 3. Under Navigation, select Dispatching Dashboard.

Navi	gation
	Workliet
	Patient Arrival Worklist
	Patient
	Fax Manager
	Schedule
2	Billing
	Payments
	Reports
	Toola
	Setup
	Portal Reg. Users
	Dispatching Dashboard
	Ourse (Bethleur

4. On the **USER ROLES** tab, open the user to modify, and then select the **Dispatching Dashboard** rights.



Technologists

<u>Configure technologists</u> in the normal manner, and then for each technologist:

- Assign a mobile schedule template.
- Go to TOOLS > MOBILE RAD, download mobilerad.apk, and send to the technologists. Referring to the Mobile Rad user's manual, technologists can use this file to install and use the app on their Android device.
- Send the technologists the company code, found in the SETUP > OFFICE > COMPANY > Code box.
- Send the technologists the hostname, which is the URL to Exa PACS/RIS.

Assign a mobile schedule template for a technologist

Mobile schedule templates are the "office hours" for technologists performing mobile radiology exams. You must assign a mobile schedule template for any technologist you want to dispatch for mobile radiology.

Procedure

- 1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Providers & Resources** > **RESOURCE**.
- 2. In the list of resources, double-click a technologist resource.
- 3. On the **MOBILE SCHEDULE TEMPLATES** sub-tab, select **NEW TEMPLATE**.
- 4. Enter the following settings.

Name *							D Institute		OF	fice *		ALFact	tes:		
From Data *			m	nbû-t	0		100					12245	ić.		ŵ.
Color Code		Allow Scheduling Argie Facility Acitha Test Facility											¥.		
									Fo	Doto *		ANAAN	00	111	
									De	alay Onler	÷.				
O. Weeks							O Cetr		200					Q 24	10
	3	2	3	4	5	6	35	(2)	5	4	5.5	÷	7		Buinday
Sunday							8. L		38	11	tż.	13	14		landoy
Mondey							15	16	17	10	19	20	21	01	Tuesday
Tuboday							22	28	24	25	żń	27	28	(E)	Nednesitaj
Wednesday							29	33	31						hursday
Thursday															fislay
Friday															Batuchay
Saturday															

Setting	Description
Name	A name for the schedule template.
From Date	The date from which the technologist is available.
Color Code	Click in the box, and then select a color to help identify the technologist on various worklists.
Allow Scheduling	Select to enable scheduling of the technologist for exams.
Office	Select facilities to which the technologist is associated.
To Date	Select the date up to which the technologist is available (set far in the future unless you know the exact date).
Display Order	The order in which the technologist appears in lists.
Weeks/Date/Day	Select time ranges during which the technologist is available for dispatch.

5. Under Schedule Time, configure specific hours of availability.

Sched	dule Time					
🔲 Sund	iay					
🛃 Mon	day ADD					
	From		То		Vehicle Name	
3	CPED AM	U	12:00 AM	Ø	Tielent Vehicle	
Tuos Wed Thur Fride Satu	ne sclay odlay sy					

6. Select **SAVE & CLOSE**.

Configure ordering facilities for Exa Mobile

To work with mobile radiology, <u>Configure ordering facilities</u> (or <u>Import ordering facilities</u>), and <u>Add</u> <u>locations or contacts to ordering facilities</u>.

Notes about Ordering Facilities (OFs)

- When configuring an OF in the topic above, make sure to configure any credentialed technologists or credentialed radiologists for access to restricted locations such as prisons.
- It is possible to have two OFs per study. For example, one OF can order the study, but the patient may reside at the other OF (and technologist would be dispatched to that OF to conduct the exam).

Add a POS map

With a *POS* (*place of service*) *map*, you can select default locations to which to dispatch mobile radiology orders from providing entities. For example, an ordering facility might typically dispatch its orders to the patient's residence rather than its own location, and you can preconfigure this arrangement by using a POS map.

Notes:

- When you create an ordering facility, you can select a default POS map.
- The POS map serves to provide the correct address for claims, and the correct address for dispatching the technologist.

• The POS map associates industry standard POS codes with *physical addresses* that you want to use.

Procedure

1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Billing** > **POS MAP**.

	(Betride	n i novini i ili novini pir	SADER INVERTORIES	PROVIDER LEVEL CODE	POEMO	*	
						() AD	Dis Illiono
	C00##	Parameter (Posicope	P054004853		NURSENAD	COPACONINE ADD
				AK.		ÀK;	19
0.0	09	Prison/Conscional Facility	.09	#acity		facility	
18	11	Office	- m			Ordering Past	By-Block
28	12	Home	12			Patient's Res	dence

2. Select ADD, and then enter the following settings.

INSUMMOLE)	ROVIDEN INSUE	ANDE PROVIDER PRAEK TYPES
Code *	00	
Name *	Prison/Com	estional Facily
PDG Code	09 👻	Principal Correctional Family
POS Address	Pacifity	v

Setting	Description
Code	Your internal code.
Name	A name for the place of service. These names appear in dropdown lists when you need to select a place of service.
POS Code	The formal standardized code for the POS.
POS Address	Select the address needed for the claim .
Mobile Rad Disp. Address	[See below]

3. In the **Mobile Rad Dispatching Address** dropdown list, select one of the following.

Setting	Description
Facility	Dispatch to the address of the Facility.
Ordering Facility - Study	Dispatch to the OF address selected for the study.
Ordering Facility - Patient	Dispatch to the OF address selected for the patient.
Ordering Physician	Dispatch to the address of the ordering physician.
Patient Residence	Dispatch to the address of the patient.

4. Select SAVE.

Configure vehicles

You must register any vehicles that you will use for mobile radiology. Perform this procedure for each vehicle in your fleet.

Procedure

- 1. Go to $\underline{SETUP} > \underline{OFFICE} > Mobile RAD$.
- 2. On the VEHICLE REGISTRATION tab, select ADD, and then enter the following settings.

Setting	Description				
Vehicle Name	Type any name.				
Make	Type the manufacturer of the vehicle.				
Model	Type the model of the vehicle.				
Device ID (GPS)	Type the ID number of the GPS device for the vehicle.				
Color	Type the color of the vehicle.				
VIN	Type the vehicle identification number of the vehicle.				
License Plate No.	Type the license plate number of the vehicle.				
License Plate State	Type the state of issue of the license plate.				
Facility	Select facilities for which the vehicle is used.				
Inspection Expiration Date	Select the date of next required inspection.				
Registration Expiration Date	Select the date of expiration of the vehicle registration.				
Vehicle Note	Types notes.				

- 3. Under **Equipment Type**, select the plus + button, and then in the **Equipment** dialog, enter settings for each piece of radiology equipment in the vehicle.
- 4. Under **Maintenance**, select the plus + button, and then in the **Maintenance** dialog, enter vehicle maintenance information.
- 5. Under **Location**, select the plus + button and then enter settings for each location where the vehicle is parked on particular days.

6. Select **SAVE**.

Configure study statuses for mobile radiology

Before using the dispatching dashboard, configure study statuses and add them to a study flow. Study statuses are customizable, but typically at least the following three statuses should be configured.

Status	Code
Assigned	AS
Accepted	ASAC
Rejected	RJ

- 1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Scheduling & Codes** > **STUDY STATUS**.
- 2. In the **Facility** dropdown list, select your facility.

Ű.	< 188 00073	NOC IDDOCES	INCODULE FILTER	STUDY STATUS	ात
-	STUDY STATUS	STUDY FLOW	-		
Facility	My Co	mpäñy	* and	a.	400
	ORD/Ordered	Older]			
	PR/Precheckin	(146)			
18	SCH/Scheduled	Citor			
	CON/Confirmed	304			

- 3. To add a status, select ADD and enter relevant settings (see Configure study statuses).
- 4. For each status in the list related to mobile orders, select the MobileRad Related checkbox.

STUDY	STATUS STUDY FLO	w.
General Information	on .	
Status Code *	SCH	Add to Study Shortcut Menu or
Statue	Scheduled	MobileRad Related
Description *		
Color Code	#7A0008	🖬 Order Related
Max Wait Time	min (25)	
		SAVE DANCEL

- 5. Select **SAVE**.
- 6. On the **STUDY FLOW** tab, add the statuses to a study flow.

Exa Mobile workflow

The topics in this section describe a typical workflow for entering and fulfilling orders.

<u>Create a new order for dispatch</u> <u>Assign an order</u> <u>After assignment</u> <u>Group orders by dispatch ID</u>

Create a new order for dispatch

You can create mobile orders in the usual manner from the worklist or schedule book, or you can use the procedure below to create an order from the dispatching dashboard. The Pre-Orders tab in the dispatching dashboard shows all studies in the Ordered status regardless of whether they were entered from the schedule book, worklist, or dispatching dashboard.

Procedure

- 1. Go to <u>DISPATCHING DASHBOARD</u> > **TO BE DISPATCHED**.
- 2. In the upper right corner, select a facility, and then select the **New Order** we button.



3. In the **New Appointment** screen, find a patient and schedule a study as you normally would, but note the following on the **APPT. INFO.** tab under **Schedule Information**.

Schedule Info	an analysis of	-	
Ordering Facility	No Oxforing Facility Location	Requesting Date	MMMMMMY E
Place of Service	Gelect 🛩		Hermind Me for Earlier Applement
Source	Telect 🛩	Transportation	Briect 👻
Patient Location	Gelect 🛩	injury Data	MRA/DOIVY"
Patient Room No.		Referral Number	
Patient Condition	select v	Patient's Condition	Employment
Supervising	Select supervising physician	Related To	Auto Accident
Physician	641		Dther Accident
		Accident State	Geloct 🖬
		Schedule Notes	Select Macro Molto
		Justification of	
		Mobile Service	

Setting	Description
Ordering Facility	Required : Select the ordering facility location that orders the exam . This is called the <i>study level ordering facility</i> .
	Note : All locations and contacts for each ordering facilities appear as separate options in the dropdown list. Take care to select the correct location.
Place of Service	Required : Select the ordering facility location where the exam is performed (where the patient is located).
	The options and default value in this dropdown list are defined by the default POS map, if one was specified for the selected ordering facility. See <u>Add a POS map</u> .
Technologist	If the Technologist field appears, do not select a technologist . Assign the technologist later in the dispatching dashboard.
Justification of Mobile Service	A note to the technologist.

- 4. Optional: If the patient lives at the OF (such as a nursing home), set the *patient level ordering facility*.
 - a. At the top of the screen, select **Show Details** to open the patient information.
 - b. Select the ordering facility.

c. Select Set	as Home Address	i.					
Note:	This is a case whe OF where the stuc The <i>study level or</i> The <i>patient level o</i>	dy is perforr <i>dering facilit</i>	ned. y is for the	OF orderi	ng the exa	m.	nd or
Personal Info	mation PortAliaco	XHUNT					
Facility*	GobiaTertFacility T						
DICOM Patient ID	Vetennisty 🙏						
Account No./Alt *	1875	10					
Name *	3107	NCOOUNTINUM (IDENTIFIERS					
D08/Sex *	test 2023-05-25 [7] F	* *					
Physician	Smith Pathing Physician						
Ordering Facility	No Gidering Facility Locati	100	NRC .				
SSN	Set as Home Address	2					

5. Select CREATE ORDER.

Result: The order appears on the **TO BE DISPATCHED** tab.

6. Continue to <u>Assign an order</u>.

Assign an order

All studies that do not have a technologist or vehicle assigned in an Ordered, Scheduled, Confirmed, or other related status appear in the To Be Dispatched tab.



- The dispatching dashboard lists *orders*, whereas the normal worklist lists *studies*. For example, one item on the TO BE DISPATCHED tab can contain multiple studies. Consequently, the order status can sometimes differ from that of the normal worklist.
- Just like the normal worklist, you can double-click an order to open the Edit Study screen.

To initiate these orders, assign them as follows.

Procedure

- 1. Go to <u>DISPATCHING DASHBOARD</u> > **TO BE DISPATCHED**.
- 2. At the top right of the screen, in the dropdown list, select your **dispatching facility**.



3. Right-click an order, hover over **Dispatch**, and then select **Create New Dispatch**.

8	04/18/2023 9:51 AM PDT	Edit Order		
0	D4/18/2023 9 15 AM PDT	Change/Edit Patient		
Ő.	04/16/2023 9:14 AM PDT	Change Status	\mathbf{A}	
0	04/17/2023 2:29 PM PDT	Cancel		
Ö.	04/17/2023 2:27 PM PDT	Stat	31	
0	04/17/2023 2 25 PM PDT	Priority	9	
0	04/17/2023223 CM(PD)	Flog as	0	
0	04/17/2023 2:21 PM PDT	Notes		
0	04/17/2023 2 18 PM PDT	Attached Files		
Ö.	04/17/2023 1:56 PM PDT	log		
0	04/13/2023 7 54 AM EDT	Dispatch		Create New Dispatch

4. In the **Dispatch** screen, enter the following settings.

Technologist	~						Depators	ed 🧮 Nat Dispetched
Today [21]	Anytime	8						
Dispatch To	Show Al	0	Dispat	ich Orders				
Dispatch To Gearch Dispate	Show Al	0	Dispet	Patient Name	Order No.	Modality	Facility	Study Description

Setting	Description			
[Dispatch resource	Select Technologist to assign the order to a technologist.			
dropdown list]	Select Vehicle to assign the order to a vehicle.			
Date/Time	Select a date and time to dispatch the resource.			
Dispatch To	Type to search for a resource of the type selected previously (e.g. a technologist), and then select a specific resource in the list of results.			
Dispatch Orders	Select all orders to which you want to dispatch the selected resource.			

5. At the bottom of the screen, select **DISPATCH**.

Results:

- The order appears on the **ASSIGNED** tab in Assigned status, with a dispatch number. If you dispatched multiple orders to a single technologist, all orders get the same dispatch ID (to facilitate easier billing).
- The order appears on the **NEW ORDER** tab of the technologist's **Mobile RAD** app.
- 6. Continue to <u>After assignment</u>.

After assignment

After assigning an order, the order moves through the following steps.

- It appears in the **ASSIGNED** tab of the dispatching dashboard, and in the mobile app.
- The technologist accepts the assignment on the mobile app, and then the order moves to the **DISPATCHED** tab on the dispatching dashboard in **Accepted** status.
- The technologist starts the exam, the study moves to the **COMPLETED** tab in the dispatching dashboard (the exam may not be complete, but the *dispatch* is complete). If the technologist rejected the order, it appears on the **REJECTED** tab.
- When all studies in an order are complete, the dispatch closes. Any applicable billing rules are automatically applied, which adds all applicable fees to the study (you can see these in Edit Study screen > Charges/Payments).
- On the standard worklist, the order appears with images on the **ALL STUDIES** tab in the **Tech Start**, **Incomplete**, **Unread**, or other related status.

Group orders by dispatch ID

When dispatch of one or more orders and exams is completed, the Exa PACS/RIS assigns the same dispatch ID to all of those orders and exams. By grouping orders and exams by dispatch ID, you can apply and evenly divide transportation fees, setup fees, and "after hours" pricing to them. These orders and exams remain open until they are set to Check Out/Complete, during which time they can be modified (by adding, editing, or removing exams or other services).

Procedure

- 1. If an open dispatch ID exists, right-click an order, and in the **Dispatch** submenu, select **Add to Open Dispatch**.
- 2. Select the open dispatch ID to add to the new dispatch group.

Mobile billing

If you purchased the necessary license, Exa PACS/RIS provides features that support billing for <u>Mobile</u> <u>Radiology</u> ("mobile billing").

This section contains the following topics.

About mobile billing What is a billing type? About billing rules Add a billing rule Copy a billing rule About the Census screen Create a claim in the Census screen Export the census queue

About mobile billing

Mobile billing means managing billing for Exa Mobile studies. It does not imply the use of a mobile device to perform billing tasks. Exa PACS/RIS provides separate functionality for mobile billing because the healthcare organizations involved are different, and because insurance companies in the US require claims for mobile services to be handled with different procedures and metadata.

Terminology

To explain mobile billing, we need to introduce the following terms.

Mobile radiology company

A company that dispatches technologists to a patient's location to perform an exam.

Ordering facility (herein, OF)

In mobile billing, a *facility* is typically a mobile radiology company, whereas an *ordering facility* is a facility that hires them. For example, a skilled nursing facility is an ordering facility that contracts with a mobile radiology company for mobile radiology services. Facilities use Exa PACS/RIS, whereas ordering facilities do not. The term *ordering facility* is not used outside the context of Exa PACS/RIS.

Ordering facility location or contact

A billable sub-division of an OF. For example, a skilled nursing facility may have multiple wings (locations), which may have multiple halls (locations), which may have multiple stations or fax lines (contacts).

Census

A billing type available to OFs in which the responsible payer and pay status may be unknown at initial billing time. The term *census* refers to a weekly report from OFs detailing the payment status of its residents, which is difficult to track because it can change over time.

Example of mobile billing

Consider the following example of a mobile billing workflow. In this example, the **OF** is a skilled nursing facility.

An OF needs an X-ray for one of its residents. The OF signs into **Ordering Facility Portal**, and orders the X-ray from **MRC** (the mobile radiology company with whom they have a contract). The order arrives at MRC in Exa PACS/RIS, and MRC uses **Exa Mobile** to dispatch a technologist to the skilled nursing facility to perform the X-ray. When the technologist changes the order status to Completed, an in-progress claim is created in the **Census** queue within Exa PACS/RIS. When the responsible payer and payment status of the claim is finalized, MRC completes the claim (or split claims) and sends them to the responsible payer or payers. Information and reports about the study and claims become available in the ordering facility portal to those with access.

What is a billing type?

The *billing type* is a setting for ordering facility locations that determines how their claims are handled. Since some insurance policies do not cover certain mobile billing charges, the **Split** billing type can be used to split charges between the ordering facility and the insurance carrier. Often you do not know the responsible payer for a mobile exam when you initially draft the invoice, and you can use the **Census** billing type to place such orders in a queue until the payer is determined. If both of the previous cases apply, you can use the **Global** billing type. Finally, you can use the **Facility** billing type to invoice the ordering facility for all charges.

Claims	Charges	Respons. Payer	Modifier	POS	Description (or Result When Census Updated)
1	Global fees	Primary insurance	None	Ordering facility location	Studies change to Billed status and are removed from the Census screen.
2	Technical charges	Ordering facility	TC saved in M1 or next available.	None	 Box 20 is checked "Yes," total claim amount filled into the charges box to the right of box 20 Box 32 filled in with rendering provider's name, address, and NPI number. Claims change to Billed status and are removed from the Census screen. Note:
		2 Technical	Global fees Primary insurance Technical Ordering	Global feesPrimary insuranceNonePrimary insurancePrimary insurancePrimary insurancePrimary insurancePrimary insurancePrimary insurancePrimary insurancePrimary insurancePrimary insurancePrimary insurancePrimary insurancePrimary insurancePrimary insurancePrimary insurancePrimary insurancePrimary 	Image: Second

The following provides more details about each billing type.

Туре	Claims	Charges	Respons. Payer	Modifier	POS	Description (or Result When Census Updated)
		Professional charges	Primary insurance	26 saved in M1 or next available.		 When there are no technical charges, the claim is created but has zero bill fees for those charges When there are no professional charges, the claim is created but has zero bill fees for those charges When there are only global charges, the claims is created but has zero bill fees for those charges
Facility	Any	Global fees	Ordering facility	None	Ordering facility location	Studies do not appear in the Census screen.
Census						Studies appear in the Census screen for delayed claim creation. Use this billing type for skilled nursing facilities and other locations that need to wait to determine the claim payer type.

See also:

<u>Configure mobile radiology</u> <u>Mobile billing</u>

About billing rules

The Billing Rules feature is available by special request. *Billing rules* apply to mobile billing claims for ordering facilities of type Split. Billing rules define automatic modifications to default charges and fees on ordering facility location claims. For example, if your ordering facility location frequently adds extra charges for X-rays performed after hours, you can create a billing rule that automatically adds the extra charge for you. The following table describes the types of billing rule that are available to configure. For instructions on how to configure a billing rule for an ordering facility, see <u>Add a billing rule</u>.

Billing Rule Type	Description				
On claims for the selected O	F location, adds a specified amount to fees for:				
After Hours	Selected CPTs when performed after hours.				
Exam Override	Selected procedures.				
Exam Type Override	Selected modalities.				
Setup	Setup for selected CPTs.				
Stat	Selected CPTs performed stat.				
Stat Exam Type	Selected modalities performed stat.				
Transport - Multi-patient	Transport to multiple patients with selected modalities. When serving multiple patients in a single dispatch, the transport fees are divided evenly between patients.				
Transport - Single	Transport to the patient with selected modalities.				

Note: For all billing rule types, percentage-based fee overrides are based on the billable fee schedule assigned to the ordering facility if one exists, and the default fee schedule otherwise.

See also:

Ê/

<u>Configure mobile radiology</u> <u>Mobile billing</u>

Add a billing rule

The <u>billing rules</u> feature is available by special request. You can add a billing rule for an ordering facility location by configuring it as shown below. When a mobile radiology order is complete, charges are automatically added to the order according to the billing rules you define here. To copy an existing rule to a new location, see <u>Copy a billing rule</u>.

Procedure

- Go to <u>SETUP</u> > <u>OFFICE</u> > **Providers & Resources** > **ORDERING FACILITY** > **ADD** (see also <u>Configure an ordering facility</u>).
- 2. On the **BILLING RULES** sub-tab, in the **Billing Rule** dropdown list, select a rule type.

		153
Billing Rule *	Transport • Multi patient	

Result: Other fields appear.

-						5840	CRIMIN	
nii-un.k -	Tenneur Wide putere	2	The Oode *	6.21(29697)			1	MAN PROVIDENT
Summy-Smilly.	Ald IN -		740.0416++	C Perman	9 Vale		1.00	
1.0000501*			initiality in site	Carrier Ballemakaren	Perto	wei Hier Dag		
Michilly/Orb	CI we	(*)						
web?	1 MA	R.						
	E 161							
	CL ar	(8)						

3. Enter the following settings.

Setting	Description
Ordering Facility Locations	Select the OF locations whose claims you want to modify by the rule.
Modalities	Select the modalities whose fee you want to modify.
Fee Code	Select the CPTs whose fee you want to modify.
Fee Basis	Percentage : Select to increase the fee by a percentage of the billing fee schedule assigned to the ordering facility, or the

Ŕ

Setting	Description			
	default fee schedule if no billing fee schedule was assigned.			
	Value : Select to increase the fee by a fixed amount, and then type the amount in the box.			
Billable Limit	If limits apply (for example by insurance companies), type the maximum amount for each patient for each dispatch that can be added to the fee.			

4. Select SAVE.

See also:

Configure mobile radiology Mobile billing

Copy a billing rule

To configure a new billing rule, see <u>Add a billing rule</u>. To copy an existing rule to a new OF location:

Procedure

- Go to <u>SETUP</u> > <u>OFFICE</u> > **Providers & Resources** > **ORDERING FACILITY**, and then doubleclick an ordering facility.
- 2. On the **BILLING RULES** sub-tab, find a rule to copy and select its **COPY RULE** button.

	or advantation elevation and a construction		200,0023 20100240253					
	auce	100.6410	-	MALINE .	FERRE	BLLARE USP	-	
2.6	Transfer, Mail pasket	1030-0	4.0°%#	9.00	19061		CONFERENCE.	

3. In the **Copy Billing Rule dialog**, select an ordering facility and then select **NEXT**.



4. On the second page, select an ordering facility location.

Copy Billing Rule		8
Ordering Facility	Locations	BACK SAVE
Anithe Ordering Facility	ANITHAORDERING	FAC -
	Q Scarch	0
	Select all	
	AnitheOrderingFac	
	A 🗇 Ascha	

5. Select SAVE.

See also:

About billing rules Configure mobile radiology Mobile billing

About the Census screen

The Census screen shows a list of studies at ordering facility locations of type Census, and lets you create claims for them. Such ordering facility locations process studies whose payer status is unknown or subject to change, even up to the day of the study (for example, Medicare patients at skilled nursing facilities). When the billing method for a study is finalized, you can create a claim directly in the Census screen, which then moves to the Claims screen for further processing in the usual manner.

Create a claim in the Census screen

You can create claims directly in the Census screen.

Procedure

- 1. Go to <u>BILLING</u> > **Census**.
- 2. In the **Ordering Facility** list, select a facility.

DENSUS MORTONO			Denast	udin Galaxi Persena	COR Rearry Selary	Ang T. Log OT.	
Orderey Facility	Constant and	C Hilelactin					
		五 - 道 - 五					
1		souther a	r Secolaria Sans	NATION .	nandikina sanatipa harazara punta		
O IMPEREMENTATION	ar sensiti	aCCESSION/ARTP	Heresur	2403.040099403	EUS/OOTPPCER	Jeess met	
5 46	2049 (22)	18	Ceraniti, Ceraniti	8002-07 (E-4-18 PM-807)	JUNTIONEY MADE VEN	Seleci	19
C te	De471124	38	Cenard (Leave)	2009-07-12-1-96 PM-207	STECHE HINDIAL EDIST?	Select	- 39
17.16	Tawer ton	67	ternel). General	week out of a local local the	REALING STATUTION (121	

- 3. Optional: In the **Ordering Facility Notes** box, type or edit notes, and then select **SAVE NOTES**.
- 4. Select a study in the list, assign the census type, and then select **CREATE CLAIM.**

Combine studies into a claim

If studies are of the same patient, date of service, reading provider, and facility, you can combine them into a single claim.

- 1. In the Census screen, select Combine studies for same patient.
- 2. In the list of studies, select studies to combine.
- 3. Select **CREATE CLAIM**.

Export the census queue

You can export the census queue to send your ordering facilities.

Procedure

- 1. On the burger menu, select **BILLING**.
- 2. On the toolbar, select Census.
- 3. In the **Ordering Facility** list, select a facility.
- 4. Select the export 🛃 button.

Result: The census queue appears in your Windows Downloads folder as an Excel file.

Work with query and retrieve

You can query a server and retrieve studies.



Prerequisite: <u>Configure DICOM settings</u> for the server to query and for Exa PACS/RIS on the server.

Procedure

ß

- 1. On the burger Hennu, select **QUERY/RETRIEVE**.
- 2. In the dropdown list on the left side of the QUERY button, select a server to query.

EV40	COM_V32/V33 (dev	(monthing the second second	quant	i nemieve	Query By Pat	ient ULE 🛞 🛄 Guerry My	Patienta Only 🔘
MURNER	NUTS PULLENT WANT				CODA	PATIENTID OR HU	.57K
44	* 1	Tint Maryo	iu)	Satta			
Vez 🛛	Easta	Road			1001-10-38	86555.212	94
D Yes	Talifas	Eluderfi			1970-12-04	1500078	v
Dw	valuoiettei	the second in			1015-04-10	accelerate	<i>2</i>

- 3. Type criteria in the search bar and select **QUERY**.
 - Use the Study Exists list to filter studies that are already in or not in the Exa platform database.
 - You can use the asterisk (*) and question mark (?) **wildcard characters** in the account number, accession number, referring physician, and study description fields.
- 4. Optional. To retrieve all matching studies, select **RETRIEVE**. To retrieve some matching studies, select the studies, right-click, and then select **Retrieve**.

Work with the fax manager: Updox

If you purchased Updox fax or reminder services, the fax manager function sends you to the My Updox website where you can manage sending and receiving of faxes and reminders. Updox is a powerful tool because it helps you sync faxed information with patient records in Exa PACS/RIS. This chapter gets you started using Updox by describing some of the main configuration and operational tasks you can perform. For full documentation, select the help ? button in the upper-right corner of Updox.

Configure Updox

Users with administrator rights can configure Updox, including users and groups, queues, fax numbers, and reminders.

Configure users for Updox

Configure users in Updox to allows assigning Updox email addresses (for example, for individual users or for an entire practice), add staff to queues, assigning sign-in credentials, and sending notifications to individuals. Configure user groups in Exa PACS/RIS grants access to Updox

In Updox

- 1. On the burger 🔲 menu, select **FAX MANAGER**.
- 2. In **Updox**, in the upper-left corner of the screen, select **MENU**.
- 3. On the Admin tab, select Users, and then select New.
- 4. On the User Information, Settings, and Notifications tabs, enter all relevant information.
- 5. Select Save.

In Exa PACS/RIS

- 1. SETUP > OFFICE > User Management > USER GROUPS.
- 2. Open a group to uses Updox.
- 3. Under Navigation, select Fax Manager.

Configure facilities for Updox

For each facility that uses Updox, configure the following.

1. SETUP > OFFICE > FACILITY.

2. Open a facility, and in the **Updox Account ID** box, type the ID.

- 3. Select the Send Fax via Updox checkbox.
- 4. Select **SAVE**.

Configure browsers to allow pop-ups

Configure your web browser to allow pop-ups. For example, in Chrome:

- 1. On the overflow menu, select **Settings**.
- 2. Under Privacy and security, select Site settings.
- 3. Select Pop-ups and redirects, and then select Sites can send pop-ups and redirects.

Configure queues

Queues are useful for organizing and routing your faxes and emails to the intended location and person. For example, you could create one queue for each department at a facility. Updox notifies specific users of the queue that they have a fax waiting. In that sense, a queue can act as a task list.



Prerequisite: <u>Configure users for Updox</u>.

Procedure

- 1. On the burger 🔲 menu, select **FAX MANAGER**.
- 2. In **Updox**, in the upper-left corner of the screen, select **MENU**.
- 3. On the Admin tab, select Queues, and then select New.
- 4. Type a name for the queue, select users who can access it, and then select **Save**.

Configure fax numbers

Konica Minolta provides you with a fax number that you can use for Updox, but your Konica Minolta sales engineer can also work with the phone company to add your existing fax numbers to Updox.

Procedure

- 1. On the burger 🔲 menu, select **FAX MANAGER**.
- 2. In Updox, in the upper-left corner of the screen, select MENU.
- 3. On the Admin tab, select Fax Settings.

A list appears of all fax numbers configured for Updox.

4. Select a fax number to configure, and then enter the following settings.

Setting	Description
Nickname	Type an easily recognizable nickname for the fax number.
Route To	Select the queue to which Updox routes inbound faxes.
Tag	Tags are customizable labels that you can add to incoming faxes. For example, if you assign a tag labeled Authorizations to a specific fax number, Updox automatically adds the tag to all incoming faxes on that number.

5. Select Save.

Configure reminders

A *reminder* is an email, text, or voice message that Updox sends automatically to patients to remind them of their scheduled appointments. Updox records the patient response to a reminder, and automatically cancels appointments in Exa PACS/RIS if appropriate. To edit the contents of reminders, see <u>Configure</u> <u>scripts</u>.

Procedure

- 1. On the burger 🔲 menu, select **FAX MANAGER**.
- 2. In Updox, in the upper-left corner of the screen, select MENU.
- 3. On the Admin tab, select Reminders.
- 4. On the **Rules** tab, enter the following settings.

Setting	Description
Practice Timezone	The time zone of the practice performing the exam.
1st Reminder	Select the type of reminder, and how long before the appointment to send the reminder.
2nd Reminder 3rd Reminder	Select the checkboxes to add a second and or third reminder.
Send reminders after confirmation is received	Select to continue to send reminders even if the patient confirms the appointment.
Blackout weekends	Select to not send reminders on Saturdays and Sundays during the times you specify in the start and end time boxes.
Allow patients to cancel an appointment from the reminder	When selected, the reminder includes a button, link, or other means to cancel the appointment.
Allow patients to call the office from the reminder	When selected, the reminder includes a button, link, or other means of calling the practice.
Filter out multiple reminders for patients within specified time frame	Select to only send a reminder for the first appointment during the time frame you select, and only once during the time frame you select. Cuts down on redundant reminders that can annoy patients.
Human Voice	Select the type of voice for voice reminders.

5. Select Save.

Configure scripts

You can configure scripts for reminders.

Procedure

- 1. On the burger 🔲 menu, select **FAX MANAGER**.
- 2. In Updox, in the upper-left corner of the screen, select **MENU**.
- 3. On the Admin tab, select Reminders.
- 4. On the **Scripts** tab, select **New**.
- 5. In the pop-up window, select the type of script you want to add.
- 6. Enter the script and other information. To use the script, select the **Active** checkbox.

You can include variables in your script, as listed on the right. The variable names in the Text Script dialog can be misleading, so note the following:

[location]	The facility in Exa PACS/RIS
[with]	DO NOT USE
[practice name]	The customer name in Exa PACS/RIS

7. Select **SAVE**.

Use Updox

View a fax or email

You can view the contents of your faxes, and any available fax-related metadata and historical activity.

Procedure

- 1. On the burger 🔲 menu, select **FAX MANAGER**.
- 2. In Updox, on the left pane, select Inbox.
- 3. In the list, select the arrow > button corresponding to the fax or email that you want to view.
- 4. Optional. If the item is a failed outbound fax, you can select RESEND to retry the fax.
- 5. Double-click the item's thumbnail to open and view it.

View an appointment reminder

Appointment reminders are preconfigured emails, text messages, or voice messages that Updox sends automatically to patients regarding their scheduled exams. If the patient cancels an appointment through an appointment reminder, the appointment is automatically cancelled in Exa PACS/RIS. To set up an appointment reminder, see <u>Configure reminders</u>. To view the contents, history, status, and patient response to an appointment reminder, do the following.

Procedure

- 1. On the burger 🔲 menu, select **FAX MANAGER**.
- 2. In **Updox**, in the upper-left corner, select **MENU**.
- 3. On the **Home** tab, select **Reminders Dashboard**.
- 4. Optional. In the left pane, enter criteria to filter the reminders list.
- 5. In the main list, select an appointment to view details about the reminder.

Process an order

Often referring physicians fax or email exam order documents, and you can attach them to the Exa PACS/RIS patient chart.

Procedure

- 1. On the burger 🔲 menu, select **FAX MANAGER**.
- 2. In **Updox**, on the left pane, select **Inbox**.
- 3. In the list, select the arrow (>) button corresponding to the order.

- 4. In the left pane, select **SEND ITEM**.
- 5. In the dialog, type the patient name to search for the patient, select a patient in the list of results, select the **Send to EHR** checkbox, and then select **OK**.



Note: If the patient does not exist, you can leave Updox open, create the patient in Exa PACS/RIS, and then return to Updox and search for the newly created patient.

6. In the form that appears, fill in options as needed (such as the subject line), and then select the checkbox for one or more relevant studies.



Note: The document is attached at the order level to the selected studies. If you do not select a study, the document is attached to the patient.

- 7. In the **Category** list, select the Exa PACS/RIS document type.
- 8. Select SEND.

Result: The fax appears in the DOCUMENTS tab of the patient chart in Exa PACS/RIS.

Exchange secure text messages

With Updox, you can exchange secure text messages directly with a patient, and the thread is stored for future reference. Like faxes and emails, if the patient texts you a document, you can add it to the patient chart.

Procedure

- 1. In the **Inbox**, select **Text Message**.
- 2. In the Send a Text Message dialog, select Secure or SMS.
- 3. In the **Send To** list, search for and select a recipient (a patient who has a mobile phone number recorded in their chart).
- 4. Type a message, or select a template or macro.
- 5. Select the Allow Responses and/or Attach a File options as needed.
- 6. Optional. If the patient attached a document that you want to attach to the patient chart, select the checkbox for the document, select **SEND ITEM**, and then follow the same procedure for attaching a faxed document.
- 7. Select Send Secure Text or Send SMS Text.

Work with scheduling

Exa PACS/RIS offers powerful features for scheduling and managing appointments. This chapter contains the following topic and sections.

About the schedule book A typical scheduling workflow Other scheduling tasks Post-scheduling tasks

About the schedule book

The schedule book is the primary starting point for most scheduling tasks. It displays a chronological list (rows) of appointments during a selected 24-hour period in each modality room (columns) at a selected

facility. You can open the schedule book from the burger 🔲 menu by selecting **SCHEDULE**.

= Sch	edule Book	Butto	rbar			
RADIOLOG	NICKITY (MY COMPANY +	SIGNALITY	-000	10023030	NALKEN APPOINTMENT	talies allocit
	Mignetic Resonance (VC) > C	Compt	(er Tomo MC)		21 2022 - LISET NO-SHOWS (LA) C Comparies Redisgraphy (MC) (a la della servici servici sa sta della servici se a la servici servici servici se a servici se a servici se a
0b45am		Navio	union has			1
0700 ám	TEST 9109 07:00 DM/SE/2000/23/A2MLM 07:30 EARDIAC MRI MORPHOLOGY E		12-17-11 BYM00			_
07.15 am	FUNCTION 18/0 CONTRAST	Appoint	mont			
0730am						
87.45						

See also:

Work with scheduling

A typical scheduling workflow

This section includes topics in the order that you might typically follow to schedule an exam.

Find an available timeslot Set up a patient Enter appointment information Enter schedule information Enter insurance information Enter other information and forms Create an order (appointment)

Confirm an appointment

See also:

About the schedule book Other scheduling tasks Post-scheduling tasks Configure a schedule filter Use the Find Slots feature About procedure codes, appointment types, and appointment type procedures

Find an available timeslot

You can begin scheduling by finding an available timeslot in the *schedule book*, which is a calendar-style view of appointments by time and modality.

Procedure

- 1. Go to burger \blacksquare > <u>SCHEDULE</u> to open the schedule book.
- 2. On the button bar, in the **FACILITY** button list, select the facility for the exam.
- 3. Optional. If you configured a schedule filter, you can select it from the **FILTER** button menu.

RADIOLOGY	Y TACKITY (NY CONTAIN)		LITY:		197	m •		-11	10 9.	100		100	100.00	0.0120		MONT	DUCK BLOCK
	AnghuthRule				Ap	- 200	29		-				y 20			CELLER COLLER	The Contraction
7.00 are	Angela Pacility		54	Mo	Ти	We	th	Fr	50	SI	Mo	Tu	Wa	Th	Fr	Sa	Prev. 4 Months
115em	Angio Facility								1			2	3	4	5	6	NINC Yeal
7:0 +*	Deepoin Foolity		2	3	4	3	6	7	8	17	8	9	10	11	12	15	Pres, Year
728 81	GobikaTest#acility		9	10	11	12	13	14	15	14	75	10	17	18	19	20	Current Month
743aa	Moore Foc Fity My Company		16	17	18	19	20	21	22	:21	22	23	24	25	26	27	
860 am	10-17-7-17-18a	•	23	24	25	30	27	28	20	28	29	30	71				05

- 4. On the navigation bar, use the date picker 🔤 to select a date.
- 5. Double-click an empty timeslot in the column corresponding to the exam's modality.



Next: <u>Set up a patient</u> Parent: <u>A typical scheduling workflow</u>

Set up a patient

To set up a patient for an appointment, you can **find** an existing patient chart, **create** a new patient chart, or **import** an existing patient chart be following one of the procedures below.

Find an existing patient chart

1. On the **New Appointment** screen, on the **SEARCH** tab, find patients by entering demographics in the Last Name and other boxes.

Optional: If the **Break the Glass** checkbox appears, you can select it to search for confidential patient records. See <u>Access confidential patient records (break the glass)</u>.

SEARCH	APPELINED.	INSURANCE	EXAM PHER #	OFFICIENCY	анша рио	
Patient S		Enter sea	rch criteria			
Starts With	All .		🕐 🗹 Show inact	vo 🔲 Break the Glass	195	
toot		First Name	MRN.	ESN	P	
Q		SCHEDULE	GLEAR SEARCH	IMPORT PATIENT	2	
the state of the s		75				
Test, T	SEARCH RESUL able - test1144 xx/09/2000					

- 2. In the list of search results, double-click the patient for whom you want to schedule an exam.
- 3. If the **Recent Schedule** dialog appears, select **NEW SCHEDULE**.

facility Mediday		Africality General			Test.	Tost. Marilyn		
40		201. I I I I I I I I I I I I I I I I I I I		1	2		01/00	/2000_0
Sinh	edulosi: à	disensel : 0	14	Converted : 11		No-libbes : 1	Walk-Inc 10	
	Fuilby	Schedde/Infered Date				Room	Stude Benchatze	Rookstreet Deter
2	Rivela Test F	04/02/0322.000/00 FM	Res.Net	ndia 23.		žių.	SD FRIDRIGHT OT MR US OT H REQ FOOTPOX	Scieklast.
史	htella Test f	04/22/2023 TT 66:00 AM	Los Nor	nita UD		Utie Sound	DOP VELOGINEETRY PTLUMINUICAL ART	Direction.
	Normala Tast P	DAUT113323 TO 16:03 AM	Key, Nor	nia UE		ditta taund	SDRMORING CTIMELED IT HIS RED PORTPOS	treat set.
th.	Revela Tarte.	64/20/2329 2 E 6:00 FM	day Nov	ella CR		692	RONE TO ESTIMATE	Senaking

4. Continue to Enter appointment information.

Create a new patient chart

If you couldn't find a patient chart, you can follow these steps to create a new one, or skip to "Import a patient chart" below.

1. On the New Appointment screen, on the SEARCH tab, select CREATE & SCHEDULE.



2. Under **Personal Information**, type the primary demographics (name, Patient ID, etc.) for the patient.

Ibbs Details			
Personal Infor	mation		
* mine*	Му Сотралу •		
DICOM Patient ID			
Account No./Alt -		N	
Name *	Test	0.46	
	Schresse	1.fb *	
008/Ses *	MALADYYY THE Sole	≠ fe	
Primary Physician	Start: Piecey Physical	+	
Ordering Facility	No Crównig Pacifiy) Joint	is Solution	2 S
454			

3. Continue to Enter appointment information.

Import a patient chart

1. On the New Appointment screen, on the SEARCH tab, select IMPORT PATIENT.

Laist Nier	ive .	First Name	MRN	35N	
(Q)	1 CREAT	E& SCHEDULE	CLEAR SEARCH	IMPORT PATIENT	7

- 2. Select Choose File, and then browse for and select a patient file.
- 3. In the **Open** dialog, browse for and select the patient file, and then select **Open**.
- 4. Optional: To merge the patient file that you are importing with an existing patient chart in Exa PACS/RIS, select the **Update Existing** checkbox.
- 5. Select **IMPORT**.
- 6. Continue to Enter appointment information.

Parent: A typical scheduling workflow

Enter appointment information

Enter general information about the appointment and exam, and add preconfigured studies to perform.

1. In the **New Appointment** screen, on the **APPT. INFO** tab, enter or modify the following settings.

	INFO INDURANCE E	KAM POET NETTUGRONE	OTHER PHYSICIAN	PROVIDENCE
Appointment I	nformation			
Unearce *	My Company +	Reading Physician	BeautoReal Physicist .	
Modelity *	en 4	Bedy Part	Abdomen 👻	
Appointment Type *	CT ADD & PELVIS W/CONTRAST	Gentation	Belect +	1
Available Date	04/28/2023 9:00 AM EDT	Ascession No.		
Repolate	Computer Taria	Study Notes	Select Macro Note	
Referring Physician	Provider, Referiting #	2	Testing - notes seanarios	
Ordering Physician	Frender, Fafering			
Atturney	Search Actioney			
Stat Level	Stat Off 🗸 🗸	Reason	Pain	
Department				
Insetation				
Code	74177			
Unite	1.0			

Fields may vary depending on version, region, and other factors.

Setting	Description			
Location	The facility where the exam will be performed.			
Modality	The modality for the exam.			
Appointment Type	Type in the box to search dynamically for an appointment type, and then select one in the list. You can search by description or code number. See also <u>About procedure codes</u> , <u>appointment types</u> , <u>and appointment type</u> <u>procedures</u> .			
Available Date	The timeslot you initially selected on the schedule book. To change the timeslot, select the calendar button.			
Resource	The modality room you selected. Not available for editing in this screen.			
Ordering Facility	Available with Exa Mobile. Type in the box to search dynamically for the ordering facility, and then select one in the list. For more advanced searching, select the magnifying glass tool.			

Setting	Description			
Referring Physician	Type in the box to search dynamically for the referring physician, and then select one in the list.			
Ordering Physician	Type in the box to search dynamically for the ordering physician, and then select one in the list. For more advanced searching, select the magnifying glass tool.			
Attorney	Select an attorney in the dropdown list, if applicable.			
Stat Level	Select an urgency for the exam.			
Code, Units, and Modifiers	When you select an appointment type, the corresponding code appears in the Code box, and any available default units and modifiers for the code appear. Edit the units and modifiers if needed.			
[Settings in the right column of the page]	The Reading Physician and other settings in this column are optional. Exa PACS/RIS will assign the accession number and other fields automatically at another time during the scheduling workflow.			
Select study detail options	(Available when studies are added in the Find Slots screen.)			
	Select details to add to the appointment information that are not included by default. Select APPLY TO ALL STUDIES to add the selected information to all studies that you schedule.			

2. Optional. Under **Diagnostic Codes**, type search criteria, select the search Q button, and then select diagnostic codes in the list.

Diagnostic Coc	les
456	
CDOG	DESCRIPTION
🗆 332.456A	Nondisplaced item verse fracture of unspecified acetabulum. Inhielence
0 832.464B	Nonetaplaced transverse fracture of unspecified acetabulum, initial enco
O	Alexandra and the second state of the second state of the second state of the second state of the second state

Next: Enter schedule information Previous: Set up a patient Parent: A typical scheduling workflow

Enter schedule information

After entering appointment information, you can enter optional details about the appointment and the patient.

Procedure

1. On the **APPT. INFO.** tab, move beyond Diagnostic Codes, and then under **Schedule Information**, enter the following information.

Ordering Facility	My Ordering Facility (My Ordering Facility) *	Requesting Date	WWWWWW 101	
	9		🔲 Semind Me for Er	siller Appointment.
Place of Senilos	Salact 💌	Transportation	Select	¥
Source	Geloct	lin(wy Date	YYYYYMM 0 100	
Petion Location	Selict 👻	Referral Number		
Patient Roone No.		≓atiant's Condition	Employment	
Patient Condition	Select v	Related To	Auto Accident	
Supervising	Belent movestning physiciles (*		D Other Acalident	
Physician		Accident State	Solum 'M	
Technologist	Select facility and modality	Schedule Notes	Salect Macro Note	
10			Enter Schodula N	otes Hera
		Justification of		
		Mobile Service		

Setting	Description			
Ordering Facility	Select the ordering facility, if applicable.			
Place of Service	Select a place of service category.			
Source	Select the event that initiated the exam.			
Patient Location	Select the intended destination (such as department) of the patient at the facility.			
Patient Room No.	Type the room number for the patient's treatment.			
Patient Condition	ondition Select the current mobility status of the patient.			
Supervising Physician	rvising Physician Type or select the supervising physician for the procedure.			
Technologist/Vehicle	Select the technologist or vehicle linked to the exam.			
Unavailable if both Mobile Billing and Exa Mobile are in use.				
Requesting Date	Select the requested date for the exam.			
Remind Me for Earlier AppointmentSelect if the patient would like to be reminded of an appointment tak place early in the morning.				
Transportation	Select the patient's mode of transportation to the facility for the exam.			
Injury Date	Select the date of onset or injury.			
Referral Number	Type the referral number, if available.			

Setting	Description	
Patient's Condition Related To	Select whether the patient's condition is related to their employment, auto accidents, or other accidents.	
Accident State	If you selected a condition previously, select the state or province where the condition occurred.	
Schedule Notes	Select a macro note, or type a note in the box.	
Justification of Mobile Service	If dispatching mobile services, type the justification.	

Next: Enter insurance information Previous: Enter appointment information Parent: <u>A typical scheduling workflow</u>

Enter insurance information

After entering appointment information, you can enter information about the insurance provider handling payment for the exam.

Procedure

1. In the **New Appointment** screen, select the **INSURANCE** tab, and then enter the following information about the patient's primary insurance policy, if available.

SEMICH APPI		EEAM	THE RETRICTIONS	OTHER	N-11220.44
Primary Insura	INCO RESET				
Existing Insurance	Beleat		Policy Number *		
Carrier *	Scenit Gamer		Ritosp No.	C	
Provider Type			Valid From Date	MM/00/791	题
Oroup Name			Valid To Date	MM/BD/092	詞
Ralatorishgi *	Select 👻		Address Line 1		
008 *	MM/00/01 III		Address Line 2		
544	Beleat 💌				
Country	United States 👻				
City/State/ZIP		Select *	7.9 bode 2.9	Phis	
Subscriber Name *	First None .	84	E Lawrine	ee.	Sills.

Setting	Description
Existing Insurance	If the patient record already includes insurance providers, they appear in the dropdown list. Select one, if available. Information for the provider you select appears in the other options.
Carrier	If no existing insurance providers are available, select a carrier here. Information for the carrier you select appears in other options, but you must enter the policy number manually.
Provider Type	The provider type of the selected carrier is entered automatically.
Group Name	Type the provider group name.
Relationship	Select the relationship of the subscriber to the patient. If available, the date of birth and address of the person you select appear in other options.
DOB	Select the subscriber's date of birth.
Sex	Select the subscriber's sex.
Country/Address	Type or select the country, city, state, and ZIP code of the subscriber.
Subscriber Name	Type the name of the subscriber.
Service Type*	Available after selecting a carrier. Select the service types available through the insurer.
Benefit On Date*	Available after selecting a carrier. Select the date that benefits start.

Setting	Description
Eligibility/Estimation	Available with Eligibility rights after selecting a carrier. Select to verify insurance eligibility and/or view estimates. See topics on Exa Clear and PokitDok for procedures.
Policy Number	Type the subscriber's insurance policy number.
Group No.	Optional. Type the subscriber's insurance group number.
Valid From/To Date	Select the dates during which the policy is valid.
[Address]	Type the street address of the subscriber.

*Available when *not* using Exa Clear.

2. Optional. Select **ADD SECONDARY INSURANCE**, and enter information for the secondary insurance provider.

Next: Enter other information and forms Previous: Enter schedule information Parent: A typical scheduling workflow

See also:

Verify eligibility and estimate coverage with Exa Clear Verify eligibility of preorders and studies with PokitDok

Enter other information and forms

You can add other physicians to the appointment, view exam prep instructions, view provider alerts, add patient alerts, and attach study forms such as for consent, HIPAA, and CT screening. You can have the patient verify the accuracy of the information in the form at the time of the exam.

Procedure

1. In the **New Appointment** screen, select a tab and perform the corresponding task.

ment Test	Encan Licenser 🔤 💽 Contacts on De			
T NTL NUBRE	PASTRICTORS DESCRIPTION PRODUCTATION PRODUCTATION PRODUCTATION			
nstructions				
Tab on the New Appointment Screen	Task			
EXAM PREP. INSTRUCTIONS	View prep instructions for the selected appointment type.			
OTHER PHYSICIAN	In the Other Physician dropdown list, select a physician, and then select the plus button to add the physician to the list. You can add multiple physicians. Note : When the exam report is completed, if Reports to me is configured (see "Create, view, and deliver reports"), Exa PACS/RIS automatically sends a copy of the report to the physicians selected here.			

Tab on the New Appointment Screen	Task
PROVIDER ALERTS	View alerts for the provider performing the exam.
PATIENT ALERTS	Select one or more checkboxes relevant to the patient undergoing the exam. Note : To add a new alert, at the bottom of the list of alerts, type a new alert and select the plus to button.
STUDY FORMS	Select a form in the dropdown list. To fill out the form, select its edit button and select SAVE . To make the form available on the patient portal, select the Patient Portal checkbox.

Next: <u>Create the order (appointment)</u> Previous: <u>Enter insurance information</u> Parent: <u>A typical scheduling workflow</u>

Create the order (appointment)

After entering all information related to the appointment, finalize by creating the order. When you create an order, the appointment is added to the schedule book (except for walk-in appointments) and becomes available in RIS and the worklist.

Procedure

1. On the **New Appointment** screen, select the **APPT. INFO** tab, and then select **ADD STUDY**.

Result: A study ("appointment card") appears based on the information entered.

Marilyn 7		10040	er 1600 (1706)		
Livenci =	Me Company	16	Suday Perisin	Verand June Hegeneric	THE STREET
Anida (Ap =	12		Bady Park	Sect 1	·
Republication Type.	Select equationer have	10.00	Distanting -	THE A	14,03/1023 0004 AM - 21/15 AM (2)/Eastern CT
Rightable Dallin	007311223-040-664 EF		Antoneovitte	Appink menteum	ATTN CTARS SPECIE AND CONTRET I - DEDVICEME
associate .	Devices Service		they ford as	Taine I Manager Factor (18)	A .1 D11

2. At the upper-right corner of the screen, select **CREATE ORDER**.

Result: The order is created and scheduled, and one of the following screens appears.

- The schedule book, showing the scheduled appointment (by default)
- The Edit Study screen (when <u>Auto-Open New Order</u> is selected)
- The confirmation screen (when <u>Auto-Open Appointment Confirmation</u> is selected)

RADICLO	TACILITY MY CONTRACT	Y- MV	RACIES IN CONTRACT	nume-
	Computer Tame (M6)	¢	×	ay (MC)
08.00 am	A scheduled ex	am app	eara	
18:15 am	as a "study blo	ck"		
98:3U am				
18:45 am				
19:00 am	TEST MARLYN 01/01/2008/2018/3000 F	09:00 09:15		
19:15 am				
09:30 am				

3. Optional. To view a quick summary of the exam in the schedule book, hover over the study block.


Next: <u>Confirm an appointment</u> Previous: <u>Enter other information and forms</u> Parent: <u>A typical scheduling workflow</u>

Confirm an appointment

After creating an order, use the Appointment Confirmation screen to confirm details with the patient, and then change the study status to Confirmed.

Open the Appointment Confirmation screen

- Automatically: See the "Auto-Open Appointment Confirmation" setting in <u>Configure other user</u> settings.
- Manually: On the **schedule book**, right-click an appointment block, and then select **Appointment Confirmation**.

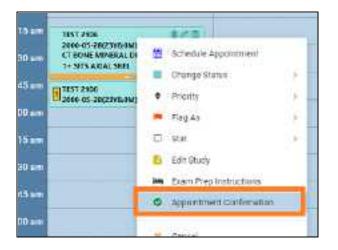

1001, 21 05 - 2000-05-28 - 10021 158, P		Patient Antrai Cate	VIING 2023-06-23 12:00 AM 807 🛞
Sales Monader	Column Planna	Steel Deares	a station mental and
Pukan Haras Tant 1906	how .	10120-01-01-01	CAN-157 All St. Enders 27 Citing MultiParty
Adverse	Avi-no.		NERINAVES : DOBLET P 10 107 1+ STUARA SHE
Harrist Theorem	Norm		4100 A184 A135
Ind Phone	Kull Press	6 39	610
Stread Addiment	failes.	Traine Village	
	Period Robiensii		ICAN - FORM PLEASER OF CTHENNETHING
	Other Physics and	de 14	00
The second secon	centael televisilentaria logicia sicel vallogi	. (1)	
til sava nucleorie		Sent Innois In	and

Confirm with the patient

You can discuss the information over the phone, or send the information:

- In the **Send Emails To** box, type an email address and select the email 🖾 button to send the confirmation.
- Select the print 🖲 button to print the information.

Optional: For more information you can:

- Verify eligibility and estimate coverage by selecting Eligibility/Estimation or Verify (if available). See <u>Verify eligibility and estimate coverage with Exa Clear</u>.
- Under **Exam Details**, select an appointment card to jump to the Edit Study screen.

Change the appointment status to Confirmed

When the patient confirms the appointment, change the status. On the **schedule book**, right-click the appointment card, select **Change Status**, and then **Confirmed**.

ə am Dum	TEST 2106 2009 65 28(22) CT BONE WINE 1+ SITS AND.	RAL DE	F SSITY 31 U 0 3:20 08:50			
5an	TEST 2106	m	Schedule Appointment		-	
13 ain	-	- 38	Charge Status			Continned -
5 am			Peartty		0	Confirmed All
met		-	Flag Ad	- 6	33	Checkin
	-	-			100	Charles and

Previous: <u>Create the order (appointment)</u>

Parent: <u>A typical scheduling workflow</u>

See also:

Other scheduling tasks Post-scheduling tasks

Other scheduling tasks

This section contains the following scheduling tasks that were not covered in <u>A typical scheduling</u> workflow.

Use the Find Slots feature Create a quick block Print the schedule book Schedule appointments back to back Schedule a walk-in appointment Change the time block duration on the schedule book Edit or delete schedule blocks About auto-blocks Track referrals

Use the Find Slots feature

Find slots is a powerful tool for finding appointment times and creating orders. It can be especially useful when you want to find a timeslot first, and then schedule an appointment in it.

Procedure

- 1. Go to SCHEDULE.
- 2. In the schedule book, on the menu bar, select **FIND SLOTS**.

FACILITY : MY COMPANY - MYFACILITY FILTER-	FOID-SLOTS	WALK-IN APPOINTMENT	OBICKBLOCK
Mammography (MC) Co	<u> </u>	FRIDAIC MAXY IN, SALE-BUILT	

- 3. In the **Available Slots** screen, in the **Patient ZIP Code** box, type the patient's ZIP or postal code.
- 4. In the **Distance from Patient** dropdown list, select a search radius.

Result: Facilities in the range you specify are automatically selected in the Facility list.

Available Slots	R.		
Distance from Patient	Patient ZIP Code	Patient ZIP Plus	Patient Time
25 👻 miles	27610	ZIP Plus	US/Eastern
Appointment Type	Foo	irty	
Select Appointmen	nt Type 💌	5 SELECTE	D v

5. In the **Appointment Type** dropdown list, select an appointment type.

Result: Available timeslots appear in a list.

6. To narrow the list of available timeslots, type or select other criteria described below.

Available Slots				
Desire contract Pases A vice as Separation Type Sear Type Sear Type Sear Type Sear Type Sear Type Sear Sear Sear Sear Sear Sear Sear Sear Sear Sear Sear Sear Sear			ensierin [11]	Accordments SEC 11 THE AN TOTAL AMUSE ANY INTERNATION SEC 11 THE ANALY TOTAL ANY
4:10 AM - 1:00 AM 1111	2003-0119	Ny Celeare (secrecestate)		Line and Lin
4c20.458 - \$:45.464.00.0	000445.1%	Mc Georger (determined)		Tente ancie deta
4585 AM - 7 00 AM 111	20119-0	My Company (Association)		
1-25.488-140.4M		Rep Conceptory: Charmonic		
1:40 AM - 7:56 AM [11]	anticia.	My Consul y (Annexandra)		

Setting	Description
Appointment Type	Selected above. The appointment to schedule.
Referring Physician	Select to display any provider alerts, and to carry over into the new appointment screen after slots are found.
Carrier	Select the insurance carrier. The insurance may also be auto-populated if the patient has a default insurance policy.
Facility	Select to schedule in specific facilities.
	To select all facilities, make no selection or select Select all.
Modality Room	Select if you want to schedule in specific modality rooms.
	To view slots in all modality rooms, make no selection or select Select all.
Date	Select to schedule on a specific date.
Start/End Time	Select a time range during which to schedule.
AM/PM	Select to schedule only in the morning or afternoon.
Day of the Week	Select one or more days on which you want to schedule.
Additional Time	Select if more time is needed for the exam than the standard duration.

🗊 Notes:

- Timeslots overlapping soft quick blocks are accompanied by a pink square.
- Timeslots overlapping hard quick blocks are omitted from the list.
- 7. In the list of timeslots, select a slot.

Result: An appointment card appears (on the right side of the screen, under Appointments).

8. Optional. To schedule another appointment in the same order, select another appointment type and then repeat the steps.

9. Select CREATE ORDER.

10. Perform scheduling procedures.

Create a quick block

You can create a *quick block* to specify timeslots for non-exam events such as lunch breaks, or for dedicated use such as all-day training.


- Soft quick blocks are partial room reservations, and you can schedule exams over them.
- *Hard quick blocks* are full room reservations, and you cannot schedule exams over them.

Procedure

- 1. Go to <u>SCHEDULE</u>.
- 2. On the button bar, select **QUICK BLOCK**, and then drag within a column to place a block.



3. In the **Create Quick Block** dialog, enter the following settings.

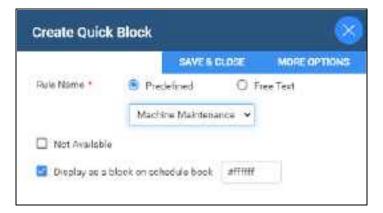

Setting	Description
Rule Name	Predefined : Select to apply a schedule rule to the block, and then select a schedule rule from the list.
	Free text : Select to apply a label to the block without applying a schedule rule.
Not Available	Select: Create a hard quick block.
	Clear: Create a soft quick block.
Display as a block on schedule book	Select to add a visible quick block. If cleared, the quick block is not visible, but still affects available slots during scheduling.
[#fffff]	Click inside the box, and then select a color for the block.

4. Optional. For detailed settings, select **MORE OPTIONS** to open the schedule rules screen.

See also:

Configure a schedule template and schedule rules Edit or delete schedule blocks About schedule blocks

Print the schedule book

Procedure


- 1. Go to <u>SCHEDULE</u>.
- 2. Under the button bar, on the navigation bar, select the print 🕒 button.

OTS WALK IN A	POINTMENT	QUICK BLOCK	_	
FEIDAY MAY 11 MRM odRoom (GT		COLOCION EMAY 19 2023 - XAModroom (GTF)	A	C TODAY PREV NEXT
205 SEC1205 -04-30(23Y&oM) f	12:00		f	
MS SECTOR	12-15		-	

Schedule appointments back to back

You can select to schedule multiple appointments back to back, which means that Exa PACS/RIS will display timeslots that are available for all of the appointments on the same day at the same facility.

- 1. Following the steps in <u>Use the Find Slots feature</u>, add the first appointment type.
- 2. Select the second **Appointment Type**, and then in the **Group Schedule** dialog, select **BACK-TO-BACK**.

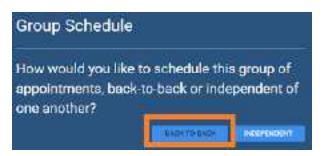

3. Optional: In the **Max Delay** dropdown list, select the maximum amount of wait time between appointments.

Distance from Patient	Patient ZP Gods	Palani ZP flas	Pedent Time Zoos	Carrier	Азразилин Туре	Appointmonte
10 • māos	28	289 Paix /		Salant surriur. *	CT ABO & PERME W/ DD	- Several
Fecility.	No	delity floor	Doix		Start Time	74177
NUME SELECT	11211	HOME STATES	MD -	臣	o	<i>3</i> 5
End Time E SEARCH SLOTTS	000	9000	0.5	Sand Time MacRel	*	Back-to-back 74776

Note: The **Same Day** option returns all available timeslots regardless of delay.

- 4. Enter additional scheduling criteria, select **SEARCH SLOTS**, and then select a slot.
- 5. Select CREATE ORDER.
- 6. Perform scheduling tasks, such as in <u>A typical scheduling workflow</u>.

Change slots part way through

If you need to change the timeslots you selected for your appointments before creating the order:

1. Select the first appointment card, and then select the 💿 button.

8023-64-19	8:00 AM - 8:30 AM US/Eastern	CI	CT Room 1, Gam r
14176	CT Abdo	men and i	Pelvis W/WD Contes 1
B; 1.0	III 1		O 20

2. Select the next appointment card, and select a new timeslot.

3. Repeat for all appointment cards.

Schedule a walk-in appointment

To immediately admit patients who do not have a previously scheduled exam, you can schedule a walk-in appointment. Walk-ins are considered exceptions and do not appear on the schedule book, but they do appear on the worklist so that staff can make accommodations.

Procedure

- 1. Go to <u>SCHEDULE</u>.
- 2. In the schedule book, on the menu bar, select WALK-IN APPOINTMENT.

	MANAGERET.	-	MUSIOIS	PALICIE APPOINTMENT	ains state	
					COLUMN DI	AN 191022- SET NO SHOWS
Manusography (MC) C						


3. Perform scheduling tasks, starting with <u>Set up a patient</u>.

Note that Exa PACS/RIS fills in the **Available Date** box with the current date and time.

Change the time block duration on the schedule book

A *time block* is the amount of time represented by each block on the schedule book. By default time blocks are 15 minutes, but you can change them.

- 1. Go to <u>SCHEDULE</u>.
- 2. On the schedule book, right-click anywhere in the left margin where the times of day appear.

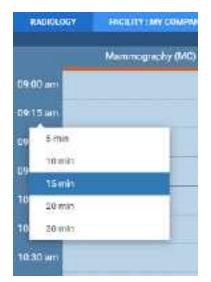

3. Select a duration.

Edit or delete schedule blocks


You can edit or delete schedule blocks directly on the schedule book.



Prerequisite: Sign in as a user with the Schedule Block right.

Procedure

- 1. Go to <u>SCHEDULE</u>.
- 2. On the schedule book, select a schedule block, and then select its edit \swarrow button.

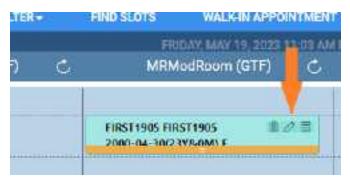

Result: Depending on the type of block, one or more editing and/or deleting option appears.

3. Select an option, and if editing, make changes in the editing screen.

See also:

Edit Study screen

About auto-blocks

When you schedule an exam in a timeslot to which a schedule rule applies, and the schedule rule specifies a particular resource to perform the exam (for example, a technologist), Exa PACS/RIS does the following.

- Checks to see if the resource is also available to work in other modality rooms at the same time.
- If so, and if Autoblocking is selected in the schedule rule, Exa PACS/RIS automatically creates schedule blocks in the other timeslots to prevent future users from double-booking the resource.

These *auto-blocks* appear on the schedule book as "Auto-Block with Study ID," and the Allow Scheduling option is disabled. You can edit auto-blocks like any other schedule block.

Track referrals

This is a legacy Meaningful Use feature and is no longer available.

Post-scheduling tasks

The following tasks can be performed (typically by front desk staff) after appointments are scheduled.

Schedule book

Check in a patient for an appointment from the schedule book Attach a document from the schedule book Make quick changes to appointments Edit an appointment from a schedule block Reschedule an appointment Schedule a follow-up Cancel an appointment Set no-shows


Patient Kiosk Use Patient Kiosk to check in patients

Edit study Enter a new charge Enter payments and print receipts

Check in a patient for an appointment from the schedule book

You can follow the steps below to check in a patient for an appointment from the schedule book. If using Patient Kiosk, see topics in <u>Use Patient Kiosk as a healthcare staff member</u>.

- 1. Go to <u>SCHEDULE</u>.
- 2. On the schedule book, on the schedule block shortcut menu, select Change Status > Check-In.

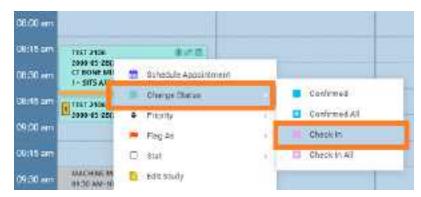


Attach a document from the schedule book

To attach a document from the schedule book, see <u>Attach (upload, scan) non-DICOM documents</u>.

Make quick changes to appointments

You can make several quick changes to the status, priority, flagging, and other aspects of appointments without having to open and edit them.

- 1. Go to <u>SCHEDULE</u>.
- 2. Right-click a schedule block and select from the options in the shortcut menu.

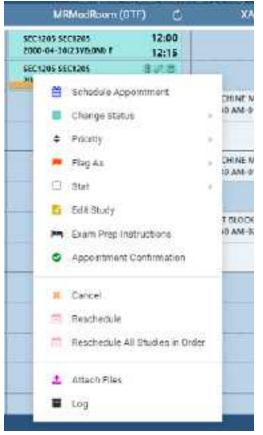

Setting	Description
Schedule Appointment	Select to change multiple properties of the appointment in the Find Slots screen. See <u>Use the Find Slots feature</u> .
Change Status	Hover over Change Status, and then select a new status. Modifies workflow elsewhere in the program.
Priority	Hover over Priority, and then select a new priority. Modifies workflow elsewhere in the program.
Flag As	Select to add flags in the submenu.
Stat	Select to assign a new urgency level.
Edit Study	Select to open the appointment in the Edit Study screen.
Exam Prep Instructions	Select to view exam prep instructions. If no instructions were entered, an " x " appears on the right side of the command in the shortcut menu.

Setting	Description
Appointment Confirmation	Select to display the appointment confirmation screen. See <u>Confirm an</u> <u>appointment</u> .
Cancel	Select to cancel the appointment. See <u>Cancel an appointment</u> .
Reschedule	Select to move the schedule block to the reschedule bucket. See <u>Reschedule</u> <u>an appointment</u> .
Attach Files	Select to open the Patient Documents screen. See <u>View or add patient</u> <u>documents</u> .
Log	Select to open the audit log screen. See <u>View and export audit log entries</u> .

See also:

About drawing attention to orders and studies Create a study flag

Edit an appointment from a schedule block

You can edit the information in an appointment directly from its schedule block on the schedule book.

Procedure

- 1. Go to <u>SCHEDULE</u>.
- 2. On the schedule book, double-click a schedule block.
- 3. In the **Edit Order** or **Edit Study** screen, edit the appointment.
- 4. Select SAVE.

See also:

Edit order information Edit study information

Reschedule an appointment

On the burger menu, select **SCHEDULE**, find a schedule block on the schedule book, and then choose one of the following methods to reschedule the appointment.

Same day: Drag the schedule block

To move the appointment to a later time within the same day and same modality room, you can drag its schedule block to a new timeslot.

Different day: Use the reschedule bucket

To move the appointment to a different day, you can use the *reschedule bucket* to "hold" your appointment while you switch the calendar.

- 1. Right-click a schedule block, and then in the shortcut menu, select Reschedule.
- 2. In the **Reschedule** dialog, select the reason for rescheduling, and select **SAVE**.

The schedule block moves from the calendar to the reschedule bucket.

		and the second second	
		Accession of the	1
		Barahadada b	
		Reschedule b	
			Minimize/Maximize
and the second se		•	
actity and other	filters for Date	filler for the	Update the reschedule bucket to the
he reschedule bu		hedule bucket	some date as the schedule book

IMPORTANT:

- The facilities, filters, and dates of the schedule book are independent from those of the reschedule bucket. Changing the date in the schedule book does not change the date in the reschedule bucket.
- If a study is in the reschedule bucket and you change the reschedule bucket date, the study will "disappear" (be filtered out), but you can simply restore the original date to show the study again.
- 3. Find an available timeslot on the calendar.
- 4. Drag the schedule block from the reschedule bucket to the new timeslot.
- 5. In the confirmation dialog, select **YES**.

Copy authorization

When rescheduling an exam, if authorization is past due, a pop-up window appears prompting you whether to copy authorization to the rescheduled exam.



- To copy authorization, select **YES**. **Result**: The authorization expiration date changes to the rescheduled appointment date.
- To not copy authorization, select NO.
 Result: The authorization status changes to Reauthorization Needed.

Shorten or lengthen the schedule block

To change only the duration of the appointment:

- 1. Select a schedule block.
- 2. Hover over the bottom edge of the block, and then drag the orange bar up or down to shorten or lengthen the appointment.

Schedule a follow-up

If users specify follow-up exams, you can view a list of the specified exams and schedule them.

Procedure

1. Go to <u>SCHEDULE</u> > **PENDING FOLLOW-UPS**.



2. On the **Pending Follow-Ups** screen, enter filtering criteria and then select **Enter**.

From Date		To Date		Facility		Contact Pref
2023-05-04	<u>19</u>	2023-05-19	121	Al	*	
1411		273		A4	~	
a -						
NAME	20	io late	000	CONTROT PIEF. VISIT DATE	PROBLEM LIST ALS	LINDY LOT


3. Double-click an exam in the list, and complete scheduling tasks.

See also:


Specify a follow-up exam

Cancel an appointment

You can cancel an appointment from the schedule book.

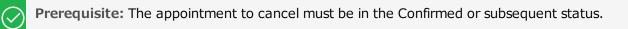

- 1. Go to <u>SCHEDULE</u>.
- 2. On the schedule book, right-click the appointment that you want to cancel, and then select **Cancel**.

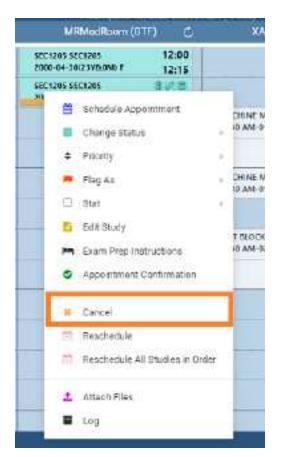

3. In the **Order** dialog, in the **Cancel/Delete Reason** dropdown list, select a reason for the cancellation.

Order		
Cancel/Delete Reason *	Select	*
	Select Adverse Reaction Child Care	
	Error Illness Physical Limitations Reschedule Scheduling Conflict Transportation others	

- 4. If you select "others" in the previous step, in the **Reason** box, type a reason.
- 5. Select SAVE.

See also:

Reason codes

Set no-shows

A *no-show* is a scheduled appointment that the patient did not attend. You can create a list of studies by type and date range, and then change the status of all studies in the list to "No-show."

- 1. Go to <u>SCHEDULE</u>.
- 2. Under or on the right side of the button bar (on the navigation bar), select **SET NO-SHOWS**.

From Date/	To Date *				
	E		E		
No Show SI	atus *				
Appoin	tment Stati	is			
	DULED HEDULED HEMED KED IN		ΞD	A351	CHECKIK GMED HOWS
Study F	Progress - 1	lech.			
STAR	T	PAUSE		END	
Rad St.	atus				
			ED CRIBED		10VED

- 3. In the filtering dialog, enter a date range, select one or more statuses, and then select **NO-SHOWS**.
- 4. In the **No-Shows for** screen, if necessary, refine the list by adjusting search criteria.

* ett Deleffield mon	9025601-01	田 8015-05-78 田	his Show Stature 1 BEH/SIGH, I	BRABCH	SECONDEL SAM
araanaaa e	000010	entroses-tim	Annalistic anti-press solar	star oxis.	A tare sure
			41. · · ·		
1934. Tast	*	CT MEDIA PRIVIS ACCONTRACT	Ω.	Schetteri	202345-111040/991101
1919.754		CTORESPONDED.	01	and we have a second se	202340-1705494007
(504, Tp#		OT ADD & PELVIS W/GOHTPACT	त	Aurorit	0022441112087W CPT

5. To change the status of all studies in the list to "No-show," select **SAVE.**

Work with billing

Exa Billing is a module of the Exa platform that lets you manage all aspects of the billing workflow from claim submission and receiving payments to preparing detailed financial reports. To configure Billing related screens and functions, see <u>Configure billing</u>.

This chapter contains the following sections.

Viewing claims Entering and submitting claims Manage payments Other billing-related tasks

Viewing claims

This section contains the following topics.

About claim summaries and inquiries Find and view studies or claims Create a filter for the study or claims list Set the default filter for the study or claims list View, print, or fax a patient claim summary View a pop-up claim summary View and modify a claim inquiry

About claim summaries and inquiries

The following describes the levels of detail at which you can edit and view claims.

Level	Description
Claims screen	The top-level list of claims. See Find and view studies or claims.
Claim detail	Double-click a claim in the Claims screen to view its contents.
Claim summary	A pop-up window on the Claims screen showing the claim's primary contents. See <u>View a pop-up claim summary</u> .
Patient claim summary	A patient-by-patient claim workspace with a billing provider focus. See <u>View, print, or fax a patient claim summary</u> .
Claim inquiry	A detailed patient-by-patient claim workspace with the ability to print statements and invoices, view claim forms, and add transaction comments and alerts. See <u>View and modify a claim inquiry</u> .

Find and view studies or claims

You can search for a study or claim and open it to view details.

- 1. Go to <u>BILLING</u> > <u>Claims</u> or **Studies**.
- 2. Optional: Select a filter tab. The default filter is **ALL CLAIMS**.

GLA	IMS		Groue	CHIIII.	Chine Permente 100	Sepirit - Rever - 14		
	ALL CLAME TOTTOM STO	ann interne int	<u>د کې</u>	XQI	001 ± 1000	TT IN ANTRONIS GU	- HILL	
E.	STREET DATE: -	SKELLET ADAM	PAVER TIME		BELKING HIOVORS	IDAM STITUE		GRAIN
	2023-81-19-1913-05-19		45	*		[] AT	1.2	43
9	or 00 2000 (10	Teal.Jultin	Princy Incodesar			AT/ Pendity-Selamanan		52(07
0	/ III III measur-no	Test Active	Pater.			Penting valiation		
0	¢≣≣ muosm	THEE 4 AM	Paters		sale new sign to scolo	ARY OVER PAYMENT		
1010	and the second se							

- 3. To find a study or claim, in the search bar, do one or both of the following:
 - Type or select one or more criteria and select Enter.
 - Select a column header to sort the list.

Note: For tips on searching, see <u>Search syntax</u>.

4. Double-click a study or claim in the list to open it for viewing.

See also:

Set the default filter for the study or claims list Create a filter for the study or claims list

Create a filter for the study or claims list

You can create a new filter for the studies or claims list.

- 1. Go to <u>BILLING</u> > <u>Claims</u> or **Studies**.
- 2. On the toolbar, select the filter W button.

CLAIM FILTER		8
Defoult Tab ALL CLAIMS: ~		ADD RELOAD
FILTER NAME	FILTER OFDER	-
2 🔒 Contenu Health	1	
🖉 📱 Isan Arna	ž.	
🖉 📋 Is Aetne		

3. In the Study Filter or Claim Filter dialog, select ADD.

					_
			SAVE	CLEAR	MO
Filter Name *	Pultor Namo	[] Inactiva			
Filter Order *	-Filler Onder	C Global Filter			

4. In the area above the blue line, enter the following settings.

Setting	Description
Filter Name	Type your name for the filter.
Filter Order	The order in which the filter appears as a tab and list item.
Global Filter	Select to make the filter available to all users.
Display as a Tab	Creates a tab on the Studies or Claims page with your filter applied.
Display in Dropdown	Adds your filter to the Default Tab dropdown list.

5. Under the blue line, enter filtering criteria.

() Claim										
O First S	Statement Cate									
🕏 Prefo	rmatted	Last 90 Days								
O Leat O Next			-		From		Ø	To		ţ
Claim Stat		YYYY MM DO	9	Facility ~	Date To	TYYYAMMAL	믭	į į		č
Ia AHS Delete Pendin AHS Delete Pendin AHS Deleted AHS Over Paid AHS Partially Paid Not				O Is O Is Not	pauthRule pe Facility skaTestFacility thick Facility	ity Fecility		1		
Billing Me	thod									
0				Ordering F	acility					
io D is Not	Oirect Billing Electronic Billing Paper Claim Patient Payment	ļ		0.16						
Modality				O Is						
0				Not						
i# D	BD III				1					
la	CR				1			-	1	

6. Select SAVE.

Note: You may have to refresh your browser for changes to take effect.

Set the default filter for the study or claims list

You can specify which tab appears when you open the study or claims screen.


- 1. Go to <u>BILLING</u> > <u>Claims</u> or **Studies**.
- 2. On the toolbar, select the filter $\boxed{}$ button.
- 3. In the **Claim Filter** dialog, in the **Default Tab** dropdown list, select the default tab.

Default Tab	ALL CLAIMS -	
RECEILNAME		FILTER ORDER

View, print, or fax a patient claim summary

The *patient claim summary* screen is a patient-by-patient workspace in which you can view, print, and fax a summary of available patient claims by billing provider and date. This screen is similar to the <u>claim</u> <u>inquiry</u> screen, but that screen provides viewing and editing of insurance, transaction, and other details.

- 1. Go to <u>BILLING</u> > <u>Claims</u>.
- 2. In the list of claims, find a claim for a patient whose summary you want to view.
- 3. Right-click the claim, and then select **Patient Claims**.

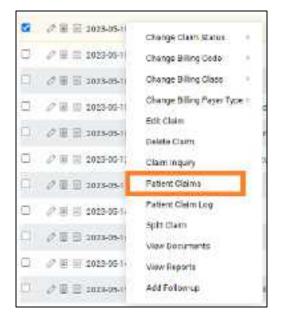

AI.	9	Unapphot Payme S65,10	ane Q	8 ALA Oxen	ctruity Ny ⊐y Data		() industs 71 () industs Al	
transinger in a jac			Tota, neuñanaz sent		9(144 Holi (99) 85.00	CONTRACT	Print Via(un)28 poto	CLAN STATUT CLAN STATUT Peoling Voltage
water t				30.04%		-		-
TENNINGY		Classical		anti-		NU DOLL		WIGHTE

4. To modify the contents of the summary, enter these settings.

Setting	Description
PREVIOUS/NEXT	Move to through patient claim summaries.
Billing Provider	Select to filter by billing provider.
All Activity	Select to show all activity.
Activity by Date	Select, then enter a range of dates in the date pickers that appear.
Include Billing Provider Info	Select to include billing providers in the summary.
Include Third Party Payers	Select to include insurance providers in the summary.
Include All Comments	Select to include claim comments.
PRINT	Select to open a screen for printing the summary.
FAX	Select to display additional fields for faxing:
	Recipient name: Type the name.
	Fax #: Type the fax number of the recipient.
	SEND FAX: Select to send.

View a pop-up claim summary

You can view a claim summary in a read-only pop-up window.

- 1. Go to <u>BILLING</u> > <u>Claims</u>.
- 2. In the list of claims, find a claim for a patient whose summary you want to view, and select its claim summary 🗐 button.

	ALL CLAIMS FOLL	IW-UP QUELIE IS AF	ETNA ISINOTAETI	NA	
1	STUDY DATE :	PATIENT	NAME	PAYER TYPE	
	2023-02-19 - 202	3-05-19		All	
0	∂ 🗏 🖹 2023-05-10	Test, John	i	Primary Insurance	
	0 🗉 🔳 2023-05-10	Test, Amb	e	Patient	
D	∂ ≣ ≣ 2023-05-10	Test, Ader		Patient	
		Const Links		1.410.02	
	7 E Test, Amber		Patient Balance	1	1.13221
	/ E	: TesA1022	S 74	4 *	6.5327
	Test, Amber F, 23Y, 2000-01-01 Account Number E CPT Codes	: TesA1022 : 77073	Patient Balance Insurance Balance	2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2	1.13221
	Test, Amber F, 23Y, 2000-01-01 Account Number	: TesA1022	Patient Balance Insurance Balance		0.00

View and modify a claim inquiry

The claim inquiry screen is a patient-by-patient workspace in which you can print statements and invoices, view claim forms, and add transaction comments and alerts.

Open the claim inquiry screen

- 1. Go to <u>BILLING</u> > <u>Claims</u>.
- 2. In the list of claims, find a claim for a patient whose history you want to view, and select its claim inquiry 🗉 button.



3. To move through patients, select **PREVIOUS** or **NEXT**.

ler No.: MC-10	000124					PARTEMENT	1473
lian liquiry	Dare Status Pendag Weldebon		iy Culu (2033-05- 10	KB Codes	PATOTOLAME	WVE PRIE	
ding Extwider	Tes StruProvie			coin.	0100001004		
tridoliking navidar							
diang Paula							
er rice Fassility		Place Of Service					
S ran	35.00	ABOVAS	50.00				
etient Faild	53.00						
thar Paid	\$3.00						
quittaine	-\$73 kHz						
stanta	30.00						
dianes.	30.00	Feloriy can	PROVABILIS I				
ding troops			1				
SUBDOR							

Add a transaction comment (claim note) or alert

1. In the Transaction Detail area, select ADD.

Trans	action De	lion.					ACD
		CONCERNED.	(100.0001109.0	COMPANY A	HIGURE ALL COMMENTER VARG	CHAND FRAME /	GENERAL SEALON PARAMONIA
			2222-01-19	31032-05-19	DIVE LENGTH OTHER DIFFERE	00.00	8
$\left \mathcal{P}_{i} \right $;D) (61+25+0505	1323425-10	Galarizmented and OS/CR.		

2. In the left pane, type a comment.



3. Optional: Select checkboxes for screens on which you want your comment to appear as an alert.

4. Select **SAVE**.

Other tasks within a claim inquiry

- To add a reminder to follow-up on the claim, in the **Follow-Up Date** box, select a date, and then select **SAVE**.
- To add billing notes, under **Billing Notes**, select the notes 🗏 button, and then select **SAVE**.
- To print a statement, select **PRINT STATEMENT**.
- To print a statement for a selected recipient, select the **PRINT STATEMENT** button shortcut
 menu, and then select a recipient.
- To print an invoice, select **PRINT INVOICE**.
- To move to the patient claims screen, select **PATIENT CLAIMS**.
- To view insurance forms, in the **Insurance** area, select a form button.

Entering and submitting claims

This section contains the following topics.

Enter a claim for a study Enter a non-study-related claim Select a delay reason for a late claim Enter claims in batches Create claims automatically with Auto Billing Validate claims Add a claim to the follow-up queue Make quick changes to claims Split a claim Submit a claim

Enter a claim for a study

For studies that are completed but unbilled, you can enter a claim based on the information in that study. For other charges, see <u>Enter a non-study-related claim</u>.



Prerequisite: Configure a billing provider.

Procedure

- 1. Go to <u>BILLING</u> > <u>Studies</u>.
- 2. Right-click a study whose billed status is Unbilled, and then select Create Claim.

	ALL STUDIES	CONTAINE HEARTH	IA HEALTH PLAN III
С:	ACCOUNT NO. :	PARIENT	011.000.877
			Unsafed
0	19071001	1904, Test	UnSiled
5	19071001	create Clarn	UnBiled
	10051001		and a second

3. Optional: In the **Claim Creation** dialog, on the **CHARGES** sub-tab, select **NEW CHARGE**, and enter a **CPT Description**, **CPT Code**, and pointers and modifiers as needed.

				PER NOT	1010		anaesin	AND REPORT	11	-546
	C.eto	Arreadon Ne.	CPI Code	CPIT Description	=	ite	83	P4	945	147
F ×	2122-10-32] (+ 11)	(4077 ÷	CT ASD & REDNOW SOM FINGT	lir_	10	i 🗆		T.	
4 x	2022-25-10	12	Salast +	Gelezt -	HIT.	1 100	1		177	100

4. In the **CLAIMS**, **INSURANCE**, **ADDITIONAL INFO**, and **BILLING SUMMARY** tabs, enter all necessary information. For a detailed description, see <u>Settings in the claim entry screen</u>.

Ciaim Creation :	1904, Test Acc-#, 190	171001 <i>2000-01-01</i> F C	Alerts Patient Char	D.		
CRANKED C.T.	ell' novecetti ko	INTOIN, SPC				-
Claims			1017 4617	ALL R	president and account	LANG
Hoty Date !	JAL2-12-12		Cluges av Cristes			
Taility Nerve *	CaluberTeenTeering	124				
Billing President	Relant	Sale				
Restarting Provider	Select Seed, Provider	1.4				
Referring Provider	Ealart Reise Provider	-				
Droken gtackty	Salaci Oxforrigi Factily					
Service Particip Localities	Sciel3	2				
POS Type	Rolant	1.11				
Primery Insuranc	e same		Secondary Insur	ance HTM		
fraingloosece	- 101.36	Access Assignment	fileing heaving	95.00t	- Chickst Aesignment	
Darket	ATTWO REACTIN PLAN	· [#]		Ved steeper		
			Gener		-	

5. Select **SAVE**.

See also:

Keyboard shortcuts for the claim entry screen

Enter a non-study-related claim

You can enter a claim for charges unrelated to studies, such as for CDs or medical records. Even though this procedure is primarily intended for unrelated charges, you can also use these steps to associate charges with a study.

Procedure

1. Go to <u>BILLING</u> > <u>Studies</u>, and then select **NEW CLAIM**.



- 2. In the **Claim Creation** dialog, search for a patient by name or MRN.
- 3. In the **PATIENT SEARCH RESULTS** list, select a patient, and then select **CREATE WITHOUT STUDY**.

Claim Creation				
Starts With 👻 🔠	Y Sho	w Inactive		
test. T	int Name	MRN		
PATIENT SEARCH RESULTS		RECE	NT CLAIMS	
Test, Rich tesr1009, DOB: 2000-09-09				No Claims For Selected Pa
Test, Patient2_tesp1058, DCB: 2000-04-30 ComboApptType1 (Act 12) ComboAppType2 (Act 12) WTH STUDY WT Test, 1205_tes11055, DCB. 2000-05-01		and for an area of		

- 4. In the Claim Creation dialog, on the CHARGES sub-tab, select NEW CHARGE.
- 5. Enter settings as needed and then select **SAVE**.

See also:

<u>Settings in the claim entry screen</u> <u>Keyboard shortcuts for the claim entry screen</u>

Select a delay reason for a late claim

If submitting a late claim, you can add a reason for the delay. Exa PACS/RIS comes with preconfigured delay reasons, and you can configure additional reasons if needed (see <u>Add codes and other basic data</u>).

Procedure

- 1. Go to <u>BILLING</u> > <u>Claims</u> and double-click a claim.
- 1. On the **ADDITIONAL INFO** tab, at the bottom of the **Additional Information** section, on the **Delay Reasons** dropdown list, select a reason.

Edit : Test, Amber Acc4: TesA1022 2000-01-07 F LiAlerts E	WARP PROMOTE		8
Addussiting 2 Accounting 2 Acco	tonos Itain Autoricensatie Frequence	Oles Arthresis VI	34/2
Service Type Hold Statemed = Service On Only to 2020-95:04 Service On Only to 2020-95:04 Billing Summary	Ddag Roosso	Seed - Used - 1-Proof of eloptanty environmentations 3-Antoropation 3-Antoropation Delayer 4-Belay in Certifying Provider 3-Delay in Supplying Billing Perms 5-Belay in Supplying Billing Perms 5-Belay in Supplying Billing Perms	
BillFes 0.00 PatientPac 0.00 Adustment 0.00	Aliyees) Othan Pag	7 - Triod Party Processing Delay III - Delay in Childry Dietermonium 9 - Original Chine Rejector's Control Civic Is a Ray 13 - Administration Delay in the Third Approval Pro 11 - Offan 15 - Hanuer Delay in 1991	

2. Click SAVE.

Enter claims in batches

To make it easier to process large numbers of claims for different patients, you can enter, edit, and validate then in batches.

- 1. Go to <u>BILLING</u> > <u>Studies</u>.
- 2. Select multiple studies in **UnBilled** status, and then select **BATCH CLAIM**.

STU	DIES			Radine Barne F		Aust - Berry	mart i New AND
	ALEIDIE			(Tyoo	87	HEALAM	BOTHELAW
57	WOODLAT HELE	PROPERTY.	PARE PARA	IN/OV MATE		COLUMN IN	
			LU .	Access - mains			
Ċ.		(Non-Section		annes a communa	etimeters	in the second	
ē.	44.00	4.00	the states	1000-0012 MARINY EST	ANY MED	CLARE COMPLETE	
a	44:00	18, 500	Unit Des	2022-05-12 11/10 CM COT		OURE CONFECTE.	
-0	CALCULATION OF THE OWNER OWNER OF THE OWNER OWNE	15.00	Include 1	metaciri il minorar	11000000	ARE OBSERVED.	

3. Go to <u>BILLING</u> > <u>Claims</u>.

4. On the search bar, in the CLAIM STATUS dropdown list, select Pending Validation.

Result: Claims for all studies that you selected earlier are listed on the screen (along with any other claims in Pending Validation status).

	Catha Parlace	and the second s	ini - sini Moteonico	 1942 - 1941 1947 - 1947
PATER TIPE	NORMOLE PROVIDED &	TLANA ST		ILLEN WA
MI ~		Pentraga	shouse	40
Prinary Disaste	AARP MEEICARS COMPLETE	Pending 14	Idaton	ED 92
Premary Insurance	EARP DEDICARS COMPLETE	Servings	sisten.	EU-SP
	a local second start days and start	- 16 11		-

5. Open the first claim in the list, edit as needed, and then select VALIDATE.

II.		Liste	Accession No.	CPT Code	CPT Decodption	FI	63	.ea	- 14	MT	þ
t	×	3020-0515	279	75520 +	DOP VELOCIVIETRY PTL UNBLUCAL ART	1	3	3	4		1
÷	×	2023-05-15	-276	70820 +	DOP VECCOVETREPTL MIDDLE CEREARE 4	1. C	ũ.	3	\mathbf{T}		

6. Select **NEXT**, and then repeat the previous step until all claims in the batch are processed.

Create claims automatically with Auto Billing

With the Auto Billing function you can create *auto billing profiles*, which are sets of criteria evaluated against study information. When a study matches all criteria in an auto billing profile, Exa PACS/RIS automatically creates a claim for the study. To create an auto billing profile:

Prerequisites:

- Configure items to use as auto billing criteria, such as facilities, modalities, and service codes.
- Obtain the Autobilling right from your administrator.

Procedure

 \checkmark

1. Go to <u>BILLING</u> > <u>Setup</u> > **Auto Billing**, and then select **ADD**.

SETUP		Ganasa Justime Babra	Twomas D	7) Second Second	T Los Off
				C ARD	muna
KLENTHER (1944)	101019/1010	antari anenan		RELEASE CLARK STREET	
milliso cobes		1 A		48	
BRENDERLANDER	2 E 1250	6204		Panding Processed	

2. Type a name for the profile in the **Description** box, and in the **Results in Claim Status** dropdown list, select the status to assign to claims created based on this profile.

Decription 7	Résulte m Claim Statue	•	Pending Velidence	¥	() inactive
Study Status *					

- 3. Under **Study Status**, **Facilities**, **Ordering Facilities**, and other categories, add one or more criteria as follows.
 - Select the **Is** or **Is Not** option (for example, **Is**).
 - Select a study status in the dropdown list (for example, Tech Start (TS))

Study Status *			
⊖ ^{Is} ⊖ Is Not	Tech Start (TS) •	Tech Start (TS)	
Facilities			
() is () is Not	AngAuthRule (ang) *		
Ordering Facil	ities		8
OB	Karthick Ordering Facility +		1
⊖ ls Not	+		
Modalities			

- 4. Select the plus + button.
- 5. Select SAVE.

Result: In this example, when a study changes to the *Tech Start* status, Auto Billing creates a claim for that study.

See also:

Add a facility

<u>Add modalities</u> <u>Configure procedure codes</u> <u>User rights</u>

Validate claims

You can validate claims to ensure that information is present in all required fields.

Procedure

- 1. Go to <u>BILLING</u> > <u>Claims</u>.
- 2. In the CLAIM STATUS column, select Pending Validation.

a	VMS		Sector 18	letter Cherry Reproptie atta	House Heart Lag	e: Logan
		THE REPORT OF	10.5	KOBOVA MAN	ELECTIONIC CLAS	462)
\sim	ER, DY NATE ±	NAMENTRAME	PAVENTYPE	HOUTHING PROVEERS	NAMONIN'S	atina
_	2020-01204-2020-0404		1.040	14.2	Pendry Welaster	
2	// E = ==25,05.10	Tast annor	Fatare		Fending Validation	
	27 EL 2 2023-0110	Even005, Fee81333	Presag Insurance	AND MENCINE CONFLETE	Turning Valention	there.
13	₽ III 2025-01-12	Lasta.extoriziyubpingiti. Auto	an sta Patient		Pending Velication	
in .	of the second and	Test Dente	Distant in Contract	ATTACAMENTAL TO AN	Tendor Valoritor	

3. Select claims to validate, and then select VALIDATE.

Result: Claims are validated, and their status updates accordingly. If any options are invalid, the program displays them in a pop-up window.

See also:

Configure claim validation

Add a claim to the follow-up queue

If you know that a claim will need further attention in the future, you can add it to the follow-up queue. To view the follow-up queue, in the upper left of the **CLAIMS** page, select the **FOLLOW-UP QUEUE** tab.

- 1. Go to <u>BILLING</u> > <u>Claims</u>.
- 2. In the list of claims, right-click a claim and then select **Add Follow-up**.

140	5. @ E 5053-02-10	iest, Amber	Patient
	ℓ 🗄 🗏 2023-05-10	Test, Aden	Change Claim Status
	⊘ 🗏 🗏 2023-05-10	Test, Aylisa	Change Billing Code
0	⊘ 🗏 🖹 2023-05-10	Test, Alaxander	Change Billing Class
	♂ 🗏 🗄 2023-05-10	First1005, First1005	Change Billing Payer Type I
۵	J 🗐 🔠 2023-05-12	Lestautotestutyutyurg	Edit Claim Adjust to Paid in Full
	⊘ 🗄 🛞 2023-05-12	Test, Brenda	Delete Claim
	⊘ 🗑 🗐 2023-05-14	Test, Mary	Claim Inquiry
D	⊘ 🗉 🗏 2023-05-14	Test, Ben	Patient Claims
ū,	J 🗄 🖹 2023-05-14	Test, Jovery	Patient claim Log Split Claim
	⊘ 🗄 🗎 2023-05-15	Test, Richard	View Documents
D.	⊘ 🗏 🗏 2023-05-17	Test, Amy	View Reports
0	₫ 🗄 🖹 2023-05-17	Test, Alton	Add Follow-up

3. In the **Add Follow-up** dialog, enter a follow-up date.

Follow-Up Date		
2023-05-01	13	
Follow:Up User	Alest	0.0

- 4. Optional. If the **Follow-Up User** list appears, select the person to whom to assign the follow-up.
- 5. Select **SAVE.**

Make quick changes to claims

You can make quick changes to claims without opening them for editing.

Procedure

1. Go to <u>BILLING</u> > <u>Claims</u>.

2. In the list of claims, right-click one (only) claim and then select one of the following.



Change Claim Status – In the submenu, select a new claim status.

Change Billing Code – In the submenu, select a new billing code.


Change Billing Class – In the submenu, select a new billing class.

Change Billing Payer Type – In the submenu, select a new payer type.

Split a claim

If a claim has more than one charge, you can *split* the claim by moving some of the charges to a new claim for the same patient.

- 1. Go to <u>BILLING</u> > <u>Claims</u>.
- 2. Right-click a claim with multiple charges, and then select **Split Claim**.

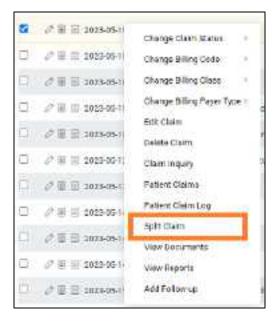

3. In the **Create/Split** dialog, find a charge that you want to move to the new claim, and select its plus (+) sign.

Create/Split					×
charge List		RELEASE	Selected Charge List		SAVE
Account No. F#F1027 Policent Hest1005, Hest1005 Referring Provider	Claim Number 1 CPT Description RADEX SEUTS SymCirc Views CPT Codec 72202	+	Account No. FirF1027 Patient : Fire11005, Pire11005 Rolenneg Providor	Camillanbe : 1 Off Deceptor RADEX SLITS 3/MORE VIEWS OFT Codes: 22202	17
Account No. FirF1027 Patient: Post2005, Hind1005 Referring Pravider	Class Humber 1 CPT Description RADEX 32 JTS 3/MCRE VEWS CPT Codes : 72202	+	~		
Account No. F#F1027 Patient - Anx1005, Mex1005 Referring Prevaler	Claim Number 1 CPT Description RADEX SEUTS XMCRE VEWS CPT Codes 172202	Ŧ			

4. Select SAVE.

The new claim is added to the **CLAIMS** page.

Submit a claim

You can submit one or more claims of the same billing method.

- 1. Go to <u>BILLING</u> > <u>Claims</u>.
- 2. On the search bar, in the **BILLING METHOD** column dropdown list, select a billing method (for example "Paper Claim").

œ	AIME			CORNEL N	utice) Coine: Provent	: LUIS Property	e disare Lar	100.01
	- 100	TAME PROVIDEN	16 (1.16) (1.16)		Mide74	VIGINATE	THESE DUASHIE	16.M) 😒
E		100/1475 ±	Estimative and	DIGITIVE	DELING APRICO	100	262314181 22440	CLAIMETATUS
		TTD+02-34 - 1183402-34		44.	· Pager Carro	-		M.
D	2日日	2019-05-36	Text Autom.	Prenety Incomes	Pape Deery	ACTH	A HEALTH PLAN	Prodist Person
D.	AUL	ANEN-CO-OC	Direct Agencia	Energia Novament	Paget Diama	ACTIN	ANCACTHIPLAN	numa

- 3. In the list of claims, select one or more claims, select **VALIDATE**, and then resolve any issues.
- 4. In the upper-right corner of the screen, select the billing method selection button, select the same billing method that you selected previously (e.g. **Paper Claim**).
- 5. Select the button (e.g. select **PAPER CLAIM (B&W)**), and then do one of the following.

- **Electronic claims** In the EDI Claims dialog, select **DOWNLOAD**, and then upload the edi.txt file the clearinghouse.
- **Other claims and invoices** Download or print the claim or invoice.

See also:

Configure columns on the Studies and Claims screens

Manage payments

This section contains the following topics.

Find and view payments Add a payment Step through claims to add payments Apply payments against a customized list of claims Adjust claims to paid in full Look up and edit applied payments Export the payments list Exchange files with healthcare agencies Apply TOS payments Show or hide charges on printed receipts Write off a small balance

Find and view payments

You can find and view existing payment information for reference or editing.

Procedure

1. Go to <u>BILLING</u> > <u>Payments</u>.

PAYMENTS							P.	-		Pagravets +111	Report- Lens	e ine inite
Fayney Da.	na ja -		erns ton Victoria	en vent		Statistic 1908	104 Factoria Applied 45278-80	Torul Dirul		01 1020	the second party of	ALMANNE PER
1 HATHER PRICE	III AMERICAN	Antise Armont	-	NAME AND DESCRIPTION OF	N-THEFT	es even	ACREASE A	a matalene.	General	were the	tear Automa	a contaction paratest
		102084-021	11								40.0	1.000
28	NORMS.	2014421	restains	ATTRACTOR PLA		000000	0010.00	1010	10.02	19.0	enia Cant	Bry Demonstra
/m	1010.00.00	2011.01.21		ITTALIATE PO		008.05	110 FT		12.05	5.5	the C field	MyDorque
/14 /11	10000	2022-0010	Palmen	Test, Tant	Test tage	110.00	60.02	100.20	in co	Tec.P	tertin dani	for eith Test

- 2. To filter the list of payments, in the search bar, do one or more of the following.
 - Type or select one or more criteria and then select Enter.
 - Select a column header to sort the list of claims.
 - In the upper-left part of the screen, select the Payment Status button and then select one or more payment status filters.



Note: For tips on searching, see Search syntax.

3. Double-click a payment in the list to open it for viewing or editing.

Add a payment

You can add a payment based on an EOB or an invoice. "Adding" a payment *enters* the bulk payment in the system, but does not *apply* the payment to claims and charges. To apply added payments, see the "Apply payments" topics later in this section.

Procedure

1. Go to <u>BILLING</u> > <u>Payments</u>, and then select **ADD**.

FAYMEN	កទ							ren via	- 20	Pagreets	ter top	ari- leng	fige light
Pagesky Sale	i Darian Billionna (n. 19	-		a na weat at kaana minin		Station 1904 10 Station 200	ACTIONS ACTION	Tread		51 10	19594	and a second second	diata no se
1.95581	NUMBER AND DESCRIPTION	. Altockymus	1.8603.79	T. POPERATE	NAME	e enterna	Concentry.	o muutee	General	anne :	-	NUMBER	contaction surplets
		NUCLEAR D	a .										1.00
2.8	(NORMS)	2014421	reations	ATTRACTOR (F)	í. –	00000	000.00	1010	10.02)	in iteration	Cent	My Contents
ina -	1012-01-02	2011-01-21		ATTACKATE P	6	000.00	110 011		12.05	1	believen.	Cart	MyDoreput
14	1002100-00	2002/00/10	Palare	Test, Tans	Tertior	110.00	10.22	101.20	in co		he, hirefu	Gani	Parentin Treat

2. In the **PAYMENTS** screen, enter the payment details.

AYMENTS		Cenaua	Studies	Claims	Paymenta	FOB Report -	Sitte
Payment ID	<u>u</u>			Ret	ference		
Payer *	Select v			Pay	ymeint ID		
S2 553 (1971)				Pay	yment	Select	×
	C EOB C Invoice No.			Rea	ason		
				Pa	id Eccation *	My Company	¥
				Pay	yment	Select	U
				Mo	de		
Accounting Date *	2023-05-24			ch	ack/Caid		
	E			Nu	mber		
Areount *	0.00			Cal	d Name		
Applied				No	tes		
Balance							
SAVE CLEAR	BACK						

- 3. Optional. New buttons appear in the middle of the screen that you can use as follows.
 - To add a new payment, select **ADD**.

- To print a record of the payment, select **PRINT**.
- To delete the current payment, select **DELETE**.
- 4. Select SAVE.

See also:

Apply payments against a list of unpaid claims Apply payments against a customized list of claims

Apply payments against a list of unpaid claims

You can apply payments to an automatically generated list of unpaid claims for the payer.

Procedure

- 1. Go to <u>BILLING</u> > <u>Payments</u>, and then double-click a payment.
- 2. In the bottom pane, select the **PENDING PAYMENTS** sub-tab.

PAYMENTS		Septim. South	. Calua Panena 198 Aurany Sela	r ille llagett
Applied	30.90	Hzisa		
Ediato	Quant as			
344C A00	000-0000007 COLUT	C SACH CARUNDATE		
Instance of the sets	the maximum state of the second second	anna ere rotentettar		-
				(WITTERSON
CLANN BARAKER	Second Second Amore	IT THE DAY	BILINE BALANE OFFICES	ABERATINE.

- 3. Optional. Sort or filter the list with search criteria.
- 4. In the list of payments, double-click the payment to apply.
- 5. In the dialog, edit payment information as needed and then select **PAY IN FULL**.

Result: Exa Billing applies payment to the line items in the list in order, until the payment balance is 0.

6. Select SAVE.

Step through claims to add payments

You can step through claims and add payments to them.

Procedure

1. Go to <u>BILLING</u> > <u>Claims</u>, and open a claim.

2. On the **PAYMENTS** sub-tab, select **NEW PAYMENT**.

				nisy liser	Name	DOBLINENTS AND REP	IONTS	TALENTE	540
npe	insite .	2.08	Later			100			
fislance		55.34			#EngCodes	20Het			
Etam?	* intes	Over Pacenes			Editory Class	delect	v		
Bilru i	and and a				Secondile*	AAPOUPPIEVE	N7.0200 +		
eyme	and and a	Parts Nature		Prat. Mode	Secondie*	Avitr Curriive	NT.GECO +	Paymini ID	

- 3. Enter payment information, select **APPLY**.
- 4. In the detail screen, enter payment details.

Optional: To attach documents, select **DOCUMENTS**.

Ţ	otal Bill Fee : 90.00		Applied	Payment: 0.00		Applied A	djudtment : 0,0	10	
elect TPC	CPT Description	Billfee	Payment	This Payment	Althred	Adjustment	This Adjustment	Balance	CAS
77073	BONE LENGTH STUDIES	0.0.0	50.00	0.80	0.00	0.00	0.00	50.00	GAS
									-
								SAVE	CLE
			4.djustm	on: Salact		t.		SAVE	OLE

- 5. Select **SAVE**, then close the details screen.
- 6. Select **PREV/NEXT** to step through claims and add payments.

Apply payments against a customized list of claims

You can apply payments to a list of claims that you can customize first by patient, and then by other criteria.

Procedure

- 1. Go to <u>BILLING</u> > <u>Payments</u>
- 2. In the list of payments, double-click a payment.
- 3. In the bottom pane, select the PENDING PAYMENTS (BY PATIENT / CLAIM / INVOICE) tab.

PRYMENTS			11999	**	en manada ISE Anor	er milde loge ligter
Saler			Deter			
24400	31,000.8					
SAVE ALL	HHMT	www.uerne	Selere Search Hill	INENTS		
10000314001043	PERDING THAN	ENTERIN PATIENTALIA	Manchaener Alvinca Para	00		
ending payments i	tur me påtiont:		078)			BATHERI BASIK TO SEARCH
stati sovern	Heroset He	even	67101,0426	014.755	ALLANDS OF DODES	4000,4780

- 4. Use the search tools to find a patient, and then double-click a patient in the list.
- 5. In the dialog, edit payment information as needed and then select **PAY IN FULL**.


Result: Exa Billing applies payment to the line items in the list, in order, until the payment balance is 0.

6. Select SAVE.

Adjust claims to paid in full

You can automatically apply an adjustment to claims to be paid in full.

- 1. Go to <u>BILLING</u> > <u>Claims</u>.
- 2. On the shortcut menu of the claim to pay, select Adjust to Paid in Full.

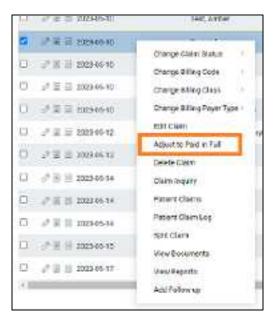

3. In the **Adjust to Paid in Full** dialog, in the **Adjustment Codes** dropdown list, select an adjustment code.

Adjust to Paid in Fu		8
Test, Aden, 2000-01-01, 23 Date of Service 2023-05-10		SAVE
Claim Balance (\$50.00)		
Adjustment Codes	SBCA - Small Balance Credit #	•

4. Select SAVE.

Look up and edit applied payments

Especially when a payment was applied in error, you can look up applied payments and edit them if necessary.

- 1. Go to <u>BILLING</u> > <u>Payments</u>
- 2. In the list of payments, double-click a payment in the list.
- 3. On the **APPLIED PAYMENTS** sub-tab, double-click a payment.

WYMENTS				(Correct)	er sikare	sound 18	aparents 6		+ sump+	teg- lig	
	11		Test	Ht.							
Aniert*	00.008		fant								
Agental	0000 63		Netze								
Device	82(11)										
THE PARTY OF T				LED MONES	m						
											1517123
CLAIM NAME IN	WARKS TO WHERE &	ILANDAD	HLEP.	- FATTERY PA	- PROVIDE	ACARCHE	THEAT	141140015	MIRKE	15141085	
22.76	billion, tarticos	2022/06/25	300109	10.00	31.92	pc.ce	28.00	001424	30.04	13079	٦

4. Edit the payment information.

CIDIII. # 19,	Billtest, Karthick	DOCUMENTS -			Total Payme	ent Record B	alanco : S	0.00
Totel	Bill Fee : 0663.00	Appled Payrie	o.0168.1m	App	ied Adjustriei	N. 56.00	ä	DR .
লশ Code	OFT Description	Bill Press	ayment. This Poyment	Newad	Adjustment)	This Adjustment	Balareo	CAS
7707E CT BORE	MINESAL DEMUTY STUDY 14	SITS AXIAL SKEL BOS 00	boos co.o	8.00	0.06	1.03	800	CAS
PAV DE PAR L							er staten	ent -
PEN IN FRILL Toto I BIT Foo	1865.90	Clam *	Dansing E		Deburhos			- 10-
1000	\$800.00	Ötatus	Panding E 🐱		Deburnos Geltaurace		uv#=	- 10-
Thto I Bill Foo	001500		Danaling F 💌 Relitions, Kasthicels/Partia		Celtsurace			- 10-
Total Bill Fao. Others Paid	\$800.00 38.00	Ötatus		•	Cellsiumoce		WE	- 10-

5. Select SAVE.

Export the payments list

You can export the contents of the currently displayed payments list to a CSV or PDF file.

Procedure

1. Go to <u>BILLING</u> > <u>Payments</u>

- 2. Filter the list of payments as needed.
- 3. Do one of the following.

		Canad	i Studieu C	niikin d	Payments E0	8 Report +	Selup + Log +	Log Off	
iymant Total \$3,058.00	Total Paymont Applied \$1,518.00	Adjustment Tot \$10.00	tal AD	0	HEITRECH	EXPORT	DENERATE PDI		
PAYMENT	an PAYMENT OFF	BALMVCE	ADJUSTMENT	NOTES	Postéo	by Permits	TMO CHECK/CARD	RECLIPT	
						ILA	- U	All	¥
\$1,000.00	80.00	00.000,00	00.00		Vidtek Pa	a Cash		Norvila Te	it F
3840.05	VAMP OF	100.000	Re IN		En Reinida	Co10		My Compa	-

- To download a CSV file, select EXPORT.
 Result: Exa Billing downloads the list to your Windows Downloads folder.
- To download a PDF file, select **GENERATE PDF**, and then download the PDF from the preview window.

Exchange files with healthcare agencies

You can exchange files such as payment records with healthcare agencies if supported by their clearinghouse (or if the healthcare agency acts as its own clearinghouse).

Procedure

1. Go to $\underline{BILLING} > EOB$.

CL.	anvs				them when the rest from the sector of the					
	- 400	TAME PELLE PRIME		EHENN			WINHES THE	molasici av jezt - 1		
C.		atvevevte -	ототные	THE STORE		04	****	CLANNETWEDO		
		2125/0224 2025-02-04		- Sal		dan 😳		at .		
P.	$\mathcal{S} \models \Xi$	2021 (9.10	Test, all te	(*****) *********		Networking	ADD DEPENDENT/DEPA	Mett Door Payment		
0	Gan	COLUMN TWO IS NOT	THE ADDRESS	- Passa - La		Partner Process		- Personal Concession		

2. In the EOB screen, do one or more of the following.

	Denia 30	dae Carrie Payments FOR De	par-s imp+ t	ar unent		
	Lowrload	OF CAD & PROCESS EDE THE VERSE	WEDEFILE	HELVE		
BEID PLEBAME	TUTE	THE IPERTODATE THE	STATLS			
11		10/3-44-13 - 3013-69-24	4	Υ.		

- To preview the file before importing it, select **PREVIEW EOB FILE**, browse for and select the file, and then select Open.
- To receive a file, select Download.
- To send a file, select UPLOAD & PROCESS EOB FILE, browse for and select the file, and then select Open.


Apply TOS payments

You can automatically apply TOS (type of service) payments to claims having the same date. If the claim has more than one charge, the payment is evenly distributed among the charges.

- 1. Go to <u>BILLING</u> > <u>Payments</u>
- 2. On the **Payment Status** button menu select **Unapplied**.

Pir	www.millBlatan	1	APPLY TOS P	NIMENT	Payna	ent Sobal Tat	GR 2003240	Adjustment		especial.
	LINAPPLED *	DAN	L BALLARDS	ADJUSTINENT	\$1,7	00.00	Applied \$0.00	Tatal \$6.00	GENERATE PO	
1	T-woles	COLN'RO	WEITH	PETERNAVE	PATIENT MITS	PANADA	A PANMENTS	BALANCE	ADJUGTME MOTES	1000
0	Downaded 🖸	1 13-04-25	λI = =							
	Partol Applied	1.160.24	Paratete	Test 1005	18511028	81,000,01	88.03	07,003.00	89.00	Votel
2000		10 CE-18	Patient	Test Tox	TesT1007	\$16.00	40.00	\$50.50	80.00	Soc.14
1	C Diel 700ies	P 445.19	Patient	Text Autorsy	LooA 1060	314.05	\$6.05	05.082	\$9.181	ing the

- 3. On the search bar, select:
 - PAYER TYPE = Patient
 - An accounting date range

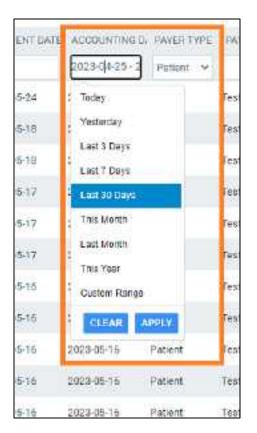

4. Select **APPLY TOS PAYMENT**.

See also:

Print TOS payment receipts

Show or hide charges on printed receipts

You can print receipts for TOS payments, and show or hide charges.

Procedure

1. Go to **Burger** > <u>Payments</u>.

Payments														
Payment Dans					Mayor Tar	tal.			8	a i waaich	11-3614 17	 9		
1 John Server	n waxaaniin	NUTRING AN	Sheet Inc.	US/WWW.B.	RATING AND	PROMINENT V	PROFESSION	autors.	100536	1000	HESER	 + (HOUCE	L SHOP	-
h.σ.	2952-27.94			Teol 5 Sartina L	Tref-SQC1	\$1.00	61.00	10.00	30.03		Baby: Banta		1 percent	-

- 2. Double-click the payment for which you want to print a receipt.
- 3. In the details at the bottom, select the CPT or service codes to print.

			Colored and Colore	101000	
-	10.000	128	annow/ intractive print.	9500	
-	11000	1.00	and an and the second s	and statistics	_
	1204-040				
•	annes.	1442			
-	ALCONOMIC -	1000	the second	1.00	
	0.04.00	-	have and have	1949	
			24949-4471 #4:0001991/PUTPE	10+	2.

4. Select **PRINT RECEIPT**.

Write off a small balance

If the patient's total account balance falls below a specified amount, you can automatically write the balance off (adjust it to zero).

Procedure

- 1. Go to <u>BILLING</u> > <u>Payments</u>
- 2. On the **Payments** screen, select **SMALL BALANCE ADJUSTMENT**.

PAYMENT	S					(ida	inin Stu	Ses Clau	Payments
Paymont S ALL BE	ostas Leotreo (4) -	P COLUMN	APPLY TOS	RAYMENT CADJUSTMENT	1	,058.00	Total Paymorit Applied	Adjustmen Total \$10.00	ADO
FAYME REP	ERI PAYMENTO,	ACCOUNTIN	PAYER TVP	PAYER NAME	PATIENT MRN		FAYMENT	BALANCE	ADJUSTRE NOT
		2021-04-25	All ~						
P 36	2023-05-24	2023-05-24	Patient.	Test, 1605	Tes11078	\$1,000.00	\$0.08	\$1,000.00	\$0.00
2.35	2023-05-23	2023-05-23	Insurance	ARTNA HEALTH	£	00,0488	8860.00	60.00	88.69

3. In the **Small Balance Adjustment** dialog, type the total patient account balance at or below which you want to write off that balance, and then select **NEXT**.

Small Balance Adjustment								
Maximum Account Balance	10.00	NECT	COMPLETE					
PATIENT BALLE	4000010100			uoe	PATENTORIALE			
+ Tezt, Ayfiza	TabA 1026			2202-01-01	\$10.00			

4. Confirm the contents of the results list, and then select **COMPLETE**.

Other billing-related tasks

This section contains the following topics.

View and edit the patient chart in Billing Display alerts in claim and payment screens View patient alerts in Billing View and enter notes and reasons for study View and download a patient document Upload a new patient document Create a billing report View approved reports associated with claims View documents and reports in billing Create and deliver a patient statement View the patient claim log Send claims to collections Open an insurance provider's website from a claim

View and edit the patient chart in Billing

You can open and edit the Exa PACS/RIS patient chart from within Billing.

Procedure

- 1. Go to <u>BILLING</u> > <u>Claims</u>, and double-click a claim.
- 2. At the top of the screen, select the **Patient Chart** link.



Result: The Patient Information tab of the patient chart opens.

3. Optional. Edit patient information and select **SAVE PATIENT**.

Display alerts in claim and payment screens

You can add comments to billing transactions and select them to appear as alerts in the Edit Claim, Claim Inquiry, Payment Reconciliation, and Patient Claims screens. Follow the steps in <u>View and modify a claim inquiry</u>.

View patient alerts in Billing

You can view patient alerts from the Studies screen. To view patient alerts from the Edit Study screen, see <u>View patient alerts</u>.

Procedure

- 1. Go to <u>BILLING</u> > <u>Studies</u>, and then open a study.
- 2. At the top of the Claim Creation or Edit screen, select Alerts.

CHI	VIGES	2.5M2	HIDURANNES AND	THE MAKE WITE	THUNG GUNNARY					
					PREV HEN	HORE	COCUM	впехир	den-divers	
		Deto	Accession No.	CPT Code	CPT Description		PT.	P2	63	P4
¥	×	2023-05-12	:200	77053 -	COMPLITER A DED O	ETECTION ON MAN. +				

View and enter notes and reasons for study

You can view previously entered patient, study, schedule, and reason for study notes, and enter new ones.

Procedure

- 1. Go to <u>BILLING</u> > <u>Studies</u> and then double-click a study in the list.
- 2. In the Edit screen, select Notes.

TARGES	autiva II	NORMAL CAR	STICKAL HITC:	BILLING ILMINAW	-				
				HER NEXT BODS	BOGIN	-	esperies	58	ar -
	Date	Addression No.	CPTCade	CPT Description	PI	62	93	PM	м
• ×	20282618	2340	770101 +	COMPUTERABLE DETECTION DICKSIN.	1				

- 3. In the Notes dialog, select a tab (such as STUDY NOTES), and then do one of the following.
 - Select ADD, type the note, and then select SAVE.
 - To find an existing note, enter criteria in the search bar.
 - To edit an existing note, double-click it in the list.

See also:


Enter notes and reasons for study

View and download a patient document

You can download any patient documents that are available in Exa PACS/RIS.

Procedure

- 1. Go to <u>BILLING</u> > <u>Claims</u>.
- 2. In the list of claims, right-click a claim and then select **View Documents**.

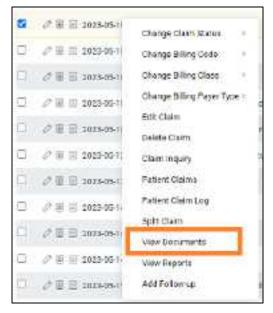

- 3. Optional. In the **Patient Documents** dialog, narrow the list of documents by entering properties in the search bar.
- 4. Double-click a document in the list.

1.	Documents Skew All Documents				NICSON T UP	oke - ROJOKO
	DOGUNERTTIPE	DOOLHEN'T NOTES	082472087	CREWITE DATE &	MPORTINILGAD BATE	bso.movrieves
	WI	1				Tan a

5. In the second **Patient Documents** dialog, in the right pane, select the link of the document to download.




Result: Exa Billing downloads the documents to the Windows Downloads folder.

Upload a new patient document

You can upload and attach a new document to the patient chart. You can upload files of the following formats: TXT, PDF, and JPG.

Procedure

- 1. Go to <u>BILLING</u> > <u>Claims</u>
- 2. In the list of claims, right-click a claim and then select **View Documents**.

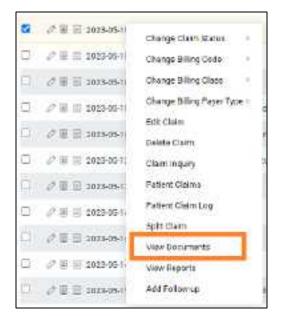

3. In the Patient Documents dialog, select UPLOAD.



4. In the second **Patient Document** dialog, enter the required metadata and then select **SELECT FILES**.

tient Document	S	
Booument Type *	Other 🗸	
Created Date/Time *	2023-05-24	ਿ
	9 35 AN EDT S	
Requested By	Provider 🛩	
Notes	Test	
	lavimum file size: 20 MB	

5. Browse for and select the file to upload, and then select **Open**.

Note: Exa Billing renames the file with a UID, and displays its icon in the right pane.

6. Select SAVE.

Create a billing report

You can create various types of reports based on configurable criteria, and then export them in formats including PDF, XLS, CSV, and XML.

Procedure

1. Go to <u>BILLING</u> > <u>Report</u>, and then select a report type in the list.

REPORT	2	Carave Blocker Charms Promerits (CE Report + Satur + Sag + Leg 0)				
MUNDONTS	Chaire Date - 2008-04-2923-05-13	Paciny Mercanolitics Mex.14 +	Siling Provide This reaction in a second second second			
ARCEAR BORNARY	Diman Ba Houseven Clair I M					
AND AN OF ALL	Notes of Great D	itaati Biaw Giaw				


- 2. Enter criteria in the options provided (such as date, facility, and billing provider), and then do one of the following.
 - To view the report in the current window, select **VIEW**.
 - To view the report in a new browser tab, select
 - To export the report to a file, select PDF, EXCEL, CSV, or XML.

View approved reports associated with claims

You can view all available reports associated with a claim.

Procedure

- 1. Go to <u>BILLING</u> > <u>Claims</u>.
- 2. In the list of claims, right-click a claim and select View Reports.

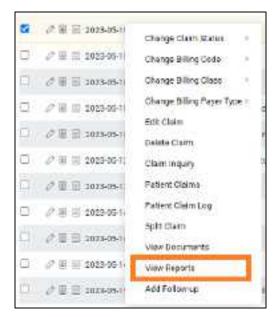

3. In the Approved Reports dialog, double-click a report to open it.

See also:

View documents and reports in the multi-panel

View documents and reports in billing

When opening claims or studies in Billing you can open documents and reports. Depending on configuration, a standard patient documents screen appears, or the multi-panel, which is an operating panel for working with dictation, transcription, documents, notes, and other information.

Procedure

1. Go to <u>BILLING</u> > <u>Claims</u>, open a claim or study, and select **DOCUMENTS AND REPORTS**.



2. If the multi-panel appears, in the left pane, select a document or report to view.



If linked prior reports are included, the main report appears at the top of the left pane with its linked reports indented below.

3. If the Patient Documents screen appears, select the PATIENT DOC. or REPORTS sub-tab.

Create and deliver a patient statement

You can generate a patient statement for one or more patients. If patients are in facilities such as nursing homes or hospice, and you send bills to their guarantor or a third party payer, you can change the mailing address when printing the patient statement.

Procedure

1. Go to <u>BILLING</u> > <u>Report</u>, and then select **Patient Statement** in the list.

Statesters Date /	Siling Provide	Techter
2923-66-24	NORE BOLDTED -	HONE VOLDETED -
in Arhourt	Sutainent Made	Palenta
	BELEDT -	
Uno Address of Pay To Provider	C Legin claim maany	
arty.		1
ORE DESIGNATION OF THE PARTY OF T		
-		

2. Select criteria for the report.



Note: In the US, some information such as the address is excluded from the patient statement if the patient is a minor.

- 4. In the **Statement Mode** button shortcut menu, select patients to include in the statement:
 - To select individual patients, select Select, and then use the Patients dropdown list to select patients.
 - To select a range of patients, select **Range by Patient Last name** and then use the **Last Name From** and **Last Name To** boxes to define a range.


Note: If you select a range, you cannot mail the report from this screen.

- 5. In the Mail To button shortcut menu, select a recipient.
- 6. Click a format button to generate the report (**PDF**, **EXCEL**, **CSV**, or **XML**).

View the patient claim log

You can view a log of patient claim activity.

- 1. Go to <u>BILLING</u> > <u>Claims</u>.
- 2. In the list of claims, right-click a claim and select **Patient Claim Log**.

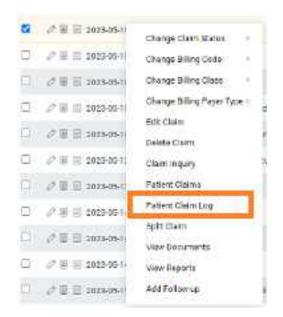

Send claims to collections

You can prepare a report to review all claims that are in Collections Review status. You can generate this report for the collection agency, change the claim statuses to Claim in Collections, and adjust off the claim balances.

Procedure

1. Go to <u>BILLING</u> > <u>Report</u>, and then select **Collections**.

					Comus Dilligis Calves	Parmin	EDD Separt+ Setur- Log+ Log D
Dairy Date +		E-sta	rane -		Tilling Provider		
2023-29-24 - 1973-02	24 🕅		NON	astastis -	NUMBER OF COLUMN	100.7	CSentabaris to collectore
NUM 3	Biter	TEXNET.	12 897	1200			

- 2. Select filtering criteria in the Claim Date, Facilities, and Billing Provider lists.
- 3. Optional. To send the report to the collections agency, select the **Send claims to collections** checkbox.
- 4. To generate the report, select a report format button (such as **PDF** or **EXCEL**).

Open an insurance provider's website from a claim

When browsing claims, you can open the website of a claim's insurance provider directly from a claim.



Prerequisite: Configure the URL of the website for the insurance provider (see <u>Add an insurance</u> <u>provider</u>).

- 1. Go to <u>BILLING</u> > <u>Claims</u>.
- 2. On the shortcut menu of a claim, select **Change Billing Payer Type**, and then select the redirect symbol.

-	N II II WYSOSHI	These, surfaces,	C MERCINE.	Caller
	⊘ 🗐 🗐 2023-05-10	Teet Adam	Patient	Pater
Q	⊘ 🗏 🗐 2023-05-10	Change Claim Status	Patient	Patier
	⊘ 🗏 🗐 2023-05-10	Change Billing Class	Patient	Pater
	夕目目 2023-05-10	Change Billing Poyer Type	Test, Aden(Patient)	ectri
	⊘ 🗏 🗐 2023-05-12	EditOlaim	C ARP SUPPLEMENT/SEC	INDARY JOHN
Π.	2日日 2023-05-12	Adjust to Paid in Full	Edmary misusone	Electo

Work with reports

Exa PACS/RIS provides tools to easily create a wide variety of reports to help you manage your healthcare facilities and resources.

Create, view, and deliver reports

This section contains the following topics.

<u>Create a report</u> <u>View your created reports</u> <u>Send reports to providers</u>

Create a report

You can create various types of report based on configurable criteria, and then export them in formats including PDF, XLS, CSV, and XML. Note that *reports* here does not include approved reports from radiologists.

- 1. On the burger 🔲 menu, select **REPORTS**.
- 2. On the top navigation bar, select Report, and then select a report type in the list.

eport Filter							i d
Filter Floids		VEW	1005	FROM	ear-	414	death famil
From Date: To Date 1 NYYYAMMOD YYYYYAMMOD Pacients	20 10					+	Report Types O Early Schedules O Patient Form O Patient Doly Scho
Scarth Patent Botannig Physisten Lad Name Finit Nerve Anam Aarte			Modelty D BB C C1 Modelty Fearm			Pathent D Pathent D	
Reaching Physician Last Norre Think Th Ordering Facility Sweeth University Pa	First Nores		Shady Stoke Invited to Believe to Defect to D	n Ay station alum alum		+	By Status By Transportation Geospleted Schedu D Incomplete Schedu Walkin Studies by CPT
	Fitter Floids Fitter Floids Fitter Floids Fitter Date/To Date T VYYY MANDO VYYY MANDO Patterts Beformes Physioner Last Name In Reacing Physioner Last Name Fitter To Date(Space)	Fitter Fields From Date (To Date + VYYY MM-OD VYYY MM-OD Patteries atteries Pat	First Florids VEW From Date (To Date + Image: Control Date + VYYYY MM-EDD Image: Control Date + Patents Image: Control Date + Security Patents Image: Control Date + Defense Physicists Image: Control Date + Image: Control Date + Image: Contr	Fitter Floids VEW ME From Bate/To Bate + Facility + VYYY MV600 ID VYYY MV600 ID VYYY MV600 ID Patentis Id Hased Patentis Id Hased Patentis Id Hased Batentis Id Hased Batent	Filter Fields Value Data Figure Data To Data * Fiscility * VYYY MVⅅ III VYYY MVⅅ III VYYY MVⅅ III Patanta IIII Scarch Patent Modelity Scarch Patent IIII Scarch Patent IIIII Scarch Patent IIIIIIIIIIIIIIIIIIIIIIIIIIIIIIIIIIII	Filter Floids Value Docal Date Figure Date 1 Figure Date Figure Date Figure Date VYYY MWODD III IIII Figure Date Figure Date VYYY MWODD IIII IIII Figure Date Figure Date VYYY MWODD IIII IIII Figure Date Figure Date VYYY MWODD IIIII IIIII Figure Date Figure Date VYYY MWODD IIIIIIIIIIIIIIIIIIIIIIIIIIIIIIIIIIII	From Date To Date * From Date To Date * VYYY MM-DDD VYYY MM-DDD Patential Patential Patential Scarch Patenti Scarch Patenti Scarch Patenti Scarch Patenti Scarch Patenti Scarch Patenti Scarch Patenti Scarch Patenti Scarch Patenti Scarch Patenti Scarch Patenti Scarch Patenti Scarch Patenti Scarch Patenti Scarch Patenti Scarch Patenti Scarch Patenti Scarch Patenti Scarch Patenti Scarch P

- 3. Below the **Filter Fields** label, select criteria to define the content of the report.
- 4. Select **VIEW**, or select an output format (**PDF**, **EXCEL**, **CSV**, or **XML**).

Result: The report is created, and automatically saved to My Reports.

See also:

<u>Create a billing report</u> <u>Create a peer review report</u> <u>Create and deliver a patient statement</u>

View your created reports

Reports that you create are automatically placed in the My Reports area where you can open and download them.

Procedure

- 1. On the burger menu, select the arrow next to **REPORTS**, and then select **MY REPORTS**.
- 2. Select a report in the list, and then:
 - Select 📥 to download

Send reports to providers

You can configure Exa PACS/RIS to automatically send reports and images through email to specified provider locations/contacts and their provider groups.



Prerequisite: Add a referring physician or ordering physician to any studies whose reports you want to send (see <u>Edit study information</u>).



Caution: A password or patient information is configured or used on this page. For recommended security, <u>turn off Chrome autofill</u>.

- 1. Go to <u>SETUP</u> > <u>OFFICE</u> > **Providers & Resources** > **RESOURCE**.
- 2. In the list of resources, double-click a resource of type **Provider-Radiology** or **Referring Provider**.
- 3. On the LOCATIONS/CONTACTS sub-tab, double-click a location/contact in the list.
- 4. In the **Contact Information** area, in the **Email** box, type the email address of the provider contact.

CONTACT INFO	MARCETING, REP.	
Contact Informatio	n	
Gade *	8P70 Directive	Offic
Provider Organization	Solari provide regarization *	Ciffi
Group	Select providur group	Mot
liema	emailgemail.com	P20
Contact Name		Eho

5. In the **Reports to Me** area, under **Contact Information**, enter the following settings. To send reports to other members of the provider location/contact's provider group, under **Reports to Group**, enter the same settings.

Reports to Me					
Contact Information	S None		Reports to Group		
	Cost Report Link	Cimal Attachment		Email Report Link	🗋 Emuil Attachment
	D Postal Uni	D ***		Pessel Mal	E Ea
	D Office Fax	□ +it7		Giffice Fait	🗆 нья
Report Password			Reports to Patient Po	anal	
			Delay	(in days)	

Setting	Description	
Email Report Link	Select to send a link to the report in an email.	
Postal Mail	Select to send a hard copy of the report to the mailing address enter in the Contact Information area.	
	Auto-Print : Select a printer in the dropdown list to automatically print reports.	
Fax	Select to send a fax of the report to the fax number entered in the Contact Information area.	
Email Attachment	Select to send the report as an attachment to an email.	
Office Fax	Select to send a fax of the report to the office fax number entered in the Contact Information area.	

6. Optional: Under **Report Password**, type a password to require to open reports.

7. Select SAVE & CLOSE.

Note: You can view your sent reports at: Burger menu > SETUP > General > REPORT QUEUE. Records are processed and sent to the specified user or group based on the email service configuration.

Work with Exa Chat

This chapter contains the following sections. <u>What is Exa Chat?</u> <u>Use Exa Chat</u> <u>Manage Exa Chat</u> <u>Use Exa Links</u>

What is Exa Chat?


Exa Chat is a program within Exa PACS/RIS that provides secure *chat* (exchange of text messages) with other members of your institution. You can hold private chats (instant messaging), or group chats with individuals or departments. You must purchase a license to use Exa Chat.

This section contains the following topics.

What is the contact list? About group (multi-user) chat rooms

What is the contact list?

The contact list is contained in the main Exa Chat window that appears in the lower left of the Exa PACS/RIS browser window, and lists chat rooms in which you participated. You can select any room to re-open it. When a user archives a room, it is removed from the contact list.

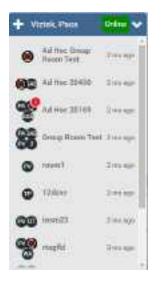

About group (multi-user) chat rooms

Group and *ad hoc* are both types of multiuser chat room. Group chat rooms are managed, and ad hoc chat rooms are not. If you start a private chat and then add a third member, the chat room becomes an ad hoc chat room. The following table summarizes other differences.

Property	Group	Ad hoc
Who can create the room?	The "manager" (user with Group Chats Management rights).	Any user with Personal Chat Allowed rights.
Who can/remove add members?	The manager. No special rights are required to be added.	Any member can add anyone else with Personal Chat Allowed rights.
Who can delete/archive the room?	The manager.	Only the last remaining member.

Use Exa Chat

This section contains the following topics.

Start a new private chat Join a group chat room Start a new multi-user chat Minimize or restore chat rooms Open a previously used chat room Add a member to a chat room Change your status Add a chat room to your favorites Use an avatar for chat Copy from one message to another

Start a new private chat

To start chatting with someone for the first time, you can create a private chat room. After creating a private chat room, it appears in the contact list for future use.

Procedure

- 1. On the contact list, select the add \blacksquare button.
- 2. In the **Add room** dialog, begin typing a contact name, and then select the name from the list. The private chat room window appears to the left of the contacts list.
- 3. In the box at the bottom of the chat room window, type a message and select Enter.

Join a group chat room

You can join a group chat room that was configured in Exa PACS/RIS.



Prerequisite: Add a group chat room

- 1. On the contact list, select the add \blacksquare button.
- 2. In the **Add private room** dialog, start typing the name of the group room, and then select the name from the list. The group chat room window appears to the left of the contacts list.
- 3. In the box at the bottom of the chat room window, type a message and select Enter.

Start a new multi-user chat

To start a group chat, you create a multi-user chat room. After creating the room, it appears in the contact list for future use.

Procedure

- 1. On the contact list, select the add \blacksquare button.
- 2. In the Add Room dialog, select CREATE ROOM.
- In the Create new room dialog, type a title for the room, and then select CREATE NEW ROOM.
 The multiuser chat room window appears to the left of the contacts list.
- 4. In the box at the bottom of the chat room window, type a message and select Enter.
- 5. The message is sent to all members in the chat room.

Minimize or restore chat rooms

- To minimize a chat room, select its minimize W button.
- To minimize Exa Chat (all chat rooms and the contact list), select the minimize button on the contact list.
- To restore a single chat room or all chat rooms, select the maximize button at the bottom of the browser window.

Open a previously used chat room

Each entry in the contact list corresponds to a chat room in which you participated. Select any room in the list to re-open it and re-initiate a chat.

Add a member to a chat room

You can add a new member to an existing chat room. You can also add a group chat room as a "member" of another chat room.

- 1. On the header of a chat room, select the settings 🙆 button.
- 2. Select Add users or Groups.
- 3. In the **Add user** dialog, begin typing a contact name, or a group chat room name, and then select the name from the list.

4. Select OK.

Change your status

Your status indicates your availability to chat. You can set a status of Busy (available for emergencies) and DND (do not disturb, do not play sounds). Unless you assign a status manually, Exa Chat automatically assigns a status of Online (available) or Offline (unavailable).

Procedure

1. On the header of the contact list, select the status button.



2. Select a status.

Add a chat room to your favorites

When you add a chat room to your favorites, it moves to the top of the contact list.

Procedure

- 1. Open the chat room that you want to add.
- 2. In the chat room window header, select the Add to favorites button.

Use an avatar for chat

An avatar is an image such as a photo that represents you in a chat room. To use an avatar in chat, follow the procedure in <u>Add an avatar</u> and the avatar will appear in your chat rooms automatically.

Copy from one message to another

You can copy all or part of a message into another message.

- Select all or some text in one message, select Ctrl+C, and then paste into another message.
- Drag a message from one chat window into the message box of another chat window.

Manage Exa Chat

This section contains the following topics.

Add user roles for Exa Chat Rename a chat window Archive a chat

Add user roles for Exa Chat

Administrators must add user roles for Exa Chat.

Procedure

- 1. On the burger 🔲 menu, select **SETUP**.
- 1. On the OFFICE menu, select User Management.
- 2. On the **USER ROLES** tab, select **ADD**.
- 3. Type a name for the role (such as Chat), and a description, and then select SAVE.
- 4. On the **USER ROLES** tab, double-click the user role you just added.
- 5. Select one or more of the following checkboxes.

Personal Chat Allowed - Create and join private and ad hoc chat rooms.

Group Chats Management - Create and join private and multiuser chat rooms.

Chat – Apply both of the previous rights.

6. Select SAVE.

See also:

Manage users

Rename a chat window

You can change the name of an existing chat room.

- 1. Select Settings in the chat room that you want to rename.
- 2. Type the new title in the box, and then select **SAVE TITLE**.

Archive a chat

To save disk storage space, you can archive your chats. Archiving saves messages older than a preconfigured date to a CSV file, and then removes them from the system. Archiving only works if the system holds more than a preconfigured number of messages.

- 1. Select Settings in the chat room that you want to archive.
- 2. Select Archive the chat.

Use Exa Links

This section contains the following topics.

What are Exa Links? Find and send an Exa Link Send a document by using an Exa Link

What are Exa Links?

Exa Links are links to Exa PACS/RIS controls that you send to chat members. Recipients can select the Exa Links to open patient charts, studies in the Edit Study screen or viewer, approved reports and addenda, notes, and other assets.

Find and send an Exa Link

If you hover over a control in Exa PACS/RIS and its tooltip ends with "... - Share to Chat," it is an Exa Link.



The following table lists the controls that are available as Exa Links at the time of this writing. More will be added in future versions.

Page	Controls			
Worklist	Study toolbar: Viewer, Edit, Approved Reports	+ 8 ■ ● ● ⊘≣ 3 ₫	☆ ⑧	

To send an Exa Link, drag it onto the message box at the bottom of a chat room window, and then select **Enter**.

Send a document by using an Exa Link

In addition to the Exa Links on the worklist, you can send Exa Links to documents from the Patient Documents screen.

- 1. On a worklist shortcut menu, select **Attached Files**.
- 2. In the Patient Document dialog, double-click a document to open it.
- 3. Drag the **Share to Chat** button into a chat message.

Work with tools

Exa PACS/RIS includes a suite of tools that provide a variety of functionality. Some tools must be installed locally on the client computer (see <u>Install and configure tools</u>).

Use import tools

With the import tools provided in the program, you can <u>Import studies from CD</u> and <u>Import studies to a</u> <u>specific AE (API mode)</u>.

Import studies from CD

You can perform a "CD import."

Caution: Ensure that the data to be imported, and the data after import, are correct.

Prerequisite: Install Opal tools.

Procedure

- 1. On the burger Honor select the arrow next to **TOOLS**, and then select **CD IMPORT**.
- If the "Open Opal Import?" message appears, select the checkbox, and then select Open Opal Import.
- 3. In the **OpalImport** dialog, select **Select Files/Folders or DICOMDIR to Import**, and then browse for and select files to import.
- 4. Enter customization settings and then select **Import**.

Import studies to a specific AE (API mode)

You can import studies to a specific AE.

Procedure

1. On the worklist, on the **PACS Actions** menu, select **Import Images**.



Note: If the "Open Opal Import?" message appears, select the checkbox, and then select **Open Opal Import**.

2. In the **OpalImport** dialog, turn on API mode by typing lq2w3e4r5t.



Note: You only need to perform this step once. API mode will remain on permanently.

- 3. Select the **Exa API transfer** option.
- 4. Select **Select Files/Folders or DICOMDIR to Import**, and then browse for and select files to import.
- 5. Enter customization settings and then select **Import**.

Work with Patient Portal

Patient Portal is a website that patients can use to view or modify information about their past and future exams, demographics, insurance, guarantors, and representatives, and to schedule appointments. For instructions on how patients can use Patient Portal, see the **Patient Portal user's manual**.

For staff, see About creating a patient portal account.

About creating a patient portal account

Before the patient can use Patient Portal, clinical staff must work with the patient to create an account. You can choose one of two workflows to create an account:

1. In-Person Registration initiated by staff: Create a patient portal account

2. Online Patient Registration initiated by the patient: Patient self-registration

Create a patient portal account

The recommended workflow to create a patient portal account begins when you are scheduling an appointment.

1. Open the patient's chart, and at the top of the screen, select **PORTAL ACCOUNT**.

■ Patient © Test, Testghvlx0ol4	X DOB:1981-10	0-11 MRN: tesu27	12	
Patient Information [more]	PORTAL ACCOUNT	SCAN LICENSE	NOURY	CHECKEN CODE

2. In the **Create Portal Account** dialog, confirm the patient's email address, and then select **CREATE.**

	A CONTRACTOR OF
1446GH V0(00L/0)	
Test	
test@123.com	
1	
10-10-10-10-10-10-10-10-10-10-10-10-10-1	
	Test

Result: An email is sent to the patient with a link to Patient Portal. The patient does the following:

- Selects the link (or pastes it into the browser) to open the create password page of the Patient Portal.
- Types a password and selects the button.
- On the Login page, enters their password and selects LOGIN to open the portal.
- 3. In the Create Portal Account dialog, select SAVE.

Patient self-registration

If the patient already has a chart in Exa PACS/RIS, they can initiate creating a portal account as follows.

1. On the patient portal sign-in screen, the patient selects **NEW USER**.

	New User Registration	
	First Hanne	Last News
	Su	Date of Date
1 Login	Select Largeogn	Deve Hell
	Seec	* (200,450,700)
2	End	Address Une 1 8
	##AT\$RD ## CIP\$	
B	Address Line 2	av.
Z Paret to the HERA Privacy Nation	State	ZE Com
C Remember Me	Select	
LOGIN	Password	Gatter Passed
	Same Reset	

- In the New User Registration dialog, the patient fills out the form, and then selects Save.
 Result: The demographics are sent to Exa PACS/RIS for linking by staff (see below), and the patient receives a verification email.
- 3. The patient opens the verification email and selects the link in the to verify their email address.

Staff: Link the new patient's information to the patient's chart

1. On the burger enu, select **PORTAL REG USERS**.

			red Users			
		STATUS	FIRST NAME	LASTNAME	DATE OF BIRTH	
		Varifiad 👻	l.	1		
A.	ź.	Venfied	1PNUPatient78330	1PNUPatient	1988-01-01	portal.exauser1@gmail.com

- 2. On the **Portal Registered Users** screen, open the user.
- 3. In the **Portal Registered Information** screen, in the **Patient Match** area, search for a matching patient. If no matching patients appear, select **CREATE NEW**, edit the patient's information as needed, and then select **CREATE PATIENT**.

Dipris Will 👻 🙏		1	Stow Wastive	Break the Glass		
1 PNUFicti ent	7 van H	10 TH 10	MRN	221	Frane	DQE
Address		7.9				
0 1 686	ATENEW	CLEAR SEARS	94			SAVE AND LIP
PATIENT SEARCH	IEBVU/TE				RESISTEATION	REGUMATION
Ipropalient, Tpr	vapatient78330 , (876)134-567			8	- Fest Name Lest Name	(PNUPatient78330 1PNUPatient

- 4. Open the matching patient, and check the **Registration Information** pane on the right; if any information appears in yellow, decide whether you want to update the patient chart with the new information.
- 5. Do one of the following:
 - To update the patient chart and link the account, select **SAVE AND LINK**, and then select **UPDATE**.
 - To continue linking the account without updating the patient chart, select **SAVE AND LINK**, and then select **CONTINUE WITHOUT UPDATING**.
 - To reject the registration request altogether, select **REJECT**.

Work with Physician Portal

For information on how physicians can use Physician Portal, see the **Physician Portal user's manual**. In Exa PACS/RIS, staff enable physicians to use Physician Portal by doing the following.

- 1. <u>Add a resource</u> of type **Referring Provider**.
- 2. On the LOGIN DETAILS sub-tab of the resource, select a user who:
 - Is a **referring provider**, and;
 - Belongs to the **Portal** user's group.
- 3. Select LINK USER.

See also: About CDSM and AUC

About CDSM and AUC

The Protecting Access to Medicare Act (PAMA) of 2014 requires practitioners of advanced diagnostic imaging services such as CT, PET, NM, and MRI to consult a qualified *clinical decision support mechanism* (CDSM) before ordering exams for Medicare patients. CDSMs are online apps that use *appropriate use criteria* (AUC) to determine whether exams are clinically relevant (or, "appropriate"). By purchasing a license, Exa PACS/RIS can offer access to the LogicNets CDSM app through Physician Portal.

Appendix

The appendix includes the following sections.

<u>User reference</u> <u>Install and configure tools</u>

Reference

User reference

This section contains the following topics.

Search syntax Keyboard shortcuts for the claim entry screen Keyboard shortcuts and commands for Exa Trans Merge fields User rights Description of reports Settings in the claim entry screen Measurement accuracy limits Viewer tools and keyboard shortcuts Details on Linked Reporting Mammography views

Search syntax

Various lists in Exa PACS/RIS have a search bar with column headers that accept search criteria. The following general rules apply when entering search criteria.

• In most numeric boxes, you can search for any part of the number. For example:

In the account number box, type: 20

- Exa PACS/RIS finds: 203, 1203, and 01203.
- In a name box (such as patient name or payer name), you can type the full name in the format:

lastname, firstname

…but not…

firstname lastname.

• In a name box, you can type the first or last name only. For example:

Type: Anderson

Exa PACS/RIS finds: Cooper, Anderson ...and...

Anderson, John.

• In a name box you can type all or part of the first or last name, but using as many letters as possible improves accuracy. For example:

Type: Oswa	
Exa PACS/RIS finds:	Oswald, Trevor
···and···	
Davis, Oswald	
However, if you type:	Os
Exa PACS/RIS finds or	nly: Oswald, Trevor

Keyboard shortcuts for the claim entry screen

When creating or editing claims, the following keyboard shortcuts are available.

Target	Shortcut		
Charges sub-tab	Alt+C		
Claims sub-tab	Alt+Shift+C		
Insurance sub-tab	Alt+I		
Additional Info. sub-tab	Alt+A		
Billing Summary sub-tab	Alt+B		
Payments sub-tab	Alt+P		
Cursor moves to the payer name option	Alt+(+)		
Previous claim	Alt+Shift+P		
Next claim	Alt+Shift+N		
Notes screen	Alt+N		
Documents and Reports screen	Alt+D		
Validate claim	Alt+V		
Save All	Alt+S		
Close without saving	Esc		

Keyboard shortcuts and commands for Exa Trans

Command	Command Text	Shortcut
Remove last dictated piece	undo that	-
	delete that	
	scratch that	
Next variable placeholder	next variable	Alt+]
Previous variable placeholder	previous variable	Alt+[
Capitalize previous word	capitalize that	-
	cap that	
Remove word left of the cursor	remove left	Ctrl+Alt+Back
Start recording		Ctrl+F1
Toggle recording		Ctrl+F3
Stop recording	stop recording	Ctrl+F2

When using Exa Trans, the following keyboard shortcuts are available.

Merge fields

The following table describes the merge fields available in various parts of the program. Some merge fields may not be available in your installation of Exa PACS/RIS depending on added modules, country, and other factors.

1. **SO** = Study/Order; **IN** = Insurance; **PA** = Patient; **PR** = Provider; **ET** = Email Template; **NT** = Notification Templates, **RT** = Report Templates; **SF** = Study Forms; **CF** = Custom Forms; **TT** = Transcription Templates

Merge Field	Formal Name	Scope1
FamilyHealthHistory	Family Health History	SO, NT, SF, CF
FollowUps	Follow Ups	SO, NT, SF, CF
GST (Canada)	General Sales Tax	SF, CF, ET
SalesTax	Sales Tax	SF, CF, ET
Immunizations	Immunizations	SO, NT, SF, CF
LabResults	Lab Results	SO, NT, SF, CF
Medications	Medications	SO, NT, SF, CF
Referrals	Referrals	SO, NT, SF, CF
PreLabResults	Pre Lab Results	SO, NT, SF, CF
Problems	Problems	SO, NT, SF, CF
VitalSigns	Vital Signs	SO, NT, SF, CF
Age	Age	PA, ET, NT, SF, CF

Merge Field	Formal Name	Scope1
AgeatDOS	Age at DOS	PA, ET, NT, SF, CF
Address1	Address 1	PA, ET, NT, SF, CF
Address2	Address 2	PA, ET, NT, SF, CF
Ciy	City	PA, ET, NT, SF, CF
State	State	PA, ET, NT, SF, CF
ZipCode	Zip Code	PA, ET, NT, SF, CF
DOB	DOB	PA, ET, NT, SF, CF
FirstName	First Name	PA, ET, NT, SF, CF
Fullname	Full Name	PA, ET, NT, SF, CF
Gender	Gender	PA, ET, NT, SF, CF
Race	Race	PA, ET, NT, SF, CF
Ethnicity	Ethnicity	PA, ET, NT, SF, CF
HomePhone	Home Phone	PA, ET, NT, SF, CF
MobilePhone	Mobile Phone	PA, ET, NT, SF, CF
WorkPhone	Work Phone	PA, ET, NT, SF, CF
Lastname	Last Name	PA, ET, NT, SF, CF
Middlename	Middle Name	PA, ET, NT, SF, CF
mrn	MRN	PA, ET, NT, SF, CF
uli	ULI	PA, ET, NT, SF, CF
ssn	SSN	PA, ET, NT, SF, CF
Suffix	Suffix	PA, ET, NT, SF, CF
Weight	Weight	PA, ET, NT, SF, CF
Height	Heigh	PA, ET, NT, SF, CF
Employer	Employer	PA, ET, NT, SF, CF
PreferredLanguage	Preferred Language	PA, ET, NT, SF, CF
Username		PA, ET, NT,
Password		PA, ET, NT,
ApprovingPhysicianAddress1	Approving Physician Address 1	PR, ET, NT, SF, CF
ApprovingPhysicianAddress2	Approving Physician Address 2	PR, ET, NT, SF, CF
ApprovingPhysicianCity	Approving Physician City	PR, ET, NT, SF, CF
ApprovingPhysicianFirstName	Approving Physician First Name	PR, ET, NT, SF, CF
ApprovingPhysicianFullName	Approving Physician Full Name	PR, ET, NT, SF, CF
ApprovingPhysicianLastName	Approving Physician Last Name	PR, ET, NT, SF, CF
ApprovingPhysicianMiddleName	Approving Physician Middle Name	PR, ET, NT, SF, CF
ApprovingPhysicianState	Approving Physician State	PR, ET, NT, SF, CF

Merge Field	Formal Name	Scope1
ApprovingPhysicianSuffix	Approving Physician Suffix	PR, ET, NT, SF, CF
CCProv1Addr1	CC Provider 1 Address 1	PR, ET, NT, SF, CF
CCProv1Addr2	CC Provider 1 Address 2	PR, ET, NT, SF, CF
CCProv1City	CC Provider 1 City	PR, ET, NT, SF, CF
CCProv1Name	CC Provider 1 Name	PR, ET, NT, SF, CF
CCProv1State	CC Provider 1 State	PR, ET, NT, SF, CF
CCProv1Zip	CC Provider 1 Zip	PR, ET, NT, SF, CF
CCProv1Alerts	CC Provider 1 Alerts	PR, ET, NT, SF, CF
CCProv2Addr1	CC Provider 2 Address 1	PR, ET, NT, SF, CF
CCProv2Addr2	CC Provider 2 Address 2	PR, ET, NT, SF, CF
CCProv2City	CC Provider 2 City	PR, ET, NT, SF, CF
CCProv2Name	CC Provider 2 Name	PR, ET, NT, SF, CF
CCProv2State	CC Provider 2 State	PR, ET, NT, SF, CF
CCProv2Zip	CC Provider 2 Zip	PR, ET, NT, SF, CF
CCProv2Alerts	CC Provider 2 Alerts	PR, ET, NT, SF, CF
CCProv3Addr1	CC Provider 3 Address 1	PR, ET, NT, SF, CF
CCProv3Addr2	CC Provider 3 Address 2	PR, ET, NT, SF, CF
CCProv3City	CC Provider 3 City	PR, ET, NT, SF, CF
CCProv3Name	CC Provider 3 Name	PR, ET, NT, SF, CF
CCProv3State	CC Provider 3 State	PR, ET, NT, SF, CF
CCProv3Zip	CC Provider 3 Zip	PR, ET, NT, SF, CF
CCProv3Alerts	CC Provider 3 Alerts	PR, ET, NT, SF, CF
CCProv4Addr1	CC Provider 4 Address 1	PR, ET, NT, SF, CF
CCProv4Addr2	CC Provider 4 Address 2	PR, ET, NT, SF, CF
CCProv4City	CC Provider 4 City	PR, ET, NT, SF, CF
CCProv4Name	CC Provider 4 Name	PR, ET, NT, SF, CF
CCProv4State	CC Provider 4 State	PR, ET, NT, SF, CF
CCProv4Zip	CC Provider 4 Zip	PR, ET, NT, SF, CF
CCProv4Alerts	CC Provider 4 Alerts	PR, ET, NT, SF, CF
CCProv5Addr1	CC Provider 5 Address 1	PR, ET, NT, SF, CF
CCProv5Addr2	CC Provider 5 Address 2	PR, ET, NT, SF, CF
CCProv5City	CC Provider 5 City	PR, ET, NT, SF, CF
CCProv5Name	CC Provider 5 Name	PR, ET, NT, SF, CF
CCProv5State	CC Provider 5 State	PR, ET, NT, SF, CF
CCProv5Zip	CC Provider 5 Zip	PR, ET, NT, SF, CF

Merge Field	Formal Name	Scope1
CCProv5Alerts	CC Provider 5 Alerts	PR, ET, NT, SF, CF
ProviderSignature	Provider Signature	PR, ET, NT, SF, CF
ReadingPhysicianAddress1	Reading Physician Address 1	PR, ET, NT, SF, CF
ReadingPhysicianAddress2	Reading Physician Address 2	PR, ET, NT, SF, CF
ReadingPhysicianCity	Reading Physician City	PR, ET, NT, SF, CF
ReadingPhysicianFirstName	Reading Physician First Name	PR, ET, NT, SF, CF
ReadingPhysicianFullName	Reading Physician Full Nae	PR, ET, NT, SF, CF
ReadingPhysicianLastName	Reading Physician Last Name	PR, ET, NT, SF, CF
ReadingPhysicianMiddleName	Reading Physician Middle Name	PR, ET, NT, SF, CF
ReadingPhysicianState	Reading Physician State	PR, ET, NT, SF, CF
ReadingPhysicianSuffix	Reading Physician Suffix	PR, ET, NT, SF, CF
Ref_PhyAddress1	Referring Physician Address 1	PR, ET, NT, SF, CF
Ref_PhyAddress2	Referring Physician Address 2	PR, ET, NT, SF, CF
Ref_PhyCity	Referring Physician City	PR, ET, NT, SF, CF
Ref_PhyFax	Referring Physician Fax	PR, ET, NT, SF, CF
Ref_PhyFirstName	Referring Physician First Name	PR, ET, NT, SF, CF
Ref_PhyFullname	Referring Physician Full Name	PR, ET, NT, SF, CF
Ref_PhyHomePhone	Referring Physisican Home Phone	PR, ET, NT, SF, CF
Ref_PhyLastName	Referring Physician Last Name	PR, ET, NT, SF, CF
Ref_PhyMiddleInitial	Referring Physician Middle Initial	PR, ET, NT, SF, CF
Ref_PhyNameOrderFMLS	Referring Physician Name Order First Middle Last Suffix	PR, ET, NT, SF, CF
Ref_PhyState	Referring Physician State	PR, ET, NT, SF, CF
Ref_PhySuffix	Referring Physician Suffix	PR, ET, NT, SF, CF
Ref_PhyZip	Referring Physician zip	PR, ET, NT, SF, CF
Ref_PhyMobileNo	Referring pHysician Mobile Number	PR, ET, NT, SF, CF
Ref_PhyOfficeNo	Referring Physician Office Number	PR, ET, NT, SF, CF
Ref_PhyAlerts	Referring Physician Alerts	PR, ET, NT, SF, CF
Report Link	Report Link	PR, ET, NT, SF, CF
SignatureText	Signature Text	PR, ET, NT, SF, CF
PreApprovedSignatureTExt	PreApproved Signature Text	PR, ET, NT, SF, CF
preApprovedProviderSignature	PreApproved Provider Signature	PR, ET, NT, SF, CF
RadiologistFullName	Radiologist Full Name	PR, ET, NT, SF, CF

Merge Field	Formal Name	Scope1
TechnologistFullName	Technologist Full Name	PR, ET, NT, SF, CF
SignatureBlockText	Signature Block Text	PR, ET, NT, SF, CF
preApprovedSignatureBlockText	PreApproved Signature Block Text	PR, ET, NT, SF, CF
AttorneyFirstName	Attorney First Name	PR, ET, NT, SF, CF
AttorneyFullName	Attorney Full Name	PR, ET, NT, SF, CF
AttorneyLastName	Attorney Last Name	PR, ET, NT, SF, CF
AttorneyMiddleName	Attorney Middle Name	PR, ET, NT, SF, CF
AttorneySuffix	Attorney Suffix	PR, ET, NT, SF, CF
AttorneyAddress1	Attorney Address 1	PR, ET, NT, SF, CF
AttorneyAddress2	Attorney Address 2	PR, ET, NT, SF, CF
AttorneyCity	Attorney City	PR, ET, NT, SF, CF
AttorneyState	Attorney State	PR, ET, NT, SF, CF
AttorneyZip	Attorney Zip	PR, ET, NT, SF, CF
AttorneyPhoneNo	Attorney Phone Number	PR, ET, NT, SF, CF
AttorneyEmail	Attorney Email	PR, ET, NT, SF, CF
AttorneyFaxNo	Attorney Fax Number	PR, ET, NT, SF, CF
AppointmentTypeCode	Appointment Type Code3	SO, RT, ET, NT, SF, CF
AppointmentTypeDescription	Appointment Type Description	SO, RT, ET, NT, SF, CF
ExpectedArrivalTime	Expected Arrival Time	SO, RT, ET, NT, SF, CF
AccessionNo	Accession Number	SO, RT, ET, NT, SF, CF
ApprovedDate	Approved Date	SO, RT, ET, NT, SF, CF
PreApprovedDate	PreApproved Date	SO, RT, ET, NT, SF, CF
TranscribedDate	Transcribed Date	SO, RT, ET, NT, SF, CF
BodyPart	Body Part	SO, RT, ET, NT, SF, CF
Department	Department	SO, RT, ET, NT, SF, CF
DictationDate	Dictation Date	SO, RT, ET, NT, SF, CF
ICDCode	ICD Code	SO, RT, ET, NT, SF, CF
ICDDescripton	ICD Description	SO, RT, ET, NT, SF, CF
StudyICDCode	Study ICD Code	SO, RT, ET, NT, SF, CF
StudyICDDescription	Study ICD Description	SO, RT, ET, NT, SF, CF
Institution	Institution	SO, RT, ET, NT, SF, CF
Modality	Modality	SO, RT, ET, NT, SF, CF
ModalityRoom	Modality Room	SO, RT, ET, NT, SF, CF
OrderedBy	Ordered By	SO, ET, NT, SF, CF
OrderingFacility	Ordering Facility	SO, ET, NT, SF, CF

Merge Field	Formal Name	Scope1
Orientation	Orientation	SO, ET, NT, SF, CF
PatientLocation	Patient Location	SO, ET, NT, SF, CF
PlaceOfService	Place of Service	SO, ET, NT, SF, CF
Priority	Priority	SO, ET, NT, SF, CF
ProcedureCode	Procedure Code	SO, ET, NT, SF, CF
ProcedureCodes	Procedure Codes	SO, ET, NT, SF, CF
ProcedureCodestabular	Procedure Codes Tabular	SO, ET, NT, SF, CF
ProcedureName	Procedure Name	SO, ET, NT, SF, CF
ProcedureNames	Procedure Names	SO, ET, NT, SF, CF
ProcedureNamesTabular	Procedure Names Tabular	SO, ET, NT, SF, CF
ReasonForStudy	Reason for Study	SO, ET, NT, SF, CF
RequestingDate	Requesting Date	SO, ET, NT, SF, CF
RoomNo	Room Number	SO, ET, NT, SF, CF
STAT	Stat description	SO, ET, NT, SF, CF
StudyDate	Study Date	SO, ET, NT, SF, CF
StudyDescription	Study Description	SO, ET, NT, SF, CF
StudyFlag	Study Flag	SO, ET, NT, SF, CF
StudyReceivedDate	Study Received Date	SO, ET, NT, SF, CF
TAT	ТАТ	SO, ET, NT, SF, CF
TranscribingUser	Trascribing User	SO, ET, NT, SF, CF
VehicleName	Vehicle Name	SO, ET, NT, SF, CF
studyUID	Study UID	SO, ET, NT, SF, CF
StudyFormsignature	Study Form Signature	SO, ET, NT, SF, CF
CustomFormSignature	Custom Form Signature	SO, ET, NT, SF, CF
ExamPrepInstructions	Exam Prep Instructions	SO, ET, NT, SF, CF
CPTCode	CPT Code	SO, ET, NT, SF, CF
CPTName	CPT Name	SO, ET, NT, SF, CF
visit_no	Visit Number	SO, ET, NT, SF, CF
CPTCodes	CPT Codes	SO, ET, NT, SF, CF
CPTNames	CPT Names	SO, ET, NT, SF, CF
CPTCodesTabular	CPT Codes Tabular	SO, ET, NT, SF, CF
CPTNamesTabular	CPT Names Tabular	SO, ET, NT, SF, CF
DateofInjury	Date of Injury	SO, ET, NT, SF, CF
Allergies	Allergies	ET, NT, SF, CF
CompanyName	Company Name	ET, NT, SF, CF

Merge Field	Formal Name	Scope1
CurrentDate	Current Date	ET, NT, SF, CF
CurrentTime	Current Time	ET, NT, SF, CF
DateAndTime	Date and Time	ET, NT, SF, CF
OfficeAddress1	Office Address 1	ET, NT, SF, CF
OfficeAddress2	Office Address 1	ET, NT, SF, CF
OfficeCity	Office City	ET, NT, SF, CF
FacilityLogo	Facility Logo	ET, NT, SF, CF
FacilityContactNo	Facility Contact Number	ET, NT, SF, CF
OfficeName	Office Name	ET, NT, SF, CF
OfficeState	Office State	ET, NT, SF, CF
Zip	Zip	ET, NT, SF, CF
FormalName	Formal Name	ET, NT, SF, CF
PracticeType	Practive Type	ET, NT, SF, CF
PrimaryInsurance	Primary Insurance	IN, ET, NT, SF, CF
PrimaryAddress	Primary Address	IN, ET, NT, SF, CF
PrimaryCity	Primary City	IN, ET, NT, SF, CF
PrimaryState	Primary State	IN, ET, NT, SF, CF
PrimaryZipCode	Primary Zip Code	IN, ET, NT, SF, CF
PrimaryPhone	Primary Phone	IN, ET, NT, SF, CF
PrimaryFax	Primary Fax	IN, ET, NT, SF, CF
PrimarySubscriber	Primary Subscriber	IN, ET, NT, SF, CF
PrimaryDOB	Primary DOB	IN, ET, NT, SF, CF
PrimaryRelationship	Primary relationship	IN, ET, NT, SF, CF
PrimaryPolicyNo	Primary Policy Number	IN, ET, NT, SF, CF
PrimaryGroupNo	Primary Group number	IN, ET, NT, SF, CF
HealthNumber	Health Number	IN, ET, NT, CF
VersionCode	Version Code	IN, ET, NT, CF
SecondaryInsurance	Secondary Insurance	IN, ET, NT, SF, CF
SecondaryAddress	Secondary Address	IN, ET, NT, SF, CF
SecondaryCity	Secondary City	IN, ET, NT, SF, CF
SecondaryState	Secondary State	IN, ET, NT, SF, CF
SecondaryZipCode	Secondary Zip Code	IN, ET, NT, SF, CF
SecondaryPhone	Secondary Phone	IN, ET, NT, SF, CF
SecondaryFax	Secondary Fax	IN, ET, NT, SF, CF
Secondarysubscriber	Secondary Subscriber	IN, ET, NT, SF, CF

Merge Field	Formal Name	Scope1
SecondaryDOB	Secondary DOB	IN, ET, NT, SF, CF
SecondaryRelatoinship	Secondary Relationship	IN, ET, NT, SF, CF
secondaryPolicyNo	Secondary Policy Number	IN, ET, NT, SF, CF
SecondaryGroupNo	Secondary Group Number	IN, ET, NT, SF, CF
FacilityNotes	Facility Notes	NT, SF, RT
FacilityFax	Facility Fax Number	RT, SF, CF, ET, NT
FacilityEmail	Facility Email Address	RT, SF, CF, ET, NT
Ord_PhyAddress1	Ordering Phys. Street Address1	RT, TT, SR, CF, ET, NT, CS
Ord_PhyAddress2	Ordering Phys. Street Address2	RT, TT, SR, CF, ET, NT, CS
Ord_PhyCity	Ordering Phys. City	RT, TT, SR, CF, ET, NT, CS
Ord_PhyFirstName	Ordering Phys. First Name	RT, TT, SR, CF, ET, NT, CS
Ord_PhyFullName	Ordering Phys. Full Name	RT, TT, SR, CF, ET, NT, CS
Ord_PhyLastName	Ordering Phys. Last Name	RT, TT, SR, CF, ET, NT, CS
Ord_PhyMiddleName	Ordering Phys. Middle Name	RT, TT, SR, CF, ET, NT, CS
Ord_PhyState	Ordering Phys. State	RT, TT, SR, CF, ET, NT, CS
Ord_PhySuffix	Ordering Phys. Suffix	RT, TT, SR, CF, ET, NT, CS
Ord_PhyZip	Ordering Phys. Postal Code	RT, TT, SR, CF, ET, NT, CS
Non-ProprietaryName	Non-Proprietary Name	RT, TT, SF, CF
AmountUsed	Amount Used	RT, TT, SF, CF
Route	Route	RT, TT, SF, CF
Supv_PhyAddress1	Supervising phys. Street Adrs. 1	RT, TT, SF, CF, ET, NT, CS
Supv_PhyAddress2	Supervising phys. Street Adrs. 2	RT, TT, SF, CF, ET, NT, CS
Supv_PhyCity	Supervising phys. City	RT, TT, SF, CF, ET, NT, CS
Supv_PhyFirstName	Supervising phys. First Name	RT, TT, SF, CF, ET, NT, CS
Supv_PhyFullName	Supervising phys. Full Name	RT, TT, SF, CF, ET, NT, CS
Supv_PhyLastName	Supervising phys. Last Name	RT, TT, SF, CF, ET, NT, CS
Supv_PhyMiddleName	Supervising phys. Middle Name	RT, TT, SF, CF, ET, NT, CS
Supv_PhyState	Supervising phys. State	RT, TT, SF, CF, ET, NT, CS
Supv_PhySuffix	Supervising phys. Suffix	RT, TT, SF, CF, ET, NT, CS
Supv_PhyZip	Supervising phys. ZIP	RT, TT, SF, CF, ET, NT, CS
Canada only:	Health Card Number	RT, SF, CF, ET, NT

User rights

Ê/

The following table describes the rights that administrator can assign to a user role. All of these rights may not be available depending on your product and configuration.

Category	Right	Description
Billing/ Setup	Billing Codes	Allows creating and editing billing codes
	Billing Class	Allows creating and editing billing classes
	Claim Status	Allows creating and editing claim statuses
	CAS Group Code	Allows creating and editing CAS group codes
	Provider ID Code Qualifier	Allows mapping provider ID code qualifiers to billing providers
	Payment Reason	Allows editing payment reasons such as co-pay, deductible, and payment plan
	CAS Reason Code	Allows creating and editing CAS reason codes
	Status Color Code	Allows adding and editing colors for payment, claim, and billed statuses
	Supporting Text Templates	Allows mapping of preconfigured supporting text to specific service codes.
	Insurance EDI Mapping	Allows insurance mapping, such as billing method and clearinghouse
	Adjustment Codes	Allows adding and editing adjustment codes
	Billing Provider	Allows adding and editing billing providers
	Billing Messages	Allows editing billing messages that print on statements
	Billing Validation	Allows selecting the data that are validated during claim validation
	Printer Templates	Allows editing some settings in printer templates such as page margins
	EDI Request Templates	Allows creating and editing EDI request templates.
	Clearing House	Allows entering clearinghouse information
	Autobilling	Allows creating auto-billing profiles for automatic generation of claims
	CollectionsProcess	Allows configuring the automatic claim collections process
	Delay Reasons	Allows adding, editing, and deleting reasons for delayed payment
Billing	Claims	Allows working with claims
	Edit Claim	Allows opening the edit claim screen
	Claim Inquiry	Allows access to patient claim information
	All Insurances	All insurances
	Coordination of Benefits	Allows coordinating benefits

Note: Shaded rows indicate that the right is not currently in use.

Category	Right	Description
	Explanation of Benefits	Allows access to billing ERA
	ERA Inbox	Allows opening the ERA inbox
	EOB Claims Process	Allows opening the EOB tab, and uploading and processing electronic payment files
	File Insurance	Allows managing billing claims
	Patient Claim	Allows viewing a patient's claim history within Billing, and makes available the Inquiry buttons on the Patient Information tab in the patient chart and Edit Study screen
	Patient Report	Allows opening patient reports and documents within Billing
	Payments	Allows viewing and processing payments
	Apply Payments	Allows applying payments to claims
	Refund	Allows processing refunds
	Back to Ready to Validate	Allows changing claim status to Ready to Validate
	Claim Validate	Allows validating claims
	Create/Split Claim	Allows splitting a claim with more than one charge
	Delete Payment	Allows deleting payment records
	Delete Claim	Allows deleting claims
	File Management	Allows opening the file management screen in Canadian versions
	Edit Claim Status	Allows manually changing claim statuses
	Invoice Activity Statement	Allows viewing and printing the activity statement
	Query Claim	Allows querying of claims.
	Adjust to Paid in Full	Adds a shortcut menu command to pay a claim in full.
	Census	Allows viewing the Census screen.
Billing/ Report		All rights in this category allow creating the corresponding report.
	Aged AR Summary	
	Aged AR Detail	
	Charges	
	Claim Activity	
	Claim Transaction	
	Collections	
	Credit Balance Encounters	
	Diagnosis Count	
	Modality Summary	
	Monthly Recap	
	Patient Statement	
	Payer Mix	

Category	Right	Description
	Payment	
	Claim Inquiry	
	Patients By Insurance	
	Payments by Insurance Company	
	Procedure Analysis by Insurance	
	Procedure Count	
	Reading Provider Fees	
	Referring Provider Count	
	Referring Provider Summary	
	Transaction Summary	
	Patient Activity Statement	
	Payments Realization Rate Analysis	
	Send claims to collections	Allows using automatic collections to change claim statuses to "Claim in Collection," and make other changes.
Billing/Log	Billing User Log	Allows viewing the billing user log
	Bulling Audit Log	Allows viewing the billing audit log
Chat	Personal Chat Allowed	Allows using Exa Chat to send and receive messages with individual users (private chat)
	Group Chats Management	Allows creating and managing group chat rooms
Dashboard	Dashboard	Allows opening the dashboard
Dictation	Approve	Allows approving a dictated study report
	Approve (Addendum)	Allows approving an addendum (Edit or Addition) on a dictated study report, and changing a study's status from Approved to Approved-Coded.
	Dictation	Allows creating a dictated addendum (Edit or Addition) on an approved study report
	Dictation (Addendum)	Allows creating a dictated report
	Dictation (Delete)	Allows deleting a dictated report
	Submit For Review	Allows submitting an addendum for review on a dictated report
	Submit For Review (Addendum)	Allows submitting a dictated report for review
	Transcription (Addendum)	Allows creating an addendum by using Web Trans
	Link Reports	Allows linking reports (and studies)
	Unlink Reports	Allows unlinking reports (and studies)
Dispatching Dashboard	Dispatching Dashboard	Allows users of Mobile RAD to open the dispatching dashboard
General	AE Scripts	Allows managing AE scripts to receive, send, or print studies
	API Users	Allows adding and editing API users and rights

Category	Right	Description
	Application Entities	Allows configuring application entities (AE).
	Assign Study to Users	Allows assigning studies to specific users from the worklist
	Company	Allows editing general settings including those related to: company contact and billing, apps, MRN, AE filters, LDAP, Rcopia, and HL7
	File Stores	Allows selecting folders to store various data (mainly images) on the server
	Notification	Allows adding notifications that appear on the dashboard
	Tasks	Allows entering tasks that appear on the My Tasks gadget of the dashboard
HL7	HL7 Global Config	Allows viewing global HL7 configurations
	HL7 Queue	Allows viewing the HL7 Outbound (Sender) log
	HL7 Receiver Log	Allows viewing the HL7 Inbound (Receiver) log
Home	Allow Reset Approved studies	Allows resetting approved studies to Scheduled status
	Allow Reset of Canceled and No Show Studies	Allows resetting studies from Canceled or No Show status to Scheduled status
	Appointment Confirmation	Allows viewing the Appointment Confirmation screen
	Can Edit Public Filter	Allows editing public filters
	Can Edit Public Template	Allows editing public templates
	Cancel Reasons	Allows canceling appointments
	CD Burn	Allows burning DICOM images to a CD or external device
	DICOM View	Allows switching between DICOM preview or priors on the worklist
	DICOM Viewer	Allows opening images on the Exa PACS/RIS viewer
	External App Icon	Allows opening third party applications from the worklist
	Image Preview	Allows previewing images on the worklist
	Import Images	Allows importing images by using the PACS Actions menu
	Import Images from Portal	Allows importing images from the provider or attorney portal into Exa PACS/RIS
	Link DICOM Study	Allows linking a DICOM study to another study
	Marketing Rep Dashboard	Allows opening the Marketing Representative dashboard
	Merge Study	Allows merging and splitting DICOM studies and RIS orders
	Technologist	Allows associating a technologist to an order
	Opal Viewer	Allows viewing studies with the Exa PACS/RIS viewer
	Order additional	Allows opening the Additional Information tab within the Edit Study screen
	Order ICD	Allows adding ICD codes to an order
	Order Referring Provider	Allows opening the Referring Provider tab within the Edit Study screen

Category	Right	Description
	Order Studies	Allows opening the Exam Information tab within the Edit Study screen
	QC Delete	Allows deleting DICOM studies within the QC tab
	QC Edit	Allows editing some information in a DICOM study within the QC tab
	QC Move to Studies	Allows manually moving a DICOM study from the QC tab to the All Studies tab
	QC Reconciliation	Allows manually matching or reconciling a DICOM study to a RIS order
	Worklist Menu	Allows opening the worklist shortcut menu
	Send Fax	Allows sending approved reports and study forms by fax
	Send Studies	Allows sending a DICOM study to another AE from the worklist
	Show Priors	Allows viewing a patients' priors in a separate study filter on the worklist
	Studies	Allows accessing Exa PACS/RIS
	Study Delete	Allows deleting studies from a patient chart, and to delete selected images from series.
	Study Purge	Allows permanently deleting a study
	Study Edit	Allows opening the Edit Study screen
	Study Forms	Allows opening the Study Forms tab within the Edit Study screen
	Change Status	Allows changing the status of a study
Log	API Log	Allows viewing the API log, which tracks API activity
	Audit Log	Allows viewing the Audit log, which tracks system-wide activity
	Rcopia Log	Allows viewing the Rcopia log, which shows Rcopia activity
	Eligibility Log	Allows viewing the Eligibility log
	User Log	Allows viewing the User log, which tracks system-wide user activity
Patient	Activity Log	Allows viewing the Activity log within the patient chart and Edit Study screen
	Advanced Search	Allows expanded searching of patients by age, allergies, lab results, problems, sex, medications, vital signs, smoking status, and race/ethnicity
	Allergies	Allows viewing the Allergies tab in the Edit Study screen, and editing patient allergies
	Allergies (Read only)	Allows viewing the Allergies tab in the Edit Study screen (only).
	Break the Glass	Allows use of the Break the Glass function to access confidential records.
	Eligibility	Allows using third-party services to request confirmation of insurance eligibility
	Eligibility (Read only)	Allows viewing eligibility requests/results if previously requested

Category	Right	Description
	Clinical Summary Transmit	Allows creating, viewing, and transmitting the Clinical Visit Summary/CCD file
	Export CCD	Allows encrypting and downloading of patient charts for physician referrals
	Family Health History	Allows completing the Family Health History form for a patient's first-degree relatives
	Film Tracking	Allows check-out and tracking for 'hard' films from the library
	Immunization	Allows adding and editing patient immunization records
	Immunization (Read Only)	Allows viewing patient immunization records
	Lab Orders	Allows adding, submitting, editing, and deleting laboratory tests and results records
	Lab Orders (Read Only)	Allows viewing laboratory tests and results records
	Medications	Allows adding, editing, and deleting prescribed medications, supplements, and OTC products through Rcopia
	Medications (Read Only)	Allows viewing prescribed medications, supplements, and OTC products through Rcopia
	Merge Patients	Allows merging of patient charts
	Outside Referrals	Allows importing CCD/CCR files into the patient chart from the referring provider
	Patient Alerts	Allows selecting alerts or adding custom alerts and notes to a patient chart
	Patient Arrival Worklist	Allows viewing the patient arrival worklist when patients use the kiosk to check in
	Patient Clinical Rules	Allows creating clinical support rules
	Patient Documents	Allows uploading and scanning documents to a study or patient
	Patient Encounter	Allows viewing fees associated with CPT codes and associated fees or charges
	Patient Guarantor	Allows adding a guarantor to a patient's insurance
	Patient Information	Allows opening the Patient Information tab in the patient chart and Edit Study screen
	Patient Insurance	Allows opening the Insurance tab in the patient chart and Edit Study screen
	Patient Prescriptions	Allows submitting prescriptions to a pharmacy, formulary checking, and adding, editing, and deleting prescribed medications through Rcopia
	Patient Prescriptions (Read Only)	Allows viewing of prescribed medications through Rcopia
	Patient Search	Allows searching for patients from the Patient tab in the patient chart
	Patient Studies	Allows opening the Studies tab within the patient chart and Edit Study screen
	Patient Orders	Allows viewing the Orders tab within the patient chart
	Payment History	Allows opening the Payment History tab in the patient chart

Category	Right	Description
	Pending Follow-ups	Allows viewing follow-up appointments
	Pending Referrals	Allows viewing referrals
	Problems	Allows editing in the Problems tab in the Edit Study screen (to add and edit diagnosis codes)
	Problems (Read Only)	Allows viewing the Problems tab in the Edit Study screen
	To Be Reviewed	Allows opening the To Be Reviewed tab in the patient chart.
	Transition of Care	Allows exporting a CCD file and sending it to a referring physician via email
	Vital Sign (read Only)	-Allows viewing a patient's vital signs on the Edit Study screen
	Vital Signs	Allows adding and editing a patient's vital signs on the Edit Study screen
	Patient Claim Inquiry	Allows viewing patient claim information
	Patient Claim Inquiry (Read Only)	Allows viewing patient claim information
	VIP	Allows designating patients as VIPs
Peer Review	Peer Review Allowance	Allows managing automatic peer review assignments
	Peer Review Manual Appointment	Allows manually assigning peer reviews from the worklist
	Peer Review Operational	Allows generating the peer review operational report
Portal Reg Users	Portal Reg Users	Allows managing registered users of portals, such as Patient Portal
Reports		All rights in this category allow creating the corresponding report
	Study Details	
	Cancellation Reason	
	My Reports	
	Completed Schedules	
	Date of Service Payment Summary	
	Export Completed Studies	
	Export Peer Review	[unused]
	Insurance Balance Aging	
	Marketing Rep Activities	
	Monthly/Daily Study Goals	
	Patients Worksheet	
	Referrals Variance	
	Referring Physician Study Count	
	Report Filter	
	Report Provider	
	Relative Value Units	

Category	Right	Description
	Scheduler Activity	
	Studies Breakdown	
	Studies by Modality	
	Studies by Modality Room	
	Modality Breakdown	
	Transcription Study Count	
	Turnaround Time (TAT) – Calculated	
	Fees by Facility and Modality	
	Fees by Radiologist and Modality	
	Unfinished Studies	
	Unsigned Orders	
	STAT Tracking	
	Insurance vs LOP	
	Marketing Report Export	
	Technologist Productivity	
	Audit Trail	
Schedule	Add Study	Allows creating new studies, such as in the Edit Study screen.
	Allow Double Booking	Allows double-booking an appointment if available for the facility
	Allow to edit Accession	Allows editing study accession numbers
	Approved Report	Allows viewing approved radiology reports
	Assign Vehicle	Allows assigning a vehicle to a study for Mobile RAD
	Available Slots	Allows using the Find/Available Slots screens
	Billing	Allows viewing information from the Edit Study screen
	Charge and Payments	Allows opening the Charges and Payments tab within the Edit Study screen
	TOS Payments	Allows opening the Payments area of the program
	Patient Payment History	Allows viewing patient payment history and printing payment receipts
	Chief Complaints	Allows adding and editing chief complaints in the patient chart
	Chief Complaints (Read Only)	Allows viewing chief complaints in the patient chart
	Clinical Overview	Allows opening the patients clinical overview
	DICOM Edit	Allows editing DICOM information within the QC and Exam Information tabs within the Edit Study screen
	Education Material	Allows opening the Educational Material tab in the Edit Study screen
	Follow Ups	Allows scheduling follow-up appointments in the Follow Up tab in the Edit Study screen

Category	Right	Description
	Follow Ups (Read Only)	Allows viewing follow-up appointments on the Follow Up tab in the Edit Study screen
	Insurance Authorization	Allows adding and editing insurance authorizations for studies
	Medical History	Allows adding, editing, and updating patient medical histories
	Medical History (Read Only)	Allows viewing patient medical histories
	New Order	Allows creating new orders
	Notes	Allows entering study notes, patient notes, schedule notes, reasons for studies, and call log entries
	Notes (Read Only)	Allows viewing study notes, patient notes, schedule notes, reasons for studies, and call log entries
	Order Forms	Allows adding custom forms to orders from the Custom Forms tab in the Edit Study screen
	Order Forms (Read Only)	Allows viewing custom forms added to a study from the Custom Forms tab in the Edit Study screen
	Order Images	Allows ordering images or reports from the Studies tab in the patient chart
	Order Summary	Allow opening the Summary tab in the Edit Study screen
	Ordering Physician Search	Allows using the magnifying glass to search the Provider Setup screen and NPPES website for an ordering physician to add to an order
	Quick Block	Allows creating a quick block (schedule block) from the schedule book
	Patient Demographics	Allows editing information in the Patient Information tab in the patient chart and Edit Study screen
	Priority	Allows assigning priority to the study from the worklist and in the Additional Information tab within the Edit Study screen
	Recent Schedules	Allows scheduling/copying appointments in the Recent Schedules screen if available for the facility
	Recent Schedules (Read Only)	Allows viewing appointments in the Recent Schedules screen if available for the facility
	Reference Document	Allows assigning a reference document to an order and providing it to the patient
	Referrals	Allows entering information for referrals to another provider
	Schedule Book	Allows viewing the schedule book
	Schedule in Non-working Time Slot	Allows scheduling appointments in non-working timeslots from the schedule book
	Stat	Allows setting STAT levels for a study or order from the worklist
	Status Validation	Allows validating a study in the Validation screen when validation rules are set within the Study Status workflow
	Study CPT Update	Allows updating the CPT of a study
	Study Education Material	Allows opening the Educational Material tab in the Edit Study screen

Exa® PACS/RIS 1.4.33_P0 User's Manual

Category	Right	Description
	Teaching Study	Allows creating a DICOM teaching study from the worklist and within the viewer
	Transcription	Allows editing report transcriptions
	ExaTrans2 Multipanel	Allows using Exa Trans2 in the multipanel, and using the multipanel without Exa Trans2.
	Vehicle Assignments	Allows viewing the list of scheduled/dispatched vehicles in using Mobile RAD
	Edit Requesting Date	Allows editing of the initially requested date for an exam
Setup	Appointment Types	Allows adding, editing, and deleting appointment types
	Body Parts	Allows adding, editing, and deleting body parts
	Cognitive Status	Allows adding, editing, and deleting cognitive statuses
	СРТ	Allows adding, editing, and deleting CPT codes
	Custom Forms	Allows adding, editing, and deleting custom forms, and attaching them to studies
	DB Totals	Allows using the DB Totals function
	DICOM Receiver Rule	Allows creating and managing DICOM receiver rules
	DM List	Allows viewing the DM Manager and list of DMs
	DM New	Allows creating a new view in the DM Manager
	EDI Translations	Allows creating and managing EDI translations
	Editor Template	Allows creating, editing, and deleting Editor templates
	EDI Rule	Allows creating and managing EDI rules
	Email Template	Allows creating and assigning email templates for the Confirmation screen
	Notification Templates	Allows creating and assigning templates sent to referring physicians
	Cover Sheets	Allows creating and assigning cover sheets for approved reports
	Exam Authorization	Allows opening the Exam Authorization tab
	Exam Authorization Override	Allows scheduling that breaks authorization rules
	Export Summary	Allows exporting summary totals
	Facility	Allows adding, editing, and deactivating facilities
	Market	Allows configuring regional markets
	Fee Schedule	Allows creating and editing billing fee schedules and contracts
	Facility Fee Schedule	Allows creating a fee schedule based on facility and modality for the Fees by Facility and Modality report
	Provider Pay Schedule	Allows creating a pay schedule for radiologists, which is used in the Fees by Radiologist and Modality report
	Provider Location Information	Allows managing provider location information
	Form Builder	Allows creating and editing electronic study forms, and managing their attachment

Category	Right	Description
	Functional Status	Allows adding, editing, and deleting functional statuses
	Gadgets	Allows adding, arranging, and viewing gadgets within the user's dashboard
	Hanging Protocol Groups	Allows editing DM (hanging protocol) settings for user groups, modalities, and other categories
	SR Report Queue	Allows opening the Structured Report Queue tab, which shows the status of outbound DICOM SRs
	Issuers	Allows adding and editing issuers of patient IDs
	ICD	Allows adding, reloading, and importing ICD, SNOWMED, and LOINC code files
	Insurance Provider	Allows adding, editing, and deleting insurance providers and assigning them to a facility
	Matching Rules	Allows creating rules for matching DICOM studies
	Macro Notes	Allows configuring macro notes
	Modality	Allows adding, editing, and reordering modality priority
	Modality Room	Allows adding and editing modality rooms
	Monthly Goals	Allows creating and editing monthly study count goals by modality for the Monthly Goals operational report
	No Shows	Allows setting an appointment to a no-show status from the worklist and schedule book
	Ordering Facility	Allows adding and editing ordering facilities
	Patient Portal Log	Allows viewing Patient Portal activity
	POS Map	Allows creating and editing place of service mappings for POS codes, addresses, and mobile dispatching addresses
	Billing Rules	Allows adding, editing, and deleting billing rules for ordering facilities.
	Provider	Allows adding, editing, and importing resources such as referring providers, reading providers, and technologists
	Provider Group	Allows adding and editing groups of providers
	Provider Organizations	Allows adding and editing provider organizations
	Queue	Allows viewing the status of outbound DICOM studies
	Rcopia Transactions	Allows viewing Rcopia transactions
	Report Template	Allows user to create and edit report headers for approved reports within the Report Template tab
	Report Queue	Allows resending and viewing the status of outbound emails and faxes such as approved reports, notifications, scanned documents, and study forms
	SR Report Queue	Allows opening the Structured Report Queue tab, which shows the status of outbound DICOM SRs
	Transcription Template	Allows adding, editing, deleting, and associating templates with facilities, modalities, and study descriptions used for transcription

Category	Right	Description
	Routing Rules	Allows creating routing rules for DICOM studies to send, print, or perform an HL7 action
	Schedule Block	Allows adding and editing schedule blocks that display on the schedule book and Find Slots screens
	Schedule Filter	Allows adding and editing filters of the schedule book
	Schedule Rules	Allows adding and editing individual rules within a schedule template
	Allow Editing of Display Block Colors	Allows editing the color of blocks on the schedule book
	Schedule Templates	Allows adding and editing scheduling templates
	Study Filter	Allows adding, editing, and deleting worklist study filters
	Update Global Study Filters	Allows updating global and shared worklist filters
	Study Flag	Allows adding and editing study flags
	Study Status	Allows adding study statuses and associating them with study flows
	Study Status - Read Only	Allows opening the Study Status tab to view study statuses and flows
	Templates	Allows adding and editing structured reporting templates
	Auto Suggestions	Allows adding and editing auto suggestions for structured reporting
	Keywords	Allows adding and editing keywords used for structured reporting
	Image Hotspots	Allows navigating through a cardiac US cine series
	SR Mapping	Allows adding and editing mappings of DICOM properties to structured reports
	Update URLs	Allows adding and updating Rcopia URLs
	Users Online	Allows viewing which users are currently logged on, and terminating connections
	Vaccines	Allows adding, editing, and marking the inactive vaccines list
	Vehicle Log	Allows opening the Vehicle Log tab in Log Setup
	Vehicle Registration	Allows adding and managing available vehicles to dispatch in Mobile RAD
	Vehicle Tracking	Allows viewing the location of a vehicle
	Locked Slots	Allows locking and unlocking slots (scheduling time ranges) for scheduling
	Insurance Provider Payer Type	Allows creating and editing payer type groups or categories that can be associated to insurance companies
	Provider Level Codes	Allows creating and editing provider level codes and percentages to associate with CPT codes, which are used in the Reading Provider Fees report
	Portal Configurations (Read Only)	Allows viewing portal configuration settings.

Exa® PACS/RIS 1.4.33_P0 User's Manual

Category	Right	Description
	Portal Configurations	Allows configuring links to appear on the patient, provider, and attorney portals.
	NDC Codes	Allows configuring NDCs.
User Mgmt.	AD Groups	Allows adding, editing, or deleting AD groups.
	AD Users	Allows adding, editing, or deleting AD users.
	User	Allows managing individual user access to the system
	User Assigned Patients	Allows viewing the list of patients who are currently assigned to specific users
	User Assigned Studies	Allows viewing the list of studies that are currently assigned to specific users
	User Group	Allows creating, managing, and assigning document types and user roles to user groups
	User Role	Allows creating and managing rights
	Copy Profile Settings	Allows copying profile settings from one user to another
	External Apps	Allows configuring Exa PACS/RIS to work with external applications
Viewer Options	Cardiogram Settings	Allows configuring cardiogram settings in Viewer Settings
	External Tools	Allows configuring options in the External Tools tab in Viewer Settings
	General Information	Allows configuring options in the General tab of Viewer Settings
	Interface Settings	Allows configuring options in the Interface tab of Viewer Settings
	Mammography Options	Allows configuring options in the Mammography tab of Viewer Settings
	Modality Options	Allows configuring options in the Interface tab of Viewer Settings
	Overlay Settings	Allows configuring options in the Overlays tab of Viewer Settings
	Tools Settings	Allows configure options in the Tools tab of Viewer Settings

Description of reports

The following table lists all of the reports available on the Exa platform. Some reports may not be available to you depending on your product and configuration.

Category	Report	Description
Billing/Reports	Aged AR Detail	AR aging by facility, responsible party, and payer by cutoff date and selected facilities, in detail
	Aged AR Summary	AR aging by facility, responsible party, and payer by cutoff date and selected facilities
	Charges	Charges for reconciling and reviewing the charges populated for any date of service
	Claim Activity	Detailed claim activity history by claim date range
	Claim Inquiry	All submitted claims, and order payments and balances
	Claim Transaction	All claims, and claim payments and balances
	Collections	Claims in the Collections Review status
	Credit Balance Encounters	Credit balance encounters
	Diagnosis Count	Aggregate diagnosis count by ICD code and facility, by order date range
	Modality Summary	Modality summary
	Monthly Recap	Monthly recap
	Patient Statement	Patient statement
	Patients by Insurance	Patients seen within a date range by insurance
	Payer Mix	Payer mix
	Payments	Totals by payment and payer type, and how each charge line item is applied by accounting date range
	Payments by Insurance Company	Calculated payments to each insurance company, by patient
	Payments Realization Rate Analysis	Accounts with a zero balance
	Procedure Analysis by Insurance	Procedures by insurance
	Procedure Count	Number of procedures approved by radiologist fee schedule
	Reading Provider Fees	Reading provider fees
	Referring Provider Count	Referring provider count
	Referring Provider Summary	Referring doctor summary information
	Transaction Summary	Transaction summary
Reports -	Daily Schedules	Information on studies performed by day
Schedule & Reports	Export Completed Studies	CSV export of patient, study, and insurance details for completed or approved studies
	Marketing Rep Activities	Notes and activities that marketing representatives enter about referring physicians
	Unsigned Orders	not signed by referring physician. No longer used.

Category	Report	Description
Operations	Cancellation Reason	Statistics on canceled studies with cancellation reason as a key filtering criterion.
	Scheduler Activity	Number of orders (based on the current order status) by time of day that a user ordered, scheduled, rescheduled, or canceled, by facility and date range
	Unfinished Studies	not having selected study statuses, by facility and date range
	Referrals Variance	Number of studies referred by referring physicians in the past 13 months, by facility, modality, and month
	Studies Breakdown	Number of studies by facility, modality, study date, and study status
	Studies by Modality	Aggregate study count by facility, modality, study date, and study status
	Studies by Modality Room	Aggregate study count by facility and modality room
	Modality Breakdown	Number of studies performed based on CPT or appointment type, by modality
	Technologist Productivity	Number of studies performed per technologist by facility, modality, appointment type, and procedure code
	Peer Review Operational	Progress and results of peer reviews performed
	Fees by Radiologist and Modality	Summary and breakdown of fees for a modality, by radiologist
	Turnaround Time (TAT) - Calculated	Shows turnaround time (TAT) calculated based on selected date, type, and study's approved date, by date range, facilities, and/or ordering facilities
	Relative Value Units	CPT and RVU totals for reports signed off, by date range
	Referring Physician Study Count	Number of studies referred by referring physicians by date range
	Completed Schedules	Shows information about completed studies
	Monthly/Daily Study Goals	Number of studies performed in the month compared to the goal set in Monthly Goals Setup
	STAT Tracking	Number of studies marked as STAT. Includes approved studies that had a STAT level at any time in the past.
	Transcription Study Count	Aggregate study count of reports transcribed by transcriptionists, by facility, modality, and study date
	Insurance vs LOP	Aggregate study count by modality for insurance paid vs letter of protection/personal injury studies
	Marketing Rep Export	Study information, bill fees, and allowed amounts for referring physicians and marketing reps
	Audit Trail	Break the glass Audit Trail report
	Same Last Name User Audit	Alberta only.
Patients	Patients Worksheet	Patient and study face sheets in form
		Allows opening the My Reports tab to view operational reports

Settings in the claim entry screen

The following table describes the settings shown in the Claim Creation screen.

Sub-Tab	Setting	Description
Charges	Date	Claim entry date, automatically assigned.
	Accession No.	Study accession number, automatically assigned.
	CPT Description	Type-to-search or select by CPT code.
	P1…P4	Diagnosis pointers link the diagnosis code to a CPT. Type the first diagnosis pointer in P1, and then add additional pointers as necessary.
	M1…M4	A modifier acts as a shortcut for making a pre-defined change to a claim. Type the first modifier in M1, and then add additional modifiers as necessary.
	Units	Type the number of procedures of the selected CPT that were performed.
	Bill Fee	The preconfigured charge for the procedure. Edit as needed.
	Total Bill Fee	The bill fee multiplied by the units.
	Allowed	The amount per-unit allowed by the insurance provider as configured in the billing fee schedule.
	Total Allowed	The total amount allowed by the insurance provider as configured in the allowed fee schedule.
	Auth No.	Type the authorization number provided by the insurance provider.
	Exclude	Select to exclude the charge from invoices or claims, such as surgical trays or other miscellaneous items not billed with the service.
Claims	Study Date	Automatically assigned.
	Claim Date	Claim entry date, automatically assigned.
	Facility Name	Select the name of the facility for scheduling purposes. It may or may not be the same as the service facility.
	Billing Provider	Select the provider issuing the invoice for the procedure.
	Rendering Provider	Select the provider who performed the procedure.
	Referring Provider	Select the referring provider.
	Service Facility Location/Ordering Facility	Select the facility where the procedure was performed. For mobile billing, select the ordering facility.
	POS Type	Select the place of service code for the service facility.
	Diagnosis Codes	Select and add all relevant ICD diagnosis codes.
Insurance	Existing Insurance	Select a previously configured insurance profile, if any.
(Primary, Secondary, and	Accept Assignment	[reserved]
Tertiary)	Medicare Payer	For secondary insurance, select if the patient has Medicare Part B , and then select the MSP rule to report in the list.

Sub-Tab	Setting	Description
	Carrier	Select the insurance carrier. The carrier's address and other information is filled in automatically.
	Policy No.	Type the insurance policy number.
	Group No.	Type the insurance group number.
	Coverage Start/End Date	Type the start and end dates of coverage, from the patient's insurance profile.
	Relationship	Select the relationship of the patient to the policy holder.
	Subscriber Name	Type the name of the policy holder.
	DOB	Select the date of birth of the policy holder.
	Gender	Select the gender of the policy holder.
	Address…	Type or select the address, city, state, and postal code of the policy holder.
	Service Type	Available when not using Exa Clear.
		Select the service type to which the procedure belongs.
	Benefit On Date	Available when not using Exa Clear.
		Select the date for which you want to verify eligibility.
	Eligibility/Estimation	Select to verify eligibility and/or estimate coverage. See <u>Verify</u> eligibility and estimate coverage with Exa Clear.
Additional Info	Patient's Condition Is Related To:	Select a source of the condition if relevant for insurance purposes.
	Accident State	If you selected a source previously, type the state/province in which the accident (source) occurred.
	Date of illness Onset, Injury/Accident, or Pregnancy (LMP)	Select the date of onset, accident, or first symptoms. In the Other Date box, enter additional dates related to the accident or injury, if needed.
	Dates Patient Unable to Work at Current Occupation	Select a date range in the From Date and To Date boxes.
	Hospitalization Patient Related to Current Services	Select a date range in which the patient was hospitalized, if relevant.
	Claim Notes	Enter notes for the claim. How different from Billing notes.
	Outside Lab	Select if entering a claim for a lab external to the current facility.
	Original Ref	The original reference number returned by Medicaid or Medicare if they are the primary payer. This number can be submitted to a secondary payer.
	Claim Authorization No.	Type the claim authorization number.
	Frequency	If you must submit a corrected claim to the insurance provider, use this option to code it as such.
		If set to 1 , remove the original ref. number.

Sub-Tab	Setting	Description
		If set to 7 , include the original ref. number.
	Delay Reasons	Reason for delayed billing. Confirm with the insurance carrier before selecting a reason.
Billing Summary	[Amounts]	Totals for fees charged, payments made, adjustments, claim balance, allowed amounts, other payments, and refunds.
		Bill Fee: Mapped to the total claim amount. If multiple accessions are billed on one claim, the claim inquiry shows the amount of the first accession.
	Claim Status	Select a claim status. You can use the status selected here for sorting in the Claims screen. This status changes as the claim moves through payment cycles.
	Billing Notes	Type notes to appear in the Notes column of the Claims screen. This is different from Claim notes, which are submitted with the claim.
	Billing Codes Select a preconfigured billing code to flag claims for your own specific purposes.	
	Billing Class Select a preconfigured billing class to flag claims for your own specific purposes.	
	Responsible	Select the party responsible for payment. For mobile billing this is especially useful to select if the ordering facility and service location are different.
Payments	Accounting Date	The entry date of the payment, assigned automatically.
	Payer Name	Select the payer.
	Pymt. Mode	Select the type of payment.
	Check/Card No.	If the payment mode is "Check," type the check number.
	Apply	Select to apply the payment to the current claim.
	This Pay.	A reference value. Select the Apply button to enter payment information.
	This Adj.	A reference value. Select the Apply button to enter an adjustment amount.
	Payment ID	Automatically assigned serial number for the payment.
	NEW PAYMENT	Select to add a new payment.

See also:

Add codes and other basic data Configure procedure codes Verify eligibility of preorders and studies with PokitDok Verify eligibility and estimate coverage with Exa Clear

Measurement accuracy limits

Measurement	Tools	Unit	Accuracy
Length	Ruler	mm	±2 mm
Angle	Protractor	0	±2°
Area	Calculated	cm^2	±5%
Ratio	Calculated		±2%

Viewer tools and keyboard shortcuts

The following table lists all tools that are available in the viewer toolbar and shortcut (context) menu, and that are available as keyboard shortcuts. See corresponding topics for the functions of the tools.

Bold = Tool appears by default, cannot be removed.

TB = Toolbar (main, top, bottom, left), **CM** = Shortcut (context) menu, **KS** = Keyboard shortcut, **ETS** = Exa Trans shortcut

Tool	Name	Availability	Function
U	3D Ray	TB, CM, KS	Draws a 3D ray on an image. Select to place a vertex on one plane, and the ray extends through the image on all planes.
٩	3D Cursor	TB, CM, KS	Displays a cursor in other open planes that follows the motion of the cursor in the active plane.
3DA	3D Angle	ТВ, СМ	Click three points to create the angle. The angle appears at the same coordinates on all series with the same frame of reference.
2014	3D Measure	ТВ, СМ	Click two points (or drag and click). The measurement appears at the same coordinates on all series with the same frame of reference.
30P	3D Point	ТВ, СМ	Click to add the point. The point appears at the same coordinates on all series with the same frame of reference.
aDR	3D ROI	ТВ, СМ	Available in a future release.
AR	AR	TB, CM, KS	Plots points to represent angular rotation.
	Actual Image Size	TB, CM, KS	Displays images in their real-life anatomical dimensions.
	Add to DICOM Print Queue - Image	CM, KS	Adds the current image to the DICOM print queue for later printing.
	Add to DICOM Print Queue - Series	CM, KS	Adds the current series to the DICOM print queue for later printing.

Tool	Name	Availability	Function	
\bigtriangledown	Angle Marker	TB, CM, KS	Measures and annotates an angle.	
AS	Annotate Series	TB, CM, KS	Select this tool, then select an annotation tool and annotate the current image to add the same annotation to all images in the series. Does not support 3D annotations.	
ATI	Annotate Tagged Images	TB, CM, KS	Select, and then apply a 2D annotation to a tagged image to annotate all tagged images.	
	Approve	ETS	Approves the current study.	
	Approve and Next	ETS	Approves the current study and opens the next study.	
\mathbf{z}	Arrow	TB, CM, KS	Draws an arrow.	
	Audio Annotatio n	СМ	[Currently unavailable]	
ant [Auto W/L	TB, CM, KS	Automatically optimizes the W/L.	
1	Bone Enhance	ТВ, СМ	Controls contrast between bone and other tissues.	
GAD	CAD	TB, KS	Displays CAD findings, with a button menu for selecting specific findings. Available when CAD findings are present in the opened study. The following can be set independently.	
			- BTO Jump to Next Finding	
			- Toggle Breast Density	
			- Toggle Breast Geometry	
			- Toggle Calcification Clusters	
			- Toggle Individual Calcification	
			- Toggle Malc	
			- Toggle Nipple	
			- Toggle All Study	
			- Toggle Current Image	
1 - 1	Center Point	ТВ, СМ	Click two points. The center point between them is indicated.	
.	Change Slice Thickness	ТВ	Sets the slice thickness.	
	Change Annotatio n Color	СМ	Right-click an annotation, select Change Annotation Color, and then select a color in the sub-menu. Prerequisite: Configure two or more annotation colors in viewer settings.	
CL	CLAHE	TB, KS	Contrast limited adaptive histogram equalization. Turns CLAHE mode on/off. Improves image contrast.	
	Close Series	KS	Keyboard shortcut for the Close Series option in the image cell's upper-right shortcut menu.	

Tool	Name	Availability	Function	
	Close Study	KS	Keyboard shortcut for Partial Close.	
Y.	Cobb Angle	TB, CM, KS	Measures a Cobb angle.	
×	Context Menu	ТВ	Displays the viewer shortcut.	
† +	Context Tool	TB, CM, KS	Displays a linked image at the same X, Y, Z angle as the current image.	
	Contrast Enhance	СМ	Applies a fixed image contrast adjustment.	
UNG	Create Teaching Study	ТВ, СМ	Creates a teaching (unq.) study.	
4	CT Ratio	TB, CM, KS	Displays the angle between two lines. Use to calculate the angle between vertebra.	
6	Curve	TB, KS	Click two points to estimate curvature from the center of the end vertebrae to the center of the apical vertebrae.	
솔	Cutlines	TB, CM, KS	Shows/hides cutlines, if present.	
k	Default	TB, KS	Selects the standard pointer tool.	
	Delete Annotatio n	СМ	Right-click an annotation and select Delete Annotation to delete.	
	Delete Same Type Annotatio n	СМ	Deletes all annotations of the same type (such as text annotations) on the current image only.	
3	Delete All Image Annotatio ns	TB, CM, KS	Deletes all annotations.	
S.	Delete Series Annotatio ns	TB, CM, KS	Deletes all annotations in the current series.	
2	Delete All Study Annotatio ns	TB, CM, KS	Deletes all annotations in the current study.	
	Delete Series	СМ	Available with Study Delete rights. Select to delete the current series.	
	Delete Image	СМ	Available with Study Delete rights. Select to delete a single image from a series.	
DP	DICOM Print	ТВ, СМ	Sends the image to a DICOM printer.	

Tool	Name	Availability	Function	
	DICOM Values	TB, CM, KS	Displays a list of DICOM values associated with the current image.	
	or			
	View			
	DICOM Values			
DN	DM	тв, см	Opens display management settings.	
	Settings			
	DM	KS	DM navigation functions:	
			DM Next	
			DM Next Current	
			DM Next Group	
			DM Next Prior	
			DM Previous	
			DM Previous Prior	
٠	Dot	TB, CM, KS	Draws a dot.	
	Download Image	CM, KS	Downloads the current image to the default download folder on the workstation.	
	Download Series	СМ	Downloads the current series to the default download folder on the workstation	
	Download Series as MP4	СМ	Downloads the current series as an MP4 video, saved in a selectable frame rate.	
2	Edit Annotatio ns	TB, CM, KS	Turns Edit mode on/off. Edit mode allows editing of previously set annotations.	
\circ	Ellipse	TB, CM, KS	Draws an ellipse.	
	Enable Cine	CM, KS	Select to turn on scrolling of series images with the mouse wheel.	
	External Tools	ТВ, СМ	Reserved for opening an external application.	
8	Fit to Window	TB, CM, KS	Restores original zoom level.	
A	Flip	TB, CM, KS	Flips the image across its horizontal or vertical axis (hover over the tool to select between axes).	
	Frames	ТВ	Selects the number and arrangement of frames to display in the viewing area.	
P	FH Measure	ТВ, СМ	Takes freehand measurements.	
Piere ROC	Freehand ROI	TB, CM, KS	Freehand-draws an ROI.	
HG	Histogram	тв, см	Displays a histogram.	

Tool	Name	Availability	Function	
${\times}$	HO Angle	TB, CM, KS	Draws a horizontal orthogonal angle.	
	or			
	Horizontal Orthogona I Angle			
L	Horizontal Line	TB, CM, KS	Draws a horizontal line.	
Hu	HU	TB, CM, KS	Displays the pixel value in Hounsfield Units.	
	or			
	Hounsfield Units			
	Invert	TB, CM, KS	Inverts black/white values.	
0	L Marker	TB, CM, KS	Adds an L (left) marker.	
1	Line	TB, CM, KS	Draws a straight line.	
	Linking	KS	After linking two series, the Linking keyboard shortcut turns linking ON and OFF	
\$^	Link Auto	TB, KS	Links two or more series that are open.	
÷,	Link Manual	TB, KS	Links user-specified frames between panels.	
	Mask Image	ТВ	[BTO only, currently unused]	
H	M Compare	TB, CM, KS	Measurement comparison. Click a first and second point, then double-click a third point. Displays the distances between the points.	
	or			
	Measure Compare			
•	Magnify	TB, CM, KS	The "magnifying glass tool," displays a movable zoomed area.	
1	Measure	TB, CM, KS	Measures between two specified points.	
MPR v	MPR	TB, KS	Multiplanar reformation. Select to generate and display coronal, sagittal, and MIP (maximum intensity projection) stacks from an axial image.	
	Next Series	KS	Keyboard shortcut for moving to the next available series in the study.	
1:1	One to One	TB, CM, KS	Displays the image in its original DICOM size.	
	Open Next Study	KS	Auto Open Next Study option.	
87	Pan	TB, CM, KS	Drags the image any direction.	
D	Patient Documen ts	ТВ	Opens an independent window for viewing, scanning in, and attaching patient documents. Turns blue when documents are available.	

Tool	Name	Availability	Function	
1	Pencil	TB, CM, KS	Freehand-draws a line.	
0	Perfect Circle	TB, CM, KS	Draws a circle.	
8	Play Frames	ТВ	Starts cine looping.	
	Pre- Approve	ETS	Pre-approves the current study.	
	Presets	СМ	Display Window/Level presets	
	Previous Series	KS	Moves to the previous series in the study.	
4	Print Images	ТВ	Sends images to a non-DICOM printer.	
R	Prior Reports	ТВ	Opens reports. Turns blue when reports are available. Opens prior approved reports regardless of DICOM status (including non-imaging priors).	
®	R Marker	TB, CM, KS	Adds an R (right) marker.	
+	Radial Angle	TB, CM, KS	Draws a radial angle, which is the angle of the distal radial surface with respect to a line perpendicular to the shaft.	
Ŧ	Radial Length	TB, CM, KS	Draws a radial length.	
	Rectangle	TB, CM, KS	Draws a rectangle.	
Ċ	Reset Frame	TB, KS	Undoes all unsaved changes to the image within the active viewing cell.	
	Reset Linked Series	TB, KS	Removes any linkages.	
\$	Reset Stack	TB, CM, KS	Restore images in a stack to their last-saved ordering.	
	Reset Study Object	СМ	Resets all unsaved changes in the series.	
\mathbf{n}^{\dagger}	Reset Viewer	TB, CM, KS	Reverts non-permanent modifications to images in the viewer (zoom, rotation, Window/Level etc.) to their original state.	
ROE	ROI	TB, CM, KS	Draws an ellipse to specify an ROI.	
C	Rotate	TB, CM, KS	Rotates the image 90 degrees clockwise.	
	(Right)		Hover over the rotate button to reveal the rotate left button.	
J	Rotate (Left)	TB, CM, KS	Rotates the image 90 degrees counterclockwise. Hover over the rotate button to reveal the rotate left button.	
	Save	ETS	In Exa Trans, saves the current transcription.	
ť	Save Annotatio ns	ТВ	Saves the current state of annotations.	

Tool	Name	Availability	Function	
	Save Image Annotatio ns	KS	Saves the current image annotations.	
	Save Series Annotatio ns	KS	Saves the current series annotations.	
	Save Study Annotatio ns	KS	Saves the current study annotations.	
	Save MPR	СМ	Saves the current MPR-generated images.	
	Skip	ETS	In Exa Trans, skips the current study and moves to the next.	
5	Span or Scroll/Spa n Images	TB, CM, KS	Switches between moving and spanning.	
тю	Set Key Image	TB, CM, KS	Sets the current image as a key image.	
	Send Image/ Annotatio ns	СМ	Opens a dialog for sending the current image to selected AEs.	
٥	Settings	тв, см	Displays the settings dialog.	
Ρ	Show Prior List	ТВ	Opens a dialog with a list of priors that you can select to display in the prior bar.	
	Shutter Box	TB, CM, KS	Adds an rectangular shutter.	
SHE	Shutter Ellipse	TB, CM, KS	Draws an elliptical shutter.	
энг	Shutter Freehand	TB, CM, KS	Draws a freehand shutter.	
1	Spine Label	ТВ, СМ	Adds vertebrae labels to spinal images.	
6	Spine Label Short	ТВ	Adds vertebrae labels from common starting points.	
2	Spine Label 3D	ТВ, СМ	Adds vertebrae labels on all series with the same frame of reference.	
2	Spine Label 3D Short	ТВ	Adds vertebrae labels from common starting points on all series with the same frame of reference.	

Tool	Name	Availability	Function	
2	Spine Label 3D Short	ТВ	Adds vertebrae labels to the same anatomy on all series with the same frame of reference.	
2	Spine Label Quick 3D Short	ТВ	Adds vertebrae labels from common starting points to the same anatomy on all series with same frame of reference.	
N	Study Notes	ТВ	Displays a dialog for reading/writing notes.	
	Switch to 2D	KS	For mammography, switches from 3D to 2D view, if available	
	Switch to BTO	KS	For mammography, switches to BTO view, if available	
	Switch to C View	KS	For mammography, switches to C view, if available	
Π	Tag Images	TB, CM, KS	Click to tag images in a series for batch annotation.	
Т	Text	TB, KS	Draws text.	
TA	Toggle Annotatio ns	TB, KS	Shows/hides annotations.	
۲	Toggle Overlays	TB, CM, KS	Shows or hides overlays. See also "Toggle DICOM Overlays."	
тро	Toggle DICOM Overlays	ТВ, СМ	Shows/hides DICOM overlays.	
т	Transcript ion	ТВ	Opens an independent dictation and transcription window.	
	Triangulati on	TB, CM, KS	Displays the context tool at the same coordinates on all series with the same frame of reference.	
UAI	Untag all Images	TB, CM, KS	Untags any images tagged (selected) for batch annotation.	
	Vertical Line	TB, CM, KS	Draws a vertical line.	
0	Window/L evel	TB, CM, KS	Adjusts the window/level.	
q	Zoom	TB, CM, KS	Zooms the image in/out.	

Details on Linked Reporting

Statuses of linked reports

• Any activity related to retaining the STAT level of a main study also applies to any of its linked studies.

- Any activity related to TAT recording (approval time) on a linked main study also applies to any of its linked studies.
- When you unlink an approved study, the study reverts to Unread status.
- When a main study changes to Approved status, so do its linked studies. In addition:
 - The order status of the linked studies changes to Check-Out.
 - The approving physician and/or pre-approving physician and approved date are added to the main and linked studies.
- When a main study changes to the following RAD-related statuses, so do its linked studies.

Draft

Transcribed

Pre-Approved

Dictated

Approved-Coded (APCD status)

• When you add an addendum to a main study and the study status changes, its linked studies change to the same status, including the following.

Draft - Addendum

Transcribed - Addendum

Approved - Addendum

Dictated - Addendum

- When you right-click a main study and select Exam > Reset Transcription, the main study and linked studies are reset to Unread status.
- When you change a main study from Approved to an earlier status, you are prompted to unlink its linked studies. The main study changes to the selected status, and the linked studies change to Unread status. The report remains with the main study.
- When you change the status of a linked study, you are prompted to unlink it from the main studies. The linked study changes to the selected status, and no other studies are affected.
- When you reset the transcription of a main study, this resets the transcription for the main and linked studies, and sets their statuses to Unread. The studies remain linked.
- When you reset a main study with an approved report, you are prompted to unlink its linked studies. The main study changes to Scheduled status, and the previously linked studies change to Unread status.
- When you reset a linked study with an approved report, you are prompted to unlink it from the main study. The study changes to Scheduled status, and other studies are unaffected.

Considerations when transcribing linked studies

• When transcribing a main or linked study, the transcription lock is applied to the main study and all linked studies (in both Exa Trans and Web Trans). The following actions release the transcription lock on all studies:

Close the transcription

Sign out of Exa PACS/RIS

Approve the study (E-sign and Approve, Approve and Next, or Pre-Approve)

Using Skip or Not Approve

- Immediately after you link studies, Exa Trans and Web Trans updated accordingly, even if a transcription is open. This includes updating of the report header with information from the linked studies.
- Exa Dictation audio files are available in the viewer and in Web Trans for all main and linked studies.

Which reports can you link?

When you select to link reports, Exa PACS/RIS automatically displays a list of reports that are available to link. These studies:

- Have a study date 3 days before or after the main study
- Have a status other than Ordered, Cancelled, Rescheduled, No Show, Read, or Approved
- Are from the same facility as the main study

Mammography views

In mammography there are two main view "families," **CC** and **MLO**. Each family contains multiple subviews.

C	C Family	M	LO Family
CC +	 Craniocaudal Superior – Inferior C-Arm Angle: 0 degrees 	MLO	 Mediolateral Oblique Superomedial – Inferomedial C-Arm Angle: 30–60 degrees
xcc	 Exaggerated Craniocaudal Superior – Inferior C-Arm Angle: 0 degrees 	ML	 Mediolateral Medial-Lateral C-Arm Angle: 0 degrees

XCCL	 Exaggerated Craniocaudal Lateral Superior – Inferior C-Arm Angle: 0 degrees 	LMO	 Lateromedial Oblique Inferolateral – Superomedial Oblique C-Arm Angle: 90–180 degrees
XCCM	 Exaggerated Craniocaudal Medial Superior – Inferior C-Arm Angle: 0 degrees 		 Lateromedial Lateral-Medial C-Arm Angle: 90 degrees
FB	 From Below: Caudocranial Inferior – Superior C-Arm Angle: 0 degrees 	SIO	 Superolateral to Inferomedial Oblique Superomedial – Inferomedial Oblique C-Arm Angle: 1–90 degrees

The following views are also available:

- **ID** Implant displacement
- CV Cleavage view
- AT Axillary tail
- RL Rolled lateral
- **RM** Rolled medial
- **RI** Rolled inferior
- RS Rolled superior
- MAG Magnification
- **S** Spot compression
- TAN Tangential

Install and configure tools

This section contains the following topics.

Add the Chrome extension Install Exa Trans Install Exa Dictation Install OPAL tools Install Exa Client Viewer Install Exa Client Viewer Install Exa Scans Install Exa Launcher Install Exa Tool Install and configure local cache Add a trusted IP or URL

Add the Chrome extension

You can add an extension that configures your Chrome browser to work with multiple monitors. After adding the extension, you can configure display settings for the monitors you will use.

Procedure

- 1. On the burger Hennu, on the **TOOLS** submenu, select **CHROME EXTENSION**.
- 2. On the Chrome_MultiMonitor page, select Add to Chrome.

à.



Note: If the 'Add Chrome_Multimonitor? message appears, select Add extension.

3. On the worklist, on the upper toolbar, select the settings button, and then in the button shortcut menu, select **Viewer Settings**.



CO

- 4. On the GENERAL tab, under Display Settings:
 - a. Select the Monitors on System box
 - b. Select the refresh \bigcirc button
 - c. Select the identify 💿 button.
- 5. In the table of monitors:
 - a. In the **Monitor** column, select all monitors that you want to use.
 - b. In the **Current** column, select the monitor to hang current images and thumbnails.

c. In the Prior column, select the monitor to hang prior images and thumbnails.

Note: When using a hanging protocol, the Current and Prior settings in the table only control where the thumbnails hang.

- 6. Optional. In the **Orders** column, select an item to auto-open in the monitor.
- 7. Select SAVE.

Install Exa Trans

Radiologists who use Dragon for transcription must install Exa Trans.

Procedure

1. On the burger 🔲 menu, on the **TOOLS** submenu, select **EXA TRANS**.



Note: Chrome downloads ExaTransSetup.msi in the lower left corner of the browser. Wait until the download is finished before continuing.

2. Select ExaTransSetup.msi. Windows installs Exa Trans.



Note: If the Windows protected your PC dialog appears, select **More info**, and then select **Run anyway**.

Install Exa Dictation

Radiologists who use Dragon for dictation must install Exa Dictation.

Procedure

1. On the burger Hann, on the **TOOLS** submenu, select **EXA DICTATION**.



Note: Chrome downloads exa_dictation_setup.msi in the lower left corner of the browser. Wait until the download is finished before continuing.

- 2. Select exa_dictation_setup.msi.
- 3. Windows installs Exa Dictation.



Note: If the Windows protected your PC dialog appears, select **More info**, and then select **Run anyway**.

Install OPAL tools

Many functions in Exa PACS/RIS require that you first install OPAL tools, such as CD import and CD burning.

Procedure

1. On the burger 🔜 menu, on the **TOOLS** submenu, select **OPAL TOOLS**.



Note: Chrome downloads OpalToolsSETUP.exe in the lower left corner of the browser. Wait until the download is finished before continuing.

2. Select **OpalToolsSETUP.exe**. Windows installs Opal tools.



Note: If the User Account Control dialog appears, select Yes.

Install Exa Client Viewer

Exa PACS/RIS uses a server-side viewer, but you can also install the client-side Opal viewer.

Procedure

1. On the burger Henn, on the **TOOLS** submenu, select **OPAL VIEWER**.



Note: Chrome downloads ExaClientViewerSETUP.exe in the lower left corner of the browser. Wait until the download is finished before continuing.

2. Select ExaClientViewerSETUP.exe. Windows installs the Opal viewer.



Note: If the Windows protected your PC dialog appears, select **More info**, and then select **Run anyway**.

Install Exa Scans

With Exa Scans, you can scan paper and other media and attach the scans to studies as DICOM data.

Prerequisite: You must uninstall any existing version of Exa Scans before installing a new one.

Procedure

1. On the burger Hondright menu, on the **TOOLS** submenu, select **EXA SCANS**.

Chrome downloads exa_docscan_setup.msi in the lower left corner of the browser. Wait until the download is complete before continuing.

2. Select exa_docscan_setup.msi.

Note: If the Windows protected your PC dialog appears, select **More info**, and then select **Run anyway**.

- 3. In the Welcome to the EXA Document Scan Setup Wizard, select Next.
- 4. On the **End User License Agreement** page, select **I accept the terms in the License Agreement** checkbox, and then select **Next**.
- 5. On the **Destination Folder** page, select **Next**.
- 6. On the Ready to Install EXA Document Scan page, select Install.
- 7. If the User Account Control dialog appears, select Yes.
- 8. On the Completed the EXA Document Scan Setup Wizard page, select Finish.

Install Exa Launcher

Exa Launcher is a background application that prepares studies to be opened by third-party applications such as PenRad, PowerScribe 360, and MModal.

Procedure

1. On the burger menu, on the **TOOLS** submenu, select **EXA LAUNCH**.

Chrome downloads ExaLaunchExtApp.msi in the lower left corner of the browser.

- 2. Wait until the download is complete, and then select **ExaLaunchExaApp.msi**.
- 3. If the UAC prompt appears, select **YES**.

Install Exa Tool

Exa Tool is a client side background application that facilitates opening of studies in OrthoView.



Prerequisite: Install and configure local cache.

Procedure

- 1. On the burger 🔲 menu, on the **TOOLS** submenu, select **EXA TOOL**.
- 2. Wait until the download is complete, and then run exa_tool_setup.msi.
- 3. Follow the on-screen prompts.

Install and configure local cache

Local cache is a service that manages the copying of studies from the server to your local workstation for local use.

Configure the server

- 1. On the burger 🔲 menu, select **SETUP**.
- 2. On the **OFFICE** menu, select **DICOM**.
- 3. On the **AE Title** tab, select **Add**, and then enter the following settings.

АЕ Туре	Local Cache
AE Title (Remote)	[Name with no spaces]
Description	[Friendly descriptive name]

4. Select SAVE.

Configure the client

1. On the burger Hann, on the **TOOLS** submenu, select **CLIENT SERVICE**.

Result: Your browser downloads **exa_localcache_setup.msi** to its specified folder for downloads (usually the Windows Downloads folder).

- 2. Run exa_localcache_setup.msi and follow the prompts to install it.
- 3. In the Exa PACS/RIS worklist, select **Settings** and select **Local Service Settings**.
- 4. In the **Rendering Options** screen, under **Prefetch Configuration**, enter the following settings.

Target	Shortcut
AE Title	Type the AE Title (Remote) name that you created in "Configure the server."
User Name	Type a user name with rights to access studies on the server.
API URL	Type the address of the server hosting the studies that you want to locally cache.
Debug Mode	Select to add additional troubleshooting information to logs.
Disable Oncall	Turns automatic caching of the user-selected On Call worklist filter ON and OFF.
Allow Opal Receive	Select to allow receiving, parsing, and storing of Opal protocol files. In the Opal Receive Port , type the port number on which to receive Opal files.
Allow DICOM Receive	Select to allow receiving, parsing, and storing of DICOM protocol files. In the DICOM Receive Port , type the port number on which to receive Opal files.

5. In the **Rendering Options** screen, under **General Configuration**, enter the following settings.

Target	Shortcut
API Timeout	Type the number of seconds to keep the connection to the server open.

Target	Shortcut
Study Timeout	Type the number of hours to keep the imageservice/study thread open.
Prefetch Interval	Type the number of minutes keep to studies in the send queue.
Cache Timeout	Type the number of days to keep locally cached studies before purging.
Cache Directory	Type the fully qualified path to the local cache
	(default = c:\viztek\exa\cache\localcache)
Study Count	Type the maximum number of studies to keep in local cache.
RAM Usage	Type the maximum number of megabytes of RAM to use for prefetching and local rendering.
Max Threads	Type the number of simultaneous threads available to download studies.
Parallel Prefetch	Leave blank.
Cache Disk Usage	Move the slider to set the maximum percentage of hard disk to use for prefetching and local rendering.
Oncall Interval	Type the number of minutes to elapse before checking for new studies to prefetch.
Prefetch Priors	Select to include priors when prefetching.
Number of Priors	Select the number of priors to prefetch.
Enable Skinline Detection	Select to make Skinline Detection available when viewing prefetched studies.
Cache Filter Page Size	

6. Select ACTIVATE.

When local cache is active, a **Disable** checkbox becomes available in local service settings to turn OFF local cache and stop related services and threads.

Add a trusted IP or URL

If access to an external server is needed to access to third-party apps, add the IP address or URL of the external server to a list of trusted resources as follows.

Procedure

- 1. On the Exa PACS/RIS server, use File Explorer to go to the ...web\config directory.
- 2. In Notepad, open the express.js file.
- In the app.use(helmet.contentSecurityPolicy({ area, under scriptSrc, add the IP or URL as in the example in the following figure.



- 4. In **Control Panel**, select **Administrative Tools**, and then double-click **Services**.
- 5. In the list of services, select **exa_web**.
- 6. Use the toolbar to **stop**, and then **start** the service.

Location and contents of screens Burger menu

The burger menu contains the following options.

Burger > Worklist



Burger > PATIENT ARRIVAL WORKLIST



Burger > PATIENT



Burger > DISPATCHING DASHBOARD



Burger > SCHEDULE



Burger > BILLING



Burger > SETUP



Burger > TOOLS




Worklist

Worklist > ALL STUDIES



Worklist > Settings > User Settings

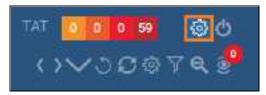

User Settings

Viewer Settings

Local Service Settings

Worklist > Settings > Viewer Settings



Worklist > Settings > Local Service Settings



Office menu

The setup area (or "office") menu contains the following options.

OFFICE > Office

	OFFICE-
	Office
	Providers & Resources
Co	Scheduling & Codes
	DICOM
	Billing
	Meaningful Use
	User Management
	General
	HL7
Co	Log

Billing

Billing > Studies

STI	DIES			Interne States Sures (Western 10	William Health Frank 100/07
	N.LUTUDIO 🖘	MANERADIANI BARTIKA	ediaren diaarvet	LCTV0B07	MINIDUM ANTOIDUCH
1	Annual M.	PERMIT.	21105.0.19	AN AVAILY PRINTERS	
			0620-94-25-2029-05-10		
0	1014-102	THE ALL PROPERTY.	102340-1911-0040-990		1
6	Further	Pourtable, Pourrath	22220519920.441107	**************************************	
D.	259/1064	15.9	2023-05-19 (0:05-89, 100	SEPTERATE ROWNING HIS	

Billing > Claims

CL.	AIMS			Danie regelerite a	OR NEWS- THEF LOS- LIGOT
	ALL CLAME FILLOW (FILLOW)		0.5	/5 B B 7 🛦 💽	LIDATE ELECTRONICCIAN (ED)
C	inter and	OLAINA WATE	OAM NAMES	Exchange and the	aurorosaus auro
	12/15/2222-00/14/2023				(A1) (41)
E.	2° 10 10 00.00.000	00/01/2000		Tesk, Richard	Walter Compression Search Develo
Ę.	2 III 🗊 econores	10.002428	1.0	1995.470	Kolenn
n	2 11 11 AL AND AND A	05/3203421	- 11	Telf anty:	Estart

Billing > Payments

PAYMEN	TS						2	per via		Pagravette	e tep	iri- leng	· leg- light
Farmer	Pela Hirmijit -			CANADADIT		Riverier (1404 - 1 \$2,698-00	Active Processon Active Active Active	Sco.08		and the second second	1 million 1	and the second se	diam'r
1.95587.7	INTER AND ADDRESS	i Altorayout	1.89/18.790	NAME AND A	NUMBER	ek Assess	at research	o muutee	Access	arres in	tint ex	SCROTTEN.	contaction paratest
		NUCLEAR D-	11										1000
2.8	NORG	2014421	restains	47060067670	6	00000	0010.00	MOR.	10.00	10	irenia.	Cent	Kry Contented
ins.	1013.0.0	201.01.21		UTBALLATE FO	6	008.00	100 00	10.00	12.05		feeties.	Cart-	MyJorques
24	1002100-0	20010010	Pulses	Test, Tant	Testagor	1000	60 E2	101.20	in co	24	Junia	Gani	furnished.

Billing > Setup

Budies Claims, Paymenta E	COB Report - Setup - Log - Log Off
V0Ø\$74 🔤 🚧	LIDATE E Refresh Settings
PATIENT NAME	CLEARIT Adjustment Codes
	All Billing Codes
Test. Richard	Billing Classes Worker C
Transfer to the second	Claim Status
Test, Jil	Collections Process
Test, Judy	Delay Reasons
Test, Judy	Billing Providers
Test, Judy	Provider Id Code Qualifiers
Test Stephane	Billing Messages
	Payment Reasons
Test, Stephane	CAS Group Codes
Test Stephene	CAS Resear Codes
Test, Helen	Status Color Codes
	Billing Validations
	EDI/ERA Templates
	EDI Clearinghouses
	Insurance Mapping
	Printer Templates
	Auto Billing

Billing > Report

Studies (Claims P	Paymenta EO	8 Report - Setup - Log - Log Off
>~>>	BILLING	METHOD	My Reports Aged A/R Summary Aged AR Detaile
	All	s Billing	Charges Claim Activity
	Patient Pa	ayment	Claim Inquiry Claim Transaction Collections
	Patient Pa Patient Pa	ayment	Credit Balance Encounters Diagnosis Count
	Electronic Patient Pr Electronic	nyment	Modality Summary Monthly Recap Patient Statement
	Patient Pa Patient Pa	ayment	Payer Mox Payments Payments Realization Rate Analysis
	Electronic	e Billing	Patients By Insurance Company Payments By Ins Company
	Electronic Patient Pa	10	Procedure Analysis By Insurance Procedure Count
	Electronic		Reading Provider Fees Referring Provider Count

Screens in the Edit Study screen

Edit Study > PATIENT INFORMATION

	Petient Inform	uefficer servi	NEWARS	r 304 LC3	99E	-qui		13 1940 GL
COMPRESERVATION OF THE PARTY OF	Nate	NONSEE	anco y				Licenia No.201	
AND AND AND ADDRESS OF	Pailine *	HICCH	19641 ⁴				the face	14,000
economia Arrange automa	CIODAL Provincial	-					Orp. Set.e.	2000
CONTRACTORY.	Asseat No. (JA.+	Peori 100	Ø				Sylastions	
THE OTHER DESIGNATION		MANUSE AL	NAME OF BRIDE	ALMAN, EDUISED			ing taken	
Arryst	Patre 1	7000	340				05/066/28	
THE CHARGE BRANNING		- Antonia	a da	*				20412
Alignet Address of	Alam	The Harm	1.00	Harted			Think Fax.	
	008.1	dianyistt.	Ω.	3m *	н	38		
	Name Texas	Select.	12	Augest Relat				C Freeze
NANCE NO.	Service 2 Service	telict	÷ 0	Weight (the)			Alatin	
LOV INTERN	Realition	🖸 tranias Informe Ababa	i i	Perettin	Selict			
ALCOHOL O		Hatse PL	- 2	Langiage	Selver			

Viewer Settings

Viewer > Settings



Viewer > Settings > General

Viewer Settings						
GENERAL	Display Setting	15				
HALEBYYCE	Monitora cri Byatem	3		6 ©		
MODALITY	Monitor	Current	Prior	Full Screen	Orders	Тури
-Manager 1 - K	DISPLAY	0	0		None .*	+
MAMMOBILAPHY	DISPLAY2	۲	0	8	Nons •	•
OVERLAYS	DISPLAY3	0	۲		None +	•
TOOLS	Autosave Optic	ons				
EXTERNAL TOOLS	Autopave Image P	hopenties on	Close			
	Autopave Image A	vinicitations o	n Close			
	Auto Mark Read S	tudy on Class	-			
	Auto Lipland Data	tion on Class	55			
	Autopany Viewers	Settinge on G	losa			
	Auto Open Next S	tudy				
	🔲 Auto Send Arredta	ited Images o	of Close			

Viewer > Settings > Interface

ETTINGS			
OUTHERIN,	Toolbar		Annotation
INTERFACE	Toolbat Button Scale		Z Show Annotation
MODALITY	Thumbrail Ger Scole	- N.	 Gra-Click Americations Show One-Click Americations Button
MANIMOGRAPHIY	Magnifying Glass Size	200 Px Factor	Annotation Efit Mode
OVER AYS	2 W/L Acceleration		 Use Det Cureer Edit Annotations after Creating
70013	100		Allow Drag Croste
10005	Span Sensitivity	51	Confirm Annotation Bulete
EKTERSAL 700LS	Auto Pdr Low	High	Peroistant Annotation Tool Show Measurements Is mm •
	Thumbouil Column Room		Colors

Viewer > Settings > Tools

SETTINGS			SWE
GENERAL	Toalbar	Tool	
INTERIO (18	Al 👻		Taols
WOOALITY	>>MAIN		3D Argla
	>>TOP		30 Measure
MANIMOGRAPHIC			30 Point
OVERLAVS	>>BOTTOM		30,801
(DELML/IV)	SHEET		3D Ree
TOOLS	SECONTEXTMENU		SC Cursor
ENTERNAL	30KYBD.SHORTCUTS		Angular Rotation
TDOLS			Aptual Intege Size
			Create Teaching Study
		-	Angle Market