

Exa® PACS RIS

Feature Summary

Exa Trans Improvements

©2023 Konica Minolta Healthcare Americas, Inc.




Exa Trans Hotkeys

Starting in version 33, you can now configure keyboard shortcuts for the following functions of Exa Trans: Approve, Approve and Next, Save, Skip, and Pre-Approve.

Note: You must have the Exa Trans application in focus to utilize the shortcuts.

- 1. Go to Worklist > Settings > Viewer Settings > Tools > Exa Trans Shortcuts.
- 2. Under **Tools**, select shortcuts and assign keyboard keys.
- 3. Select **SAVE**.

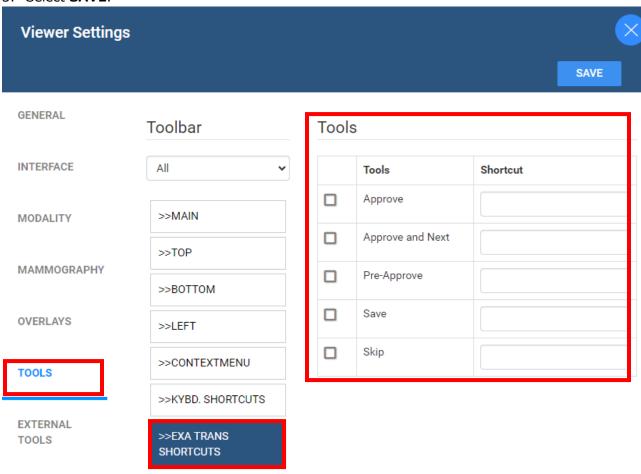


All caps button




500-000712A 2




New Voice Rec Option for Exa Trans without Dragon or Exa Voice

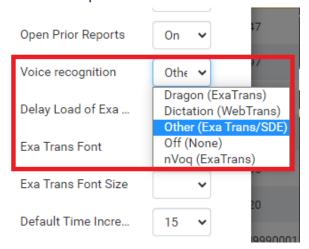

New Options Menu

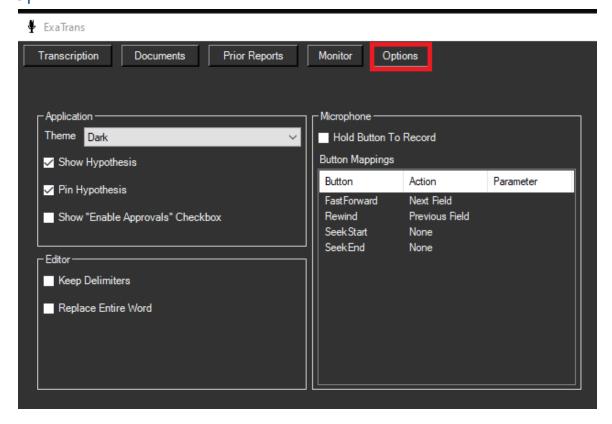


Improved Template Manager

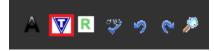

500-000712A 3



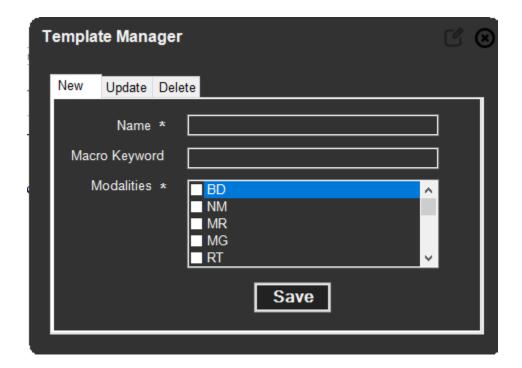

500-000712A 4