

Exa® PACS/RIS

Feature Summary


Move Studies between Charts

©2024 Konica Minolta Healthcare Americas, Inc.


Move Studies

Studies can sometimes accidentally appear in the wrong patient chart. If you need to move some (but not all) studies under an account number to another patient chart, follow this procedure.

1. Determine the correct patient account number for the studies.
2. Right-click a study to move and select **Patient Information**.

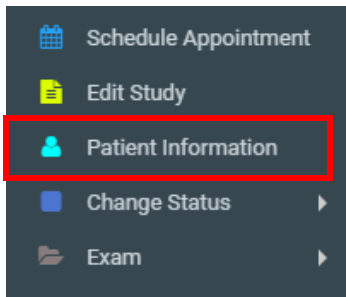

3. Type the correct account number in the **DICOM Patient ID** box, and then press **Tab**.

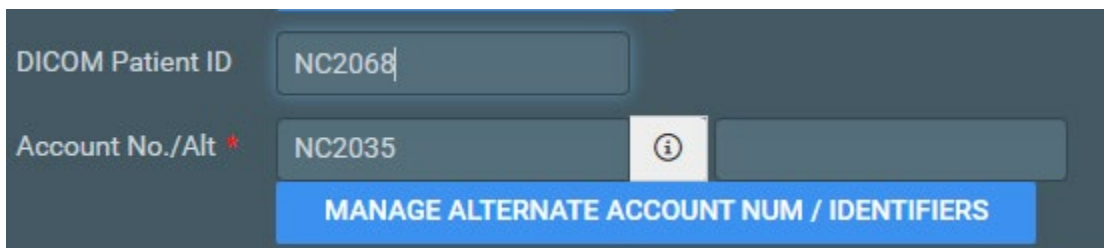

4. In the **Existing studies for patient** list, select studies you want to move and the select **SAVE SELECTED**.

A screenshot of a table titled 'Existing studies for patient: NC2035'. The table has columns for 'accessionNo', 'Study Date', 'Study Description', and 'Referring Physician'. There are buttons for 'SAVE', 'SAVE SELECTED', and 'CLEAR' at the top right of the table area.

accessionNo	Study Date	Study Description	Referring Physician
6113	11/20/2023 11:00 AM EST	MRA Abdomen with and without Contrast	Ref*Test***
6171		CT Orbits or IAC with and without Contrast	
6129	02/22/2024 4:10 PM EST	XR Abdomen (Flat)	
6359	04/18/2024 4:10 PM EST	XR Abdomen, 2 View	
6161	01/03/2024 9:15 AM EST	XR Abdomen (Flat)	Ref*Test***
6174	01/09/2024 8:40 AM EST	CT Chest with Contrast	
6173	01/09/2024 9:20 AM EST	CT Abdomen with Contrast	Ref, Test
6265	02/15/2024 12:45 PM EST	CT Abdomen and Pelvis with Contrast	
6175	01/08/2024 8:45 AM EST	XR Chest, 2 View	Ref, Test
6060	10/18/2023 2:20 PM EST	CT Abdomen and Pelvis with Contrast	

Note: Selecting **SAVE** moves all studies listed to the new chart regardless of which are selected.