

Exa® PACS RIS

Feature Summary


Exa PACS/RIS on Mobile Devices

©2024 Konica Minolta Healthcare Americas, Inc.

When you sign into Exa PACS/RIS on a tablet or other mobile device, it opens its mobile-optimized site. This site offers touch screen functionality with a simplified interface and intuitive navigation for smaller screens. You can easily access features and tools with touch screen gestures and simple controls.

Worklist

You can search for a patient by name on the All Studies tab or select a worklist filter from the drop-down menu.

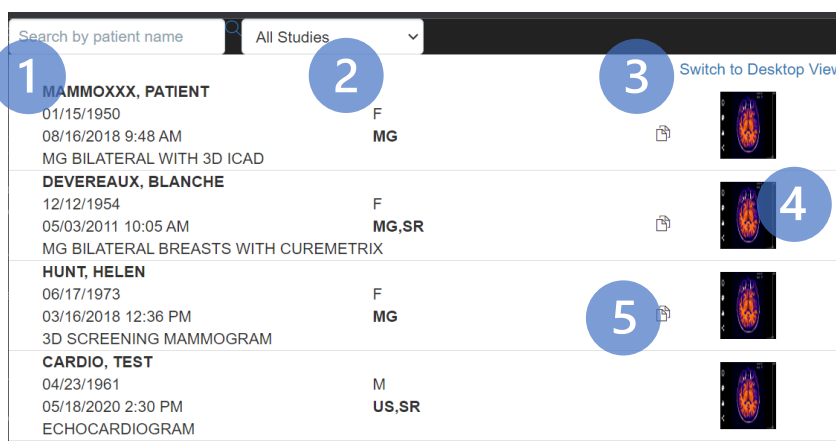

- 1 Search the list by patient name
- 2 Access filter tabs
- 3 Switch to desktop view (website)
- 4 Open approved reports
- 5 Open images



In the desktop view, you can select the ellipsis button to open the shortcut menu.


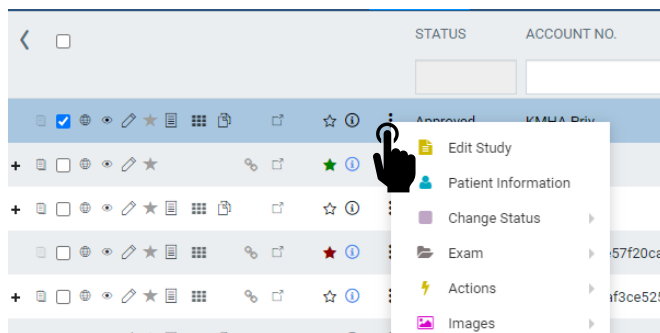



Image view

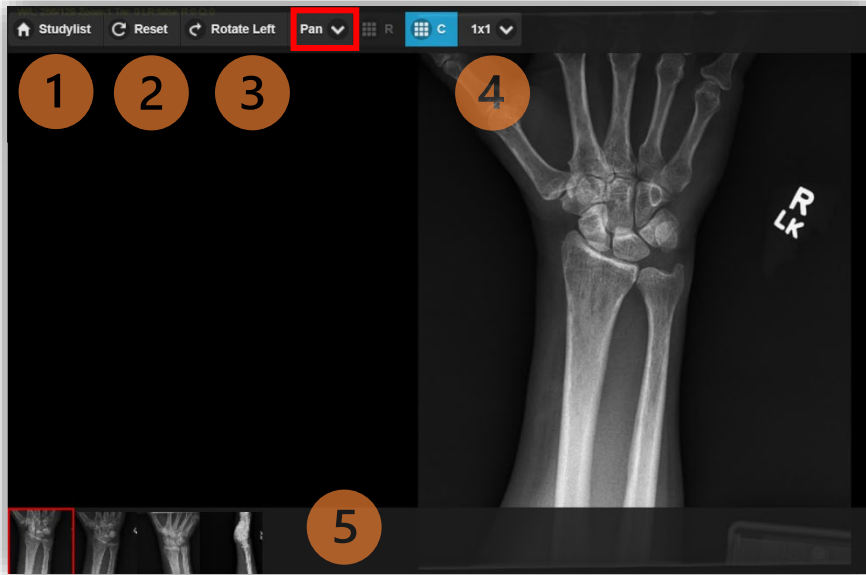

- 1 Close the image and go back to the worklist
- 2 Reset image manipulation
- 3 Rotate image clockwise
- 4 Change image layout
- 5 Image/series selection

Viewer tools

Select tools by dragging your finger on the touch screen.

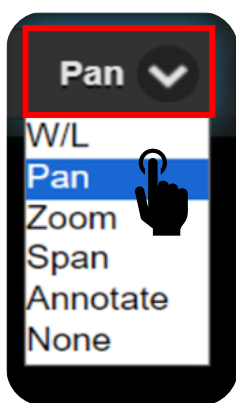

- W/L** Adjusts the window/level
- Pan** Pan the image within the viewing window
- Zoom** Enlarge/reduce the view of the image its center
- Span** Navigate through a series of images (CT, MR, US)
- Annotate** Draw on the image. Annotations are not saved when using the mobile viewer
- None** Disable toolset