



Exa® PACS|RIS

34.1.7

Customer Release Notes

Table of Contents

Browser recommendations 3

Specifications in this release 4

New Features 5

Customer Requested Improvements 5


Other Improvements 5

Bug Fixes 5

Browser recommendations

To ensure uninterrupted functioning of your Exa platform product, we strongly recommend the following:

- Manage and deploy updates to Chromium in a test environment prior to full deployment.
- On all clients, turn off automatic updates in Google Chrome by following these steps:
 1. In Windows, press Windows + R.
 2. In the Run dialog, type `msconfig` and press Enter.
 3. In System Configuration, on the Services tab, select the Hide all Microsoft services checkbox.
 4. Clear the following checkboxes as shown in the figure below:
 - Google Updater Internal Service
 - Google Updater Service

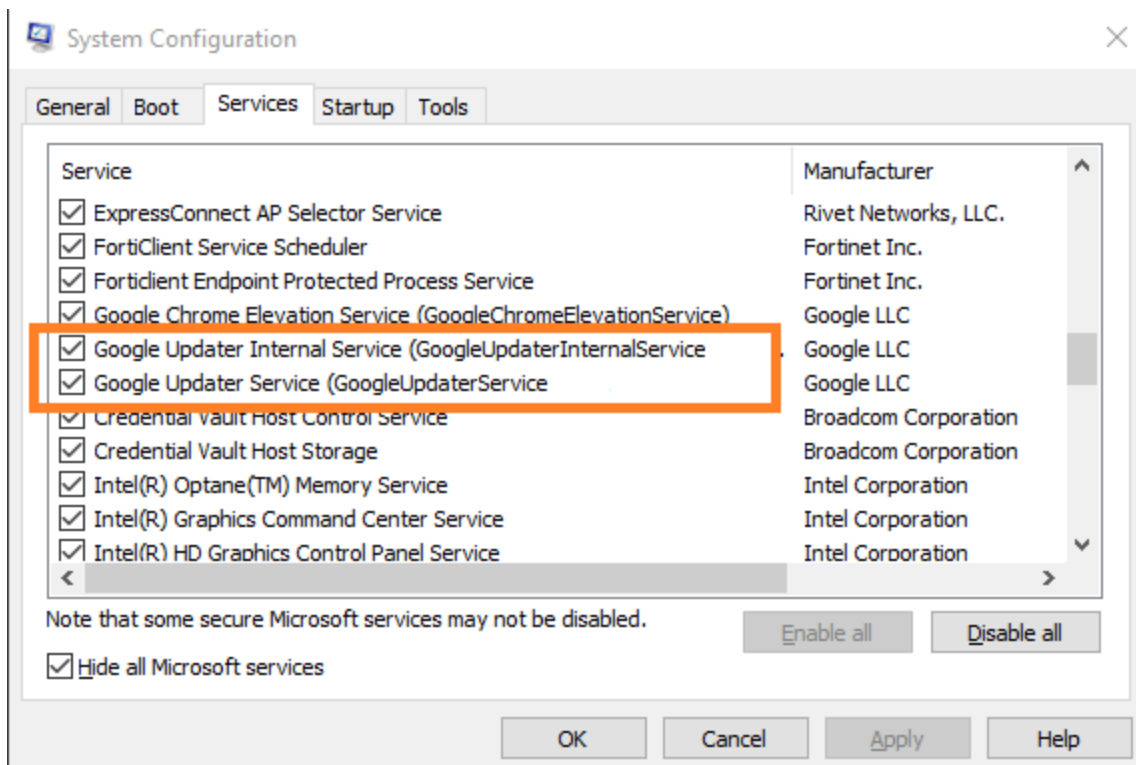

5. Select Apply, OK.
6. If prompted, restart your computer.

Specifications in this release

This version of your Exa platform product was validated using the product versions listed below.

Application	Version
Google Chrome (client browser)	Version 151.0.7922.76 (Official Build) (64-bit)
Windows Server	Windows Server 2022 Standard
Windows Client	Windows 10 Pro (19045.5011,19045.4651) Windows 11 Pro (22631.4391,22631.4317)

For system requirements, see the dedicated system requirements document.

New Features

Summary

- No new features in this release.

Customer Requested Improvements

Summary

- When administrators configure LCM, they now have greater control of how files are purged.

Other Improvements

Summary

- Various improvements to performance and the UI.

Bug Fixes

Summary

- Patient documents (upload or scan) are no longer erroneously saved with the same study ID.
- In the WebTrans multipanel, restored proper functionality of the Approve & Next button.
- When in M-Mode, disabled measurement and other non-supported tools.
- If a user worklist filter includes Is Not operator, matching studies are now excluded without requiring a refresh.
- Made various security improvements.
- When logged in as a radiologist, restored the ability to create an XML drop file automatically for Pen Rad when opening a study.
- When searching for a patient chart with the same MRN as other patients, moving to different pages of results and adding search criteria no longer causes failure to find the matching chart.
- When an appointment extends into a schedule block configured as Not Available, a warning is no longer displayed if the appointment type is not restricted by the schedule rule.
- When scheduling an appointment that extends into a timeslot set as Non-Working in a schedule rule, a warning now appears whether or not the non-working schedule rule is displayed as a block.
- On the schedule book, after configuring a quick block, the block now consistently appears without having to refresh the modality room.
- After merging 2 patients, each at a different facility, both patients can now be found on the patient chart search screen.
- On the schedule book, fixed a problem in which double-clicking a slot resulted in a server-side error at certain sites.

Summary

- When adding a macro note, if the user filters by modality, the Modality column in the Edit Appointment Types dialog now only displays the selected modalities.
- On the worklist, after uploading a document to a patient and then deleting the study, the document is no longer deleted from the patient chart.
- Using Query and Retrieve, restored the ability to query patients using name, DOB, and gender as criteria.
- On the schedule book, after rescheduling an appointment to a different day by using its shortcut menu, the appointment block no longer remains visible on the original day.
- LDAP users can now access the worklist and viewer through encrypted URLs.
- In Web Trans, restored functionality of the Approve & Next button.
- Restored functionality of disabling the Schedule in a non-working timeslot user right.
- In the HL7 sender log/receiver log, when there are 500 or more records, No records found no longer repeatedly appears at the end of the list, and the page no longer becomes unresponsive.
- Restored the ability to send outbound HL7 messages at certain sites.
- In the viewer, restored the ability to automatically or manually link certain studies when open in a 2 x 2 layout.
- Restored the ability to launch a third-party application from the worklist, the viewer, and a new Chrome tab.
- Fixed a problem in which, with some large studies, dragging thumbnails between a current and prior monitor resulted in unresponsiveness.
- You can now merge or split an AHI study into a RIS study (a destination study that was created through an HL7 order).
- When searching for a patient chart by last name, first name, and ZIP code, the search no longer times out.
- In Concord, fixed a problem in which reports faxed to providers were blank.
- When a document is sent from Concord to Exa PACS|RIS, clarified the text of a potential error message to include the phrase Unexpected non-whitespace.
- After a query changes, the provider ordering no longer changes from the configuration in provider and resource filters.
- When Updox appointment reminders are enabled and a patient chart is updated, no calls are made (no patient details are sent) to Updox if the appointment sync cron job is running.
- In the viewer, when drawing ROI or FROI annotations on a CT/PT study with WB_NAC series images, the measurements are now shown in CENTS units.
- The DICOM compression service no longer generates excess .bak files that prevented images from loading.
- When viewing a referring physician's worklist filter, the server no longer times out with an error.
- In an MG DM in the viewer, the system no longer erroneously hangs empty US breast prior cells set to Any (it no longer ignores other logic requirements).