


Exa[®] PACS|RIS

Feature Summary

Add Tools to the Viewer Toolbar

By following the procedure below, you can make modality-specific keyboard shortcuts that select viewer tools. See also “Assign toolbar keyboard shortcuts,” and “Configure standard tools” in the user’s manual.

1. On the worklist, select the settings  button, and then select **Viewer Settings**.

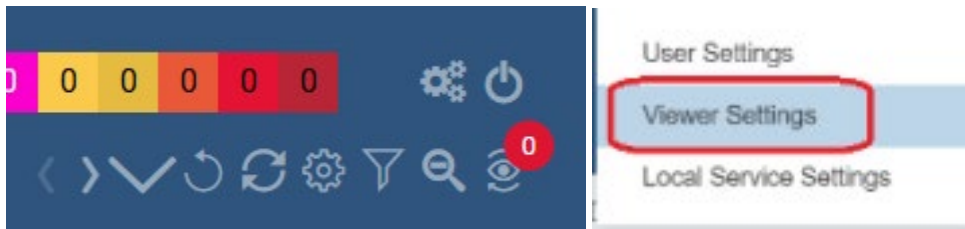

2. In the **Viewer Settings** dialog box, on the left pane, select **Tools**.
3. Under **Configure For**, select a modality if needed.
4. Under **Tool Settings**, select or clear check boxes to add or remove tools from the Main, Left, Top, or Bottom toolbar.


Main – The main viewer toolbar

Left, Top, Bottom – The popup toolbars within viewer frames

5. Optional. In the **Keyboard Shortcut** box, type a key that selects the tool.

Note:

- You can type a single letter, or Ctrl + a single letter.
- Keyboard short cuts are case sensitive.

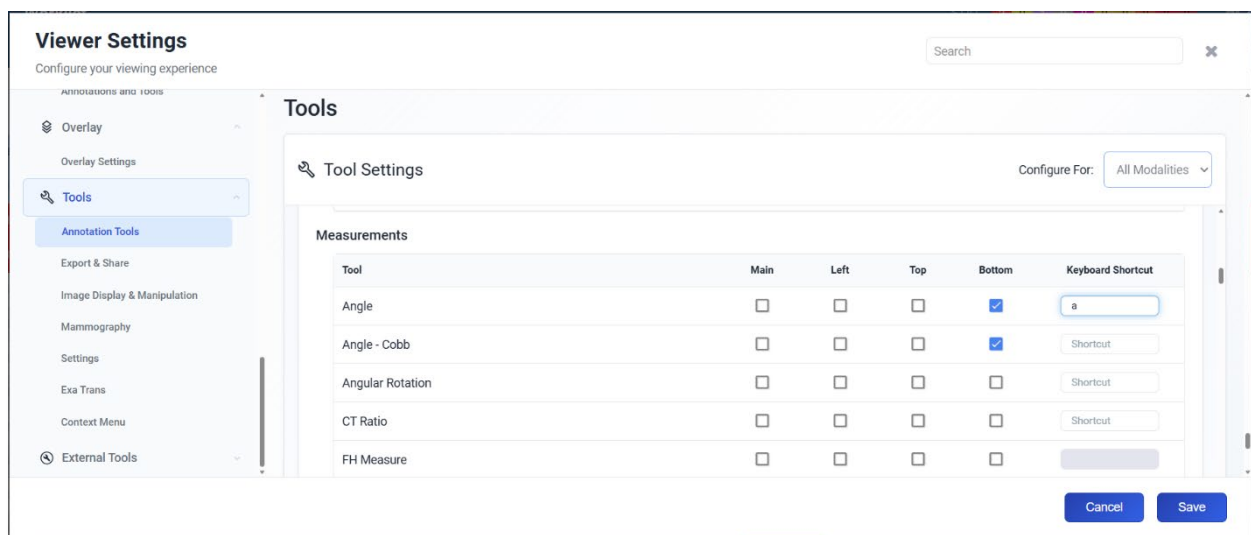

6. Select **SAVE**.