




Exa[®] PACS|RIS

Feature Summary


Change Annotation Color and Size

You can change the colors and sizes of annotations by following the procedure below. This can be useful for non-color monitors and/or colorblind personnel. See also “Configure annotation colors” in the user’s manual.

1. On the settings  menu, select **Viewer Settings**.

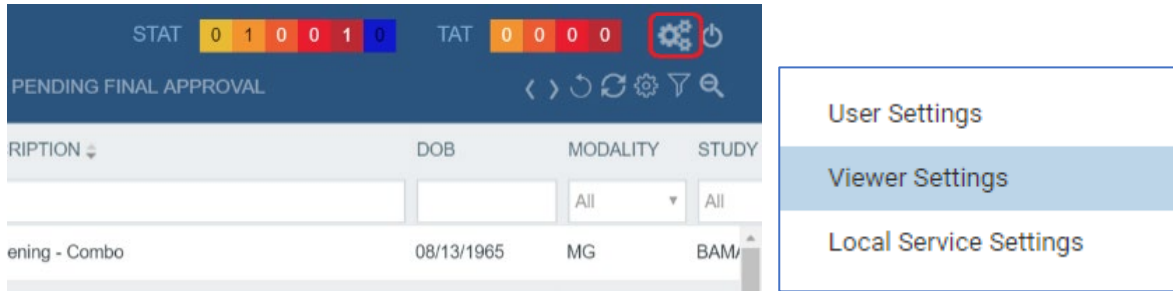

2. In the **Viewer Settings** dialog box, on the **INTERFACE SETTINGS** tab, make changes and then select **SAVE**.

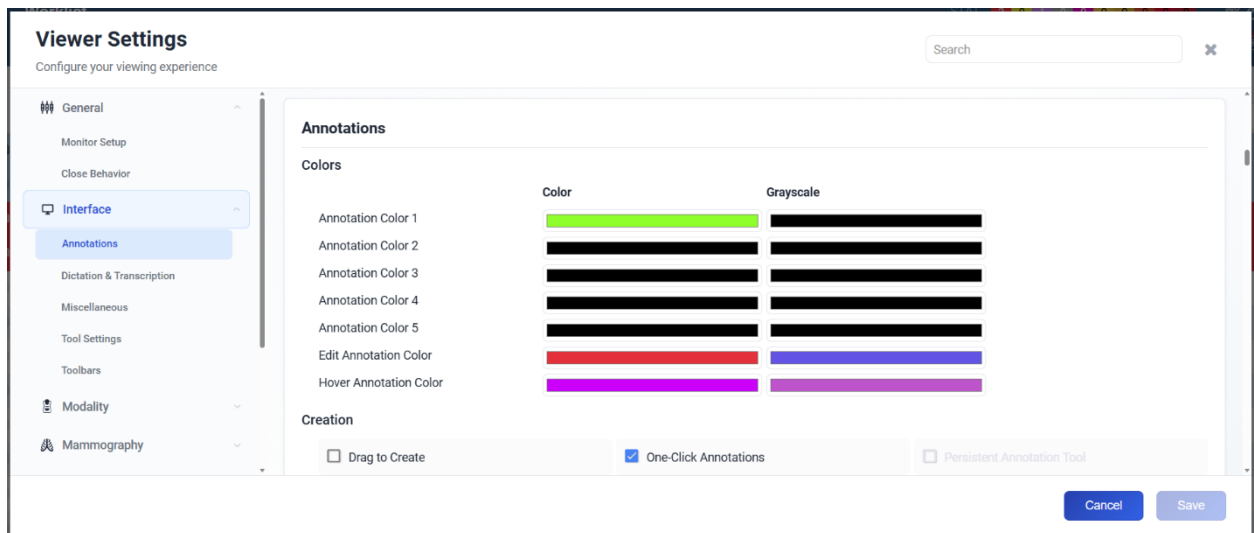

3. To change the size and other properties, scroll down to **Visibility & Display**.

