



# **Exa® PACS|RIS**

## **Feature Summary**

**Version 35.0.1 and Later**


**Customize the Viewer Context Menu**

The context menu (opened by right-clicking) in the viewer provides quick access to commonly used tools and functions within the image area. Customizing this menu allows you to add, remove, or rearrange options to better match your workflow, reducing menu clutter, and provide faster access to the features you use most often.


See also “Add or remove tools from the image shortcut menu” in the user’s manual.

1. Go to Viewer Settings:


- From the worklist, select **Settings > Viewer Settings**

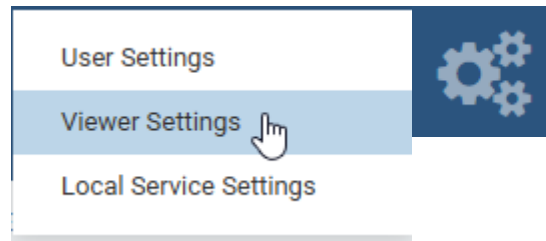

- From the viewer, select the Settings button in the main toolbar

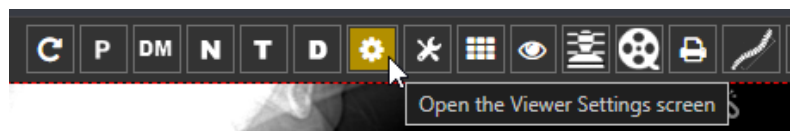


2. Select **Tools > Context Menu**

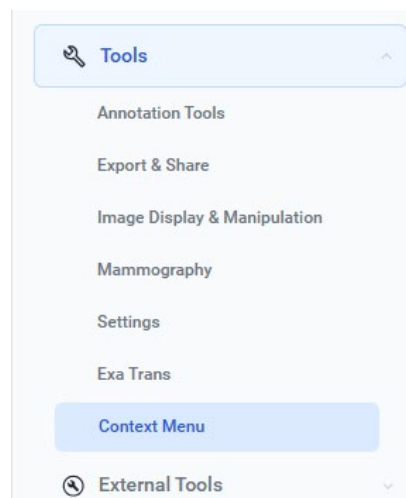

## Add/Remove options

You can select or clear the checkboxes to add or remove options from the context menu. Note that some menu items are only available with certain user rights, even if selected in the configuration.

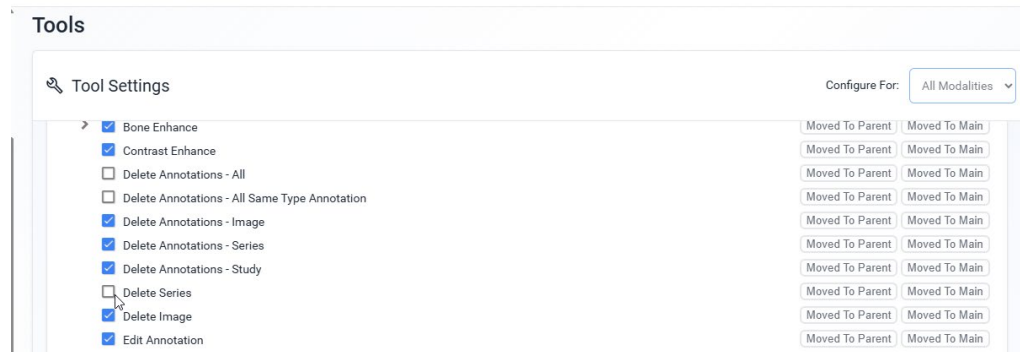

## Rearrange Options


Some options are located within submenus. If you use these options frequently, you can move them closer to the top level of the context menu for quicker access. Select **Move to Parent** to move an option up one menu level or **Move to Main** to place it directly in the root menu.

### Example with “View DICOM Values”

Original:

#### Context Menu

##### Tools

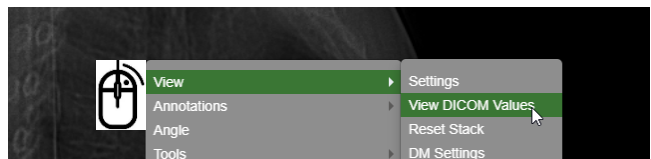

### Moved to Main

#### Context Menu

##### Tools

