




Exa[®] PACS|RIS


Feature Summary Customize the Viewer Toolbar

You can change the button size, thumbnail location, and DM toolbar location on the viewer toolbar by following the procedure below. See also, “Configure the toolbar” in the user’s manual.

1. On the settings  menu, select **Viewer Settings**.

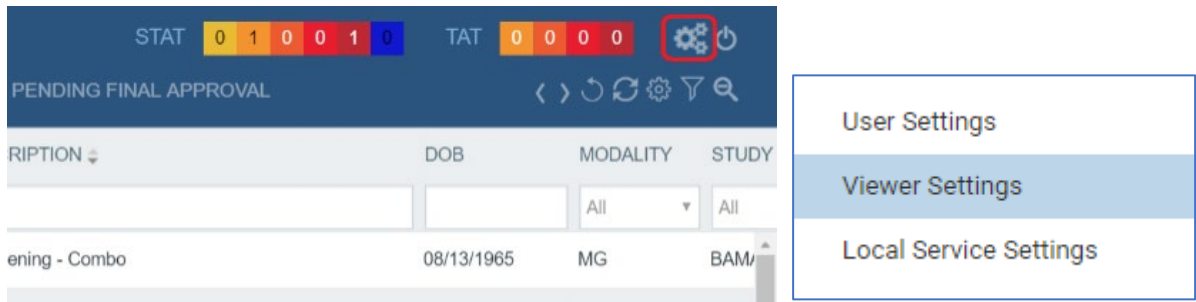

2. On the **INTERFACE SETTINGS** tab, in the **Toolbars** section make changes and then select **SAVE**.

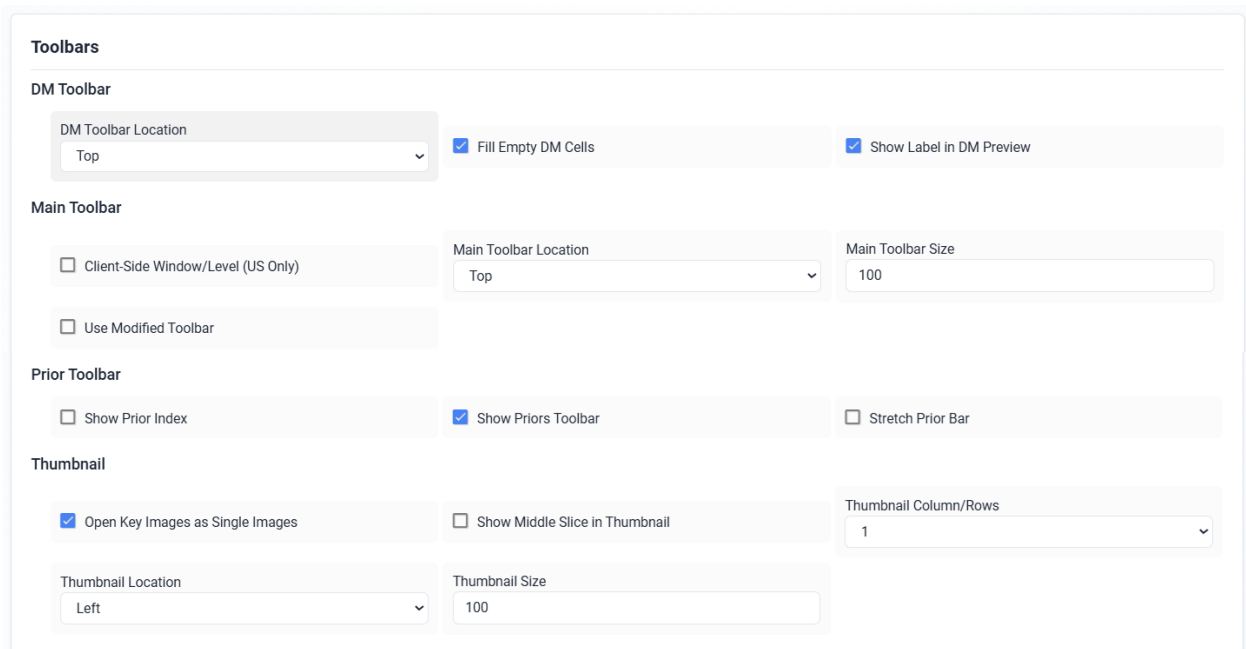The screenshot shows the "Interface Settings" window, specifically the "Toolbars" section. It is divided into four sub-sections: "DM Toolbar", "Main Toolbar", "Prior Toolbar", and "Thumbnail".
DM Toolbar: Includes a dropdown for "DM Toolbar Location" set to "Top", a checked checkbox for "Fill Empty DM Cells", and a checked checkbox for "Show Label in DM Preview".
Main Toolbar: Includes two unchecked checkboxes: "Client-Side Window/Level (US Only)" and "Use Modified Toolbar". It also has a dropdown for "Main Toolbar Location" set to "Top" and a text input for "Main Toolbar Size" set to "100".
Prior Toolbar: Includes an unchecked checkbox for "Show Prior Index", a checked checkbox for "Show Priors Toolbar", and an unchecked checkbox for "Stretch Prior Bar".
Thumbnail: Includes a checked checkbox for "Open Key Images as Single Images", an unchecked checkbox for "Show Middle Slice in Thumbnail", a dropdown for "Thumbnail Column/Rows" set to "1", a dropdown for "Thumbnail Location" set to "Left", and a text input for "Thumbnail Size" set to "100".