



Exa® PACS|RIS

Feature Summary

Study Flags

©2026 Konica Minolta Healthcare Americas, Inc.

What is a study flag?

A *study flag* identifies a study, such as for medical records or as a potential teaching study. You can add multiple study flags to a study.


Why use study flags?

In addition to identifying studies, study flags can drive your workflow. A more detailed guide on building worklist filters can be found here and in the user’s manual. You can make a worklist filter that lists studies having a study flag you create. The Study Flag option is found in Study Information within the Study Filter option.

Where do study flags appear?

Study flags appear:

On the worklist

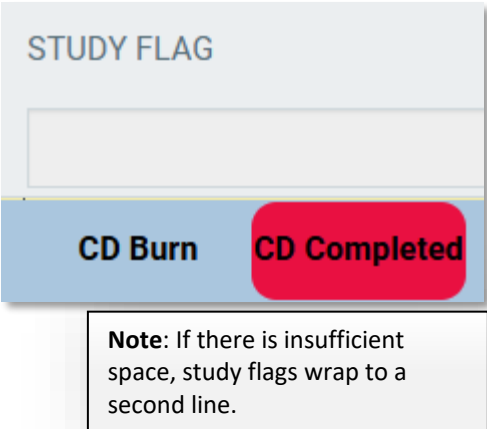

On the schedule book



Create a study flag

1. Go to **Setup > OFFICE > STUDY FLAG.**

DESCRIPTION	INSTITUTIONS	FACILITIES	MODALITIES	COLOR CODE	ACTIVE
1 test		1202 E. Nasa Parkway,AutoFacNoValidation,Autotest			✓
11 all facilities		(CPS) 1034 E. Hwy 67,(CPS) 11211 S Hwy 6, Ste B,(CiBone Densitometry,Bloodwork,C			✓
11 all facilities and future		All	All Modalities		✓
1st call attempt	1241 Campbell RdALEX	(CPS) 1034 E. Hwy 67,(CPS) 11211 S Hwy 6, Ste B,(CiAll Modalities			✓
2nd call attempt		All	All Modalities		✓
3rd call attempt		All	All Modalities		✓
A POTENTIAL REREAD					✓
AI FOLLOW UP	All	All	All Modalities		✓
Approved pending LOP or documents		All	All Modalities		✓
Attorney Referral		All	All Modalities		✓
Auth Required		All	All Modalities		✓
CD Burn		All	All Modalities		✓

Showing 52 of 52

OFFICE- STUDY FLAG

SAVE CLEAR BACK

Description * ☐ Inactive

Markets

Facilities ☐ All Facilities

Modalities ☐ All Modalities

☐ View Study Flag on Schedule Book

Color Code

Institutions

☐ All Institutions

Select institutions

When creating a study flag you must type a description, and you may want to add a color to help you recognize the study flag on the schedule book.

If you want a study flag to appear on the schedule book, you must choose **View Study Flag on Schedule Book.**

Additional considerations

- If your organization has multiple facilities defined in the system, you can choose for which facilities the study flag is available. This can shorten the list of available study flags, which helps users on the front end. If you don't want to filter by facility, you do not need to select All Facilities.
- The same is true for modality type. For example, you can have a study flag you want to be used only for mammography, which shortens the list for other modality types.
- Again, the same is true for the institution field, if your organization needs to have separate study flags for each institution.
- Available in Portal makes the study flag available in the physician portal.

For more, check out the [Healthcare IT Learning Center](#)