



Exa® PACS|RIS

Feature Summary

Version 35.0.1 and later

Annotation Settings

©2026 Konica Minolta Healthcare Americas, Inc.


Markup settings in Exa

Markups are essential for highlighting and annotating medical images within a PACS. This guide covers how to customize markup settings based on user preferences. By understanding these options, you can enhance your workflow and communication within Exa PACS|RIS.

Autosave Options


Any remaining markups can automatically be saved when you close the image area.

- 1. Go to **Settings > Viewer Settings**.
- 2. Select **Close Behavior**.
- 3. Select **Save Image annotation on Close**.
- 4. Select **Save**.

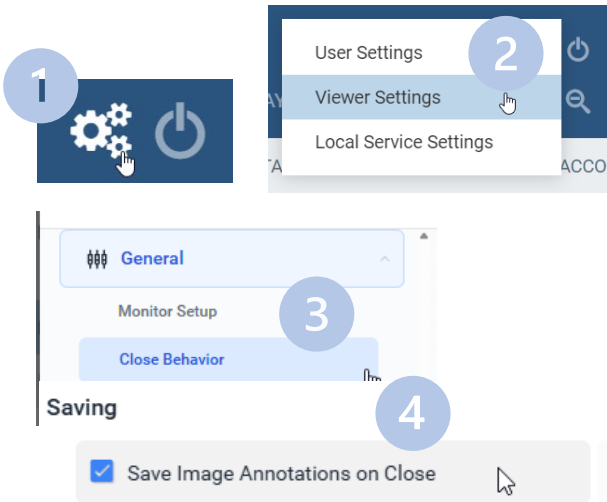

Annotation Settings

- 1. Go to **Settings > Viewer Settings**.
- 2. Select **Interface > Annotations**.

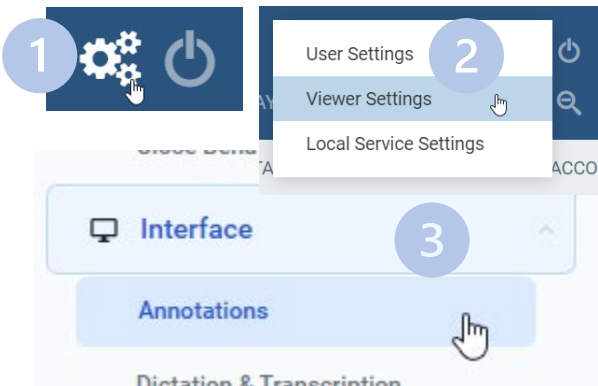

Show Annotation

This setting allows you to use and see the annotation tools and markups.

One-time use vs persistent settings

You can choose whether the selected tool is used for a single annotation, or stays active until selecting another tool or manually deselecting it. Bellow is a table outlining all possible interactions for the two settigs that affect that behavior.

One click annotation	ON	OFF	OFF
Persistent annotation tool	OFF	OFF	ON
Result	One-time use	Tool active until deselect or right click	Tool active until deselect

Edit Annotation mode

This setting allows for editing of a completed markup.

Create an annotation

When creating an annotation, you can drag to create or “Click-Move-Click.”

- **Drag to create**

☒ Drag to Create

Drag the cursor from the starting point of your annotation to the endpoint and release the mouse button to complete the annotation.

- **Click-Move-Click**

☐ Drag to Create

Click once to set the starting point, move the cursor to the endpoint, and then click again to complete the annotation.

Confirm annotation delete

Prompts for confirmation when deleting an annotation.

Measurement Units

Sets the units for any length measurements taken (mm, cm, inches).

Configure Annotation Colors

Annotation Color 1 : Annotations appear in this color by default

Annotation Color 2–5: Annotations can be changed to these colors

Annotation Color Edit: Annotations change to this color when you edit them

Annotation Color Hover: Annotations change to this color when you hover over them

Colors

	Color	Grayscale
Annotation Color 1		
Annotation Color 2		
Annotation Color 3		
Annotation Color 4		
Annotation Color 5		
Edit Annotation Color		
Hover Annotation Color		

Configure annotation size and scaling

Annotation Font Size

Sets the font size for text annotations.

- Scale Pixel Size (ON): Default is approximately 10
- Scale Pixel Size (OFF): Default is approximately to 1.5

Annotation Size

Sets the thickness of annotation handles (from 0.1 to 5 mm).



Visibility & Display

Annotation Font Size

2

Annotation Size

1

Default SR Report Font Size

12

☒ Invert SR Reports

Measurement Units

Millimeters

☐ Scale by Pixel

☒ Show Annotation

Scale by Pixel

Determines the annotation font size by the pixel size.



You can select SAVE ANNOTATION to save all annotations.

This button automatically appears in the main toolbar whenever changes are made to annotations.

