



# **Exa® PACS|RIS**

## **Feature Summary**

### **V35**


**Auto-Change Study Status to Read**

In facilities where there is no radiologist reporting in Exa, Exa can streamline workflow by configuring provider account settings to automatically change the study status to Read upon closing the study image area.


## User Configuration

### 1. Go to Viewer Settings:

- From the worklist, select **Settings > Viewer Settings**.

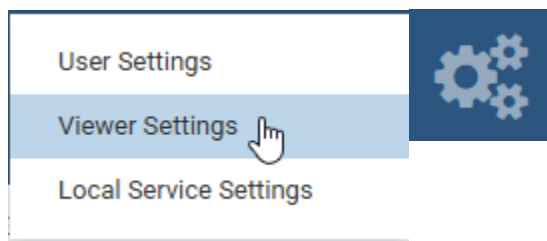


- From the viewer, select the settings button in the main toolbar

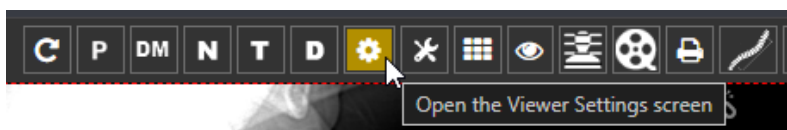


### 2. In the left pane, select **Close Behavior**.

### 3. Select **Change Status to Read on Close**.

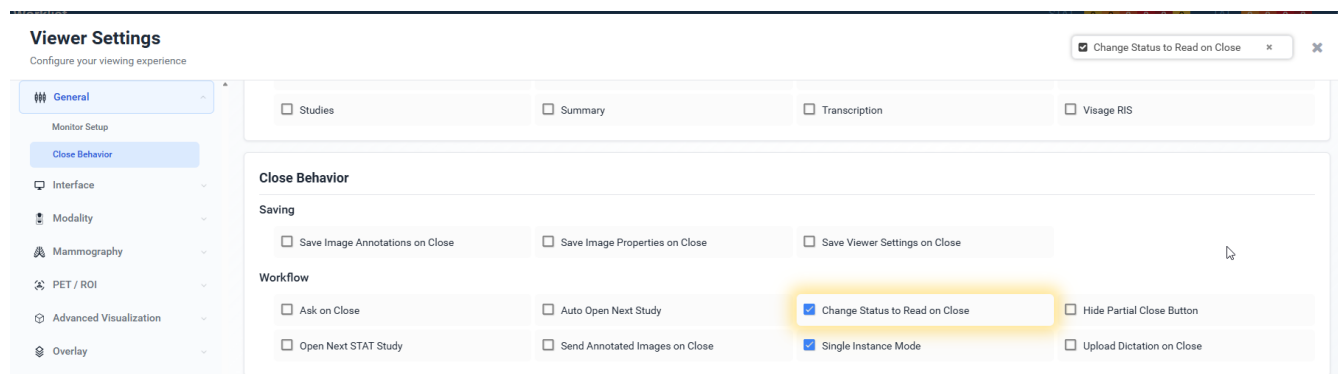

## Workflow


1. On the worklist, open a study in **Unread** status.

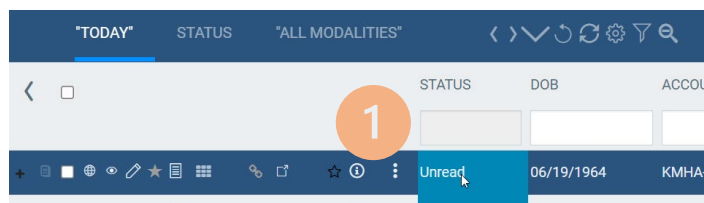

2. Close the study using the **close image area button**.\*



The study status automatically changes to Read.

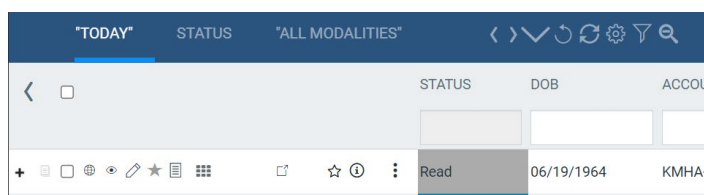

\* You must use the close image area button; closing the browser window or tab will not trigger the study status change.