

Exa® PACS|RIS

Feature Summary

Opening Trauma CAD


©2025 Konica Minolta Healthcare Americas, Inc.

Open Trauma CAD from the Exa PACS|RIS worklist


1. In Exa, add an external apps button for Trauma CAD.

Integration Name: TraumaCAD

URL: `http://TC server URL/TraumaCadServer/Main/index.aspx?autostart=1&kill=1&studyuid={{study_uid}}`


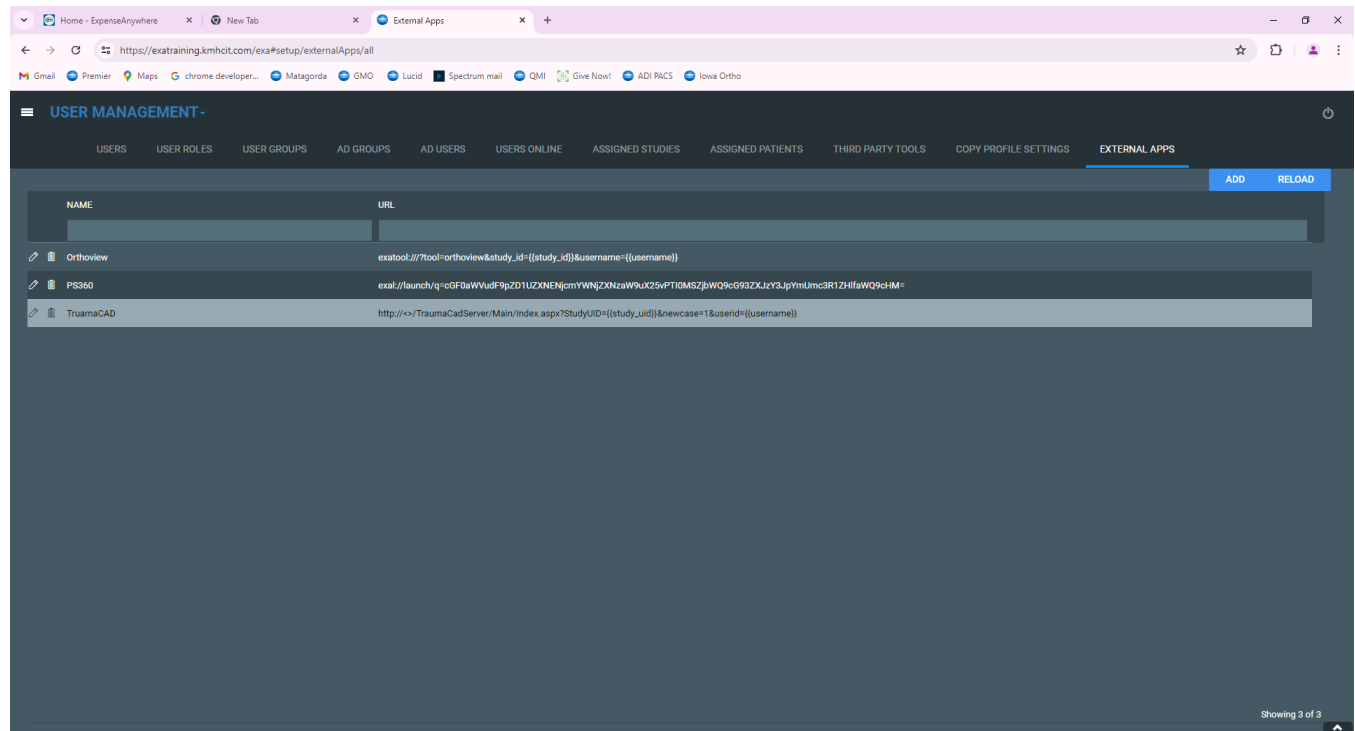


2. From the worklist, select the external app button and then select **TraumaCAD**.

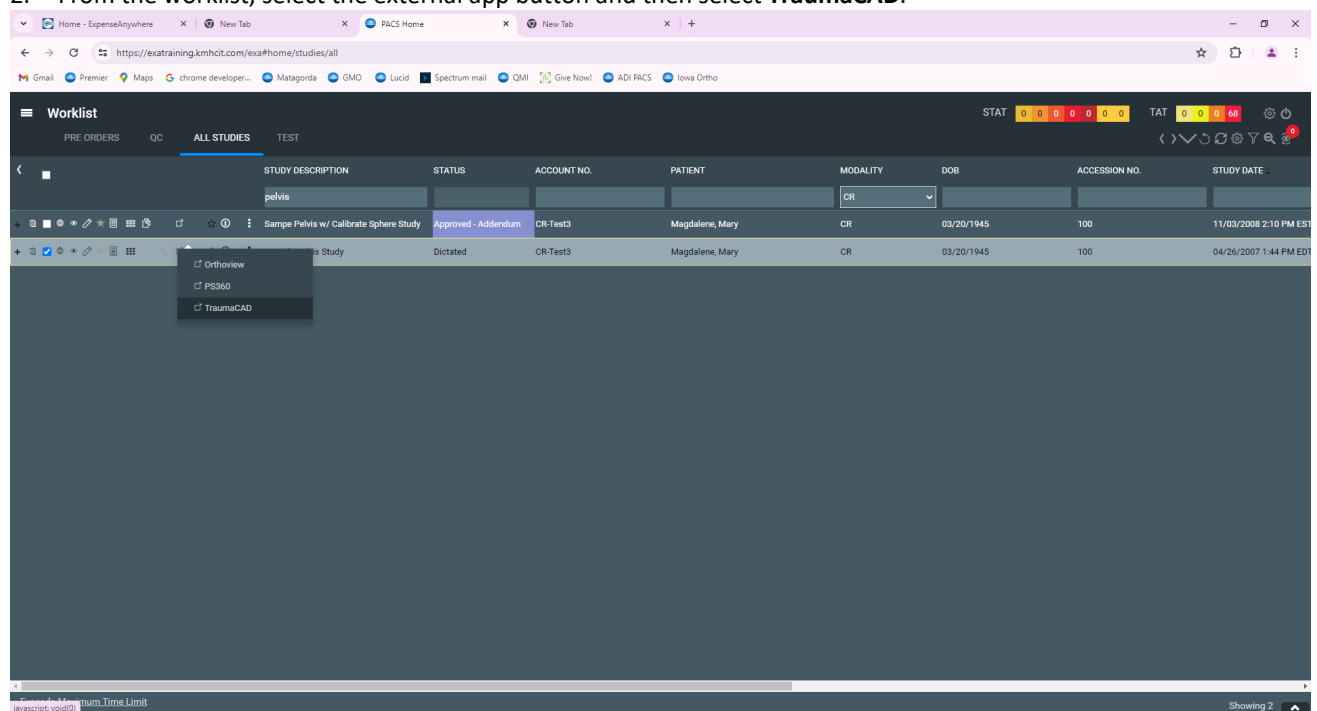

Open Trauma CAD from the viewer

1. Go to **Viewer Settings > External Tools**, and add an entry for TraumaCAD.

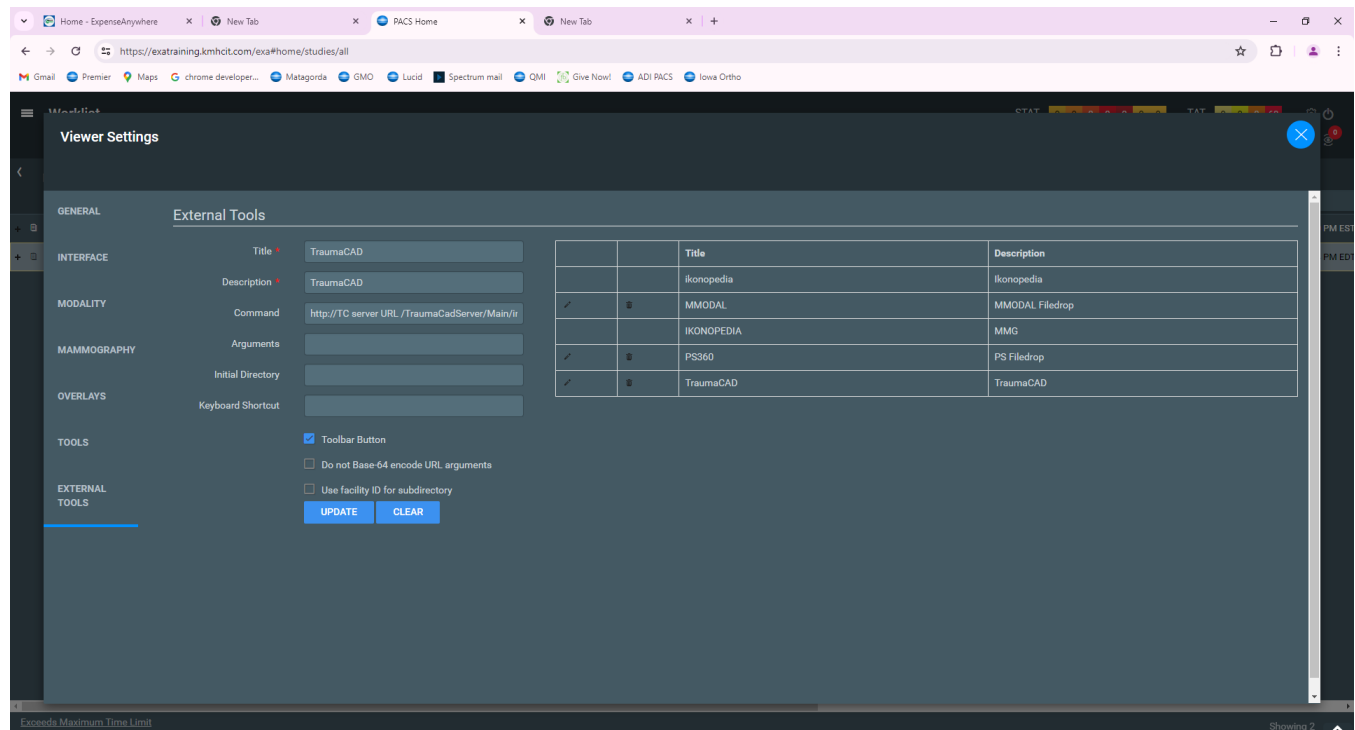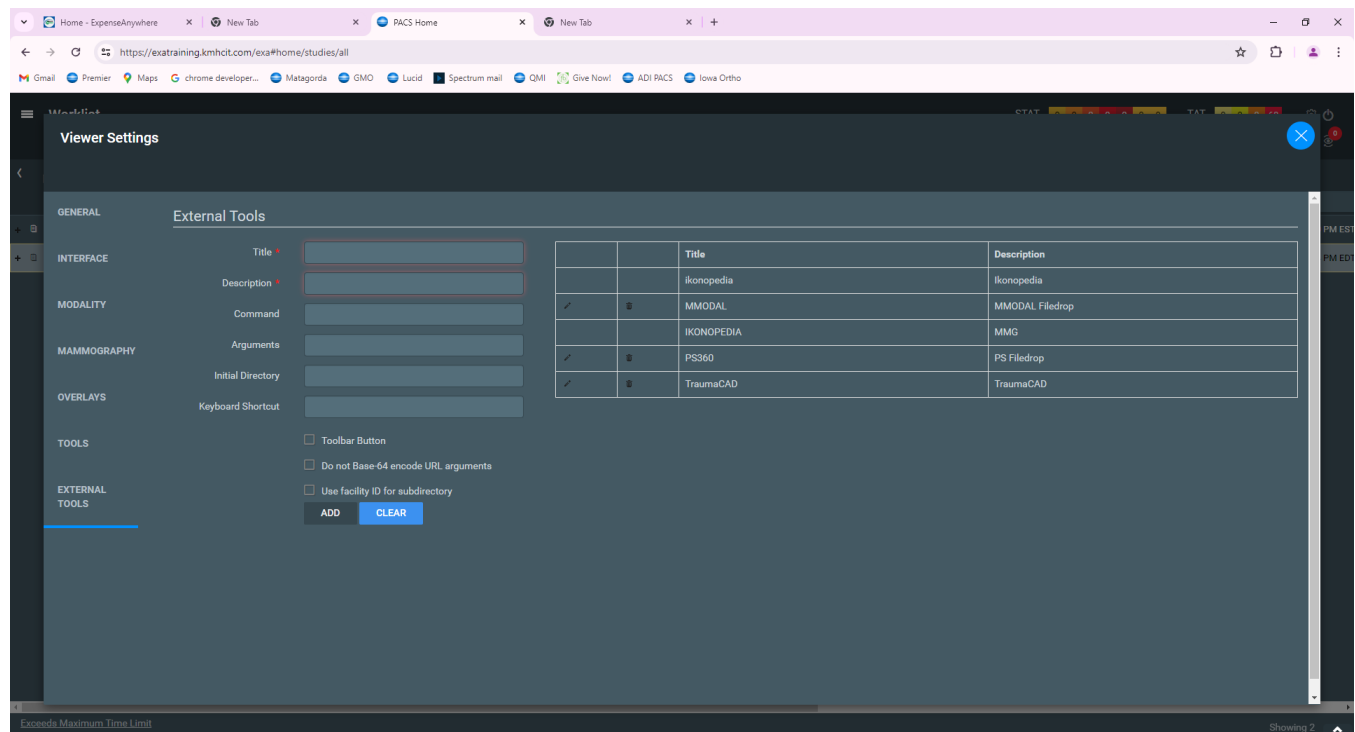

Command variable = `http://TC server URL /TraumaCadServer/Main/index.aspx?autostart=1&kill=1&studyuid={{study_uid}}`