

# COVID-19 Support from 20/20 Imaging.

## Cleaning materials for the SR-130 X-ray System.

When performing the following procedures, be sure to follow safe practices and wear the correct Personal Protective Equipment (for example, Safety Eyewear) according to your Company's Standard Operating Procedures.

**This document provides a list of approved cleaning agents for the SR-130 X-ray System.**



**MISTY MULTI-PURPOSE  
DISINFECTANT CLEANER**  
– AMREP INC.




**MISTY DISINFECTANT AND  
DEODORANT RTU**  
– AMREP INC.




**PRECISE HOSPITAL FOAM  
CLEANER DISINFECTANT**  
– CALTECH INDUSTRIES, INC.



**STERIS® 1608-G4  
SURFACE WIPES**  
– STERIS CORPORATION

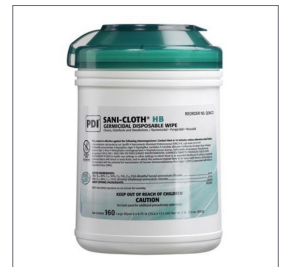

**SANI-CLOTH® AF GERMICIDAL  
DISPOSABLE WIPES**  
– PROFESSIONAL DISPOSABLES  
INTERNATIONAL INC.

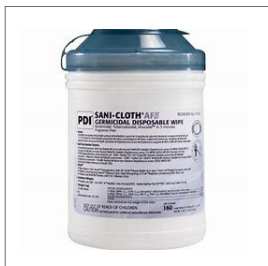

**SANI-CLOTH® HB GERMICIDAL  
DISPOSABLE WIPES**  
– PROFESSIONAL DISPOSABLES  
INTERNATIONAL INC.



**VIREX® II 256  
JOHNSON PROFESSIONAL  
OXIVIR TB WIPES**  
– DIVERSEY, INC.



**70% ISOPROPYL ALCOHOL**



**10% BLEACH SOLUTION  
(DILUTE ONE (1) PART OF 5.25%  
SODIUM HYPOCHLORITE WITH  
NINE (9) PARTS OF WATER**



**MILD SOAPSUDES**

### NOTES:

- The cleaners listed above ensure that the paint finish does not fade nor cause harm to the plastic
- Take special care when cleaning display windows to prevent fogging
- Do not soak parts, make sure to wipe dry with a clean dry cloth
- We understand that each facility may have its own requirements, guidelines and approved cleaners. The manufacturer will not be responsible for replacement of parts that are shown to be damaged by using improper cleaning agents

If you have any questions, please contract our **Customer Care Center at 1.866.734.6234**

**The SR-130 X-ray System is manufactured by Source-Ray, Inc.**

MISTY® is a registered trademark of the Amrep Company, Sani-Cloth® AF and Sani-Cloth® HB are registered trademarks of PDI, Inc., Steris is a registered trademark of Steris Corporation, Virex® is a registered trademark of Diversey, Inc., Precise is a registered trademark of Caltech Industries, Inc.